

M8 Slimming System



Important Safety Notice

- I. Please don't disassemble the equipment or try to do some other operation, which have been indicated in our instruction. All the repairing work should be done by our admitted professional personnel**
- II. Please don't assemble or operate this equipment while you are near the water or your hands are wet. Notice that don't pour the liquid on it.**
- III. Please pull out the plug from the socket when you met such conditions as below. And in touch with professional maintenance Station.**
 - A. The equipment touched liquid**
 - B. The equipment has abnormal smell, smoking or loud and strong noise.**
 - C. Cable is broken**
 - D. The equipment is fallen down and broken**
- IV. Please don't put anything on the cables, don't place the equipment at any place where someone can step on the cable.**
- V. Please pull out the power cable after using it for your safety**

VI. Don't put anything into the equipment's jack, otherwise it may cause fire or touch the electricity. If something enters the equipment, please don't pick it out by yourself, contract distributor or our company immediately

VII. Please don't put the equipment on the anything that can't stand steadily, avoid it turns over and damage.

VIII. Our company has the right to modify the information in this introduction before notify! And keep the explanations right for the above.

Safety Notice

Before operation, please read the following content.

- 1. It is essential to make different setting according to specific treatment demand.**
- 2. The application of The machine must never be painful and cause redness to the skin or leave marks.**
- 3. Move the treatment heads slowly over the skin of the body or the face without ever letting them stop in any one place on the skin.**
- 4. Do not apply the device on children less than 7 years old. The treatment to children over 7 requires medical advice.**
- 5. Do not use this device if suffering from a fever, inflammation, dermatitis (acne, eczema, herpes), warts, open or recently stitched wounds, serious and progressive illnesses (cancer, AIDS, hepatitis, blood diseases, etc. or blood coagulation problems. Do not apply the device if you have or have had phlebitis, after veins surgery within the last two months, if you have varicose veins or tingling in your limbs, if you have any concerns consult a physician.**
- 6. Never use the device on a pregnant woman.**
- 7. May be used on all external and healthy areas of the body except the eyes,**

throat, ears, glottis and the genital area.

8. Must not be applied inside the mouth, or on mucous membrane, or on nipples, or moles without medical advice.

9. Never use the device if you suffer from epilepsy. The device must not be used by any person suffering from mental illness.

10. In the case of long-term treatment with drugs such as anti-inflammation, anticoagulants, antibiotics or if your skin bruises at the slightest shock, ask advice of your doctor/physician before using.

11. Never apply to the body parts with jewelry or ring.

12. Regular use of The machine enhances well-being; it is better not to stop its use for a prolonged period of time. Not following operating instructions and usage recommendations may bring lesser results than expected. Do not use the device longer than the recommended time.

13. If your symptoms have no improvement, or if in any doubt on the use, consult your doctor.

14. Effect may vary depending on your age, life habits (level of physical activity, diet, etc.).

15. The machine can be used after any plastic surgery only while following the instruction of the surgeon.

16. The machine is designed for professional use only, any personal use of this device is prohibited.

17. The Machine continuous working time shall not exceed 2 hours. The best way is cut the power off, let the machine have a rest for 10-15 mins. After the machine have worked for a hour,

SUMMARIZE

Cellulite is one of medicinal social problems in the developed countries.

Obesity, a stumbling block in the way of beauty pursuing, and as a devil that cause thousands of women to fear of spending summer. Taking exercise is easy to reduce weight while its rebound is also very quick; liposuction, a way to shaping waist, while skin is easy to slack. without care; on diet harms our health Usually, obesity is caused by too much food and too little exercise, or some disease.

When the calorie is too much, it will be converted into fat and stored under subcutis or around bowels, the result including thick fat level, heart encased by fat or fatty liver. As to the modern people, obesity is becoming a chronic disease, and it can also generate various side effects. What on earth cause obesity?

- 1. Inherence: The percentage of children obesity is 70% if the parents are both fat; the percentage of children obesity is 50% if the parent is fat; while it's also greatly influenced by eating habit in families.**
- 2. Too much food and drink: sweet food, beverage, and foods those are high in calories, oil, sugar, protein and low fiber.**
- 3. Lack of exercise: if you lack exercise while your daily food are unchanged or contain too much calories, then extra calories are converted into stored fat, your weight will be gained more and more gradually.**
- 4. Midnight snacks: eating too late at night over a long period of time or just before sleep is easily to get weight for the physical activity level are decreasing at that time.**
- 5. Emotional eating: that weight is gained gradually by way of uncontrolled immoderate eating and drinking for happiness or sadness when the extra calories are converted into fat.**
- 6. Central nervous dysfunction of feeding: people with this disease can't help stopping eating.**
- 7. Hormonal abnormalities or abnormality in female sex hormones secretion.**
- 8. Hormones secretion abnormalities or use of medicines, for example, epinephrine, thyroxin, insulin or steroid.**

WHAT DOES THE OBESITY MEAN?

The obesity means a certain extent overweight and fatness layer overdepth, it is

fatness in the body, especial is caused by the triglyceride accumulation too much. The obesity can be divided into two, Single Obesity and Secondary Obesity. Normall we see the former. Single Obesity occupies 99% in the obesity, we cannot find what causes the single obesity. In medicine, it is also named Primary Obesity. It maybe relevant to Descendibility, foods, sporting habits. So called S Secondary Obesity, that is caused by some healthy problems, that is to say, we can source the reasons of Secondary Obesity. It occupies 1% in the obesity, can be classified into Hypothalamus Obesity, Pituitary Obesity, Hypothyroidism Obesity, Cushing's Syndrome Obesity, Gonad Functional Retardation Obesity etc, separately caused by the thalamencephalon, pituitary, hypothyroid and gonad diseases. Among them. Mostly Cushing's Syndrome Obesity, Hypothyroidism Obesity happen to the adults, Hypothalamus Obesity always to the children, which is caused by the craniopharyngioma

WORK THEORY

Tripolar RF handpiece, adopts advanced output mode which could treat the tissue

more accurately, enhance blood circulation, generate more collagen, at the same time

protect the epidermis with cooling, heat up the dermal layer but keep the epidermis at

normal temperature. The dermal layer could be heat up to 45°C-60°C, to generate more

collagen, increase the thickness, giving wrinkle removal, skin tightening and lifting resu

-lts.

Vacuum & RF handpiece, combined 1MHz RF with vacuum treatment , gives more

accurate & positioning treatment and heat up to fat tissue, accelerate blood circulation,

collagen production. At the same time, the handpiece gives cooling to protect epidermis.

Finally, more collagen is produced, wrinkle disappears, skin becomes tight and firm.

High power vacuum gives massage to fat tissue in different depth, to dissolve fat tissue, reduce cellulite, smooth the lymph gland. Fat cell is dissolved into fatty acid and toxin which will be ejected out of the body by lymphatic system, to give instant slimming result . This handpiece is designed for deep fat dissolving , fat melting , detoxifying, skin tightening and body shaping.

Cryotherapy can selectively damage subcutaneous fat while the skin remains intact. Based on this premise, namely fat cells are more easily damaged by cooling than skin cells, cryolipolysis was developed applying cooling to tissue via thermal conduction in a controlled fashion, selecting temperature level ,and exposure to avoid skin damage or frostbite resulting only to damage to the underlying fat cells.

TECHNICAL ADVANTAGES

Adopt the top ultrasonic liposuction technique in the world. , Vacuum + dipolar RF and Triplar RF and cooling treatment perfect combination

Suitable for all kinds of skin.

Comfortable, painless, woundless during the treatment.

Easy operation, easy study.

Non- exhaustion, low cost investment and quick high returns..

- 1. Replace and exceed the OPS liposuction;**
- 2. Tighten the slack skin.**
- 3. Enhance the speed of the body metabolism, speed up discharging of waste and moisture from the body.**
- 4. Repair pregnancy wrinkle;**
- 5. Relax muscles, ease muscle pain;**
- 6. Partial or the whole body weight slimming**
- 7. Effectively improve the buttocks and the legs orange skin, effectively solve the problem of abdomen skin slack after liposuction etc.**

Market advantages

1. Absorbing new guest, improving the beauty salon visibility.
2. Improve beauty parlour levels, expand profit space.
3. Strengthen product efficacy, the promotion product price, promote consumer spending.
4. To open a new course, add new project.
5. But with any product cycles for nursing is tie-in, synergistic, enhance self-confidence, thus active pin beautician
6. Importing foreign latest vacuum negative pressure, mechanical scroll and skin folding technology leading counterparts.

Good traits

1. Non-surgical; non-invasive;
2. Bloodless; painless, seamless, woundless;
3. Do not need recovery period;
4. Short treatment time, quick results;
5. Do not need dieting.

Treated scope

Vasodilator

Fatness of orange skin

Tighten the skin tissue, prevent slack.

Lipolysis

Enhance Flexibility of the fibroblast

Edema obesity

Solid obesity


Decomposition of the fatness on the whole or partial body

After ultrasonic liposuction treatment

85% fatness will be dissolved and removed through lymph system.

Through the liver, 15% fatness will be transformed to be carbohydrate, to supply the people's daily necessary consumption.

Obvious effect

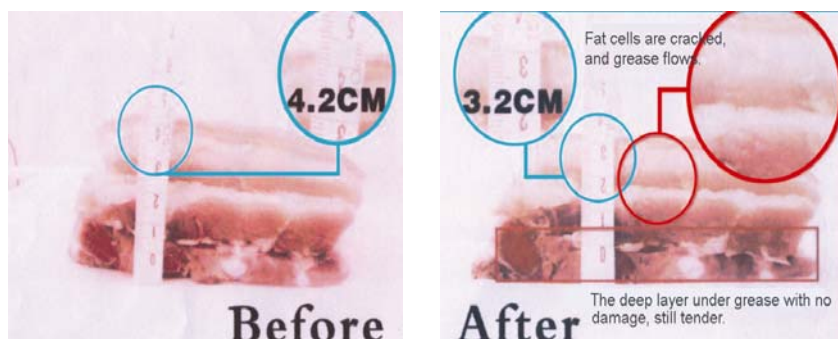

1. Cavitation (40KHz)



For Face

2. Tripolar RF



For Body



For Face(Eye)

3. Freezing probes



ACCESSORIES LIST

No.	Description	Qty.
1	40KHz	1pc
2	Tripolar RF Head for face	1pc
3	Tripolar RF Head for body	1pc
4	Tripolar RF Head for face	1pcs
5	Cooling treatment head	1pc

6	Negative plate	1pc
7	Power cord	1pc
8	User manual	1pc

MAIN TECHNICAL PARAMETER

Ultrasonic Frequency:40KHz RF Frequency :0.5-10KHz

Cooling Intensity : 1-10℃

Voltage :110v~120v/60HZ 220v~240v/60HZ

Power: 100W

G.W: 18KG

MEAS:58cm×43cm×38cm

OPERATION STEPS

- 1) **Check all accessories whether they are connected right or not, clean or disinfect the accessories.**
- 2) **Clean the skin first, then smear the products onto the treated skin, please use ultrasonic fat burning gel.**
- 3) **Plug in the power and switch on the general power on of the control panel.**
- 4) **According the client's fat part status, adjust the Intensity and the Wave and Working time.**
- 5) **After treatment, please pause the working, then switch off the power. Clean the spare parts and place them in original.**

System Enter



System Selection Interface


Choose the System for treatment





Cavitation(40KHZ)




A. Please analyze the fat part status, then according it, to adjust the Intensity and the and Working time and body parts.


B. Then click  key ,The 35K+Bipolar RF will be working,and the time will be countdown,





C. When click  key,the machine will be Pause , time pause.

D. After treatment.please click  key, the machine will be stop .the time will be zero.

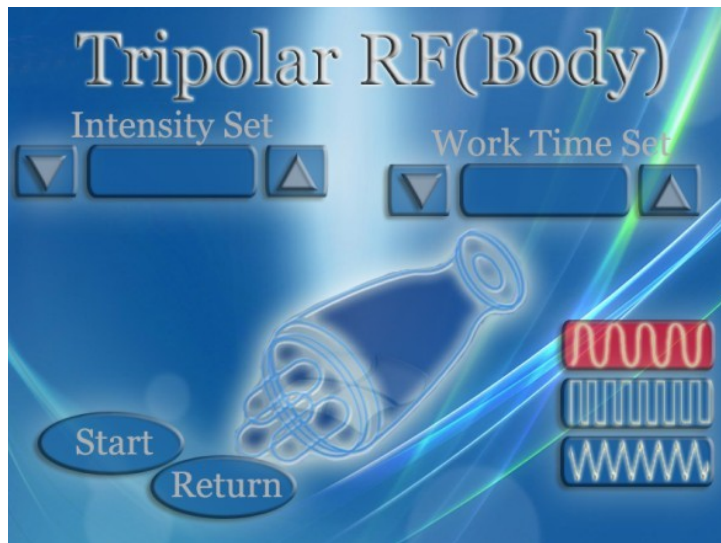
E. Then click  key,the system will be back to main menu

Tripolar RF(Body)





- A. Please analyze the fat part status, then according to it, to adjust the Intensity and the Working time and body parts.
- B. Then click  key, The vacuum +Bipolar RF will be working, and the time will be countdown,
- C. When click  key, the machine will be Pause, time pause.
- D. After treatment, please click  key, the machine will be stop. the time will be zero.
- E. Then click  key, the system will be back to main menu

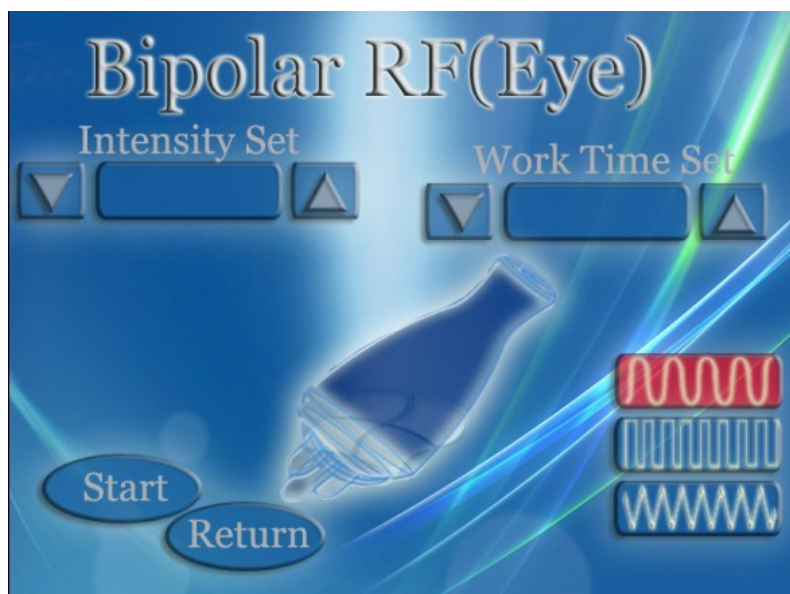
Tripolar RF(Eye)




A. Please analyzer the fat part status, then according it, to adjust the Intensity and the Working time.

Then click  key, The Tripolar RF for body will be working , and the time will be countdown.

C. When click  key, the machine will be Pause , time pause.


D. After treatment. please click  key, the machine will be stop .the time will be zero.


E. Then click  key, the system will be back to main menu


Tripolar RF(face)




A. Please analyzer the fat part status, then according it, to adjust the Intensity and the Working time.

Then click  key, The Tipolar Rf for Face will be working , and the time will be countdown.




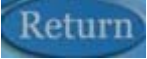
When click  key, the machine will be Pause , time pause.

D. After treatment. please click  key, the machine will be stop .the time will be zero.

E. Then click  key, the system will be back to main menu

Cryotherapy



- A. Please analyze the fat part status, then according it, to adjust the Intensity and the Working time and body parts.
- B. Then click  key, The Cryotherapy will be working , and the time will be countdown.
- C. When click  key, the machine will be Pause , time pause.
- D. After treatment, please click  key, the machine will be stop .the time will be zero.
- E. Then click  key, the system will be back to main menu

Suggest Operation Method

THIGH

Treated time: 30 minutes

1. **From down to upper, pushing to the groin to dredge the lymph.**




2. **From down to upper, circling push by anticlockwise, can help decomposing fatness.**



3. Also can push by come-and-go to decompose fatness.

4. Pull from the knee and muscle texture to upper, can improve the curve.

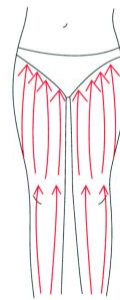

BUTTOCKS


Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.


3. Along the muscle of arm, pull up to lift and tight the muscle.

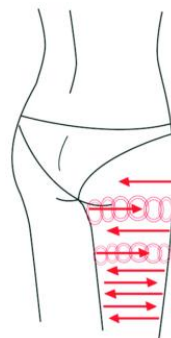

BACK


Treated time: 20 to 30 minutes

1. First by come-and-go, pushing

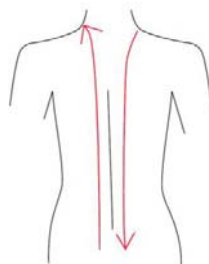


the bladder nerve 2 to 3 times.


2. Circling by anticlockwise to stimulate the underarm lymph node.

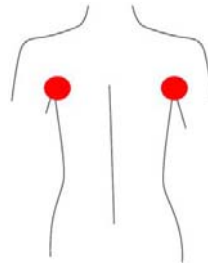

3. On the back, by anticlockwise circling, can help to decompose the fatness.

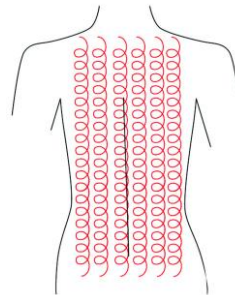

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help improving the back curve.

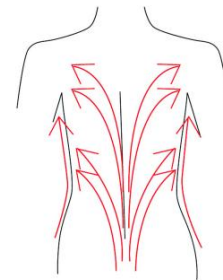

ARM

Treated time:30 minutes

BACK SIDE

1. From the inner elbow to the armpit, doing lymph drainage.



2. Circling to stimulate the lymph node.



3. By anticlockwise or come-and-go pushing, decomposing the fatness

4. From the elbow to the outer, doing tightening gesture.



Facade


1. From the elbow to the outer, doing tightening gesture.

Abdomen


Treated time: 20 to 30 minutes

1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.


2. From small to large, by anticlockwise direction, circling push down.

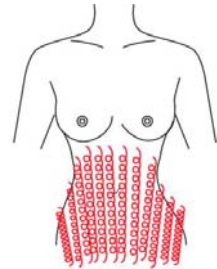

3. From the belly and muscle texture, pull to the groin.

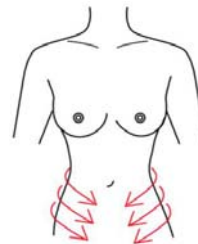

4. Through lymph drainage gesture, taking the toxin to the place of groin.

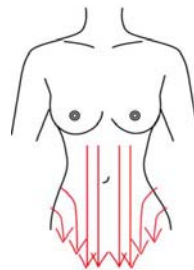

Contraindication

1. Pregnant women or women during in menses.
2. Epileptic
3. Patients with malignancy.
4. Patient whose wound after operation has not healed up.
5. Acute inflammation or epidemical patients.
6. Whom with heart diseases or with heart pacemaker..
7. Whom with kidney (gall-stone) disease.
8. Who was embedded metal object or silica gel.
9. Who in menses, birth control period, emiction incontinence period, or accepting the belly operation.
10. Whose body always take much inner hot.
11. Who has the genetic hypersensitivity.

NOTICE

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasoinc gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
7. If the machine will not use for a loing time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.

AFTER-SALE SERVICE AND MAINTENANCE

After-sale service

We will guarantee the quality of our equipment, normally there is no quality problem

1. If there is a deficiency, please email or call us on the reasons, it would be best if this can be dealt with by email or telephone. Because some of the problems are caused by using the machine in a wrong way.
2. We suggest the client to take photos of the area where the problem occurs, we

will pass them to our technician , our technician will give a satisfactory answer within 1-7 working days

3. If the problem comes from the spare part, we could consider in changing the spare part for our client, which could be sent with his/her next order. If the next order is uncertain, we could pay freight cost to send the spare parts.
4. If the problem comes from the main device, we could advice our client to post the device back to our company for repair by ordinary post, or we could re-send a new.