

Potable VelashapeMachine

User Manual

Mode: WL-V012B



The treatment handles:

LCD Touch Screen Auto



Functions

1. Cellulite Reduction
2. Lymphatic drainage, body massage
3. Skin tightening, stretch marks removal
4. Body lifting

Technical Parameters

1. Air Pressure : 0 – 85Kpa
2. Red laser light : 650nm; Out power : 3W
3. Blue laser light : 480nm; Out power : 3W
4. Green laser light : 560nm; Out power : 3W
5. Torque: 5KG/M
6. Rotate speed: 40 rolls/Min
7. RF Frequency: 2MHZ; Output power: 60W

Treatment Area
Belly, Back, Thigh, Byuuocks

Middle Auto Roller Head



Functions

1. Orange Cellulite Removal
2. Body Sculpting
3. Cellulite Massage and Lymphatic drainage

Technical Parameters

1. Air Pressure : 0 – 85Kpa
2. Red light: 650nm; Out power : 3W
3. Rotate speed: 40 rolls/Min
4. Torque: 5KG/M
5. RF frequency: 2MHZ; Output Power: 60W

Treatment Area
Calf, Thigh, Arm, Neck

Vacum RF pen

Functions

- 1.Face Lifting
- 2.Wrinkle Removal
- 3.Eyes'care

Technincal Parameters

- 1.Air Pressure : 0 – 85Kpa
- 3.RF frequency:2MHZ; Out power : 20W
- 3.Blue light:480NM; Out power : 10MW

Treatment Area
Neck,Face,Eyes



40KHZ Cavittion Head

Treatment Area
Belly,Thigh,Arm,Back,Byttocks,Calf

Technincal Parameters

- 1.Cavitation FrEQUENCY: 40KHZ
- 3.Out power : 50W
- 3.Cavitation Output:
Level 1-3:continuous; Level 1-3:discontinuous
- 4.Red lighth: 650NM; Output power: 3W

Functions

- 1.Fat Explording
- 2.Body Sculpting
- 3.Skin Tightening



WL-V012B has combined with the RF, vacuum, outer automatic roller, infrared light, green light and blue light, cavitation, BIO, lipo laser. It is the new updates of V9 velashape technology. It also combines with RF and 3 photons (red, green, blue) to generate heat. It can penetrate the skin to a depth of 15mm by mechanical movement, which is far more than the depth of the other devices currently on the market. While the heat is conducted, the mechanical movement is carried out simultaneously and so is the fat. In this way, it makes the skin more flexible. It can improve the figure and the skin surface. Although the weight of the patient does not reduce, the patient will look much slimmer because the whole fat area has been re-assigned. We have selected 20 women of different ages and skin types for a test. Against their main fat parts – thighs and buttocks, we conduct 3-6 treatment times and each person's body curve has improved to a great extent. Meanwhile, the fat around the thigh has obviously reduced. The side effect of WL-V012B treatment is minimal. The skin surface of each treatment receiver will only appear flushed from some minutes to an hour or appear slightly edema, but a few hours later, the appearances of the fat accumulation parts will be greatly improved. For the past few years, we have been using the method of deep fat massage, which is very time-consuming. Also, the treatment receiver will feel uncomfortable for long duration of treatment. However, the time for WL-V012B (B) treatment has greatly shortened. It only takes 30 minutes to do thigh and buttock treatment so the treatment receivers will feel more comfortable compared with the deep fat massage. According to their description of the treatment process, most people are very comfortable and even sleepy when the massage is up and down the thigh.

I. The operation handle features

1. Combined with bi-polar RF and infrared, Blue and Green energy; the vacuum suction coordinated with mechanical outer automatic rollers
2. The RF and Red, Blue and Green light therapy generate heat, increasing the oxygen content of skin cells.
3. The special vacuum roller massage soothes the skin and makes the heat conduction effect better.
4. It increases the metabolic rate effectively and reduces fat accumulation through lymphatic drainage.
5. It increases the skin tissue elasticity and makes the skin more smooth and delicate.

II. The Advantages of Instrument WL-V012B(B)

1. It is the good body sculpting and firming instrument verified, with the purpose of improving Cellulite and reducing fat accumulation effectively;
2. It is unanimously recommended by World Medical Organization;
3. It is the slimming instrument sought after by many Hollywood stars and reported in many European and American magazines;
4. It combines light and radio frequency interaction and lowers the impedance of the radio frequency from the epidermis;
5. It eliminates the influence of therapeutic effects due to body resistance force;
6. The energy is concentrated in the treatment of epithelial tissue and connective tissue;
7. It adopts Bi-polar and polar radio frequency to improve the treatment of security;
8. The 5MHZ RF penetrates the skin depth with 15mm, which effectively treats adipose tissue;
9. The vacuum roller absorbs the skin tissue, which leads the radio frequency and infrared light into a further treatment;
10. The light and radio frequency can be controlled separately;
11. The immediate effect is obvious and the long-term efficacy could be predicted;
12. It is a non-invasive treatment with a comfortable process;
13. There are more than 10,000 women adopting V9 care every day.

III. Synergistic Use of Energies - The Velasmooth Pro device



employs four treatment modalities

-Infrared, Blue and Green lights heats the tissue up to 2mm depth.

-Radio Frequency (RF) heats tissue from 5 to ~ 15mm depth.

-Vacuum +/- massage mechanisms enable precise targeting of energy to the tissue.

-Mechanical massage provides lymphatic drainage (dermal and hypodermal).



Biological Impact of Treatment –Increased -abolism leading to demarcated desired ana-bolic and catabolic alterations at predeter-mined skin components and depths.

Mechanical Manipulation (Vacuum +/-

Massage mechanism

- a. Stimulates lymphatic and blood circulation
- b. Facilitates fibroblast activity
- c. Reduces viscosity of fat cell clusters
- d. Promotes vasodilation and extravasation of oxygen and nutrients
- e. Enables heating at different depths

Heating (infrared + radio frequency energies

- a. Enhances blood circulation and increases




oxygen dissociation from oxyhemoglobin

- b. Facilitates fibroblast activity
- c. Increases metabolism of fat cells
- d. Improves skin texture

40khz ultrasonic cavitation for fat exploding
Ultrasonic cavitation combined with RF vacuum roller;
new and groundbreaking treatment to obtain a slim
body by intensifying collagen, liquefying fat, and
compression the skin.


Lipo-Cavitation or Ultra-Cavitation may be the innovative
world-shattering procedure in which may take people to a
fresh level proposing unsettled slimming results and the
aptitude to look ages younger in a few weeks. This latest
revolution designed to give people great results without
surgery has been extremely successful.

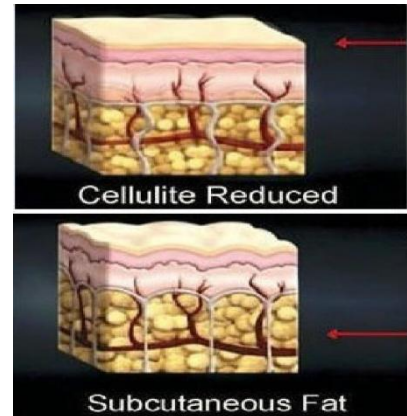

VI. Functions


1. Body slimming, contouring & shaping
2. Fat and cellulite reduction
3. Skin Tightening
4. Wrinkle Removal
5. Warm vacuum cellulite massage
6. Eyes' area treatments
7. Face wrinkle removal & lifting
8. Promote cell metabolism, improve blood circulation
9. Increase skin elasticity

VII. Operations:
1. Operating Instruction 1

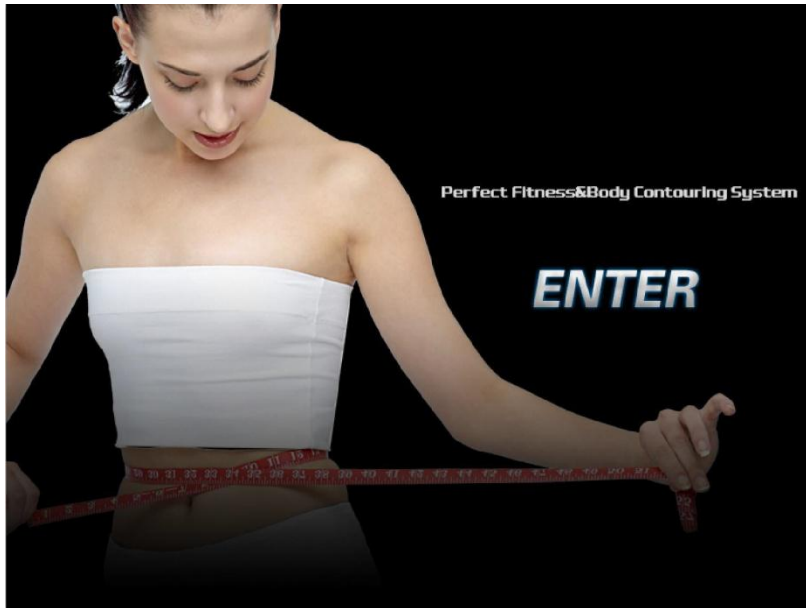

(Picture 1)

Turn on the switch on the back of the machine, it will enter into this interface (Picture 1) , press “Enter” it will turn into the next operation interface (Picture 2)

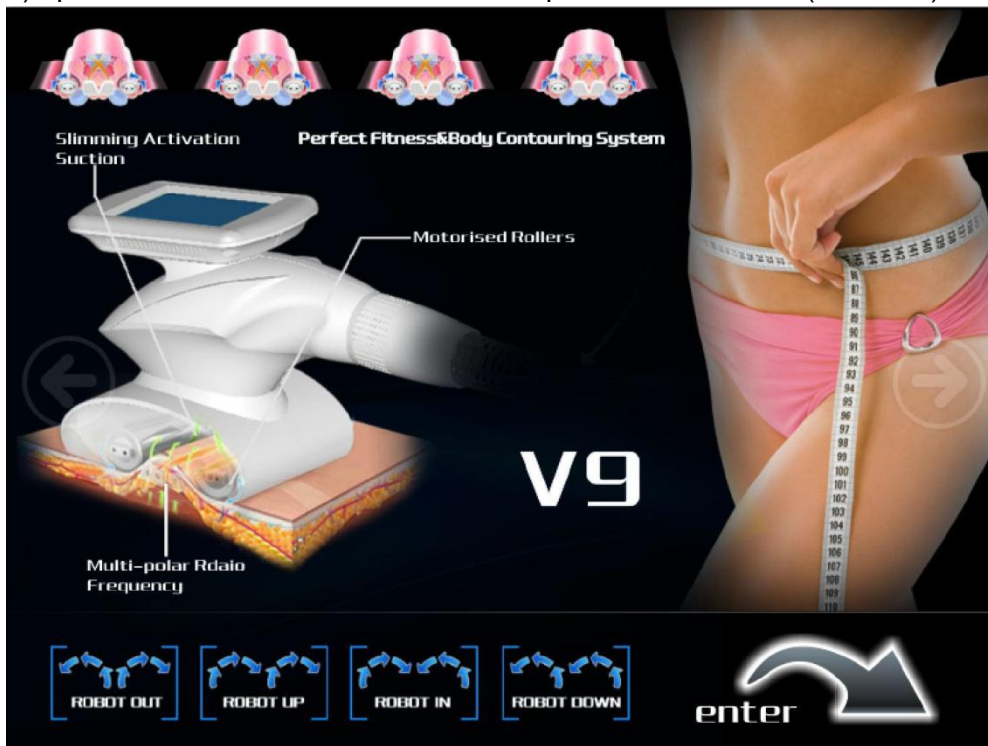

(Picture 2)




(Picture 3)

2. Operating Instruction 2





press “enter” button to enter the operation interface of the velashape vacuum roller handle and the cavitation handle (Picture 3)

1). the middle vacuum roller handle operation details

press “” button to enter the operation interface of the Middle vacuum roller handle (Picture 3)

RF:

press **R.F (w)** button and set the intensity of RF through  (the up and down

button), then press  button to start RF, you can set the RF to 3, if don't feel warm, you can add it one by one every 1 minute.

Red light laser:

Red Light: Can increase skin elasticity and smooth the concave and convex surface.




Press this button to start and stop the red light.


Vacuum:


Model A: long suction mode.

Model B: inhale 0.5 seconds and deflate 0.5 seconds.




Press  button to start and stop vacuum suction, there are 2 vacuum suction

models, continuous and discontinuous, you can press  to change different

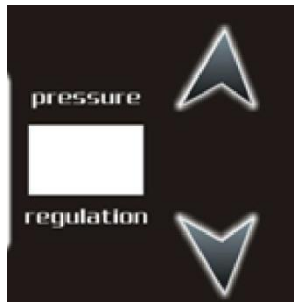

vacuum suction models; Set the vacuum pressure by this button up and down, you can use it with 35kpa at the beginning, if it is not enough, you can add little by little.


Treatment time:


Press **Time** button, set the treatment time by  button. You can use this handle for treatment of the arms, thighs, etc. The treatment time is about 20-30 minutes for each part.

The drawing of how to operate this handle:

Pull and push the fat to the nearest lymphatic.



2). the operation details of small vacuum facial handle

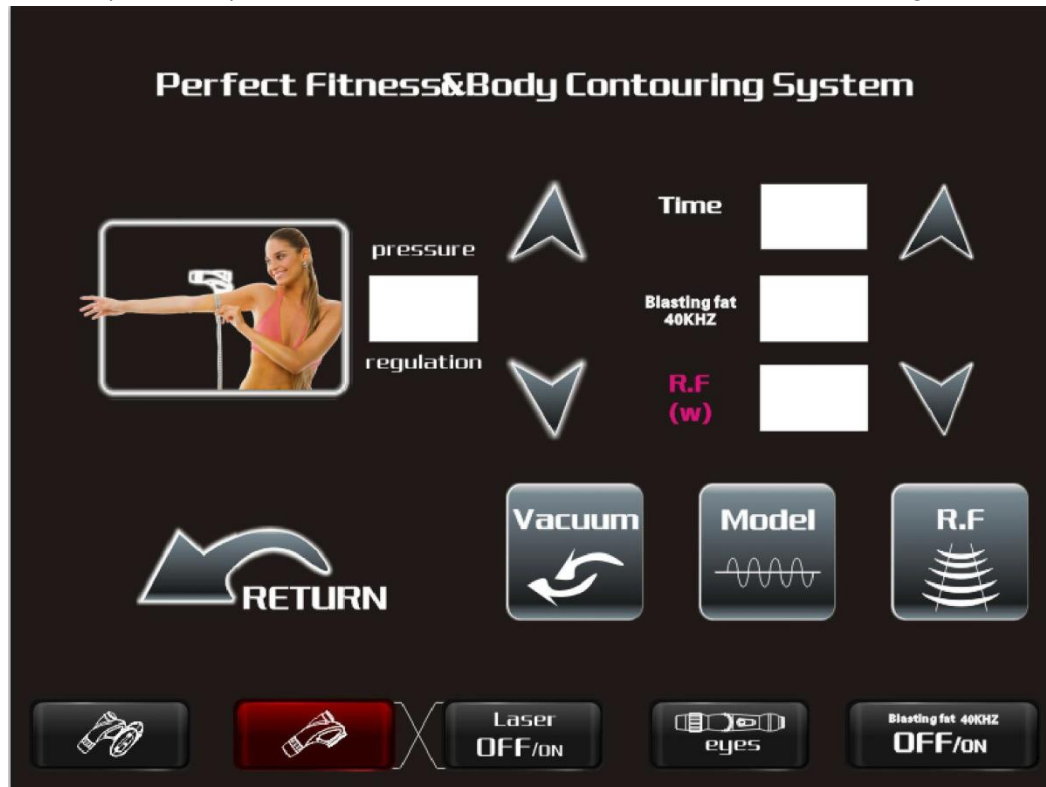
Press  this button to enter into the operation interface of vacuum facial handle (Picture 4). This handle is used for face wrinkle removal & lifting.



(Picture 4)


RF:

Press  button and set the intensity of RF through  (the up and down

button), then press  button to start RF, you can set the RF to 1 or 2, if don't feel warm, you can added one 1 minute later. Normally the RF with 1 or 2 is enough for face treatment.

Red light laser:

Red Light: Can increase skin elasticity and smooth the concave and convex surface.


 Press this button to start and stop the red light.

Vacuum:


Model A: long suction mode.


Model B: inhale 0.5 seconds and deflate 0.5 seconds.



press  button to start and stop vacuum suction, there are 2 vacuum suction models, continuous and discontinuous, you can press



 to change different vacuum suction models; Set the vacuum pressure by this

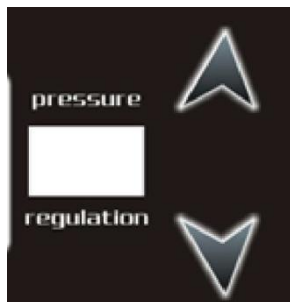


button up and down, you

can use it with 25 at the beginning, if it is not enough, you can add little by little.

Treatment time:

Press **Time** button, set the treatment time




by  button. The treatment time is about 10-15 minutes for whole face.



The drawing of how to operate the vacuum facial handle:

The first step lifting directionally:

- a). Moving upward along the skin texture, pulling to the front of ears, back of ears, and the fixed point of temple.
- b). During operation another hand need to lift skin upward subsidiarily so that handle folded skin can stimulate the collagen production by RF heating. The effect is better.
- c). Face part need to be repeated 2-3 times

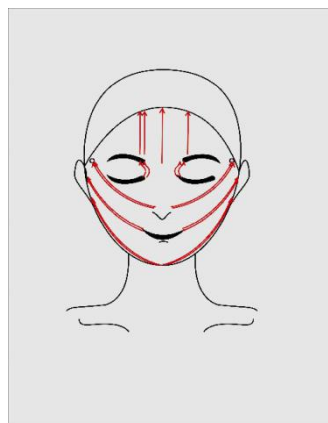

The second step:

Aim at the facial parts which are not easy to operate can repeat for several times, such as nasolabial folds, forehead wrinkles, facial wrinkles, thick pores and so on.


The effective features:


- a) Reddish facial skin, thermal feeling;
- b) Facial wrinkles has be decreased;
- c) Face has been lifted and tightening
- d) Promote cell metabolism
- e) Improve blood circulation
- f) Increase skin elasticity

Neck operation method:

Use linear method to adsorb the neck part skin; When operating another hand need to lift skin subsidiarily it will have the better effect; Entire neck part need to repeat for 2-3 times.




3). the operation details of vacuum eyes pen

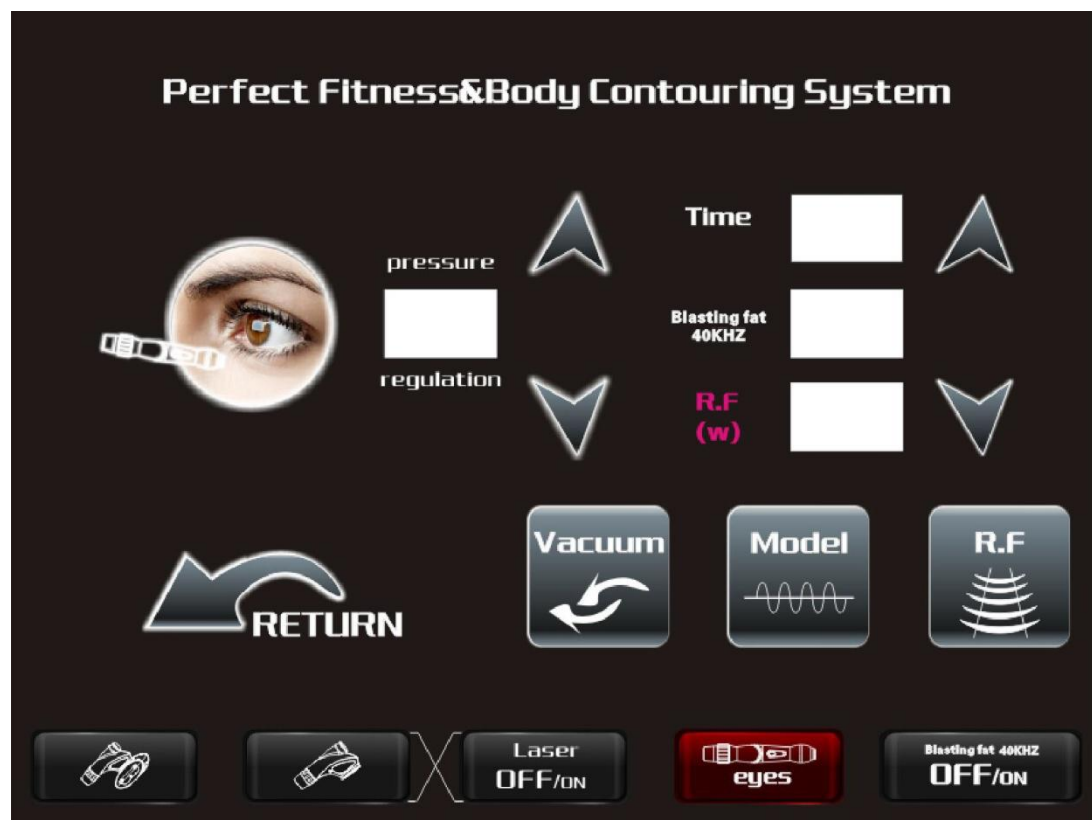Press  this button to enter into the operation interface of this eyes' handle (Picture 5). This handle is used for eyes' wrinkle removal, lifting and other eyes' problem solution.



(Picture 5)

RF:

Press  button and set the intensity of RF through  (the up and down button), then press  button to start RF, you can set the RF to 1, if don't feel warm, you can added one 1 minute later. Normally the RF with 1 or 2 is enough for

eyes' treatment.

Vacuum:

Model A: long suction mode


Model B: inhale 0.5 seconds and deflate 0.5 seconds



press button to start and stop vacuum suction, there are 2 vacuum suction



models, continuous and discontinuous, you can press different

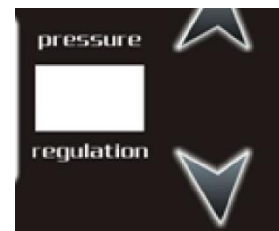

vacuum suction models; Set the vacuum pressure by this

button up and down, you can use it with 25Kpa at the beginning, if it is not enough, you can add little by little.

Treatment time:

Time



Press button, set the treatment time by button. The treatment time is about 10 minutes for two eyes.

The drawing of how to operate the vacuum eyes pen:

The first step lifting directionally:

- a). Moving from medial of the eye conner to the direction of outboard of the eye conner, lifting upward, pulling to the fixed point of temple or hair edge.
- b). During operation another hand need to lift skin upward subsidiarily so that handle folded skin can stimulate the collagen production by RF heating. The effect is better.
- c). Eye part need to be repeated 3-4 times.

The second step:

Aim at the Eye parts which are not easy to operate can repeat for several times, such as lower eyelids, upper eyelids, outer eye wrinkles and stubborn eye bags.

The effective

features:

Reddish eye parts skin, thermal feeling;

Eye parts wrinkles ha be decreased;


Eliminate eye dark circles, bags under eyes, eye pattern and relieve edema



Upper eyelids and lower eyelids have been lifted and tightening

Decrease eyes' wrinkles;




4). the operation details of small 40KHZ cavitation handle


Press  this button to enter into the operation interface of this 40KHZ cavitation handle.

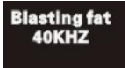Press  until it turns red. Then, press  adjust the intensity 4~5 or higher according to clients' request.

Treatment time:

Press **Time** button, set the treatment time by  button. You can use this handle for treatment of the belly, buttocks, arms, thighs, etc. The treatment time is about 20 minutes for each part.

The drawing of how to operate the 40KHZ cavitation handle:

Moving this handle on treat parts with circle slowly. 


3.Operating Instruction 3

the operation details of big velashape vacuum roller handle:



Four models working way of the rollers:

- 1). dual rollers rotate inwardly;
- 2). dual rollers rotate outwardly;
- 3). dual rollers rotate clockwise simultaneously;
- 4). dual rollers rotate anticlockwise simultaneously.

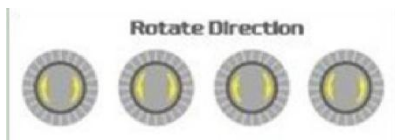

Press these buttons to change different roller models.

Rollers: The rollers have strengthened the RF and the thermal penetration depth so as to reach the fat layer more effectively.

Vacuum models:

Model A: long suction mode

Model B: inhale 0.5 seconds and deflate 0.5 seconds




Press this button to start and stop the vacuum suction,



Press

here to change the vacuum models. Set the vacuum pressure by this

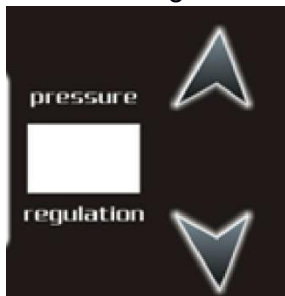

button up and down, you can use it with 35 Kpa at the

beginning, if the customer feels not enough, you can try to adjust it with 40-45.

Vacuum: The vacuum can increase blood circulation and promote cell metabolism and do vacuum cellulite massage.

3 photons:

Red, Green and Blue Light: Can increase skin elasticity and smooth the concave and convex surface.



Press this button to start and stop the right, press this button to change different lights.

5MHZ RF:

Bi-polar RF: 5MHZ RF penetrates the skin depth with 5-15mm. It focuses on heating the fat cells and treats cellulite intensively.



Pls touch here to start and stop the RF, then adjust here to change RF intensity, you can set it with 4 at the beginning, if customs doesn't feel warm, you can add it one by one every 1-2 minute.


Treatment time:



Pls touch here, then adjust this two parts to change the treatment time.

Click (Time -- normally set the operation time to be 20-30 minutes) or (R.F <w/cm²>, all the contact area of the skin is 3 cm²) select any one and then set the value – start and stop infrared light – start and stop R.F – start and stop vacuum

The drawing of how to operate the touch screen vacuum roller RF handle:

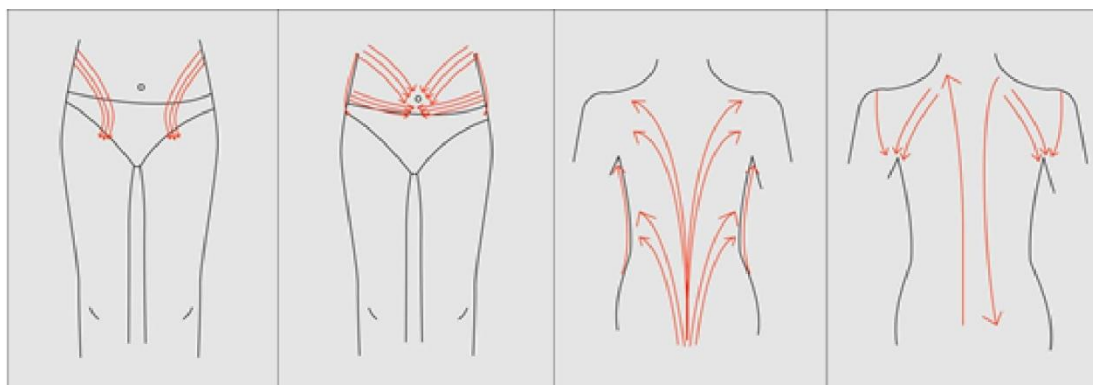

(Pls do not use the gel as medium when operate the handle with Vacuum, the slimming oil will be the perfect choice. Because the treated handles combined the technology of Vacuum, if using the gel with vacuum, the gel will be blocked up in the vacuum as the gel is insoluble substance and then make the vacuum can not work.)

VIII. Technical Parameters:

Input voltage:	AC220V, 50HZ/AC110V, 60HZ
Power consumption:	1000W
Fuse size:	10A
Large vacuum roller handle:	
Vacuum pressure:	0-100 (KPA)
Infrared light wavelength:	700nm
Green light wavelength:	560nm
Blue light wavelength:	490nm
Roller speed:	50r/min
R.F Energy:	30W
R.F Frequency:	5MHZ
Middle vacuum roller handle:	
Vacuum pressure:	0-100 (KPA)

R.F Energy:	20W
Roller speed:	30r/min
R.F Frequency:	5MHZ
Infrared light wavelength:	700nm
Small vacuum facial handle	
Vacuum pressure:	0-100(KPA)
RF frequency:	5MHZ
Infrared light wavelength:	700nm
Vacuum eyes pen (same as small vacuum facial handle)	
Small 40KHZ cavitation handle:	
Cavitation frequency:	40KHZ

Pictures of All 7 units of The Accessories:

Large vacuum roller handle



Middle vacuum roller handle



Small vacuum facial handle



Vacuum eyes pen



Small 40KHZ cavitation handle



Please take and put the handle gently and do not fall it on the floor.

1. The equipment shall stop for 10 minutes after one hour's work. Continuous high-intensity use will shorten the service life of the equipment.
2. Do not use the viscous media such as gel, which will block the tracheal.
- 3.** You can use the alcohol to clean the roller, please avoid the alcohol to drop into the vacuum pipe, or touch the plastic which is protect the lights on this handle.

Notes after operation:

- 1) Avoid using cold water to clean the operation area within eight hours after operation(except facial part);
- 2) Prohibit to do massage in the operation parts ;
- 3) Do not bathe in the sun within one week after treatment;
- 4) Can smear moisturizing skin care products in the treatment parts;
- 5) Prohibit to eat raw and spicy food within 8 hours after operation

Forbidden group

- 1). People with heart disease or high blood pressure, or configured cardiac pacemaker.
- 2). Patients with acute inflammation, asthma, deep vein thrombosis, cancer, etc.
- 3). People with hemorrhagic disease, trauma or who is bleeding.
- 4). Pregnant women
- 5). Medical Plastic parts, or parts with artificial in filling inside.
- 6). People with metal inside the body(include metallic tooth)
- 7). Patient with skin inflammation or with edema.
- 8). People with an abnormal immune system.
- 9). Numb or insensitive to heat.

After-sale

One year free warranty for main part of machine; half a year warranty for accessories;
(Except consumables) Perfect after-sale training

After-sale maintenance can be effect in the way of providing enough preparing apparatus.
Guarantee: operating rate is above 95% of actual working days in warranty period (not including human errors, unqualified environment, etc.)

Response time of maintenance in expired warranty: in 24 hours (Response time of maintenance in expired warranty beginning from when telephone ring, and include confirmation of payment and other matters as well as dispatch of preparing apparatus.)