

# **AR-NVA04 BIO Function**

## **# Function**

Skin tightening after cellulite and fat breaking down:

Lifting up. For example, faces, breast. Hips ect.

## **#Accessories**

Spot handle X 2 pair

## **#Operating Instruction**

1. Connect one pair handles to BIO outlet

There are 1 pairs of (extra for replacement) Handles available:

Spot handles: For small area, such as nasal fossa, and so on.

2. Select the times as you like about 15 minutes or 30 minutes.

3. Select the strong or weak intensity, Adjust the BIO intensity slowly from minimum to maximum, for commonly use, do not set the intensity to maximum unless necessary.

4. Select the mode just you like such as hip, bust, neck or face.

## **Can be used on:**

-Micro wave Anti-wrinkle massage suitable for sagging, ageing. Excrescence skin and eye bag muscle.

-Skin Maintain the pores and vessels, Clean up pores and vessels.

Close up pores tighten muscles, Improve nutrition absorption and permeability. At the same time, prevent aging and help face reforming.

-Body Purify our body and blood, Smash fat cells and burn the fat have a same effect of arranging then makes our body look more airy-fairy.

## **Specification:**

Measurement: 26\*25\*16cm

Power: 13.5w

Gross Weight: 2KGS

Voltage: 220-240v

Specification: 110v/220v Fre.50 HZ/60HZ