

Instruction of NF-600 Multi-function Anti-pressure Slimming Machine

Preface

I. Function and Theory:

The specially designed vacuum pump can quickly improve size and shape of breast, without any aching and health threat, it does not involve silicon injection or pushing and filling bra, it would make you have natural and pretty breast according to physiological period.

1. The vacuum pump can absorb fat particle and liquid, and then make them enter breast tissue and cells, leave breast more developed.

2. Expulse toxin and purify lymphatic system, refine skin pore, re-build skin, improve micro-circulation, enhance collagen elasticity, lift drooping muscle, resist Wrinkle, activate skin, remove eye bag and double jaw, eliminate wrinkle and telangiectasis, and treat sub-health.

3. Stimulate pituitary to produce hormone, enhance muscle tissue elasticity and fiber, resume breast elasticity

and softness, lift and develop breast, improve breast shape, leave improve suck nipple, stabilize treatment effect.

4. Expedite fatty acid transformation, safely and effectively remove excessive fat, reduce weight, slim up body, eliminate excessive fat on belly, improve leg shape, firm up skin, it can be applied for scrub, point percussion therapy, lymphatic detoxification, treat cellulites, improve buttocks to realize perfect body contour.

III Instruction:

(I) Breast Development

1. Choose breast cup according to customers' needs, connect the cup and aspirator socket.

2. Let customers half-sleep to stretch out breast, cover breast with plastic cloth, don't apply on people with sunken breast.

3. Turn the air pressure knob to the highest point in the cup, turn the strength knob to the lowest point, then press" to choose inspiration and turn the rhythm knob to "breast lifting" (or press body rhythm 1, body rhythm 2 and breast lifting alternately).

4. Put the cup on breast, and leave breast and the cup closely touched. For people with different breast sizes, it is suggested to apply the instrument on smaller side for 5

times before applying on two sides together.

5. Adjust strength knob from “weak” to “strong”, and then adjust air pressure until customers feel comfortable.

6. After 20 minutes, turn the air pressure knob to the highest point, turn the strength knob to the lowest point to take off the cup, the apply on breast lifting cream.

(II) Rhythm Massage Treatment (Fingertip Moving)

1. Choose appropriate type “moving fingertip”, connect the cup and inspiration socket with flexible tube.

2. Turn the air pressure knob to the highest point in the cup, turn the strength knob to the lowest point, then press “power on” and “functional choice” to choose inspiration and turn the rhythm knob to “face rhythm” or “body rhythm” (or press body rhythm 1, body rhythm 2 alternately, face rhythm I and 2 alternately).

3. Adjust strength knob from “weak” to “strong”, and then adjust air pressure until customers feel comfortable.

4. After 20 minutes, turn the air pressure knob to the highest point, and turn the strength knob to the lowest point to take off the moving finger.

(III) Scrub Treatment (Gourd Cup)

1. Choose appropriate type cup, connect the cup and inspiration socket with flexible tube.

2. Turn the air pressure knob to the highest point in the

cup, turn the strength knob to the lowest point, then press “power on” and “functional choice” to choose inspiration and turn the rhythm knob to “treatment”.

3. Adjust strength knob from “weak” to “strong”, and then adjust air pressure until customers feel comfortable.

(IV) Gas Cup (Long-necked Cup)

1. Choose appropriate type cup, connect the cup and inspiration socket with flexible tube.

2. Turn the air pressure knob to the highest point in the cup; turn the strength knob to the lowest point, then press “power on” and “functional choice” to choose inspiration.

3. Adjust strength knob from “weak” to “strong”, and then adjust air pressure until customers feel comfortable.

(V) Attention:

1. Keep the machine level to avoid shaking and collision.

2. Turn off the power and pull out plug when washing the machine.

3. People with serious heart disease, diabetes mellitus and hepatitis are not suitable for treatment with the machine.

4. Women during menstruation and pregnancy are not suitable for treatment.

5. Scrub time should be controlled within 30 minutes with appropriate strength that customers feel comfortable.

6. Don't apply the machine half hour before or after meals, people with empty stomach, fatigue; insufficient sleeping and weak health are not suitable for treatment.

7. People with and blood productivity and spontaneous hemorrhage are not suitable for treatment, including hemophilia, purpuric disease and leukemia.

8. People with serious skin allergic symptoms or acariasis are not suitable for treatment.

9. People with malignant skin tumor or partial skin cancer, fracture, varicosity, partial skin elasticity loss are not suitable for treatment. People in drunk, thirsty or hungry situation should pay special attention.

10. Wash filter regularly to ensure the machine effect and lifetime.

11. When pairing with essential oil, don't apply too much essential oil or cosmetics to avoid damage the machine.

12. After application of the machine, turn the air pressure to the lowest point.

13. Don't apply lymphatic detoxifying treatment when you are annoyed with lymphadenitis.

14. Patients with myocardial infarction, phlebitis depravation and intravenous tumor are not suitable for treatment.

15. Don't apply the treatment within three weeks after surgery for hyperthyroidism, pregnant women.

16. Patient with serious heart disease, diabetes mellitus and hepatitis are not suitable for scrub treatment.

17. Women during menstruation are not suitable for scrub treatment. Pregnant women can have light scrub treatment to promote blood circulation.

18. The scrub time shall not be too long, about half hour is ok with appropriate strength that customers feel comfortable.

19. Don't apply the treatment half hour before or after meals, people with empty stomach, fatigue; insufficient sleeping and weak health are not suitable for scrub treatment.

20. After scrub treatment, have a glass of warm water after test for a while, and then have warm bath to enhance the effect.

21. You would feel a little bit aching one or two days after scrub treatment that is normal and would automatically disappear.