
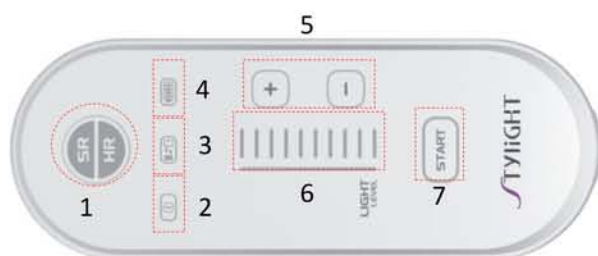


# Chapter 1 Product Introduction

## 1.1 Parts Introduction

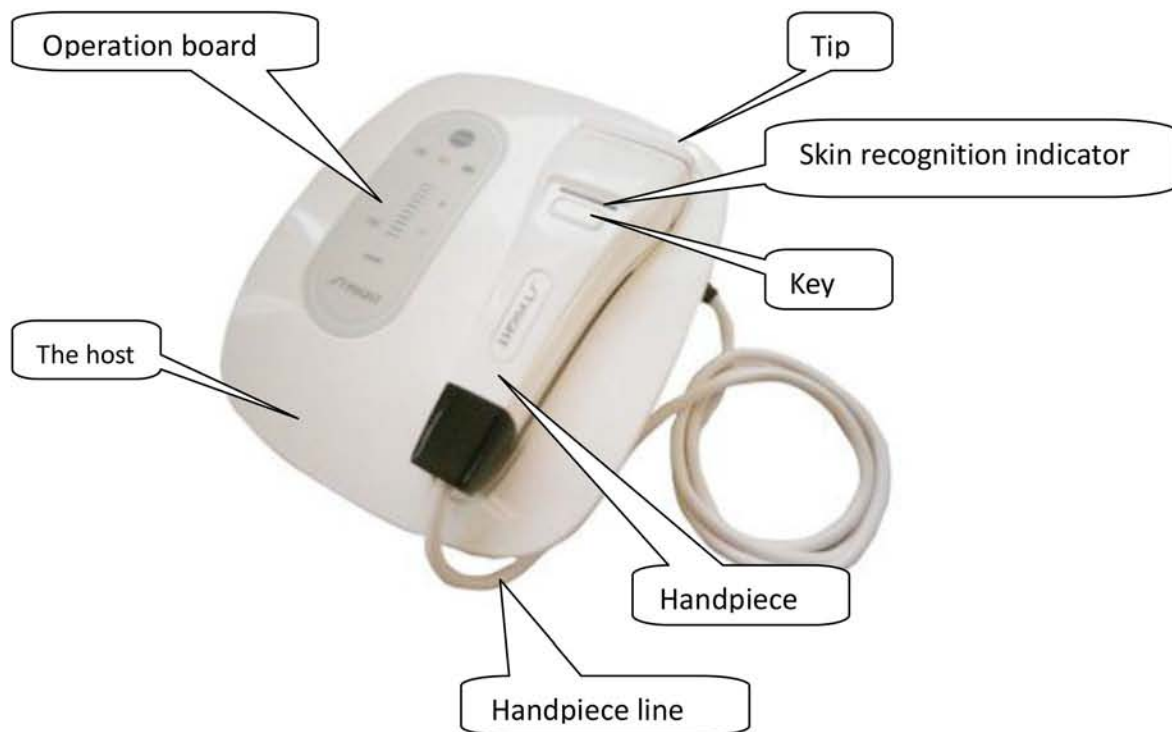

- 1、 Mode indicator
- 2、 Power supply light
- 3、 Tip life indicator
- 4、 Charging indicator
- 5、 Energy level adjusting
- 6、 Energy level
- 7、 Standby/Ready

# Chapter 2 Installation


## 2.1 Connect Power Line

Connect power line to the host (following picture):



## 2.2 Change Tip

1) Connect A with A1 and connect B with B1 (following picture)

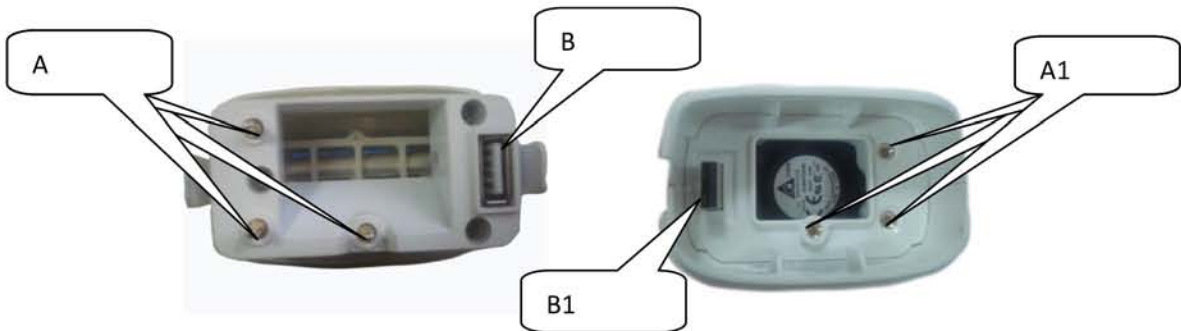

2) The change is finished (following picture)



# Chapter 3 Operation

## 3.1 Start Device



- 1) Connect power supply; indicator light is red and SR or HR will start;



## 3.2 Operation Board

- 1) **Mode selection:** Connect different tips, the equipment will change to relative working mode automatically (SR light means skin rejuvenation mode, HR light means hair removal mode)


- 2) **Standby/Ready selection:** Press  key to change to Standby or Ready

- 3) Energy level adjustment (Ready situation): Press  /  key to adjust energy level

## 3.3 Handpiece Operation



(This is the light in handpiece and the key in handpiece)

- Start the equipment and press  ;





c) Deep color skin shall use low energy level;

## 2) Note during operation

- 1) Long hair can absorb more light energy and this may hurt skin. So before hair removal, cut off the longer hair (leave hair 2~3mm long) ;
- 2) Forehead, upper lip, chin —Skin is very thin, so use low energy level; Customer may feel a little pain during hair removal;
- 3) Make the tip fully touch skin to ensure skin absorb enough light and heat;
- 4) During HR operation mode, delay between two treatments shall be 30s

## 4.3 Operation Procedure

- 1) Clean skin;
- 2) Start equipment and adjust relative energy level;
- 3) We advise users to wear goggles;
- 4) HR mode: first make test on skin with no hair: if skin turns a little red and a little heat, it means the energy level you set is available, then you can remove hair;
- 5) SR mode: first make test on skin under jawbone: if skin turns a little red and a little heat, it means the energy level you set is available, then you can treat with face from the bottom up;
- 6) Make treatment on entire parts; During treatment watch skin's reflection for adjusting in time;
- 7) Clean treated parts after treatment;
- 8) Treated skin needs moisturizing and avoid sunshine

### Suggestion :

- A) Before treatment, smear some beauty products on skin with moisturizing , whitening or renovating functions according to skin's situation;
- B) After treatment, use some beauty products on skin with moisturizing , whitening or renovating functions to make skin smooth and shiny

## 4.4 Result

After treatment if skin turns a little red, a little heat and last for long, a little pain, hair is curve, it means the treatment is successful;  
Normally each treatment needs 1-2 times' continual shots.

## 5.5 Discomfortable Feeling

During treatment, customer may feel differently;

Some may feel himself is bit by small mosquito;  
After hair removal, skin may feel hot for an hour

## 4.6 Notes after Treatment

1. If there are blain and ulceration coming into being in the skin after the treatment, it is recommended to use antibiotic ointment and anti-burn ointment to treat.
2. Customer can use high multiple sunscreen and keep the skin moistly.
3. After treatment, skin around the treated parts may be fragile, if this, customer shall not make up and avoid friction (since it may cause skin hurt)