



MS-4272



MS-4271

MS-4271/4272

Instructions

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆ How to contact us?

Phone: (+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!


Table of Contents:

- 1. Introduction to Manual
- 1.1 Purpose
- 1.2 Conventions
- 2. Medical Safety.....
- 2.1. Indications for Use.....
- 2.2. Contraindications
- 2.3. Precautions
- 2.4. Patient Safety.....
- 2.5. Potential Side Effects.....
- 2.6. Complaints and Adverse Events.....
- 3. System Overview.....
- 3.1. System Description.....
- 3.2. System Components and Features.....
- 4. Specification.....
- 5. Package Listing.....
- 6. Treatment Skills and Operation guide
- 6.1. Treatment Skills.....
- 6.2. Operation guide




1. Introduction to Manual

1.1 Purpose

This instructions for use manual provides a description of the system components, its controls and displays, instructions for its operation, and other equipment information important to the user

-  **Warning:** Do Not operate this machine before reading this manual thoroughly. In addition to this manual,additional clinical training may be available by the company or your local distributor.
For more information on training available please contact your local representative.

1.2 Conventions

-  **Note:** Notes designate information of special interest.
-  **Caution:** Cautions alert the user to precautionary steps necessary to properly operate the system.Failure to observe these cautions may void the warranty
-  **Warning:** Warnings alert the user to information that is of the highest importance and vital to the safety of the patient and user

All procedures are broken down by numbered steps.Steps must be completed in the sequence they are presented

Bulleted lists indicate general information about a particular function or procedure.They do not imply a sequential procedure.

Control names are spelled as they are on the system,and they appear in **Bold text**

2. Medical Safety

2.1. Indications for Use

The Hydro+Dermabrasion system is indicated for use as a non-invasive dermatological aesthetic treatment to:

- Skin deep cleaning, remove blackhead, hoary head, acarid

- Dispel folliculitis, eliminate and enhance rough pore, acne and scar

2.2. Contraindications

The Hydro+Dermabrasion system is contraindicated for use in patients with:

- Open wounds or lesions on the face and/or neck
- Severe or cystic acne on the face and/or neck

2.3. Precautions

The Hydro+Dermabrasion system has not been evaluated for use over various materials. Therefore, treatment is not recommended directly over those areas with any of the following:

- Mechanical implants
- Dermal fillers
- Implanted electrical devices in the face and/or neck
- Metal stents in the face and/or neck area

Treatment energy is not recommended for use directly on an existing keloid.

The Hydro+Dermabrasion system has not been evaluated for use in patients on an anticoagulant treatment plan

It is recommended that the following areas should be avoided during treatment:

- Thyroid gland, thyroid cartilage and trachea
- Major vessels

The Hydro+Dermabrasion system has not been evaluated for use in the following patient populations:

- Pregnant or breast feeding women
- Children
- Those with the following disease states
 - A hemorrhagic disorder or hemostatic dysfunction
 - An active systemic or local skin disease that may alter wound healing
 - Herpes simplex
 - Autoimmune disease
 - Diabetes
 - Bell's palsy

2.4. Patient Safety



Warning: Hydro+Dermabrasion should not be used on a patient's eyes or in a location or technique where ultrasound energy can reach the eye



Warning: Use this system only if you are trained and qualified to do so.



Warning: If any problems occur during system operation, ask immediate action(s): lift the transducer off the patient's skin, press the STOP button to completely halt system operation.

2.5. Potential Side Effects

Side effects reported in the clinical evaluation of the Hydro+Dermabrasion system were mild and transient in nature. These were limited to:

- Erythema (redness): The treated area may exhibit erythema immediately following Treatment. This typically resolves within a few hours of treatment.
 - Edema (swelling)rythema (redness): The treated area may exhibit erythema immediately following Treatment. This typically resolves within a few hours of treatment.
 - Pain: Momentary discomfort may be experienced during the procedure while energy is being deposited. Post procedure discomfort or tenderness to the touch is also possible.
 - Bruising: Mild bruising, which is caused by damage to soft tissue blood vessels, may occur occasionally and typically resolves within a few days of treatment.
 - Nerve Effects:
 - Transient local muscle weakness may result after treatment due to inflammation of a motor nerve.
 - Transient numbness may result after treatment due to inflammation of a Sensory nerve.
 - Transient pain, paresthesia and/or tingling may be experienced
- No permanent injuries to facial nerves have been reported
- Scarring: The possibility for scar formation (which may respond to medical care) May exist if incorrect treatment technique is used

3. System Overview

3.1 System Description

Hydro+Dermabrasion beauty system uses an advanced new hydro-dermabrasion facial technology, which can reach the efficacy as following:

- Deep cleaning can help gently exfoliate stratum corneum, improve skin fatigue and stress due to rough phenomenon. result in get a health natural beauty skin
- Remove blackhead, hoary head, acarid

- Renovates skin with effects of whitening, moisturizing and lifting
- Dispel folliculitis
- Eliminate and enhance rough pore, acne and scar

3.2 System Components and Features

3.2.1 Control Unit & Handpiece



Fig 1.0 control unit



Fig 2.0 control unit

3.2.2 Accessories connecting/disconnecting



Fig 3.0 Hydro facial tips



Fig 4.0 Hydro facial installation

Tip	Size	Function	Size	Function
Focus bevel head	M	Red swollen&pimple	L	Suitable for delicate skin cleansing + hydrating
Straight line bevel head	M	acne vulgaris	M	Second cleaning,suitable for large area with red and swollen acne
Plate suction head	S	Purulent blackhead acne	M	Suitable for large area with purulent blackhead acne
spiral bevel head	S	Obstinate blackhead acne	L	Suitable for second cleaning



3 x Diamond wands

Fig 5.0 Dermabrasion tips installation

D120 D180	Body	Especially for acne,knee,elbow areas and striae of pregnancy
D180 D240	Face	For the larger blemishes
D120 D180 D240	Face&body	Scars, fine lines and imperfections
D200	Eyes and both sides of nose,	For wrinkles,oil tablets, blackhead and acne
D240	Eyes and both sides of nose	For wrinkles,oil tablets, blackhead and acne

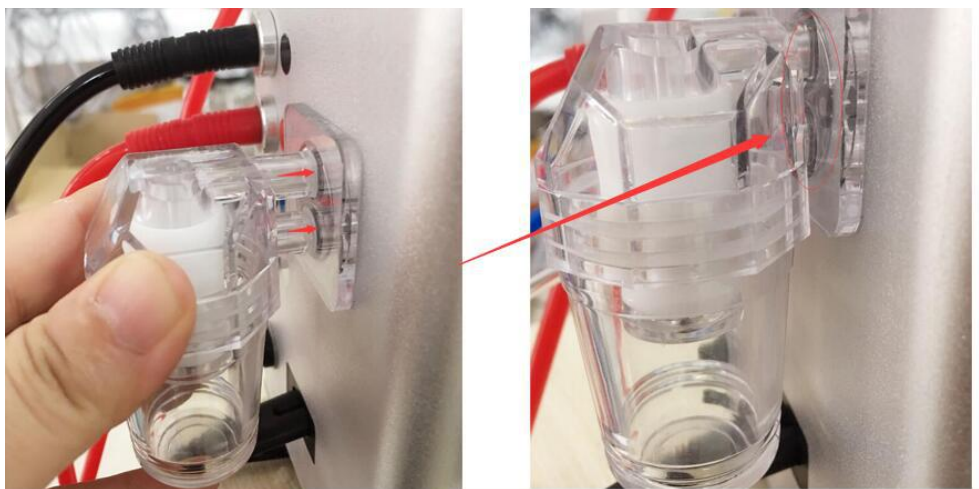


Fig 6.0 Vacuum filter installation



Fig 7.0 Fresh&waste tank installation



Fig 8.0 Water sprayer jet installation

4. Specification

Voltage: 220v/110v

Power: >45w

Vacuum 0--75cmHg

Tank capacity: 100ml

Machine size: 28*29*13cm

5. Package Listing

- 1 x Hydro-demabrasion machine
- 1 x Waste bottle
- 1 x Fresh bottle
- 1 x Spray jet
- 1 x Red tube for spray jet
- 3 x Dermabrasion wands
- 1 x Black tube for dermabrasion wand
- 8 x Hydro tips
- 1 x Hydro wand
- 1 x Holder for accessories
- 1 Packet cotton filter
- 1 Packet seal ring
- 1 x Power supply cord

6. Treatment Skills and Operation guide

6.1 Treatment Skills

6.1.1 Deep Cleaning

6.1.1.1 Course of treatment recommended

1st Course of Treatment Recommended

Hydro +Dermabrasion facial cleaning: 5 sessions

Ultrasonic facial care: 5 sessions

RF Skin lifting : 5 sessions

Second Course of Treatment Recommended

Hydro +Dermabrasion facial cleaning: 5 sessions

RF Skin lifting : 3 sessions

Bio skin tightening: 2 sessions

1 st Course of Treatment Recommended			
	MON		THU
1 Session	Hydro+Dermabrasion	2 Session	Ultrasonic facial care
3 Session	Hydro+Dermabrasion	4 Session	Ultrasonic facial care
5 Session	Hydro+Dermabrasion	6 Session	Ultrasonic facial care
7 Session	Hydro+Dermabrasion	8 Session	RF Skin lifting
9 Session	Hydro+Dermabrasion	10 Session	RF Skin lifting
Second Course of Treatment Recommended			
1 Session	Hydro+Dermabrasion	2 Session	RF Skin lifting
3 Session	Hydro+Dermabrasion	4 Session	RF Skin lifting
5 Session	Hydro+Dermabrasion	6 Session	RF Skin lifting
7 Session	Hydro+Dermabrasion	8 Session	Bio skin tightening
9 Session	Hydro+Dermabrasion	10 Session	Bio skin tightening

6.1.1.2 Skin care cycle

10 sessions / course of treatment

2 courses of treatment are needed

hydro+dermabrasion skin treatment:

Oil skin, big pore skin, black pore skin,thick stratum corneum skin:

2 sessions /week for earlier 4 weeks

Normal skin : 1 session/week

Sensitive skin: 1 session /10 days

The course of treatment includes reinforcement stage of 10 sessions and consolidation of 10 sessions, adds up to 20 sessions:

Reinforcement stage: earlier 10 sessions, 2 session/week Consolidation stage: later 10 sessions, 1 session/7-15 days

The course can be adjusted according to the skin improvement during the course of treatment.

6.1.1.3 Skin care process:

Hydro +Dermabrasion facial cleaning

Step 1. Makeup removal+face wash+toner work time:5 minutes

Step 2. Warm spray work time:3-5 minutes

Cold spray(sensitive skin)

Step 3. Hydro facial deep cleaning work time: 20 minutes

AHA normal skin

BHA sensitive skin

Step 4. Dermabrasion exfoliation work time: 5 minutes

Step 5. Firming moisturizing essence by spray jet work time: 1 minute

Step 6 . Soothing gel/mask+LED photon, select LED photon, use a point-by point

moving method, especially for treatment area, work time should last for about 10-15 minutes

Step 7. Moisturizing mask work time: 10minutes

Step 8. Basic skin care

Ultrasonic facial care

Step 1. Makeup removal+face wash+toner work time: 5 minutes

Step 2. Warm spray work time 3-5 minutes

Cold spray(sensitive skin)

Step 3. **Ultrasound deep cleaning with facial wash**

work time 3 minutes (eg model:SR-CM02)

Ultrasound deep cleaning work time: 20minutes

Step 4. **Import HA essence** by Ultrasound device (eg model:SR-CM02)

Step 5. Soothing gel/mask+LED photon,select LED photon, use a point-by point moving method, especially for treatment area, work time should last for about 10-15 minutes (eg model:SR-CM02)

Step 6. Moisturizing mask work time: 10minutes

Step 7. Basic skin care

RF Skin lifting

Step 1. Makeup removal+face wash+toner work time: 5 minutes

Step 2. Warm spray work time: 3-5 minutes

Cold spray(sensitive skin)

Step 3. **Ultrasound deep cleaning with facial wash**

work time 3 minutes (eg model:SR-CM02)

Ultrasound deep cleaning work time: 20minutes

Step 4. **Import HA essence** by RF device (eg model: NM-SN6)

Step 5. Soothing gel/mask+LED photon,select LED photon, use a point-by point moving method, especially for treatment area, work time should last for about 10-15 minutes (eg model: NM-SN6)

Step 6. Moisturizing mask work time: 10minutes

Step 7. Basic skin care

Bio skin tighteing: 2 sessions

Step 1. Makeup removal+face wash+toner work time: 5 minutes

Step 2. Warm spray 3-5 work time: 5 minutes

Cold spray(sensitive skin)

Step 3. **Ultrasound deep cleaning with facial wash**

work time 3 minutes (eg model:SR-CM02)

Ultrasound deep cleaning work time: 20minutes

Step 4. Import HA essence by bio device (eg model: SR-MACL263)

Step 5. Soothing gel/mask+LED photon by bio device, select LED photon, use a point-by point moving method, especially for treatment area, work time should last

- for about 10-15 minutes (eg model : SR-MACL263)
- Step 6. Moisturizing mask work time: 10minutes
- Step 7. Basic skin care

6.1.2 Remove blackhead

6.1.2.1 Course of treatment recommended

1st Course of Treatment Recommended
 Hydro +Dermabrasion facial cleaning: 5 sessions
 Ultrasonic facial care: 5 sessions
 RF Skin lifting : 5 sessions

Second Course of Treatment Recommended
 Hydro +Dermabrasion facial cleaning: 5 sessions
 RF Skin lifting : 3 sessions
 Bio skin tightening: 2 sessions

1 st Course of Treatment Recommended			
	MON		THU
1 Session	Hydro+Dermabrasion	2 Session	Ultrasonic facial care
3 Session	Hydro+Dermabrasion	4 Session	Ultrasonic facial care
5 Session	Hydro+Dermabrasion	6 Session	Ultrasonic facial care
7 Session	Hydro+Dermabrasion	8 Session	RF Skin lifting
9 Session	Hydro+Dermabrasion	10 Session	RF Skin lifting
Second Course of Treatment Recommended			
1 Session	Hydro+Dermabrasion	2 Session	RF Skin lifting
3 Session	Hydro+Dermabrasion	4 Session	RF Skin lifting
5 Session	Hydro+Dermabrasion	6 Session	RF Skin lifting
7 Session	Hydro+Dermabrasion	8 Session	Bio skin tightening
9 Session	Hydro+Dermabrasion	10 Session	Bio skin tightening

6.1.2.2 Skin care cycle

10 sessions / course of treatment
 2 courses of treatment are needed
 hydro+dermabrasion skin treatment:
 Oil skin, big pore skin, black pore skin,thick stratum corneum skin:
 2 sessions /week for earlier 4 weeks
 Normal skin : 1 session/week
 Sensitive skin: 1 session /10 days
 The course of treatment includes reinforcement stage of 10 sessions and consolidation of 10 sessions, adds up to 20 sessions:
 Reinforcement stage: earlier 10 sessions, 2 session/week Consolidation stage: later

10 sessions, 1 session/7-15 days

The course can be adjusted according to the skin improvement during the course of treatment.

6.1.2.3 Skin care process:

Hydro +Dermabrasion facial cleaning

Step 1. Makeup removal+face wash+toner work time: 5 minutes

Step 2. Warm spray work time: 3-5 minutes

Cold spray(sensitive skin)

Step 3. Apply blackheads-removing lotion on nose with makeup cotton for about 5 minutes

Step 4. Hydro facial deep cleaning work time: 20 minutes

AHA normal skin

BHA sensitive skin

Step 5. PH balancing essence by spray jet Work time : 1minute

Step 6. Soothing gel/mask+LED photon, select LED photon, use a point-by point moving method, especially for treatment area, work time should last for about 10-15 minutes (eg model:SR-CM02)

Step 7. Moisturizing mask work time: 10minutes

Step 8. Basic skin care

RF Skin lifting

Step 1. Makeup removal+face wash+toner work time: 5 minutes

Step 2. Warm spray work time: 3-5 minutes

Cold spray(sensitive skin)

Step 3. [Ultrasound deep cleaning with facial wash](#)
work time 3 minutes (eg model:SR-CM02)

[Ultrasound deep cleaning](#) work time: 20minutes

Step 4. Import firming moisturizing essence by RF device (eg model : NM-SN6)

Step 5. Soothing gel/mask+LED photon, select LED photon, use a point-by point moving method, especially for treatment area, work time should last for about 10-15 minutes (eg model : NM-SN6)

Step 6. Moisturizing mask work time: 10 minutes

Step 7. Basic skin care

Bio skin tighteing: 2 sessions

Step 1. Makeup removal+face wash+toner work time: 5 minutes

Step 2. Warm spray work time: 3-5 minutes

Cold spray(sensitive skin)

Step 3. [Ultrasound deep cleaning with facial wash](#)
work time 3 minutes (eg model:SR-CM02)

[Ultrasound deep cleaning](#) work time: 20minutes

- Step 4. Import HA essence by bio device (eg model: SR-MACL263)
- Step 5. Soothing gel/mask+LED photon,select LED photon, use a point-by point moving method, especially for treatment area, work time should last for about 10-15 minutes (eg model : SR-MACL263)
- Step 6. Moisturizing mask work time: 10 minutes
- Step 7. Basic skin care

6.1.3. Renovates skin

6.1.3.1 Course of treatment recommended

1st Course of Treatment Recommended

Hydro +Dermabrasion facial cleaning: 5 sessions

RF Skin lifting : 5 sessions

Second Course of Treatment Recommended

Hydro +Dermabrasion facial cleaning: 5 sessions

RF Skin lifting : 3 sessions

Bio skin tightening: 2 sessions

1 st Course of Treatment Recommended			
	MON		THU
1 Session	Hydro+Dermabrasion	2 Session	RF Skin lifting
3 Session	Hydro+Dermabrasion	4 Session	RF Skin lifting
5 Session	Hydro+Dermabrasion	6 Session	RF Skin lifting
7 Session	Hydro+Dermabrasion	8 Session	RF Skin lifting
9 Session	Hydro+Dermabrasion	10 Session	RF Skin lifting
Second Course of Treatment Recommended			
1 Session	Hydro+Dermabrasion	2 Session	RF Skin lifting
3 Session	Hydro+Dermabrasion	4 Session	RF Skin lifting
5 Session	Hydro+Dermabrasion	6 Session	RF Skin lifting
7 Session	Hydro+Dermabrasion	8 Session	Bio skin tightening
9 Session	Hydro+Dermabrasion	10 Session	Bio skin tightening

6.1.3.2 Skin care cycle

10 sessions / course of treatment

2 courses of treatment are needed

hydro+dermabrasion skin treatment:

Oil skin, big pore skin, black pore skin,thick stratum corneum skin:

2 sessions /week for earlier 4 weeks

Normal skin : 1 session/week

Sensitive skin: 1 session /10 days

The course of treatment includes reinforcement stage of 10 sessions and consolidation of 10 sessions, adds up to 20 sessions:

Reinforcement stage: earlier 10 sessions, 2 session/week Consolidation stage: later 10 sessions, 1 session/7-15 days

The course can be adjusted according to the skin improvement during the course of treatment.

6.1.3.3 Skin care process:

Hydro +Dermabrasion facial cleaning

Step 1. Makeup removal+face wash+toner work time: 5 minutes

Step 2. Warm spray work time: 3-5 minutes

Cold spray(sensitive skin)

Step 3. Hydro facial deep cleaning work time: 20 minutes

AHA normal skin

BHA sensitive skin

Step 5. Apply liquid revitalizing essence with spray jet work time: 5 minutes

Step 6. Age control gel/mask+LED photon, select LED photon, use a point-by point moving method, especially for treatment area, work time should last about 10-15 minutes (eg model : SR-MACL263)

Step 7. Refreshing facial mask work time: 10minutes

Step 8. Basic skin care

RF Skin lifting

Step 1. Makeup removal+face wash+toner work time: 5 minutes

Step 2. Warm spray work time: 3-5 minutes

Cold spray(sensitive skin)

Step 3. **Ultrasound deep cleaning with facial wash**

work time 3 minutes (eg model:SR-CM02)

Ultrasound deep cleaning work time: 20 minutes

Step 4. Import Revitalizing essence by RF device (eg model : NM-SN6)

Step 5. Age control gel/mask+LED photon, select LED photon, use a point-by point moving method, especially for treatment area, work time should last about 10-15 minutes (eg model : NM-SN6)

Step 6. Refreshing facial mask work time: 10minutes

Step 7. Basic skin care

Bio Skin Tightening: 2 sessions

Step 1. Makeup removal+face wash+toner work time: 5 minutes

Step 2. Warm spray work time: 3-5 minutes

Cold spray(sensitive skin)

- Step 3. **Ultrasound deep cleaning with facial wash**
work time 3 minutes (eg model:SR-CM02)
Ultrasound deep cleaning work time: 20 minutes
- Step 4. Refreshing facial mask work time: 10mins
- Step 5. Age control gel/mask+LED photon, select LED photon, use a point-by point moving method, especially for treatment area, work time should last about 10-15 minutes (eg : SR-MACL263)
- Step 6. Refreshing facial mask work time: 10minutes
- Step 7. Basic skin care

6.1.4 Dispel folliculitis

6.1.4.1 Course of treatment recommended

Hydro +Dermabrasion facial cleaning: 5 sessions MTS skin care 5 session
MTS Skin care : 1 session

Course of Treatment Recommended			
	MON		THU
1 Session	MTS+Hydro+Dermabrasion	2 Session	RF Skin lifting
3 Session	Hydro+Dermabrasion	4 Session	RF Skin lifting
5 Session	Hydro+Dermabrasion	6 Session	RF Skin lifting
7 Session	Hydro+Dermabrasion	8 Session	RF Skin lifting
9 Session	Hydro+Dermabrasion	10 Session	RF Skin lifting

6.1.4.2 Skin care cycle

10 sessions / course of treatment
2 courses of treatment are needed
hydro+dermabrasion skin treatment:
Oil skin, big pore skin, black pore skin,thick stratum corneum skin:
2 sessions /week for earlier 4 weeks
Normal skin : 1 session/week
Sensitive skin: 1 session /10 days
The course of treatment includes reinforcement stage of 10 sessions and consolidation of 10 sessions, adds up to 20 sessions:
Reinforcement stage: earlier 10 sessions, 2 session/week Consolidation stage: later 10 sessions, 1 session/7-15 days
The course can be adjusted according to the skin improvement during the course of treatment.

6.1.4.3 Skin care process:

Step 1. Disinfection+toner work time:1 minute

- Step 2. Warm spray work time: 3-5 minutes
Cold spray(sensitive skin)
- Step 3. Please use micro needle to open the pores, work time: 2-3 minutes
- Step 4. Hydro facial deep cleaning work time: 20 minutes
AHA normal skin
BHA sensitive skin
- Step 5. Dermabrasion exfoliation work time: 5minutes
- Step 6. Soothing essence by spray jet work time: 1 minute
- Step 7. Basic skin care

RF Skin lifting

- Step 1. Cleaning+toner work time: 1 minute
- Step 2. Warm spray work time: 3-5 minutes
Cold spray(sensitive skin)
- Step 3. **Ultrasound deep cleaning with facial wash**
work time 3 minutes (eg model:SR-CM02)
Ultrasound deep cleaning work time: 20 minutes
- Step 4. Import Revitalizing essence by RF device work time: 3 minutes
(eg model: NM-SN6)
- Step 5. Basic skin care

6.2 Operation Guide




1. Neck

Move in small circular motions from the inside of the neck to the outside, then lift from the neck upward to the cheek, neck treatment area can be divided into three lines(up,medium,down areas) to operate
Attention: strength should be top heavier and down lighter, operation electrodes should tight skin, repeat 8-10 times




2.Chin

Make operation head move in small circular motions by pressing tightly against the double chin and then pull back to the back of the ear, repeat 5-8 times

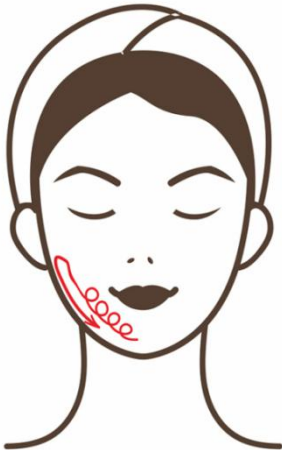


Make operation head move in small circular motions by clinging to the chin and circle out to the ear beads. Then pulled down to the chin with light force,repeat 5-8 times

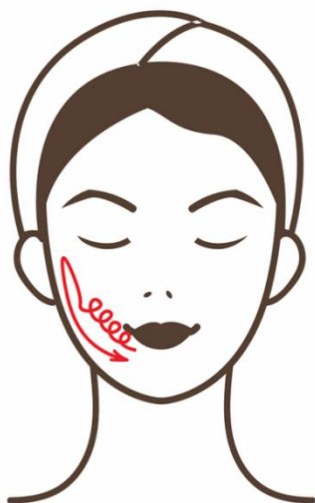


3.Mouth

Make operation head move in small circular motion by clinging to the corner of mouth, and circle out to the ear beads. Then pulled down to the chin with light force,repeat 5-8 times

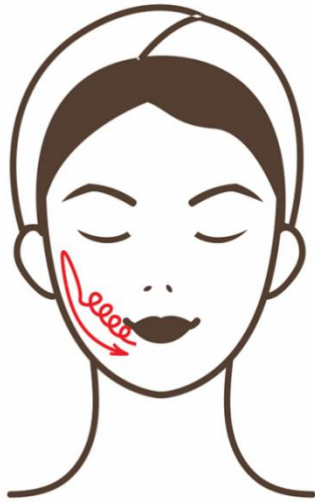

3.Mouth

Make operation head move in small circular motion by clinging to the corner of mouth, and circle out to the ear beads. Then pulled down to the chin with light force,repeat 5-8 times

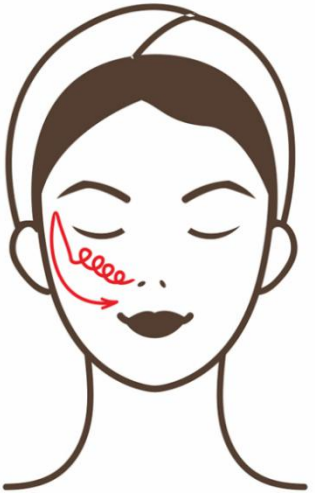


4.Cheek

Make operation head move in small circular motion by closing to the light bone of nose, circle out to the ear beads. Then pulled down to the chin with light force, repeat 5-8 times



5.Forehead

Make operation head move in small circular motion by closing to the light bone of nose, circle out to the ear beads. Then pulled down to the chin with light force, repeat 5-8 times

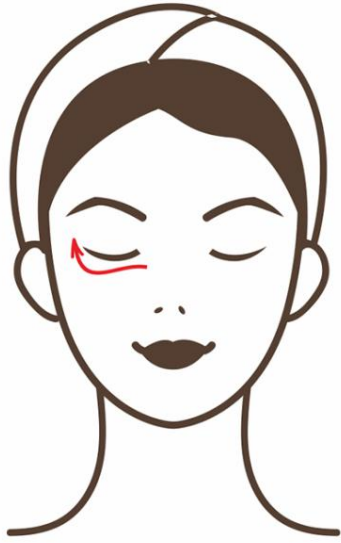

6.Eyes' around

Dredge the meridian and collaterals of eyes,
repeat 5-8 times