



MS-65W1

Instructions

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆ How to contact us?

Phone:(+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!

Introduction

FEATURES:

1. Painless treatment focus RF energy at the correct position, compared to other radio frequency technology. It uses low energy and high frequency, safe and effective.
2. Aim at the skin surface and deep location, using a complex method to control the different currents and energy, direct access to the different skin layer. Does not cause the phenomenon of uneven skin.
3. Selectively targeting fatty tissue, to avoid heating other fat, to achieve the fastest treatment effect.
4. The entire process is completed without the need of surgery and anesthesia.
5. With the most useful 40KHZ ultrasonic cavitation system.
6. Without side effects and the risk of gaining the weight back, the results are rather obvious. Does not affect the normal work and life.

FUNCTIONS:

Firming the arms, legs, thighs, buttocks, waist, back, abdominal muscles and reshape the body figure Improve and treat the orange peel like fatty tissues of different serious levels With the effects of soothing, decompression and pain relief (such as joint pain, sciatica), promote the regular mechanical aerobic exercise of skin tissue, rapidly relieve stress and remove fatigue. Firming, lifting and compact the skin and mould the face shape circulate and activate cells, promote metabolism and so that 100% more collagen and elastic fibers may be gained for improving the facial and body skin

PACKAGE INCLUDING:

Multipolar RF Head for Body x 1
40KHz Cavitation Head x 1
Vacuum Bipolar RF Head x1
Tripolar RF Head for Face and Eyes' Around x 1
Quadrupole RF Head for Face and Body x 1
Cold cryo probe x 1
Power cord x 1

Operation Guide

Accessories



40khz cavitation



Quadrupole rf for face



Sextupole rf for body



Tripolar rf for eye around



Cooling head




Vacuum rf

Please connect all the handles well, and turn on the machine
The will turn to this page:





1. 40Khz Cavitation



Please press  to enter into the cavitation page

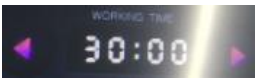

Please click  to set working time.

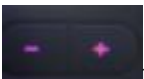Please click  to increase or decrease the ENERGY

Please click  to chose working model.


Please click  to start or pause function.


40Khz Ultrasonic handle-strong sound wave explosion fat head
 Model A is with continuous wave, which is used for the area with thin fat;
 Model B is with the dis-continuous wave, which is mostly used for the area with thick fat.
 Operate with the 40KHZ cavitation head for 15-20 minutes with mainly the spiral movement on body.

2, Tripolar rf operation:

Please press  to enter in to the eye rf page



Please click  to set working time.

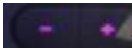Please click  to increase or decrease the ENERGY

Please click  to chose working model.

Notice: when choose NOR mode, the led photon will be turn on immediately, and the energy will reach immediately to the energy you set. When choose PRO mode, only touch the skin with the handle, then the led photo will come on. And the energy will reach the energy you set gradually.

Please click  to start or pause function.


Spread gel or wrinkle smoothing essence on the treated positions for hairdressing(generally 20-30G each time), operate with the facial RF head for 15-20 minutes with mainly the spiral movement on face abd eye around.

3.Quadrupolar RF for face

Please press  to enter into the face rf page



Please click  to set working time.

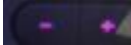Please click  to increase or decrease the ENERGY

Please click  to chose working model.


Please click  to start or pause function.

Use the probe to massage the specific part of skin circularly after applying the sliming cream for about 30 to 40 minutes. It could quicken the blood circulation in the part of skin treated and has the effect of decomposing fat, etc.


Matters needing attention:

1. Apply sliming cream or cooling gel to the part of skin to be treated before the treatment.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Please do not irradiate eyeball directly with colored light.
5. Operate with the tripolar RF head for 15-20 minutes with mainly the spiral movement on face.


4. Sextupolar Body RF handle

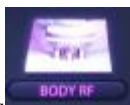


Press  button then login Sextupole RF operation display.

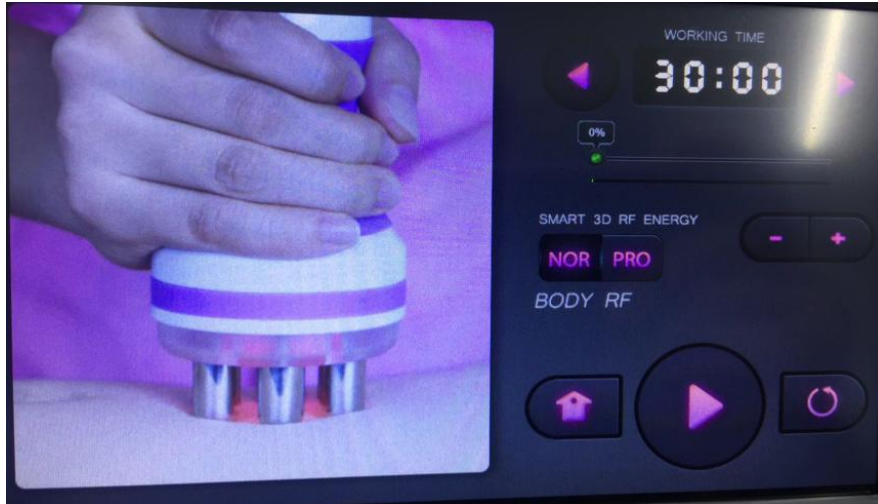

Please click  to set working time.

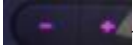Please click  to increase or decrease the ENERGY

Please click  to chose working model.

Please click  to start or pause function.


Apply gel to the negative pressure vacuum RF head, point it to the position for losing weight and operate in pulling technique(drag fat at the position for losing weight to the position where it is closer to the lymph gland) for about 20 minutes.

Notes:

1. Spread gel or essence on the treated positions for hairdressing evenly.
2. In use, start with low-energy. We suggest to try for the proper energy level on the back of clients' hands before using on body of them.
3. Do not use essential oil as the medium.
4. In operation, start pulling only after negative pressure head is attached to skin.
5. Turn off the instrument probe when it is not in use.
- 6 Operate with the tripolar RF head for 15-20 minutes with mainly the spiral movement on body.


5.Vacuum rf head




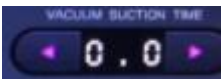
Press  button then login Quadrupole RF Vacuum operation display




Please click  to set working time.

Please click  to increase or decrease the ENERGY

Please click  to rf energy mode. Nor means the energy will increase suddenly
Pro mean the energy will increase gradually.

Please click  and  to adjust the suction and release.

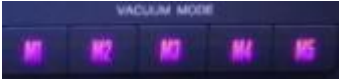Please click  to chose working model.

Please click  to start or pause function.

Use the probe to massage the specific part of skin circularly after applying the sliming cream for about 30 to 40 minutes. It could quicken the blood circulation in the part of skin treated and has the effect of decomposing fat, etc.

Matters needing attention:

1. Apply sliming cream or cooling gel to the part of skin to be treated before the treatment.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Please do not irradiate eyeball directly with colored light.
- 5 Operate with the bipolar vacuum head for 15-20 minutes with mainly the spiral movement on body.

6.Cold probe for calming down the skin and shrinking pores.



Press the red on/off button on the probe to start/pause the probe,operate 5-10 minutes.

See the charts below instructions on how and where to use the probes

FACE

Treated time: 30 minutes



1. Massage the mandible center in circles to produce deep heat.
2. Massage lower jaw in lines.




3. Massage the triangle zone of both sides of the face in circles.
4. Massage from jaw to angulus oris to ear in lines.

THIGH

Treated time: 30 minutes



1. From down to upper, pushing to the groin to dredge the lymph.
2. From down to upper, circling push by anticlockwise can help decomposing

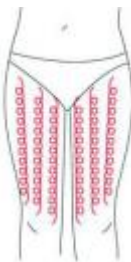

fatness.



3. Also can push by come-and-go to decompose fatness.
4. Pull from the knee and muscle texture to upper, can improve the curve.

Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.




3. Along the muscle of arm, pull up to lift and tight the muscle.

BACK

Treated time: 20 to 30 minutes



1. First by come-and-go, pushing the bladder nerve 2 to 3 times.

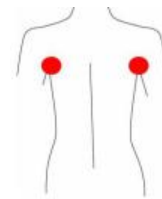


2. Circling by anticlockwise to stimulate the underarm lymph node.



3. On the back, by anticlockwise circling, can help to decompose the fatness.

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help improving the back

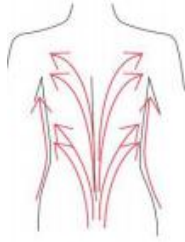

curve.

ARM

Treated time: 30 minutes

BACK SIDE

1. From the inner elbow to the armpit, doing lymph drainage.
2. Circling to stimulate the lymph node.
3. By anticlockwise or come-and-go pushing, decomposing the fatness.
4. From the elbow to the outer, doing tightening gesture.




Facade

1. From the elbow to the outer, doing tightening gesture.


Abdomen

Treated time: 20 to 30 minutes

1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.

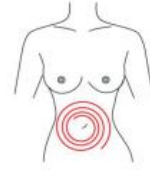


2. From small to large, by anticlockwise direction, circling push down.

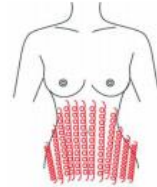

3. From the belly and muscle texture, pull to the groin.



4. Through lymph drainage gesture, taking the toxin to the place of groin.

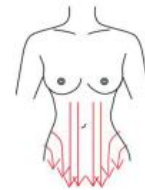

Attention:

1. Pregnant women or women during in menses.
2. Epileptic.
3. Patients with malignancy.
4. Patient whose wound after operation has not healed up.
5. Acute inflammation or epidemical patients.
6. Whom with heart diseases or with heart pacemaker.
7. Who with kidney (gall-stone) disease.
8. Who was embedded metal object or silica gel.
9. Who in menses, birth control period, emiction incontinence period, or accepting the belly operation.
10. Whose body always takes much inner hot.
11. Who has the genetic hypersensitivity.

Notice:

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
7. If the machine will not use for a long time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.


Notice:

1,Please insert the filter to the back of the machine at your first use. Or the vacuum bipolar RF probe wont have suction.



2,please use gel before you use the RF probes, and make sure the probes completed touched with skin, or you will not feel the heat

3,If the vacuum suction is weak, please adjust the following knob:

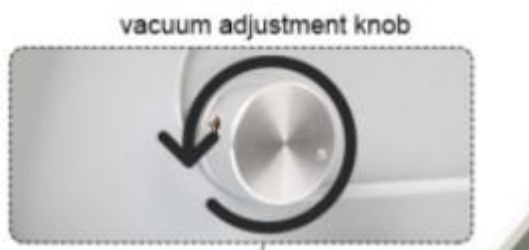

4,If the vacuum probe still dosent have suction after inserting the filter,adjusting the suction,release time,adjusting the vacuum knob, please hear the vacuum pump is working or not. If the vacuum pump is working, please take off the vacuum probe from the machine, then use your finger to touch the vacuum connector on the machine to see whether you can feel the suction(when you test, please make sure the vacuum function is on). If you can feel the suction on the vacuum connector on the machine, it shows it will be the problem of the vacuum probe, then just need to exchange one new vacuum probe;

If you cant feel the suction, please take one video for your operating and testing, then send the person who sale you the machine. Then we solve the problem very fast.