



SR-AF1313

# Instructions

# Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

- ◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

- ◆ How to contact us?

Phone:(+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!


Table of Contents:

- 1. Introduction to Manual .....
- 1.1 Purpose .....
- 1.2 Conventions .....
- 2. Medical Safety.....
- 2.1. Indications for Use.....
- 2.2. Contraindications .....
- 2.3. Precautions .....
- 2.4. Patient Safety.....
- 2.5. Potential Side Effects.....
- 3. System Overview&operation guide .....
- 3.1. Control Unit & Handpiece.....
- 4. Package Listing.....
- 5. Treatment Skills .....

# 1. Introduction to Manual


## 1.1 Purpose


This instructions for use manual provides a description of the system components, its controls and displays, instructions for its operation, and other equipment information important to the user


 **Warning:** Do Not operate this machine before reading this manual thoroughly. In addition to this manual,additional clinical training may be available by the company or your local distributor.

For more information on training available please contact your local representative.

## 1.2 Conventions

 **Note:** Notes designate information of special interest.

 **Caution:** Cautions alert the user to precautionary steps necessary to properly operate the system.Failure to observe these cautions may void the warranty

 **Warning:** Warnings alert the user to information that is of the highest importance and vital to the safety of the patient and user

All procedures are broken down by numbered steps.Steps must be completed in the sequence they are presented

Bulleted lists indicate general information about a particular function or procedure.They do not imply a sequential procedure.

Control names are spelled as they are on the system,and they appear in **Bold text**

## 2. Medical Safety

### 2.1. Indications for Use

This machine is indicated for use as a non-invasive dermatological aesthetic treatment to:

- Hydra Dermabrasion Remove debris from pores and absorb the dirt.
- Sebum removal ,black head removal ,white head removal.
- Increase skin elasticity.
- Face lifting ,Ease dropsy.
- To improve blood circulation and activate collagen by microcurrent.

## 2.2. Contraindications

This machine is contraindicated for use in patients with:

- Open wounds or lesions on the face and/or neck
- Severe or cystic acne on the face and/or neck

## 2.3. Precautions

This machine has not been evaluated for use over various materials. Therefore, treatment is not recommended directly over those areas with any of the following:

- Mechanical implants
- Dermal fillers
- Implanted electrical devices in the face and/or neck
- Metal stents in the face and/or neck area

Treatment energy is not recommended for use directly on an existing keloid.

This machine has not been evaluated for use in patients on an anticoagulant treatment plan

It is recommended that the following areas should be avoided during treatment:

- Thyroid gland, thyroid cartilage and trachea
- Major vessels

This machine has not been evaluated for use in the following patient populations:

- Pregnant or breast feeding women
- Children
- Those with the following disease states
  - A hemorrhagic disorder or hemostatic dysfunction
  - An active systemic or local skin disease that may alter wound healing
  - Herpes simplex
  - Autoimmune disease
  - Diabetes
  - Bell's palsy

## 2.4. Patient Safety



Warning: this machine should not be used on a patient's eyes or in a location or technique where ultrasound energy can reach the eye



Warning: Use this system only if you are trained and qualified to do so.



Warning: If any problems occur during system operation, ask immediate action(s): lift the transducer off the patient's skin, press the STOP button to completely halt system operation.

## 2.5. Potential Side Effects

Side effects reported in the clinical evaluation of the RF system were mild and transient in nature. These were limited to:

- Erythema (redness): The treated area may exhibit erythema immediately following Treatment. This typically resolves within a few hours of treatment.
- Edema (swelling)rythema (redness): The treated area may exhibit erythema immediately following Treatment. This typically resolves within a few hours of treatment.
- Pain: Momentary discomfort may be experienced during the procedure while energy is being deposited. Post procedure discomfort or tenderness to the touch is also possible.
- Bruising: Mild bruising, which is caused by damage to soft tissue blood vessels, may occur occasionally and typically resolves within a few days of treatment.
- Nerve Effects:

Transient local muscle weakness may result after treatment due to inflammation of a motor nerve.

Transient numbness may result after treatment due to inflammation of a Sensory nerve.

Transient pain, paresthesia and/or tingling may be experienced

No permanent injuries to facial nerves have been reported

- Scarring: The possibility for scar formation ( which may respond to medical care) May exist if incorrect treatment technique is used

## 3. System Overview&operation guide


### 3.1 Control Unit & Handpiece




(1) Choice the  first, then press , then the fan  will be

flashing .---this function only can continuous working 5 minutes ,then it will automatic stop working.you just need press this function again to work again.

\*  Bio Micro Current Roller (divided into 9 file)

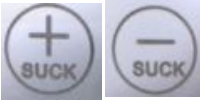
\*  Hydra Dermabreasion handle (divided into 3 file)

\*  Coloured light Psychotherapy- there is six colors for you choice .as:

Red ,Yellow ,Aqua ,Green,Blue,Pink.



(2) There are two bottles, fresh bottle and waste bottle. You can adjust the bottle down, and unscrew, fill with clean water in fresh bottle, discard and clean the waste bottle.

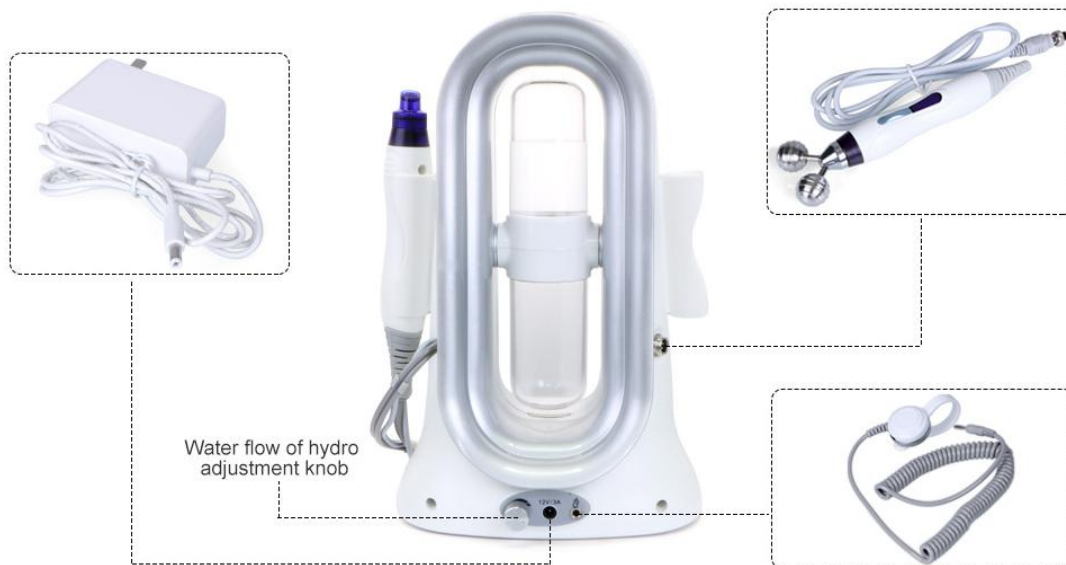

(3) Water flow of hydro adjustment knob.





(4) Bio Micro Current Roller --press the on/off button ,touch the sensing hammer at same time.



(5)Hydra Dermabreasion --press the on/off button.

N.W: 5.84 lbs  
G.W: 7.05 lbs




(6)Size and weight.

#### 4. Package Listing

- 1 x Main Machine
- 1 x Hydra Dermabrasion
- 1 x Bio Micro Current Roller
- 1 x Power Supply Cord

#### 5 Treatment Skills

##### Treatment Skills for Face

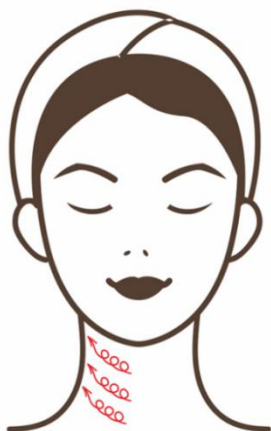


##### 1. Neck

Move in small circular motions from the inside of the neck to the outside, then lift from the neck upward to the cheek, neck treatment area can be divided into three lines(up,medium,down areas) to operate  
Attention: strength should be top heavier and down lighter, operation electrodes should tight skin, repeat 8-10 times



## 2.Chin

Make operation head move in small circular motions by pressing tightly against the double chin and then pull back to the back of the ear, repeat 5-8 times

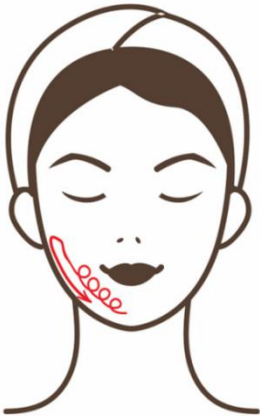


Make operation head move in small circular motions by clinging to the chin and circle out to the ear beads. Then pulled down to the chin with light force,repeat 5-8 times



## 3.Mouth

Make operation head move in small circular motion by clinging to the corner of mouth, and circle out to the ear beads. Then pulled down to the chin with light force,repeat 5-8 times

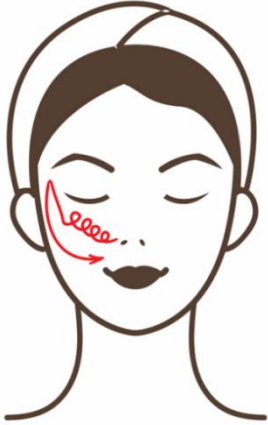

#### 4. Cheek

Make operation head move in small circular motion by closing to the light bone of nose, circle out to the ear beads. Then pulled down to the chin with light force, repeat 5-8 times



#### 5. Forehead

Make operation head move in small circular motion by closing to the light bone of nose, circle out to the ear beads. Then pulled down to the chin with light force, repeat 5-8 times



#### 6. Eyes' around

Dredge the meridian and collaterals of eyes, repeat 5-8 times



Move in small circular motion in the lower eye then lift to the temple, repeat 5-8 times