



SR-AF1315

# Instructions

## Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆ How to contact us?

Phone:(+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!

## Chapter 1: Safety Warnings

### I. Electricity & Machinery Safety

1. Please keep the voltage at stable condition.

### II. Contraindication Safety


1. People with heart disease, high blood pressure, or configured cardiac Pacemaker.
2. People with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
3. People with hemorrhagic diseases, trauma or who is bleeding.
4. Pregnant women
5. Medical plastic artificial or metal parts inside the body such as metal teeth or silicone breast implants.
6. People with abnormal immune system.
7. People with numb or insensitive to heat.
8. If you are feeling unwell, Prohibited in the ears, nose, eye throat and do not use this device in advance and tell your doctor.
9. Children under 12 years old are not allowed.

### III. Packing Including and parameter

#### 1. Packing list:

- 1x main machine
- 1x power cord
- 1x handle display holder
- 1x bio handle
- 1x rf handle
- 1x ultrasonic handle
- 1x cooling handle
- 8x hydro tips



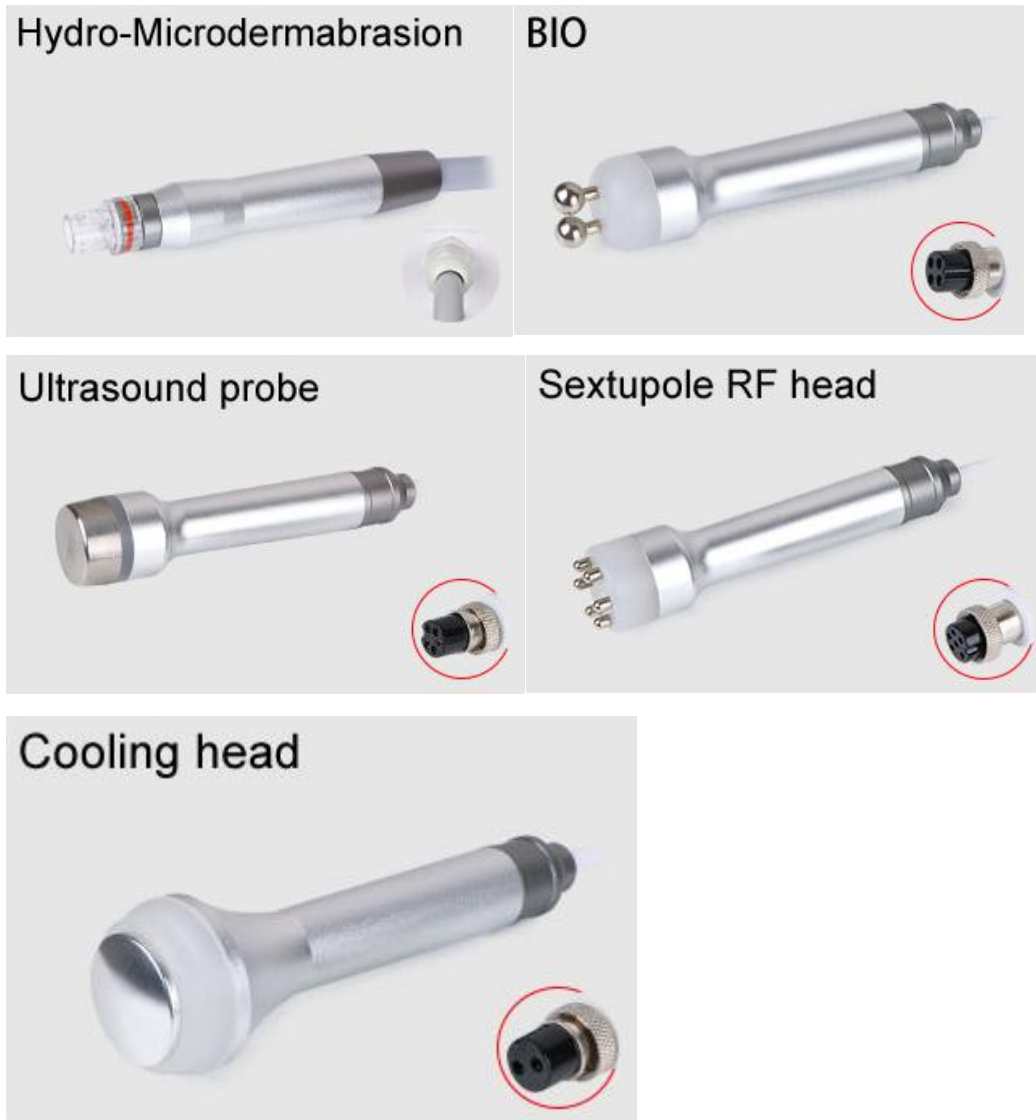

## 2. Parameter:

Working power: 26W

Carton size: 50x41x37cm

## Chapter 2: Before using

**I.** Before use, clean face with lotion and finish massage face to accelerate blood circulation.

**II.** How to install accessories.

1. install all the handles to the machine.



drainage, turn to this side, means valve is close.

turn to this side, turn on the valve of waste water.



3 bottles could removal from base by anticlockwise rotation. ( Notes: avoiding the damage, not suggest to take out the bottles. )



the center of waste water bottle caps have a hole in it, there is no sealing ring in it.

The silicone sealing ring can not be lost which inside the container lid and the container base, avoid the sealing of the container.

2. Put all the handle to the holder stand, just display like below picture show:






3. Put water into clean bottle on the right of the machine:



4. Connect power cord with machine, and then press red bottom to turn on the machine.

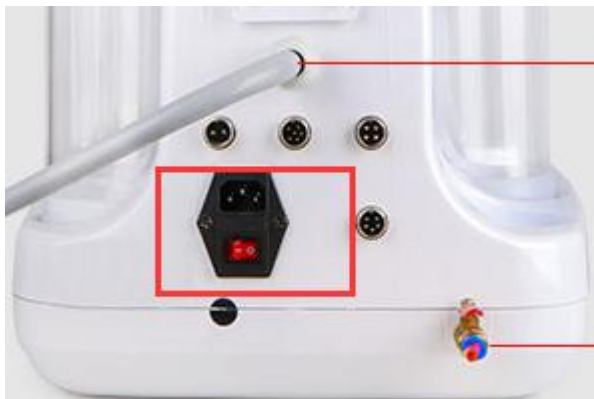



3. Press the power button to start/finish treatment.


## Chapter 3: program setting

### I. Menu setting


1. When turn on the machine, menu will show like this:

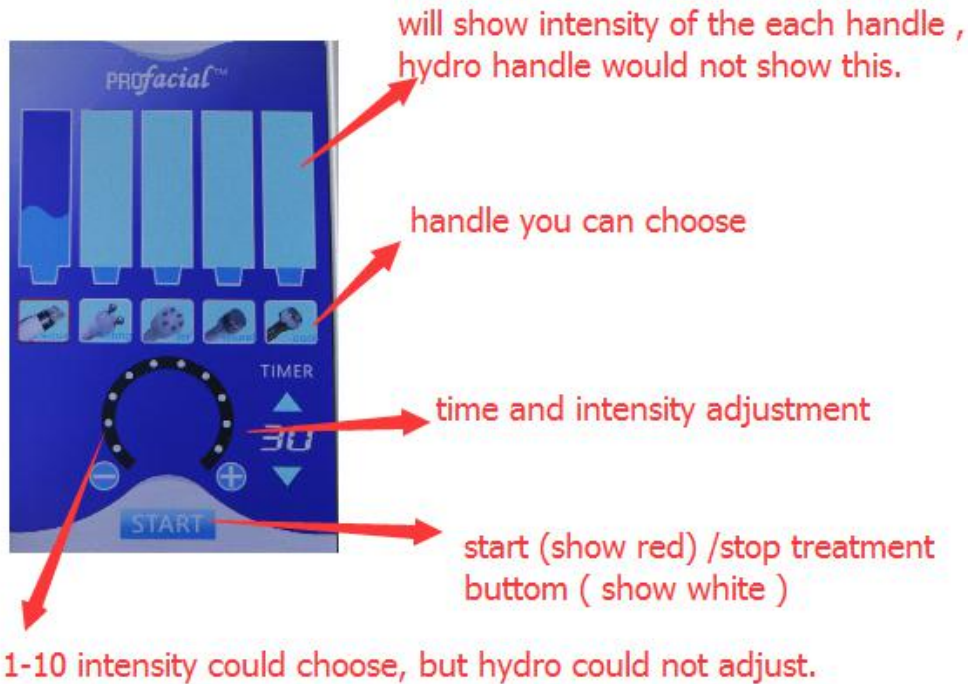


2. Choose handle to do treatment,  from left to right is: hydro, bio, RF, ultrasonic, cooling, when select the exactly function, the handle picture will show in red mark.

3. Click  to choose the intensity u need (0-10 intensity for adjustment. )

Notes: hydro function could not adjustment.







4. Click  to set the time u need ( 1-60mins could be setted, default time is 30mins)

5. click “ start”  to beginning / stop your treatment. ( when “ start” show in red, means treatment is working, show in white, means stop. )

6. take out all the handle to clean well after treatment. ( hydro handle could not remove. )

## II. Function of each handle



<p>Hydro-Microdermabrasion</p> 	<p>use the hydro pen to clean skin first, from middle skin to side, move quickly, dont stay too long in one area. There are 8pcs tips for the pen to fit for different skin and area.</p>
	<p>Tip 1-2, 7-8 for sensitive skin, tips 3-4, 5-6 for oil skin, nose area, black heads skin.</p>
<p>BIO</p> 	<p>Skin Lifting, improve cell penetration, help the body absorb nutrients and water better, improving the level of plateau protein synthesis, reduce wrinkles, make skin softer, elastic and glossy.</p>
<p>Sextupole RF head</p> 	<p>Open pores: Positioning the tissue to heat, open the pore to promote the skin collagen contraction tension, alleviate the wrinkles, produce new collagen, the later skin will be more compact.</p>
<p>Ultrasound probe</p> 	<p>The ultrasonic wave is introduced into the subcutaneous tissue, promoting microcirculation and the skin of the skin. The product can be imported into the skin to penetrate deep into the skin and accelerate absorption.</p>
<p>Cooling head</p> 	<p>Ice covers the skin, contracts the pores, releases the ice force, and penetrates the skin deep into the skin, which instantly causes the skin pores to contract forcefully, forming a natural indifference on the surface of the skin.</p>

### III. Checking out simple problems.

Problems	Checking way
Power button is on, main machine could not turn on	Checking the power cord if connect well
	Checking out the power button if turn on
No output	Checking out the handle if connect well with machine
Main machine being hot during treatment	Using too long time, turn off machine for cooling down
	Checking out the voltage if normal
No power of main machine	Checking the fuse if burned, if yes, change it

### III. Funcion of the machine

1. Used for acne, seborrheic hair loss, hair folliculitis, clear mite and clear skin allergy source.
2. Skin whitening, improve skin black, dark yellow, improve skin quality。
3. Clean the skin deeply and moisturize and moisturize skin and promote skin metabolism.
4. skin lifting, shrink pores and improve the chin.
5. It is used to remove the skin reconstruction and the preoperative nursing of non-stripping skin reconstruction.