



SR-M51708

Instructions

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆Please follow the instruction to fit and operate the machine correctly.
- ◆◆Please do not remove and change any accessories of the machine.
- ◆◆It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆It forbids others to change and maintain the machine except the professional people.
- ◆◆Please contact us if there is any trouble during operation.

After Sale service

◆We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆How to contact us?

Phone:(+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!

I. Function Introduction:

1. RF

The radio frequency face lift was a non-surgical treatment for wrinkles and facial skin imperfections. It is a proven medical procedure that requires no scalpels or sutures. It's also a popular choice for busy professionals who care about their looks. The treatment only takes about an hour, and full recovery is expected in a few days. The results take a bit of time to show fully though. Some results are visible immediately, while the full results take a few months to develop as the deep tissue layers heal.

2. Diamond Dermabrasion

Diamond Dermabrasion provided a non-surgical skin refinish procedure, by using sterile diamond heads to abrade or rub off the top skin layer, then vacuuming out the particles along with any dirt and dead skin back up. This procedure removes skin debris, imperfections, blemishes, wrinkles and unwanted pigmentation on the skin. Moderate usage of Diamond Dermabrasion, accompany with skin products that penetrate through top skin layer that reach the dermis level, helps to replenish natural nutrients, restore cell activity, and promote skin health. The results after Micro Dermabrasion treatment is usually a healthy, glowing, beautiful skin.

3. LED Photon

The Photon Micro current Beauty Device, designed for facial beautification, uses natural light waves, which are transmitted by LED into the skin. The light activates photoreceptors in skin cells, producing energy for absorption by skin components, to beautify your complexion.

4. Cold Hot Treatment

Cool hammer edema, relieve the pain in the part of body, shrink pores, calm inflammation and abate hyper susceptibility; Hot hammer accelerate blood circulation and enhance metabolism. To smooth the wrinkle and make the skin fresh through cool and hot massage alternatively.

5. Skin Scrubber

With Ultrasonic Scrubber begins a new generation of aesthetic soft peeling. Using the high performance of ultrasounds, this exciting cosmetic breakthrough provides a safe painless and perfect physiological skin regeneration. By removing the superficial scaly layers of the skin, the Scrubber stimulates the revival of the skin tissue rendering it youthful and luminous from the first treatment. The top of the metallic probe produces extremely rapid moments at a very low amplitude when it is exposed to ultrasound vibration. This ultrasonic emission has a spectacular effect on water molecules, which are pulverized. This phenomenon is known as micronization.

6. Spot Removal Pen

By means of coagulation of high-frequency electric current, urges the cell necrosis of the epidermis, which can thoroughly eliminate all kinds of spot on the face and the other skin surface.

7. Magic Gloves

Bio Magic Gloves is a very flexible and useful tool to tighten your facial muscle.

8. High Frequency

Use high-frequency current and infrared ray to diminish inflammation and ease pain. The generated Ozone kill the bacteria, heal and astringe the acne vulnus. It has an effect on regulate sebum secretion, improve the texture of skin, accelerate the blood circulation and speed up metabolism, anti-wrinkle and anti-aging.

II. Package list

1. Host 1set
2. Diamond tips 9pc
3. Stainless steel wands 3pc
4. 7 Color photon LED 1pc
5. Skin scrubber 1pc
6. High frequency 1pc
7. Glass tubes for high frequency 3 pc
8. Holder for handles 1 pc
9. Hot Cold hammer 1pc
10. Spots removal pen 1pc+10 needles
11. Bipolar RF handle 1 pc
12. Quadruple RF handle1 pc
13. Magic gloves 1 pair
14. Skin scrubber 1 pc
15. Power cord 1pc

III. How to install the Accessories



IV. Accessories



V. Treatment Introduction

Please press the power switch behind machine, then the screen will show the following menu image:



1. RF



1.1 Operation:

1.1.1 please chose the appropriate RF head, and connect with output socket behind machine (Bipolar RF is used for face, Quadrupole RF is used for face and body.)

1.1.2 please set up the treatment time, energy and mode. (please adjust the energy from low to high intensity under customers' tolerance.

1.1.3. please press the start button, then the head will work

2. Diamond Dermabrasion



2.1 Operation:

2.1.1 Push in the movable ring of copper nozzle at the hand piece

2.1.2 Put a drop of cream or milk lotion on the vacuum hose nozzle, push in the movable ring and insert in the vacuum hose to the limit.

2.1.3 Insert a cotton filter into the Diamond Head, then tight up the Diamond Head on the hand piece. Remove the cotton swab after every treatment. (like #1)

2.1.4 Turn on the power switch.



#1



2.1.5 Adjust the Vacuum intensity by Vacuum Regulator (The big silver button behind machine) according to the following reference table, the intensity displays on Vacuum gauge and check with the client for comfort.

2.1.6 Use the incline area of Hand piece to contact with the skin surface. Proceed with a slow sliding motion. Do not place the hand piece at one spot for too long, doing so will cause harm to the skin.

2.1.7 Adjust the intensity slowly from minimum to maximum. For commonly use, do not set the intensity maximum unless it is of necessity. This is for better comfort.

2.1.8 Gently stretch the skin with thumb and finger and glide the Hand-piece 2 to 3 times over the area being treated. Then cross again in different directions to prevent streaking lines as in the reference picture. Brush strokes or polishing motions may be used in different sites.

2.2 Recommended vacuum intensity on treatment area:

Treatment Area	Normal Skin (inHg)	Sensitive Skin (inHg)
Face	12-22	8-12
Forehead	12-22	10-12
Chest	15-20	15-18
Nose	10-15	10-14
Neck	18-18	10-15
Near Eye Area	8-12	8-10


2.3 Treatment schedule table:

Treatment	Frequency (days)	Total Visits
Face Peeling	6-10	3-5
Fine Lines	7-13	6-8
Neck	12-14	3-5
Acne Scars	7-12	8-10
Hyper pigmentation	7-12	8-10

2.4 Diagram of suggested movements and areas for dermabrasion:



3. Photon LED

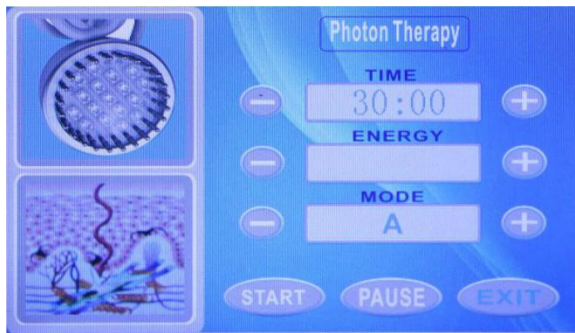

3.1 Operation:

3.1.1 connect with output socket behind machine

3.1.2 please chose the appropriate color. It is not recommended to put the probes on a same position for longtime. Probes may move on the skin in regular line or circle.

	<ul style="list-style-type: none"> • WAVE <p>There are 2 kinds of waves to choose, Continuous Wave & Intermittent Wave</p>
	<ul style="list-style-type: none"> • MODE/ON <p>MODE: Choose different color (among 7 colors)</p> <p>ON: Start the photon function</p>
	<ul style="list-style-type: none"> • AUTO <p>Those 7 colors will operated automatically .</p>
	<ul style="list-style-type: none"> • OFF <p>Stop the photon function</p>

3.1.3. please press the start button, then the LED head will work

3.2 Choose the proper spectrum reaching different depth on skin.

Orders of color by pressing MODE/ON button	Depth In Skin (Max 6, Min 1)	Suitable Treatments For
1 Red	6	Aging, Loose, Lead In Nature, Burn Fat
2 Green	4	Capillary vessel dilation, Splash, Detumescence, Diminish inflammation, Eye lifting, Dark eye rim, Pouch, Face, Light speckle, Breast lifting
3 Blue	2	Acne, Susceptivity skin, Diminish inflammation and pox
4 Yellow	5	Dim skin, Lark skin, Improve lymph, Activate cells
5 Purple	1	Accelerate cell regeneration
6 Cyan	3	Anti hypersensitivity, Reducing appearance of capillary vessel
7 White	1~6 Combination	Whitening, Light speckle, Reducing melanin

4. Cold & Hot Hammer



4.1 Operation:

- 4.1.1 please connect with output socket behind machine
- 4.1.2 please set up the treatment time
- 4.1.3 please press the start button, then the head will work

5. Skin Scrubber



5.1 Operation

5.1.1 After Cleaning Skin, Apply essential gel on the skin and massage parallel with the back of the scrub head. Apply cleansing lotion on the skin for 20 seconds; gently rub the skin with the front of the scrub head at 35 to 40 degree angle

5.1.2 Please setting up the treatment time, energy and mode

5.1.3 Please press start button and put the tips on skin.

5.1.4 Please adjust the energy from low to high under customer's tolerance.

5.2 Mode button option:

Mode A: continued vibration

Mode B: short pause vibration.

Mode C. A bit long short pause vibration.

6. Spot Removal



6.1 Operation:

Please setting up the treatment time and energy, then press start button and put the spot tips on skin.

Please adjust the energy from low to high under customer's tolerance.

7. Magic Gloves



7.1 Operation:

7.1.1 please connect with output socket behind machine

7.1.2 please set up the treatment time, energy and mode. (please adjust the energy from low to high intensity under customers' tolerance.)

7.1.3. please press the start button, then the head will work

Note: mode option is different output frequency, and the customers could try one by one, then chose the appropriate option under tolerance.

8. High Frequency



8.1 Operation

8.1.1 please chose appropriate high frequency tips replacement, and connect with handle piece.

8.1.2 Please setting up the treatment time and energy

8.1.3 Please press start button and put the tips on skin.

8.1.4 Please adjust the energy from low to high under customer's tolerance.

8.2 Usage of high frequency tips:

A. Mushroom Tube for large facial areas

B. Tougue Tube for small facial areas

C. Bend Tube for spot areas



Mushroom Tube



Tongue Tube



Bend Tube

VI. Attention:

Never operate this equipment on a person has or suffers from the following medical conditions described below. (The salon or operator may consider having a client sign a waiver to the effect that he/she is free of any poor health conditions.)

1. Heart Disease
2. Heart Murmur or Irregular Heart Rate
3. Has a Pacemaker or other type of heart rate balancer.
4. Pregnancy
5. Epilepsy
6. Alcoholism
7. Ulcers
8. Disease of Stomach, Intestine, Liver, or Pancreas.
9. Breast Cancer.
10. Undergoing treatment for any form of Cancer.
11. Had major surgery within the past year.
12. Asthma
13. Cold, Flu, or Upset Stomach
14. Any serious illness or disease not listed above.

VII. Do not use on these areas:

1. Sensitive Skin
2. Skin Rash
3. Damaged, Inflamed, or Infected Skin.
4. Surgical scar less than 12 months old.
5. Skin Swollen.
6. The probes can not point to Eye ball.
7. Genitals.