

# Suitcase Spots Removal Freckles Pigment Age Remove Beauty Facial Skin Care Device



## Introduction

MS026B ,This Beauty Facial machine is capable of effectively removing all kinds of spots such as freckles, senile plaque and flat nevus. The outcomes are instantly noticeable and no special skill is required to operate the machine to achieve astonishing results.

## Function

Remove freckles.

Remove spots

Remove pigments

Remove aging spots

Remove nevus

## **2.Before operation**

**1,Disinfection:** Use 75% alcohol to disinfect the instrument

**2.Connect** power cable and spot pen with needles

**3,turn** on machine,

**4,Adjust** intensity from lower to higher , find the comfortable intensity for skin.

**5,Handle** spot pen, use it on mole,until melanin distribution

**6,after** operation ,use of anti-inflammatory ointment until skin recovery.

## **Attention**

**1,** before operation,disinfect skin and spot needles.

**2,after** operation,avoid strong sunlight

**3,Doing** spot removal, do not let needles go into skin depth,and using time not long in one area, avoid burn skin.

**4**,if do one treatment which is not good result, until skin recovery then do spot removal again, but can not let needles go into skin depth.

**5**,one treatment can not removal more spot, just 3-5 nevi;

**6**,after operation, do not use any water on treatment area, and do not eat food which contain dark pigment ,ginger;

**7**,after scabs from treatment are,do not remove scabs,until scabs fall off naturally.

**8**,Skin inflammation,purulent , pain and others, stop treatment.