



Shishi Goddess

MODEL: SR-NV17

diamond dermabrasion suction black heads instrument
Please read the instruction before operate

Diamond dermabrasion beauty instruments instruction:

Diamond dermabrasion beauty item use diamond dermabrasion combine with different diamond heads. By the way of back and forth friction, with suction and release strength, the diamond head will grind off a layer of shallow skin , it is non-invasive and physical shallow.

The principle of pure physical,mechanical compared to use fruit acid,the chemical way to removal horny, is more mild and would not stimulate the skin. With the different level of suction, and different sizes of the diamond heads to control the depth of dermabrasion, less irritating. After treatment, it would not hurt skin or deepen the pigment problem. At the same time, the crystal particles would not broken and fall out, all treatment process is clean and safe.

Diamond dermabrasion beauty instruction can use to improve skin pigments, slightly youth pock, the elderly spots, skin tough after sun exposure and facial fine lines.It also can promote skin renewal, you will feel the skin is more smoother after treatment, if you do the ultrasonic whitening lead in or whitening preparations then with ice compress, the result is better. Faster to achieve your beautiful wise.

In addition to diamond dermabrasion, this instruction also with special head for black heads, strong suction to suck acne, black heads and skin excess oil, to keep the skin lean and fresh.

1

Functions of diamond dermabrasion:

Diamond dermabrasion working on the leather to activation, repair, regeneration

Microcirculation (nutrition, waste exchange), lymphatic circulation(passive drainage system), affiliated glands(sebaceous glands, hair follicles)

Promote metabolism, smooth the lymphatic circulation, removal 3 obstruction of esophageal gland, improve the supply of oxidized nutrients, eliminate excess water and old waste, sebum normal secretion nourish the skin, quickly removal the metabolism of aging material, lead out and increase nutrient supply function.

Acne, pock skin treatment:

1. In depth removal skin which around the pores of accumulation, prevent pore abstraction, and strength to removal pock and surface dead cells

2. To stimulate metabolism, promote oil and bacteria, toxins and old dead cells to exclude, increase the rate of cell regeneration water, and fade the dimples on the face.

3. Increase cell permeability and cell absorption, into skin the effective ingredients, inhibit bacterial growth, smooth skin and to heal wounds.

4. Promote renew skin cells, lighten acne pock, renew skin, improve skin.

2

Water deficient skin, dry skin treatment:

1. Removal aging skin horny, Avoid excessive accumulation of keratinocytes, keep the skin brighten, meticulous;

2. Improve blood circulation, lymphatic circulation, faster water nutrients, brighten and shiny skin.

3. Moisturize and repair skin, increase cell penetration, strengthen skin absorption, improve skin moisture.

4. Promote the regeneration of epidermal basal cells, accelerate the skin renewal, repair skin tissue, balance skin water transfer.

Loosed skin, wrinkle skin treatment:

1. Accelerate the removal of dust, pollutants and free radicals that damage cells on the surface of the skin, promote the natural defense ability of cells and avoid the degeneration of collagen fibers;

2. Improve blood, lymphatic circulation, accelerate metabolism, then the dermis layer will get enough nutritional supplements, and lighten fine lines.

3. Increase cell penetration,let the product into depth skin efficiently, promote collagen fiber regeneration, smooth wrinkles

4. Promote renew skin cells, lighten acne pock, renew skin, improve skin.

3

Dull skin, uneven color skin treatment:

1. Removal aging skin horny, Avoid excessive accumulation of keratinocytes, keep the skin brighten, meticulous;

2. Improve blood circulation, lymphatic circulation, increase the content of skin oxygen, make the skin fine and glossy

3. Increase cell penetration, strengthen absorption, improve skin moisture, brighten and shiny skin

4. Promote the regeneration of epidermal basal cells, accelerate the skin renewal, make the skin even color and whitening

Melanin skin treatment:

1.Accelerate the elimination of melanin in the epidermis and the environment pollution and oxidation of lipid which remain on the skin and easy to hurt the skin cells, during treatment, you can add the embryo frost to fade melanin spots quickly.

2. Improve the skin microcirculation, promote the metabolism of melanin, strengthen deep whitening, restore melanin, so that the spot effect is more significant

3. Increase skin permeability, so that whitening products into the cell

4

Specification:

1. It is only 160g for the main case, small and light. No matter in home or go out, easy to bring.

2. Diamond tips suitable for whole, the eyes around, face and neck

3. 3 different suction head, big round hole, small round hole and oval hole

4. Single layer filter to facilitate the operation of clean, while effectively prevent the face dirt into the machine inside

5. Suitable for all skin type as well as the sensitive skin.

6. No irritation, no tingling, no anesthesia

7. Physical mechanical dermabrasion, freely to control, without side effect, sage and reliable

8. Would not influence the normal life or work after treatment

How to do if the skin get red after treatment ?

Apply the moisture gel on the skin, will smooth the skin, smaller the skin pores, or you can use the towel to wrap some ice to cool facial to eliminate redness quickly.

5


Install instruction:



Step 1. take off the parts
Step 2. put the filter in the hole
Step 3. install the filter cap
Step 4. install the transparent part
Step 5. install the white case
Step 6. put the suction head in the machine
Step 7. put the protective cover on the head
Step 8. take off the protective cover for the power hole
Step 9. connect the power line to operate the machine

Each silicone right on the ring part can ensure the integrity of the entire item, if the silicone right off when installing or operating, please re-adjust well
Need to change the cotton filter after 3-5 treatments
Regularly check the filter outlet if clogged, if so, please change the cotton filter timely

Parts instruction:

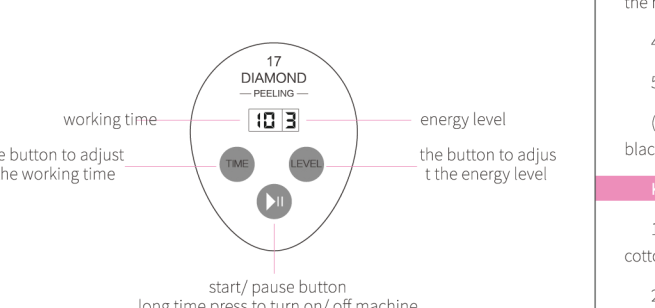

working time
energy level
the button to adjust the working time
the button to adjust the energy level
start/ pause button
long time press to turn on/ off machine

7

Operation:

1.put some water of 60 degrees on the small towel, deposited on the nose for 3-5 mins to open the pores

2.Put some derived lotion on the makeup cotton, deposited on the area with black heads. Take it off for 5-8 mins

3.Use the bid round suction head, put the head on the treatment area, turn on the machine and suck skin

4.Move slowly to suck out the black heads

5.After treatment, use astringent water or apply mask to shrink pores

(note: change to small around suction heads to the small area, Stubborn blackhead can be adsorbed by nose paste)

Key Tips:

1.After cleansing the skin, apply black head export liquid to the black heads by cotton, take off it after 5-8minis, removal on the fixed point.

2.After removal back heads, use the ice towel and moisturizing products to shrink the pores, relieve discomfort.

8

Warranty card

Basic information

Product name	Product model
purchased date	Maintenance date
Purchased platform	contact phone number
Buyer name	
Address	
Problem description	

Warranty history

Maintenance date	Maintenance items	Maintenance organization

9

※ The warranty card is an important document for customer get the free maintenance services, please keep it safe.
※ Please fill in the basic information completely before you return the machine, so that we can arrange shipping after maintenance.

Warranty instruction:

◆ If machine damage during normal usage, customers can enjoy one year maintenance according to the content of warranty card.

◆ But the spare parts is not within this warranty.

◆ Other cost parts not including in the warranty.

◆ Please keep the receipt and warranty card safe, and show this document when need warranty service.

Non warranty range:

◆ Faulty use and damage by man-made;

◆ Unauthorized repair personnel improper repairment and unauthorized modification caused by the failure;

◆ Damage cause by not follow instruction of user manual;

◆ Damage cause by force majeure disaster.

10

