

Instruction

RU-668

Dear customer, Thanks for your choosing our product!
For the best capability and safety, please read the
instruction before operation.
Please reserve the instruction book for the reference.

Function :

Remove spot

Instruction Procedure :

1、 Insert the pin into the handle.

1. Clean the treat area with alcohol then spread it with anesthetic 3 minutes.

2. Turn on the machine and ready for operation.

3. When operating, care the currency lever and adjust a proper energy lever to avoid hurt.

Spot: special care to test and analysis spot's conditions, some spots on the surface of skin but some spots in the deep under skin issue. If spot small, might remove it one time, if spot big and pigments deposits deep, should remove it 2-3 times. Don't cut through and break the spot's surface one time, should burn and remove with small frame way. To spot in the deep under skin, should spread proper cell cream on it before the wounded cure well 2~3 days, to avoid inflammation or skin hollow happen. When spread the cream, if the wounded inflame, don't spread it.

Caruncle: gently touch the end with pin and kill the caruncle cell, later about 6-7 days, the caruncle drops. If the caruncle is too big, should remove it several times and not finish it in one time to avoid the wounded covers bad.

Caution :

The pin should be sterilized before use.

Beautician should keep the handle balance not shake to avoid burning the wounded.

Parts :

Sweep the spot pen: 1