

RU-8202 Beauty equipment

Instruction

Dear customer, Thanks for your choosing our product!
For the best capability and safety, please read the instruction before operation.
Please reserve the instruction book for the reference.

Accessories list:

Mainframe: 1 set

Power Line: 1 item

Ultrasound head: 1 round-headed, 1 cusp head

Scrub skin knife: 1 set

Hand exchange line: 1 item

Brief introduction :

RU-8202 beauty equipment has ultrasonic and skin scrubber two function. Each functions has independent switch to convenient use. Resolve the traditional functions shortcomings. The machine contour design generosity, appearance, novelty, panel color soft, convenient operation, fully embodies the machine practical value.

Function :

1. Ultrasonic use of high frequency ultrasonic wave (1000000 / 3000000) shocks to promote human blood circulation and speed up metabolism. Application ultrasound, with all kinds of cream cosmetics, penetrate into the skin to strengthen the organization to treatment, beauty effect.

2. Ultrasonic scrubber instrument is the leader in numerous beauties equipment at present, use ultrasonic scrubber instrument beauty ensure had a noticeable effect after a long period time testify , The equipment is to use ultrasonic transducer technology will be 28000 times per second mechanical shock replaced 28,000 times per second electric shock promote our muscle infiltration. Make deeper under the skin cells to produce exciting, therefore, as long as the use of this apparatus in conjunction with the corresponding liquid medicine or distillate element function , can quickly eliminate black spot, wrinkles, acne cuticle pores dirt, thoroughly clean skin, the bright white skin and restore flexibility.

This equipment is also available on current injection button; make the synchronization use of current and the ultrasound. Make the result more notable. Its design appearance generous, beautiful, innovative, soft color panels, convenient operation, fully reflects the value of this machine.

Method of use:

1. Ultrasonic

Make the two ultrasonic head plug on the before the mainframe corresponding ultrasonic socket (L / R). By "POWER" connected to power. Then press Start / pause button to select start work or suspended. By MODE button to select different functions of the ultrasonic waveform (Continuous wave or intermittent waves), use time switch selects the time. Please tune the size of the output power from CHAO window beneath two keys when Operation need to adjust output power, At the same time attention CHAO window display file (1-10 file)

2. Scrub skin, clean skin

Plug the power line on the mainframe back socket and connect power. The shovel head and hand ring plug corresponding socket on the mainframe. Press "power" switch connect power. Press the have up and down direction arrowhead switch to adjust power.(corresponding PEEL windows display your need power). Let treatment wear hand ring. The machines are automatic device, When the shovel head come into contact with the treatment skin, immediately enter the working conditions, when the shovel head leave treatment skin, immediately stop working. Press the "Start / pause" button (the corresponding CLEAN light), then press IN/PUT switch option (Continuous wave or intermittent waves). Then smear tighten cleaning fluid or decomposition fluid on the skin. About after half one minute, let shovel head positive and skin of 35-45 degree angle gently push forward. Let need Clean off adverse material on the skin together atomization with the liquid.

Precaution:

1. Ultrasound head maintain clean, not use high-temperature water disinfection and use cotton lightly clean when cleaning.
2. First clean the operation site, Painted cream in the operation site, Ultrasonic head not empty burning, Such as to pause when in the operation, according to the "POWER" to pause work.
3. Don't make the ultrasonic head directly to eyeball use
4. Please removable metal watches and jewelry before operation
5. Have cardiovascular disease and sensitive skin disease and not be used
6. Do not placed machines on damp places
7. Not to shovel head contact to the eye when use
8. Do not operate in the same location on the skin for too long.

Maintenance services:

- 1, since the instrument from the date of purchase (by invoice) mainframe warranty one year , parts and consumables for non-warranty scope of warranty coverage, the period has expired, can be responsible for the maintenance and update (will be charge).
2. Please contact dealers if necessary, supplement or purchase, Accessories and consumables.
3. The following factors do not provide warranty service free of charge:
 - A. self-entry and modification of the equipment failure caused
 - B. improper use, the lack of reasonable maintenance caused fault
 - C. inadvertent knock, falling caused fault