

8 IN 1 BEAUTY EQUIPMENT

RU-8208

- but payable.
2. If you want to replenish or purchase the consumables and accessories, please contact the distributor for advice.
 3. Breakdown cause by human as below is out of our warranty.
 - a. Breakdown cause by dismounting or transforming the instrument by customer himself/herself.
 - b. Breakdown cause by incorrect or unreasonable maintenance.
 - c. Breakdown cause by beating or dropping the instrument.
 - d. Breakdown cause by incorrect operation.
 - e. Damage cause by irresistible disaster like earthquake, fire disaster.
 4. Welcome for any comment and suggestion.

Instruction Manual

Accessories:

Name	Qty
Mainframe	1 pc
Power line	1 pc
Ultrasonic probe	2 pcs
High frequency electrotherapy stick	1 pc
High frequency electrotherapy pipe	3 pcs
Spot pen	1 set
Vacuum glass pipe	3 pcs
Spray bottle	1 pc
Red & black pipe	1 set
Brushing accessories	1 gun; 5 pcs brush
Galvanic accessories	1 set

Dear customer, thanks for buying our product!
For the best capability and safety, please read this instruction manual before any operation.
Please reserve the instruction manual for future reference.

Brief Introduction

RU-8208, an 8 in 1 multifunctional beauty machine, is a new cosmetic machine and widely used. It was designed according to the common and necessary functions used in Beauty Salon. It includes the functions as Vacuum, Spray, High Frequency Electropathy, Brushing etc.

Each function has a separated switch. Easy and simple to handle and overcome the defect of traditional multifunctional instrument that with one function break down, all the others could not be used. The steamer arm is adjustable and rotated, with autognosis function to ensure the machine would not burnout. It has wheels at the bottom for easily moving. And there is a tray to place and store the accessories. Make it more convenient for a beautician while using the machine.

High Frequency Electropathy

High frequency electropathy is a good assistant of healing the acnes. The high frequency current could infiltrate into deep skin layer, generate heat, and accelerate the blood circulation, help to absorb the nutrition and expel the waste. The heat could relax the tense nerves, help the dissolvent to infiltrate the skin, and regulate sebum production. It could disinfect and accelerate the wound healing.

Usage:

1. Connect the power line and the electropathy stick.
2. Plug the glass pipe into the stick.
3. Press the Power switch.
4. Press the Pause/Power button, adjust the working time with ▲/▼ buttons under the TIME display frame and the working intensity with ▲ /▼ buttons

3. If want to use the spray function, connect the sprayer to the tube, then connect the tube to the spray outlet. Hold it with thumb and long finger, and press the breather hole with forefinger. Air pressure will be engendered inside the bottle and spray out the toning lotion.

Attentions:

1. There are three types of glass pipe, choose the proper one according to the treating parts.
2. Lift the tube with finger then pull out the pipe. Do not pull it out directly, or it might break the pipe.
3. While using the vacuum function, do not stay at the same place for too long or it might cause bruise.
4. While using the spray function, ask the customer to close the eyes and mouth in case the liquid will spray into the eyes and mouth. Start from the forehead and spray down, keep a proper distant between the sprayer and face, do not spray into the nose.
5. Keep the glass pipes clean and dry.
6. Do not use the toning lotion with impurity and sediment or the spray hole will be jammed.
7. Never connect the sprayer to the vacuum outlet or the toning lotion will be vacuumed into the pump and damage the machine.
8. Cut the power if no using.

Maintenance Service:

1. The instrument is under warranty for one year except the accessories and the consumables. We would provide maintain service after the warranty period,

3. Turn on the brushing gun, adjust the intensity and the rotated frequency according to the customer's skin condition.
4. Shut the machine off and clean and disinfect the brushing heads.

Attentions:

1. Do not let the hair draw into the brush.
2. Use vapor to soften the cuticle before using the brush.
3. 1-2 days after this treatment, please don't do any facial or shave, don't use the scrub cream and keep away from the stuffy place.
4. Take one treatment every 3 weeks.
5. The working time on the whole face is better to be 15 to 20 minutes. It would not affect your daily life after the treatment.

Vacuum & Spray

1. The glass pipes could use to vacuum the comedones, acnes, pustules and clean the dirt and oil contamination, make the skin clean and fresh.
2. The sprayer could convert the toning lotion, anti-inflammatory lotion or shrinkage water into vapor and spray.

Usage:

1. Connect the power and switch on. Press the Pause/Power button, while the indicator light is on, vacuum and spray functions are ready to work.
2. Connect the glass pipe to the rubber tube at one end and the filter at the other and then the filter to the vacuum outlet, press the hole on the pipe, it will generate attractive force.

under the High Frequency display frame. It will start to work.

Mode:

1. Direct therapy

The beautician holds the stick and massage on the face, adjust the intensity. While the ultraviolet light occurs, rotate for 5 minutes. The high frequency current will act on the skin epidermis, generate vibration and heat, and the ozone will kill the bacteria. It could use to treat oily and acne skin.

2. Salutatory therapy

Touch the skin with the electrode pipe intermittently, while the pipe leaves the skin, sparks will occur. It could kill the bacteria and diminish inflammation, accelerate the wound healing. Treat for about 30 minutes. If the patient is afraid of the sparks, cover the eyes with towel. This therapy is suitable to cure the acnes on the face.

Attentions:

1. While doing the direct therapy, clean the skin and make sure there is no dirt or makeup remain, ensure the pie could slide smoothly and steadily.
2. While doing the salutatory therapy, tell the patients what would occur in case they might be scared by the sparks and noise. Do not touch the patient randomly to avoid interference and influence the curative effect.
3. Adjust the operating time and intensity accordingly to tolerance and sensibility of the skin. Dehydrated skin should be treated with short time and weak intensity. And oily skin with longer time and stronger intensity.
4. Please take off the wearing jewelry before the treatment in case conduction of current occurs and hurt the skin.

5. Keep the accessories clean and dry, disinfect before and after use.
6. Do not hold the junction of the pipe and stick in case of electric shock after power on.
7. Do not touch the eye part while operating.
8. Patients with sensitive skin, vascular disease, acne rosacea and serious skin diseases could not use this machine.

Galvanic induct & educe

In accordance with opposites attract principle, it helps to induct the necessary essence which heal skin disease into deep skin and accelerate the absorption, meanwhile, it will educe the impurity.

The positive electrode will generate acid reaction, stabilize the nerves, reduce the blood supply, and strengthen the fibrous tissue. Mainly use to shrink the pore, prevent inflammation, induct the essence into deep layer and accelerate the absorption after cosmetic treatment.

The negative electrode will generate alkaline reaction, stimulate the nerves, increase the blood supply, and soften the fibrous tissue. Mainly use to treat aged or oily skin.

Usage:

1. Connect the galvanic stick to the outlet.
2. Take some cotton, moisten with essence and embrocate on customers face.
3. Connect the power, switch on. Press the Pause/Power button, adjust the working time with ▲/▼ buttons under the TIME display frame and the working intensity with ▲/▼ buttons under the Galvanic display frame. It

2. Keep the probes clean and dry. Do not wash by water, disinfect and clean with cotton, do not drop or hit the probes.
3. Do not let the probes work without any operation, or the machine may get broken.
4. Do not use around the eyeball.
5. Person with hypohepatia, cirrhosis and kidney problem could not use this machine.

Brushing



1. It could do high frequency micro massage to the skin, eliminate the aging cells, expel the toxin and refine the wrinkles. Accompany usage with cosmetic product or medicine, it could treat and improve many skin problem.
2. With the force of the rotated brush, it could massage the skin.
3. It could clean and exfoliate the aging cuticle.
4. After clean the dirt and dead cuticle, it could refine the skin condition, accelerate blood circulation and remove fatigue.
5. By the massage, it could improve the skin elasticity, make the skin smooth tender and shiny.

Usage:

1. Connect the power line and the brushing gun.
2. Press the “Pause/Power” button, adjust the working time with “▲/▼” buttons under the TIME display frame and the working intensity with “▲/▼” buttons under the Brush display frame.

3. Dyschromatosis
4. Chloasma
5. Aestates
6. Under-eye dark circle
7. Swollen eyelid
8. Ichthyosis

Usage:

1. Before using this function, clean the skin where would be treated by cleanser.
2. After cleaning the skin, choose a proper couplant according to the skin condition and disease to achieve the best effect.
3. Connect the power and switch on. Press the Pause/Power button, adjust the working time with ▲/▼ buttons under the TIME display frame and the working intensity with INTENSIFY▲/WEAKEN▼ buttons. Press the  /  button to select pulse wave or continuous wave.
4. Select the proper probe according to the treating part, small part use the smaller one, adjust from lever 1 to 5; large part use the bigger one, adjust from leave 5 to 10. Adjust the working time 8-10 minutes.
5. While operating, the beautician should hold the probe steadily, rotate slowly and evenly, let the skin have a sufficient ultrasonic treatment.
6. Take the probe away after finishing the treatment, keep the essence on the skin and let it penetrate thoroughly.

Attentions:

1. Patient with angiocardopathy, hemorrhagic disease, wounds and the pregnant woman could not take this treatment.

will start to work.

4. Adjust the intensity to lowest, let the customer hold the metal stick, and the beautician hold the other one with roller. (If operating by oneself, hold one with each hand.)
5. Select anode or cathode according to customers need, While the instrument indicate anode, the wand is cathode and the roller anode. Embrocate liquid cosmetic essence on the skin, use a piece of cotton fleexe moistened by water or essence to wrap the roller and induct the nutrition. On the contrary, use cotton fleexe moistened by germicidal solution or toning lotion to wrap the roller and educe the impurity. Press the +/- to switch between anode and cathode.
6. Adjust the intensity according to customers skin sensitivity, while he/she have a little sting feeling, that would be fine. Roll the roller slowly, and operate for 5-6 minutes. Keep the cotton fleexe moistened while operating to achieve the best effect.
7. After finishing the treatment, turn of the power, but do not take away the roller immediately in case the customer may feel uncomfortable.

Attentions:

1. After the treatment, clean the roller and stick and place on the dry environment, or the electroplated layer may be corroded.
2. Person with sensitive skin, inflammation, wounds, serious skin disease and angioelectasis and the pregnant women please do not take this treatment.
3. Please take of all metal wearing jewelry before the treatment. Person with metal matter inside the body is not suitable to take the treatment.
4. Acidity and alkaline of the essence should apply to the anode and cathode

usage. While the roller is anode, it will induct the acid essence and educe the alkaline. On the contrary, it will induct the alkaline essence and educe the acid. If the essence does not acid or alkaline, please test with PH paper.

5. While select to be anode, sting feeling will be stronger than as cathode. Please select accordingly.

Spot removal

By means of coagulation of high-frequency electric current, urges the cell necrosis of the epidermis, which can thoroughly eliminate all kinds of spot on the face and the other skin surface. The effect is obvious with no bleeding or scar left.

Usage:

1. Connect the spot pen to the corresponding outlet.
2. Connect the power and switch on.
3. Press the Pause/Power button, adjust the working time with ▲ / ▼ buttons under the TIME display frame and the working intensity with ▲ / ▼ buttons under the Remove spot display frame. It will start to work.
4. Touching the spot with the pen. After treatment, coating antimicrobial for 2 or 3 days.

Attentions:

1. While doing the treatment, please keep a proper distance between the pen and the skin. Do not insert too deep into the skin or stay at the same position in case the dermis gets burn.
2. If the spot could not be removed by one time, do again after the scab peel off.

Do not scan too deep in case there might be scar left.

3. After the treatment, please avoid strong sunshine or coating with sunscreen cream. Let the scab peel off automatically. Do not peel by hand in case it may be infected and leave a scar.
4. Before the treatment, please disinfect the spot area and the spot pen.

Ultrasonic

1. By the high frequency vibration (1MHz -3MHz), generate ultrasonic with strong penetrability. It is a special instrument that could radiate an alternate wave and spread around. It is much stronger than the normal sonic wave, with high frequency, good directivity, strong penetrability and great expand ability.
2. Because of the high frequency vibration, it will do soft and micro massage to the skin cells, accelerate the blood circulation, facilitate the metabolism, activated the cells and improved the absorption of nutrition.
3. High frequency micro massage could eliminate the aged cells, expel the toxin and fine the wrinkles. Accompany usage with cosmetic product or medicine, it could treat and improve the skin problems.
4. The frequency of ultrasonic could also generate resonance vibration of the cells, consume the fat and improve water absorbing capacity of cells, rejuvenate the skin and recover the elasticity.

Symptoms could be treated by ultrasonic:

1. Red face
2. Acne