



SR-SP18

Instructions

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges). If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆ How to contact us?

Phone:(+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!

Principle

OxyGeneo

Gene+ takes its inspiration from natural hot springs known for inducing skin oxygenation. By simulating this effect, Gene+ triggers a body response that sends oxygen to the treated area. At the same time Gene + exfoliates the skin and creates an optimal environment for infusion of essential nutrients both during and after treatment. The ability to layer Radio Frequency induces collagen and elastin regeneration in the dermis layer, which leads to smoother and tighter skin and a reduction in fine lines and wrinkles.

Radio Frequency

The multi-polar RF delivering thermal energy into the subcutaneous tissue makes the tissue's natural resistance movement produce heat, the heated collagen shrink immediately and stimulate dermal layer to excrete more collagen to make up for contraction and draining collagen, thus to hold the bracket of the skin, recover skin elasticity. When the collagen are produced continuously, it will increase the thickness and density of the dermal layer, fill wrinkles, eliminate scars, recover skin elasticity and gloss, making the skin white and smooth.

Functions

- a. Whitening and anti-wrinkle
- b. Ascension, tightening
- c. Sculpture facial contour
- d. Convergence chin
- e. Contractive pore, calm, allergy
- f. Improve eye problems (eye end lines, pouch, black rim of the eye)
- g. acne removal, dark circle removal, wrinkle removal, whitening, rejuvenation, lifting,
- h. cleansing skin
- i. rejuvenation, lightening
- j. skin whitening, skin lightening

Package list


- 1set x Host
- 1pcs x BOHR effect C & O handle
- 1pcs x 4D - circle RV handle
- 1pcs x Power supply line
- 2pcs bandage for tighten hair
- 6pcs x Accessories counterunk hexagonal screw
- 1pcs x screw wrench

2pcs x Silicone massage head
2pcs x skin rejuvenation gel(NeoBright, NeoRevive)

Machine Instruction



Handles Instruction

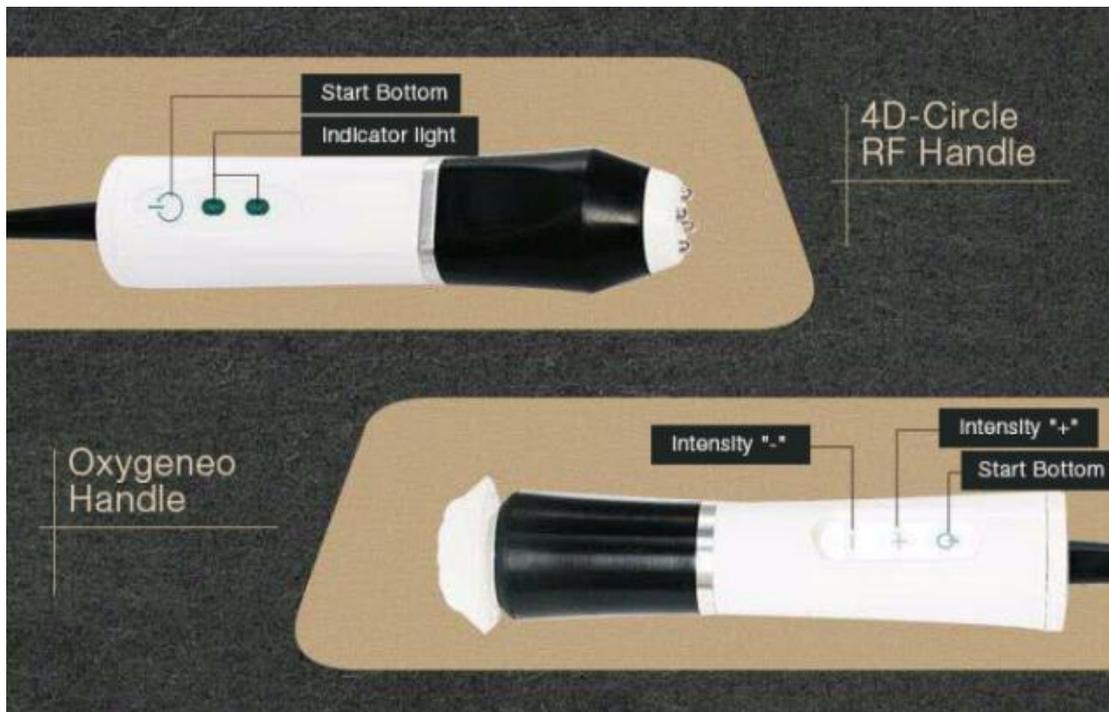

Products Instruction



NeoBright Formula

NeoBright reduces skin pigmentation and sun damage while lightening the skins complexion.

Azelaic Acid = Even Pigmentation

Azelaic Acid is a well-known agent that is used to treat skin hyper pigmentation conditions that decreases the activity of pigment-producing cells.

Kojic Acid = Skin Lightening

Kojic Acid is a naturally occurring agent produced by fungi which inhibits Melanin production. When combined with Azelaic acid, there is a synergistic lightening and bleaching effect.

Retinol = Anti-Aging

A derivative of Vitamin A, Retinol has been clinically proven to increase collagen and skin cell production. It is also known as one of the best anti-aging solutions.



NeoRevive Formula

NeoRevive smooths the appearance of fine lines and wrinkles, and revitalizes dull complexion.

Hyaluronic acid = Wrinkle Reduction

Naturally produced by the body and responsible for fluid retention and skin moisturizing, as we age, the body's capacity to produce Hyaluronic Acid decreases. This results in thinner, dryer and wrinkled skin. By infusing Hyaluronic acid, the skin is able to regain lost moisture and volume, thereby reducing wrinkles.

Peptides = Skin Repair

Peptides are small protein fragments which strengthen the skin's ability to retain fluids. Combined with Hyaluronic Acid, a synergetic effect of dermal repair is created.

Retinol = Collagen Renewal

A derivative of vitamin A, Retinol has been clinically provide to increase collagen and skin cell production. It is known as one of the best anti-wrinkle solutions.

Operation

1. Use the screw wrench to install the holder and machine
2. Install the machine holder and all handles
3. Connect the power line with machine and turn on the machine
4. Machine screen show you this :



Start to home page



Info



Setting

Home page




OxyGeneo included 3 different treatments, NeoRevive, NeoBright and NeoMassage

NeoRevive : Skin lifting firming

NeoBright : Skin brighten


NeoMassage: Absorption massage

OxyGeneo:

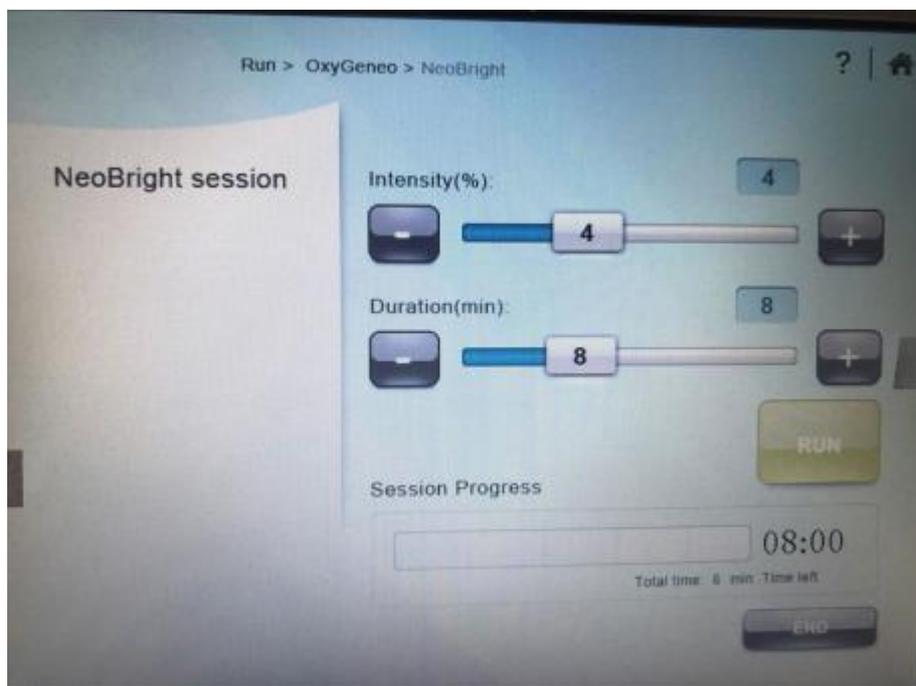



adjust the intensity, 1- 10 available

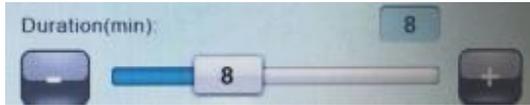


setting the duration/treatment time, 1-20

available




Start/ Pause session, when click the RUN, screen will show you to press the start button of handle, as below :

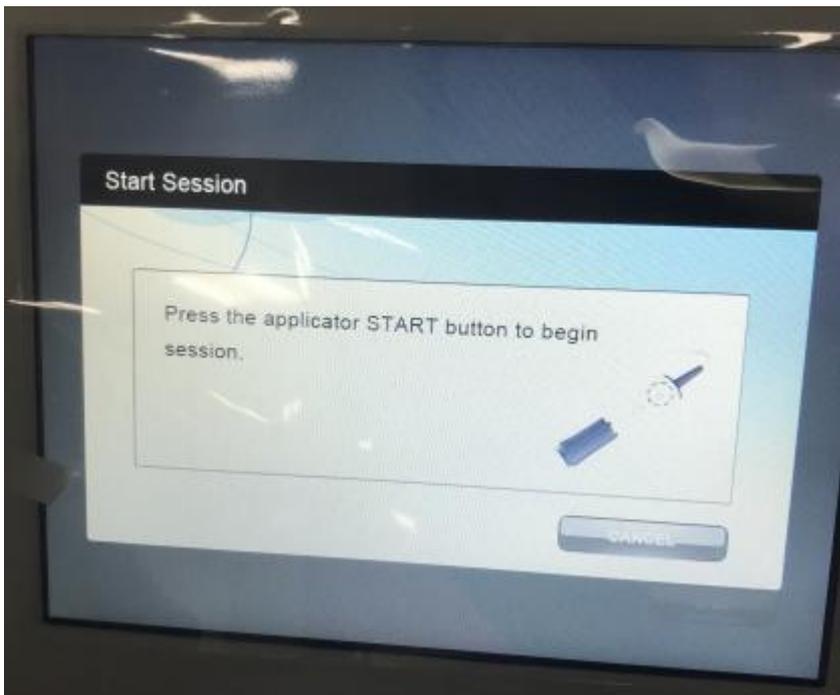

Press Start button to start treatment

If not, click CANCEL back to setting page

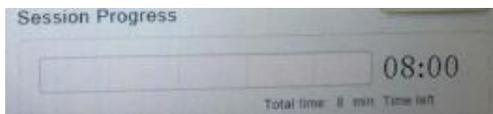

the progress for the treatment, show you the rest

time of what your set




End session




Back to home page


TriPollar:

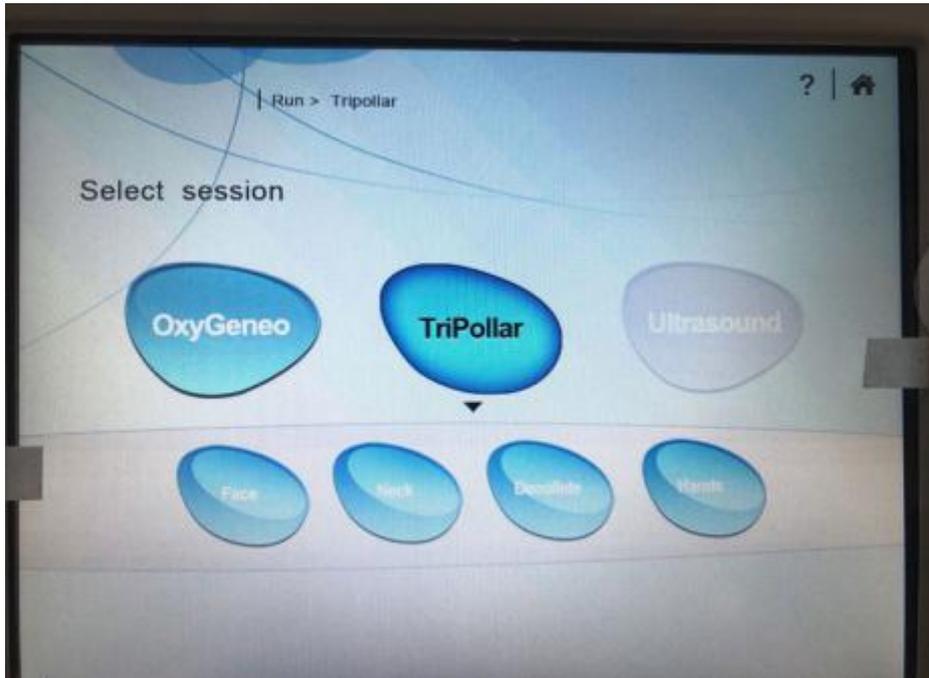

TriPollar function included 4 treatment area, Face, Neck, Decollete and Hands
Choose one of that to enter treatment setting page :

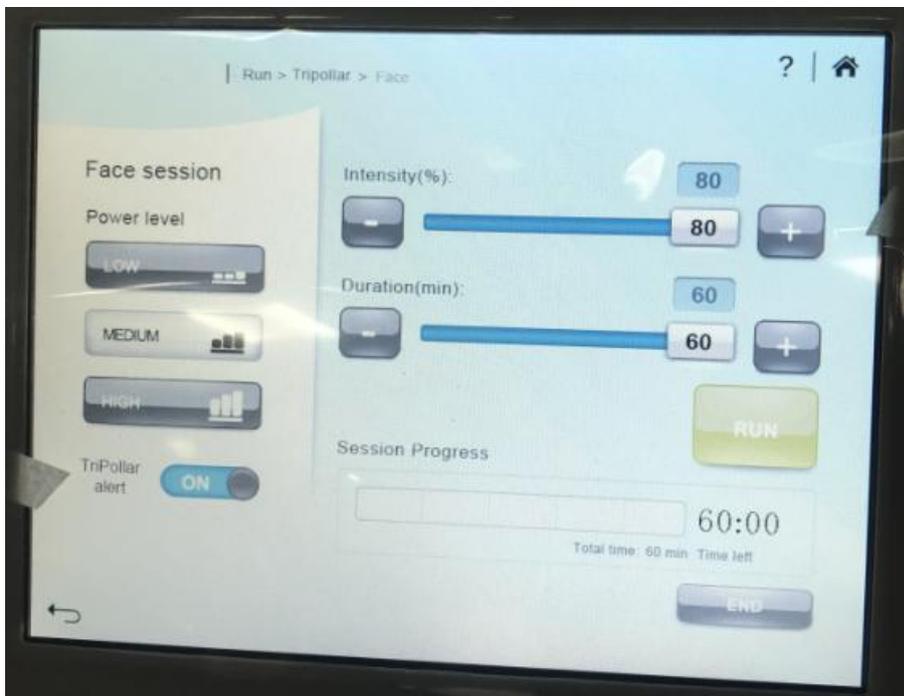

click this to adjust the intensity, 1-80 available

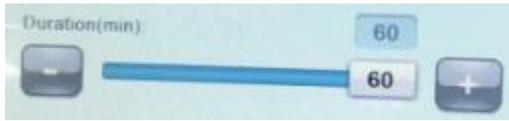


click this to adjust the treatment time, 1-60


available



Start/ Pause session, when click the RUN, screen will show you to press the start button of handle, as below :

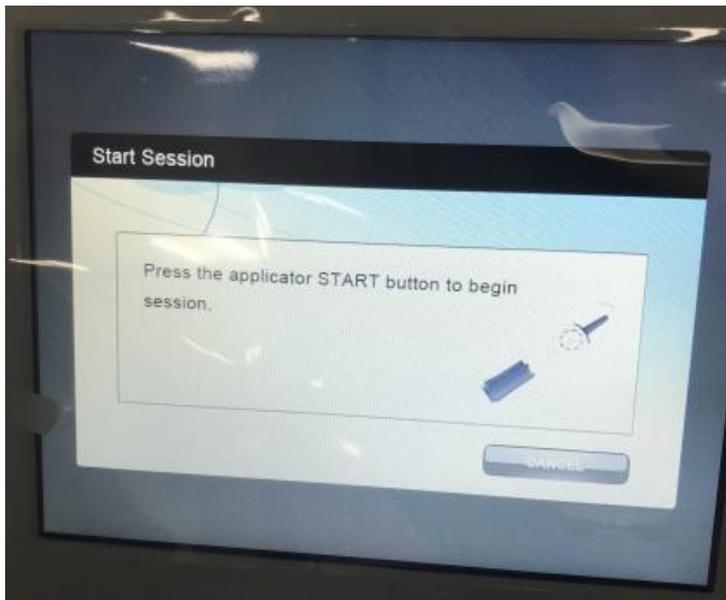


Press Start button to start treatment, during treatment, can not change the parameter setting
If not, click CANCEL back to setting page

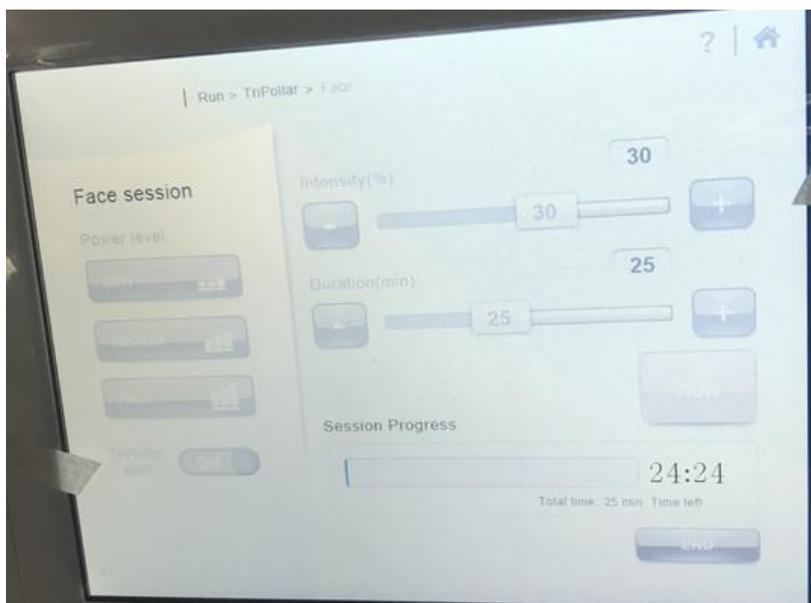


Paused session

Press the start/pause button of handle, then screen will show you picture as below:

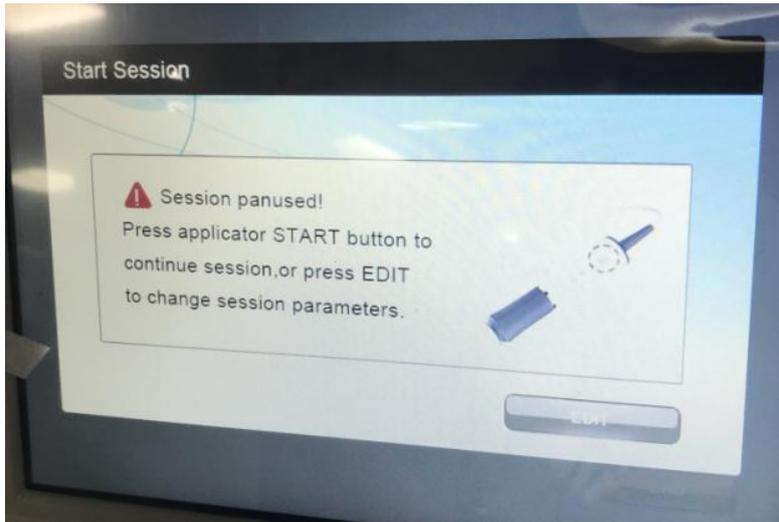

If you want to exit the treatment, then press the "EXIT" back to home page, if not, click the button again to continuous treatment

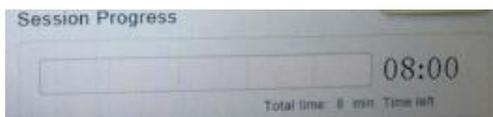

the progress for the treatment, show you the rest

time of what your set




End session




Back to home page



Power level, regular setting when you choose the level

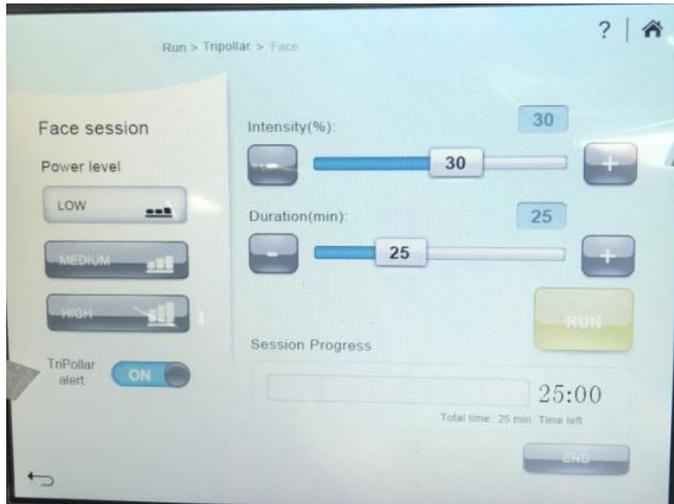

Low power level, intensity at 30 and time for 25 mins

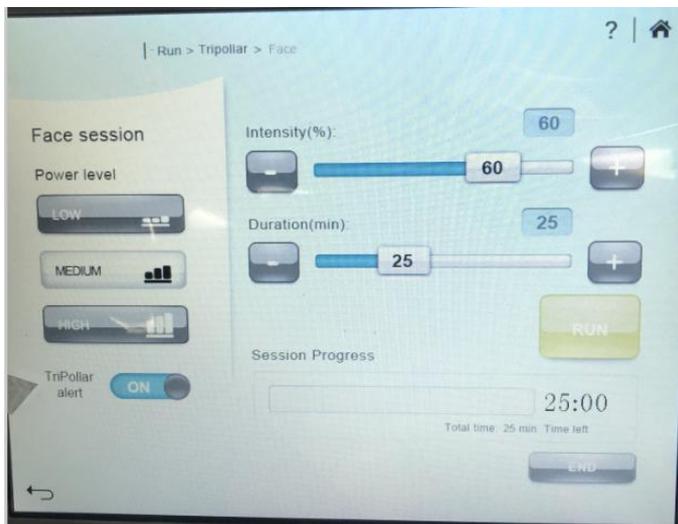


Medium power level, intensity at 60 and time for 25 mins



High power level, intensity at 70 and time for 25 mins


Treatment

Step 1:

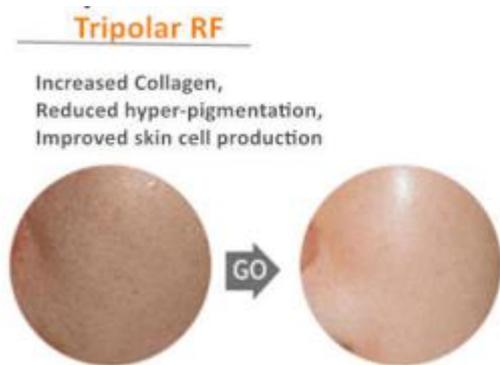


Tripollar

1. Facial cleansing, use the washing cleaning products to clean facial
2. Apply the special gel on treatment area
3. Massage the mandible center in circles to produce deep heat.
4. Massage lower jaw in lines.

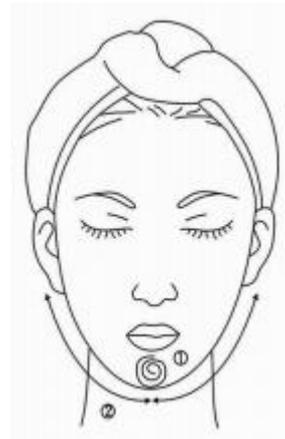

5. Massage the triangle zone of both sides of the face in circles.
6. Massage from jaw to angulus oris to ear in lines.



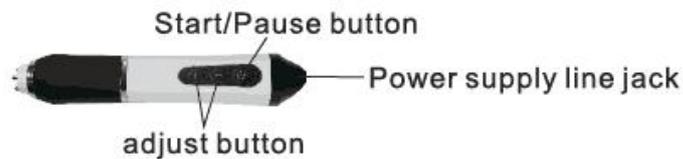

1. Insert the power cord into the socket, and the LED light will flash two times, indicating the pre-start state.
2. Press the "start" button.
3. Set intensity. Press "-" and "+" to adjust the intensity, each time adjusting a strength LED light to display different brightness. According to customer's needs, choose different intensity, strength control in the customer can withstand the range.
4. When the intensity is adjusted, the LED lamp flashes two times, indicating that the intensity has been adjusted to the lowest or highest state.
5. During operation, pause and click "pause" button.

Cautions:


1. Beautician should wear insulated gloves during operation.
 2. Do not wear any aglet, both for the beautician and the users.
 3. The electrode probe must be fully in contact with skin.
 4. Do not make the electrode stay on any part without moving. Keep it moving on skin slowly.
 5. Skin around eyes and forehead is very thin, the output intensity should be relatively weakened.
 6. Do not contact skin with unipolar when using of Micro heat work head otherwise it will produce pulse discharge, gel paste etc. Products like gel maybe evaporated into a mist.
 7. Do not let the laser light straightly shine upon the eyes in case of eyes hurt.
-

Step 2:



Co2 Oxygen

1. Facial cleansing, then apply 2/3 skin rejuvenation gel (NeoBright or NeoRevive) on skin
2. Do treatment in circle on skin
3. Massage the mandible center in circles to produce deep heat.
4. Massage lower jaw in lines.

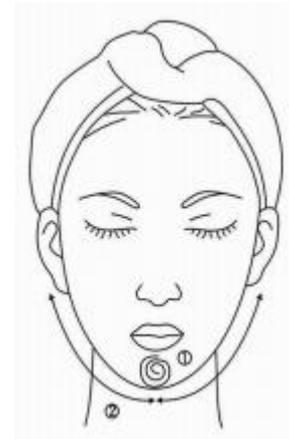


5. Massage the triangle zone of both sides of the face in circles.
6. Massage from jaw to angulus oris to ear in lines.




How the Oxygeneo Work ?

1. EXFOLIATE

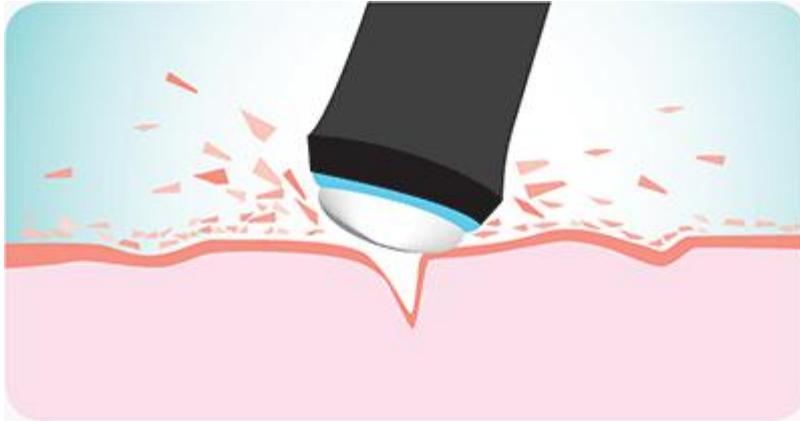

Similar to microdermabrasion, the OxyGeneo™ exfoliates the upper skin layer to remove dead cells, smooth and renew the skin, preparing it to receive active nutrients. The OxyGeneo™ is safe on all skin tones and can treat sensitive skin without any irritation.

2. INFUSE

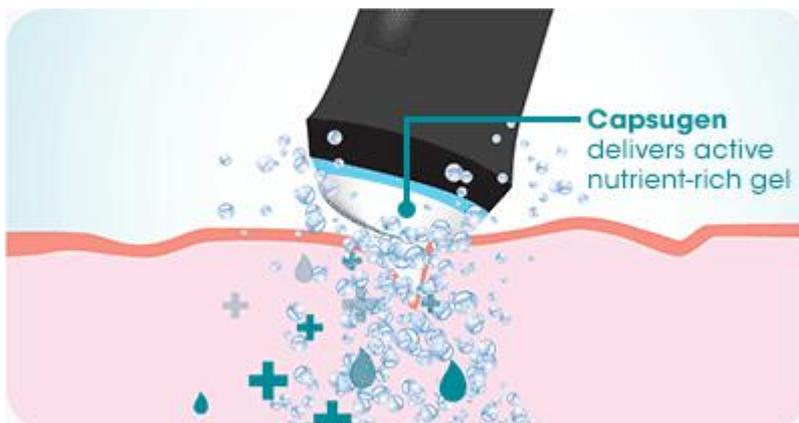

Through our exclusive Capsugen, the OxyGeneo™ cleanses the skin while infusing nutrient-rich active ingredients. You choose between the NeoRevive™ for rejuvenation and anti-aging or NeoBright™ for skin brightening and texture improvement.

3. OXYGENATE



The OxyGeneo™ produces CO2 bubbles which gently burst on the skin surface creating a physiological response, sending oxygen-rich blood to the area, increasing capillary flow and skin metabolism. Oxygenation results in optimal absorption of active ingredients.


Before and after :

NeoRevive Results



Improve the skin elasticity, clear the diffuse reticular pigmentation,
skin area looks more even and shiny.

NeoBright Results

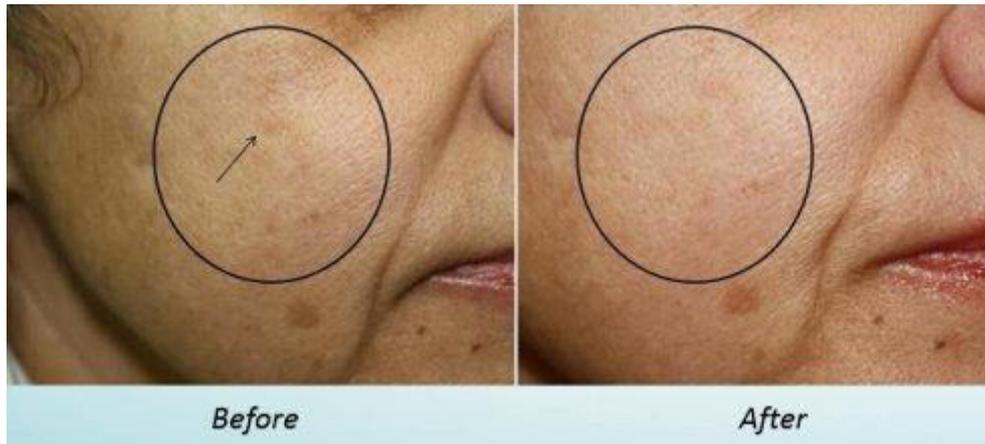

Improve skin texture and brighten the skin color, the pores look smaller than before treatment, lighten the freckles and pigmentation

Tripollar Result



Wrinkles of eyes around is smaller, skin smoother, skin lifting

Notice

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
7. If the machine will not use for a long time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.