

## **2 in1 Diamond Dermabrasion unit**

The Diamond Peel treatment erases epidermal layers at varying depths in a controlled manner, minimizing skin trauma. It mechanically alters the epidermis and invigorates cell renewal to create a healthy, balanced skin surface. It is then possible to improve or eliminate many skin conditions and skin imperfections. The machine also has the hot & cold treatment functions. The cold treatment function is specially designed for claming down the skin after peeling. The skin is sensitive after peeling treatment, so the cold treatment procures is necessary. The hot treatment function is designed for accelerating the absorbtion of nutrition.



### **Operating Instruction for Diamond peel**

1. Push in the movable ring of copper nozzle at the hand piece, and pull out the 6x4 size vacuum hose. Then push in the black plastic ring on the vacuum hose.
2. Put a drop of cream or milk lotion on the

vacuum hose nozzle, push in the movable ring and insert in the vacuum hose to the limit.

3. Insert a cotton swab into the diamond head, then tight up the diamond head on the hand piece. Remove the cotton swab after every treatment.
4. Adjust the vacuum intensity then place the hand piece on the treatment area and follow the suggested movements as in the reference picture.
5. To clean the diamond head, use toothbrush or cotton swab and damped with a little alcohol to wipe off the dirty impurity, then use UV light or high temperature for sterilization.

#### Hot & cold treatment instruction

1. Connect the hot & cold plug to the machine; please note the socket is in the "Hot & Cold" Area.
2. Start the machine: To turn on the machine, press the "on" button. And then start the hot & cold treatment function.
3. Please wait 3 minutes, and then you can apply the treatment. For the machine need time to warming up or cool down the treatment probe.
4. Normally one treatment program is about 5 ~10 minutes. Please note that do not stay in one treatment place over time when applying the treatment to avoid any hurt caused by cool head.
5. After treatment, please turn off the machine and then pull out the plug, make clean the probe.

#### DIAGRAM OF SUGGESTED MOVEMENTS AND AREAS FOR DERMABRASION

