

Instruction of VS-905 5 In 1 Beauty Equipment

Preface

The Micro-crystal Dermabrasion was designed by Italian Florence's Mattioli at first, until now had over 20 years' history. This kind of technology only supplies to the dermatology expert and doctor at first uses, but more and more SPA House & Beauty Saloon also gradually uses, it was called the best method of medical beauty. This method successfully has developed in Europe for several years, cures patient thousands of examples, and has obtained the extremely satisfactory effect.

Now we want to introduce the **Diamond Dermabrasion**, it is a creative improvement of the Micro-crystal Dermabrasion. The Diamond Dermabrasion provided a non-surgical skin refinish procedure, by using sterile diamond heads to abrade or rub off the top skin layer, then Vacuuming out the particles along with any dirt and dead

Skin back up. This procedure removes skin debris, imperfections, blemishes, wrinkles and unwanted pigmentation on the skin. Moderate usage of Diamond Dermabrasion, accompany with skin products that penetrate through top skin layer that reach the dermis level, helps to replenish natural nutrients, restore cell activity, and promote skin health. The results after Micro Dermabrasion treatment is usually a healthy, glowing, beautiful skin.



Functions Introduction:

*** Preparation before treatment

1. A consultation included the examination of the skin, a plan for treatment.
2. On oily or acne skin, a steamer may be used prior to the treatment, or a degreasing lotion may be applied prior to the cleanser.

1.The Diamond Dermabrasion. (See the preface)



Diamond Dermabrasion Operating Instruction:

1. Push in the movable ring of copper nozzle at the hand piece, and pull out the 6x4 size vacuum hose. Then push in the black plastic ring on the vacuum hose.
2. Put a drop of cream or milk lotion on the vacuum hose nozzle, push in the movable ring and insert in the vacuum hose to the limit.
3. Insert a cotton swab into the Diamond Head, then tight up the Diamond Head on the hand piece. Remove the cotton swab after every treatment.
4. Turn on the power switch.
5. Adjust the Vacuum intensity by Vacuum Regulator according to the following reference table, the intensity displays on Vacuum gauge and check with the client for comfort.

Recommended Vacuum intensity on treatment area

Treatment Area	Normal Skin (inHg)	Sensitive Skin (inHg)
Face	12-22	8-12
Forehead	12-22	10-12
Chest	15-20	15-18
Nose	10-15	10-14
Neck	18-18	10-15
Near Eye Area	8-12	8-10


Treatment Schedule Table

Treatment	Frequency (days)	Total Visits
Face Peeling	6-10	3-5
Fine Lines	7-13	6-8
Neck	12-14	3-5
Acne Scars	7-12	8-10
Hyperpigmentation	7-12	8-10

6. Use the incline area of Hand piece to contact with the skin surface. Proceed with a slow sliding motion. Do not place the hand piece at one spot for too long, doing so will cause harm to the skin.
7. Adjust the intensity slowly from minimum to maximum. For commonly use, do not set the intensity maximum unless it is of necessity. This is for better comfort.
8. Gently stretch the skin with thumb and finger and glide the Hand-piece 2 to 3 times over the area being treated. Then cross again in different directions to prevent streaking lines as in the reference picture. Brush strokes or polishing motions may be used in different sites.

DIAGRAM OF SUGGESTED MOVEMENTS AND AREAS FOR DERMABRASION

(see next page)

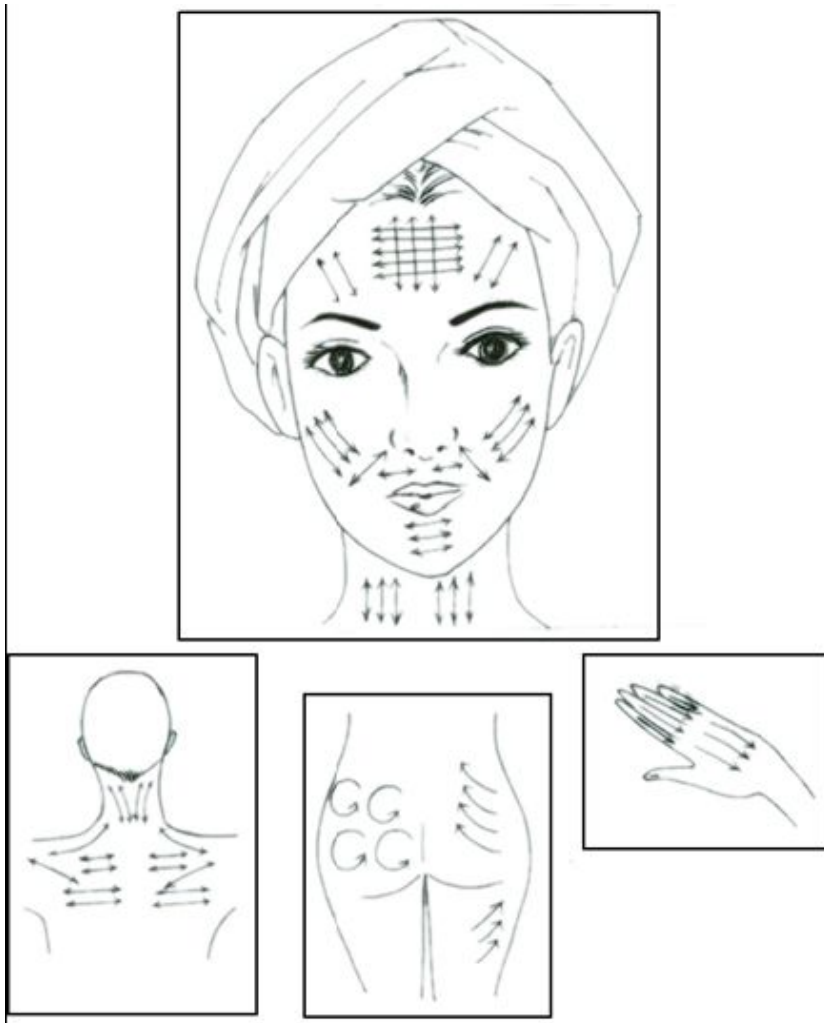

***Please follow direction of arrows.**

9. After treatment, brush off all remaining dead skin with a soft brush and rinse with damp sponges or damp cool towel.

10. Allow skin to dry then apply desired skin product.

11. To clean the Diamond Head, use toothbrush or cotton swab damped with a little alcohol to wipe off the dirty impurity, and then use UV light or high temperature for sterilization.

***** Important:**

1. Clean skin with clear water.

2. Never stop the electrode on the skin in one spot.

3. Always keeps the electrode moving across the skin.

4. Never point the electrode towards chest area.

5. Do not immerse the handle or the machine into water. Doing so could cause serious injury or death.

6. Store in a dry cool place.

Safety Warning

Important Notice of Caution before using this machine:

Never operate this equipment on a person has or suffers from the following medical conditions described

below. (The salon or operator may consider having a client sign a waiver to the effect that he/she is free of any poor health conditions.)

1. Heart Disease
2. Heart Murmur or Irregular Heart Rate
3. Has a Pacemaker or other type of heart rate balancer.
4. Pregnancy
5. Epilepsy
6. Alcoholism
7. Ulcers
8. Disease of Stomach, Intestine, Liver, or Pancreas.
9. Breast Cancer.
10. Undergoing treatment for any form of Cancer.
11. Had major surgery within the past year.
12. Asthma
13. Cold, Flu, or Upset Stomach
14. Any serious illness or disease not listed above.

Do not use on these areas:

1. Sensitive Skin
2. Skin Rash
3. Damaged, Inflamed, or Infected Skin.
4. Surgical scar less than 12 months old.
5. Skin Swollen.
6. The probes can not point to Eye ball.
7. Genitals.

1. Always begin treatment conservatively and observe the effects by the touch and appearance of the skin. A vacuum power that is too high or too big may cause an inflammatory response.

2. Edema and erythema can occur following treatment, but will resolve in few hours.

3. Overly aggressive techniques can cause mild flaking over the next 3-4 days. Epidermis that is treated too aggressively can lead to bleeding in the superficial dermis.

4. Diamond Dermabrasion should not be used on sensitive areas such as eyelids. The eyelids should be closed at all times during the procedure and may be covered with damp cotton pads.

5. The following areas are considered inappropriate for treatment: Skin tumor, moles, birthmark, angioma, lips, eyes, herpes, skin cancer, and liver spots.

6. Do not modify or attempt to repair the machine, contact the technician for maintenance.

2. BIO:



Skin tightening after cellulite and fat breaking down;
Lifting up, for example, faces, breasts, hips and etc.

BIO Operating Instruction:

Operating Instruction

1. Connect the spot handle to BIO outlet.

Spot handles: For small area, such as nasal fossa, and so on.

3. Select the time as you like about 15 minutes or 30 minutes.

4. Select the strong or weak intensity.

5. Select the mode just you like such as hip, bust, neck or face.

Can be used on:

- Micro wave Anti-wrinkle massage suitable for sagging, ageing, excrescence skin and eye bag muscle.

- Skin Maintain the pores and vessels. Clean up pores and vessels.

3. Ultrasound



Ultrasound utilizes sonic vibrations; it works one million times per second (1 MHz). The ultrasonic wave could stimulate skin tissue and improve permeability of the membrane to promoted collagen tissue, reduced wrinkle and invisible appeared fine line, also lift the skin result for younger and healthy look. Ultrasonic generate frictional heat which accelerate blood circulation, muscle relaxation, and decompose the cellulite to break down exceed fat.

It can help the beautician apply the recover treatment after peeling treatment. **Note:** Let the client **hold the Negative Stick**, and then operator hold the ultrasound probe.


4. Photon



Senility-resisting, anti-allergic introducing nutrition, losing skin, degreasing, improving skin, activating cell, reducing inflammation, accelerating cell reproduction and cell membrane reproduction, written the skin, reducing spot and melanin.

Photon Operating Instruction:

1. It is not recommended to put the probes on a same position for longtime. Probes may move on the skin in regular line or circle.

	<ul style="list-style-type: none">● WAVE There are 2 kinds of waves to choose, Continuous Wave & Intermittent Wave
	<ul style="list-style-type: none">● MODE/ON MODE: Choose different color (among 7 colors) ON: Start the photon function
	<ul style="list-style-type: none">● AUTO Those 7 colors will operated automatically .
	<ul style="list-style-type: none">● OFF Stop the photon function

2. Choose the proper spectrum reaching different depth on skin.

Orders of color by pressing MODE/ON button	Depth In Skin (Max 6, Min 1)	Suitable Treatments For
1 Red	6	Aging, Loose, Lead In Nuture, Burn Fat
2 Green	4	Capillary vessel dilation, Splash, Detumescence, Diminish inflammation, Eye lifting, Dark eye rim, Pouch, Face, Light speckle, Breast lifting
3 Blue	2	Acne, Susceptivity skin, Diminish inflammation and pox
4 Yellow	5	Dim skin, Lark skin, Improve lymph, Activate cells
5 Purple	1	Accelerate cell regeneration
6 Cyan	3	Anti hypersensitivity, Reducing appearance of capillary vessel
7 White	1~6 Combination	Whitening, Light speckle, Reducing melanin

Note:

- Before use, choose color according to the skin status.
 - The unsuitable setting may cause rubefaction on sensitive skin. When this occur, can use the cold side of Hot & Cold Hammer to cool down the area. The rubefaction will disappear in 24 hours.
- Photon Function can be combined with "MICRO ELECTRIC CURRENT", which will make the photon function more effective.

5. Skin Scrubber:



It removes dead skin cells, smooth wrinkles, treats pimples and clears the complexion. Mscrubber technology uses ultrasonic waves to shake loose dead skin cells, it also create heat and vibration to give a pleasant massage which will drive creams and ampoules into the skin.

Super Skin Scrubber Operating Instruction:

Super Skin Scrubber is a superior beauty instrument of the same kinds. Each Person after treatment a period of time, can proved to be having significant effects on skin, especially for facial treatment. It employs a variety of currents, to produce vibrations with a frequency of 28000 times per second. It provides the ability to penetrate skins and activate pores underneath. Moderate usage of ultrasound, accompany with suitable

emulsion or gel can eliminate pigmentation, wrinkle, acne, dead skin and dirt, making the skin shining and elastic.

Preparation

(1) Set the machine to No time limited or Time limited mode. The machine is set to No time limited mode at default, the

Time window displays “— —”.

To set Time limited mode, use Time adjuster to set desired time. When the time is up, the Time window displays “00”. To operate the machine again, use Time Adjuster to adjust.

Cleaning Skin

(1) Plug the Scrub Head into Scrubber OUTPUT.

Note: Let the client **hold the Negative Stick**

(as the right photo shows) and then operator hold the scrubber.



(2) Apply cleansing lotion on the skin for 20 seconds; gently rub the skin with the front of the scrub head at 35 to 40 degree angle.

Skin Care

After Cleaning Skin, Apply essential gel on the skin and massage parallel with the back of the scrub head.

3. Cleansing the skin with Cleansing gel or water, then wipe off with cotton pad. Allow the skin to dry.

Treatment Reference Table

Function	Operate Mode	Output Intensity	Time
Lifting and Melanin Cure	Cleaning	Level 1-4	2-3 Mins
Nutrition Conduction	Massaging	Level 1-4	2-3 Mins
Remove Acarid Worms	Cleaning	Level 5-6	2-3 Mins
Clear Lead Poison	Cleaning	Level 7-8	2-3 Mins
Caution: Do not over adjust the intensity, should be adjust from low to high till reach comfort zone.			

Safety Warning:

1. Never operate this equipment on a person has or suffers from the following medical conditions: Heart diseases, Sensitive Skin, Damaged, Inflamed, or Infected skin.
2. When proceed deep cleaning or Galvanic, accompanies with suitable emulsion or gel.
3. Never stop the handle's blade on the skin in one spot, always keep the blade moving on the skin.
4. Never point the blade towards the eyes or throat.
5. Do not modify the equipment.
6. Do not immerse the handle or the machine into water.
7. After usage, clean the Handle and store in a cool and dry place. The machine should be grounding.

Trouble Shooting

If the machine does not function properly, please follow these procedures before requesting maintenance.

1. Low suction power/ low vacuum pressure:

Please check again the vacuum hose should be connected well to

the socket and then hand pieces. And then turn the Vacuum Regulator to the maximum, and then start the Diamond Dermabrasion, and then use finger to block the hole of the hand pieces. Normally the vacuum Gauge can reach the 24 inch Hg.

**** Please check the O ring in the Diamond pens & O ring in the socket!**

If you cannot settle the problem finally, please contact your distributor. Don't open the machine by yourself without the technician guidance.

2. No response when turn on the power.

Please exchange the fuse. And connect the power cord correctly. If cannot settle the problem, please contact to your distributor.

3. Supersonic hand piece cannot work properly.

Please connect again the plug. If cannot settle the problem, please contact to your distributor.

4. Hot & Cold treatment cannot use.

Please connect again the plug. If cannot settle the problem, please contact to your distributor.

Specifications

Packing Size: L49.5* W38.5* H29 cm

Voltage: 240V/50/60Hz □ 220V/50/60Hz □ 115V/60Hz □

Power: 145W

Fuse: 2A

Total Supplies and Accessories

Machine x 1

Manual x 1

Fuse x 3

Power cord x 1

1- Diamond Dermabrasion

PVC vacuum hose 6*4 x 1

Diamond pen x 2

O-rings x 3

Diamond Head x 9

Pens & Heads box 1

Cotton Filter x 1

2- Ultrasound Sonic Probe x 2

3- Photon Photon Probe x 1

4- BIO Handle x 3 pairs (6pcs)

5- Skin Scrubber Scrubber Handle x 1
Negative Stick x 1

, BIO Face Lift (NA-A04)

There are 3 pairs different handles available,

Mushroom handles: For large area such as the cheeks, forehead, neck, etc.

Spot handles: For small area, such as nasal fossa, and so on.

Handles without Cap: Please insert the Cotton stick (which you can find them in the manual book plastic bag), then dampen the cotton with anti-wrinkle essence or other kind of essences. Its operating method is same as the other 2 kinds of handles.

Note: Handles with damp cotton stick conducts electricity stronger than the other 2 kinds of handles.

3. Select the time as you like about 15 minutes or 30 minutes.

4. Select the strong or weak intensity.

5. Select the mode just you like such as hip, bust, neck or face.

Can be used on:

- Micro wave Anti-wrinkle massage suitable for sagging, ageing, excrescence skin and eye bag muscle.
- Skin Maintain the pores and vessels. Clean up pores and vessels.
Close up pores tighten muscles. Improve nutrition absorption and permeability. at the same time, prevent aging and help face reforming.
- Body Purify our body and blood. Smash fat cells and burn the fat have a same effect of arranging then makes our body look more airy-fairy.

Operation

You can turn on the NV-A04 by press the button of "POWER" on the panel. Press the button again to make the NV-A04 "OFF" .

Press the "PROG" button and use UP or DOWN button to set the working time. Every setting to increase or decrease 1 minute for the working time.

Press the "PROG" button again and use UP or DOWN button to adjust the output power. Now you can start the procedure.

Press the "PROG" button again and use UP or DOWN button to choose the model, there are three models of the BIO function.

Now you can start the procedure.