

# SR-WD60 Diamond Dermabrasion

## Machine Structure



## Face Panel Instruction



## For Function & Operation Details please refer to:

### Contents/ Function & Treatment/ ①

**1-Finish all accessories connection before turn on the machine**

→ Press power button to start → Adjust the Intensity Knob for the suction you want → Turn off the power button after use;

**2- Disconnect the power cord, then disconnect and clean accessories.**

## Package Included

- Diamond Dermabrasion Machine X 1
- PVC vacuum hose 6\*4 X 1
- Stainless steel wands X 2
- Diamond tips X 9
- Cotton filter X 2 packages (1big, 1 small)
- Rubber rings and fuses
- Full detailed English manual/ instructions

※ **Operating Instruction**

1- Please first connect all accessories to the machine.



1) Unscrew the vacuum connector, make sure a big cotton filter is in the center & a large black ring is around connector head



2) Screw down the connector after check. Then attach the PVC hose to the connector.



3) Place the other end of PVC hose in the end of wand.



4) Place a small cotton filter on the head of wand. (see, a small black ring is around the head)



5) Screw the diamond tip into the wand (choose proper size tip for customer, see behind **diagram of tips set**)

2- Put the plug into the power socket and turn on the power button. Turn the intensity knob to left side (minimum).

3- Adjust the vacuum intensity slowly from minimum to maximum, the vacuum meter will show the data of pressure. For commonly use, do not set the intensity to maximum unless necessary.

**Recommended Vacuum intensity on treatment area**

Treatment Area	Normal Skin (inHg)	Sensitive Skin (inHg)
Face	12-22	8-12
Forehead	12-22	10-12
Chest	15-20	15-18
Nose	10-15	10-14
Neck	18-18	10-15
Near Eye Area	8-12	8-10

**4- Treatment Schedule Table**

Treatment	Frequency (days)	Total Visits
Face Peeling	6-10	3-5
Fine Lines	7-13	6-8
Neck	12-14	3-5
Acne Scars	7-12	8-10
Hyperpigmentation	7-12	8-10

5- Place the diamond head on skin surface (start from forehead), proceed with a slow sliding motion. Do not leave the diamond tip at one spot for too long, doing so will cause harm to the skin.


6- Gently stretch the skin with thumb and finger and glide the Hand-piece 2 to 3 times over the area being treated. Then cross again in different directions to prevent streaking lines as in the reference picture. Brush strokes or polishing motions may be used in different sites.  
(See behind **diagram of suggested movements and areas**)

7- After treatment, brush off all remaining dead skin with a soft brush and rinse with damp sponges or damp cool towel.

8- Allow skin to dry then apply desired skin product.

9- To clean the Diamond Head, use toothbrush or cotton swab damped with a little alcohol to wipe off the dirty impurity, and then use UV light or high temperature for sterilization.

**DIAGRAM OF SUGGESTED MOVEMENTS AND AREAS FOR DERMABRASION**

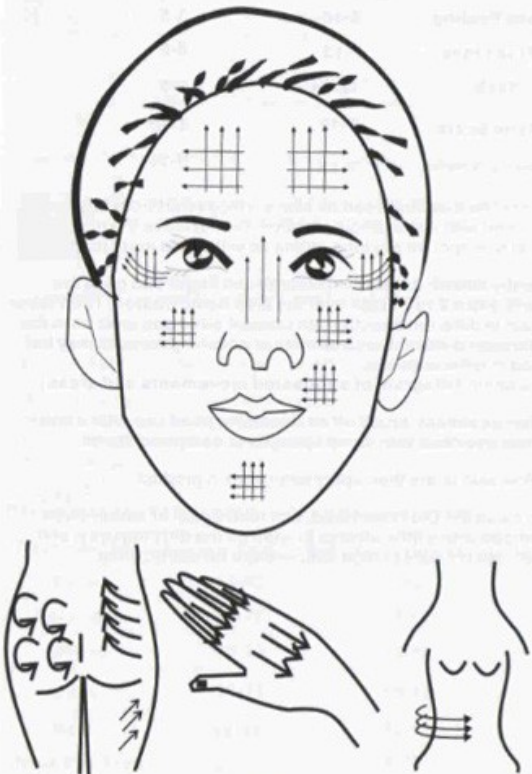

**Diagram of Diamond Dermabrasion Tips & Wands Set**

2 stainless steel wands which are used with 9 interchangeable diamond tips.

1		D80	Body. Especially for acne, knee, elbow areas and striae of pregnancy.
2		D100, D120	Face. For the larger blemishes.
3		D150, D180, D220	For Scars, fine lines and imperfections.
4		D160, D180	Used on the eyes and both sides of nose. For wrinkles, oil tablets, blackhead and acne.
5		Smooth Tip	After the D tips peeling procedure, use this smooth tip to vacuum/clear away the dead abraded skin

1- The higher the D number, the finer the grit.

2- Clean diamond tips with Alcohol, then sterilize them with UV light or high temperature.

3- Replace the big cotton filter inside the vacuum connector after every five to six treatment.

4- Replace the small cotton filter on the wand head after every treatment.