



SR-WO3

Instructions

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆ How to contact us?

Phone: (+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!

Principle

OxyGeneo

Gene+ takes its inspiration from natural hot springs known for inducing skin oxygenation. By simulating this effect, Gene+ triggers a body response that sends oxygen to the treated area. At the same time Gene + exfoliates the skin and creates an optimal environment for infusion of essential nutrients both during and after treatment. The ability to layer Radio Frequency induces collagen and elastin regeneration in the dermis layer, which leads to smoother and tighter skin and a reduction in fine lines and wrinkles.

Water Dermabrasion

Water dermabrasion is an exciting NEW technology that combines the efficacy of Microdermabrasion, vacuum system and new hydration system. It is a lot gentler than crystal microdermabrasion or diamond dermabrasion as only water is used, and we use professional quality machines.

Radio Frequency

The multi-polar RF delivering thermal energy into the subcutaneous tissue makes the tissue's natural resistance movement produce heat, the heated collagen shrink immediately and simulate dermal layer to excrete more collagen to make up for contraction and draining collagen, thus to hold the bracket of the skin, recover skin elasticity. When the collagen are produced continuously, it will increase the thickness and density of the dermal layer, fill wrinkles, eliminate scars, recover skin elasticity and gloss, making the skin white and smooth.

Mesotherapy Injector

1. Needle free injection skin beauty treatment= hyaluronic acid+ many kinds of skin care method
2. International high-end beauty import materials, a security for your beauty.
3. Fine injection, quick and convenient
4. Non-surgical treatment, tiny traumatic, high safety
5. Refined product, non-corroding, no metal residual, no sensitivity.
6. Just like medical beauty instrument, painless, noninvasive, no bleeding .
7. Effective permeability, open the tiny pore and enhance the absorption.
8. Aim at the root of skin problem, the absorption rate is 12 times of the normal beauty product.
9. Whole body can do the treatment.
10. Recover immediately, won't affect your normal life and work.


Functions

- a. Whitening and anti-wrinkle
- b. Ascension, tightening
- c. Sculpture facial contour
- d. Convergence chin
- e. Contractive pore, calm, allergy
- f. Improve eye problems (eye end lines, pouch, black rim of the eye)
- g. acne removal, dark circle removal, wrinkle removal, whitening, rejuvenation, lifting,
- h. cleansing skin
- i. rejuvenation, lightening
- j. skin whitening, skin lightening

Package list

- 1 x Main machine
- 1 x Hydro peeling handle
- 6 x Hydro tips
- 1 x Oxygeneo handle
- 2 x Oxygeneo tips
- 1 x Mesotherapy injector handle
- 1 x Injector tip
- 1 x Massage tip
- 6 x Syringe

Handles introduction

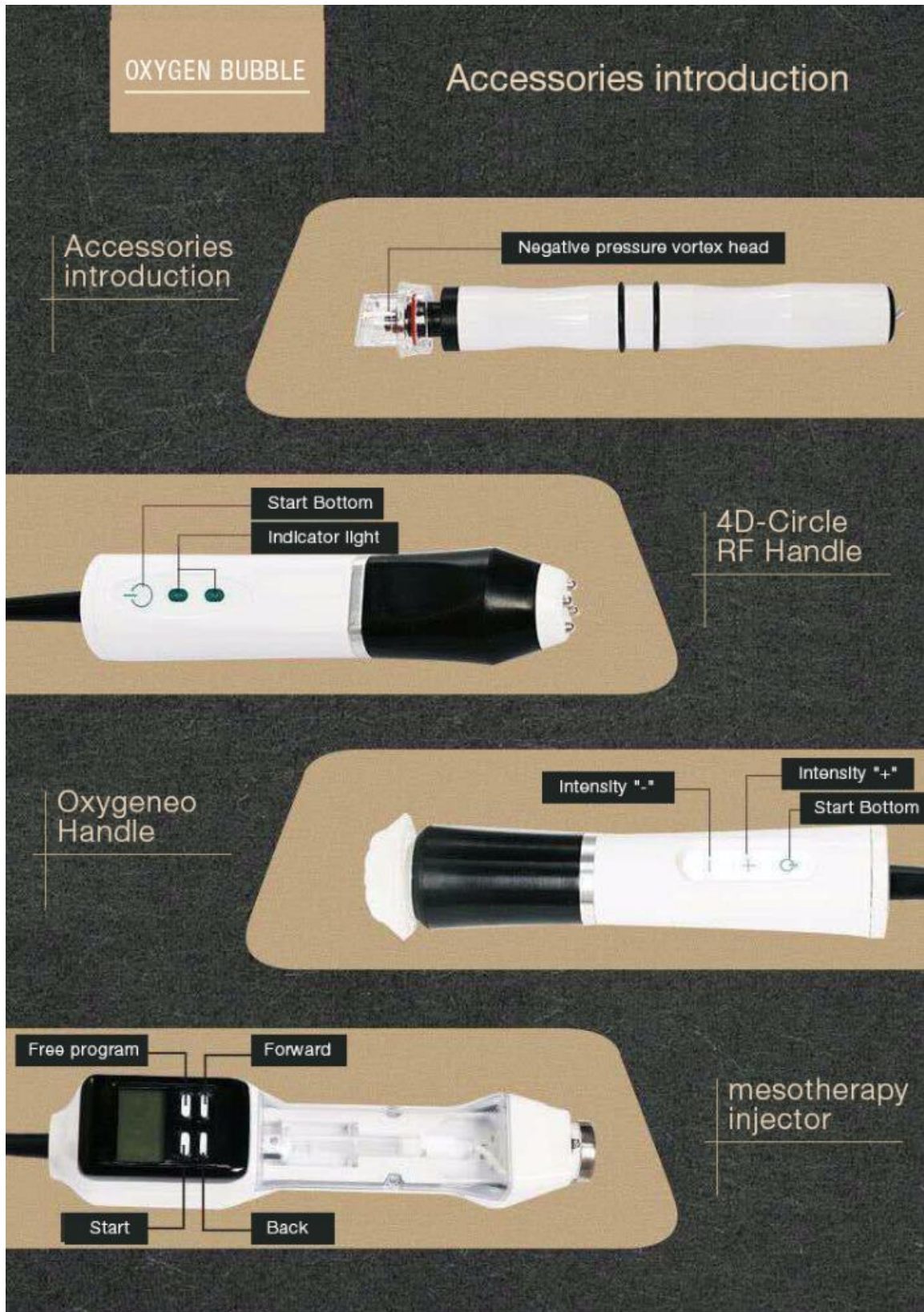

Product details

DETAILS

Every place is carved with care

Brake pedal control



Hold it here by hand
Insert and pull the
handle wire
Insert transparent
tube directly



Power Switch




Proeuct introduction



Operation

1. Plug in the power cord, turn on the earth leakage protection
2. Turn on the on/off button, "I" means on, "o" means off
3. Machine turn on, and show you the page to choose the language , English and Chinese available.
4. Choose the proper language then enter the main page

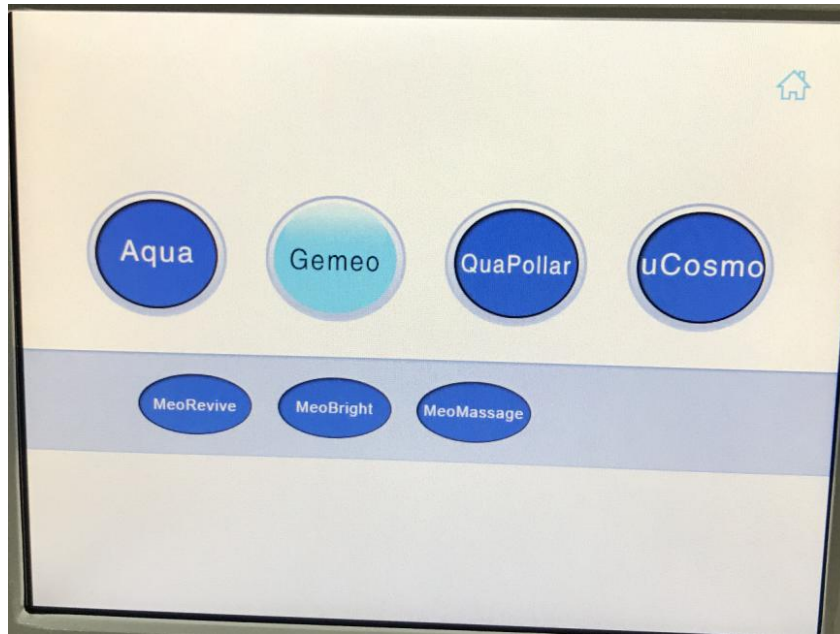


Aqua: means hydro dermabrasion function

Gemeo: means the Oxygeneo function


QuaPollar: means the multipolar RF function

uCosmo: means mesotherapy injector function

A: Press the Aqua button, enter hydro peeling page:

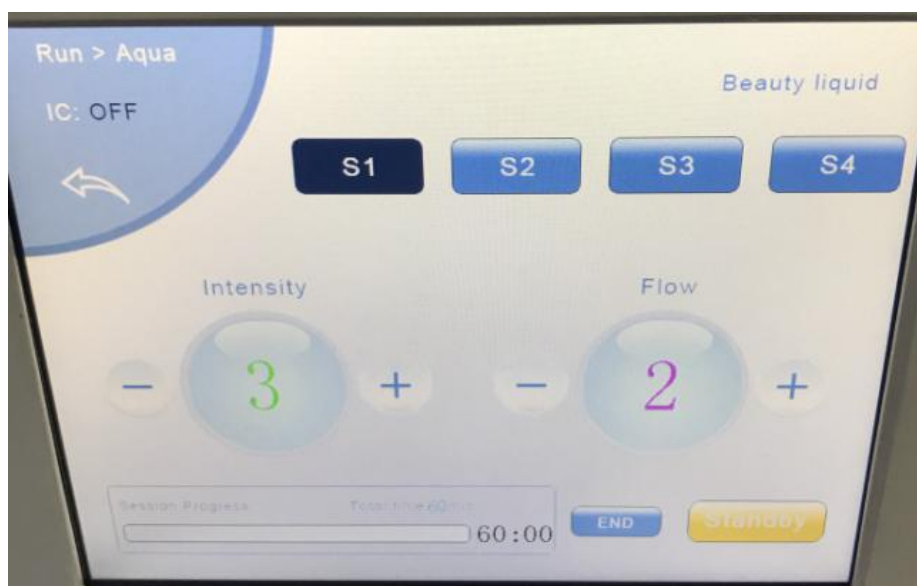

1. Adjusting the beauty liquid,

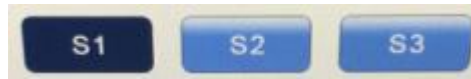


for beauty liquid,



For cleansing liquid

2.

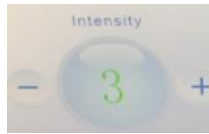

Press this image to adjust the intensity, 1-8 level available

3.



Press this image to adjust the water flow, 1-3 level optional


4. After adjusted all parameter, click "standby" to start treatment

5. Working time, the machine will stop automatically after 1 hour, then return to 60 minutes working time. If you need to do longer treatment, click "Standby" again




6. Click to return the main page

Note: how to change the hydro tips ? Do it like the below picture:

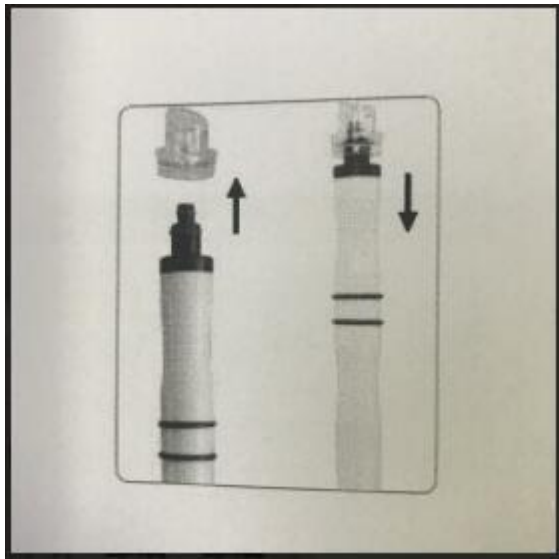

B: Press Gemeo function, enter the Oxygeneo function:

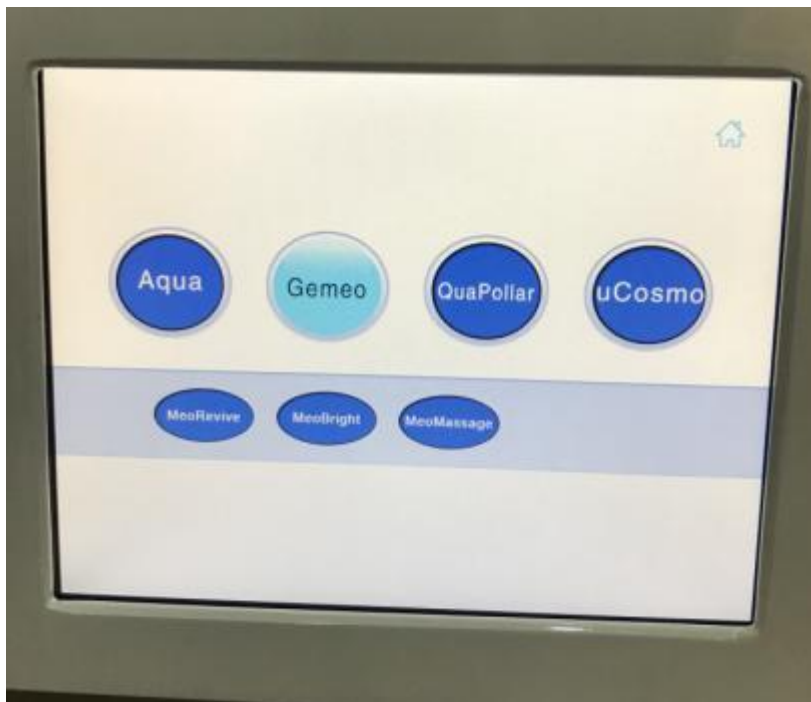

1. Gemeo included 3 different treatment, MeoRevive, MeoBright and MeoMassage


MeoRevive : Skin lifting firming

MeoBright : Skin brighten

MeoMassage: Absorption massage

2. MeoRevive / MeoBright / MeoMassage



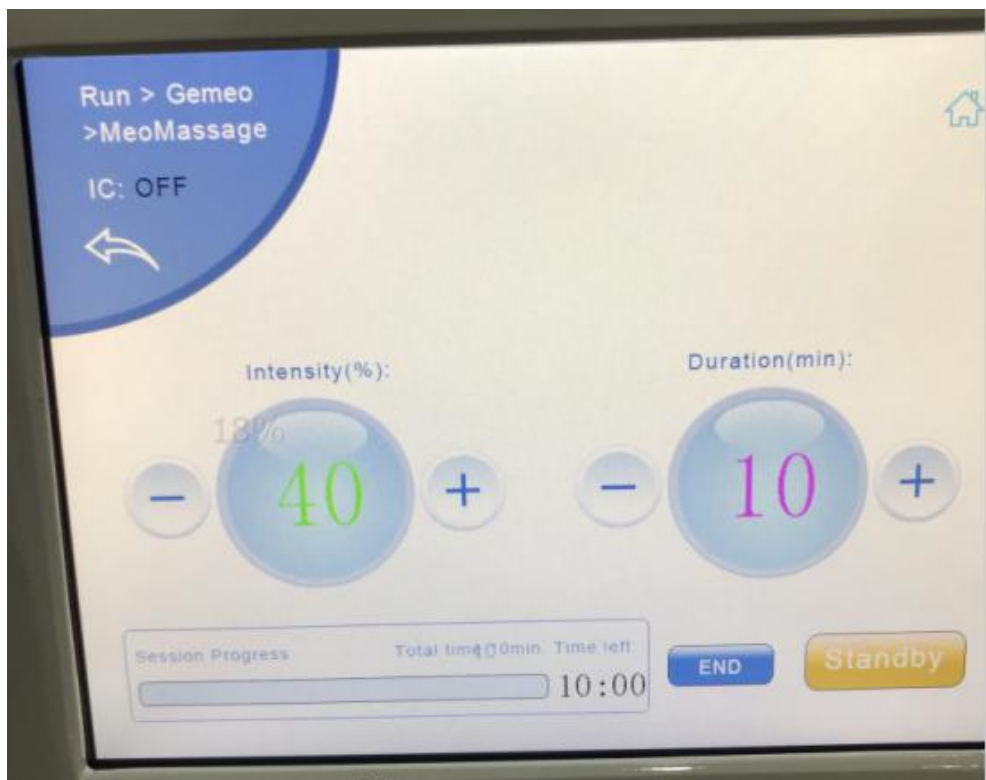


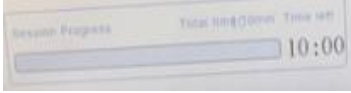
Press



to adjust the intensity, vibrate intensity for the working handle, 10-100 available, set up 10 by 10.


Press  to set up the treatment time, 2-20 minutes available, adjust 1 by 1.



 This to show you the working time for treatment, the machine will stop automatically when time over


Press  to start treatment

Press  to stop treatment

Press  return to main page


a. Start Session

When you click Standby to start treatment, the screen will you the page to remind you to connect the proper head to handle, **confirm OK then click the start button on handle**, handle light on, machine READY for treatment, if NOT, click Cancel, machine will return to STANDBY.

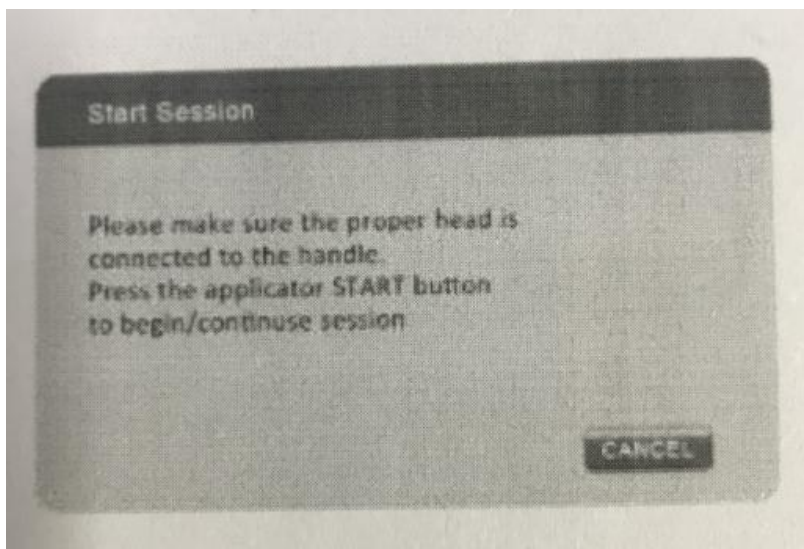


b. Session Paused

First click the Start button on handle, machine working, press again, session stop, and machine screen will you this page, if you want to continue the treatment, press the handle start button again, then screen page will disappear. Click EDIT button then will return the main operation page of Gemeo.

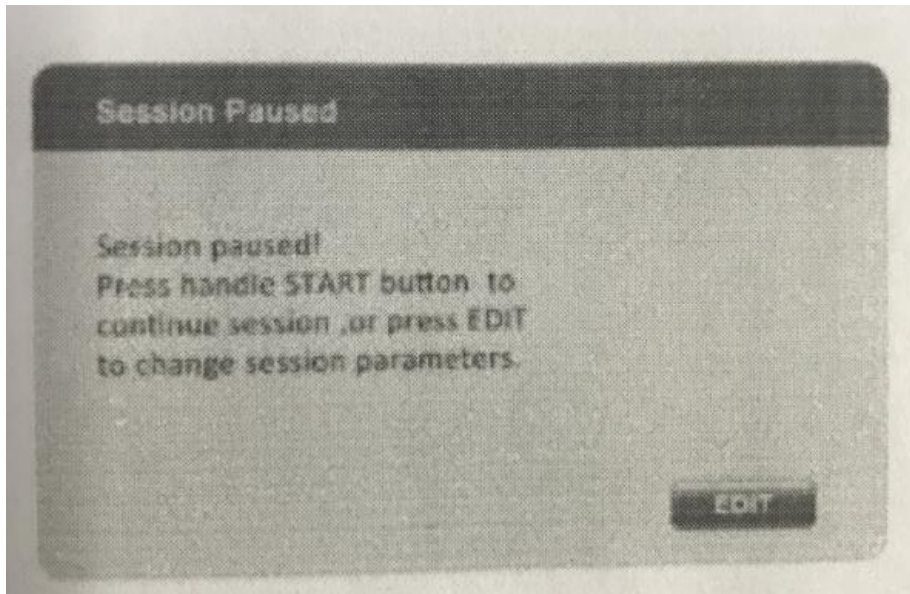



c. Session End

When treatment time over, machine screen show you the below page, press YES enter the MeoMessage operation page, press NO will return to the main page

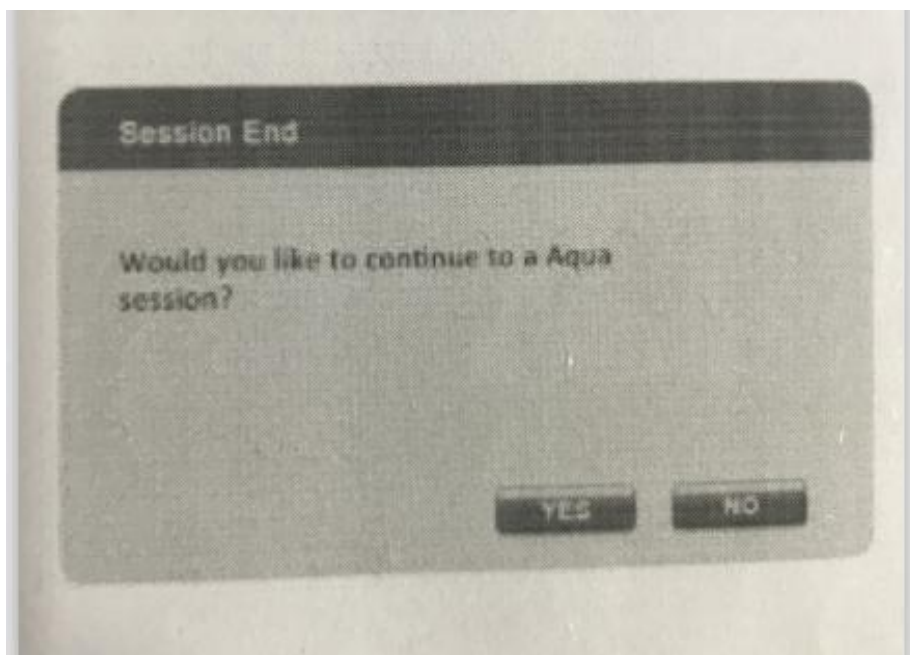


d. Stop Session

When you press the END button, screen show you the confirm page, if you still wanna exit the treatment, click EXIT, click CANCEL, treatment continue.

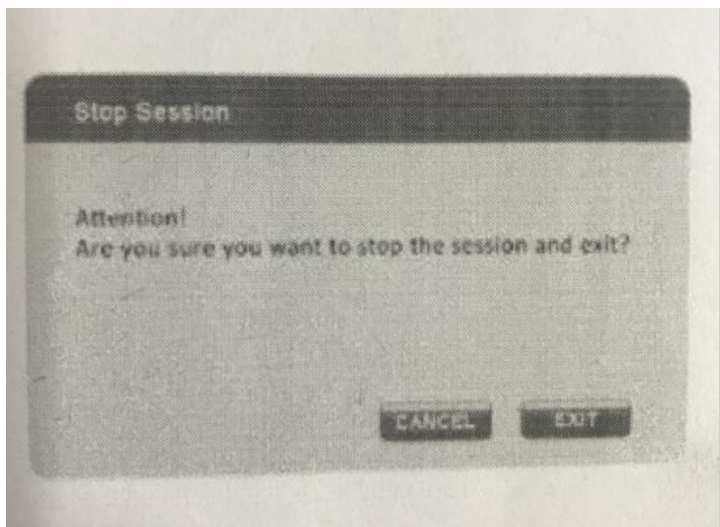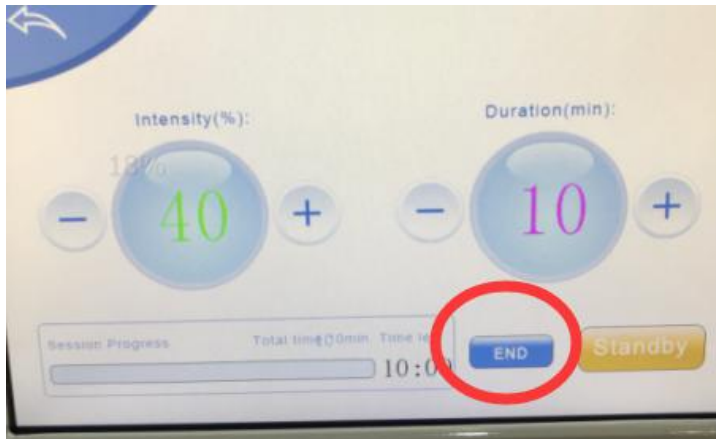


e. Session End


After MeoRevive, MeoBright and MeoMassage treatment, screen show you the session time is over, click DONE to return the main page

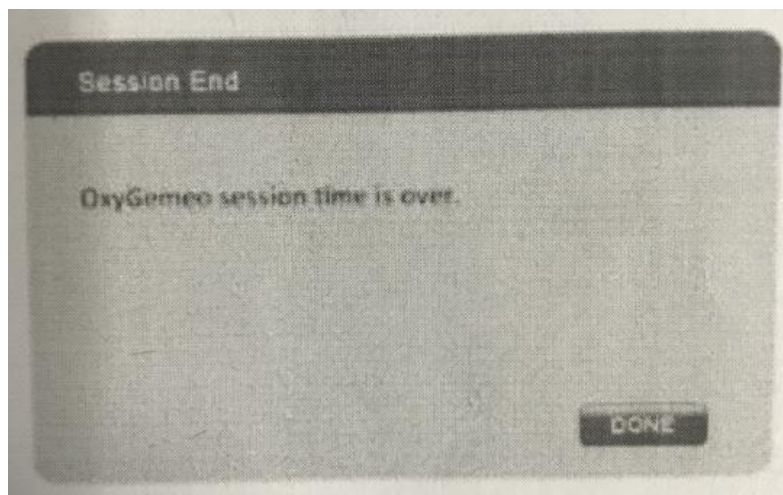


f. Oxygeneo handle button

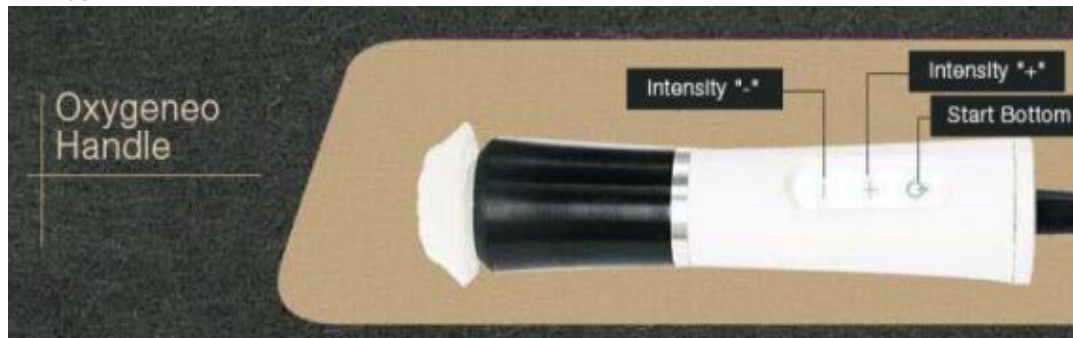


g. How to change the oxygeneo head:

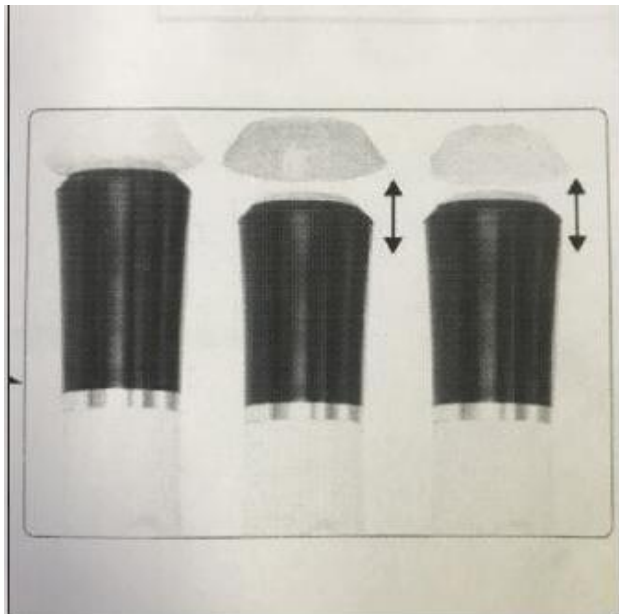

C. Press QuaPollar enter the Multipolar RF function

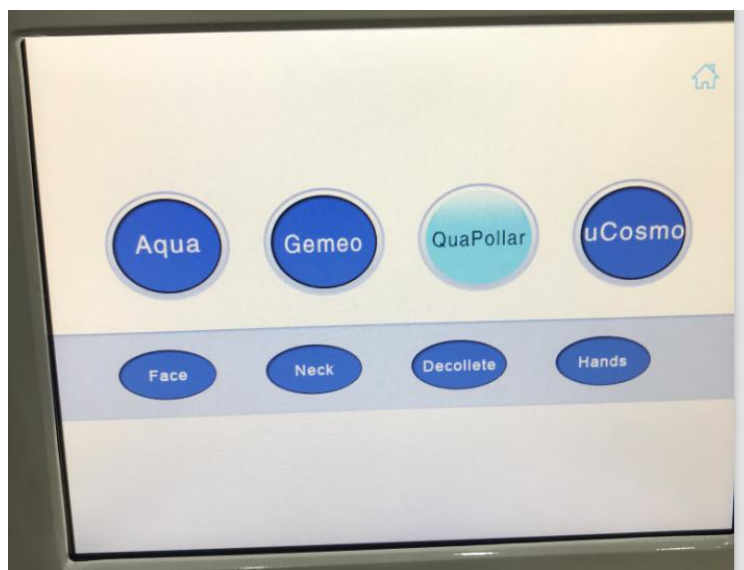

1. QuaPollar for 4 treatment area: Face, Neck, Decollete and Hands
2. Face, Neck, Decollete and Hands with same setting, for example the Face treatment page:

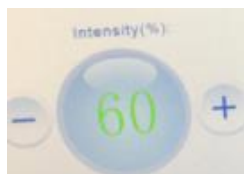

Press this to decrease/ increase the intensity, 5-100 available, 5 /step




Press this to adjust the treatment time, 2-60 mins available, 1/ step



This to show you the working time for treatment, the machine will stop automatically when time over

Press  to stop treatment

Press  return to main page




LOW/ MEDIUM/HIGH for the intensity range,


LOW: 5-50

Medium:55-80

High: 85-100


Press  to start, screen will show you the page to confirm if you already connect the

proper handle, the handle will light on, if OK, press the start button on handle to start the treatment, if NOT, click CANCEL to back to the main page of QuaPollar

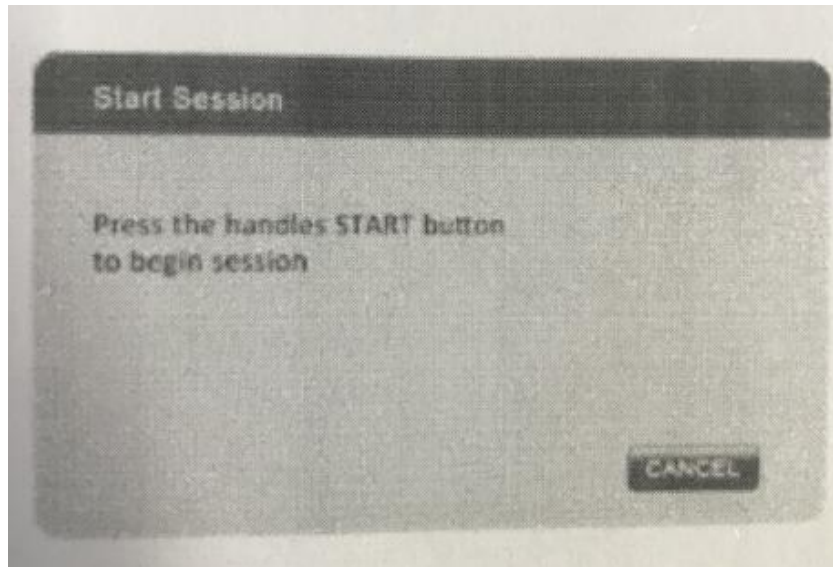


Session Paused and adjust the parameter

First click the Start button on handle, machine working, press again, session stop, and machine screen will you this page, if you want to continue the treatment, press the handle start button again, then screen page will disappear. Click EDIT button then will return the main operation page of QuaPollar, and you can adjust the parameter.

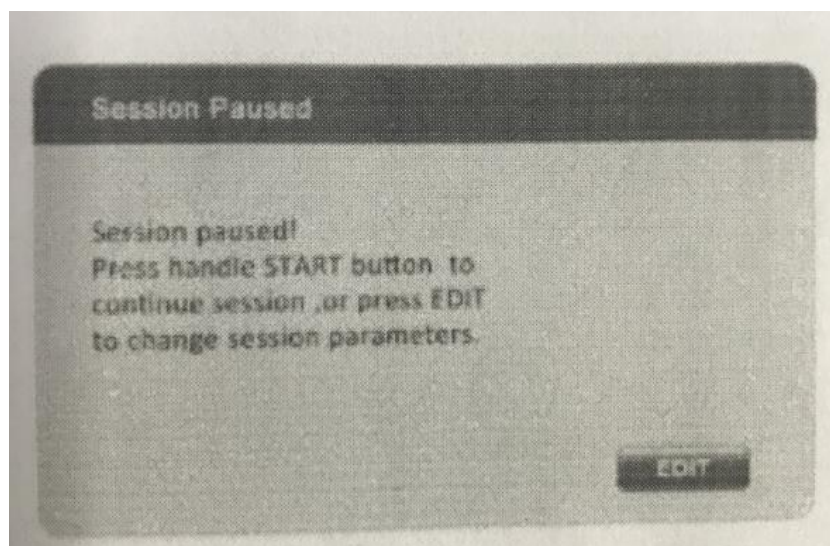



Session End

The last 1 min before treatment time over, machine will alarm, machine screen show you the below page, click DONE to return the main page

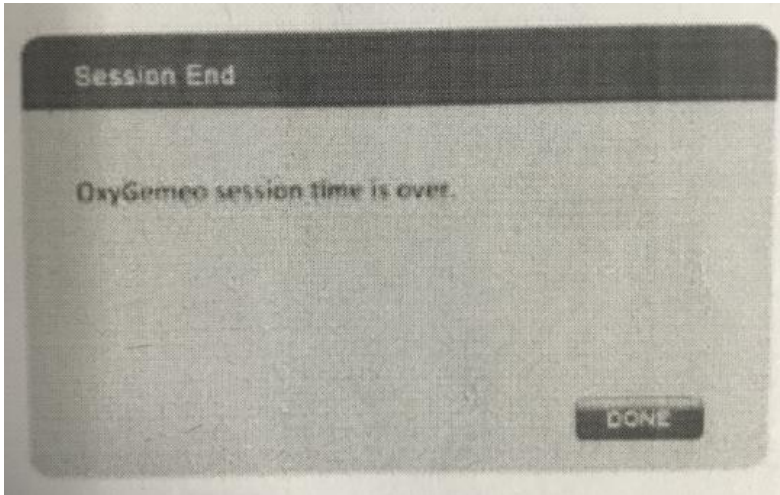


h. Stop Session

When you press the END button, screen show you the confirm page, if you still wanna exit the treatment, click EXIT, click CANCEL, treatment continue.

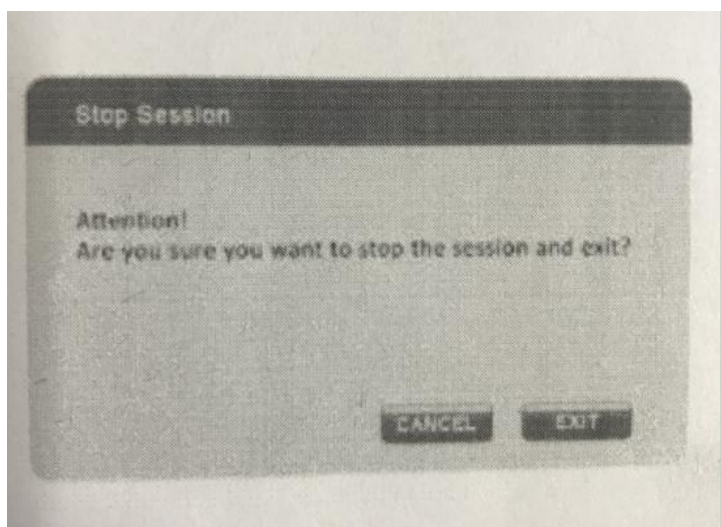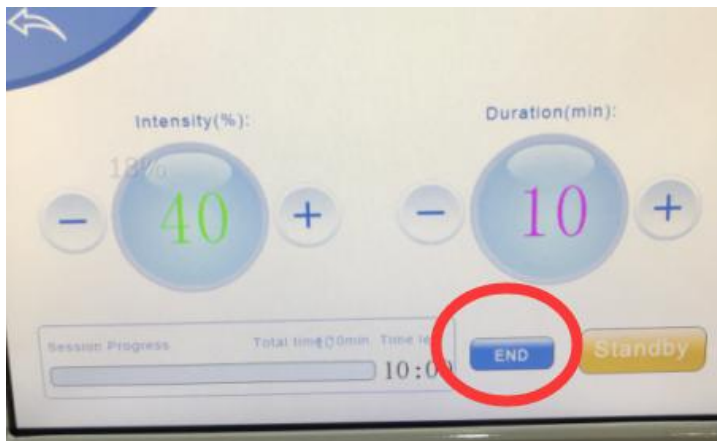

Smart QuaPollar handle


The handle indicator light will keep on when the head touch on skin, handle heat up normal, Indicator light blinking when head leave treatment area, and handle will not warm up.

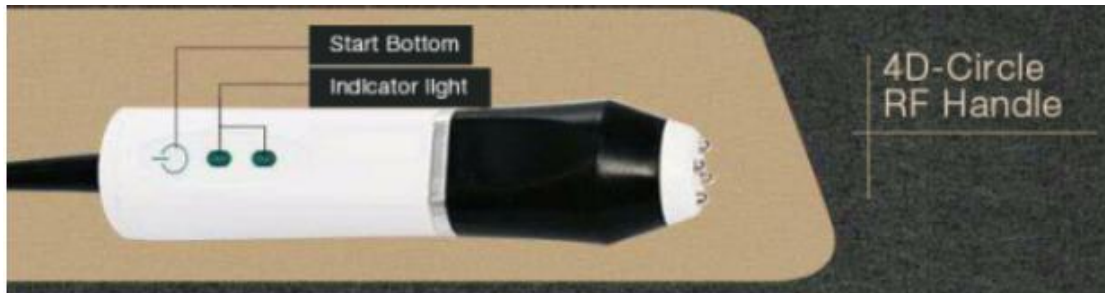


D. uCosmo

uCosmo come with 2 different heads, uCosmo injector and uCosmo massage


1. uCosmo injector head

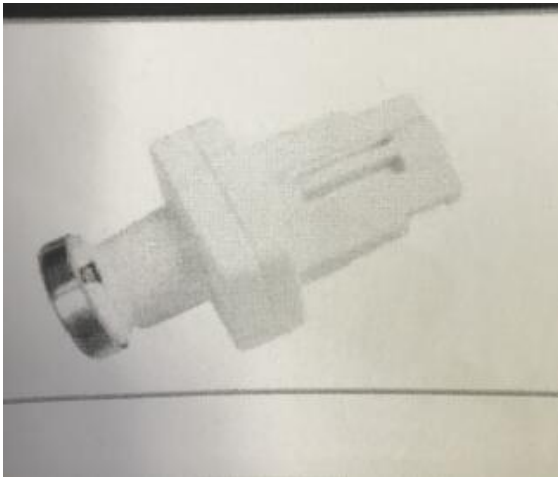


uCosmo massage head

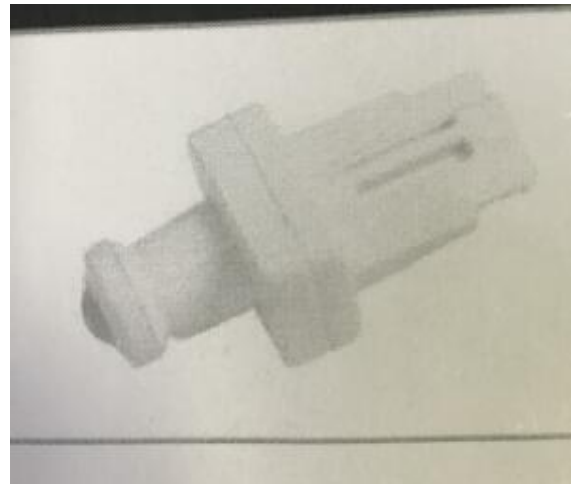


2. How to install head, and syringe :

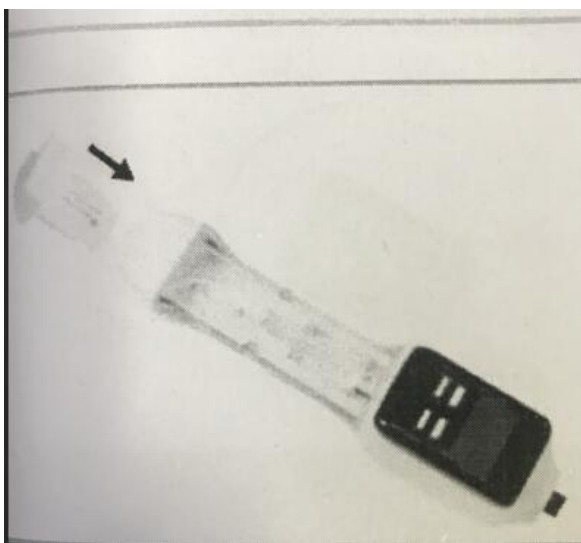


Choose the proper head and put it to the handle

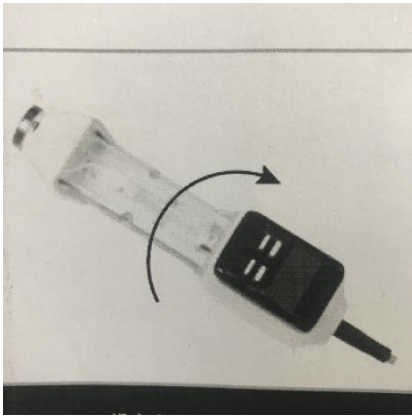


Open the syringe cover of the handle

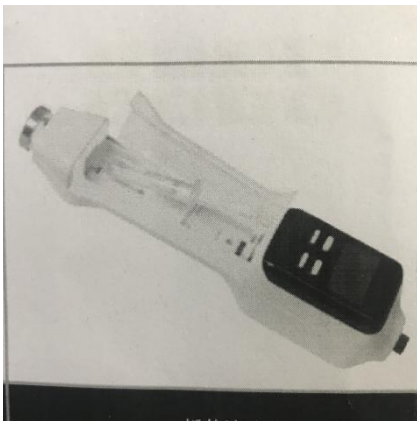

Put out the syringe, and add the proper liquid

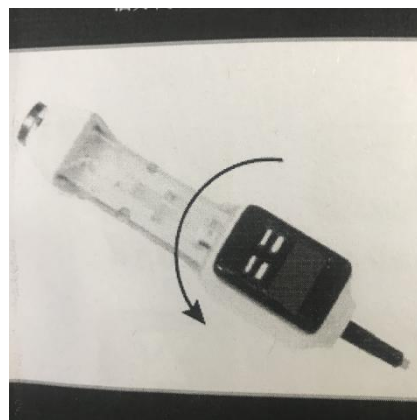


Then put back the syringe and the handle cover

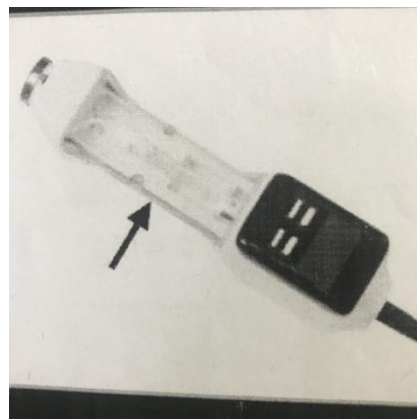



Check if already tighten the handle cover, fit the handle


3. uCosmo operate page:




 Skin Moisturized program


 Active program

 Anti-age program

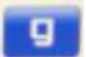 Free setting program

 Dry skin


 Neutral skin

 Oily skin

 Synthetic skin

1. When you choose    working program, choose the proper skin

Situation     , treatment with Fixed parameters

2. When you choose  working program, you can not choose the skin situation, you should set up the parameter




Press this to decrease/ increase the intensity,10-100 level available, 10/step




Press this to adjust the treatment time, 2-20 mins available, 1/ step




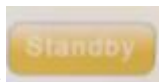
This to show you the working time for treatment, the machine will stop automatically when time over

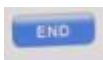

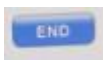
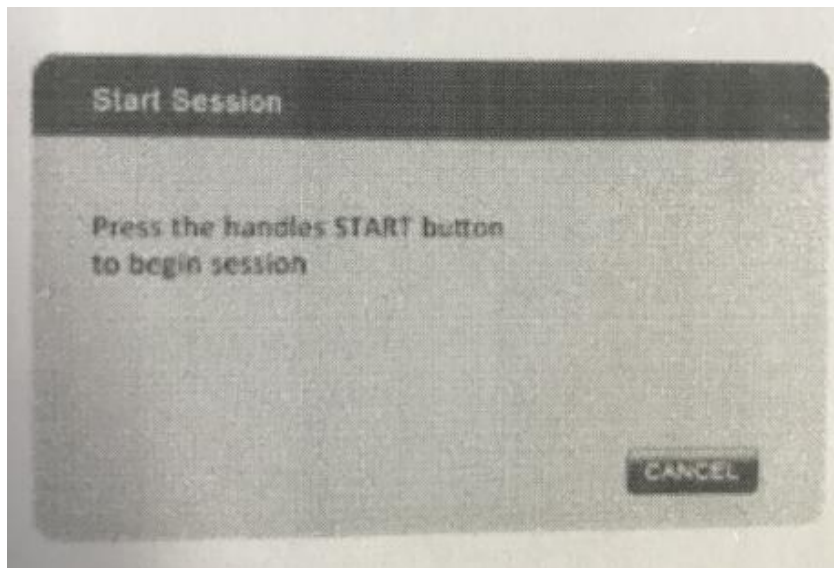
Press  to stop treatment



Press  return to main page




Press  then press the start button on handle to start the treatment, if NOT, click CANCEL to back to the main page of uCosmo




Session Paused

First click the Start button on handle, machine working, press again, session stop, and machine screen will you this page, if you want to continue the treatment, press the handle start button again, then screen page will disappear. Click EDIT button then will return the main operation page of uCosmo, and you can re choose the working program and skin situation.

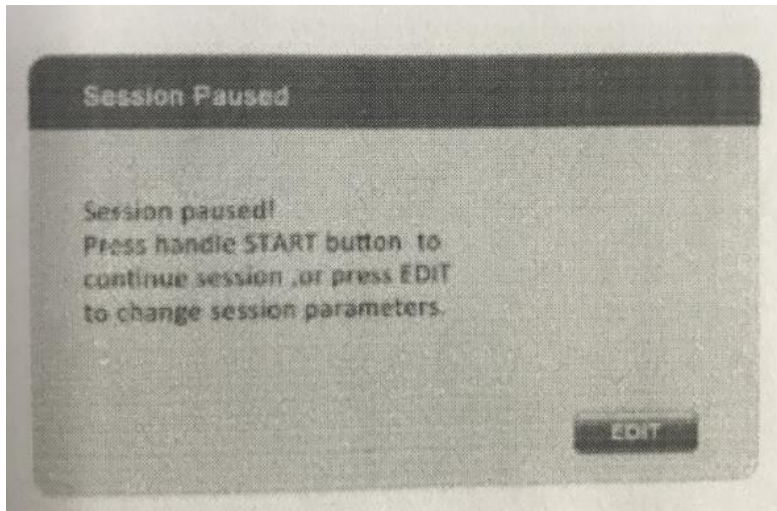


Session End

The last 1 min before treatment time over, machine will alarm, machine screen show you the Session Over page, click YES, change the uMessage head to enter the uMessage function page, Click NO return the main page


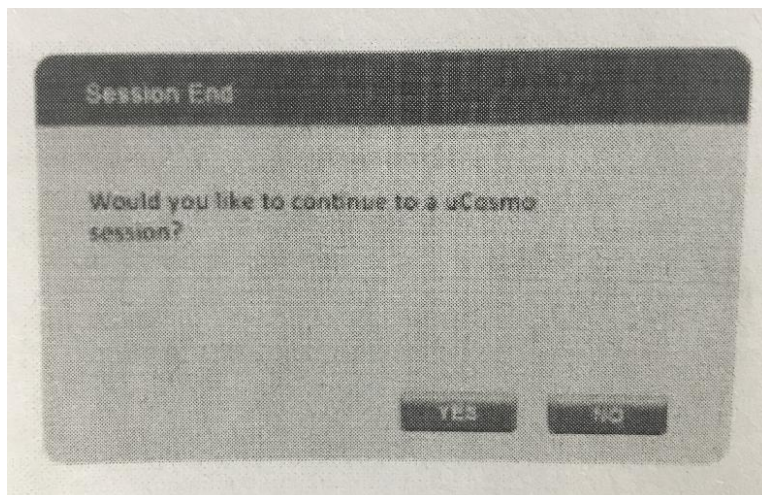


After uMessage treatment, screen will show you this page again, click DONE return to main page.

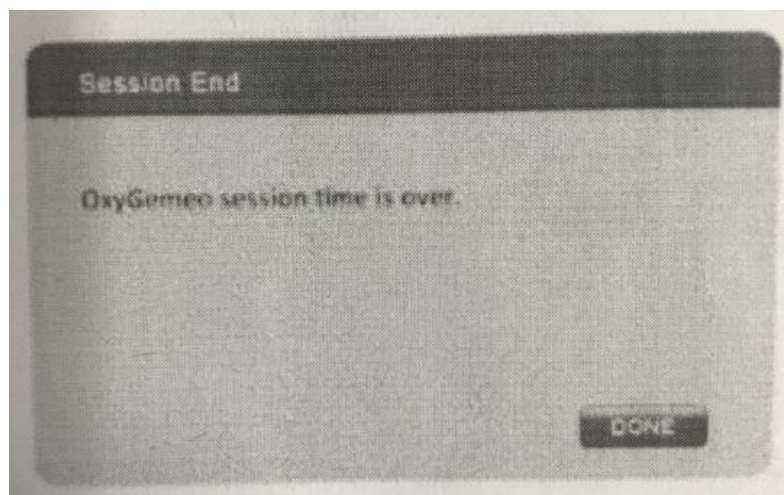


Stop Session

When you press the END button, screen show you the confirm page, if you still wanna exit the treatment, click EXIT, click CANCEL, treatment continue.

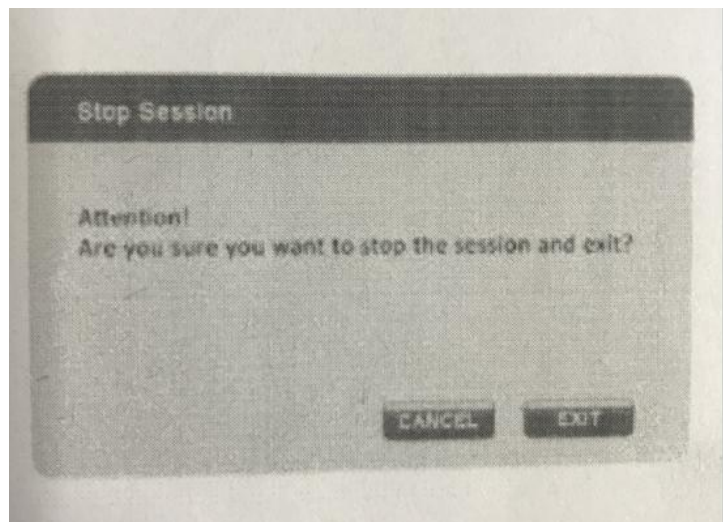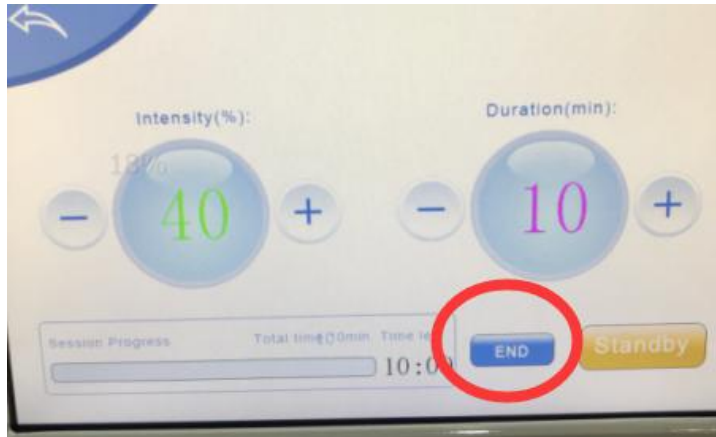

Handle of uCosmo

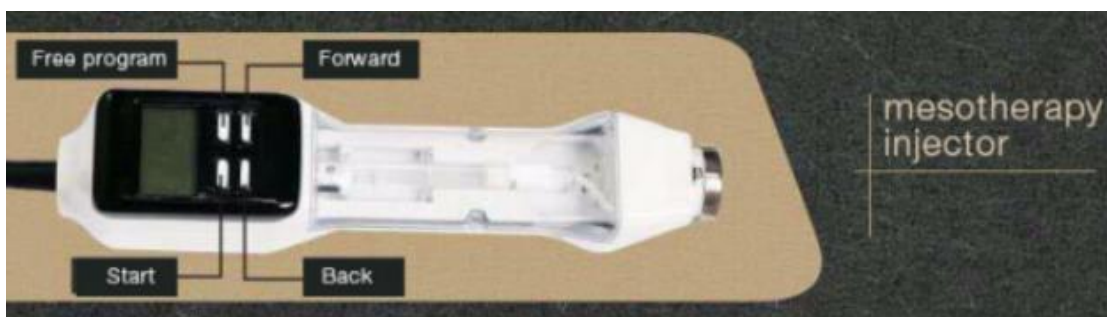

1. Use handle to change the working program: click Forward button to change the working program, you can change




working program

2. Continued press the Forward button can speed up the liquid forward, special for cleaning handle or the liquid in handle is less than 2ml

3. Press the back button can choose the skin situation when machine standby

LCD Screen of handle

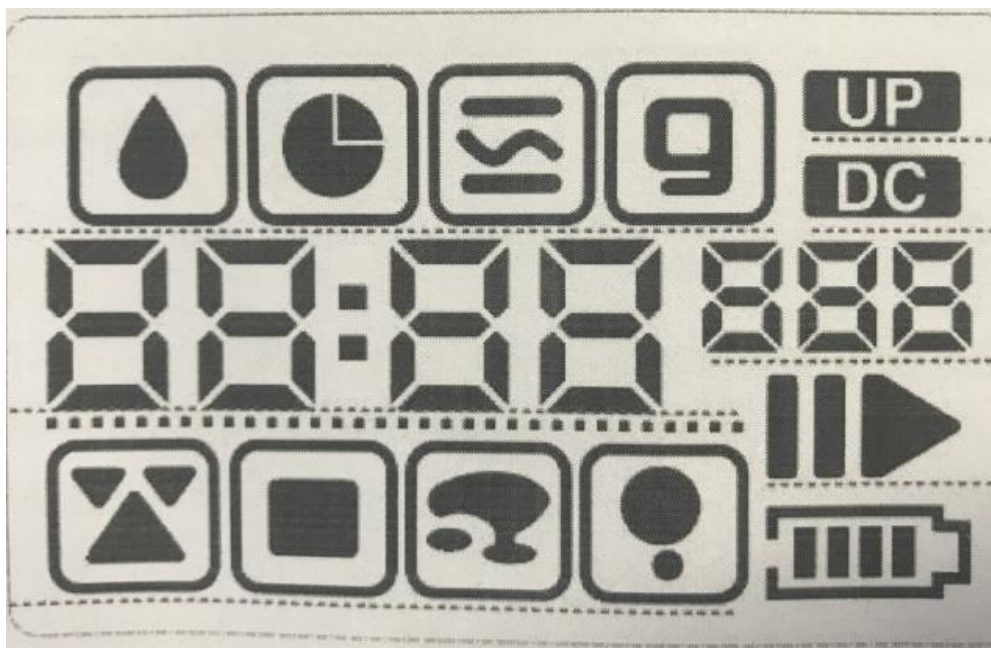

Working program



uCosmo injector



uMassage function



Working time




Intensity



Skin situation



Start / stop

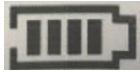

Power

After Treatment

1. Take out the liquid from the uCosmo handle
2. Put the handles on the machine holder
3. Return to main page or standby page
4. Turn off the machine
5. Turn off the earth leaking protection
6. Put out the power cord

Machine parts cleansing

1. Waste water bottle

The machine will stop automatically when the waste water bottle full, machine also will have the “DiDi” alarming. Turn to the standby page, turn off the machine and clean the waste water bottle and put it back



2. Beauty liquid/ cleansing liquid bottles

Turn right the handle and put them out to clean and put them back




3. uCosmo injector and uMassage head

Turn on the machine and change the Aqua function, choose the S4, set up the intensity and flow to the largest, then start the cleansing


Finally, use the syringe to add the S4 cleansing liquid, turn to the uCosmo injector function, long press the Forward button to speed up the syringe liquid spray, stop cleansing until the syringe liquid run out.

4. Change the filter


Must change the filter every 3 months, take out the waste water bottle first.

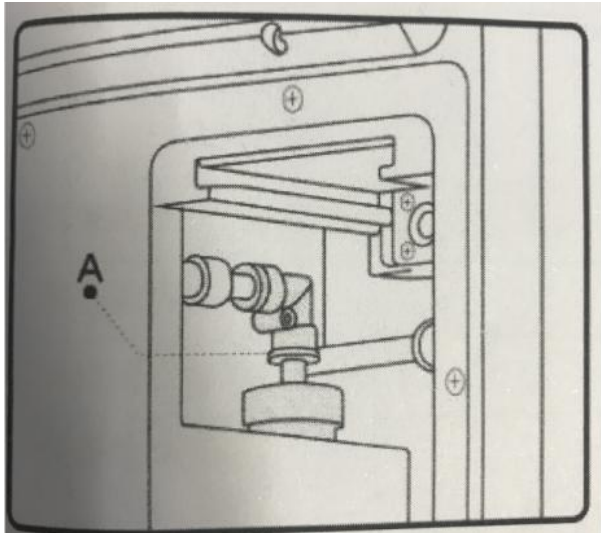


Push up the plastic ring (Marked A)

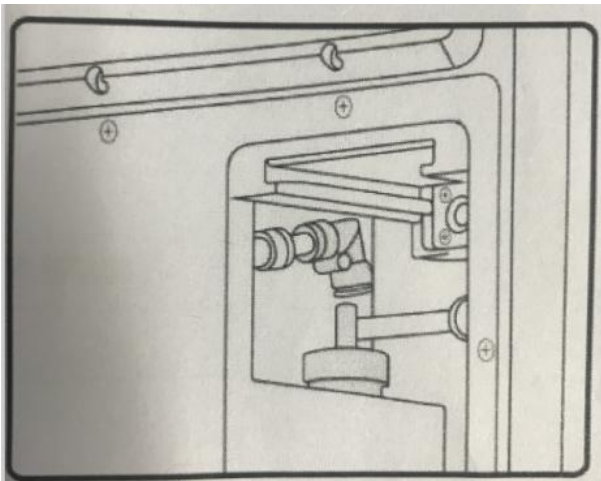

Separate the elbow from the filter

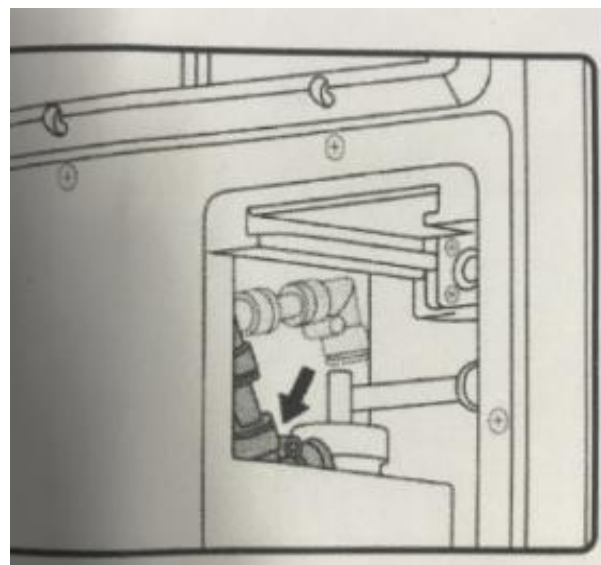

Elbow folded down about 60 degrees, so that you can put out the filter

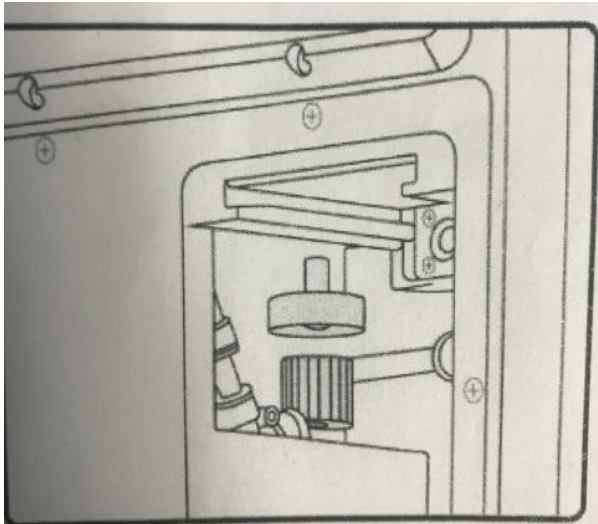

Turn right the filter cover to put out the filter, put it back and connect with the elbow tightly after you change the filter

Treatment operation



1. Massage the mandible center in circles to produce deep heat.
2. Massage lower jaw in lines.



3. Massage the triangle zone of both sides of the face in circles.
4. Massage from jaw to angulus oris to ear in lines.