

**New Cavitation Ultrasonic 6in1 RF Vacuum Lipo
laser**

Model: WL-939S



Chapter 1: Safety Warnings

I. Electricity & Machinery Safety

1. The machine is either 110V or 220V. So please check the machine voltage before starting the machine.
2. Please keep the voltage at stable condition.

II. Contraindication Safety

1. People with heart disease, high blood pressure, or configured cardiac Pacemaker.
2. People with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
3. People with hemorrhagic diseases, trauma or who is bleeding.
4. Pregnant women
5. Medical plastic artificial or metal parts inside the body such as metal teeth or silicone breast implants.
6. People with abnormal immune system.
7. People with numb or insensitive to heat.
8. If you are feeling unwell, Prohibited in the ears, nose, eye throat and do not use this device in advance and tell your doctor.
9. Children under 12 years old are not allowed.

III. Treatment Safety

1. Treatment Time is less than 30mins/Body Part/Each Treatment.
2. The energy setting is from low to high gradually which should be subject to skin comfortableness for different people.
3. The patient had better wear the protective eye glasses.
4. If you cannot tolerant heat or uncomfortable, please stop the treatment in advance and turn to your doctor for help.

Chapter 2: Installment and Use

I .Working Condition

Using at normal temperatures and pressure. Please do not stay in too cold, too hot, too dry and too wet.

II. Technical Parameters

Power supply Input:100V-240V

Power :180W

Unioisetion Probe:

Power supply output:150V

Frequency:40KHz

Power:25W

Vacuum RF for body

Power supply output:66V

Frequeuncy:3MHz

Power:30W

Sextupolar RF for body

Power supply output:66V

Frequeuncy:3MHz

Power:30W

Quadrupole RF head for Arm/face/small area of body Power supply output:66V

Frequeuncy:3MHz

Power:30W

Tripolar RF for face and eyes

Power supply output:66V

Frequeuncy:3MHz

Power:30W


Red Wavelength:650 - 730 nm

IV. How to install the Accessories?




V. Program Setting


1. Use the red power button to start the machine, the screen will show “Welcome”.

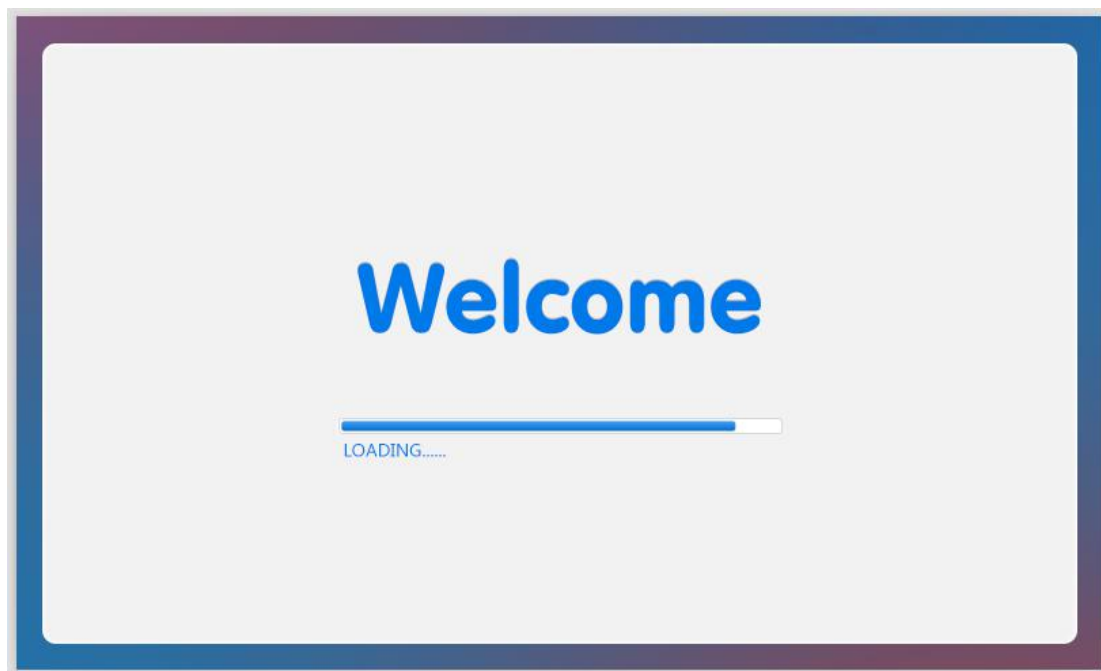




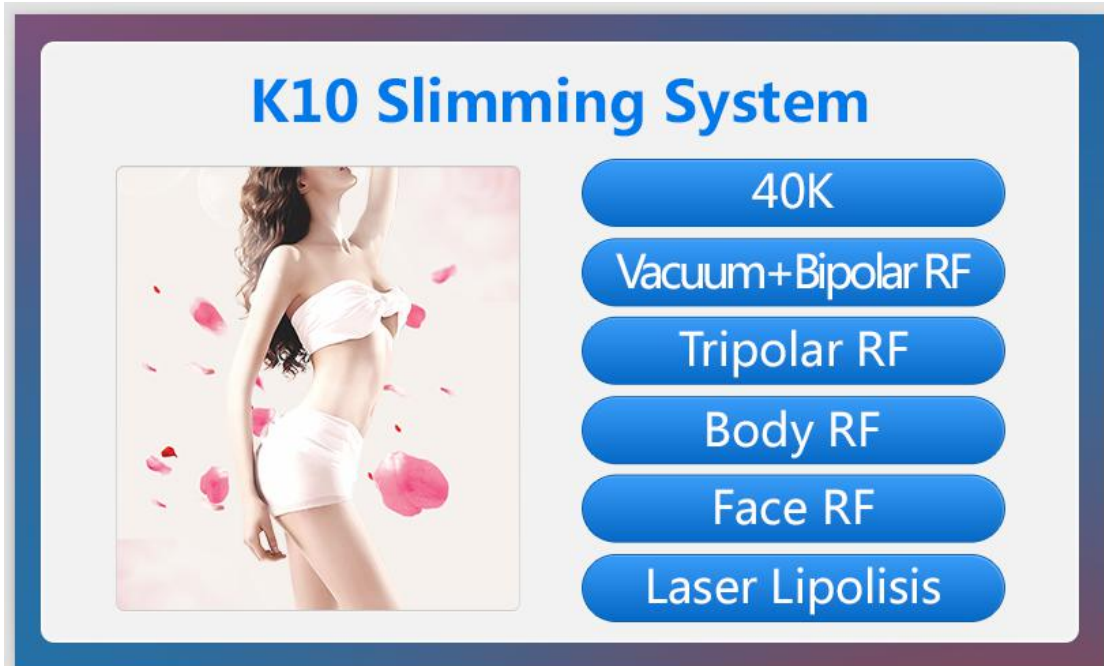





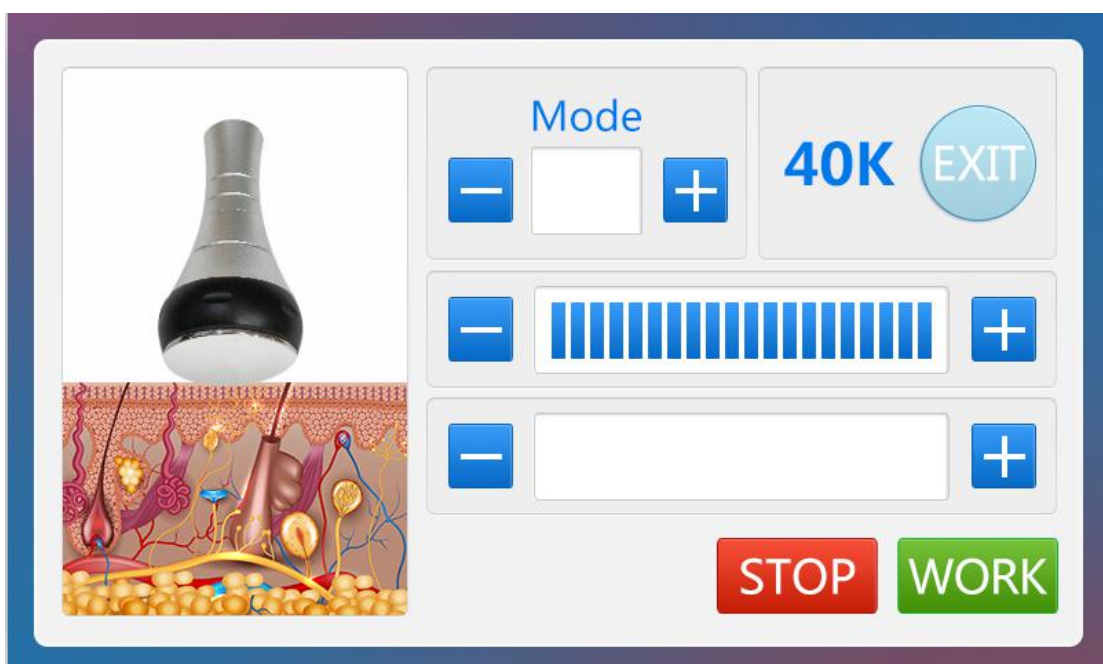

2. Click any place of the screen to enter into the main interface.



A. 40K Cavitation


1. Press  button to enter into 40k cavitation function display as bellow:



2. Please click   to set working time.
3. Please click    to increase/decrease ENERGY
4. Please click   to choose working mode
5. Please click  to start treatment
6. Please click  to pause treatment
7. please click  to return to main page




B. Body vacuum bipolar

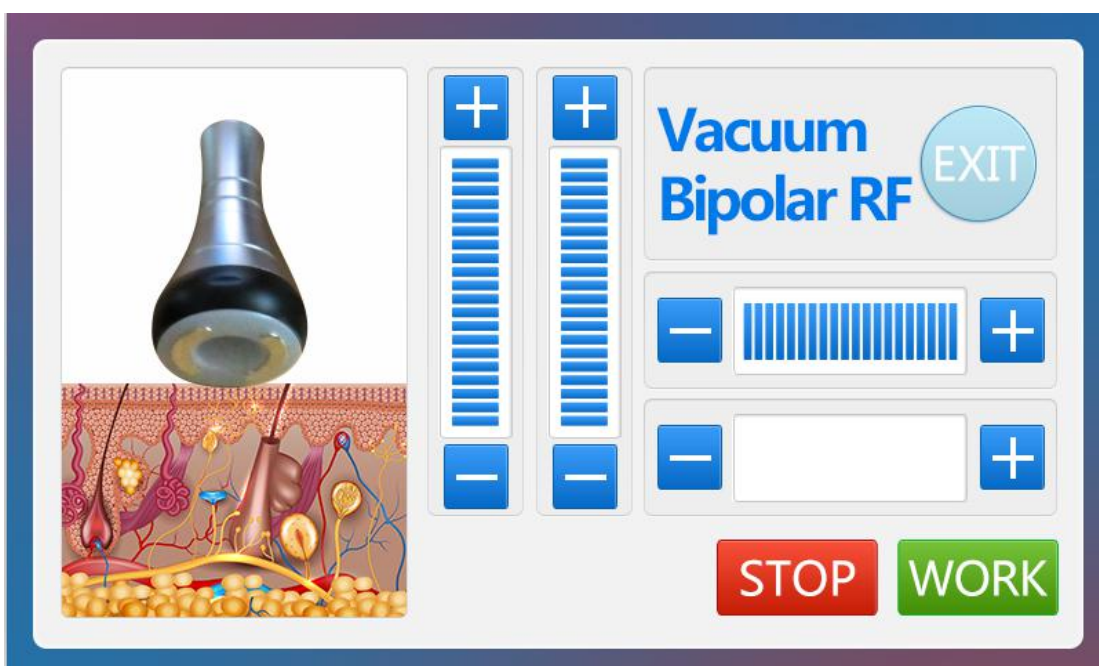


1. Press  button to enter into Vacuum bipolar function display as bellow:



2. Please click   to set working time.

3. Please click    to increase/decrease ENERGY

4. Please click    to increase/ decrease VACUUM RELEASE TIME


5. Please click    to increase/ decrease VACUUM SUCTION TIME


6. Please click  to start treatment


7. Please click  to pause treatment

8. please click  to return to main page

C. Body Radio Frequency

1. Press  button to enter into Body RF function display as bellow:

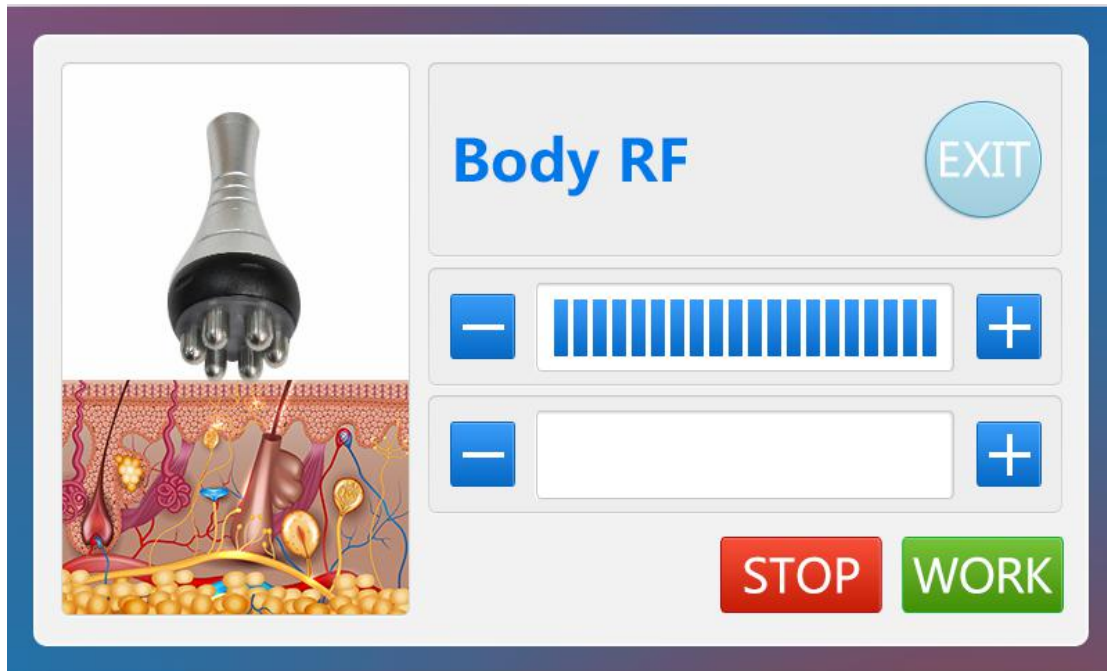


2. Please click   to set working time.


3. Please click    to increase/decrease ENERGY


4. Please click  to start treatment



5. Please click  to pause treatment

6. please click  to return to main page

D. Face RF

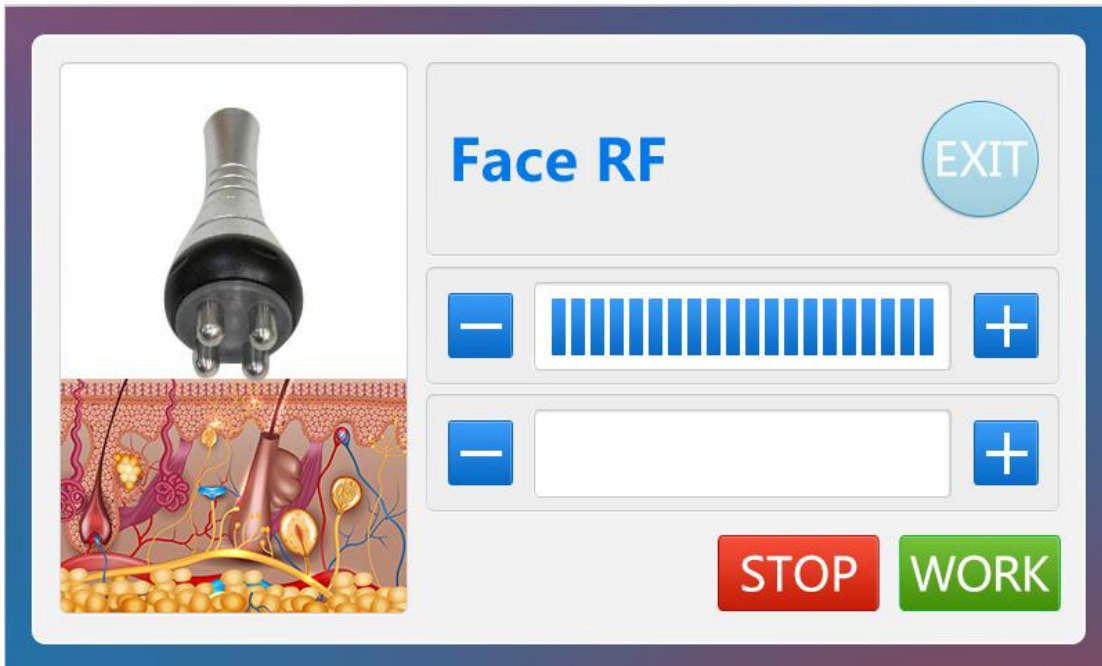1. Press  button to enter into Body RF function display as bellow:



2. Please click   to set working time.


3. Please click    to increase/decrease ENERGY


4. Please click  to start treatment



5. Please click  to pause treatment

6. please click  to return to main page

E. Tripolar RF

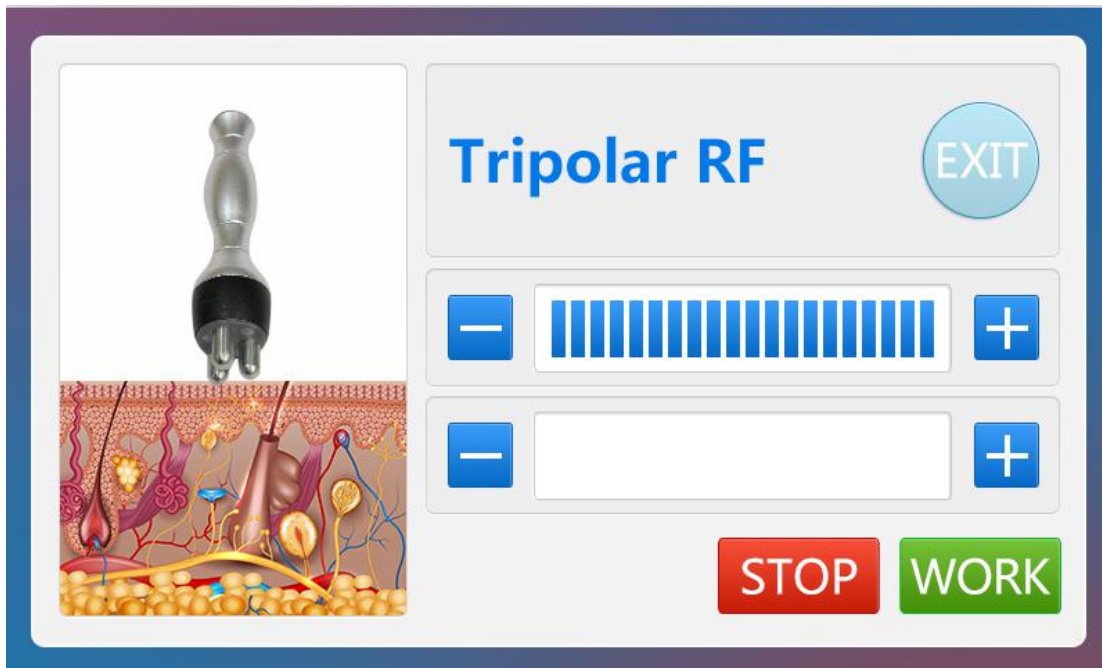1. Press  button to enter into Body RF function display as bellow:



2. Please click   to set working time.


3. Please click    to increase/decrease ENERGY


4. Please click  to start treatment

5. Please click  to pause treatment

6. please click  to return to main page

F. Lipo laser

1. Press  button to enter into Body RF function display as bellow:

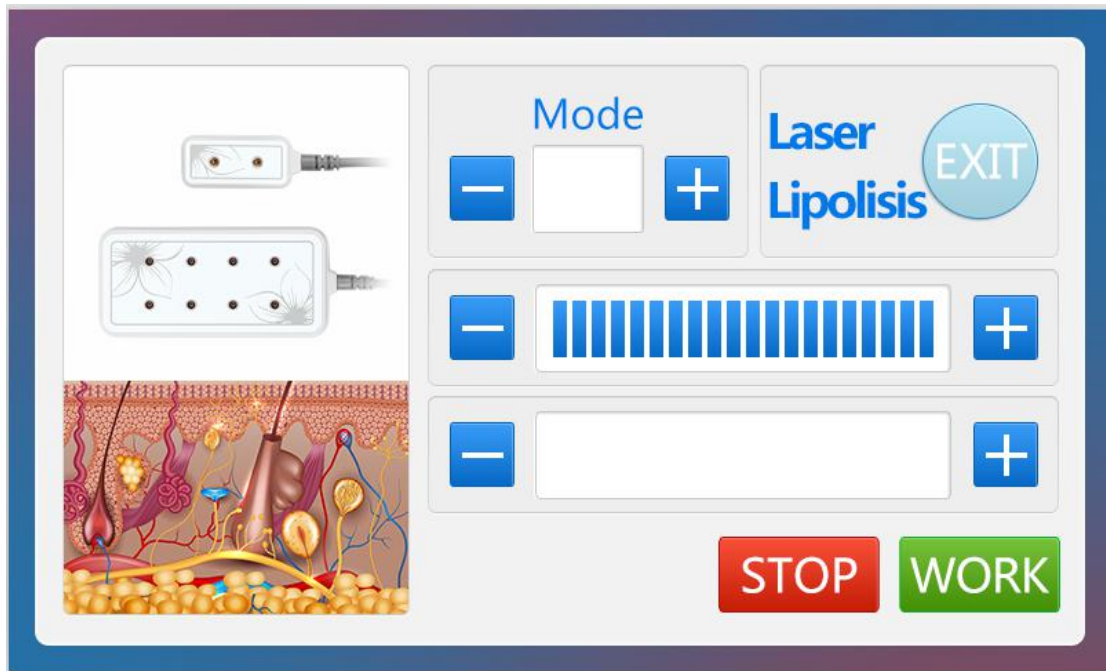

2. Please click   to set working time.

3. Please click    to increase/decrease ENERGY

4. Please click    to change the working mode

5. Please click  to start treatment

6. Please click  to pause treatment

7. please click  to return to main page

About Treatment

1) Abdomen, thigh, buttock, arms slimming

Please clean the skin and then put the ultrasound gel on the treatment area, then turn on the machine, set the time and energy. Please set the energy from low to high to get the best comfortable temperature, then follow this steps:

1. Use the Body RF head 5 mins
2. Use the unoisetion 40k cavitation head 20-25min
3. Use the Body RF Vacuum head 10-15min
4. Use the Body RF head 15~20 min.
5. Use the lipo laser for 15~20 min.

Work time: 2-3 times/ week

2) Face, eyes around, neck skin lifting:


Please clean the skin and then put the rf gel on the treatment area, then turn on the machine , set the time and energy, pls set the energy from low to high to get the best comfortable temperature, then follow this steps:

1. Use face radio frequency head for face for 15-20 min
2. Use eye radio frequency head for eye around for 10-15min

Work time: 2-3 times/ week

FACE

Treated time: 30 minutes

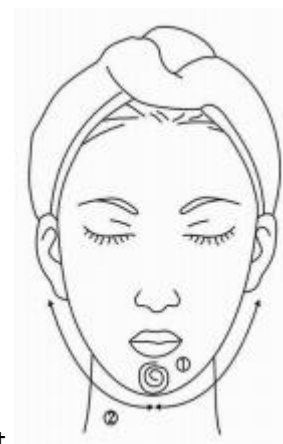

Massage the mandible center in circles to produce deep heat.

2. Massage lower jaw in lines.

3. Massage the triangle zone of both sides of the face in



circles.

4. Massage from jaw to angulus oris to ear in lines.


THIGH

Treated time: 30 minutes



1. From down to upper, pushing to the groin to dredge the lymph.

2. From down to upper, circling push by anticlockwise can help decomposing

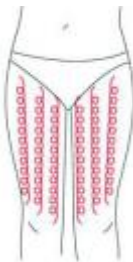

fatness.



3. Also can push by come-and-go to decompose fatness.

4. Pull from the knee and muscle texture to upper, can improve the curve.

Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.



3. Along the muscle of arm, pull up to lift and tight the muscle.

BACK

Treated time: 20 to 30 minutes




1. First by come-and-go, pushing the bladder nerve 2 to 3 times.




2. Circling by anticlockwise to stimulate the underarm lymph node.

3. On the back, by anticlockwise circling, can help to decompose the

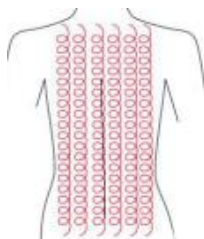

fatness.

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help

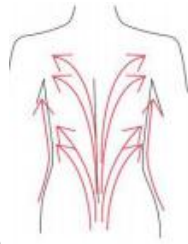

improving the back curve.

ARM

Treated time: 30 minutes

BACK SIDE

1. From the inner elbow to the armpit, doing lymph drainage.
2. Circling to stimulate the lymph node.
3. By anticlockwise or come-and-go pushing, decomposing the fatness.
4. From the elbow to the oexter, doing tightening gesture.




Facade

1. From the elbow to the oexter, doing tightening gesture.


Abdomen

Treated time: 20 to 30 minutes

1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.

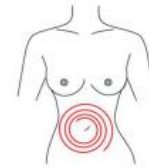


2. From small to large, by anticlockwise direction, circling push down.

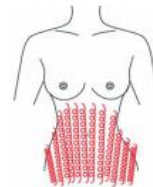

3. From the belly and muscle texture, pull to the groin.



4. Through lymph drainage gesture, taking the toxin to the place of groin.

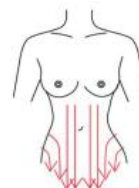

Attention:

1. Pregnant women or women during in menses.

2. Epileptic.
3. Patients with malignancy.
4. Patient whose wound after operation has not healed up.
5. Acute inflammation or epidemical patients.
6. Whom with heart diseases or with heart pacemaker.
7. Who with kidney (gall-stone) disease.
8. Who was embedded metal object or silica gel.
9. Who in menses, birth control period, emiction incontinence period, or accepting the belly operation.
10. Whose body always takes much inner hot.
11. Who has the genetic hypersensitivity.

Notice:

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
7. If the machine will not use for a long time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.