



Many thanks for purchasing our company' s product,please kindly read the manual instructions carefully before any operation.And please operate the system in the right way.


✧ **Treatment Principle:**

Nowadays, this is the most popular multi-function weight-loss and health-care beauty equipment, which combines the functions of Far-infrared ray heated therapy and Air-pressure vacuum massage therapy. The air-pressure therapy is also called pressotherapy that aims to massage full body so that to promote blood circulation, eliminate body muscle tiredness, tightening skin, improve body metabolism... Etc; The Far-infrared heated therapy further helps to speed up the body discharges sweat so that to get rid of body toxins, quickly burns redundant fatness/cellulite so that to achieve the effects of losing weight, slimming body and keeping health.

✧ **Treatment Applications and Effects:**

1. Burning fats, weight loss and body contouring, keeping fit.
2. Perspiration detoxification, discharge of heavy metals and various of toxins along with sweat, promote the metabolism.
3. Relieve rheumatism and cold, unclog the meridian, reducing body fatigue and relieving stress
4. Activate, purificate and optimize the blood cells and body tissues, improve body immunity
5. Stimulate body circulation system, promote blood circulation, strengthen and cultivate the Viscera.
6. Enhancing medication/nutrition lead-in and absorption.


✧ Panel Introduction:

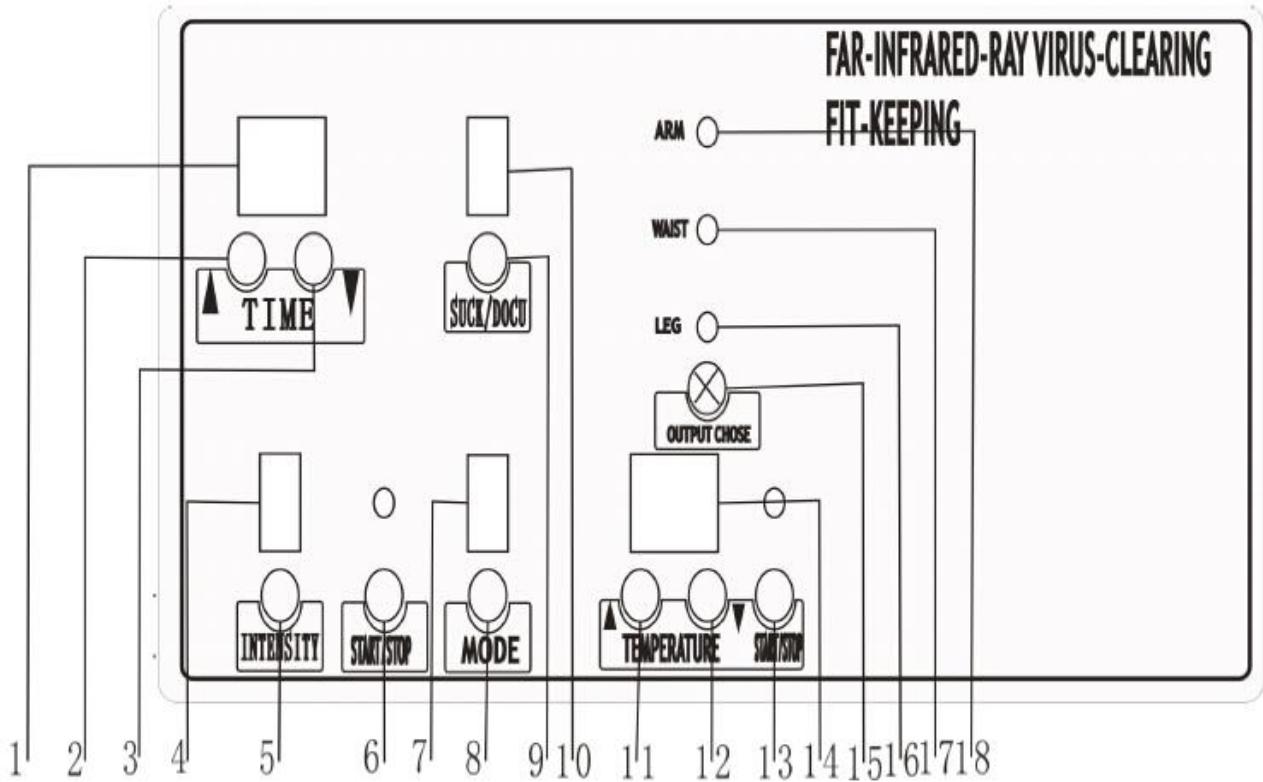

1.TREATMENT TIMER	2,3.TIME UP/DOWN
4.AirPressure INTENSITY	5.Airpressure INTENSITY button
6.Airpressure START/STOP	7.Airpressure MODE
8.Airpressure MODE button	9.Air Inflation/Deflation button
10.Air Inflation/Deflation TIMER	11,12.Temperature UP/DOWN
13.Heating START/STOP	14.Temperature Display
15.Infrared MODE button	16.Infrared LEG mode
17.Infrared WAIST mode	18.Infrared ARM mode



✧ Airtube Connection Figure:

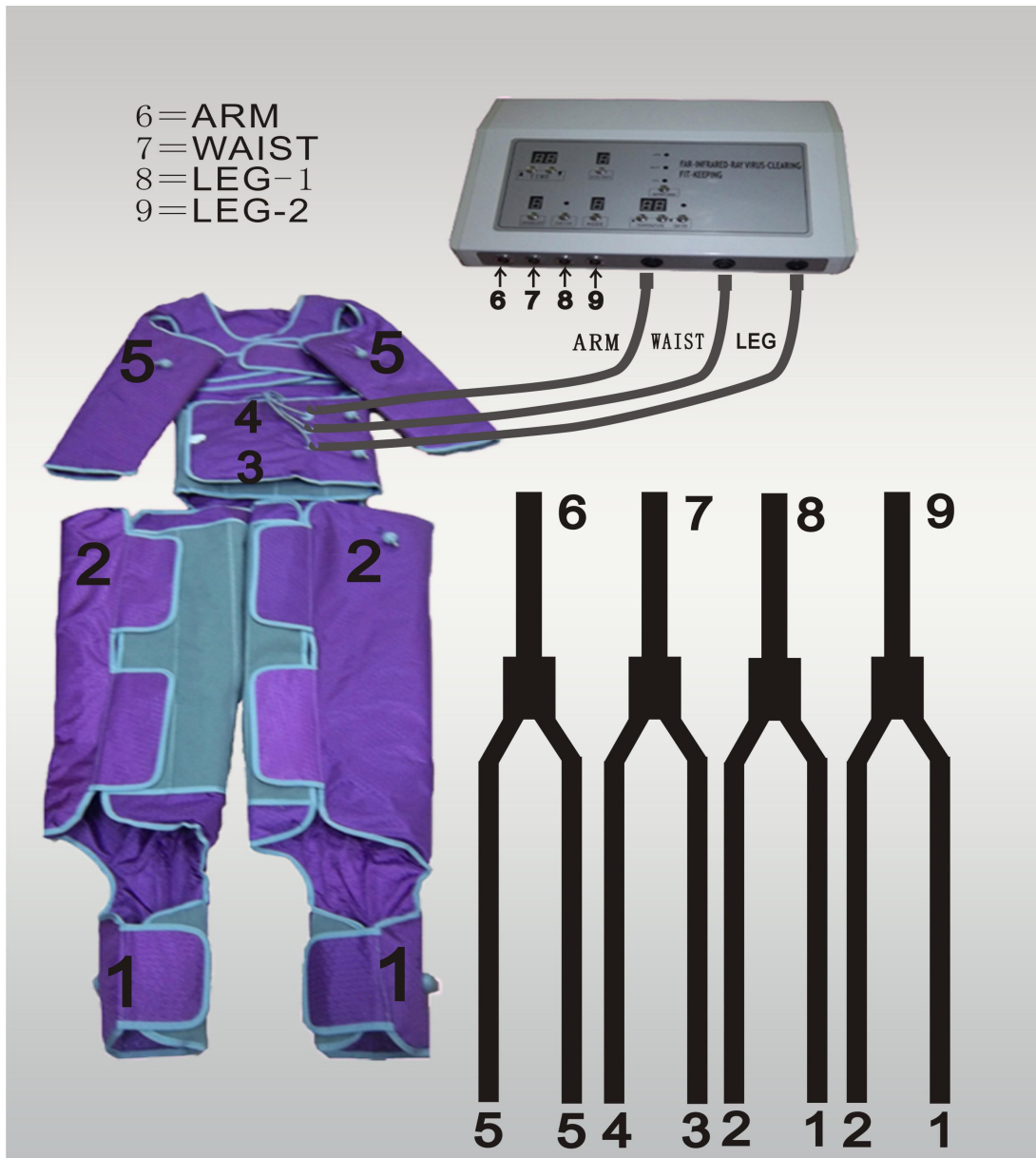


✧ Operation Instructions:

✓ AirPressure Operation Steps:


1. Correctly connect the air tubes to the according outlets **ARM WAIST LEG-1 LEG-2**.
2. Connect the power wire, press power switch to start the machine.
3. Set **TREATMENT TIMER** to the desired time using the ▲ and ▼
4. Set desired **Airpressure MODE** using **Airpressure MODE button**.

"1" : ARMS massage mode

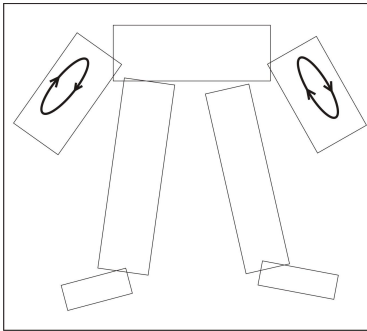


You can feel arms have airwave massage, which tightens the arms muscle and make arms slender.

"2" : LEGS(FEET) massage mode

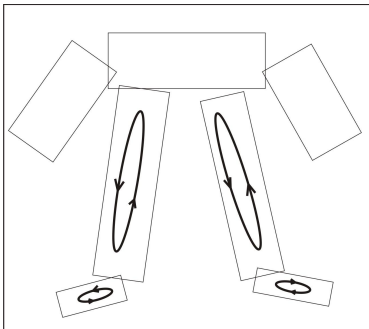


You can feel legs(feet) have airwave massage, which tightens double legs muscle and make legs slim.

"3" : WAIST massage mode

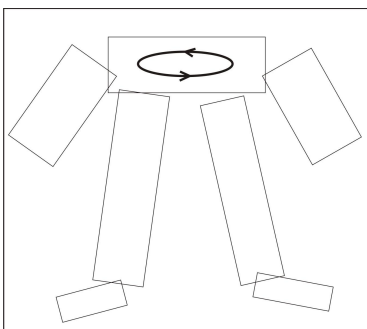

You can feel waist have airwave massage, which tightens waist muscle and make waist slim.

"4" : Circulation mode from ARMS-LEGS-WAIST

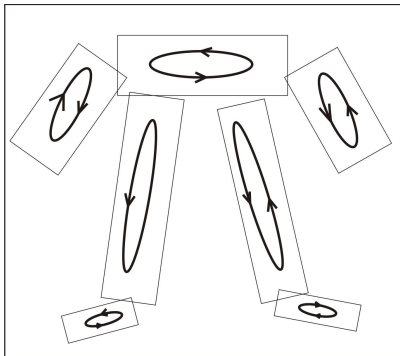

The circulation airwave massage from Arms to Legs, then to Waist, which makes whole body feel the airwave massage and make body relax, tighten body muscle and loss weight.

5. Set the **Air Inflation/Deflation TIMER** by pressing **Air Inflation/Deflation button**.

To set 1 second, meaning both air inflation time and deflation time are 1 second

6. Set the desired **Airpressure INTENSITY** (1~7 levels)

7. Press **START** to begin the airpressure treatment

8. Press **STOP** to pause the airpressure treatment

✓ Infrared Operation Steps:

1. Connect the Infrared Heating wires to the according outlets **ARMS WAIST LEG**

REMARKS:

- Have to connect each Infrared Heating wire to the according outlet so that can set the temperature. If the digital tube shows **"EO"**, meaning the wrong connection of wire and its temperature can not be set at the same.
- The temperature (**35°C to 80°C**) of each part (ARM, WAIST, LEG) can be set freely, and the temperature of each part can be different.
- Each part can work independently or can be set to work together as your treatment needs.

2.Press **OUTPUT CHOSE**,the **ARM** indicator lamp is on.

Set the desired temperature for ARMS.

Press **START** to begin the ARMS Infrared treatment . (Press **STOP** to pauze the ARMS)

3.Press **OUTPUT CHOSE** ,the **WAIST** indicator lamp is on.

Set the desired temperature for WAIST

Press **START** to begin the WAIST Infrared treatment (Press **STOP** to pauze the WAIST)

4. Press **OUTPUT CHOSE**,the **LEG** indicator lamp is on

Set the desired temperature for LEGS

Press **START** to begin the LEGS Infrared treatment (Press **STOP** to pauze the LEGS)

5.After doing the above operations,the Infrared Heating will work on different part of body.

6.Turn off the power when stop using the machine,pull off the attachment and store the machine and suit well.

✧ Technical Parameters:

Voltag Supply	AC220V-240V/50HZ
Power Supply	500W
Maximum Treatment Time	90 minutes
Infrared Temperature	35°C to 80°C,optional
Airpressure Intensity	1 to 7 levels,optional
Airpressure Inflation/Deflation Time	1~6 seconds,optional
Machine Dimension	40cm(L)X27cm(W)x23cm(H)
Net.Weight	13.5KG
Gross.Weight	15KG

✧ Caution Items:

- 1.To make sure the safety, the operator has to be near the machine when working, never leave away when it is working;
- 2.The suggested time of using this machine should not be over 30 minutes;
- 3.Never pull out the power plug or other connected pipes as the machine is power on;
- 4.Before doing the therapy,the customers must take down the sharp objects,which would probably puncture the therapy suit and cause serious consequences;
- 5.Stop using the unit if the suit cracks to avoid the danger of electric shock;
- 6.People wearing metal decorations can do the therapy with this unit,but the decorations can not be sharp.

✧ Packing List:

Item Name	Quantity	Remarks
Machine Controller	1(set)	
Infrared Pressotherapy Suit	1(piece)	
Airtube	1(set)	
Power Wire	1(piece)	
Spare Fuse	2(pieces)	
Spare White Wing	5(pieces)	
English Manual Instructions	1(copy)	