

VS-8008 5 In 1 Slimming Machine

Manual

Functions Include:

- 1- laser lipolysis
- 2- Multi-polar RF (Tripolar, Five Polar, Seven Polar)
- 3-Vacuum
- 4-40K Cavitation
- 5-Ultrasonic



Warning

Thanks for using our company's products, in order to fully make use of the products, we sincerely suggests follow:

- ◆◆Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆Please follow the instruction to fit and operate the machine.
- ◆◆Please do not remove and change any accessories of the machine.
- ◆◆It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆It forbids others to change and maintain the machine except the professional people.
- ◆◆Please contact us if there is any trouble during operation.

Working principal

SKETCH of the CONTROL PANEL



Click on the picture of the corresponding the handle enter the operation interface

Instrument Function and Treatment Principles

1. Vacuum Bipolar RF handle

Dissolve fat, lymphatic drainage, firm skin and enhance skin elasticity

Body-shaping Equipment adopts the most advanced high-speed turbine negative pressure technology, and innovative three-dimensional conversion technology, combining with microcomputer chip controlled patent roller axis, and has an anti-geocentric attract effect on the skin.



Attracting and outspreading the fibrillar connective tissue of different cortexes, through the rhythmic massage and effectively break down subcutaneous fat and reduce cellular tissue accumulation of fat, use of the patents to massage deeply roller different parts of the body, bring up fat layer in dermis, curium and subcutaneous, and release micro-vascular lymphatic vessels at the same time, promoting metabolism and to make fat cells in a passive movement to fat acids, even within the cells of toxins, also to be eliminated from lymphatic system in vitro, the detoxification enhance the fuction of as many as four times. Can be done manually or any other instruments can not achieve the perfect results. At the same time, skin is more elastic and luster. It is the "healthy body sculpture management" expert

Bipolar RF handle mainly used for the body, limbs, buttocks and other parts of concentrated fat, chieve slimming firming effect. Turned negative and the function of detoxification massage.



Recommended working time settings:

Arm	Shoulder	Back	Chest	Waist	Abdomen	Buttock	Thigh	Leg
10Min	10Min	10Min	10Min/side	20Min	20Min	10Min/side	15Min/side	10Min/side

2. Sextupole RF handle



Being integrated with the most advanced RF technology and radio frequency, the instrument may directly reach the deep-seated fat body and have the excellence of targeted positioning RF. In the fast-active state, fat cell tissue may generate friction heat, increase local temperature and remove excess fat and toxin from the body through sweat gland, enterohepatic circulation and lymph and ultimately achieve the effect of dissolving fat. With controlled depth technology, inlaid diamond particles of different sizes may rub skin along its pattern, adjust suction strength by means of vacuum draw and rub with negative pressure strong force to directly explode thick fat. Thus, the effects are very obvious: Enhance blood circulation, accelerate the decomposition of fat, take away wastes, enhance the effect of removing cellulite, effectively improve the state of areolar tissue, remove stubborn fat and improve the sufficient absorption condition of diet products.

Then RF waves at frequency of 1M HZ is used for exhausting the integrated glycerol and free fatty acids through hepatoenteral circulation. Finally, vacuum RF and energy electrode are used for positioning and tightening fat.



Recommended working time settings:

Facial ascension anti-wrinkle	Improve anti-wrinkle eye	The neck anti-wrinkle regeneration
20Min	10Min	10Min

3. 40Khz Ultrasonic handle

40K Cavitation – 40K wave fat system

With collective strong sound wave head, strong sound wave of 40KHZ may be emitted to vibrate fat cells at top speed and produce numerous vacuum air pockets inside and outside the fat cells, robustly impact fat cells to generate introverted blast and disintegrate triglyceride into glycerol and free fatty acids. In physics, it is known as "cavitation". Micropore introverted blast inside and outside cell may lead to enhanced molecular motion and a higher energy level and this will finally cause fat cell rupture and thereby achieve ly building and losing weight.



With collective strong sound wave head, strong sound wave of 40KHZ may be emitted to human body for impacting fat cells fiercely and causing friction motion between fat cells. This may cause effective consumption of calories and moisture in fat cells and reduce the size of fat cells. What's more, sound wave vibration Vmay cause fierce impact of fat cells to make them be exploded instantaneously, reduce the amount of fat cells and thereby achieve the effects of removing fat.



Recommended working time settings:

Arm	Waist	Abdomen	Buttock	Thigh	Leg
4-8Min	4-8Min	5-10Min	10-15Min	10-15Min	4-8Min

4. Quadrupole RF handle --Facial wrinkle elimination, tightening and lifting



A capacitive coupling electrode is used for transmitting radio-wave energy and generating an electric field for entering into subcutaneous tissue through the skin surface. In the high-frequency of 450khz, this electric field changes its polarity for 450,000 times per

second. To reply to the rapid change in electrode, directions of electrical particles the skin are also changed. By this time, natural electric resistance in the subcutaneous tissue moves and generates heat energy. As the papillary dermis collagen may immediately shrink when the temperature is within the range from 60 to 70 Celsius degree, after wrinkle treatment, client may immediately sense the skin tightening effects as it is being lifted and firmed. When collagen is produced continuously, thickness and density of the skin papillary dermis may be increased to remove wrinkles, eliminate scars, restore skin elasticity and gloss and make it be blonde and smooth. While collagen is increased, fresh skin is generated in the position of operation and wrinkles are removed by large amount of cells. In addition, when cortex without elasticity or that with thick horny layer in the area with wrinkles is separated, surrounding skin will also be renewed.



5. Tripolar facial handle



Stick to enhance rejuvenation: use of positive and negative micro-current activation energy release skin cells, so that rapid synthesis of fibroblasts Collagen, elastic skin revert to the original compact, the skin causing t the skin to stimulate the muscle tension is increased, thereby gradually eliminate wrinkles, prevent skin aging, restoring skin elasticity, facial improvement.



Caution for using RF

1. Beautician should wear insulated gloves during operation.
2. Do not wear any aglet, both for the beautician and the users.
3. The electrode probe must be fully in contact with skin.
4. Do not make the RF electrode stay on any part without moving. Keep it moving on skin slowly.
5. Skin around eyes and forehead is very thin, the RF output intensity should be relatively weakened.
6. Do not contact skin with unipolar when using of bipolar, tripolar,

hexpolar RF work head otherwise it will produce pulse discharge, gel paste etc. Products like gel maybe evaporated into a mist.

Recommended working time settings:

Facial ascension anti-wrinkle	Improve anti-wrinkle eye	The neck anti-wrinkle regeneration
20Min	10Min	10Min



Ultrasonic utilizes sonic vibrations; it works one million times per second (1 MHz). The ultrasonic wave could stimulate skin tissue and improve permeability of the membrane to promoted collagen tissue, reduced wrinkle and invisible appeared fine line, also lift the skin result for younger and healthy look. Ultrasonic generate frictional heat which accelerate blood circulation, muscle relaxation, and decompose the cellulite to break down exceed fat.

Ultrasound with 2 probes provided non-surgical lifting. This machine utilizes sonic vibrations to reduce lines, leaving the skin firm and smooth. Feature Intensity control, 2 sonic variations, and 2 sonic probes.

I. Preparation

- (1). Before operating this machine, cleanse the area to be worked. After cleansed, apply suitable lotion and gel to the skin.
- (2). Press Power On/Off [7] to turn the machine.
- (3). Set the working time by Time adjust button [16], working Time display on Time window [6].
- (4). Output intensity display on Output intensity window [5], Press Power up [15] and Power down [17] to set output intensity, output intensity classify into 8 levels. For Facial work, low to mid intensity is recommended. For Body work, mid to maximum intensity is recommended.

II. Face Treatment

- (1). Connect Face Electrode [13] into Face/Eye Electrode output [1]
- (2). Press Face/Eye mode [20] to select Face mode, Face function [9] will light on.
- (3). Press Pulse/Continuing mode [20] to select Pulse or Continuing wave, Pulse/Continuing wave indicator [11] will light on.
- (4). Let the client hold the handle, the operator hold Face Electrode and working on the face.

III. Eye Treatment

- (1). Connect Eye Electrode [14] into Face/Eye Electrode

- output [1].
- (2). Press Face/Eye mode [20] to select Eye function, Eye function [10] will light on.
 - (3). Press Pulse/Continuing mode [20] to select Pulse or Continuing wave, Pulse/Continuing wave indicator [11] will light on.
 - (4). Let the client hold the handle, the operator hold Eye Electrode and working around the Eye area.

IV. Body Treatment

- (1). Connect Body Electrode [12] into Body Electrode output [3].
- (2). Press Body mode [18] to select Body function, Body function [8] will light on.
- (3). Press Pulse/Continuing mode [20] to select Pulse or Continuing wace, Pulse/Continuing wave indicator [11] will light on.
- (4). Let the client hold the handle, the operator hold Body Electrode and working on th body.

5;Laser

Lipolisis;

Diode Laser Safety Guidelines

The Following information is provided for the correct utilization of IPL Skin treatment system.the Information includes not only the accident protection regulations the products comply with,but also the effective regarding proper use of the products.

1)Electric safety regulation

The EMC Directive 89/336/EEC amended by 92/31/EC &93/68/EECEN55015:2000+A1+A2/LUM200704025/EMCEN61000-3-2:2000+A2/LUM200704025/EMCEN61000-3-3:1995+A1+A2/LUM200704025/EMCEN61547:1997+A1/LUM200704025/EMC

2)Material ROHS test2002/95/EC

Restriction of use of certain Hazardous Substance in Electrical&electronics Equipment/ROHS.

3)Electromagnetic radiation safety regulation LVD/ EN60601-1:2006/AGC10430904GZ05E3EMC/EN60601-1-2:2001+A1:2006/AGC10430904GZ05E1

4)ISO13485:2003 MEDICAL/33021200005459

5)ITC MEDICAL CE

IPL SKIN TREATMENT SYSTEM belongs to II a according to MDD directive,It designed according to EN 980:2003; EN 1041:1998; EN ISO 14971:2007; EN 60601-1-2:2007; EN 61000-3-2:2006;EN 61000-3-3:1995+A1:2001+A2:2006; EN 60601-2-22:1996;

EN 60601-1-4-1996+A1-1999; EN 60601-1:2006.

1.LIPOLASER common sense

Is LipoLaser Safe?

Yes, LipoLaser treatments are completely non-invasive and clients feel nothing. LipoLaser's low level laser technology has been studied and used for over 30 years for many biomedical purposes. FDA approved for pain and inflammation, doctors can use it for fat loss .

Do I have to do other things to lose the fat?

Only for bigger results - 1 inches without changes, 5inches with major lifestyle changes, like diet and exercise - we have support if you choose to change your life while doing the sessions

What do you do to keep the fat off?

Normal things you need to keep weight off - diet and exercise, drinking water, etc.

Where Does the Fat Go?

That fat can be burned as energy in the body, or normal metabolic processing —just like when you exercise and release fat

How Quickly Will I See Results

Many clients see measureable results after their first session.

What Kind of Results are Produced?

1-5 inches average-the most lost in 9 sessions is 23 inches
People who make minor lifestyle changes, are on the higher end of the average, and those who don't, the lower end.

How many times do I need it ?

3times per week,9times per course of treatment.



3.1. Turn the key on the key switch and urgency Button to power on the machine, and the led on the screen will display.

3.1.Adjust the timer for big probe,then press On to make 6 piece big probe working.

3.2.Press the on/off button to control the small probe working.

4.1 Cleanse the skin, Put the Lipolaser probe to body where you want lost fat.then use a belt keep the pad to body.



4.2 You relax in a comfortable position while a trained LipoLaser technician places two multi-laser paddles and two smaller lasers to the target areas. Then lay back, relax, read, or whatever you like.

5.Before and after compare




- 6.1. Whole ABS Plastic Mould.
- 6.2. It does not present any contagious risk
- 6.3. stable laser energy output
- 6.4. Good energy and effect
- 6.5. Super treatment effect

We Are Looking for Sale agent over the world.

Using Guidance


1. Operation schematic drawing of 40K cavitation explode work head
Massage the fat of thigh in circles

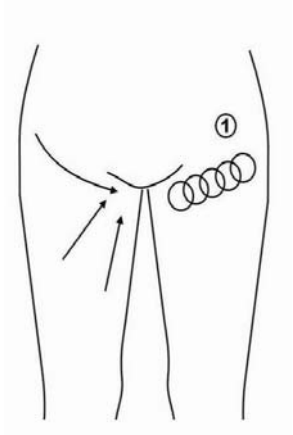


2. Operation schematic drawing of sextupole RF laser work head for body

Waist and abdomen care (25minutes, 2-3 days/time)

- ① Massage to navel as center and gradually expanding the range

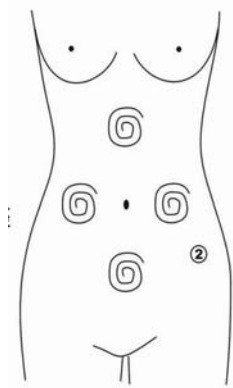


② **Massage chest nest, both sides of former abdomen and below the navel(into diamond) from inside to outside in circles.**

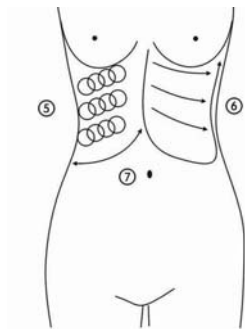


- ② Concentrated the fat from 4 directions to navel as center.
- ④ Massage from end of pelvis to groin in circles.
- ⑤ Massage from inner rib to outer edge in circles.
- ⑥ Massage back and forth from inner rib to outer edge in straight line
- ⑦ LiftiHip and he lower part of the body care (30minutes, 2-3days/time) used the ΦD50mm probe

Concentrated the fat to groin in circles or straight line.ng from inner rib to outer edge after massage.

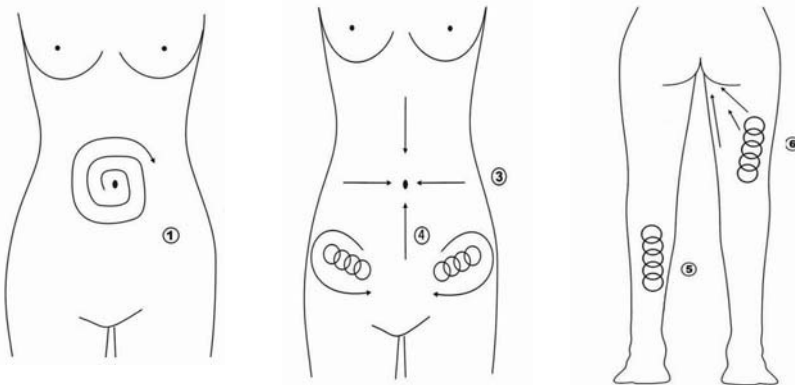

3. Operation schematic drawing of Tripolar RF work head for face

- ① Massage the mandible center in circles to produce deep heat.
- ③ Massage lower jaw in lines.
- ③ Massage the triangle zone of both sides of the face in circles.
- ④ Massage from jaw to angulus oris to ear in lines.
- ⑤ Massage from jaw to angulus oris to ear in lines.

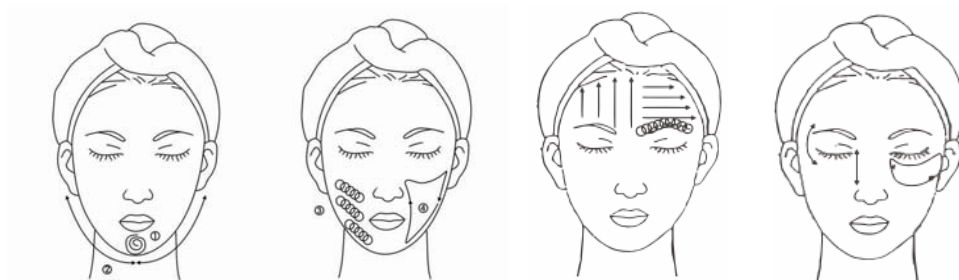



1. Pregnant women or women during in menses.
2. Epileptic
3. Patients with malignancy.
4. Patient whose wound after operation has not healed up.
5. Acute inflammation or epidemical patients.
6. Whom with heart diseases or with heart pacemaker..
7. Whom with kidney (gall-stone) disease.
8. Who was embedded metal object or silica gel.
9. Who in menses, birth control period, emiction incontinence period, or accepting the belly operation.
10. Whose body always take much inner hot.
11. Who has the genetic hypersensitivity.

NOTICE

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasoinc gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
7. If the machine will not use for a loing time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.

Safety Warning

Important Notice of Caution before using this machine:

Never operate this equipment on a person has or suffers from the following medical conditions described below. (The salon or operator may consider having a client sign a waiver to the effect that he/she is free of any poor health conditions.)

1. Heart Disease.
2. Heart Murmur or Irregular Heart Rate.
3. Has a Pacemaker or other type of heart rate balancer.
4. Pregnancy.
5. Epilepsy.
6. Alcoholism.
7. Ulcers.
8. Disease of Stomach, Intestine, Liver, or Pancreas.
9. Breast Cancer.
10. Undergoing treatment for any form of Cancer.
11. Had major surgery within the past year.
12. Asthma.
13. Cold, Flu, or Upset Stomach.
14. Any serious illness or disease not listed above.

Do not use on these areas:

1. Sensitive Skin.
 2. Skin Rash.
 3. Damaged, Inflamed, or Infected Skin.
 4. Surgical scar less than 12 months old.
 5. Skin Swollen.
 6. The Sound head can not never tower to Eyes ball.
 7. Genitals.
1. Clean skin with clear water.
 2. NEVER STOP THE ELECTRODE ON THE SKIN IN ONE SPOT.
 3. ALWAYS KEEP THE ELECTRODE MOVING ACROSS THE SKIN.
 4. NEVER POINT THE ELECTRODE TOWARDS CHEST AREA.
 5. Do not immerse the handle or the machine into water. Doing so could cause serious injury or death.
 6. Store in a dry cool place.

Packing Contents

1. Face Electrode
2. Eye Electrode
3. Body Electrode
4. Power Cord
5. Manual