

Your satisfaction, my pursuit

User Manual

YH-17401

MYCHWAY®



Tel : 86-189 4875 5929

Fax : 86-755-8636 8550

Email : sales@mychway.com

Http : //www.mychway.com

Address : RM406,F4,#6 bldg,zhongxin Industry city,Chuangye Rd,Nanshan Distr,Shenzhen,China 518054

Chinese Shenzhen mychwayTechnology Co., Ltd.

<http://www.mychway.com>

Preface

Dear users:

Welcome to choose our latest product YH-17401 which is multi frequency radio frequency technology, negative pressure and low frequency ultrasonic cavitation application integration technology of the latest slimming products. YH-17401 is the instrument of professional beauty, need to be trained operators hand operation if not the proper use of this instrument has the potential to cause adverse effects on the human body, so we hope that any person in the operation of the YH-17401, the operator first read this manual carefully and strictly comply with the manual inside the guide to operate the equipment.

We believe that our products will bring good benefits for you, our perfect after sale service will make you worry free.

Thank you!

Catalog



Preface



The first part

Lipo Laser
Applicable to the crowd



The second part

RF Unoisetion Cavitation 2.0
Package Including
Installment and Use
Skills&Notice


Lipo Laser

I. Principle

Lipo Laser Machine involves the application of a highly sensitive laser to dissolve the fat deposits situated in the upper layers of the human skin. The energy emanating from the laser breaks up the fat cells to produce an oily material. This substance is either naturally eliminated through normally bodily processes or can be removed via a small incision made by the practicing surgeon. This leads to a permanent loss of fat as long as a healthy diet is maintained after treatment.

It penetrates into skin surface and stimulates the fat cell membranes, changing their permeability, the fat cells reduce their overall size and intracellular fat is released then the fatty triglycerides flow out of the disrupted cell membranes and into the interstitial space, where they gradually pass through the body's natural metabolic functions with no harmful physiological effects, this process is resulting in inch loss of patients. The subtlety of the treatment carries various advantages over the problems often associated with liposuction. Damaged blood vessels and allied physical risks are significantly reduced, and recovery time is condensed. It is also acknowledged that the laser itself is important in encouraging collagen production, thus strengthening the skin.

Lipolaser Theory

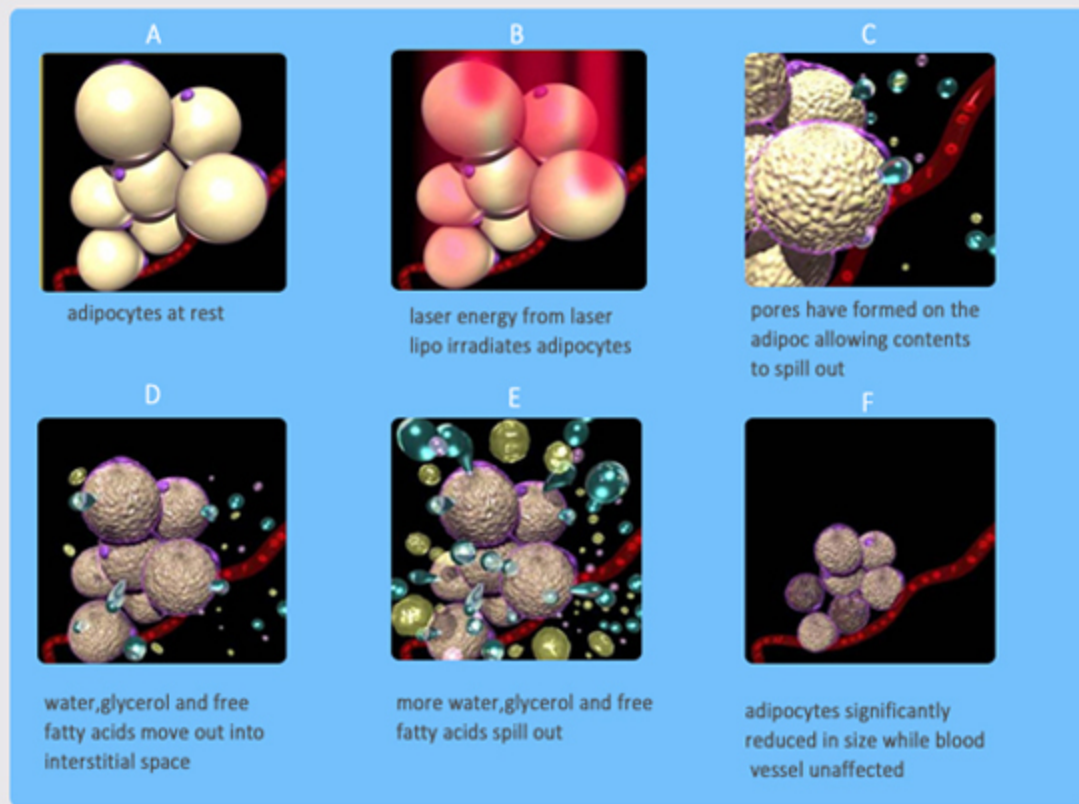

Applicable to the crowd

Application

Lipo laser is a non-invasive or minimally invasive procedure, depending on the amount of fat. And it is a method to get rid of the fat deposits that affect the aesthetic of the body contour and cause the cellulite "dimples" in the skin. And The laser method is for mild to moderate fat conditions and skin tightening. But it cannot remove larger amounts of dis-proportioned fat.

Treatment Range



RF Cavitation 2.0

RF cavitation is the most veteran of all non-invasive medical aesthetic body contouring treatments. There is a high demand for body contouring treatments. Such popularity is only to be expected due to the importance we attribute to the shapes of our bodies.

How RF Cavitation Lipo laser Works

By projecting RF (Radio Frequency) electromagnetic waves through the

Package Including

the skin and into underlying stubborn deposits of cellulite the molecules of fat are heated and broken down into substances which can then be absorbed into the blood stream and disposed of through normal metabolic function.

III. Treatment Safety

1. Treatment Time is less than 30mins/Body Part/Each Treatment.
2. The energy setting is from low to high gradually which should be subject to skin comfortableness for different people.
3. The patient had better wear the protective eye glasses.
4. If you cannot tolerant heat or uncomfortable, please stop the treatment in advance and turn to your doctor for help.

1 x 4OK +RF Cavitation Head
4 x Small Laser Pads
12 x Big Laser Pads
1 x Holder
1 x Power Supply Cord (Both 110v and 220v voltage (frequency: 50/60Hz) version are available. Also we stock the USA, UK, AU, EU plug to suit for the country all over the world.)

Installment and Use

I .Working Condition

Using at normal temperatures and pressure. Please do not stay in too cold, too hot, too dry and too wet.

II. Technical Parameters

Cavitation 2.0 Unoisetino
Ultrasound Frequency: 35 - 40 Khz
Ultrasound Power: 30 - 60 w/cm²

3D SMART RF
RF Frequency: 6 - 10 Mhz (Max.)
RF Power: 100 J/cm³
RF Output: Series / Pulse
Pulse Re-cycle Time: 0.3S

Lipo laser

Laser Lipo Wave Length: 635-650nm

Maximum Output Power: 150W

Single Lipo Output: 160 mw

128 Lipos Total Max. Output: 128*160mw=20480 mw

Numbers of Treatment Pad: 16 Pieces (12 Pieces large pads + 4 Pieces small pads)

Numbers of Lipos: Total 128 Lipos (120 LargeLipos + 8 Small Lipos)

Timer: 30 minutes adjustable




III Before Use

1. Connect the pads with the jacks,connect the RF cavitation head with the main machine,Please do make sure the connecting is in good condition
2. Plug the power supply
- 3.Please clean skin of treatment area and put ultrasound gel on the skin before do cavitation treatment
- 4 Please use the ultrasound gel on the treatment area before do RF treatment
- 5 Please drink about 500-700ml water before treatment,Please do treatment after 40-50 minutes after drink water
- 6 After treatment, please do drink water enough
- 7 Please keep fit and do exercise after treatment,if not,the result will keep a short time
- 8 Please do stop treatment if you feel something uncomfortable, it is vital important
- 9 Please do clean the parts and main machine after treatment finishing

IV. How to Put the Pads on Body?

1. Big pads for big body part such as belly, waist and etc. Small pads for small body part such as face.
2. Using the belts and adjust it in a proper place.
3. Insert the pads in and close them to the skin

How to Use

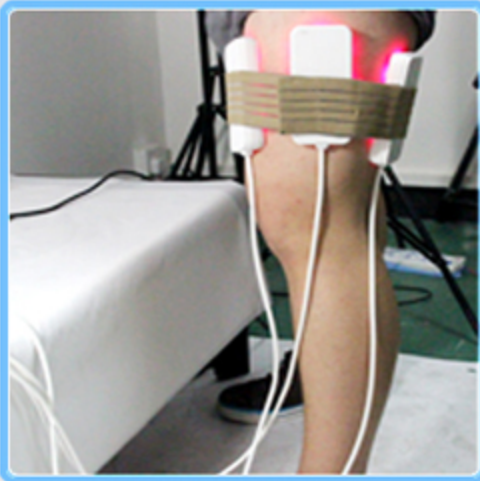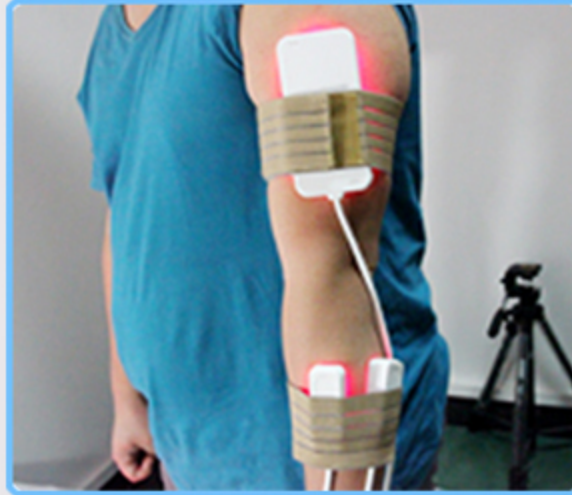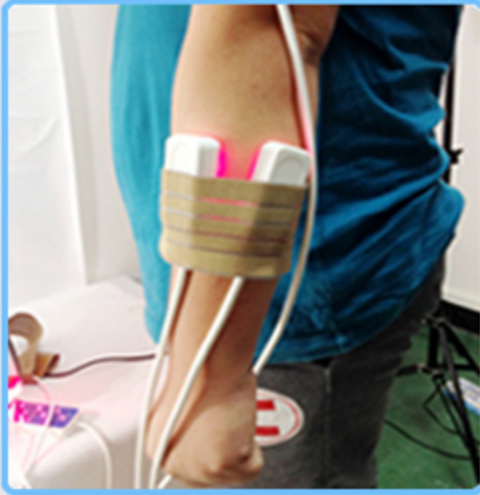

VI. How to Start the Machine?

Turn on the power supply button  on the back of the machine the button will be red if it is on

VII. Program Setting


1. Start the machine, the screen will show like below

Individually control your led light output power. However powerful you want, choose yours individually at every single paddle.



I can only choose to use the big paddles to do the treatment

Or I can choose to use all of the paddles but all are different intensity of the power.

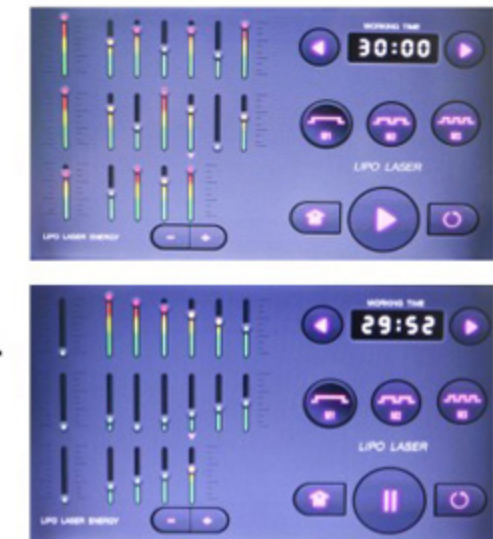




Or i can only choose the small paddles





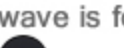





or i can choose several big paddles (and adjust different intensity)

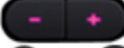

Cavitation RF

- 1 Please press RF+CAvitation button then login unoisetion RF+4OK cavitation function display
- 2 Please click  to set working time

- 3 Please click  to increase or decrease the ENERGY
- 4 Please click  or  to chose working model,  continuous wave is for normal fat ,  pulse wave is for very fat, Please chose the right mode according to fat condition
- 5 Please click  to start or pause function
- 6 Press  will exist working function display then enter menu screen
- 7 Press  means pause and all setting parameters will be zero

Skills

THIGH

Treated time: 30 minutes

1. From down to upper, pushing to the groin to dredge the lymph.



2. From down to upper, circling push by anticlockwise can help decomposing fatness.



3. Also can push by come-and-go to decompose fatness.



4. Pull from the knee and muscle texture to upper, can improve the curve.



Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.

3. Along the muscle of arm, pull up to lift and tight the muscle.



ARM

Treated time: 30 minutes

1. From the inner elbow to the armpit doing lymph drainage.



2. Circling to stimulate the lymph node.

3. By anticlockwise or come-and-go pushing, decomposing the fatness.




3. By anticlockwise or come-and-go pushing, decomposing the fatness.




Abdomen

Treated time: 20 to 30 minutes

1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.

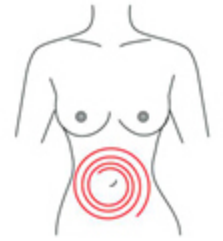


2. From small to large, by anticlockwise direction, circling push down.

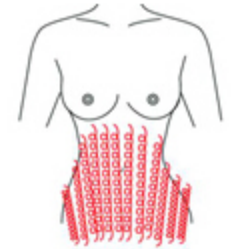

3. From the belly and muscle texture, pull to the groin.

4. Through lymph drainage gesture, taking the toxin to the place of groin.

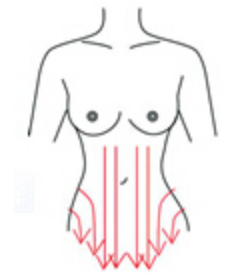

Notice

Notice:

1. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
2. Don't stay in one place, avoid treating on the bone.
3. Don't use disinfectant product to disinfect the head. We suggest the wet cotton or dry towel enough.
4. Check the machine power cord if it is connected well.
5. If the machine will not use for a long time, please switch off it, and power off.
6. Please take off all metal decoration from the operator and patient's body.
7. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
8. During the operation, be sure not to accept other treatment.