

---

**3 in1 Unoisetion Cavitation 40K 3D RF Radio  
Frequency Slim Weight Loss**

**Model: YH-3351**



---

## **Chapter 1: Safety Warnings**

### **I. Electricity & Machinery Safety**

1. The machine is either 110V or 220V. So please check the machine voltage before starting the machine.
2. Please keep the voltage at stable condition.

### **II. Contraindication Safety**

1. People with heart disease, high blood pressure, or configured cardiac Pacemaker.
2. People with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
3. People with hemorrhagic diseases, trauma or who is bleeding.
4. Pregnant women
5. Medical plastic artificial or metal parts inside the body such as metal teeth or silicone breast implants.
6. People with abnormal immune system.
7. People with numb or insensitive to heat.
8. If you are feeling unwell, Prohibited in the ears, nose, eye throat and do not use this device in advance and tell your doctor.
9. Children under 12 years old are not allowed.

### **III. Treatment Safety**

1. Treatment Time is less than 30mins/Body Part/Each Treatment.
2. The energy setting is from low to high gradually which should be subject to skin comfortableness for different people.
3. The patient had better wear the protective eye glasses.
4. If you cannot tolerant heat or uncomfortable, please stop the treatment in advance and turn to your doctor for help.

---

## **Chapter 2: Installment and Use**

### **I. Working Condition**

Using at normal temperatures and pressure. Please do not stay in too cold, too hot, too dry and too wet.

### **II. Technical Parameters**

Power supply Input: 100V-240V

Power : 110W

Unoisition Probe:

Power supply output: 150V

Frequency: 40KHz

Power: 25W

Quadrupole 3D SMART RF head for Arm/face/body

Power supply output: 66V

Frequeuncy: 3MHz

Power: 30W


Bipolar 3D SMART RF for face and eyes

Power supply output: 66V

Frequeuncy: 3MHz

Power: 30W


#### IV. Installation

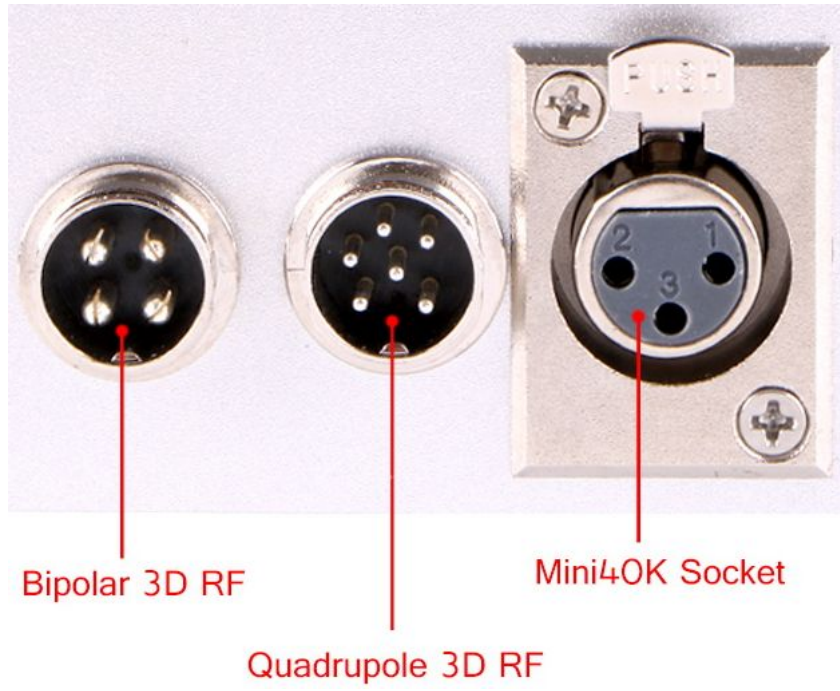






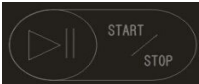
---
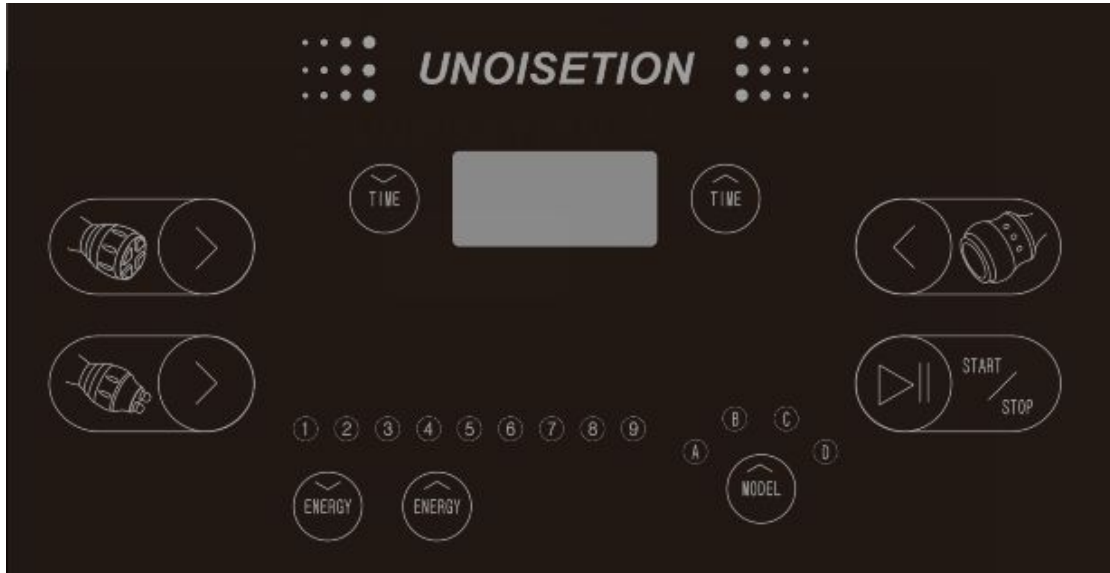
## Chapter 3: Operation

1. Connect the power supply.
2. Turn on the red button of power switch.




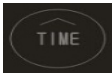



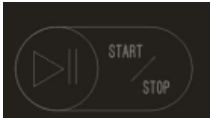


### Unoisetion 40K cavitation

1. Press  button to login unoisetion 40k cavitation function display.
2. Please click  to set working time.the time range is (0-60)mins. We recommend set it with 30mins. and the default time is 30mins.
3. Please click  and  to increase or decrease the time.
4. Please click  to chose working model, press this button. A means continuous working mode , B means pulse working mode, please chose the most comfortable mode upon your feeling.
5. Please click  to start or pause function.









---

## Quadrupole Face RF

1. Press  button to start Face RF function;
  2. Please click  to set working time, the original scheduled time is 30mins;
  3. Press  and  to increase or decrease the energy.
  4. please click  to increase and decrease the energy, the level of energy will show in the display that , from 1-9 as increase and 9-1 as decrease .
  5. Click , from Mode A to Mode B. You will see that the energy from level 1 to level 9 is increasing step by step in Mode A while touching the handle with your hand or skin ; and energy from level 1 to level 9 is increasing rapidly in Mode B while touching the handle with your Facial skin or hands.
- 3.5.6 Please click  to start or pause function.

---

## Bipolar RF for Eye

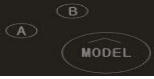- 1 Press  button to start eyes RF function;
2. Please click  to set working time, the original scheduled time is 30mins;
3. Press  and  to increase or decrease the energy.
4. please click  to increase and decrease the energy, the level of energy will show in the display that , from 1-9 as increase and 9-1 as decrease .
5. Click , from Mode A to Mode B. You will see that the energy from level 1 to level 9 is increasing step by step in Mode A while touching the handle with your hand or skin ; and energy from level 1 to level 9 is increasing rapidly in Mode B while touching the handle with your Facial skin or hands.
- 3.5.6 Please click  to start or pause function.

## Chapter 4: Treatment time and operating

1) Abdomen, thigh, buttock, arms slimming

Please clean the skin and then put the ultrasound gel on the treatment area, then turn on the machine, set the time and energy. Please set the energy from low to high to get the best comfortable temperature

---

1. Please use the unoisetion 40k cavitation head 25-30min

Work time: 2-3 times/ week

2. Face, eyes around, neck skin lifting:

Please clean the skin and then put the rf gel on the treatment area, then turn on the machine , set the time and energy, pls set the energy from low to high to get the best comfortable temperature, then follow this steps:

A: Use face radio frequency head for face for 25-30 min


B: Use eye radio frequency head for eye around for 15-20min

Work time: 2-3 times/ week

## **OPERATION**

### **FACE**

Treated time: 30 minutes

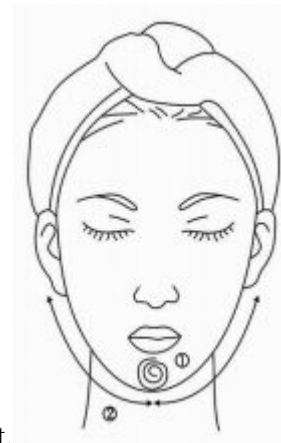

1. Massage the mandible center in circles to produce deep heat.

2. Massage lower jaw in lines.





3. Massage the triangle zone of both sides of the face in circles.
4. Massage from jaw to angulus oris to ear in lines.

## **THIGH**

Treated time: 30 minutes



1. From down to upper, pushing to the groin to dredge the lymph.
2. From down to upper, circling push by anticlockwise can help decomposing



fatness.



3. Also can push by come-and-go to decompose fatness.
4. Pull from the knee and muscle texture to upper, can improve the curve.

---

Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.



3. Along the muscle of arm, pull up to lift and tight the muscle.

## **BACK**

Treated time: 20 to 30 minutes



1. First by come-and-go, pushing the bladder nerve 2 to 3 times.



2. Circling by anticlockwise to stimulate the underarm lymph node.


3. On the back, by anticlockwise circling, can help to decompose the



fatness.

---

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help

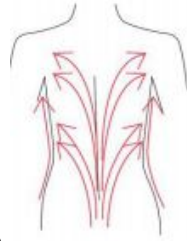

improving the back curve.

**ARM**

**Treated time: 30 minutes**

**BACK SIDE**

1. From the inner elbow to the armpit, doing lymph drainage.
2. Circling to stimulate the lymph node.
3. By anticlockwise or come-and-go pushing, decomposing the fatness.
4. From the elbow to the outer, doing tightening gesture.



---


### Facade

1. From the elbow to the outer, doing tightening gesture.


### Abdomen

Treated time: 20 to 30 minutes

1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.

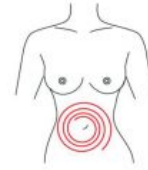


2. From small to large, by anticlockwise direction, circling push down.

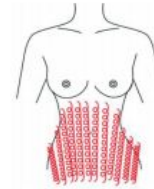

3. From the belly and muscle texture, pull to the groin.



4. Through lymph drainage gesture, taking the toxin to the place of groin.

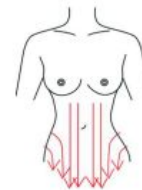

### Attention:

1. Pregnant women or women during in menses.
2. Epileptic.
3. Patients with malignancy.
4. Patient whose wound after operation has not healed up.
5. Acute inflammation or epidemical patients.
6. Whom with heart diseases or with heart pacemaker.
7. Who with kidney (gall-stone) disease.
8. Who was embedded metal object or silica gel.
9. Who in menses, birth control period, emiction incontinence period, or accepting the belly operation.
10. Whose body always takes much inner hot.
11. Who has the genetic hypersensitivity.

### Notice:

- 
1. Be sure to use the special ultrasonic gel.
  2. Avoid knocking the us head.
  3. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
  4. Don't stay in one place, avoid treating on the bone.
  5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
  6. Check the machine power cord if it is connected well.
  7. If the machine will not use for a long time, please switch off it, and power off.
  8. Please take off all metal decoration from the operator and patient's body.
  9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
  10. During the operation, be sure not to accept other treatment.