

Preface

Dear users:

Welcome to choose our latest product YH-4202 which is multi frequency radio frequency technology, negative pressure and low frequency ultrasonic cavitation application integration technology of the latest slimming products. YH-4202 is the instrument of professional beauty, need to be trained operators hand operation if not the proper use of this instrument has the potential to cause adverse effects on the human body, so we hope that any person in the operation of the YH-4202, the operator first read this manual carefully and strictly comply with the manual inside the guide to operate the equipment.

We believe that our products will bring good benefits for you, our perfect after sale service will make you worry free.

Thank you!

Catalog

A

Preface

B

The first part

Working principle
Advantages

C

The second part

Functions
Parameters
Package including
Parts introduction
Basic operation
Forbidden group

Cavitation Working Principle

Cavitation: Many people has the experience of unbearable ring or noise sound in ears,it is the normal cavitation,our professional research team has overcome this problem and the ring has reduced yet after years of study,It is caused Intelligence frequency adjustment technology which is updated from normal cavitation

Its advantage are

- 1 Annoying noise in ears caused by cavitation will be reduced 3 times,the clinical date indicated, more comfortable weight loss experience for you
- 2 Intelligence frequency adjustment technology can adjust the frequency according to the temperature of the derma during fat dissolve, people will not feel very hot during treatment, it is comfortable warm
- 3 As we all know,more hotter the cavitation head, more shorter lifetime for cavitation head, so about 10 times working time can be prolonged by this new intelligence frequency adjustment technology and resulted in much money can be saved
- 4 On the other hand, the output power will be enhanced and its effective result can be seen fast

3D RF adopts scanning variable RF technology,its frequency can be changed automatically upon the temperature of penetration depth at some range

Normal RF adopts single working mode,the depth of penetration by RF can not be changed,so the people will feel electric shocking pulse discharge when do treatment

Comparing to the traditional fixed radio-frequency which can only treat on a certain depth, this is an advanced radio-frequency control technology which can effect different depth of body with changing frequency. The fast change of radio-frequency scanning and energy output in segment equal the energy in treating area, then strength the curative effect .

Advantages

- 1 Intolerable noise in ears caused by cavitation will be reduced 3 times
- 2 3D RF system for body, face and eyes,no pulse discharge,no electric shock,more safe
- 3 Intelligence frequency adjustment technology ,no skin hurt or burned
- 4 10 times working time can be prolonged,high rate of return on investment
- 5 Effective result can be seen immediately, save time and money

1. Skin Layer

Air pressure's suction and release process ----- (1) 、 Improve the fluidity and movement of cells, treat the relative disease like congestion and stagnation of blood.

(2)、 Enlarge lymph vessels and veins.

Results: expelling excess water from tissue fiber.

2. Vascular Layer

Air pressure's suction and release process----- (1)、 No only improve the blood

Functions

For Body

- Eliminate excess fat, fat burning, sculpture, perfect curve
- Body lymphatic drainage and promote blood circulation
- Explode fat cell and dissolve stubborn cellulite.
- Removal excess fat of arm, thigh, waist and hip, shape the body lines, prevention of relaxation
- Improve stretch marks; strengthen the elasticity of tissue fibroblasts.
- Body Shaping: lift and tighten arms, thighs, sculpture waist and abdomen where relaxation after pregnant, cellulite improvement.

For Face

- Whitening and anti-dullness

3D RF System

Parameters

- Skin tightening and lifting, reduces thin wrinkles
- Expel the extra toxin and resolve the pigment and stain
- Restores derm tonification and elasticity
- Lymph drainage for face and reshapes the silhouette
- Improves skin texture and removal eyes' sagging
- Immediate lifting effect revitalized skin and uplifted eye-contour area
- Powerful solution against environmental stressors

For Eyes' Around

- Improvement in circulation
- Compact skin, shrink pores and eye puffiness
- Eliminate wrinkles, dark circle, eye pouches
- Make the skin smooth and delicate, flexibility

Dual Voltage input: AC 240 V- AC 110 V 50-60HZ

Rated input power:400VA

Cavitation frequency : 40KHz

Cavitation Power:up to 80W

Sextupole RF for body

RF frequency:1 MHz

RF power:up to 180W

RF type:six polar

Package including

Quadrupo RF for face
RF frequency:1MHz
RF power:up to 130W
RF type: Quadrupo

Bipolar RF for eye
RF frequency:1MHz
RF power:up to 80W
RF type:bipolar

- 1 x Main machine
- 1 x Cavitation head
- 1 x Sextupole head
- 1 x Quadrupo head
- 1 x Bipolar head
- 1 x Power cord

Parts introduction



Basic Operation



- | | |
|------------------------------|---------------------------|
| 1. Cavitation head connector | 2. Sextupole RF connector |
| 3. Quadropole RF connector | 4. Bipolar RF connector |
| 5. decrease | 6. Start/Pause |
| 7. Set | 8. Increase |
| 9. Instensity | 10. Time |
| 11. Function | 12. Working mode |

Setting

1. Please connect the four handles and power
2. Please turn on the machine.
3. Click button (7) to choose time, fuction, mode and instensity.
4. When the time indicator gliter, please click button (8 and 5) to increase and decrease the working time
5. Then click button (7) , you will see the mode indicator gliter, you can lick button (8 and 5) to choose the working mode.
6. Then click button (7) , you will see the function indicator gliter, you can lick button (8 and 5) to choose the working head, (1 means cavitation head, 2 means Sextupole RF, 3 means Quadropole RF, 4 means Bipolar RF) which head you want to use, please choose the corresponding number)
7. Then click button (7) , you will see the Intensity indicator gliter, you can lick button (8 and 5) to increase and decrease the working energy.
8. Then please click Button (6) to start treatment.

Basic Operation

1. Place annex well and connect to mainframe
2. Please connect the heads well with the main machine
3. Connect power line, turn on general power switch
4. Setting work time, work mode and output intensity, press start working.

Intensity 1-cavitation function

intensity 2-Sextupole function

intensity 3- Quadrupo function

intensity 4- bipolar function

5. Turn off the general power switch, unplug the power line when finished. Sterilize the used work head.

1.Cavitation: 1. Applied gel on fleshy part s(eg.abdomen,hip,t high) The amount depends on moving flexibili of the cavitation head.

2.Set work time(10 min every part),Adjust output by + or -

3.Click "Start", the power indicator will be on

4. Beautician hold the cavitation head to move slowly on skin, in circle or beeline repeatly,the other hand push the fat towards the cavitation head.

5.Do not use on back,avioid using on bones.

6.Do not use on uterus part when women do abdomen care. Abdomen treatment during menst ruate should be avoided.

* Enough gel are needed, you may feel painful if gel is not enough

* The total treatment time for daily care should be within 30 min.

Functions &operation

7.The machine will pause automat ically if the setting time is over.

8.Clean the leftovers with hot towel, use warm towel to clean the ultrasou nd head.

2 3D RF

1. Applied gel on fleshy part s(eg.abdomen,hip,thigh,back,face,eyes' around) The amount depends on moving flexibili of the RF head.

2.Set work time(10 min every part),Adjust output by + or -

3.Click "Start", the power indicator will be on

4. Beautician hold the RF head to move slowly on skin, in circle or beeline repeatly,the other hand push the skin towards the RF head.

5.RF heads can work for back.

* Enough gel are needed, you may feel painful if gel is not enough

* The total treatment time for daily care should be within 30 min.

Treatment procedure

Abdomen ,thigh,buttock ,arms slimming

Pls clean the skin and then put the ultrasound gel on the treatment area ,then turn on the machine, set the time and energy,pls set the energy from low to hight to get the best comfortable temperature ,then follow this steps

1 Pls use the multipolar RF head 5 mins,

2 Pls use 4OK cavitation head 15-30min

3 Pls use the multipolar RF head 10-15min

Work time:2-3 times/week

Face,eyes around,neck skin lifting

Pls clean the skin and then put the RF gel on the treatment area,hen turn on the machin
e, set the time and energy,pls set the energy from low to hight to get the best comfortabl
e temperature ,then follow this steps

1 Pls use the tripolar RF for face for 25-30mins

2 Pls use the bipolar RF for eyes around for 25-30mins

Work time : 2-3 times/week

- 1.People with heart disease or high blood pressure, or configured cardiac pacemaker.
- 2.Patients with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
- 3.People with hemorrhagic disease, trauma or who is bleeding.
- 4.Pregnant women
- 5.Medical Plastic parts, or parts with artificial in filling inside.
- 6.People with metal inside the body(include metallic tooth)
- 7.Patient with skin inflammation or with edema.
- 8.People with an abnormal immune system.
- 9.Numb or insensitive to heat.

Forbidden group