

Your satisfaction, my pursuit

User Manual

YH-4403



T e l : 86-189 4875 5929

Fax : 86-755-8636 8550

Email : sales@mychway.com

Http : //www.mychway.com

A d d : RM406,F4,#6 bldg,zhongxin Industry city,Chuangye Rd,Nanshan Distr,Shenzhen,China 518054

Chinese Shenzhen mychwayTechnology Co., Ltd.

<http://www.mychway.com>

Preface

Dear users:

Welcome to choose our latest product YH-4403 which is multi frequency radio frequency technology, negative pressure and low frequency ultrasonic cavitation application integration technology of the latest slimming products. YH-4403 is the instrument of professional beauty, need to be trained operators hand operation if not the proper use of this instrument has the potential to cause adverse effects on the human body, so we hope that any person in the operation of the YH-4403, the operator first read this manual carefully and strictly comply with the manual inside the guide to operate the equipment.

We believe that our products will bring good benefits for you, our perfect after sale service will make you worry free.

Thank you!

Catalog

A

Preface

B

The first part

Unoisetion Cavitation Principle
3D Radio Frequency
Sextupole Radio Frequency
Bipolar RF
LED Photon

C

The second part

Parameters
Package include
Features
Operation
Forbidden group
Skills&Notice

Unoisetion Cavitation Principle

Many people has the experience of unbearable ring or noise sound in ears,it is the normal cavitation,our professional research team has overcome this problem and the ring has reduced yet after years of study,It is caused Intelligence frequency adjustment technology which is updated from normal cavitation

Its advantage are

1 Unoisetion New Cavitation Technology--Annoying noise in ears caused by cavitation will be reduced 3 times,the clinical data indicated, more comfortable weight loss experience for you

2 Intelligence frequency adjustment technology can adjust the frequency according to the temperature of the derma during fat dissolve, people will not feel very hot during treatment, it is comfortable warm

3 As we all know,more hote the cavitation head, more shorter lifetime for cavitation head, so about 10 times working time can be prolonged by this new intelligence frequency adjustment technology and resulted in much money can be saved

4 On the other hand, the output power will be enhanced and its effective result can be seen fast

3D RF System

3D RF adopts scanning variable RF technology,its frequency can be changed automatically upon the temperature of penetration depth at some range

Normal RF adopts single working mode,the depth of penetration by RF can not be changed,so the people will feel electric shocking pulse discharge when do treatment

Comparing to the traditional fixed radio-frequency which can only treat on a certain depth, this is an advanced radio-frequency control technology which can effect different depth of body with changing frequency. The fast change of radio-frequency scanning and energy output in segment equal the energy in treating area, then strength the curative effect .

Sextupolar Vacuum RF

1Multiolar RF energy penetrates the skin layers to selectively heat the skin tissue and the extra cellular matrix where the collagen fibers are embedded. The local heating causes an immediate contraction of the collagen fibers, and at the same time increases the metabolism of the fibroblasts thereby accelerating the production of new collagen and elastic fibers.

The contraction of collagen fibers and the regeneration of the collagen and elastic deposits, tighten the skin layers creating smoother, healthier and younger looking skin.

Bipolar RF

Facial RF head capacitive coupling electrode is used for transmitting radio-wave energy and generating an electric field for entering into subcutaneous tissue through the skin surface. In the high-frequency of 1 MHz, this electric field changes its polarity for 1 M times/second . To reply to the rapid change in electrode, directions of electrical particles the skin are also changed. By this time, natural electric resistance in the subcutaneous tissue moves and generates heat energy. As thepapillary dermis collagen may immediately shrink when the temperature is within the range from 60 to 70 celcius degree, after wrinkle treatment, client may immediately sense the skin tightening effects as it is being lifted and firmed. When collagen is produced continuously, thickness and density of the skin papillary dermis may be increased to remove wrinkles, eliminate scars, restore skin elasticity and gloss and make it be blond and smooth. While collagen is increased, fresh skin is generated in the position of operation and wrinklesare removed by large amount of cells. In addition, when cortex without elasticity or that with thick horny layer in the area with wrinkles is separated, surrounding skin will also be renewed.

LED Photon

Red Light (650 - 730 nm):

Stimulate the production of collagen. Collagen is the essential protein used to repair damaged tissue and to replace old tissue. Best for fine lines, reduce pore size, swollen after surgery.

Parameters

Power supply Input:110V/220V 60HZ/50HZ

Power :180W

4OK cavitation head:

Power supply output:150V

Frequency:40KHz

Power:25W

Sextupolar Vacuum RF for body

Power supply output:66V

Frequency:3MHz

Power:30W

Quadrupole RF head for body:

Power supply output:66V

Frequency:3MHz

Power:30W

Bipolar RF for face and eyes

Power supply output:66V

Frequency:3MHz

Package Include

1 x Main Machine

1 x 4OK Cavitation Ultrasound Probe

1 x Quadrupole RF for body

1 x Sextupolar Vacuum RF for body

1 x bipolar RF for face and eyes

1 x Holder

1 x Power Cord

Features

1. Optimized for skin tightening and cellulite treatment.
2. Together with the fat reduction, you can also enjoy tighter, toner skin. Immediate visible results !
3. More safe and more effective, no downtime, no side effect, no scar, no rebound
4. Long term and natural results.
5. Versatile and mobile.
6. Easy and user friendly LCD touch screen, displaying preset treatment guidelines that help to optimize the results


Operation

1 Before Operation

- 1.1> Please connect all parts well
- 1.2> Please connect the power cord with the power supply
- 1.3> Please turn on machine
- 1.4> Then will be enter this screen below

Welcome

>1.5 Please click anywhere ,then u will enter the function screen below

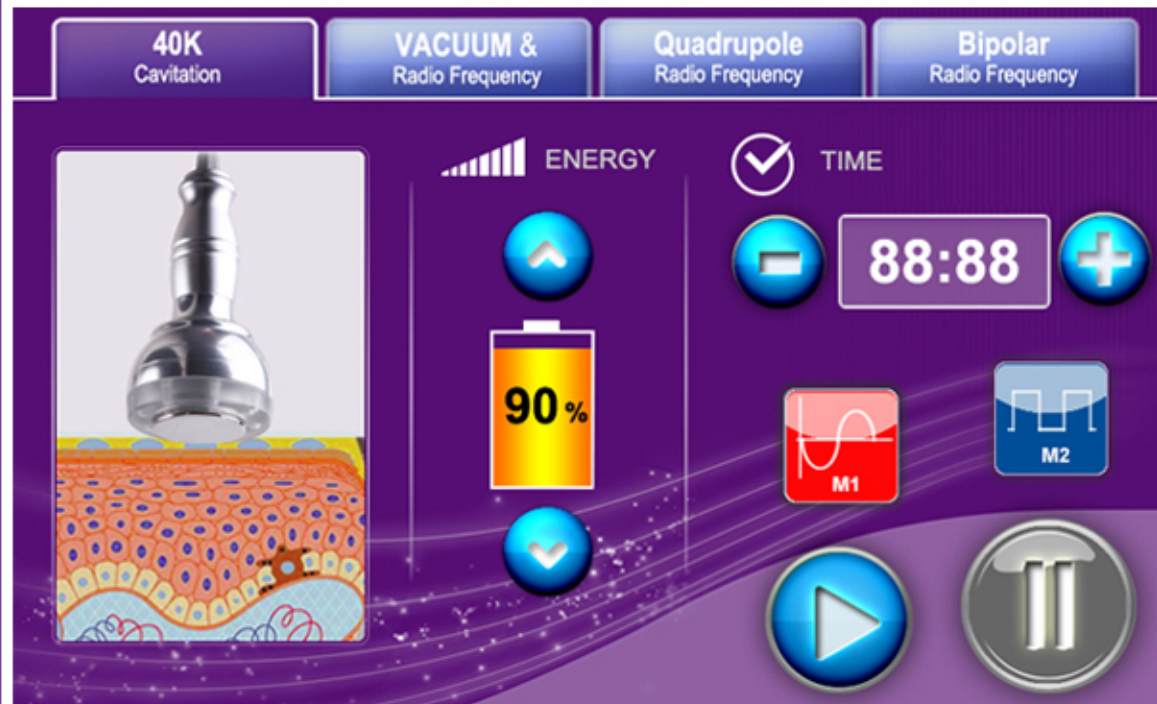

>2 Unoisetion 4OK Cavitation

>2.1 Please press button then login unoisetion 4OK cavitation function display

>2.2 Please click to set working time

>2.3 Please click to increase or decrease the ENERGY

>2.4 Click M1 M2 to chose working mode,M1 means continous working, M2 means pulse working mode,please chose the most comfortable mode upon your feeling

>2.5 Press and to start or pause function

3 Vacuum &Radio Frquency

>3.1 Please press button then login Vacuum& Radio Frequency operation display



>3.2 Please click to set working time


>3.3 Please click to increase or decrease the ENERGY

>3.4 Pleas e click to adjust the working time of SUCTION & RELEASE

>3.5 Press and to start or pause function

4 Quadrupole Radio Frequency

>4.1 Please press button then enter Quadrupole Radio Frequency function operation screen

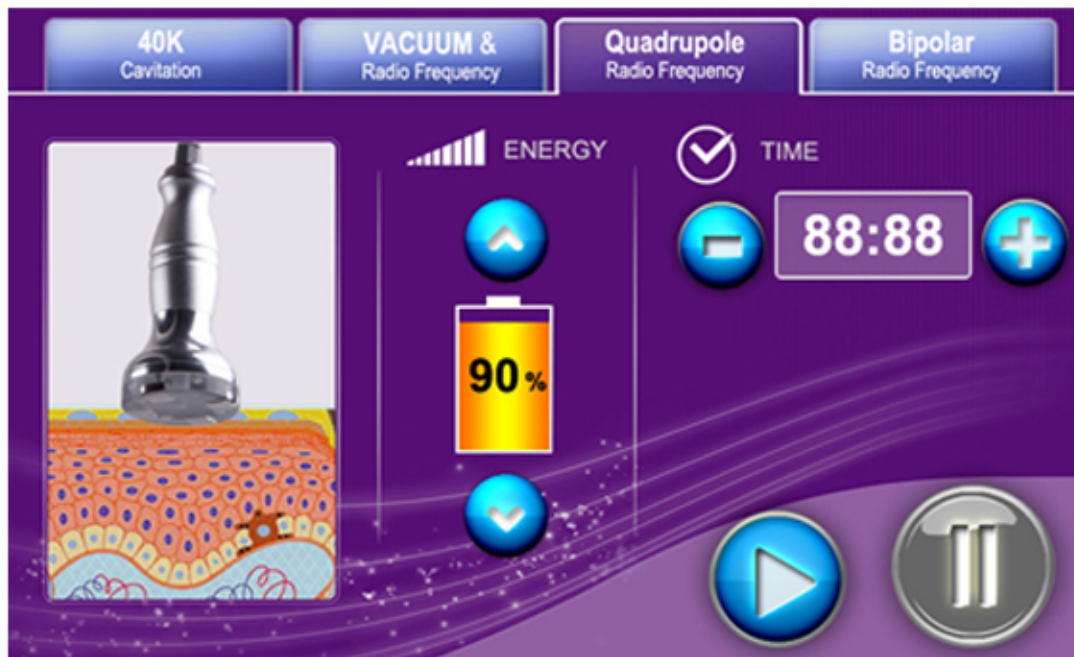


>4.2 Please click 88:88 to set working time

>4.3 Please click and to increase or decrease the ENERGY

>4.4 Press and to start or pause function

5 Bipolar Radio Frequency

>5.1 Please press button then login Bipolar Radio Frequency operation display

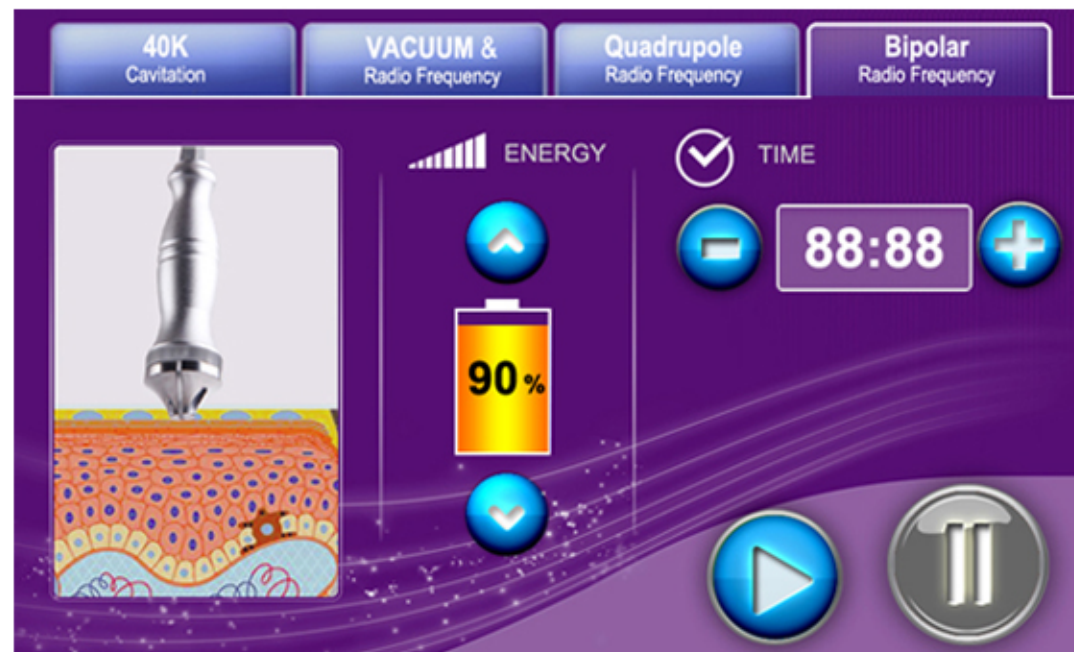

>5.2 Please click 88:88 to set working time

>5.3 Please click and to increase or decrease the ENERGY

>5.4 Press and to start or pause function

Treatment time and times

Abdomen ,thigh,butttock ,arms slimming

Pls clean the skin and then put the ultrasound gel on the treatment area ,then turn on the machine, set the time and energy,pls set the energy from low to hight to get the best comfortable temperature ,then follow this steps

- 1 Please use the vacuum RF head 5 mins,
- 2 Please use unoisetion 4OK cavitation head 15-30min
- 3 Please use the vacuum RF head 10-15min

Work times:2-3 times/week

Face,eyes around,neck skin lifting

Pls clean the skin and then put the RF gel on the treatment area,then turn on the machine, set the time and energy,pls set the energy from low to hight to get the best comfortable temperature ,then follow this steps

- 1 Please use the quadrupole RF for face for 25-30mins
- 2 Please use the bipolar RF for eyes around for 25-30mins

Work times : 2-3 times/week

Forbidden group

- 1.People with heart disease or high blood pressure, or configured cardiac pacemaker.
- 2.Patients with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
- 3.People with hemorrhagic disease, trauma or who is bleeding.

Skills

4. Pregnant women
5. Medical Plastic parts, or parts with artificial in filling inside.
6. People with metal inside the body (include metallic tooth)
7. Patient with skin inflammation or with edema.
8. People with an abnormal immune system.
9. Numb or insensitive to heat.


BUTTOCKS

ILLUSTRATION of OPERATION

FACE

Treated time: 30 minutes

1. Massage the mandible center in circles to produce deep heat.
2. Massage lower jaw in lines.

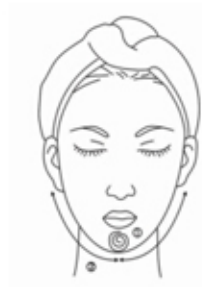

3. Massage the triangle zone of both sides of the face in circles.



4. Massage from jaw to angulus oris to ear in lines.

THIGH

Treated time: 30 minutes

1. From down to upper, pushing to the groin to dredge the lymph.



2. From down to upper, circling push by anticlockwise can help decomposing fatness.



3. Also can push by come-and-go to decompose fatness.



4. Pull from the knee and muscle texture to upper, can improve the curve.



Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.

3. Along the muscle of arm, pull up to lift and tight the muscle.




BACK

Treated time: 20 to 30 minutes


1. First by come-and-go, pushing the bladder nerve 2 to 3 times.



2. Circling by anticlockwise to stimulate the underarm lymph node.

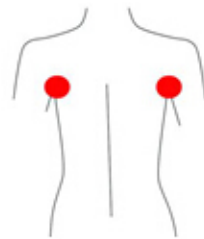

3. On the back, by anticlockwise circling, can help to decompose the fatness.

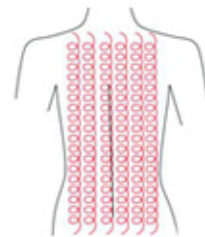


4. Using lymph drainage gesture, pull the toxin to the lymph node, can help improving the back curve.



ARM

Treated time: 30 minutes

1. From the inner elbow to the armpit doing lymph drainage.

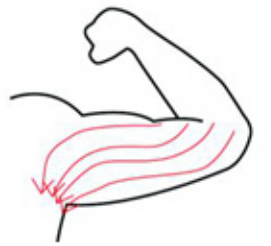

2. Circling to stimulate the lymph node.

3. By anticlockwise or come-and-go pushing, decomposing the fatness.




3. By anticlockwise or come-and-go pushing, decomposing the fatness.



Abdomen

Treated time: 20 to 30 minutes

1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.

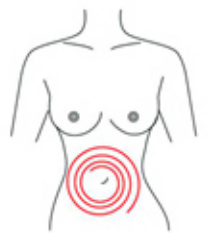

2. From small to large, by anticlockwise direction, circling push down.




3. From the belly and muscle texture, pull to the groin.



Notice

4. Through lymph drainage gesture, taking the toxin to the place of groin.

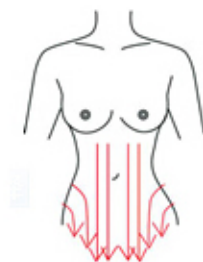

Notice:

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
7. If the machine will not use for a long time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.