

Your satisfaction, my pursuit

User Manual

YH-5301

MYCHWAY[®]



Tel : 86-189 4875 5929

Fax : 86-755-8636 8550

Email : sales@mychway.com

Http : //www.mychway.com

Address : RM406,F4,#6 bldg, zhongxin Industry city, Chuangye Rd, Nanshan Distr, Shenzhen, China 518054

Chinese Shenzhen mychwayTechnology Co., Ltd.

<http://www.mychway.com>

Preface

Dear users:

Welcome to choose our latest product YH-5301, which is multi frequency radio frequency technology, negative pressure and low frequency ultrasonic cavitation application integration technology of the latest slimming products. YH-5301, is the instrument of professional beauty, need to be trained operators hand operation if not the proper use of this instrument has the potential to cause adverse effects on the human body, so we hope that any person in the operation of the YH-5301,, the operator first read this manual carefully and strictly comply with the manual inside the guide to operate the equipment.

We believe that our products will bring good benefits for you, our perfect after sale service will make you worry free.

Thank you!

Catalog

Preface

The first part

Working Principle
Parameters
Pacckage Listing

The second part

Operation
Skills
Recommendation Treatment
Time& Times
Notice

Working Principle

Unoisetion Cavitation 2.0

Unoisetion Cavitation 2.0 is offering a new treatment which we believe can potentially replace traditional cavitation. Unoisetion customers can get results they are satisfied with, without having to undertake buzzing and noise in ears

Benefits can include:

A. SAFETY

Unoisetion New Cavitation Technology--Annoying noise in ears caused by cavitation will be reduced 3 times,the clinical data indicated, more comfortable weight loss experience for you

B INTELLIGENCE

Intelligence frequency adjustment technology can adjust the frequency according to the temperature of the derma during fat dissolve, people will not feel very hot during treatment, it is comfortable warm

C LONG WORKING LIFE TIME OF PROBES

As we all know,more hotter the cavitation head, more shorter life time for cavitation head, so about 10 times working time can be prolonged by this new intelligence frequency adjustment technology and resulted in much money can be saved

D BETTER VISIBLE EFFECT

On the other hand, the output power will be enhanced and its effective result can be seen fast

E. DIGITIZATION CALIBRATION

unoisetion utilizes the intelligent calibration technology what means it can adjust the

make the result of body shape more effective(3 times as long as normal cavitation) and the handles' lifetime will be 10 times more than normal cavitation

Quadrupole RF with Vacuum

YH-5301 combines the mechanism of advanced 3D RF energy and vacuum therapy that work with the body's natural regeneration process to deliver safe and efficient treatments for cellulite, body contouring and tightening of the skin.

YH-5301 is powered by 3D RF technology featuring AUTOMATICALLY ,INTELLIGENT,CONTROL SYSTEM that simultaneously RF energy output and Photon Therapy ,with the ability to deliver safe and effective treatments with SUPERIOR treatment CONTROL, PRECISION TARGETING and BROADER DERMAL PENETRATION for maximum results in minimal time.

Simultaneously, the unique vacuum therapy maximizes the penetration of 3D RF energy in the treatment area and also increases local blood circulation, stimulates lymphatic drainage and assists in the shrinkage of fat cell volume.

Quadrupole 3D RF with vacuum & photon assists in the Metabolism of Large Fat Granules previously ruptured by cavitation in treatment of Unoisetion Cavitation 2.0
4OK

Quadrupolar 3D RF for Face

3D RF adopts ART RF technology measures skin resistance automatically, No energy emit if tip don't touch skin well. RF energy release rate is adjustable, and increase the treatment comfort.

1 Effective

The 3D RF energy output reaches ultimate results rather than normal RF

This designed tip of different sizes are respectively suitable for face and body treatments.

3 Comfortable

RF energy release rate is adjustable, and increase the treatment comfort.

Carefully selected RF energy frequency can greatly reduce stimulation to nerve fiber.

4 Economical

Treatment tips can be used limitless within its lifespan

The equipment has few consumables

Bipolar 3D RF for Eyes

The heat generated by passing 3D RF energy through the skin shrinks dermal collagen and promotes the formation of new collagen

The treatment head draws the skin between two parallel sets of electrodes located in the tip. High frequency electrical (3D RF) current is passed through the skin. A flat surface delivers current more efficiently, so suction induced skin deformation provides an effective contact. As such, the 3D RF treatment depends somewhat on the thickness and coarseness of the skin in the target area. This mechanism induces mechanical stress on fibroblasts, leading to collagenesis and possibly a more dramatic improvement of the acne scar.

Features:

1ART technology make discomfort during treatment was minimal and adverse effects were limited to mild and temporary erythema.

2 The bipolar 3D RF appears to be a safe and effective method for reducing mild to moderate facial acne scars.

3 The deepest energy penetration , greatly improve effect and safety, even for the upper eyelid area treatment.

4 The synergistic combination of mechanical stress and conducted 3D RF energies increase the oxygen intracellular diffusion by heating the skin result in refresh skin

Sextupole 3D RF for Body

Sextupole 3D RF is applied to the surface of the skin tissue causing lift and stimulation of the under-lying structures. Meanwhile, Sextupole 3D RF is aimed at improving blood flow and lymph circulation. The dermis and subcutaneous layer are heated simultaneously forcing collagen fibers to contract. The contracting fibers pull and tighten the skin, creating an immediate effect. Fibroblasts' metabolism increases, producing new collagen fibers leaving a long term effect.

Most of the Sextupole 3 D RF energy enters into the deeper dermal layer, causing the dermal layer of skin to thicken, decreasing wrinkle depth, leaving the skin firmer with contour lifted.

The original collagen protein becomes strengthened to increase the new collagen protein as a result of the radio frequency stimulating the skin.

Parameters

Parameters

Power supply Input:100V-240V

Power :190W

Package Listing

Uoisetion 4OK Cavitation Head

Frequency:40KHz

Power:15W

Sextupolar 3D RF for Body

Frequeuncy:1MHz

Power:50W

Quadrupole RF with Vacuum

Frequeuncy:1MHz

Power:50W

Vacuum: <-80kPa

Pressure: >250kPa

Air flow: >10L/minute

Noise level: <70dB (30cm away)

Quadrupole 3D RF for face

Frequeuncy:1MHz

Power:30W

3D Bipolar RF

Frequeuncy:1MHz

Power:10W

Package List

1 x Host

1 x Uoisetion 4OK Cavitation Head

1 x Sextupolar 3D RF Head

1 x Bipolar 3D RF Head

1 x Quadrupolar 3D RF Head

1 x Quadrupole RF with Vacuum Head

1 x Holder

1 x Power Supply Cord

Operation

Before operation

- 1 Please connect all parts well
- 2 Please connect power cord with power supply
- 3 Run on the machine
- 4 Then will be enter this menu screen as below




- 5 Enter single function display after press one of this five function

Unoisetion Cavitation 2.0

- 1 Please press **40K UNOISETION** button then login unoisetion 4OK cavitation function display
- 2 Please click to set working time
- 3 Please click to increase or decrease the ENERGY
- 4 Please click or to chose working model, continuous wave is for normal fat, pulse wave is for very fat, Please chose the right mode according to fat condition
- 5 Please click to start or pause function
- 6 Press will exist working function display then enter menu screen
- 7 Press means pause and all setting parameters will be zero



Quadrupole RF with Vacuum

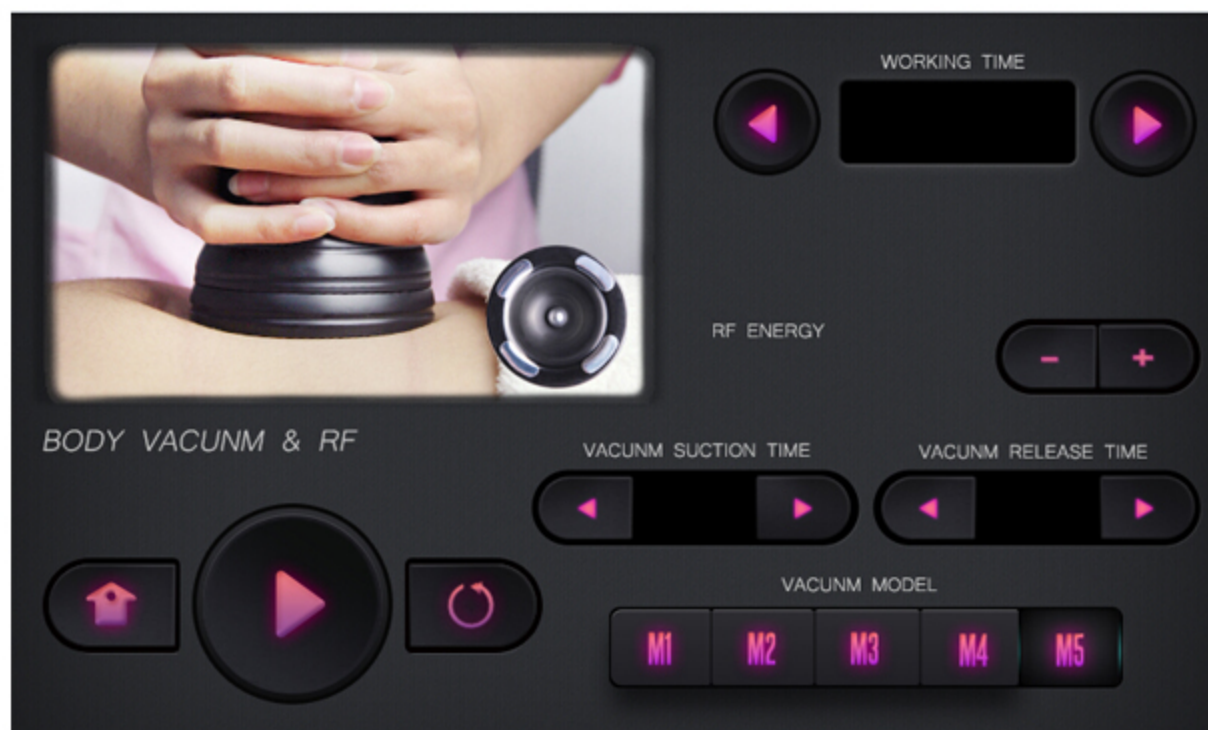

1 Please press **BODY VACUNM & RF** button then login unoisetion body vacuum &RF display

2 Please click to set working time

3 Please click to increase or decrease the ENERGY

4 Please click to set vacuum suction time and release time

5 Please click to start or pause function

6 Press will exist working function display then enter menu screen

7 Press means pause and all setting parameters will be zero

8 Vacuum mode

M1 Massage Mode

Improving the fluidity and movement of cells, treat the relative disease like congestion and stagnation of blood

M2 Drainage Mode

Enlarge lymph vessels and veins. result in expelling excess water from tissue fiber

M3 Whitening Mode

Not only improve the blood circulation of capillary, but also the flow between the deeper veins and lymph node.

M4 Detox Mode

Expelling excess toxic inside body so strength the blood vessel, treat the varicosity

M5 Skin Lifting Mode


Recover the skin elasticity, insist the skin stretch. collagen could give skin its strength and elasticity,destroy and elastic the tough fiber of cellulite to shaping body

Quadrupolar 3D RF for Face

1 Please press **FACE RF** button then login quadrupolar 3D RF face function display

2 Please click to set working time


3 Please click to increase or decrease the ENERGY

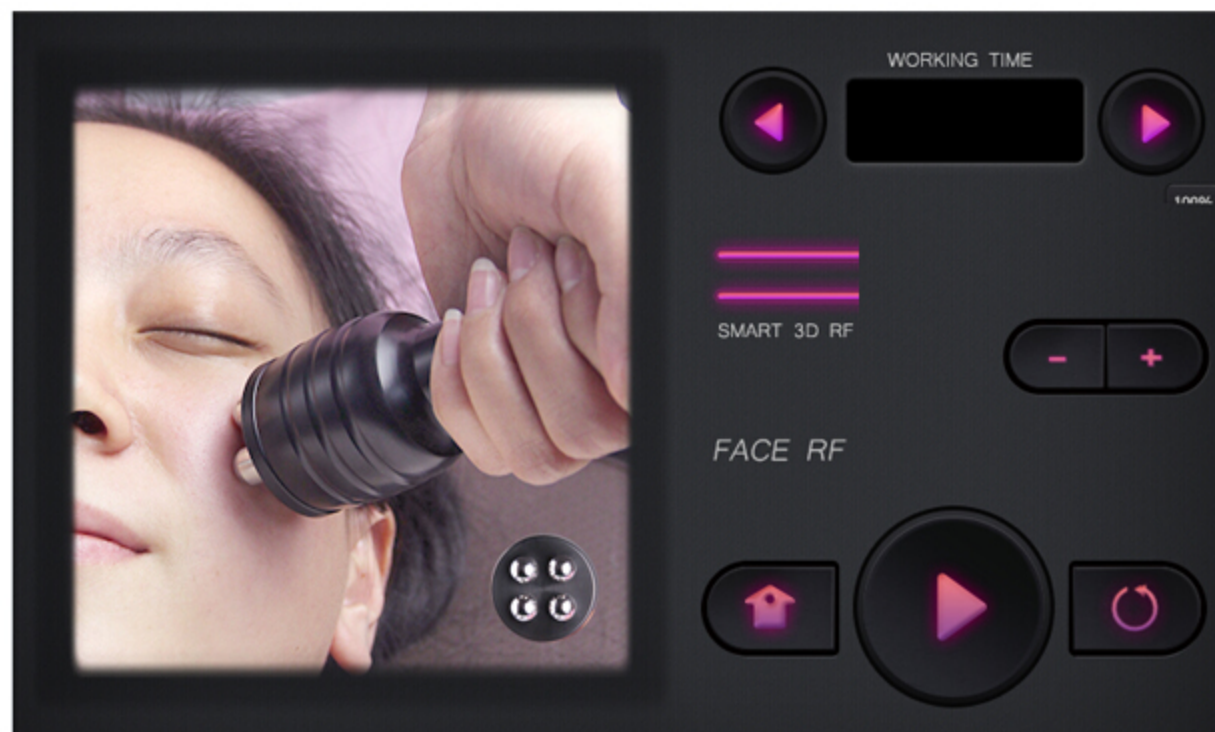

- 4 Please click to start or pause function
- 5 Press will exist working function display then enter menu screen
- 6 Press means pause and all setting parameters will be zero
- 7 SMART 3D RF energy indication bar

Bipolar 3D RF for Eyes

- 1 Please press button then login bipolar 3D RF face function display
- 2 Please click to set working time
- 3 Please click to increase or decrease the ENERGY
- 4 Please click to start or pause function
- 5 Press will exist working function display then enter menu screen
- 6 Press means pause and all setting parameters will be zero

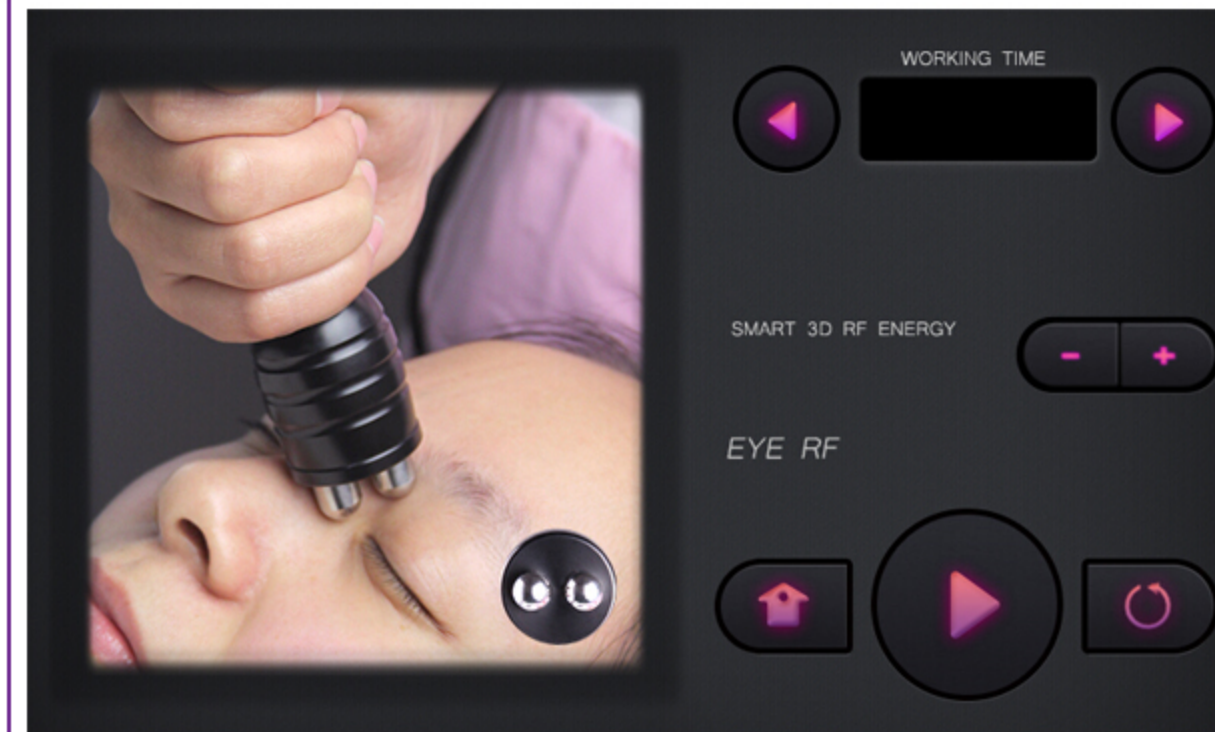

Sextupole 3D RF for Body




- 1 Please press button then login quadrupolar 3D RF face function display

Skills

2 Please click  to set working time

3 Please click  to increase or decrease the ENERGY

4 Please click  to start or pause function

5 Press  will exist working function display then enter menu screen

6 Press  means pause and all setting parameters will be zero

BUTTOCKS

ILLUSTRATION of OPERATION

FACE


Treated time: 30 minutes

1. Massage the mandible center in circles to produce deep heat.

2. Massage lower jaw in lines.



3. Massage the triangle zone of both sides of the face in circles.

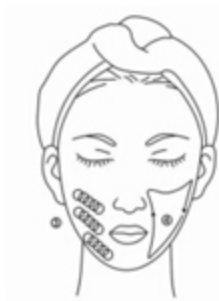

4. Massage from jaw to angulus oris to ear in lines.

THIGH

Treated time: 30 minutes

1. From down to upper, pushing to the groin to dredge the lymph.



2. From down to upper, circling push by anticlockwise can help decomposing fatness.



3. Also can push by come-and-go to decompose fatness.



4. Pull from the knee and muscle texture to upper, can improve the curve.



Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.

3. Along the muscle of arm, pull up to lift and tight the muscle.




BACK

Treated time: 20 to 30 minutes


1. First by come-and-go, pushing the bladder nerve 2 to 3 times.



2. Circling by anticlockwise to stimulate the underarm lymph node.

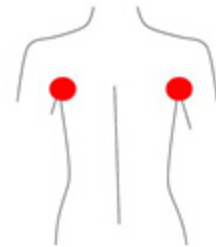

3. On the back, by anticlockwise circling, can help to decompose the fatness.

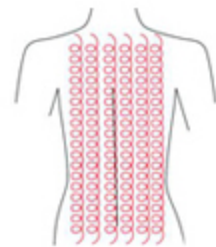

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help improving the back curve.



ARM

Treated time: 30 minutes

1. From the inner elbow to the armpit doing lymph drainage.



2. Circling to stimulate the lymph node.

3. By anticlockwise or come-and-go pushing, decomposing the fatness.




3. By anticlockwise or come-and-go pushing, decomposing the fatness.



Abdomen

Treated time: 20 to 30 minutes


1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.

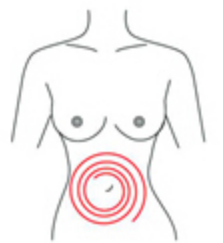


2. From small to large, by anticlockwise direction, circling push down.



3. From the belly and muscle texture, pull to the groin.

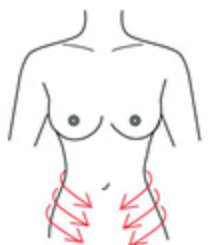

4. Through lymph drainage gesture, taking the toxin to the place of groin.

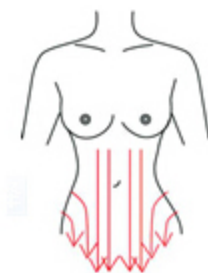

Treatment time and times

Abdomen ,thigh,buttock ,arms slimming

Pls clean the skin and then put the ultrasound gel on the treatment area ,then turn on the machine, set the time and energy,pls set the energy from low to hight to get the best comfortable temperature ,then follow this steps

- 1 Please use the vacuum RF head 5 mins,
- 2 Please use unoisetion 4OK cavitation head 15-30min
- 3 Please use the vacuum RF head 10-15min

Work times:2-3 times/week

Face,eyes around,neck skin lifting

Pls clean the skin and then put the RF gel on the treatment area,then turn on the machine, set the time and energy,pls set the energy from low to hight to get the best comfortable temperature ,then follow this steps

- 1 Please use the quadrupole RF for face for 25-30mins
- 2 Please use the bipolar RF for eyes around for 25-30mins

Work times : 2-3 times/week

Notice:

- 1.Be sure to use the special ultrasonic gel.
- 2.Avoid knocking the us head.

3.For avoiding burning the head, during operation, please prepare enough ultrasonic gel.

4.Don't stay in one place, avoid treating on the bone.

5.Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.

6.Check the machine power cord if it is connected well.

7.If the machine will not use for a long time, please switch off it, and power off.

8.Please take off all metal decoration from the operator and patient's body.

9.If continuous using 1 hour, please pause the machine about 10 minutes, then using again.

10.During the operation, be sure not to accept other treatment.

Recommendation Treatment Time &Times

Notice