

Tel: 86-15012486303 Fax:86755-8636 8550
Email: sales@mychway.com Http://www.mychway.com
Add: RM406, F4, #6 Bldg, zhongxin Industry city, Chuangye Rd, Nansha Distr, Shenzhen, China
518054

**Unoisetion Caviation Smart 3D RF Vacuum Slimming Skin
Rejuvenation BIO Machine**

Model: YH-6501



Part one Introduction

1.1 unoisetion cavitation principle

Unoisetion Cavitation 2.0 is offering a new treatment which we believe can potentially replace traditional cavitation. Unoisetion customers can get results they are satisfied with, without having to undertake buzzing and noise in ears

Benefits can include:

A. SAFETY

Unoisetion New Cavitation Technology--Annoying noise in ears caused by cavitation will be reduced 3 times,the clinical data indicated, more comfortable weight loss experience for you

B INTELLIGENCE

Intelligence frequency adjustment technology can adjust the frequency according to the temperature of the derma during fat dissolve, people will not feel very hot during treatment, it is comfortable warm

C LONG WORKING LIFE TIME OF PROBES

As we all know,more hotter the cavitation head, more shorter life time for cavitation head, so about 10 times working time can be prolonged by this new intelligence frequency adjustment technology and resulted in much money can be saved

D BETTER VISIBLE EFFECT

On the other hand, the output power will be enhanced and its effective result can be seen fast

E. DIGITIZATION CALIBRATION

unoisetion utilizes the intelligent calibration technology what means it can adjust the frequency during whole treatment and calibratethe error of frequency by itself to make the output frequency keep the same frequency as before,in this way, the unoisetion cavitationwill make the result of body shape more effective(3 times as long as normal cavitation) and the handles' lifetime will be 10 times more than normal cavitation

1.2 Quadruple &Bipolar 3D SMART RF with Photon

This RF face eyes' around treatment is aimed at further forcing the excretionof triglycerides and skin tightening with Quadruple or Bipolar 3D SMART RF &Photon High-power 3D SMART Quadruple &Bipolar RF (radio frequency) currents preferentially and intensively heat the sub-dermis (where cellulite is located), exerting a triple slimming/tightening effect on the cellulite tissues. Specifically, these currents:

- A- Immediately shorten the existing collagen fibers, thereby offering an immediate skin tightening/lifting effect.
- B- Stimulate the fibroblasts to produce new collagen and elastic fibers, thereby offering a separate skin tightening effect.
- C-Stimulate the adipocytes to release fat into the bloodstream and physically damage a number of them (adipocytes are very “fragile” cells, in comparison to fibroblasts or other tissue cells).

1.3 Sextupole 3D SMART RF with Vacuum&Photon

YH-5401 combines the mechanism of advanced 3D SMART RF energy and vacuum therapy and Photon that work with the body's natural regeneration process to deliver safe and efficient treatments for cellulite, body contouring and tightening of the skin.

Unlike any other normal RF vacuum system,YH-5401 is powered by 3D SMART RF technology featuring AUTOMATICALLY ,INTELLIGENT,CONTROL SYSTEM

that simultaneously RF energy output and Photon Therapy ,with the ability to deliver safe and effective treatments with SUPERIOR treatment CONTROL, PRECISION TARGETING and BROADER DERMAL PENETRATION for maximum results in minimal time.

Simultaneously, the unique vacuum therapy and photon therapy maximizes the penetration of 3D SMART RF energy in the treatment area and also increases local blood circulation, stimulates lymphatic drainage and assists in the shrinkage of fat cell volume.

Sextupole 3D SMART RF with vacuum & photon assists in the Metabolism of Large Fat Granules previously ruptured by cavitation in treatment of Unoisetion Cavitation 2.0 40K

The dermis and subcutaneous layer are heated simultaneously forcing collagen fibers to contract. The contracting fibers pull and tighten the skin, creating an immediate effect. Fibroblasts' metabolism increases, producing new collagen fibers leaving a long term effect.

Most of the Sextupole 3 D SMART RF energy enters into the deeper dermal layer, causing the dermal layer of skin to thicken, decreasing wrinkle depth, leaving the skin firmer with contour lifted.

The original collagen protein becomes strengthened to increase the new collagen protein as a result of the radio frequency stimulating the skin.

1.4 Photon

Red Light(650-730nm):

Stimulate the production of collagen. Collagen is the essential protein used to repair damaged tissue and to replace old tissue. Best for fine lines, reduce pore size., swollen after surgery.°

Part Two Function&Parameters&Package Including

Unoisetion Cavitation 2.0 40K Function

- Instant Inch-Loss
- Non-Invasive Fat Removal | Cellulite Reduction
- Body Sculpting | Slimming Treatments

SMART RF (radio frequency) Function

- Reduce fine lines and wrinkles
- Lift, tone and brighten the face
- Tighten the skin on the underarms and legs
- Reduce cellulite
- Contour and tighten post-pregnancy tummy
- Treatment on face, neck, hands, knees etc

Parameters

Power supply Input:100V-240V

Power :180W

- 40K cavitation head:

Frequency:40KHz Power:25W

- Sextupolar Vacuum RF for body:

Frequeuncy:3MHz Power:30W

- Quadrupole RF head for body:

Frequeuncy:3MHz Power:30W

- Bipolar RF for face and eyes

Frequeuncy:3MHz Power:30W

Tel: 86-15012486303 Fax:86755-8636 8550
Email: sales@mychway.com Http://www.mychway.com
Add: RM406, F4, #6 Bldg, zhongxin Industry city, Chuangye Rd, Nansha Distr, Shenzhen, China
518054

●Red Wavelength:650 - 730 nm

●Bipolar 3D SMART RF probe for face and eyes' area

Power supply output:66V

Frequeuncy:3MHz Power:30W

●BIO Probe:

Power supply output:24V

Frequeuncy:1.5K Power:1W

Package Including

1 x Host

1 x Unoisetion Cavitation 2.0 Probe

1 x Setupole 3D SMART RF

1 x Bipolar 3D SMART RF Probe with Photon for Eye Area

1 x Quadrupole 3D SMART RF Probe with Photon for Face

1 x Sextupole 3D SMART RF Probe with Vacuum &Photon

1 x BIO

1 x Power Supply Cord

Part Three Operation

3.1 Before operation


- 1.1 Please connect all parts well
- 1.2 Please connect power cord with power supply
- 1.3 Run on the machine
- 1.4 Then will be enter this screen below






3.2 40K Unoisetion



2.1 Press  button then login 40k unoisetion function display, please see the following:



2.2 Please click  to set working time.

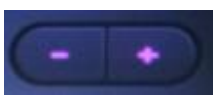
2.3 Please click  to increase or decrease the ENERGY




2.4 Please click  and  to chose working model.




2.5 Please click  to start or pause function.

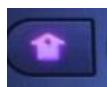

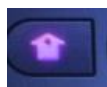
2.6 Please click  to come back to the main function interface.


3.3Body vacuum&RF




3.3.1 Press  button then login Body RF Vacuum operation display



3.3.2 Please click  to set working time.

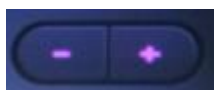

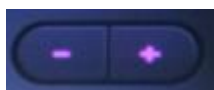3.3.3 Please click  to increase or decrease the RF ENERGY.



3.3.4 Please click  and  to set the time for the

suction and release.



3.3.5 Please click M1,M2,M3,M4 to choose the model.




3.3.6 Please click to start or pause function.



3.3.7 Please click to come back to the main function interface.


3.4 BODY RF

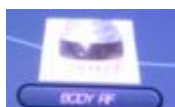

3.4.1 Press button then login BODY RF operation display



3.4.2 Please click to set working time.

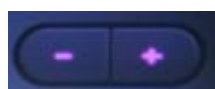


3.4.3 Please click to increase or decrease the RF ENERGY.




3.4.4 Please click to control the led light on the head.

3.4.5 Please click  to start or pause function.

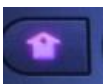
3.4.6 Please click  to come back to the main function interface.

3.5 Face RF

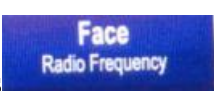3.5.1 Press  button then login Face Radio Frequency operation display




3.5.2 Please click  to set working time.

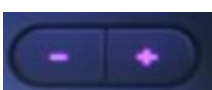
3.5.3 Please click  to increase or decrease the ENERGY.

3.5.4 Please click  to start or pause function.

3.5.5 Please click  to control the led light on the head.

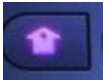3.5.6 Please click  to come back to the main function interface.

3.6 Eye Radio Frequency

3.6.1 Press  button then login Eye Radio Frequency operation display.




3.6.2 Please click  to set working time.


3.6.3 Please click  to increase or decrease the RF ENERGY.


3.6.4 Please click  to control the led light on the head.

3.6.5 Please click  to start or pause function.


3.6.6 Please click  to come back to the main function interface.


3.7 BIO

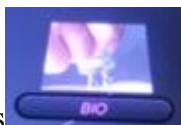


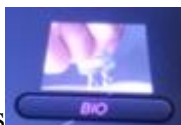3.7.1 Press  button then login BIO operation display



3.7.2 Please click  to set working time.

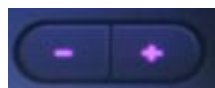

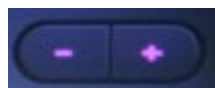3.6.3 Please click  to increase or decrease the BIO ENERGY.



3.6.4 Please click  to set the BIO model.



3.6.5 Please click  to start or pause function.



3.6.6 Please click  to come back to the main function interface.

Part Four

Treatment time and times

1) Abdomen, thigh, buttock, arms slimming

Please clean the skin and then put the ultrasound gel on the treatment area, then turn on the machine, set the time and energy. Please set the energy from low to high to get the best comfortable temperature, then follow this steps:

- 1, Please use the Body RF Vacuum head 5 mins
- 2, Please use the unoisetion 40k cavitation head 15-30min
- 3, Please use the Body RF Vacuum head 10-15min

Work time: 2-3 times/ week

2) Face, eyes around, neck skin lifting:

Please clean the skin and then put the rf gel on the treatment area, then turn on the machine , set the time and energy, pls set the energy from low to high to get the best comfortable temperature, then follow this steps:

- 1, use face radio frequency head for face for 25-30 min
- 2, use eye radio frequency head for eye around for 25-30min
- 3, use BIO for face and eye around for 10-15min


Work time: 2-3 times/ week

BUTTOCKS

ILLUSTRATIN of OPERATION

FACE

Treated time: 30 minutes

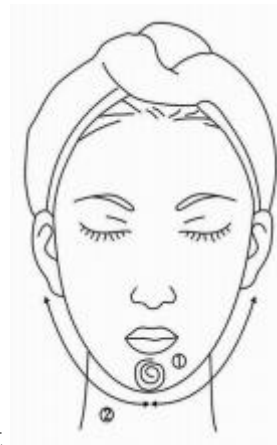

1. Massage the mandible center in circles to produce deep heat.
2. Massage lower jaw in lines.



3. Massage the triangle zone of both sides of the face in circles.
4. Massage from jaw to angulus oris to ear in lines.

THIGH

Treated time: 30 minutes



1. From down to upper, pushing to the groin to dredge the lymph.

2. From down to upper, circling push by anticlockwise can help decomposing



fatness.



3. Also can push by come-and-go to decompose fatness.

4. Pull from the knee and muscle texture to upper, can improve the curve.

Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.




3. Along the muscle of arm, pull up to lift and tight the muscle.

BACK

Treated time: 20 to 30 minutes



1. First by come-and-go, pushing the bladder nerve 2 to 3 times.

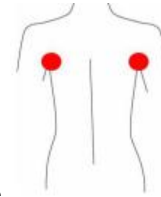


2. Circling by anticlockwise to stimulate the underarm lymph node.

3. On the back, by anticlockwise circling, can help to decompose the



fatness.

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help

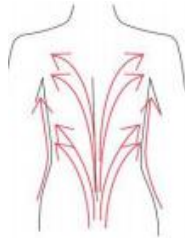

improving the back curve.

ARM

Treated time: 30 minutes

BACK SIDE

1. From the inner elbow to the armpit, doing lymph drainage.
2. Circling to stimulate the lymph node.
3. By anticlockwise or come-and-go pushing, decomposing the fatness.
4. From the elbow to the oster, doing tightening gesture.




Facade

1. From the elbow to the oster, doing tightening gesture.


Abdomen

Treated time: 20 to 30 minutes

1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.

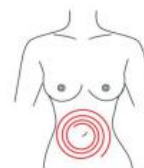


2. From small to large, by anticlockwise direction, circling push down.

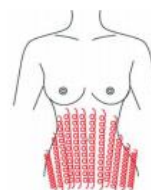

3. From the belly and muscle texture, pull to the groin.



4. Through lymph drainage gesture, taking the toxin to the place of groin.

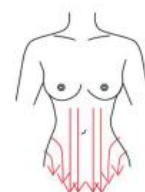

Attention:

1. Pregnant women or women during in menses.

2. Epileptic.
3. Patients with malignancy.
4. Patient whose wound after operation has not healed up.
5. Acute inflammation or epidemical patients.
6. Whom with heart diseases or with heart pacemaker.
7. Who with kidney (gall-stone) disease.
8. Who was embedded metal object or silica gel.
9. Who in menses, birth control period, emiction incontinence period, or accepting the belly operation.
10. Whose body always takes much inner hot.
11. Who has the genetic hypersensitivity.

Notice:

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
7. If the machine will not use for a long time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.