

Your satisfaction, my pursuit

User Manual

YH-6601

MYCHWAY®



Tel : 86-189 4875 5929

Fax : 86-755-8636 8550

Email : sales@mychway.com

Http : //www.mychway.com

Address : RM406,F4,#6 bldg,zhongxin Industry city,Chuangye Rd,Nanshan Distr,Shenzhen,China 518054

Chinese Shenzhen mychwayTechnology Co., Ltd.

<http://www.mychway.com>

Preface

Dear users:

Welcome to choose our latest product YH-6601, which is multi frequency radio frequency technology, negative pressure and low frequency ultrasonic cavitation application integration technology of the latest slimming products. YH-6601, is the instrument of professional beauty, need to be trained operators hand operation if not the proper use of this instrument has the potential to cause adverse effects on the human body, so we hope that any person in the operation of the YH-6601,, the operator first read this manual carefully and strictly comply with the manual inside the guide to operate the equipment.

We believe that our products will bring good benefits for you, our perfect after sale service will make you worry free.

Thank you!

Catalog

Preface

The first part

Working Principle
Parameters
Package Listing

The second part


Operation
Before vs After
Skills
Notice

Working Principle

Unoisition Cavitation 2.0

Unoisition Cavitation 2.0 is offering a new treatment which we believe can potentially replace traditional cavitation. Unoisition customers can get results they are satisfied with, with less buzzing and noise in ears

Ultrasonic penetration into skin, according to selectively destroy fat cells

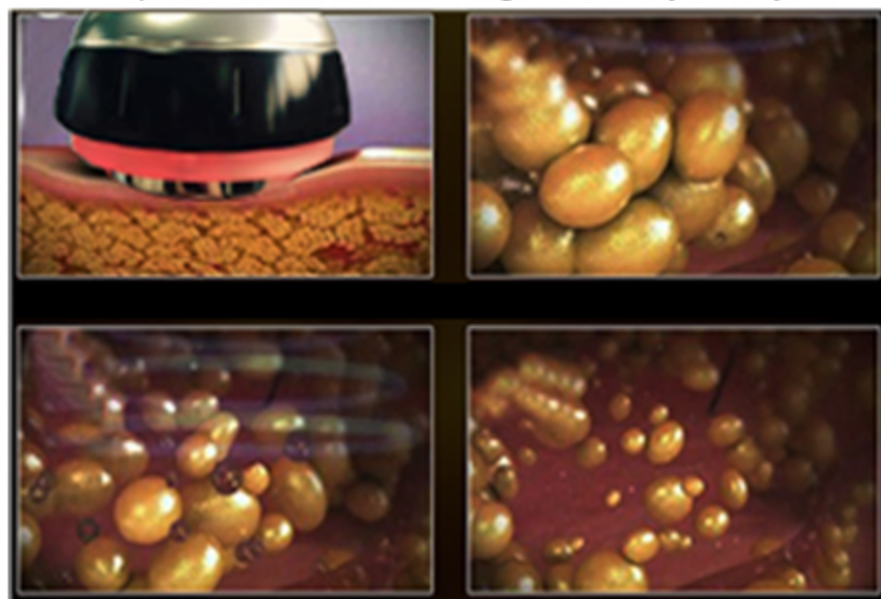


Fat cells are decomposed, liquefaction, and to retain the integrity of blood vessel, nerve, lymphatic vessels and fibrous tissue. The use of negative pressure suction, the fat liquefaction of fluid aspirated

Features: Safe, intelligence, digitizing calibrations simple treatment, Incredible Results long lifetime duration, save money and easy to use, no heat to burn skin, 2015 year the newest innovative technology

3D SMART RF technology featuring AUTOMATICALLY, INTELLIGENT, CONTROL SYSTEM that simultaneously RF energy output and Photon Therapy, with the ability to deliver safe and effective treatments with SUPERIOR treatment CONTROL, PRECISION TARGETING and BROADER DERMAL PENETRATION for maximum results in minimal time.

Bipolar SMART RF with Photon Probe for Eyes' Area

This RF face eyes' around treatment is aimed at further forcing the excretion of triglycerides and skin tightening with Bipolar 3D SMART RF & Photon. The treatment is both anti-aging and relaxing, resulting in long lasting youthful looks. The device significantly improves skin texture whilst reducing wrinkles and fine lines as well as creating a noticeable clinical effect from the very first treatment

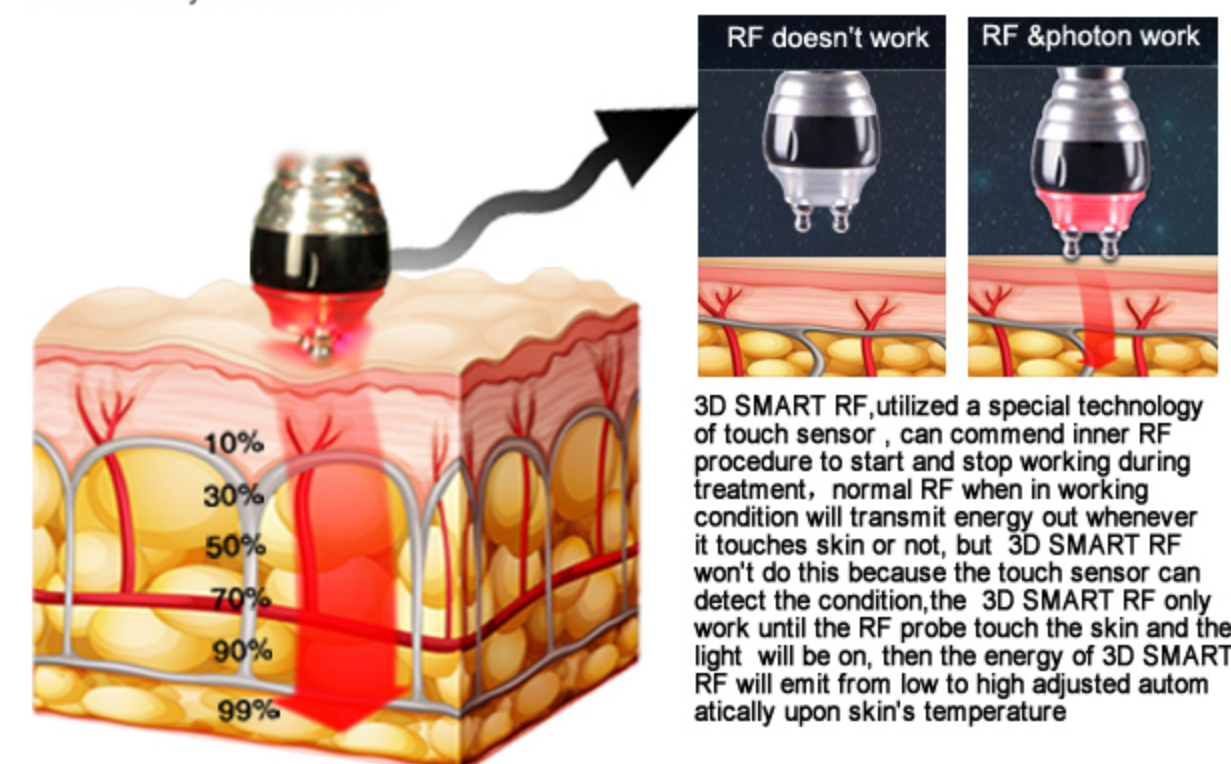

3D SMART RF, utilized a special technology of touch sensor, can commend inner RF procedure to start and stop working during treatment, normal RF when in working condition will transmit energy out whenever it touches skin or not, but 3D SMART RF won't do this because the touch sensor can detect the condition, the 3D SMART RF only work until the RF probe touch the skin and the light will be on, then the energy of 3D SMART RF will emit from low to high adjusted automatically upon skin's temperature

Quadrupole SMART RF with Photon for Facial Care

High-power 3D SMART Quadruple RF (radio frequency) currents preferentially and intensively heat the sub-dermis (where cellulite is located), exerting a triple slimming/tightening effect on the cellulite tissues. Specifically, these currents: Stimulate the fibroblasts to produce new collagen and elastic fibers, thereby offering a separate skin tightening effect. Stimulate the adipocytes to release fat into the bloodstream and physically damage a number of them (adipocytes are very "fragile" cells, in comparison to fibroblasts or other tissue cells).



Sexupolar SMART RF Vacuum with Photon Probe for Body Shape

It combines the mechanism of advanced 3D SMART RF energy and vacuum therapy and Photon that work with the body's natural regeneration process to deliver safe and efficient treatments for cellulite, body contouring and tightening of the skin.

the unique vacuum therapy and photon therapy maximizes the penetration of 3D SMART RF energy in the treatment area


and also increases local blood circulation, stimulates lymphatic drainage and assists in the shrinkage of fat cell volume.

Sextupole 3D SMART RF with vacuum & photon assists in the Metabolism of Large Fat Granules previously ruptured by cavitation in treatment of Unoisetion Cavitation 2.0 40K



Dermabrasion Microdermabrasion Skin Rejuvenation

Offer a high-quality treatment that exfoliates the skin and whisks away dead cells from the skin surface. The tips of YH-6601 was designed with a specific coarseness grade and tip size to allow a wide range of treatment options, allowing you to reach the crevices around the nose and hairline to exfoliating larger body surfaces such as the décolleté and buttocks.

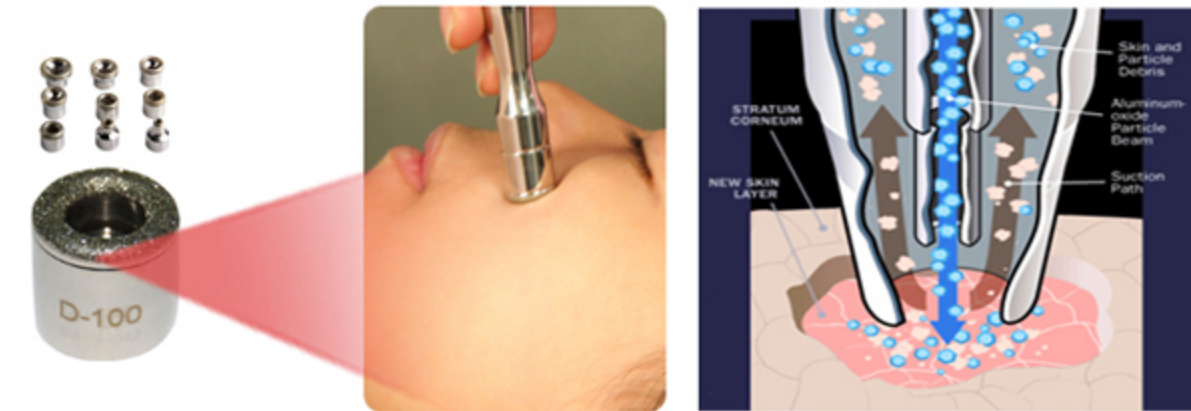

Vacuum Spray Diffuser for Cooling and Hydrating Effect

Spray diffuser mist function provides a light refreshing mist you can apply for a cooling and hydrating effect. Easy to use facial option



Parameters



Technical Parameters

- Power Supply Input	100V-240V
- Output power	190 W
- Uoisetion 40K Cavitation Head	
Frequency	40KHz
Power	15W
- Sextupolar Vacuum 3D Smart RF for Body w/Photon	
Frequeuncy	1MHz
Power	50W
Vacuum	<-80kPa
Air flow	>10L/minute
Noise level	<70dB (30cm away)
- Quadrupole 3D Smart RF w/Photon for Face	
Frequeuncy	1MHz
Power	30W
- 3D Smart Bipolar RF w/Photon for Eye Area	
Frequeuncy	1MHz
Power	10W
- Microdermabrasion Dermabrasion	
Power	45W
Vacuum	<-80kPa
Pressure	>250kPa
Air flow	>10L/minute
Noise level	<70dB (30cm away)
- Red Wavelength	650 - 730 nm

Package Listing

- 1 x Host
- 1 x Unoisetion Cavitation 2.0 with Photon Probe for Body Sculpture
- 1 x Sexutpolar SMART RF Vacuum with Photon Probe for Body Shape
- 1 x Bipolar SMART RF with Photon Probe for Eyes' Area
- 1 x Quadrupole SMART RF with Photon Probe for Facial Care
- 9 x Dermabrasion Tips
- 3 x Dermabrasion Wands

Operation

- 2 x Spray Bottles
- 2 x Spray Hose
- 1 X Power Supply Cord


1 Before operation

1.1 Please connect all parts well





1.3 Please click anywhere ,then u will enter all function screen

2 Unoisetion 4OK Cavitation

2.1 Please press  button then login unoisetion 4OK cavitation function display

2.2 Please click  to set working time


2.3 Please click  to increase or decrease the ENERGY


2.4 Please click M1  M2  to chose working model, M1 means continuous working, M2 means pulse working mode, please chose the most comfortable mode upon your feeling.

2.5 Please click  and  to start or pause function.





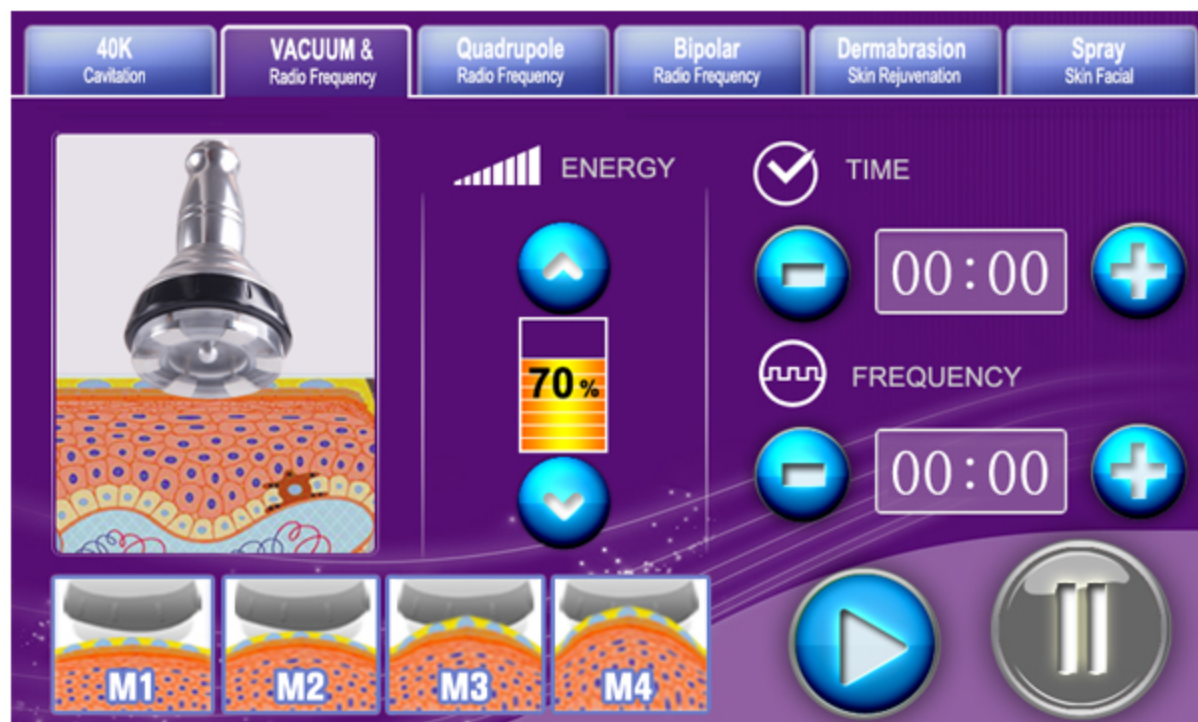



3 Sexupolar SMART RF Vacuum with Photon for Body Shape



3.1 Please press  button then login sextupolar SMART RF vacuum with photon for body shape

3.2 Please click  to set working time

3.3 Please click  and  to increase or decrease the ENERGY and the frequency

3.4 Please click M1  M2  M3  M4  to chose working model

M1  Massage Mode

Improving the fluidity and movement of cells, treat the relative disease like congestion and stagnation of blood

M2  Drainage Mode

Enlarge lymph vessels and veins. result in expelling excess water from tissue fiber

M3  Whitening Mode


Not only improve the blood circulation of capillary, but also the flow between the deeper veins and lymph node.


M4  Detox Mode



Expelling excess toxic inside body so strength the blood vessel, treat the varicosity

3.5 Please click  and  to start or pause function.

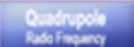
4 Quadrupole SMART RF with Photon for Facial Care

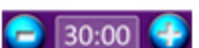4.1 Please press  button then login quadrupole SMART RF with photon for facial care display


4.2 Please click  to set working time


4.3 Please click  and  to increase or decrease the ENERGY and the frequency

4.4 Please click  and  to start or pause function.

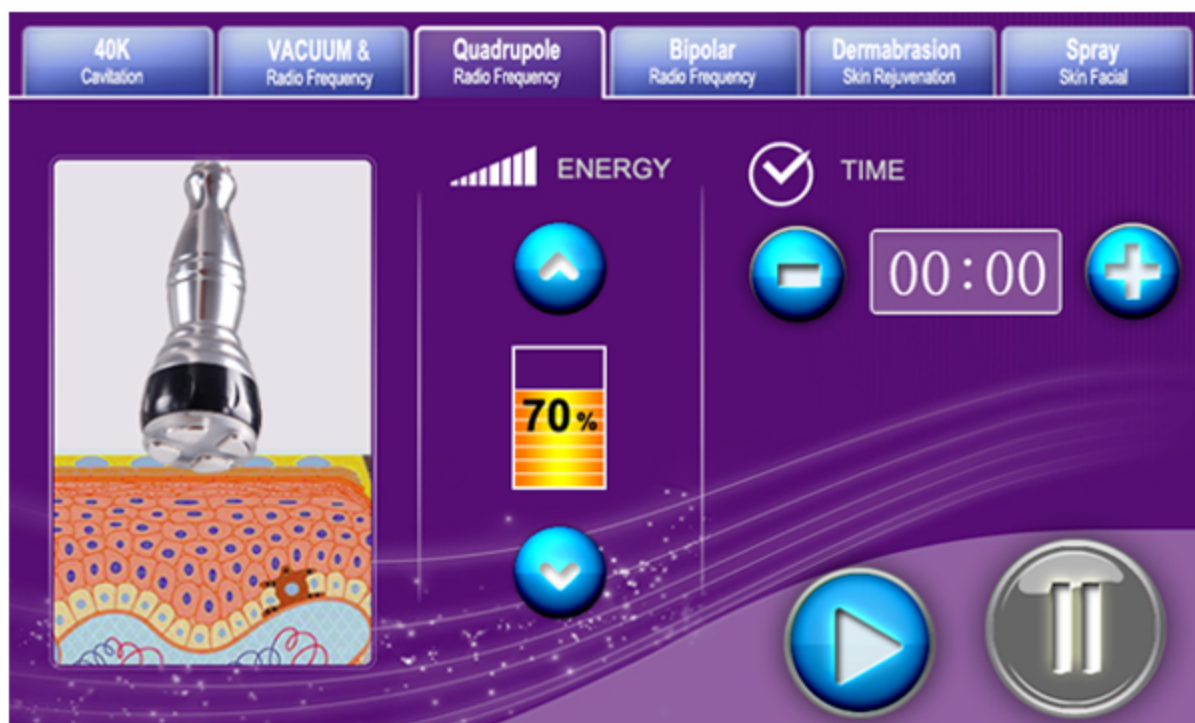


5 Bipolar SMART RF with Photon for Eyes' Area

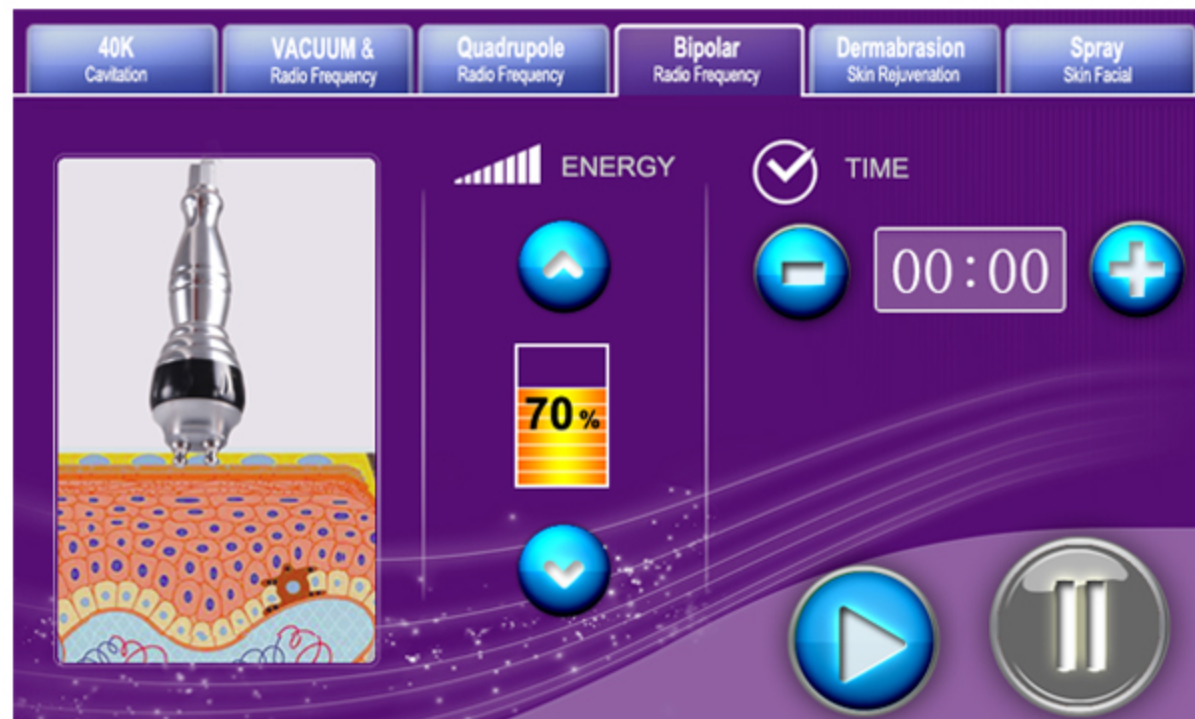


5.1 Please press  button then login bipolar SMART RF with photon for eyes' area screen


5.2 Please click  to set working time


5.3 Please click  to increase or decrease the ENERGY and the frequency

5.4 Please click  to start or pause function.


6 Dermabrasion Microdermabrasion Skin Rejuvenation

6.1 Please press  button then login quadrupole SMART RF with photon for facial care display

6.2 Please click  to set working time

6.3 Please adjust  inhale button from left to right(min to max) to set the intensity

6.4 Please click  to start or pause function

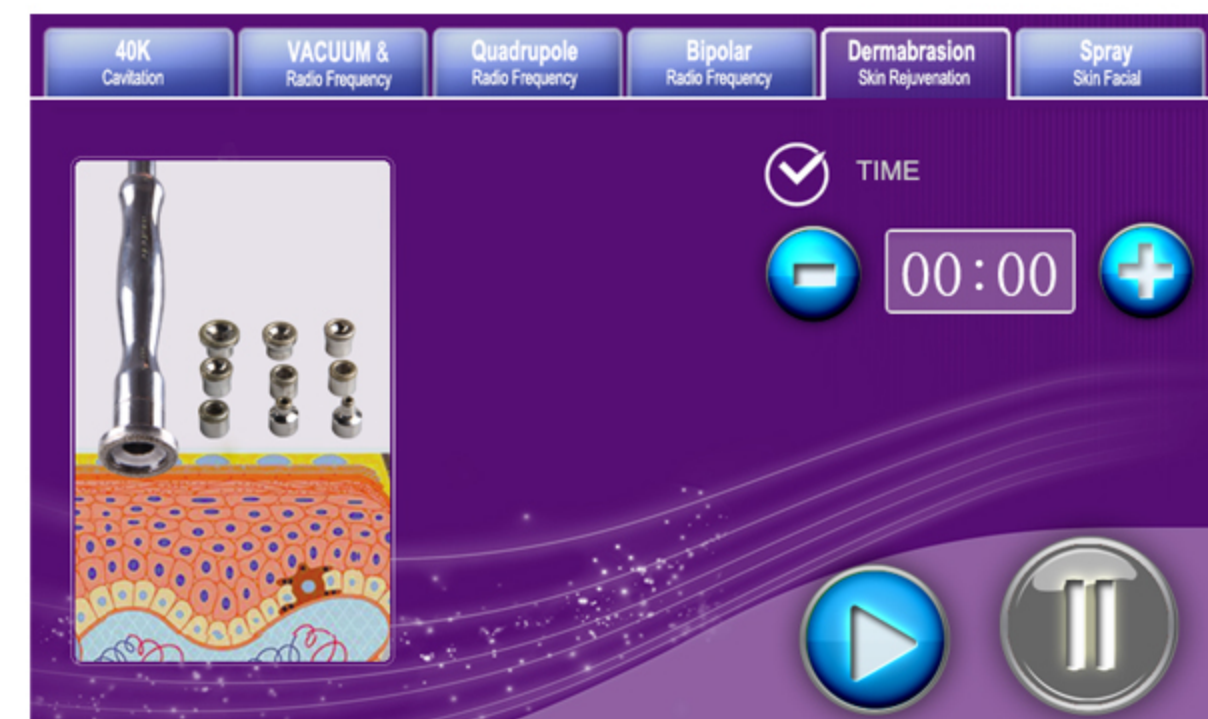

6.5 How to connect the derambrasion tips and cotton



Recommended Vacuum intensity on treatment Area

Treatment Area	Normal Skin(inHg)	Sensitive Skin(inHg)
Face	12-22	8-12
Forehead	12-22	10-12
Chest	15-20	15-18
Nose	10-15	10-14
Neck	18-18	10-15
Near Eye Area	8-12	8-10

Treatment Schedule Table

Treatment	Frequency(days)	Total Visits
Face Peeling	6-10	3-5
Fine Lines	7-13	6-8
Neck	12-14	3-5
Acne Scars	7-12	8-10
Hyperpigmentation	7-12	8-10

6.6 Place the diamond head on skin surface(start from forehead) proceed with a slow sliding motion,don't leave the diamond tip at one sport for too long,doing so will cause harm to the skin

6.7 Gently stretch the skin with thumb and finger and glide the hand piece 2-3 times over the area being treated,then cross again in different directions to prevent streaking lines as in the reference pictures(as below) brush strokes or polishing motions may be used in different sites

6.8 After treatment,brush off all remaining dead skin with a soft brush and rines with damp spones or dam cool towel

6.9 Allow skin to dry then apply desired skin product

Tips:after the completion of microdermabrasion,don't forget to protect your new skin.The new skin,which may appear pink for a day,is more prone to sun damage than was your old skin.It is recommended that you apply hydrating moisturize which is emollient-rich and has SPF 30 at least.You may also use oil such as almond oil or johoba oil once a day to ensure proper nourishment of your skin


To clean the diamond head, use toothbrush or cotton swab damped with a little alcohol to wipe off the dirty impurity and then use UV light or high temperature for sterilization

7 Vacuum Spray Diffuser for Cooling and Hydrating Effect

Make liquid product through a lance become a mist sprayed on the skin surface, absorbed through the skin microcirculation, treat various pruritus, sensitive skin, neurodermatitis, contact dermatitis, acne-prone skin



7.1 How to install the vacuum spray diffuser

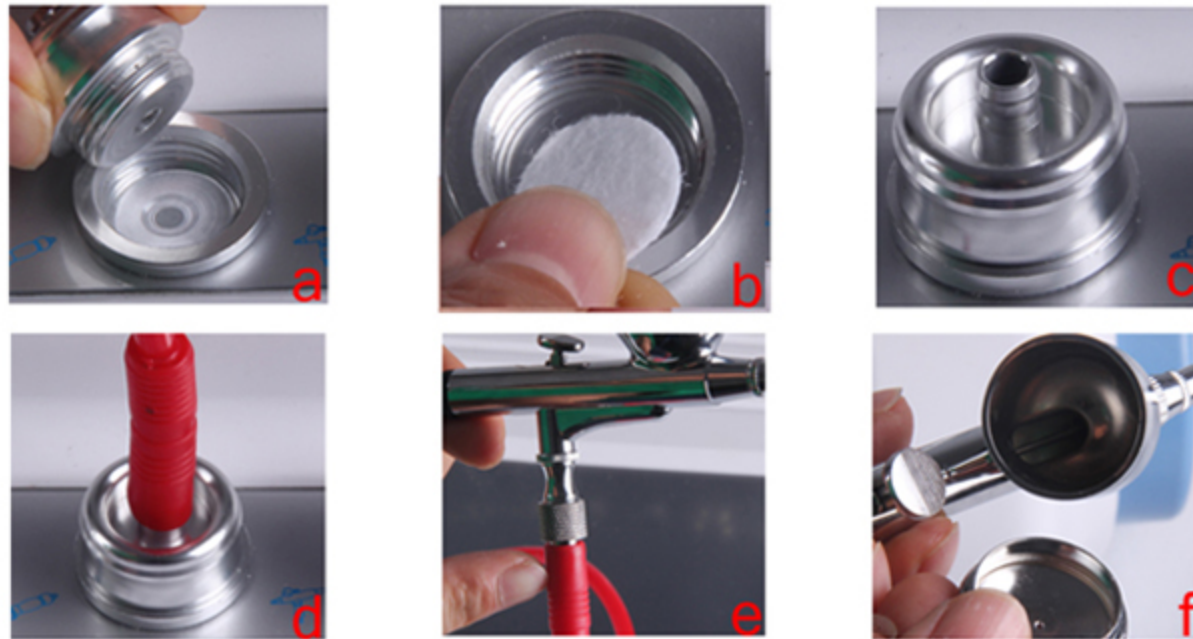

7.2 How to operate the spray diffuser




7.2.1 To circle the entire face in the form of spray

- a Put diluted solution into a solution of the cup;
- b Gently hold the gun;
- c Raised spray injection mode, solution will be ejected;
- d Press the control button to the rear position, the gun will spray oxygen and cream;
- e Keep a distance 5 cm between the skin and the spray gun;
- f For corner site, hold the gun and close to the skin surface to spray

Recommend treatment time 10 minutes/time

Before
vs
After

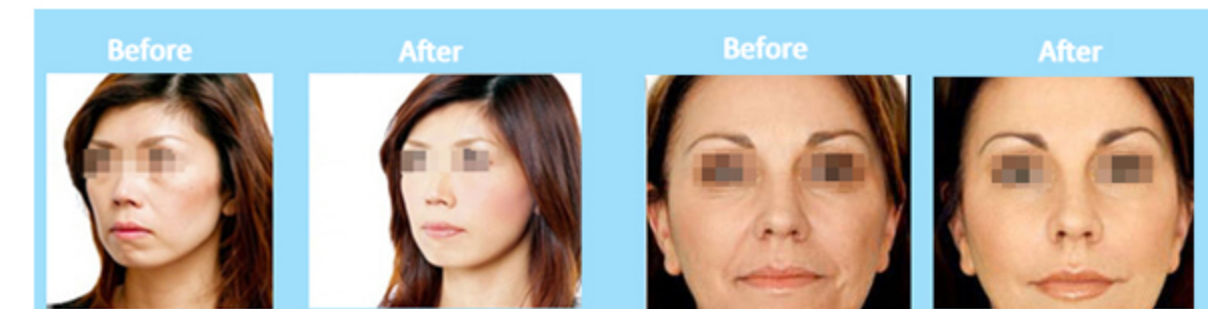

Skills



BUTTOCKS

ILLUSTRATION of OPERATION

FACE

Treated time: 30 minutes

1. Massage the mandible center in circles to produce deep heat.
2. Massage lower jaw in lines.



3. Massage the triangle zone of both sides of the face in circles.



4. Massage from jaw to angulus oris to ear in lines.

THIGH

Treated time: 30 minutes

1. From down to upper, pushing to the groin to dredge the lymph.



2. From down to upper, circling push by anticlockwise can help decomposing fatness.



3. Also can push by come-and-go to decompose fatness.



4. Pull from the knee and muscle texture to upper, can improve the curve.



Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.

3. Along the muscle of arm, pull up to lift and tight the muscle.




BACK

Treated time: 20 to 30 minutes


1. First by come-and-go, pushing the bladder nerve 2 to 3 times.



2. Circling by anticlockwise to stimulate the underarm lymph node.

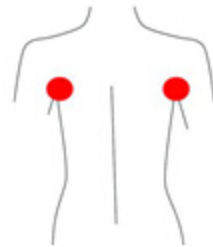

3. On the back, by anticlockwise circling, can help to decompose the fatness.

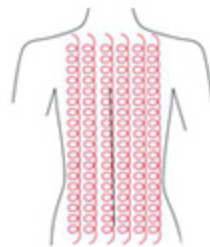

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help improving the back curve.



ARM

Treated time: 30 minutes

1. From the inner elbow to the armpit doing lymph drainage.



2. Circling to stimulate the lymph node.

3. By anticlockwise or come-and-go pushing, decomposing the fatness.




3. By anticlockwise or come-and-go pushing, decomposing the fatness.



Abdomen

Treated time: 20 to 30 minutes

1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.

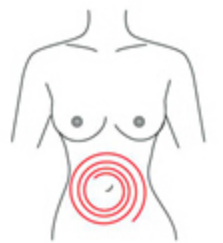

2. From small to large, by anticlockwise direction, circling push down.




3. From the belly and muscle texture, pull to the groin.



Notice

4. Through lymph drainage gesture, taking the toxin to the place of groin.

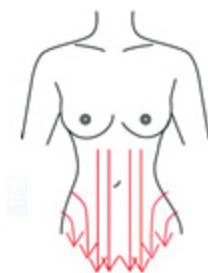

Notice:

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
7. If the machine will not use for a long time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.