

# User Manual MS-2183



# Content

Part I
Safety precautions
Contraindications
Overview
Functions of interfaces
Technical specifications
Precautions
Maintenance & Cleaning
FAQ
Ask & Answer
Operation guide
Warranty and support

## Safety precautions

1. Please read this manual carefully before use and familiarize yourself with the operation of this instrument and related technology.
2. This instrument is a high-precision electronic device that requires proper storage. Please avoid storing it in humid conditions, direct sunlight, or near inflammables and explosives to prevent damage.
3. Always use the designated accessory provided or recommended by the manufacturer to ensure the performance throughout its service life.
4. Do not place the instrument in high-temperature conditions, as this may negatively impact its service life and performance.
5. Do not apply the instrument on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pacemakers, etc.
6. Patients should refrain from using this instrument unless they have obtained permission from their doctor.

7. Ensure to apply the instrument on moist skin and avoid the operation on dry skin.
8. Power off the instrument when not in use to ensure its safety.
9. If you are currently taking weight-loss drugs, please discontinue their use 1 - 2 months before using this instrument. Failure to do so may result in an extended course of treatment.
10. To maintain hygiene and extend the instrument's service life, sterilize it with saline.

## Contraindications

**Before using this instrument, it is essential to consult your doctor or seek advice from professionals. Details as below:**

1. Pregnant or lactating woman.
2. Individuals with heart disease or a pacemaker.
3. Individuals with unhealed wounds or those in the recovery period from surgery.
4. Individuals suffering from epilepsy, severe diabetes and hyperthyroidism.
5. Individuals suffering from malignant tumors, hemophilia and hemorrhage.
6. Individuals with skin disease or infectious disease.
7. Individuals have had a cesarean section within six months, a miscarriage within three months, or a natural birth within two months.
8. People who had plastic surgery, with implants or metal objects at the treatment area in 3 months.

## Overview

With the most popular and safest technology, this vacuum beauty instrument ensures you achieve your desired body figure. It provides the perfect solution for people seeking improvements in skin appearance and body figure without injections, drugs, or side effects. For external use with immediate effects.

## Functions of interfaces



## How to replace the cotton filter



Connect the transparent hose



The long Y-shaped hose, connecting the vacuum cup.



The short Y-shaped hose, connecting the breast cup.



The straight hose, connecting the grease cup.

**How to connect the long/short Y-shaped hose:** connect one end of the hose to the inlet on the main unit, and connect another two ends with two vacuum cups/breast cups.

(In case only one cup is needed for the treatment, tie a knot on the spare end. And untie the knot if 2 cups are needed for the treatment.) )

**How to connect the straight hose:** connect one end of the hose to the inlet on the main unit, and connect another end with the grease cup.

1. Turn on the switch, the left one is suction hole, the right one is the outlet hole.



When the right switch is off, only suction function works.

2. Choose required mode by "Function choose knob".

Face rhythm 1: Intermittent suction and release, for face massage and relax.

Face rhythm 2: Relax mode, for face lifting and tightening.

Body rhythm 1: Detox mode, drain meridians.

Bodyrhythm2 : Dredge mode, unclog meridians.

B reastlifting: For breast and buttocks sculpting and firming.

Treatment: massage and relax mode. Used for relaxing whole body, or first-time user.



Rotary knob for adjusting suction intensity (turn left to decrease intensity, turn to right to increase intensity), (Only suction mode: Turn off the right switch, and set suction intensity based on individual's needs. )

# How to choose breast cup/vacuum cup/grease cup



large

middle

small

3 pairs of breast cup



9 pairs of vacuum cup



Cup No. 1 is suitable for individuals with larger-sized breasts who seek firmer and sculpted breasts.

Cup No. 2 is suitable for individuals with normal-sized breasts who desire firmer and more augmented breasts.

Cup No. 3 is suitable for individuals with smaller-sized breasts who want to enjoy a massage and relaxation.

Cup No. 4 is used for cupping therapy, moving cupping therapy (for large body area , e.g., thighs, buttocks, abdomen)

Cup No. 5 is used for cupping therapy, moving cupping therapy (for body areas like legs, back)

Cup No. 6 is used for cupping therapy, moving cupping therapy (for small body area, e.g., arms, lower legs)

Cup No. 7 is used for moving cupping therapy (for body areas like neck, arms)


Cup No. 8 is used for moving cupping therapy, massage (for body areas like face, neck)

Cup No. 9 is used for moving cupping therapy, cleansing (for face area)

Cup No. 10 is used for secret area.

Cup No. 11 is used for facial cleansing.

Cup No. 12 is used for moving cupping therapy, cleansing.

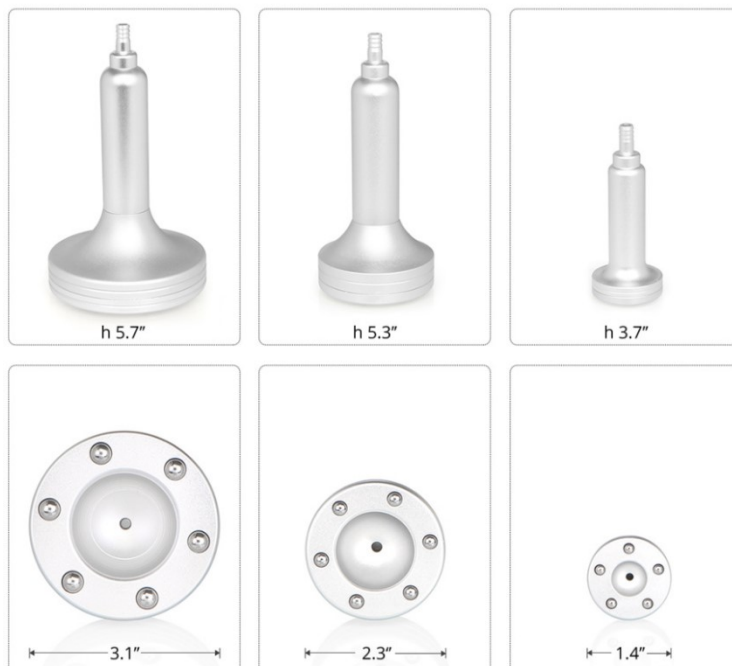

Select the appropriate grease cup from the three available sizes based on the size of the treatment area.



The large cup is used for large body areas, such as arms, abdomen, abdomen, waist, buttocks, legs, back.



The medium cup is used for face or parts of body area, such as arms, neck, face, lower legs.



The small cup is used for face area, such as cheeks, around eyes, forehead, chin, neck.

## Technical specifications



Power supply Input: 110v-220v

Net Weight: 3.30kg

Power: 15W

Gross Weight: 5.40kg

Dimensions: 30×30×15cm (40×35×28cm)

Input Power: 15W



length:2m



length:2m



length:2m

## Precautions

1. Strictly follow the instructions in the manual to use the instrument and train operator.
2. Before using the instrument, you must use a plug with a grounding pin and ensure that the power socket is grounded.
3. Make sure the voltage of the instrument is adapted. If the local power supply voltage is unstable, we advise you to add a power-matched regulated power supply between the mains and the instrument.
4. Remove all the metal objects inside the body to avoid causing accident or affecting treatment effect.
5. Do not use the instrument on empty abdomen, and take the treatment 1 hour later after the meal.
6. Start with the lowest intensity of vacuum.



# Maintenance & Cleaning

1. Unplug the power cord when not in use, store the instrument in dry, cool and airy place.
2. To clean the vacuum cup and grease cup, wipe them with a wet wipe or saline to clean and sterilize, then dry them with a dry towel.
3. After cleaning the instrument, cover it with a dustcloth or plastic wrap. Before use, ensure to sanitize the handle again for hygienic reason.
4. Disassembling the instrument and attempting to clean the inside is strictly prohibited. This task should only be performed by professionals.

## FAQ

### 1. Are there any side effects associated with vacuum breast enlargement treatment?

A: No side effect at all. The vacuum breast enlargement therapy is entirely based on the physical massage, suction and release under negative pressure. With the integration of multiple advanced technologies, it provides users with immediate effects without any side effects or rebound effects.


### 2. When do I expect to see the result of facial treatment with the vacuum cup?

A: Usually you will feel that your pores are purified, and your skin is getting smoother. It is recommended to take at least 1 treatment per week, and you will see more visible result day after another.

### 3. How does the negative pressure help in detoxification?

A: Negative pressure can make the capillaries congested, stimulate the cells to increase the vitality, the operation of inhalation and deflation, the increase and disappearance of negative pressure alternately, so that the local pores continue to open and close, promote skin breathing, increase the amount of oxygen absorption of the skin, and accelerate the elimination of waste. Therefore, it can move Qi and blood, dispel dampness by cold, dredge meridians and remove toxins and heat, etc.

## Ask & Answer

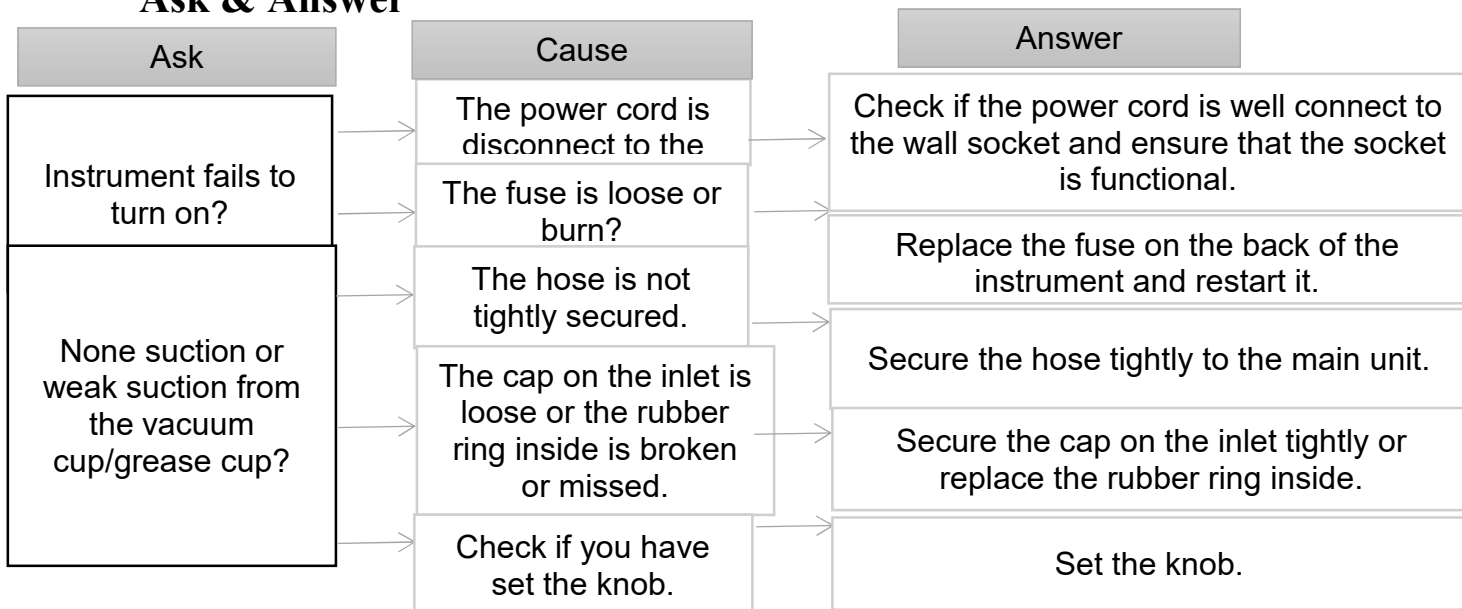


# Operation guide

**Breast enlargement (20 minutes)**




**Butt augmentation (Step 1: 15 minutes)**

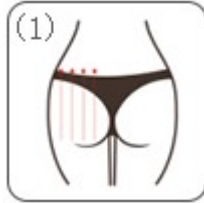


**( > Step2: 20 mins )**



**Face (Step1: 3mins )**

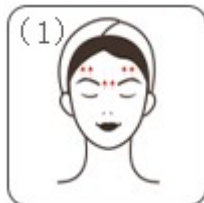


**( > Step2: 10 mins > Step 3 : 10 mins )**

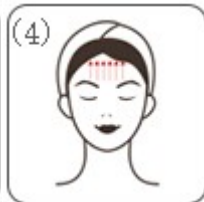


**Back (Step1:15 mins > Step2: 15 mins )**



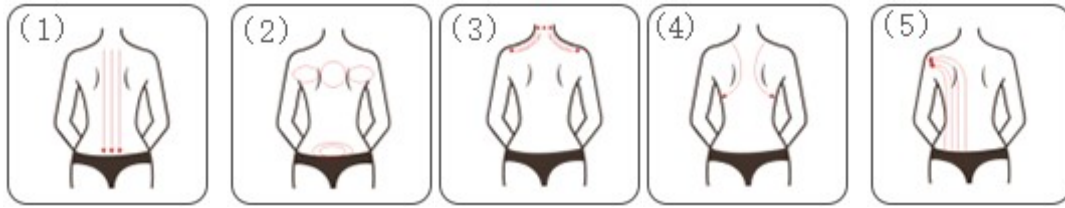

**Leg (Step1:15 mins > Step2: 15 mins )**

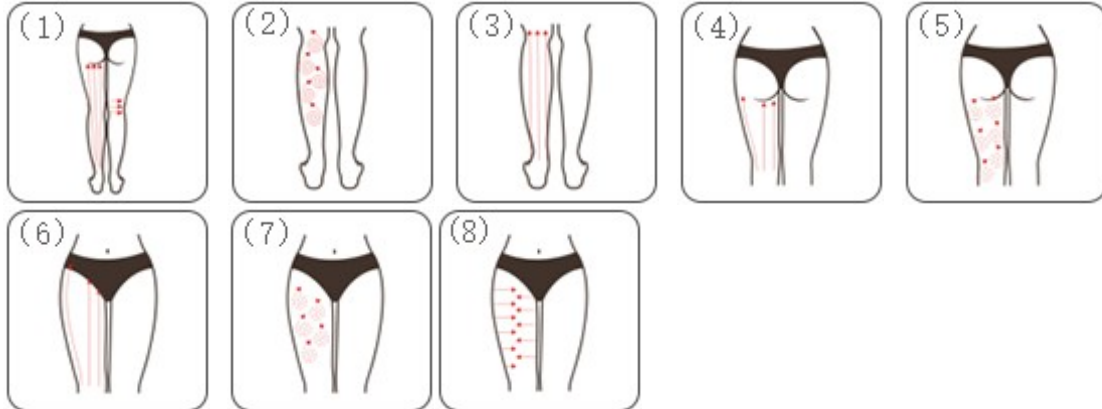

## Warranty and support

**A one-year warranty and a three-month free return and replacement is guaranteed from the date of purchase.**

Product name: Vacuum Therapy Instrument

Product model: MS-2183

Production date:

Quality Inspection:

