



MS-33F3

Instructions

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆Please follow the instruction to fit and operate the machine correctly.
- ◆◆Please do not remove and change any accessories of the machine.
- ◆◆It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆It forbids others to change and maintain the machine except the professional people.
- ◆◆Please contact us if there is any trouble during operation.

After Sale service

◆We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆How to contact us?

Phone:(+86) 189 -487-
55929 Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!

I. Function Introduction:

1. 40K Cavitation + RF

When the 40k cavitation breaks the fat cells, rf can heat up the fat at the same time, which can accelerate the blood circulation in the fat accumulative zone, then speed up burning the fat.

2. Vacuum + RF

When the vacuum open the gap between the cells, rf can heat up the fat at the same time, which can accelerate the blood circulation in the fat accumulative zone, then speed up burning the fat.

3. 40K Cavitation + Vacuum

By using ultrasound cavitation 2.0 and vacuum technologies combined, direct access is significantly increased to the deeper fat cells, bringing fat cells to a fast-active state from the ultrasonic waves and production of localised heat, so that effective blood circulation occurs.

This increased localised blood circulation within the dermis tightens sagging or loose aging skin. Your skin will have its elasticity improved aided by an increased metabolic activity. Then with the aid of the vacuum, the sweat glands and lymphatic system the excess fat and toxins are excreted out of the body. This vacuum suction and cavitation 2.0 further dissolves fat cells, strengthens the integrity of cells mitochondria, enhancing blood and lymph circulation with metabolism benefits.

II. Package list

- 1 x Host
- 1 x Vacuum+cavitation Head w/ vacuum
- 1 x Sextupolar 3D RF Head
- 1 x Quadrapolar 3D RF Head
- 1 x Holder
- 1 x Power Supply Cord

III. How to install the Accessories



Cavitation&Vacuum Body Slimming Head



Vacuum Filter



Key



Cavitation&RF Body Slimming Head



Strut



Shelf



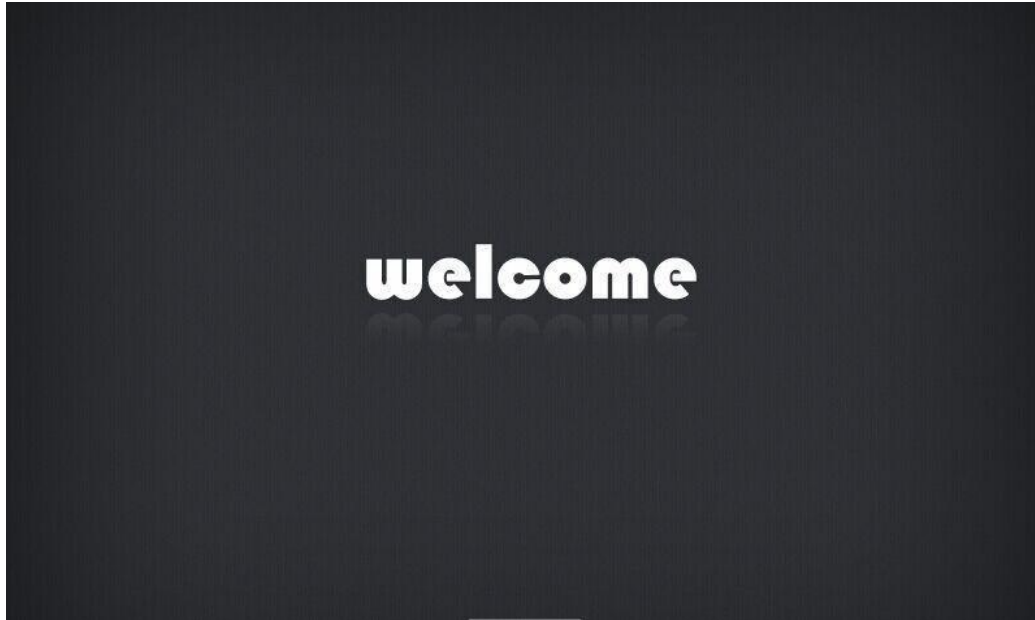
Vacuum w/RF Lymphatic Drainage Head



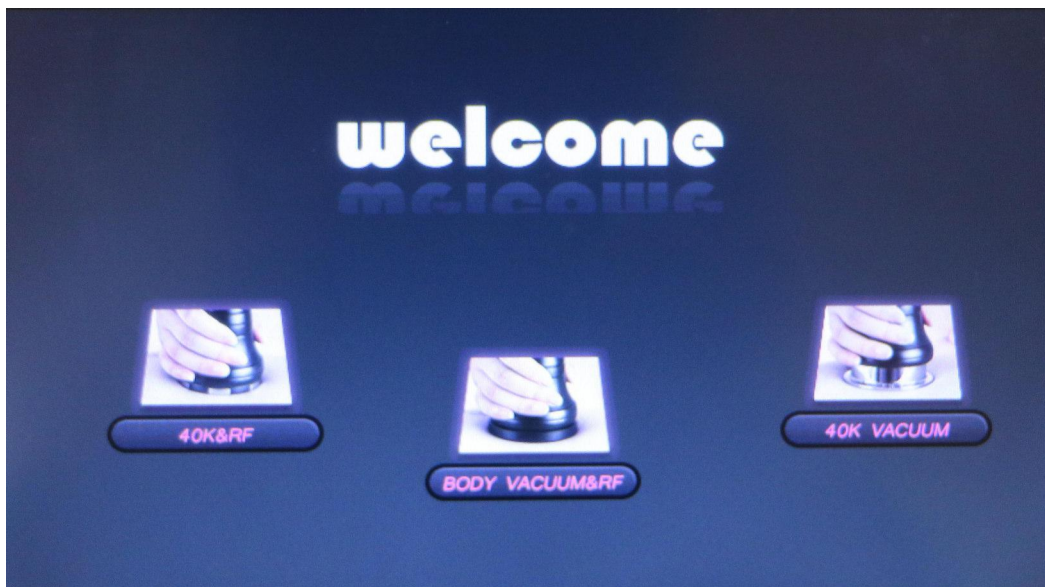
- ① Cavitation&RF Body Slimming Head
- ② Vacuum w/RF Lymphatic Drainage Head
- ③ Cavitation&Vacuum Body Slimming Head

IV. Treatment Introduction


A. Please install all the handles and press the power switch to start the machine, the screen will show "Welcome".



B. Click any place of the screen to enter into the main interface.

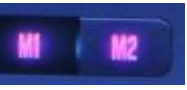



1. 40K Cavitation & RF

1.1. Press button  to enter into 40k cavitation function display as bellow:




1.2. Please  click to set working time.


1.3. Please click  to increase or decrease the energy of cavitation and RF.

1.4. Please click  to chose 40k cavitation working mode. M1 is continuous wave, M2 is pulse wave.

1.5. Please click  to start or pause function.


1.6. Please click  to come back to the main function interface.

2. Body Vacuum & RF

2.1 Press  button to enter into Body RF Vacuum operation display as bellow:




2.2 Please click  to set working time.

2.3. Please click  to increase or decrease the RF energy.


2.4. Please click  and  to set the time for the suction and release.

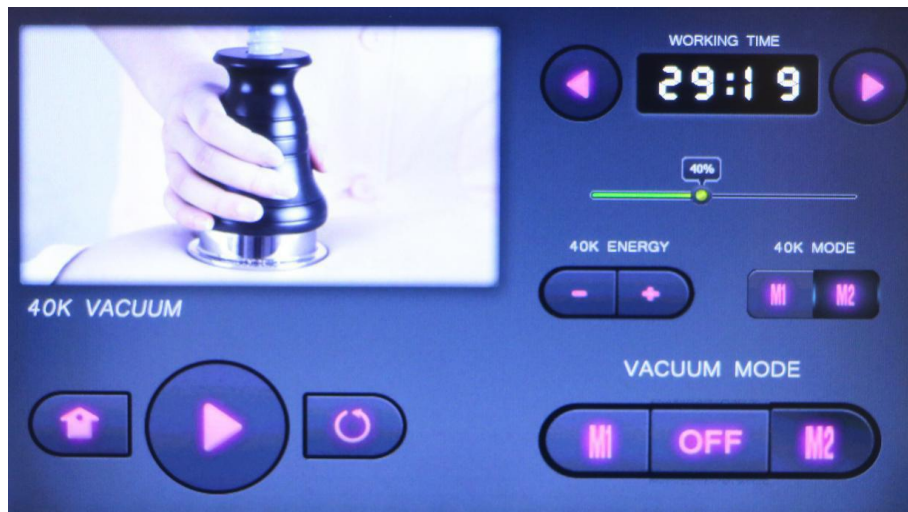
2.5. Please click  M1, M2, M3, M4, M5 to choose the Vacuum mode.

2.6. Please click  to start or pause function.

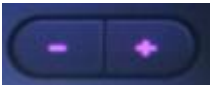
2.7. Please click  to come back to the main function interface.


3. 40K Cavitation & Vacuum

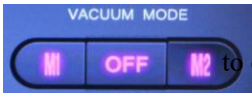
3.1 Press  button to enter into Body 40K cavitation vacuum operation display as below:



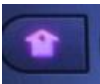
3.2 Please click  to set working time.

3.3 Please click  to increase or decrease the 40k cavitation energy.

3.4 Please click  to set the cavitation mode. M1 is continuous wave, M2 is pulse wave.

3.5. Please click  to choose the cavuum mode. M1 is continuous suction; M2 is pulse for suction and release

3.6 Please click  to start or pause function.

3.7 Please click  to come back to the main function interface.

V. Attention:

Never operate this equipment on a person has or suffers from the following medical conditions described below. (The salon or operator may consider having a client sign a waiver to the effect that he/she is free of any poor health conditions.)

1. Heart Disease
2. Heart Murmur or Irregular Heart Rate
3. Has a Pacemaker or other type of heart rate balancer.

4. Pregnancy
5. Epilepsy
6. Alcoholism
7. Ulcers
8. Disease of Stomach, Intestine, Liver, or Pancreas.
9. Breast Cancer.
10. Undergoing treatment for any form of Cancer.
11. Had major surgery within the past year.
12. Asthma
13. Cold, Flu, or Upset Stomach

VI. Do not use on these areas:

1. Sensitive Skin
2. Skin Rash
3. Damaged, Inflamed, or Infected Skin.
4. Surgical scar less than 12 months old.
5. Skin Swollen.
6. The probes can not point to Eye ball.
7. Genitals.