

User Manual

MS-33J6



Preface

To our valued users,

Thanks for choosing our latest integrated beauty machine. It's the latest skin cleansing and rejuvenation product with integration technology, which combines together positive & negative ions, high frequency, low pressure jetting and ultrasound technology. MS-33J6 is a professional beauty machine which needs to be operated by people with professional training. Any improper use of this machine may bring adverse consequence to human body. Therefore, we advise all the people to read this manual thoroughly and strictly follow the instructions specified hereby before operation.

We believe our quality product will offer you good returns, and our perfect after-sales service will make you worry-free.

Thank you and best regards.

Table of Contents

Preface
Part I
Brief Introduction
Advantages
Skin Cleansing & Acne Removing
Eyes Wrinkle Removal
Head Therapy
Part II
Detailed Operations
Technical Parameters
Contraindications
Precautions for Use
Troubleshooting & Solutions
FAQs
Packing List
Operational Diagrams

Part I

Brief Introduction

Instruments, working on appearance improving and anti-aging, with positive and negative ions, high frequency, low pressure jetting and ultrasound technology are currently the most popular and most extensively used. This kind of instrument can effectively replace all the facial care programs in beauty salons, and can be operated easily, conveniently and swiftly. It solves skin problems for people pursuing beauty. This instrument works safely and effectively, and requires no skin breaking, no injection, no medication, no operation, and without side-effects. It's used externally during the whole process and has an immediate effect.

Advantages

1. 4-in-1 multifunctional beauty machine that quickly fix facial care problems.
2. It's has multiple operating heads, which can be changed according to different demands.
3. It suits all kinds of skin, and it can improve skin and flatten fine lines.
4. Comfortable, painless, and non-invasive during the process, and there is no repairing period, and you can leave immediately after treatment, and it will not affect your work and life.
5. No consumption, and with low cost and quick returns.
6. It has a wider treatment range, and with an instant and obvious effect.
7. Unevenness, bleeding, and swelling will not occur after treatment.
8. Special-designed probe can effectively improve facial skin problems and restore your clean, tender and white skin.

Skin Cleansing & Rejuvenation

Principle

Electric Face Washing Brush

The face washing brush adopts skin penetration technology with ultrasonic pulse. Its working principle is acting directly on tiny brush by using ultrasonic wave, making it vibrate and exerting the effect on tiny pores on face so as to impose high frequency vibration on pores and achieve the effects of pores cleansing and facial muscles massaging.

Positive & Negative Ion Handle

Positive and negative ion probe adopts shift theory of positive and negative ion. With the same electrical charge contradicts each other, and the opposite electrical charge attracts each other. Electrons are all flow from negative charge to positive charge. When is set to "+", it's export which can be used for skin cleaning, while when is set to "-", it's import which can be used for skin maintenance and nutrient absorption.

Transparent Tube Head with High Frequency

Electro-therapeutic apparatus with high frequency contacts skin through high frequency micro-current generated by glass electrode tube and after have the power on. Micro-current in glass electrode tube ionizes oxygen in the air producing ozone, which will improve blood circulation and exert physical therapy on the skin.

Spray Bottle

Extracting oxygen with a concentration of only 21% of the air by using the principle of low pressure jetting. The oxygen, passing through a molecular filter made with special technology, will be compressed into oxygen active factor with a purity over 95% which then will be released under a pressure of 0.2 Pa. It turns aqueous nutrients into nebulosus oxygen which will be transmitted to skin stratum basale, which quickly opens the channel for skin absorption, kills anaerobion in the skin, activates cells, metabolizes old and useless cutin, deeply and densely replenishes moisture, shrinks pores, and clean the deep skin.

Effects

1. Clean deep skin and dredge pores.
2. Balance cutin and improve oil and fat secretion.
3. Induct nutrition into deep skin and speed up absorption.
4. Improve coarse and large pores.
5. Remove black head, lessen acnes, and improve strawberry-like nose.
6. Accelerate blood circulation and increase skin metabolism.
7. Brighten skin color and make pores smaller.
8. Replenish water and supplement nutrition.

Indications

1. People with exuberant oil and fat secretion.
2. People with thick corneum and clogged pores.
3. People with black head, acnes, and heavy pore obstruction.
4. People with coarse and rough pores.
5. People with poor skin absorption and metabolism.
6. People who always wear makeup and who don't always do skin cleaning.
7. People with uneven, dim and lackluster skin color.
8. people with dry skin and who apply masks without effects.

Contraindications

1. People who have severely sensitive skin or in allergic period.
2. People whose nose has prosthesis, and with extensive trauma on the face.
3. People in pregnancy, with serious heart disease, with severe Three Highs (namely hypertension, hyperlipidemia and hyperglycemia), with serious diseases and etc.
4. People with skin diseases and infectious diseases.
5. People who just had mesotherapy, microneedling, and with trauma on the skin.

Notes (after treatment)

1. Replenish water and protect yourself from the sun.
2. Don't use irritative skin care products within a week, such as exfoliants, exfoliator, AHA (Alpha Hydroxyl Acid) and etc.
3. Avoid staying up late, smoking, and drinking alcohol, and eat more fruits and light food.
4. Apply moisturizing facial mask three days in a row.
5. Please wash the face with warm water within three days.

Eyes Wrinkle Removal

Principle

Positive & Negative Ion Handle

Positive and negative ion probe adopts shift theory of positive and negative ion. With the same electrical charge contradicts each other, and the opposite electrical charge attracts each other. Electrons are all flow from negative charge to positive charge. When is set to "+", it's export which can be used for skin cleaning, while when is set to "-", it's import which can be used for skin maintenance and nutrient absorption.

Effects

1. Relieve eye fatigue, black eyes, eye bags and edema.
2. Improve periorbital lines and crow's feet.
3. Fade pigment and accelerate blood circulation.
4. Accelerate metabolism and prevent pigment from sediment.
5. Replenish nutrition for eyes and moisten skin.
6. Tighten skin and make it tender, and lift the corner of eyes.
7. Accelerate blood circulation of the eyes, and help with the efficient absorption.

Indications

1. People with wrinkles, fine lines, eye bags and black eyes.
2. People with dry skin, dry lines and false wrinkles.
3. People whose eyes are prone to fatigues and dryness.
4. People who always face computer and cellphone.
5. People who always stay up late and with black eyes.
6. People who always exposure to dry or high temperature environment.

Contraindications

1. People who just undergone plastic surgery.
2. People with hypertension, heart disease, diabetes, severe thyroid, malignant tumors and etc.
3. People in allergic period, or have severely sensitive skin or are allergic to metals.
4. People with skin trauma or wound.
5. People who are overly aging.
6. People in pregnancy or surgical recovery.
7. People with skin diseases and infectious diseases.

Notes (after treatment)

1. Avoid being under the blazing sun and protect yourself from the sun.
2. Do hydrating to maintain moisture and protect yourself from the sun because the treated parts are relatively dry.
3. It's better not to use alcohol, AHA (Alpha Hydroxyl Acid) or scrub cream within 1 to 3 days.
4. Avoid washing with overheated water, or enjoying hot springs and saunas, or doing strenuous exercise within 7 days.
5. Drink more water and apply eye masks which should do at least 3 times in a week.

Head Therapy

Principle

Transparent Tube Head with High Frequency

Electro-therapeutic apparatus with high frequency contacts skin through high frequency micro-current generated by glass electrode tube and after have the power on. Micro-current in glass electrode tube ionizes oxygen in the air producing ozone, which will improve blood circulation and exert physical therapy on the skin.

Effects

1. Improve qi and blood, and accelerate qi and blood circulation of the head.
2. Relax one's mind, and improve one's mental state.
3. Lessen dandruff, and prevent hair loss or greasy hair.
4. Improve hair quality, and prevent gray hair.
5. Improve insomnia and dreaminess, and dizziness and headache.
6. Prevent cerebrovascular disease.

Indications

1. People with poor sleep quality, and who always have insomnia.
2. People with dizziness and headache, and insomnia and dreaminess.
3. People with bad mental state, and poor memory, and who always engage in brainwork.
4. People who always sit and with his eyes staring at something.
5. People who always drive.
6. People who highly rely on cellphone.
7. People who always have hair loss, and with poor hair quality.
8. People who always feel dizzy and giddy, and experience tinnitus.

Contraindications

1. People with abnormally sensitive skin.
2. People with weak physique.
3. People with festered and infected epicranium, and with contact skin infectious diseases.
4. People in pregnancy, with severe heart disease, with serious Three Highs (namely hypertension, hyperlipidemia and hyperglycemia), serious diseases and etc.
5. People who are overly aging.
6. People in surgical recovery.
7. People with metal implants inside the body, such as stent, pacemaker, and etc., and who are allergic to metals.

Notes (after head therapy)

1. It's prohibited to stand up immediately.
2. Don't wash the face and head within 6 hours.
3. Avoid catching a cold and having electric fan and AC blowing exactly at the person.

Part II

1. Detailed Operations


Check the instrument and make sure it's properly connected. The following initial page will appear after having the power switch on.



1.1 Functions Selection



Modes Selection

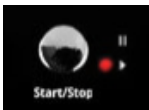

Start/ Pause



Time Setting



Energy Level Setting & Display

1.2 Detailed Operations for Electric Face Washing Brush



Install the proper alternative face washing brush, and apply cleansing mousse on the face using five points method. Have the brush head touched water and left it half-dry, and then press the switch on the handle to operate.

1.3 Detailed Operations for Positive & Negative Operating Head



Select the proper operating head

and install it on the handle

(as

picture).



Set the Proper Time



Set the Proper Energy Level



Click Start

Then select negative ion"-"





Set the Proper Time



Set the Proper Energy Level



Click Start

The Functions for Different Ion Operating Heads

Round Ion Operating Head: suitable for eyes import

Flat Ion Operating Head: suitable for face import

Roller-shaped Ion Operating Head: suitable for lifting and tightening

Straight Ion Operating Head: suitable for nasolabial folds and glabellum wrinkles and folds removal

1.4 Detailed Operations for Transparent Tube with High Frequency



Select the proper operating head and install it on the



handle (as picture).

Press Start on the handle to operate.

The Functions for Different Transparent Tube Heads

Comb-shaped Tube Head: used for head, suitable for people with insomnia and dreaminess, and wanting to improve hair quality.

Curved-shape Tube Head: used for sterilization and acne removal of nasal fossa.

Tongue-shaped Tube Head: used for face, diminishing inflammation and removing acne on small area.

Mushroom-shaped Tube Head: used for face, diminishing inflammation and removing acne on big area.

1.5 Detailed Operations for Spray Bottle



Click "Mode" and select spray bottle



Set the Proper Time



Click Start to Operate

2. Technical Parameters

3. Contraindications

People with the following situations should use the instrument with caution. Please consult a doctor or professional before using this instrument. The details are as follow:

- (1) Women in pregnancy, or lactation;
- (2) People with heart disease or with heart pacemaker;
- (3) People with unhealed surgical wound or in surgical recovery;
- (4) People with epilepsy, hypertension, severe diabetes, and hyperthyroidism;
- (5) People with malignant tumors, hemophiliac or severe bleeding;
- (6) People with skin diseases and infectious diseases should use with caution;
- (7) People who have metal implants inside the body, such as stent, pacemaker, and are allergic to metals.

4. Precautions for Use

- (1) Plug with ground pin must be used, and ensure power socket is already factually grounded before using the instrument.
- (2) To ensure voltage of the instrument is adaptive. If the voltage of local power supply is unstable, we suggest users add voltage regulator with matching power between the mains supply and the instrument.
- (3) To guarantee the curative effect and normal service life of the instrument, please uniformly use specified parts provided or suggested by original manufacturer.
- (4) The instrument can't be placed in damp places or near water sources, and also can't be exposed to sunlight directly.
- (5) Don't place the instrument near a strong heat source since this may affect its service life and normal use.
- (6) Please remove all the metal objects from the body before treatment so as to avoid unexpected situations which may affect the curative effect.
- (7) Please restrain from using the instrument aiming at eyes, thyroid, parathyroid, testicles, abdomen of pregnant woman, pacemaker, and etc.
- (8) People who are suffering from illness should use with caution unless he gets permission from a doctor.
- (9) Please turn off the power switch of the instrument if no one uses it, and ensure the main power is off after someone used it and before he left, thus the safety of the instrument can be guaranteed.
- (10) Using the instrument and training the operators in strictly accordance with the instructions

specified in the manual.

- (11) Start with the lowest energy level when just operated, and increase the level gradually after the client adjusting himself to it.
- (12) It's prohibited to use the instrument on the rim of eyes, ears, the periphery of parts that have skin diseases, and parts that have plastic surgery.
- (13) Clean the instrument with normal saline after operation so as to ensure its cleanliness and hygiene, as well as extending its service life.

5. Troubleshooting & Solutions

- (1) The instrument cannot be started, and the lights on its back is not working?
 - A. Make sure the power cord is plugged into a good power socket.
 - B. Check whether the fuse tube on the back of the instrument is loose or burnt out.

- (2) The instrument can be started, but there is error message on the monitor?
 - A. Take out the plug in the back of the instrument, and wait for about 1 minute and re-plug and restart it.
 - B. If the above-mentioned solutions are not working, please contact the instrument dealer for assistance.

- (3) The handle is not working?
 - A. Check whether the start button is on.
 - B. If the handle is broken, then you have to contact the manufacturer for repair or reorder.

- (4) The probe is without output?
 - A. Check whether the handle is correctly installed. If no, install it correctly and then operate it.
 - B. If the operating head is damaged, then replace a new one.
 - C. Check whether the power cord is connected, and whether the switch or the switch on the handle is on.

6. FAQs

- (1) Q: Is there any dependency for this treatment?
A: There is no dependency. It's target at skin problems and help it improve, lift and tighten. It maintains and prevents the skin from having problems caused by other factors. It helps the skin absorb nutrients, solve yellowish skin and lack of moisture, as well as rough skin. If quit in later stage, the skin will only back to previous conditions of secretion and senility, and it will not cause dependency.

- (2) Q: Can I put on makeup after the treatment?
A: You can wear makeup without questions. The skin will become rejuvenated and plump. It's more appropriate to put on makeup at this moment. It's a kind of care that without broken skin or wound, and it's safe, healthy, fast and effective.

- (3) Q: How long does it take to see the effects?
A: a-It has an instant effect after the treatment. Different kinds of skin can have different levels of effects.
b-Level of expertise of the operator: It needs people with high level of expertise to operate whatever the instruments are. People with low level of expertise will bring a low


level of effects.

c-Stick to the treatment. No matter what and whatever the instruments are, you need stick to it rather than just do it for one time and hope to maintain the effects for a long time. After all, different persons have different skin, and likewise different skin and whether you stick to it or not can achieve different effects.

7. Packing List

- Main Machine x1
- Round Ion Operating Head x2
- Roller-shaped Ion Operating Head x2
- Straight Ion Operating Head x2
- Flat Ion Operating Head x2
- Comb-shaped Tube Head x1
- Curved-shape Tube Head x1
- Tongue-shaped Tube Head x1
- Mushroom-shaped Tube Head x1
- Spray Bottle x1
- Ion Handle x2
- High Frequency Handle x1
- Electric Face Washing Brush x1
- Alternative Brush Head x6
- Power Cord x1
- Fuse x2
- Connecting Wire for Spray Bottle x1

8. Operational Diagrams

Parameters Setting	Product	Techniques	Diagrams
Skin Cleansing & Acne Removing: 60 Minutes/ Once a Week			
Electric Face Washing Brush: Advised time: 3 to 5 minutes Positive & Negative Ion: Advised time: 10 to 15 minutes Advised energy	Makeup Remover + Facial Cleanser + Toner + Essence + Facial Mask + Instrument	1. Remove makeup, 2 minutes. 2. Clean pores with electric face washing brush. Apply cleansing mousse using five points method, and clean the whole face with hands moving circlewise, and do it for 3 to 5 minutes (use cleaning mousse). 3. Apply educe liquid and use hot steam (apply the liquid on parts with large pores or with	Technique 9, 18, 23  Technique 2, 7, 16, 21

level: between 3 and 7.

Transparent Tube with High Frequency

Advised time: 10 to 15 minutes

Spray Bottle

Advised time: 5 to 8 minutes.

severe clog. Cheeks can be applied with toner, and face with sensitive skin can be applied with toner and used with cold steam. Hot steam and educe liquid is not advised.), 5 to 8 minutes.

4. Get rid of the ripe acne with acne pin.

5. Wipe the face with normal saline.

6. **Techniques for Transparent Tube with High Frequency:** Select tongue-shaped tube head, and press the switch on the handle to start.

7. Starting with chin, moving upwards and circlewise, and moving one line after another. Repeat 3 times.

8. Move circlewise from forehead towards hairline. Repeat 3 times. (between the eyebrows can be exercised with more treatment. Repeat 1 to 2 times.)

9. Starting with chin, lifting upwards, and lifting one line after another. Repeat 3 times.

10. Lift from forehead towards hairline. Repeat 3 times. (between the eyebrows can be exercised with more treatment. Repeat 1 to 2 times.)

11. Treat the other side.

12. Choose curved-shape tube head, and press the switch on the handle to start.

13. Techniques: Treat the nasal fossa with stamp method.

14. Select mushroom-shaped tube head, and treat the whole face with stamp method. Repeat 3 to 5 times.

15. **Techniques for Positive & Negative Ion:** Choose straight Ion probe. Set energy parameters and start to operate.



Technique 2, 8, 17, 22







		<p>16. Starting with chin, moving upwards in circles, and moving one line after another. Repeat 3 times.</p> <p>17. Move circlewise from forehead towards hairline. Repeat 3 times. (between the eyebrows can be exercised with more treatment. Repeat 1 to 2 times.)</p> <p>18. Techniques: Starting with chin, lifting upwards, and lifting one line after another. Repeat 3 times.</p> <p>19. Lift from forehead towards hairline. Repeat 3 times. (between the eyebrows can be exercised with more treatment. Repeat 1 to 2 times.)</p> <p>20. Select roller-shaped Ion head, and set energy parameters and start to operate.</p> <p>21. Starting with chin, moving upwards in circles, and moving one line after another. Repeat 3 times.</p> <p>22. Lift from forehead towards hairline. Repeat 3 times. (between the eyebrows can be exercised with more treatment. Repeat 1 to 2 times.)</p> <p>23. Techniques: Starting with chin, lifting upwards, and lifting one line after another. Repeat 3 times.</p> <p>24. Lift from forehead towards hairline. Repeat 3 times. (between the eyebrows can be exercised with more treatment. Repeat 1 to 2 times.)</p> <p>25. Select straight Ion head, and exercise more treatment with nasolabial folds and forehead and glabella wrinkles and folds.</p> <p>26. Spray Bottle Techniques: Set the time to 5 to 8 minutes, and click start to operate.</p>	
--	--	--	--

		<p>27. Apply facial mask, and wait for 15 minutes.</p> <p>28. Remove the mask and clean the face, 2 minutes.</p> <p>29. Apply toner, essence and facial cream to prevent from the sun.</p> <p>30. Treatment done.</p>	
--	--	--	--

Course of treatment recommended:
 Ten treatments is a course. After one treatment, the pores will become clean that is visible, and the skin replenished with enough moisture. After one course, the skin color will become bright little by little, and the pores will be improved. After three courses, the skin will become plump and ruddy, and the water in the skin can be locked as long as 24 hours, which make the skin turn rejuvenated, bright and elastic.

Eyes Wrinkle Removal: 45 Minutes/ 2 to 3 Times a Week

<p>Positive & Negative Ion Advised time: 10 minutes Advised energy level: between 3 and 7.</p>	<p>Cleansing Oil + Facial Cleanser + Essence/Eye Cream + Instrument + Eye Mask</p>	<ol style="list-style-type: none"> 1. Remove make-up and clean the face, 5 minutes. 2. Apply toner, 1 minute. 3. Apply essence (eye cream) evenly on the eyes with hands moving in circles. Repeat 3 times. 4. Techniques for Positive & Negative Ion: Choose round Ion probe, and set energy parameters, and click to start. 5. Lift from lower eye lid moving in circles to Temple. Repeat 3 to 6 times. 6. Move the instrument circlewise from brow head to brow tail. Repeat 3 to 6 times. 7. Lift the instrument from the lower eye lid to Temple. Repeat 3 to 6 times. 8. Lift the instrument from brow ridge towards hairline direction. 9. Clean the eye, 1 minute. 	<p>Technique 3</p>  <hr/> <p>Technique 5</p>  <hr/> <p>Technique 7</p>  <hr/> <p>Technique 8</p>
---	--	---	---

		<p>10. Apply eye mask, and wait for 15 minutes.</p> <p>11. Remove the mask and clean the eyes, 2 minutes.</p> <p>12. Apply essence on the eyes.</p> <p>13. Treatment done.</p>	
--	--	--	---

Course of treatment recommended:

It's advised to do it for 2 to 3 times every week. After one treatment, the eye will be tightened and lifted, and eye blood circulation will be accelerated. After a month, the fine lines and black eyes will be faded, and skin color will be improved. After three months, the eye will be tightened and lifted, and be rejuvenated and with gloss. To stick to it, the eye can be improved and eyes aging can be prevented.

