



MS-3493

Instructions

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆ How to contact us?

Phone: (+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!


Table of contents

1. Introduction to Manual.....	- 3 -
1.1 Purpose.....	- 3 -
1.2 Conventions	- 3 -
2. Medical Safety.....	- 3 -
2.1 Indications for Use.....	- 3 -
2.2 Contraindications.....	- 4 -
2.3 Precautions.....	- 4 -
2.4 Patient Safety.....	- 4 -
1. Features &Function.....	- 5 -
3.1 Features.....	- 5 -
3.2 Function.....	- 5 -
2. Control Unit & Handpiece.....	- 6 -
5. Principle & Operation pictures:.....	- 6 -
5.1 Unoisetion Cavitation 40k probe.....	- 7 -
5.2 Vacuum RF handle, L/M/S.....	- 8 -
5.3 Face RF.....	- 9 -
6. Package list.....	- 10 -
7. Notice.....	- 10 -
8. Treatment Operation.....	- 11 -
8.1 Face.....	- 11 -
8.2 Thigh.....	- 12 -
8.3 Back.....	- 13 -
8.4 Arm.....	- 13 -
8.6 Abdomen.....	- 14 -




1. Introduction to Manual

1.1 Purpose

This instructions for use manual provides a description of the system components, its controls and displays, instructions for its operation, and other equipment information important to the user

-  **Warning:** Do Not operate this machine before reading this manual thoroughly. In addition to this manual,additional clinical training may be available by the company or your local distributor.
For more information on training available please contact your local representative.

1.2 Conventions

-  **Note:** Notes designate information of special interest.
-  **Caution:** Cautions alert the user to precautionary steps necessary to properly operate the system.Failure to observe these cautions may void the warranty
-  **Warning:** Warnings alert the user to information that is of the highest importance and vital to the safety of the patient and user

All procedures are broken down by numbered steps.Steps must be completed in the sequence they are presented

Bulleted lists indicate general information about a particular function or procedure.They do not imply a sequential procedure.

Control names are spelled as they are on the system,and they appear in **Bold text**

2. Medical Safety

2.1 Indications for Use

The cavitation vacuum RF photon machine is indicated for use as a non-invasive dermatological aesthetic treatment to:

- Body slimming,body sculpture
- Face care,lifting,whitening
- Neck skin lift

2.2 Contraindications

The cavitation vacuum RF photon machine is contraindicated for use in patients with:

- Open wounds or lesions on the face and/or neck
- Severe or cystic acne on the face and/or neck

2.3 Precautions

The cavitation vacuum RF photon machine has not been evaluated for use over various materials. Therefore, treatment is not recommended directly over those areas with any of the following:

- Mechanical implants
- Dermal fillers
- Implanted electrical devices in the face and/or neck
- Metal stents in the face and/or neck area

Treatment energy is not recommended for use directly on an existing keloid.

The cavitation vacuum RF photon machine has not been evaluated for use in patients on an anticoagulant treatment plan


It is recommended that the following areas should be avoided during treatment:

- Thyroid gland, thyroid cartilage and trachea
- Major vessels

The cavitation vacuum RF photon machine has not been evaluated for use in the following patient populations:

- Pregnant or breast feeding women
- Children
- Those with the following disease states
 - A hemorrhagic disorder or hemostatic dysfunction
 - An active systemic or local skin disease that may alter wound healing
 - Herpes simplex
 - Autoimmune disease
 - Diabetes
 - Bell's palsy

2.4 Patient Safety

 Warning: cavitation vacuum RF photon machine should not be used on a patient's eyes or in a location or technique where ultrasound energy can reach the eye.



Warning: Use this system only if you are trained and qualified to do so.



Warning: If any problems occur during system operation, ask immediate action(s): lift the transducer off the patient's skin, press the STOP button to completely halt system operation.

3. Features &Function

3.1 Features

- Painless treatment focus RF energy at the correct position, compared to other radio frequency technology. It uses low energy and high frequency, safe and effective.
- Aim at the skin surface and deep location, using a complex method to control the different currents and energy, direct access to the different skin layer. Does not cause the phenomenon of uneven skin.
- Selectively targeting fatty tissue, to avoid heating other fat, to achieve the fastest treatment effect.
- The entire process is completed without the need of surgery and anesthesia.
- With the most useful 40KHZ ultrasonic cavitation system.
- Without side effects and the risk of gaining the weight back, the results are rather obvious. Does not affect the normal work and life.

3.2 Function


- Circumference and Cellulite Reduction
- Body Contouring and Slimming
- Belly fat removal, Weight Loss, body sculpting
- Skin rejuvenation, skin tightening, wrinkle removal, face lift
- Big Vacuum RF Probe: Belly/Leg cellulite removal, Shaping Double S Waist, Dredging lymph of back-- Relax back muscle, Massage Shoulder, Tighten Bottom Skin. Shap Bottom.
- Middle Vacuum RF Probe: Whole Arm Cellulite Removal, Neck Muscle Relax for improving Cerebral blood circulation. Face Tighten.
- Small Vacuum RF Probe: Face/ eye around/forehead Skin Tighten, V face Shaping .

4. Control Unit & Handpiece



- A. On/ Off button
- B. Vacuum adjusting button



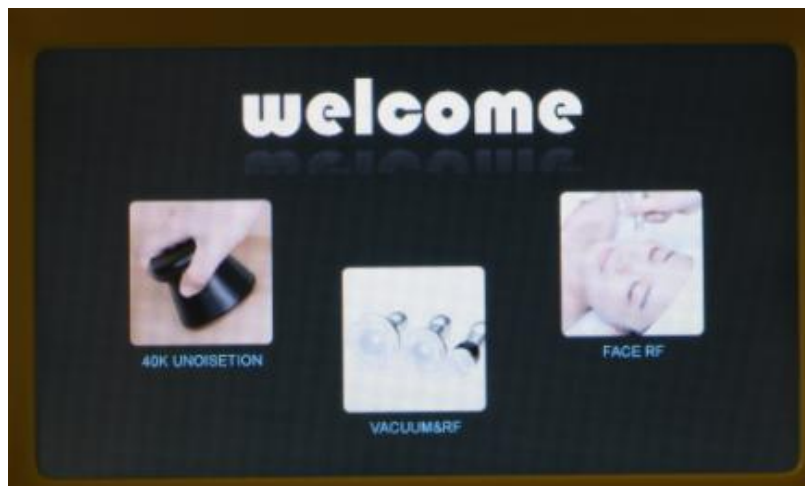


5. Principle & Operation pictures







5.1 Unoisetion Cavitation 40k probe

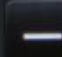

Operation:

- Press    to adjust the working time.
- Press  to decrease the energy, and press  to increase the energy.

Level 1~10 can be adjusted.

- Press  to select the model. Two models 1,2 can be selected.
- Press  to start or pause the operation.
-
- Press  to back to home page
- Press  to reset parameter






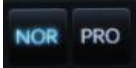









Notes:

Model 1:continuous wave, for the area with thin fat;
Model 2:dis-continuous wave, for the area with thick fat.
Operate with the 40KHz cavitation head for 15-20 minutes.

5.2 Vacuum&RF L,M,S Handles



Operation:

- Press    to adjust the working time.
- Press  to decrease the RF energy, and press  to increase the RF energy. Level 1~10 can be adjusted.
- Press  to select the model. Two models, NOR, PRO can be selected.
- Press    to adjust the suction time
- Press    to adjust the release time
- Press  to start or pause the operation.
- Press  to back to home page
- Press  to reset parameter

Notes:


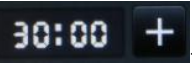
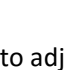
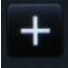

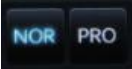



- NOR means photon keep lighting, and RF temperature will go slowly to the energy setting, PRO means photon light on when touch the treatment skin, RF temperature will arrive to the energy setting directly

- Setting the suction time longer than release time
- Apply slimming cream or cooling gel to the part of skin to be treated before the treatment.
 - Keep close contact between the probe and the skin.
 - Adjust the intensity from low level to high level.
 - Please do not irradiate eyeball directly with colored light.
 - Operate with the vacuum rf photon head for 15-20 minutes on body, arm, leg, face and eyes around.

5.3 Face RF



Operation :

- Press    to adjust the working time.
- Press  to decrease the energy, and press  to increase the energy. Level 1~10 can be adjusted.
- Press  to select the model. Two models, NOR, PRO can be selected.
- Press  to start or pause the operation.
- Press  to back to home page
- Press  to reset parameter

Notes:

- Spread gel or essence on the treated positions for hairdressing evenly.

- In use, start with low-energy. We suggest to try for the proper energy level on the back of clients' hands before using on body of them.
- Do not use essential oil as the medium.
- In operation, start pulling only after negative pressure head is attached to skin.
- Turn off the instrument probe when it is not in use.
- Operate with the multipolar RF head for 15-20 minutes on body.

6. Package list

- 1 x Yellow main machine
- 1 x 40K Unoisetion Cavitation 2.0 Handle
- 1 x Vacuum&Sextupolar RF L Handle with Photon
- 1 x Vacuum&Sextupolar RF M Handle with Photon
- 1 x Vacuum&Quadrupole RF S Handle with Photon
- 1 x Bipolar Face RF Handle
- 1 x Holder
- 1 x Power line

7. Notice

1. Please insert the filter to the back of the machine at your first use. Or the vacuum RF probe wont have suction.



2. please use gel before you use the RF probes, and make sure the probes completed touched with skin, or you will not feel the heat
3. If the vacuum suction is weak, please adjust the following knob:
Clockwise rotation: Min to Max.



4. If the vacuum probe still doesn't have suction after inserting the filter, adjusting the suction, release time, adjusting the vacuum knob, please hear the vacuum pump is working or not. If the vacuum pump is working, please take off the vacuum probe from the machine, then use your finger to touch the vacuum connector on the machine to see whether you can feel the suction (when you test, please make sure the vacuum function is on). If you can feel the suction on the vacuum connector on the machine, it shows it will be the problem of the vacuum probe, then just need to exchange one new vacuum probe; If you can't feel the suction, please take one video for your operating and testing, then send the person who sold you the machine. Then we solve the problem very fast.



5. If your machine has some problems, or you have some doubts for the machine urgently, please contact +86 18948755929 (whatsapp, Wechat) directly.

8. Treatment Operation

8.1 Face

Treated time: 30 minutes

Massage the mandible center in circles to produce deep heat



2. Massage lower jaw in lines.



3. Massage the triangle zone of both sides of the face in circles

4. Massage from jaw to angelus oris to ear in lines.

8.2 Thigh

Treated time: 30 minutes

1. From down to upper, pushing to the groin to dredge the lymph



2. From down to upper, circling push by anticlockwise can help decomposing fatness.



3. Also can push by come-and-go to decompose fatness.



4. Pull from the knee and muscle texture to upper, can improve the curve.

Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.



2. From upper to down, by anticlockwise gesture, pull come-and-go.




3. Along the muscle of arm, pull up to lift and tight the muscle.

8.3 Back

Treated time: 20 to 30 minutes



1. First by come-and-go, pushing the bladder nerve 2 to 3 times.

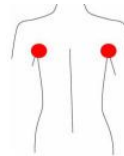


2. Circling by anticlockwise to stimulate the underarm lymph node.



3. On the back, by anticlockwise circling, can help to decompose the fatness.

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help improving the back

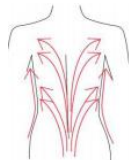

curve.

8.4 Arm

Treated time:30 minutes

1. From the inner elbow to the armpit, doing lymph drainage.



2. Circling to stimulate the lymph node.



3. By anticlockwise or come-and-go pushing, decomposing the fatness.



4. From the elbow to the outer, doing tightening gesture.

8.5 Facade

1. From the elbow to the outer, doing tightening gesture.

8.6 Abdomen

Treated time: 20~30minutes

1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of

the large intestine.



2. From small to large, by anticlockwise direction, circling push down.



3. From the belly and muscle texture, pull to the groin.



4. Through lymph drainage gesture, taking the toxin to the place of groin.

