

New 635nm-650nm Lipo Laser 5mw Lipolaser Slimming Machine 8big 4 Small Pads 440mv

Model :MY-12171



Catalogue

Introduce

Chapter 1: Safety Warnings

- I. Electricity & Machinery Safety
- II. Contraindication Safety
- III. Treatment Safety

Chapter 2: Installment and Use

- I. Working Condition
- II. Technical Parameters
- III. Program Setting
- IV. How to Put the Pads on Body

Chapter 3: Caution

Chapter 4: Accessory List

Chapter 5: Maintenance

- I. General Maintenance
- II. Troubleshooting

Introduce:

Lipo laser (or laser lypolysis) is a revolutionary treatment for the removal of excess fatty tissue in areas that are normally resistant to dieting and physical exercise.

The lipo laser body sculpting procedure uses 635nm-650nm nm laser that heats the fat cells in the targeted area causing them to break down. Over a short period of time the fat deposits are simply absorbed and expelled through our body's natural metabolic process.

During the procedure the laser seals off minor blood vessels significantly reducing bruising and patient downtime. The introduction of heat into the treated area stimulates collagen production to give a smooth, tightened and sculptured result.

Chapter 1: Safety Warnings

I. Electricity & Machinery Safety

- 1.The machine is suitable for 110V / 220V.
2. Please keep the voltage at stable condition.

II. Contraindication Safety

1. People with heart disease, high blood pressure, or configured cardiac Pacemaker.
2. People with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
3. People with hemorrhagic diseases, trauma or who is bleeding.
4. Pregnant women
5. Medical plastic artificial or metal parts inside the body such as metal teeth or silicone breast implants.
6. People with abnormal immune system.
7. People with numb or insensitive to heat.
8. If you are feeling unwell, Prohibited in the ears, nose, eye throat and do not use this device in advance and tell your doctor.
9. Children under 12 years old are not allowed.

III. Treatment Safety

1. Treatment Time is less than 30mins/Body Part/Each Treatment.
2. The energy setting is from low to high gradually which should be subject to skin comfortableness for different people.

3. The patient had better wear the protective eye glasses if need.
4. If you cannot tolerant heat or uncomfortable, please stop the treatment in advance and turn to your doctor for help.

Chapter 2: Installment and Use

I .Working Condition

Using at normal temperatures and pressure. Please do not stay in too cold, too hot, too dry and too wet.

II. Technical Parameters

Laser wavelength: 635nm ~650nm

Energy output : 64x5mw

Mains power output: 100VA


Classification :Electrical Class 1, Type B applied part

Cooling requirements :Air cooled

Electrical requirements: 100v--240V

III. Program Setting

1. Pls connect with the power supply, and turn on the power button behind machine.
Then the screen will show “ Welcome” ,and pls click the screen, then you will see the following main image:

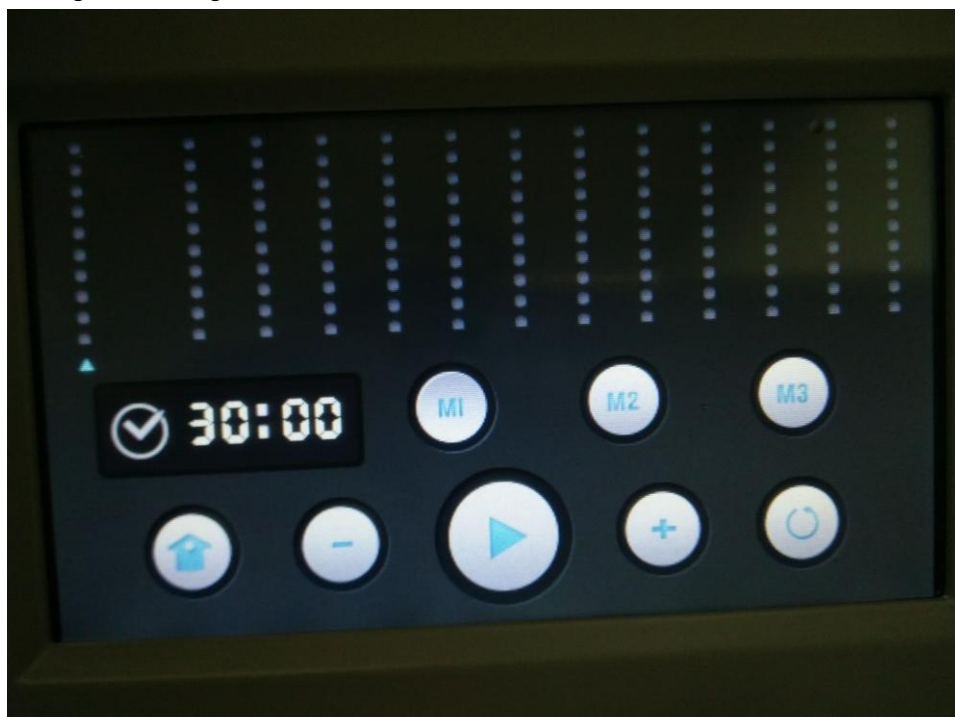

1. Please click  to set working time.

2. Please click  to increase the ENERGY intensity

3. Please click  to decrease the Energy intensity

4. Please click  to chose different modes

(**M1: Continue; M2: Flash; M3: fast Flash**)

5, Please click  to start on/ off

IV. How to Put the Pads on Body?

1. Big pads for big body area, such as belly, waist and etc. Small pads for small body area such as arms.
2. Using the belts and adjust it in a proper place.
3. Insert the pads in and close them to the skin.



Chapter 3: Caution

We normally suggest that the intensity should be adjusted from low to high gradually.

And recommended intensity level for big pads is 40-60% and 20-30% for small pads.

Chapter 4: Accessory List

1 x Power line

8x Big paddle with 10 //5mw Each (Total 80)

4x Small paddle with 2 //5mw each (Total 8)

1 x Metal holder for paddles

1 x Power Supply Cord

Chapter 5: Maintenance

I. General Maintenance

1. After the treatment, please turn off the machine, laser pads and the power supply.
2. Use the clean towel to clean the pads and the machine. Please do not clear the pads by alcohol.
3. Put back all of the parts in original place, at normal temperatures and pressure.
Please do not stay in too cold, too hot, too dry and too wet.
4. Machine needs 30mins break after 2-3hr operating.
5. Everyday do not operate the machine over 9hrs.
6. If it is not non-normal use and irresistible factors We offer One years' warranty for the units include the power, capacitor, touch-screen, and main board and inside the machine. And offer free repairment, one to one exchange or return.
7. The Pads belong to consumable electronic products, we only have three months warranty and offer one to one exchange or return during the warranty period. To know more, please check our warranty card that come with the machine.

II .Troubleshooting

1. How many treatments will i need to see the results?

- You will see the results of the treatment periodically. Initially you may notice some minor changes. But normally We recommend 2 treatments per week for 8 weeks.

2. Can i have more than one area treated at one time?

- Yes . if you are prepared to do more cardio vascular exercise.

3. Is it a long-term solution?

- Yes, if you follow our instruction, maintain a healthy diet and exercise routine.

4. Does the treatment hurt?

- No, it is a non-invasive treatment and here is no pain, bleed at all. But you may feel some heat when it worked for a while.

5. Will i have to keep coming back to keep my results?

- Generally, one treatment per a single area is usually adequate to achieve the required results. However, we recommend you more than 2 months of treatment for a better results. If the results are unsatisfactory there is the opportunity for repeat treatments. Because it depends on different body.

6. What is the difference between liposuction and Lipo laser?

- The procedure is similar to liposuction with some differences that may make Lipo laser a good alternative to traditional liposuction. Lipo laser essentially permanently removes your excess fat however without the suction removal of the fat. The body fat is dissolved or liquefied (turned into liquid) using a medical laser instead of suction.