


Meso
SKNV-919



Instruction




Keyboard of operation LCD screen

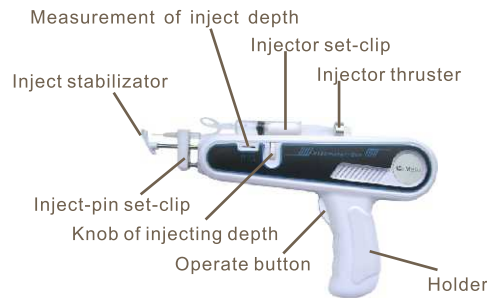

Measurement of inject depth
Inject stabilizator
Inject-pin set-clip
Knob of injecting depth
Operate button
Holder
Injector set-clip
Injector thruster

Operate instruction

1. Connect the Meso gun wire to the equipment, link the power wire, as following :

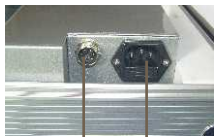


Meso Gun connection Power socket



Connection well

2. Power on, LCD screen light on, showing "install progress" or "install back", as following :

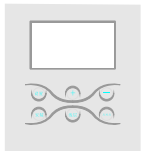

Press the "INSTALLATION" button, it's exchange of "install progress" and "install back", at the same time, it convey injuction to Meso Gun, press operate button, the injector thruster will "install progress" or "install back" according to the LCD screen.

3. Fix injector




Put injector into "injector set-clip", according the length of injector piston, adjust the injector thruster "progress" and "back", then press operate button of gun, set injector, adjust "knob of inject depth", toward "+" means increase inject depth, toward "-" means decrease inject depth.

4. Set Meso-Pin stable




Press the inject-pin set-clip, put the Meso-pin into the slot, adjust well the position of Meso-Pin, loosen the inject-pin set-clip, move off the pin-cap to operate.

5. According to our operate project to choose the "SUPERFICIAL" or "POINT TO POINT".

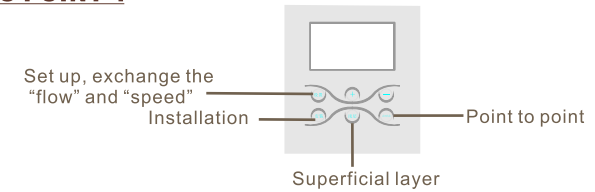


① when choose "superficial layer", as picture analyze :

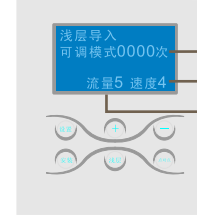


Record the inject points-quantity of operation
1-6 grade, number more large, speed more fast
1-9 grade, number more large, flow more big.

②When choose "Point to Point", as picture analyze:

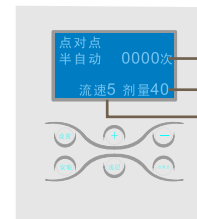

Record the inject points-quantity of operation
Each inject can choose setting 10, 20, 40, 60, 80, 100ul
1-5 grade, number more large, speed more fast

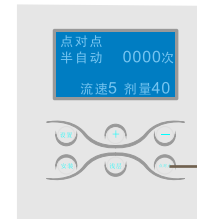

Press "point to point" button, exchange hand-use and semi-automatic, press "operate button" according to the flow speed setting, adjust the parameter, then start to work. After the inject, keep some little time to stop in order to suck products, then the Meso-pin will auto-retract, no need to forcely pull out.

Notice of Caution

1. This equipment must be operated by professional Meso-beautician with training or certificated people.
2. When operate the inject, be careful, if violent shake or drop-dropped, the proper functions will change possibly, pay attention.
3. Don't use the power over stated (AC220V, 50/60Hz)
4. Forbid to drag the power wire or wet-hand touch the power plug, otherwise it will cause fire or electric shock .
5. In order to avoid disconnection, don't distort the signal line of equipment or heavy pressing on it. If the signal line winding, firstly untie before use, don't freely pull the signal line or forely excessive connection
6. During the operation or after operation, must prevent liquid or other foreign matter into the Meso inject equipment interior, if there is liquid or foreign matter, or adhesive into interior, the Meso inject equipment must be cleaned immediately.
7. Don't lift up the Meso equipment, in order to prevent the liquid flowing into the device, otherwise it will cause trouble.