

User Manual

SR-AF1310



Preface

Dear users,

We're pleased to present to you our multifunctional beauty machine SR-AF1310 featured with skin rejuvenation, brightening, blackhead and acne removal, and so on. It is a multifunctional beauty machine, targeted at deep cleansing, hydrating and moisturizing, and skin rejuvenation. We aim at safe and, at the same time, effective beauty results by using high-end technologies. Hydro facial and ultrasonic devices are for professional use, which needs to be handled by professionals with proper training. Any improper use will result in adverse outcomes. Therefore, we advise any personnel to read this guide thoroughly and follow the instructions strictly before the operation.

We believe our quality product will bring in good returns on your end. Also, you can count on our worry-free after-sales service.

Thank you!

Table of Contents

| |
|---------------------------------|
| Preface |
| Part I |
| Brief Introduction |
| Advantages |
| Working Principles & Treatments |
| Cleansing and Hydrating |
| Soothing and Repairing |
| Skin Rejuvenation |
| Part II |
| 1. Detailed Operations |
| 2. Technical Parameters |
| 3. Safety Precautions |
| 4. Dos and Don'ts |
| 5. Troubleshooting & Solutions |
| 6. FAQs |
| 7. Packing List |
| 8. Operational Diagrams |

Part I

Brief Introduction

Hydro facial, ultrasonic, LED photon beauty machines are currently the most popular beauty machines that work on cleansing and infusion, hydrating, and moisturizing. It's an effective replacement of all the facial cleansing and care programs in beauty salons. Moreover, they are convenient and easy to operate. It solves both skin and body management issues for beauty-lovers. This equipment requires no injection, no medication, no operation, thus having no side-effects. It is used externally during the whole process and has an immediate effect. Hydro facial cleaning uses the exact principle in the state of no pain, achieving deep removal of various impurities in the hair follicle, mites, and grease residue. It combines with cold hammer, soothing and repairing the skin, shrinking pores, and having overall care.

Advantages

1. Easy to operate with obvious effect.
2. Multifunctional beauty devices meet different needs.
3. Specially designed water circulation+vacuum technology provides moisture to the skin while sucking blackheads, making the pores to close quickly.
4. 6 probes with 3 different models, meeting different types of skins.
5. Ultrasonic probe can improve absorption, accelerate nutrients infusion, and revitalize the skin.
6. Unevenness, bleed, swelling, and stasis will not appear after treatment.
7. Painless, non-invasive during the process. No downtime, without affecting any work and life plans.
8. LED photon masks adopt nano refraction technology to brighten the skin.

Cleansing and Hydrating

Principles

Hydro Facial

The vacuum circuit is formed through vacuum, and the ultra-small bubbles and nutrient solution are fully combined, and the spiral-shaped suction tip is specially designed to act directly on the skin, and the ultra-small bubbles can be kept in contact with the skin for a long time to induce the peeling effect. (Vacuum: it's below normal pressure. The use of vacuum is very common. People often make use of the omnipresent pressure by creating vacuum at some space. For example, when people

breathe, vacuum occurs when the lungs are in an expanded state. A difference in pressure is formed between the inside and outside of the lungs, and fresh air is pushed into the lungs.) Hydro facial cleansing uses the exact principle in the state of no pain, achieving deep removal of various impurities in the hair follicle, mites, and grease residue.

Biological Effect

With the most advanced moisture stripping technology, the skin can fully absorb the moisture of nutrient solution. Because the diameter of ultra-micro small bubble is only 10 microns, thus can easily penetrate the pores, take away the dirt within the pores. A large number of bubbles dissolved in water will rupture, during which ultrasonic energy released can produce a lot of oxygen anion. This kind of bubble water can kill bacteria, enhance human immunity, purify the blood, improve allergies, beauty and skin care, moisturize skin, and reduce acne.

Hydro Spray Jet

Using high-pressure oxygen and water, it sprays water flow on the skin with molecular particles less than 80 microns. The active water molecules and pure oxygen then quickly enters into the deep layer of the skin, accelerating the skin nutrition absorption. Through high-pressure force into the pores of the skin and skin crack by skin penetration into the dermis layer to promote the cell regeneration, it directly and quickly provides adequate nutrition for the skin, removes rubbish from the deep skin layer at the same time. Hyperbaric oxygen and nutrients can stimulate the fibrous tissue of the dermis regeneration, promote metabolism, thus achieves effects like hydrated, tender, and glowing skin.

Biological Effect

The hydro spray jet is equipped with a 8 kg pressure oxygen pump. It adopts the advanced principle of pressure swing adsorption (PSA, quick separation of high concentration of pure oxygen in the air, and through the pulse type push to mix 80% of pure oxygen molecules and 0.9% of the water molecules to water and transport them into cells in deeper layers of skin and replace waste toxins wherein, quickly produce energy, promote metabolism to achieve the clean, translucent and glowing skin.

Effects

1. Deep cleansing, dredge pores.
2. Improve skin tone to make skin brighter.
3. Deeply moisturizing and brightening the skin.
4. Improve oil secretion and skin quality.
5. Shrink pores and improve coarse skin.
6. Remove blackheads and acne.

Applicable Range

1. Those with dehydrated and malnourished skin.
2. Those with large pores and rough skin.
3. Those with poor skin absorption and lack of water in the skin base.
4. Those with dark and uneven skin tone.
5. Those with excessive oil secretion, oily skin.
6. Those with sallow skin that lacks in glowing.

Inapplicable Range

1. Those with severely sensitive skin or those in the allergic phase.
2. Those with extensive facial trauma.
3. Those who are pregnant and those suffering from serious heart disease, severe three H's (hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc.
4. Those with skin diseases and infectious diseases.
5. Those who have just received injection, such as intradermal injection, micro-needling, or with skin cuts.

Notes after Operation

1. Strengthen hydration and pay attention to sun protection.
 2. Do not use stimulating skin care products including scrub, exfoliating and AHAs within one week after operation.
 3. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
 4. Apply a hydrating mask once a day for three days.
- Wash your face with warm water within 3 days after operation.

Soothing&Repairing

Principles

Hydro Spray Jet

Using high-pressure oxygen and water, it sprays water flow on the skin with molecular particles less than 80 microns. The active water molecules and pure oxygen then quickly enters into the deep layer of the skin, accelerating the skin nutrition absorption. Through high-pressure force into the pores of the skin and skin crack by skin penetration into the dermis layer to promote the cell regeneration, it directly and quickly provides adequate nutrition for the skin, removes rubbish from the deep skin layer at the same time. Hyperbaric oxygen and nutrients can stimulate the fibrous

tissue of the dermis regeneration, promote metabolism, thus achieves effects like hydrated, tender, and glowing skin.

Biological Effect

The hydro spray jet is equipped with a 8 kg pressure oxygen pump. It adopts the advanced principle of pressure swing adsorption (PSA, quick separation of high concentration of pure oxygen in the air, and through the pulse type push to mix 80% of pure oxygen molecules and 0.9% of the water molecules to water and transport them into cells in deeper layers of skin, and replace waste toxins wherein, quickly produce energy, promote metabolism, so as to achieve the clean, translucent and glowing skin.

Cold Hammer

Use the principle of air conditioner. Special wafer ceramics are used to absorb energy and refrigerate at - 5 degrees in an instant. The method of physical heat accumulation produced by wafer energy absorption makes one side produce heat up to 40 degrees. The fan discharges excess heat energy and achieves the effect of constant temperature and freezing alternately. It also promotes metabolism, alleviates allergic reaction, shrinks pore, calms skin, whitens and tenders skin.

Biological effects

During the cooling process, tissue temperature cools down, blood vessels and veins contract, metabolic rate reduces, cell activity inhibits, peripheral nerve sensitivity reduces, thus having a sedative and soothing effect, counteracting or reducing the symptoms of an allergic reaction, such as redness, swelling, and pain, therefore achieving certain effects, such as calming the skin, reducing redness and swelling, shrinking pores and so on.

LED Photon Mask

Using the photodynamic principle, it activates deep cells and boosts skin metabolism. After the light is absorbed by the skin, the light energy is converted into cellular energy, which can promote the metabolism and synthesis of cells, promotes the fiber cell and endothelial cell proliferation to achieve the effect of repairing the aging cells and acne-prone skin, fading the spots, lifting and firming the skin, repairing sunburn. It relaxes and strengthens the microvessels and produces photochemical reactions with the skin -- enzymatic reactions, increasing the content of glycogen and protein, thus promoting the metabolism and synthesis of cells.

Red LED: Red LED of specific wavelength heats skin bottom, activates collagen regeneration, accelerates skin's microcirculation, increases skin elasticity, leaving skin rosy, smooth, young, and has fewer wrinkles.

Promote the regeneration of skin cells and activate active ingredients.

Blue LED: Blue LED of specific wavelength shrinks pores, calm skin, fight acne, balance sebum production, quickly lock in moisture and essence, leaving skin more elastic. Diminish skin inflammation, fight bacteria, fight acne.

Green LED: Green LED of specific wavelength helps to calm, repair skin, relieve pain, and promote blood circulation. It is suitable for those with thin corneum, and who are prone to allergy and redness.

Purple LED: Dual frequency of red and blue LED. It combines the efficacy of two kinds of phototherapy and has a particularly good and repair effect especially for the treatment of acne and acne marks.

Yellow LED: It can add energy to skin cells, promote the role of glands, assist digestion, treat skin diseases, enhance immune function, accelerate blood circulation, effectively repair red marks, red spots, relieve swelling, pigmented spots, and is suitable for removing spots and brightening.

Turquoise LED: Dual frequency of green and blue LED. It gradually enhances cell energy, and works great on metabolism.

White LED: It reaches deep into skin, accelerated metabolism of active cells, decompose pigments, reduce fine lines, tight skin. It can also relax skin, balance sebum production and control oil secretion.

Effects

1. Fast calming, strong moisturizing.
2. Improve dull and sallow skin.
3. Improve and repair sensitive skin.
4. Improve large oil fields and shrink pores.
5. Soothe and repair skin after photoelectricity operation.
6. Relieve and repair slightly sensitive skin.

Applicable Range

1. Those with dry skin and lack of moisture.
2. Those with uneven, dull complexion.
3. Those with sensitive skin, acne-prone skin, and easily inflamed, or allergic skin.
4. Those with large, coarse pores, and vigorous oil secretion.
5. Those who have just received the photoelectric treatment.
6. Those with skin peeling and dandruff.

Inapplicable Range

1. Those who have just undergone invasive plastic surgery with unhealed wounds.
2. Those with skin diseases and infectious diseases.
3. Those who are unrealistic about the results.
4. Those who have just received injection products, such as hyaluronic acid, intradermal injection, wrinkle removal, or plastic surgery.
5. Those who are having a severe allergy (red rash).
6. Use with caution for those who are allergic to lights.

Notes after Operation

1. Wash your face with warm water within 3 days after operation.
2. Strengthen hydration and pay attention to sun protection.
3. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
4. Apply a hydrating mask once a day for three days.
5. Avoid eating spicy or greasy food.

Skin Rejuvenation

Principles

Hydro Facial

The vacuum circuit is formed through vacuum, and the ultra-small bubbles and nutrient solution are fully combined, and the spiral-shaped suction tip is specially designed to act directly on the skin, and the ultra-small bubbles can be kept in contact with the skin for a long time to induce the peeling effect. (Vacuum: it's below normal pressure. The use of vacuum is very common. People often make use of the omnipresent pressure by creating vacuum at some space. For example, when people breathe, vacuum occurs when the lungs are in an expanded state. A difference in pressure is formed between the inside and outside of the lungs, and fresh air is pushed into the lungs.) Hydro facial cleansing uses the exact principle in the state of no pain, achieving deep removal of various impurities in the hair follicle, mites and grease residue.

Biological Effect

With the most advanced moisture stripping technology, the skin can fully absorb the moisture of nutrient solution. Because the diameter of ultra-micro small bubble is only 10 microns, thus can easily penetrate into the pores, take away the dirt within the pores. A large number of bubbles dissolved in water will rupture, during which ultrasonic energy released can produce a lot of oxygen anion. This kind of bubble water can kill bacteria, enhance human immunity, purify the blood, improve allergies, beauty and skin care, moisturize skin, and reduce acne.

Hydro Spray Jet

Using high-pressure oxygen and water, it sprays water flow on the skin with molecular particles less than 80 microns. The active water molecules and pure oxygen then quickly enters into the deep layer of the skin, accelerating the skin nutrition absorption. Through high-pressure force into the pores of the skin and skin crack by skin penetration into the dermis layer to promote the cell regeneration, it directly and quickly provides adequate nutrition for the skin, removes rubbish from the deep skin layer at the same time. Hyperbaric oxygen and nutrients can stimulate the fibrous tissue of the dermis regeneration, promote metabolism, thus achieves effects like hydrated, tender, and glowing skin.

Biological Effect

The hydro spray jet is equipped with an 8 kg pressure oxygen pump. It adopts the advanced principle of pressure swing adsorption (PSA, quick separation of high concentration of pure oxygen in the air, and through the pulse type push to mix 80% of pure oxygen molecules and 0.9% of the water molecules to water and transport them into cells in deeper layers of skin, and replace waste toxins wherein, quickly produce energy, promote metabolism to achieve the clean, translucent and glowing skin.

Facial Ultrasonic

Ultrasonic facial beauty instrument is based on the theory of high-frequency vibration. It reaches deep into the subcutaneous tissues, 4-6 mm to be exact, with a strong vibration wave of 2.8MHz. With its high energy and permeability, it vibrates tissues in every layer of your skin. When using ultrasonic beauty instrument for massage, the warmth of ultrasonic wave can penetrate to 6MM under the skin, make the blood and lymph flow smoothly, accelerate the blood circulation of fine cells, start the biological effect of cell tissue, fade all kinds of color spots, whiten the skin and activate and regenerate the cells of deep tissue.

LED Photon Mask

Using the photodynamic principle, it activates deep cells and boosts skin metabolism. After the light is absorbed by the skin, the light energy is converted into cellular energy, which can promote the metabolism and synthesis of cells, promotes the fiber cell and endothelial cell proliferation to achieve the effect of repairing the aging cells and acne-prone skin, fading the spots, lifting and firming the skin, repairing sunburn. It relaxes and strengthens the microvessels and produces photochemical reactions with the skin -- enzymatic reactions, increasing the content of glycogen and protein, thus promoting the metabolism and synthesis of cells.

Red LED: Red LED of specific wavelength heats skin bottom, activates collagen regeneration, accelerates skin's microcirculation, increases skin elasticity, leaving skin rosy, smooth, young, and has fewer wrinkles.

Promote the regeneration of skin cells and activate active ingredients.

Blue LED: Blue LED of specific wavelength shrinks pores, calm skin, fight acne, balance sebum production, quickly lock in moisture and essence, leaving skin more elastic.

Diminish skin inflammation, fight bacteria, fight acne.

Green LED: Green LED of specific wavelength helps to calm, repair skin, relieve pain, and promote blood circulation. It is suitable for those with thin corneum, and who are prone to allergy and redness.

Purple LED: Dual frequency of red and blue LED. It combines the efficacy of two kinds of phototherapy and has a particularly good and repair effect especially for the treatment of acne and acne marks.

Yellow LED: It can add energy to skin cells, promote the role of glands, assist digestion, treat skin diseases, enhance immune function, accelerate blood circulation, effectively repair red marks, red spots, relieve swelling, pigmented spots, and is suitable for removing spots and brightening.

Turquoise LED: Dual frequency of green and blue LED. It gradually enhances cell energy and works great on metabolism.

White LED: It reaches deep into the skin, accelerated metabolism of active cells, decompose pigments, reduce fine lines, tight skin. It can also relax the skin, balance sebum production and control oil secretion.

Effects

1. Deeply clean and shrink pores.
2. Reduce blackheads and acne.
3. Fast moisturizing and accelerate absorption.
4. Improve skin tone and skin quality.
5. Improve dry and innutritious skin.
6. Fade pigmentation and brighten the skin.
7. Improve oil secretion and shrink pores.
8. Delicate the skin, improve coarse skin.
9. Activate cells, increase skin immunity and improve skin quality.

Applicable Range

1. Those with high oil secretion and thick corneum.

2. Those with blackheads, acne, and with coarse skin.
3. Those with clogged, coarse pores.
4. Those with dehydrated, innutritious skin.
5. Those with uneven, dull skin.
6. Those with poor absorption and fast moisture loss.

Inapplicable Range

1. Those who are during menstruation, pregnancy, lactation.
2. Those who are pregnant and those suffering from serious heart disease, severe three H's (hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc.
3. Those with heart disease or who are equipped with a pacemaker.
4. Those who have just received injection, such as intradermal injection, wrinkle removal, or plastic surgery.
5. Those with skin diseases and infectious diseases.
6. Those who are in the allergic phase.

Notes after Operation

1. Wash your face with warm water within 3 days after operation.
2. Strengthen hydration and pay attention to sun protection.
3. Do not use stimulating skin care products including scrub, exfoliating and AHAs within one week after operation.
4. Apply a hydrating mask once a day for three days.
5. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.

Part II

1. Detailed Operation

The detailed installment is as follows:



1.1 Detailed hydro facial installment is as follows:



Device On/Off



Hydro Facial On/Off



Suction +/- . Rotate anticlockwise to increase suction, rotate clockwise to reduce it. 20%-80% is suggested. Adjust according to your comfort level.



Water Flow +/- . Rotate anticlockwise to increase water flow, rotate clockwise to reduce it.



Focal probe. Suitable for dead skin and acne-prone skin.



Flat probe. Suitable for exfoliating and removing acne.



Vortex probe. Suitable for cleansing and removing blackheads in the small treatment areas.



Clean Bottle/Waste Bottle




Clean Bottle is on the right side.

Fill the clean bottle with needed nutrient solution

Fill deep cleansing liquid (e.g., AHA): suitable for all skins with plant-derived, mild and non-irritating nature, cutin, sebum removal, and deep cleansing.

Fill normal saline when cleaning the pipe of the instrument: after using, put about 50ml of normal saline in the clean bottle, adjust the water flow to the maximum. Click to turn on the instrument, block the probe with hand, let the water return automatically to clean the pipe about 1 minute. Avoid blockage and damage to instrument pipes to prevent bacterial growth and infection.

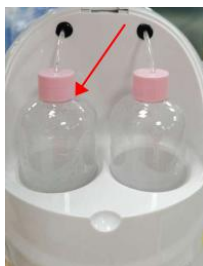

Waste Bottle is on the left side.

No fill in waste bottle. It is for storing sewage. When the waste bottle is full, the instrument will not work. Ensure to remove the waste and clean the bottle in time.

1.2 Detailed hydro spray jet installment is as follows:



Device On/Off



Hydro Spray Jet On/Off.



When installing the nozzle, snap the convex point on the nozzle to the groove on the handle of the hydro spray jet and rotate it to the left until the convex point is not visible.



After installation



Point the glass bottle of hydro spray jet upward to ensure the water spray works.

1.3 Detailed cold hammer installment is as follows:



Device On/Off



Cold Hammer On/Off



Do not block the air outlet when operating cold hammer to ensure exhausting normally.

1.4 Detailed ultrasonic installment is as follows:



Device On/Off



Ultrasonic On/Off

1.5 Detailed LED photon mask installment is as follows:



Device On/Off



LED Photon Mask On/Off. Click to turn on LED photon mask, and then click to change LED lights. The eighth one is 7 LED lights flashing circularly, click for 3 seconds to turn off the LED photon mask.

2. Technical Parameters

Item Type: Beauty Machine

Material: ABS

Color: As Pictures Show

Input: AC 100-240V 0.3A

Machine size: 275*325*238mm

Frequency: 50/60Hz

Power: 20W

3. Safety Precautions

Please consult your doctor or professionals before using the device if you have the following symptoms:

1. Those who have severe diseases such as severe “three Hs” (hypertension, hyperlipidemia, hyperglycemia), malignant tumors, or hemophilia.
2. Those with epilepsy and severe diabetes and hyperthyroidism.
3. Those with heart disease or who are equipped with a pacemaker.
4. Those with skin diseases and infectious diseases.
5. Those with allergic and severely sensitive skin.
6. Those with are pregnant, please ask for the doctor’s advice.

4. Dos and Dont’s

1. Use a grounding pin and make sure the power socket of the device is properly connected before each operation.
2. Ensure that the voltage of the device is appropriate. If the voltage of the local power supply is unstable, we recommend that users increase to the matching power voltage.
3. To ensure the therapeutic effect and normal service life expectancy of the device, please use the specified accessories provided or recommended by the original manufacturer.
4. The device should not be placed in a wet environment or near water, nor should they be directly exposed to sunlight.
5. Do not place the device near a strong heat source, because this may affect the life and normal use of the device.
6. Please restrain from using the device on the eyes, thyroid, parathyroid, testicles, the abdomen of pregnant women, and pacemakers.
7. Patients who are suffering from an illness should consult with a doctor and gain permission from the doctor before using it.
8. Please turn off the power switch of the device, and ensure that the total power supply of the device is turned off before the staff leaves after daily use to ensure the safety of the electrical products.
9. Use device or train device operators in strict accordance with instructions in the user manual.
10. It is recommended to turn on the instrument continuously for no more than 2 hours. When it reaches 2 hours, please shut down the instrument, cut off the power for 10-30 minutes, and then restart to use.
11. The tips of hydro facial can be cleaned with water and soaked in 75% alcohol.
12. Please clean the waste bottle in time after treatment.
13. Do not stay in the same area for more than 3 seconds to avoid detoxification when operating.
14. Try the suction on your arm and adjust it before operating on face to avoid detoxification.

15. Hydro facial, ultrasonic, and cold hammer etc. operating heads can be used separately. Use several operating heads together will have a better effect.
16. Screw head cleaning: basic cleaning must be done to maintain the prevention of infection. After using the screw head, disposable toothbrush can be used to brush out the dirty stuff in the groove of the screw head, rinse it off with clean water, then soak with alcohol for 3 minutes, and finally rinse the alcohol ingredients.
17. Internal cleaning (after each bottle of solution is used):
Replace the probe with a cleaning head, fill the solution bottle with half a bottle of purified water, turn on the device, plug the probe by hand, adjust the water flow to the maximum, and let the water flow back to clean the residual solution in the pipe. Around a minute.
18. Do not block the air outlet when operating cold hammer to ensure exhausting normally.
19. When operating ultrasonic, do not stay in the same treatment area for 5 seconds to avoid burn or freeze the skin.

5. Troubleshooting & Solutions

1. The instrument can't start?

- A. Ensure that the power cord is connected to a working power outlet.
- B. Ensure whether the connection of the power cable is loose.

2. No suction when the device is on?

- A. Check whether the clean bottle and waste bottle are tightened.
- B. Check whether the suction button stays at the minimum, and then adjust it to a larger value.
- C. Check whether the water flow button stays at the maximum, and then adjust it to a lower value.

3. No nutrients outflow when the device is on?

- A. Check whether the clean bottle and waste bottle are tightened.
- B. Check whether the water flow button stays at the minimum.
- C. If the above methods cannot address the issue, please contact the instrument dealer for assistance.

5. No spray coming from the polymer atomizing pen?

- A. Please check whether the nozzle is blocked.
- B. Please check whether the regulating part of the nozzle tail is tightened.

6. No ultrasonic energy output?

- A. Please check whether the connecting wire of the handle is well connected and whether there is any damage.

B. If the above methods cannot address the issue, please contact the instrument dealer for assistance.

6. FAQs

1. Q: Can I see the result after one hydro facial treatment? How often should I do it?

A: Hydro facial cleansing uses the exact principle in the state of no pain, achieving deep removal of various impurities in the hair follicle, mites and grease residue. One treatment helps increase cleaning to 60%. Operate once a week. If you have serious blackheads and clogged pores, you can operate once in 4 to 5 days in the early stage, and you can treat once a week after 4 continuous treatments.

2. Q: Will hydro facial treatment enlarge pores?

A: No. Hydro facial deep cleansing treatment follows the sequence of three steps. The first step is to use the export liquid to soften skin cutin layer; the second step is coupled with different solutions (AS1 AHA, AS2 BHA, AS3 nutrient solution; choose different solutions according to different types of skin); the third step is to hydrate the skin. Hydro facial is suitable for all skin types and will not enlarge pores but rather refine skin.

3. Q: Can ultrasonic import help remove acne marks?

A: Ultrasonic import channels whitening nutrients to the skin by ultrasonic high-frequency vibration, accelerate the absorption, inhibit melanin formation and fade spots, effectively brighten dull skin, and works wonder especially on dark spots caused by acne. It is green, ecological, simple, safe, reliable, and has no side effect. General outdoor activities can be conducted after treatment.



4. Q: Can I wear makeup after treatment?

A: Absolutely. The skin is moisturized and plump after treatment, and the makeup will be more natural. This treatment does not break the skin and no wounds, it is safe, healthy, fast, and effective.

7. Packing List

1 x Main Machine
1 x Power Cord
1 x Hydra Dermabrasion Pen
1 x Polymer Atomizing Pen
1 x Cold Hammer Handle
1 x Ultrasonic Handle
1x Led photon mask 7 colors

8. Operational Diagram



| Parameter Adjustment | Product | Technique | Diagram |
|--|---|--|--|
| Cleansing and Hydrating: 30 min, 1-2 times/week | | | |
| <p>Hydro facial: Suction: 20%~80%</p> <p>Water Flow: 20%~80%</p> <p>Hydro spray jet: No energy adjustment required</p> | <p>Makeup remover+ Face wash+ Blackhead export liquid+ Mask</p> | <ol style="list-style-type: none"> 1. Remove makeup and clean face, 5 min. 2. Apply toner, 1 min. 3. Apply blackhead export liquid with a cotton pad on T-zone, 5 min. 4. Remove cotton pad. 5. Turn on the device, adjust energy parameters and start operation. 6. Hydro facial: Rotate the device on the nose area, 1-2 times. 7. Move from the bottom to the top on the nose area, 1-2 times. 8. Rotate the device on the chin once, and then move slowly for 1-2 times. 9. Rotate the device on the forehead, 2-3 times. 10. Operate another 1-2 times clogged pores at the beginning of the eyebrows. 11. Rotate the device on the cheeks, 2 times. 12. Pause the device. 13. Wash face, 1 min. 14. Fill toner or dilute essence in the glass bottle of hydro spray jet. | <p>Technique 16</p>  <hr/> <p>Technique 18</p>  |




| | | | |
|--|--|---|--|
| | | <p>15. Start the device to operate.</p> <p>16. Operate with hydro spray jet: Lift with the device in small circles, 2 times.</p> <p>17. Do the same on the other side.</p> <p>18. Move on the forehead in small circles in left-right order, 2 times.</p> <p>19. Apply mask, 15 min.</p> <p>20. Remove mask and wash face.</p> <p>21. Apply toner, essence, emulsion, cream, and sunscreen.</p> | |
|--|--|---|--|

Suggested Treatment:

1 treatment accounts for a full course, treat 1-2 times a week. After one treatment, the pores will be visibly clean, and the skin will be fully hydrated. One course of treatment helps brighten the skin and shrink pores. Three courses of treatment help keep the skin rosy, moisture for 24 hours, and the skin is bright and elastic.


Soothing and Repairing: 40min, 2~3 times/week




| | | | |
|---|--|--|--|
| <p>Hydro spray jet: No energy adjustment required</p> <p>Cold hammer: No energy adjustment required</p> <p>LED photon mask: Green LED</p> | <p>Makeup remover+ Cleanser+ Chamomile toner+ Repair essence+ Repair mask+</p> | <p>1. Remove makeup with a cotton pad.</p> <p>2. Wash face, 2 min.</p> <p>3. Hydro spray jet hydrating: Fill chamomile toner in the glass bottle.</p> <p>4. Turn on the device and start operation.</p> <p>5. Move the device from the bottom to the top in small circular motions, 2 times.</p> <p>6. Move from the bottom to the top on lines on the face, 1 time.</p> <p>7. Do the same on the other side.</p> <p>8. Move in small circular motions in left-right order on the forehead, 2 times.</p> | <p>Technique 5</p>  <hr/> <p>Technique 6</p>  <hr/> <p>Technique 8</p> |
|---|--|--|--|

| | | | |
|--|--|---|---|
| | | <p>9. Pause the device. 10. Shrink pores and repair the skin with cold hammer: Apply repair essence on face. 10. Turn on the device and start operation. 11. Move the probe from jaw to earlobe, mouth corner to ear gate, nose wing to temple in circles, 3 times. 12. Lift towards hairlines on the forehead, 3 times. 13. Do the same on the other side. 14. Apply repair facial mask+LED photon mask to calm the skin: Wear the LED photon mask. 15. Turn on the device, adjust to green LED to start operation. 16. Wait for 15 min. 17. Remove LED photon mask and facial mask, wash face. 18. Apply toner, repair essence, cream, and sunscreen.</p> |  <hr/> <p>Technique 12</p>  <hr/> <p>Technique 13</p>  |
|--|--|---|---|

Suggested Treatment:
Treat 2-3 times a week, one-month treatment accounts for one full course. After one treatment, the skin will be glossy and smooth. After one course, it calms the skin, and the skin will not be allergic due to seasonal change, makeup, dehydrated skin. Two courses help increase skin smoothness, elasticity, brighten dull skin, increase skin metabolism. Three courses help stabilize skin state, prevent skin allergy.

Skin Rejuvenation: 30 min, 1-2 times/week

| | | | |
|---|---|---|---|
| <p>Hydro facial: Suction: 20%~80%</p> <p>Water Flow: 20%~80%</p> | <p>Makeup remover+ Cleanser+ Blackhead export liquid+ Essence+</p> | <p>1. Remove makeup and clean face, 5 min. 2. Apply toner, 1 min. 3. Apply blackhead export liquid with a cotton pad on T-zone, 5 min. 4. Remove the cotton pad.</p> | <p>Technique 15</p>  |
|---|---|---|---|

| | | | |
|---|-------------|--|--|
| <p>Hydro spray jet: No energy adjustment required</p> <p>Ultrasonic: No energy adjustment required</p> <p>LED photon mask: 7 LEDs</p> | <p>Mask</p> | <p>5. Hydro facial: Turn on the device, adjust energy parameters and start operation.</p> <p>6. Rotate the device on the nose area, 1-2 times.</p> <p>7. Move from the bottom to the top on the nose area, 1-2 times.</p> <p>8. Rotate the device on the chin once, and then move slowly for 1-2 times.</p> <p>9. Rotate the device on the forehead, 2-3 times.</p> <p>10. Operate another 1-2 times clogged pores at the beginning of the eyebrows.</p> <p>11. Rotate the device on the cheeks, 2 times.</p> <p>12. Pause the device.</p> <p>13. Wash face, 1 min.</p> <p>14. Fill toner or dilute essence in the glass bottle of hydro spray jet.</p> <p>15. Move the device from the bottom to the top on the face in small circular motions, 2 times.</p> <p>16. Do the same on the other side.</p> <p>17. Move in left-right order on the forehead in small circular motions, 2 times.</p> <p>18. Pause the device.</p> <p>19. Ultrasonic infusion: Apply essence on the face.</p> <p>20. Move the probe from jaw to earlobe, mouth corner to ear gate, nose wing to temple in circles, 3 times.</p> <p>21. Do the same on the other side.</p> <p>22. Move horizontally in circles on the forehead, 3</p> | <p>Technique 17</p>  <hr/> <p>Technique 20</p>  <hr/> <p>Technique 22</p>  |
|---|-------------|--|--|

| | | | |
|---|--|---|--|
| | | <p>times.</p> <p>23. Apply facial mask+LED photon mask to face pigmentation and brighten the skin.</p> <p>24. Turn on the device and adjust 7 LED lights flashing circularly to start operation.</p> <p>25. Wait for 15 min.</p> <p>26. Remove facial mask+LED photon mask, wash face.</p> <p>27. Apply toner, essence, cream, and sunscreen.</p> | |
| <p>Suggested Treatment:</p> <p>Treat 1-2 times a week, 12 treatments account for one full course. After one treatment, the skin will be clean, translucent, and moisture. After one course, it cleans the pores, brightens the skin. Two courses help keep skin delicate, bright, and fine lines faded. Three courses help stabilize skin state, prevent clogged pores, and delay aging.</p> | | | |