

Hydra peel Dermabrasion Machine **INSTRUCTION**



Dear customer,
Thanks you very much for choosing our product!
For the best capability and safety, please read the instruction before
operation.
Please reserve the instruction book for the reference.

Introduction of panel and accessory




Touch-screen:


1, main screen:




2, Click anywhere to enter the next interface:

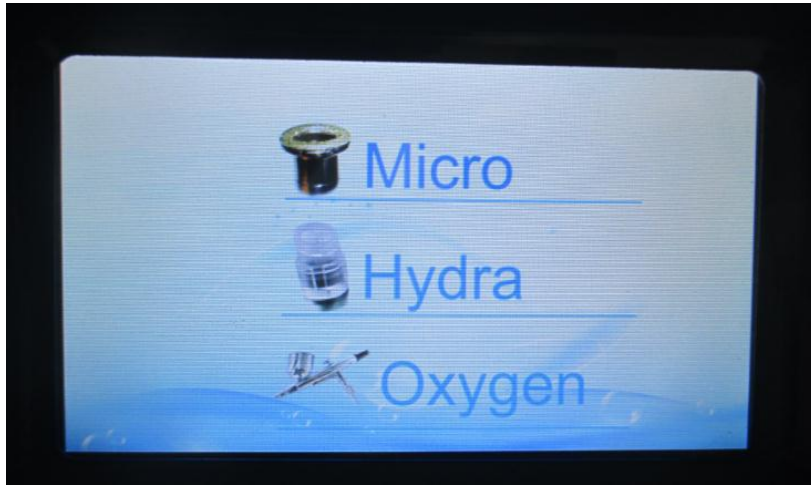


3, diamond microdermabrasion:

Click  to enter the interface:

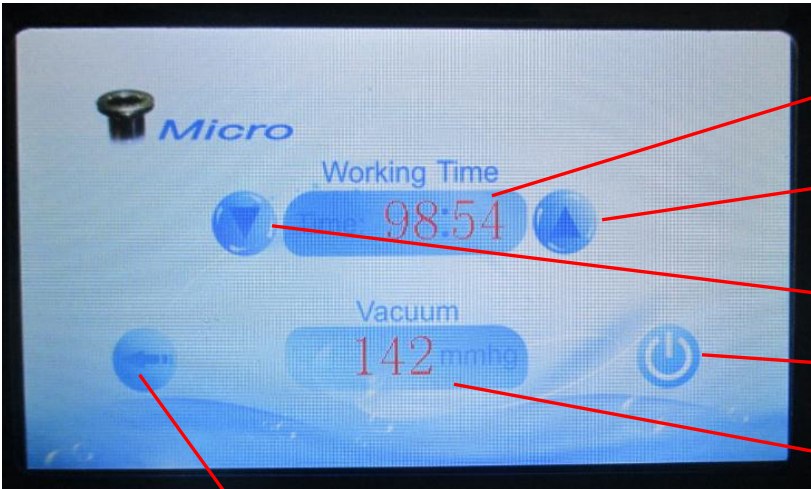


The screenshot shows the control interface for the "Micro" treatment. It features a "Working Time" display showing "98:54" with up and down arrow buttons on either side. Below that is a "Vacuum" display showing "142 mmHg" with a power button to its right. A "Return button" is located at the bottom left. Red arrows point from these elements to labels in boxes on the right: "From 00:00-99:00" (pointing to the time display), "Time increase" (pointing to the up arrow), "Time reduced" (pointing to the down arrow), "Turn on/off button" (pointing to the power button), "Vaccum display" (pointing to the vacuum display), and "Return button" (pointing to the bottom left button).


4, hydra facial :

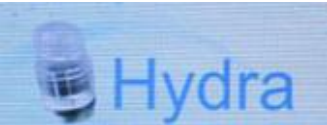
Click  to enter the interface:

From 00:00-99:00

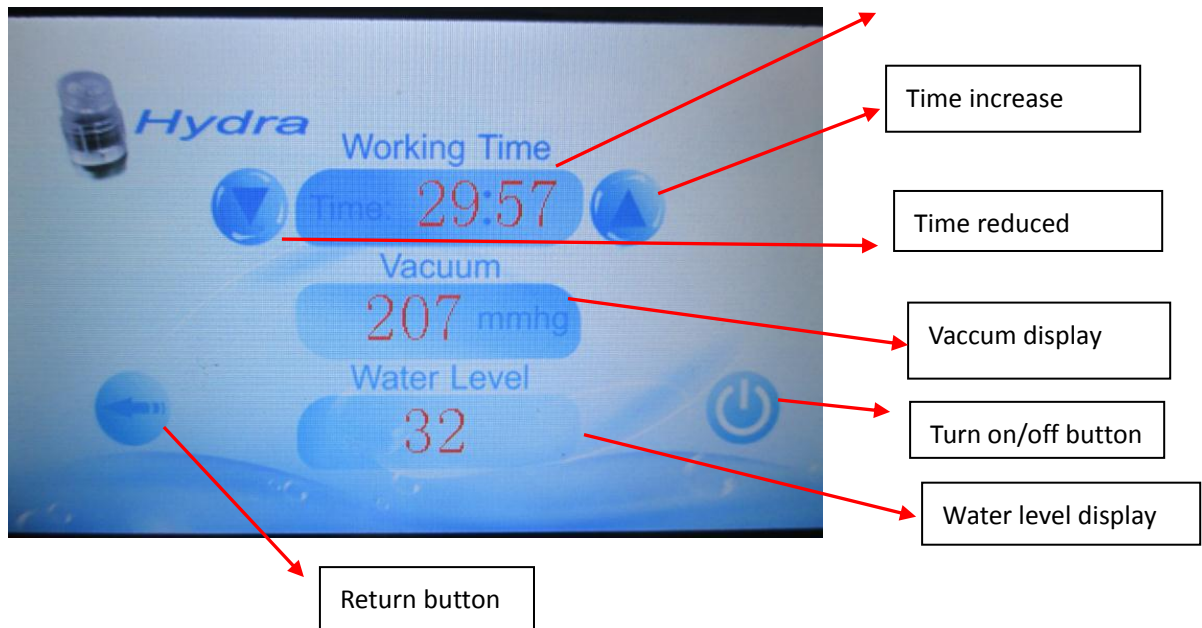

5, oxygen spray:

Click  to enter the interface:

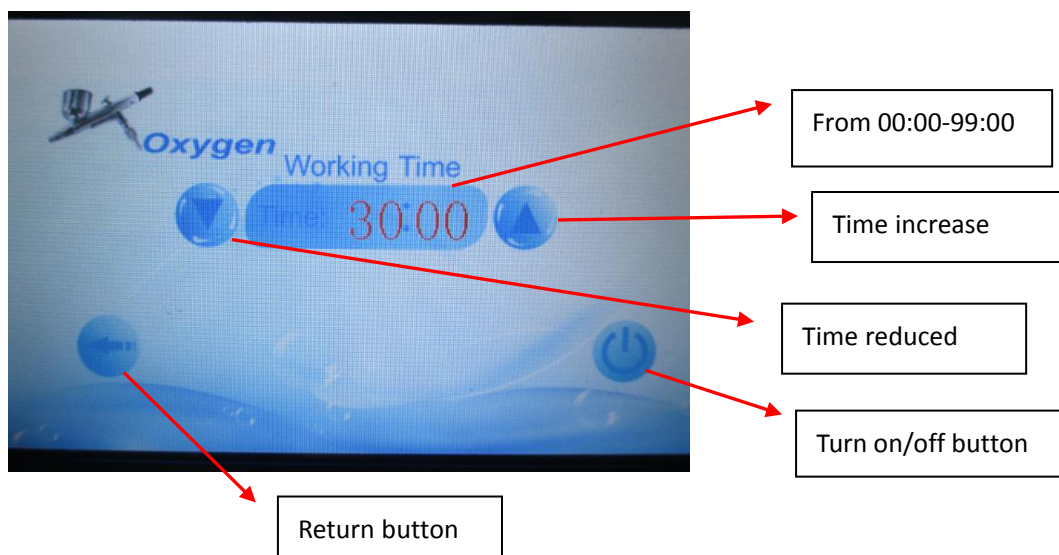

Hydra facial dermabrasion functions:

- 1) hydra facial
- 2) Diamond micro-dermabrasion
- 3) Oxygen spray

4) Cold hammer

Effect:


Hydra facial dermabrasion help to improve the appearance of

- fine lines & wrinkles
- hyperpigmentation & melasma
- sun damaged skin
- age spots
- blackheads & whiteheads
- enlarged pores
- excess oil
- hydration
- puffy eyes
- rejuvenation, overall skin health

Operation skills:

1) diamond micro-dermabrasion



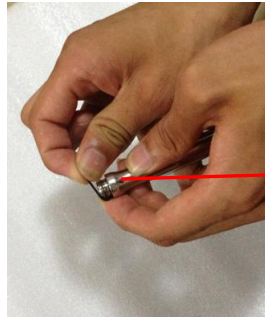


Important tips:
The black ring is necessary before use, otherwise it may cause air leakage and affect using effect. Please remember installing it.



Diamond tips

Step 2: put diamond wand tightly onto corresponding skin according to different tips

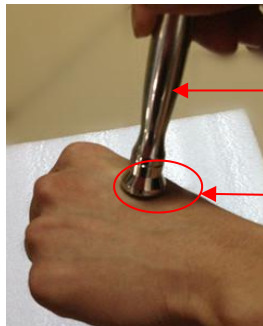


Diamond peel handle

Touch the skin

Tips disinfected:
Wipe the tip after each use treatment then Sanitize with alcohol or a good germicide solution. Dry tips and wands completely before storage and/or before use.


2) Hydra facial dermabrasion

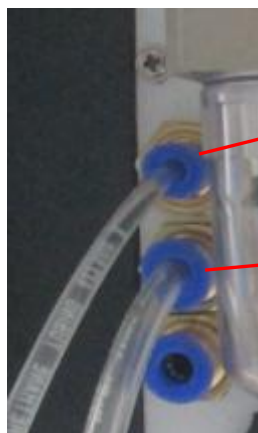


Water in

Water out

PVC tips connect to aqua peel handle

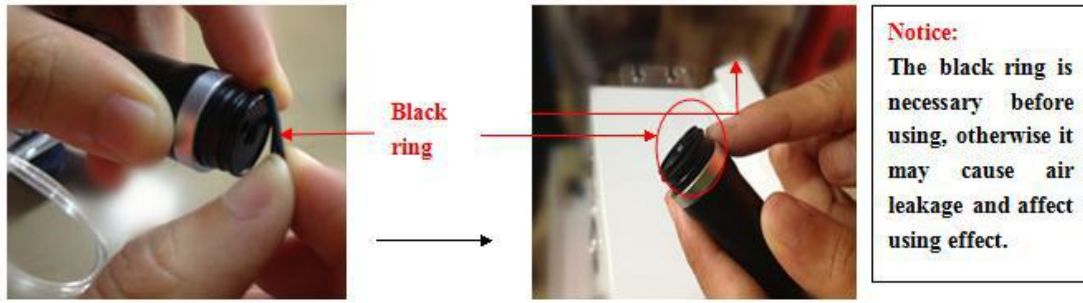


put physiological saline or other essence into clean water bottle and then connect clean and recycle water bottle to mainframe

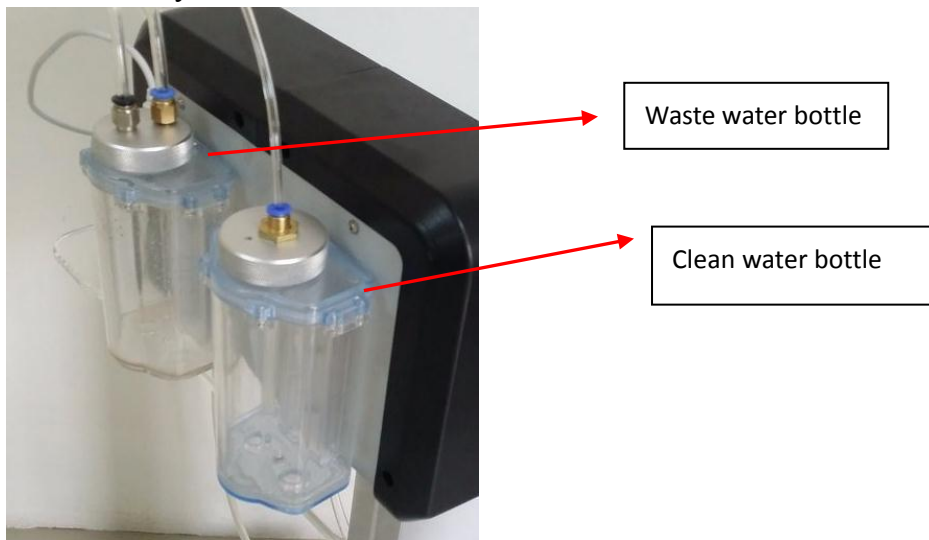

3) Oxygen spray



connect pipe to spray and turn on the machine

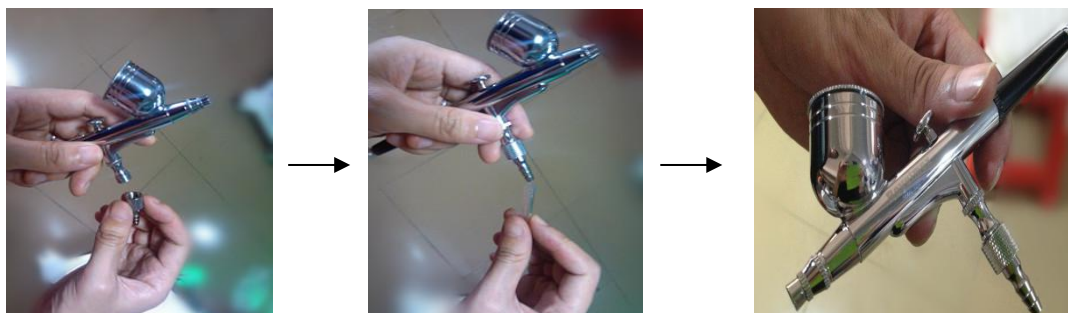

connect the spray on the back of the machine



Oxygen spray handle pipe connector


add 2-3ml serums/lotion to spray



Remark:

You can put some essence, nutrient solution and other water-soluble essence into it.

start spray and you can see the nice effect of water/serums spray

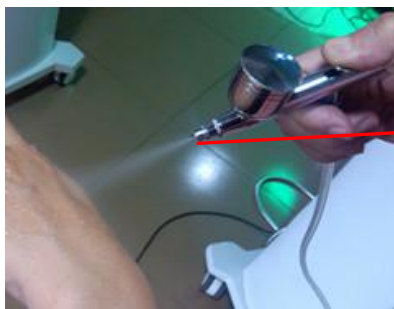

Water /serum spray

4) cold hammer



Technique parameter

Electricity voltage: AC90V-220v

Working frequency: 50Hz/60HZ

Power: 250w

Packing size: 72*46*43cm

Gross weight: 25KG

Treatment parameter reference:

1 hydro facial: water flow 0~50 (recommended water flow regulation 10~15 (0~600):

facial pressure suction: regulating 150~200

Nose: regulating 200~300

Chin: 200~250

2 diamond microdermabrasion: 70~600 (general diamond head pressure we adjusted to 70~200 suction)

Treatment course reference:

Hydro facial recommended 2 times / week

Microdermabrasion recommended 3-5 days / times, 8-10 times / course.

spray oxygen is recommended to do after the above treatment to strengthen the skin moisture.

Cold hammer calm skin, shrink pores.

Treatment Process:

- 1, Royal makeup
- 2, clean, toner
- 3, No. 1 with liquid + compressed facial mask 5-8 minutes to soften the cuticle completely
- 4, the cuticle hypertrophy, more rugged, black, rough skin, can be used to clean the drill carving. (around the eye is not recommended to use the drill)
- 5, with 2 of the liquid into the water bottle, the use of small bubble head to clean the skin! (there is an inflammation or is not used in the purulent skin)
- 6, the pure water is used to clean the face, adding 3 liquid to spray water, full wet and gently pat face make nutrition evenly absorbed into every pore of skin
- 7, ice hammer can be based on the skin texture of the overall slide, you can instantly lock the skin moisture and nutrition! Shrink pores, back red calm!

Accessories list accessories

Main body	1pcs
Clean water bottle	1pcs
Waste water bottle	1pcs
Hydra handle	1pcs
Hydra facial tips	8pcs
Micro-dermabrasion handle	3pcs
Micro tips	9pcs
Oxygen spray handle	1pcs
Cold hammer	1pcs