

User Manual

SR-DF901



Preface

Dear users:

We're pleased to present to you the 5-in-1 *Hydro Dermabrasion Skin Rejuvenation Machine* that's featuring in deep cleaning, hydration, skin rejuvenation and anti-aging. It is a multifunctional beauty machine, targeted at various skin problems such as dehydrated skin, sagging and loose skin, uneven skin tone and so on. Aqua Clean, in particular, can be used to eliminate active free radicals, repair from the most basic cell level and to combat the skin damage caused by free radicals. We aim at a safe and, at the same time, effective beauty result by using high-end technologies. 5-in-1 *Hydro Dermabrasion Skin Rejuvenation Machine* is specialized skin care machine for professional use, which needs to be handled by professionals with proper training. Any improper use will result in adverse consequences. Therefore we advise any personnel read this guide thoroughly and follow the instructions strictly before operation.

We believe our quality product will bring in good returns on your end. Also, you can count on our worry-free after-sales service.

Thank you!

Table of Contents

Preface
Part I
Brief Introduction
Advantages
Working Principles & Treatments
Skin Replenishment
Soothing & Repairing
Cell Activation
Skin Anti-aging
Skin Rejuvenation
Overall Facial Care
Part II
1. Detailed Operations
2. Technical Parameters
3. Safety Precautions
4. Dos and Don'ts
5. Troubleshooting & Solutions
6. FAQs
7. Packing List
8. Operational Diagrams

Part I

Brief Introduction

Hydro dermabrasion, RF and ultrasonic beauty machines are currently the most popular beauty machines that work on deep cleaning, hydrating, brightening and skin rejuvenation. It's an effective replacement of all the traditional facial cleaning and facial care programs in beauty salons. Moreover, it's convenient and easy to operate. It solves various skin issues for beauty-lovers. This equipment requires no injection, no medication, no operation, thus having no side-effects. It is used externally during the whole process and has an immediate effect. Through the heating effect caused by high-frequency vibration, RF can effectively introduce beauty nutrients into the skin quickly, so that the skin can better absorb the nutrients in the skin care products, so as to remove the dullness of the skin, repair skin, brighten skin and so on. At the same time, the combination of spray jet can also help inject nutrients into skin, achieving effects such as brightening, hydrating and repairing and so on.

Advantages

1. Aqua Clean can deep clean active free radicals, effectively help skin detoxify, rejuvenate and brighten.
2. Hydrogen-oxygen separation technology, which separates oxygen-rich ionized water while producing hydrogen ionized water can remove environmental free radicals.
3. 5-in-1 beauty machine that works on your overall facial care.
4. Multiple working probes can be replaced in different parts according to different requirements.
5. Suit all skin type and help improve skin quality and soothes skin.
6. The high frequency radio waves of radio frequency make the dermis get hot and warm, promote the metabolism of skin and the regeneration of collagen, thus lift the skin, dispel wrinkles, combat micro-aging, and make the skin firm and delicate.
7. Painless, non-invasive during the process. Skipping any recovery periods without affecting any work and life plans.
8. No consumption, low cost and quick returns.
9. Wider range of treatment and rapid and significant effect.
10. It will not cause any lumpiness, bleeding and swelling and stasis.
11. The heating effect of ultrasonic wave generated by high-frequency vibration can effectively introduce beauty nutrients into the skin quickly, so that the skin can better absorb the nutrients in skin care products.
12. The spray jet can inject nutrients into skin through stronger pressure, achieving skin brightening, hydrating and repairing and so on.

Skin Replenishment

Principles

Hydro Dermabrasion

Hydro dermabrasion utilizes water through purification into hydrogen and oxygen ionized water. It produces H₂ fine molecules in the skin surface, so that the water molecules cells can quickly penetrate into the skin, thus achieve the efficacy of hydration and brightening. Based on the traditional hydro dermabrasion, Aqua Clean added the rotary cleaning head, which doubles the cleaning effect.

The vacuum circuit is formed through vacuum negative pressure, and the ultra-small bubbles and nutrient solution are fully combined, and the spiral-shaped suction tip is specially designed to act directly on the skin, and the ultra-small bubbles can be kept in contact with the skin for a long time to induce the peeling effect. (Negative pressure: it's below normal pressure. The use of negative pressure is very common. People often make use of the omnipresent pressure by creating negative pressure at some space. For example, when people breathe, negative pressure occurs when the lungs are in an expanded state. A difference in pressure is formed between the inside and outside of the lungs, and fresh air is pushed into the lungs.) Hydro facial cleaning uses the exact principle in the state of no pain, achieving deep removal of various impurities in the hair follicle, mites and grease residue.

Physical Effects

With the most advanced moisture stripping technology, the skin can fully absorb the moisture of nutrient solution. Because the diameter of ultra-micro small bubble is only 10 microns, thus can easily penetrate into the pores, take away the dirt within the pores. A large number of bubbles dissolved in water will rupture, during which ultrasonic energy released can produce a large number of oxygen anion. This kind of bubble water can kill bacteria, enhance human immunity, purify the blood, improve allergies, beauty and skin care, moisturize skin, and reduce acne.

Hydrogen Water:

Rich in hydrogen, it also known as water element. Added in an appropriate amount of hydrogen water, it tastes very neutral and is similar to boiling water or pure water, colorless and tasteless.

Properties: Hydrogen water is the best antioxidant, combing high hydrogen, weak alkalinity, negative potential, and small molecules of water, it helps balance the body PH. It enters easily to the cell channel, into our body, where the reactive oxygen free radicals will be H (atomic hydrogen) that is, the active hydrogen decomposed eliminated, thus improving the body's immunity. Hydrogen water has more than all the known antioxidants such as vitamin A, C, E, green tea, and so on. The negative potential surprisingly reaches -500mv, comprehensively clearing the body of malignant reactive oxygen species (free radicals).

Effects:


Clean: Repair the defense function of epidermal layer, purify the keratosis insufficiency and fibrotic cells caused by free radicals

Protect: Protect and nurture dermis, prevent free radicals from degrading collagen, fiber bundles and hyaluronic acid.

Clear: Target environmental free radicals, clean PM2.5, dust, mites, bacteria, exhaust.

Detox: Block the attack of free radical to cell membrane, nucleus, DNA, detox the cells from inside out.

Water Vacuum To Attract Accumulation Of Sweat Pores And Skin Pores, Removal The Accumulation Of Dirt, Purifying Pores Blackheads, Tigh Pore.

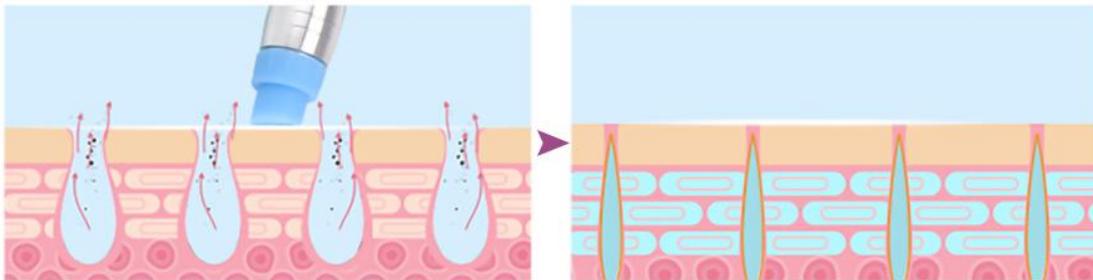


Hydro Spray Jet

Using high pressure oxygen and water, it sprays water flow on the skin with molecular particles less than 80 microns. The active water molecules and pure oxygen then quickly enters into the deep layer of the skin, accelerating the skin nutrition absorption. Through high pressure force into the pores of the skin and skin crack by skin penetration into the dermis layer so as to promote the cell regeneration, it directly and quickly provides adequate nutrition for the skin, removes rubbish from deep skin layer at the same time. Hyperbaric oxygen and nutrients can stimulate the fibrous tissue of the dermis regeneration, promote metabolism, thus achieves effects like hydrated, tender and glowing skin.

Physical Effects

The hydro spray jet is equipped with a 8 kg pressure oxygen pump. It adopts the advanced principle of pressure swing adsorption (PSA, quick separation of high concentration of pure oxygen in the air, and through the pulse type push to mix 80% of pure oxygen molecules and 0.9% of the water molecules to water and transport them into cells in deeper layers of skin, and replace waste toxins wherein, quickly produce energy, promote metabolism, so as to achieve the clean, translucent and glowing skin.

The Result Of Absorb Of The Serum Or Essence Will Be More Better If the Waste And Dirt Of The Pores Have Been Removed Totally.

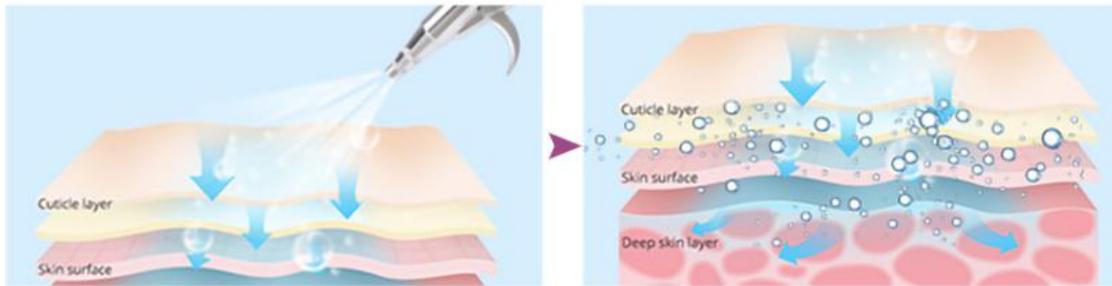

Hot Hammer

Special wafer ceramics are used to absorb energy. The method of physical heat accumulation produced by wafer energy absorption makes one side produce heat up to 40 degrees. The heat energy help imports nutrients into the skin for better absorption. It also boosts blood circulation and skin metabolism.

Biological Effects

Hot hammer heating can make you feel warm, comfortable, relax muscles, balance the autonomic nerve, make pore spread on the skin, blood vessels, nutrient is absorbed quickly. It is the most gentle and comfortable import method. Alternating use of beauty has the effect of addition and multiplication, which can naturally thoroughly activate cells, improve skin quality, stimulate blood circulation acceleration, and increase the skin's metabolism.

Effects

1. Deep cleanse gunks and exfoliate.
2. Dredge pores and promote absorption.
3. Improve oil secretion and skin quality.
4. Accelerate absorption and boost metabolism.
5. Clean cosmetic residues in the skin and remove skin allergens and mites.
6. Improve hair folliculitis, dull skin, and improve skin quality.
7. Remove blackheads, shrink pores and increase skin transparency.

Applicable Range

1. Those with dehydrated and malnourished skin.
2. Those with large pores and rough skin.
3. Those with poor skin absorption and lack of water in the skin base.
4. Those with dark and uneven skin tone.
5. Those with excessive oil secretion, oily skin.
6. Those with sallow skin that lack in glowing.

Inapplicable Range

1. Those with severely sensitive skin or those in allergic phase.
2. Those with extensive facial trauma.
3. Those who are pregnant and those suffering from serious heart disease, severe three H's(hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc.
4. Those with skin diseases and infectious diseases.
5. Those who have just received injection, such as intradermal injection, micro-needling, or with skin cuts.

Notes after Operation

1. Strengthen hydration and pay attention to sun protection.
2. Do not use stimulating skin care products including scrub, exfoliating and AHAs within one week after operation.
3. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
4. Apply a hydrating mask once a day for three days.
5. Wash your face with warm water within 3 days after operation.

Soothing&Repairing

Principles


Cold Hammer

Use the principle of air conditioner. Special wafer ceramics are used to absorb energy and refrigerate at - 5 degrees in an instant. The method of physical heat accumulation produced by wafer energy absorption makes one side produce heat up to 40 degrees. The fan discharges excess heat energy and achieves the effect of constant temperature and freezing alternately. It also promotes metabolism, alleviates allergic reaction, shrinks pore, calms skin, whitens and tenders skin.

Biological effects

During the cooling process, tissue temperature cools down, blood vessels and veins contract, metabolic rate reduces, cell activity inhibits, peripheral nerve sensitivity reduces, thus having a sedative and soothing effect, counteracting or reducing the symptoms of allergic reaction, such as redness, swelling, and pain, therefore achieving certain effects, such as calming the skin, reducing redness and swelling, shrinking pores and so on.

Use Cold Hammer on whole face, it can close pores and lock nutrition.

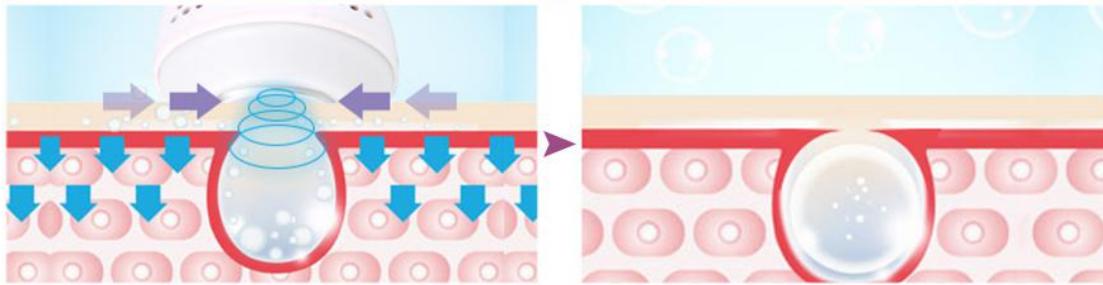


Hydro Spray Jet

Using high pressure oxygen and water, it sprays water flow on the skin with molecular particles less than 80 microns. The active water molecules and pure oxygen then quickly enters into the deep layer of the skin, accelerating the skin nutrition absorption. Through high pressure force into the pores of the skin and skin crack by skin penetration into the dermis layer so as to promote the cell regeneration, it directly and quickly provides adequate nutrition for the skin, removes rubbish from deep skin layer at the same time. Hyperbaric oxygen and nutrients can stimulate the fibrous tissue of the dermis regeneration, promote metabolism, thus achieves effects like hydrated, tender and glowing skin.

Physical Effects

The hydro spray jet is equipped with a 8 kg pressure oxygen pump. It adopts the advanced principle of pressure swing adsorption (PSA, quick separation of high concentration of pure oxygen in the air, and through the pulse type push to mix 80% of pure oxygen molecules and 0.9% of the water molecules to water and transport them into cells in deeper layers of skin, and replace waste toxins wherein, quickly produce energy, promote metabolism, so as to achieve the clean, translucent and glowing skin.

The Result Of Absorb Of The Serum Or Essence Will Be More Better If the Waste And Dirt Of The Pores Have Been Removed Totally.

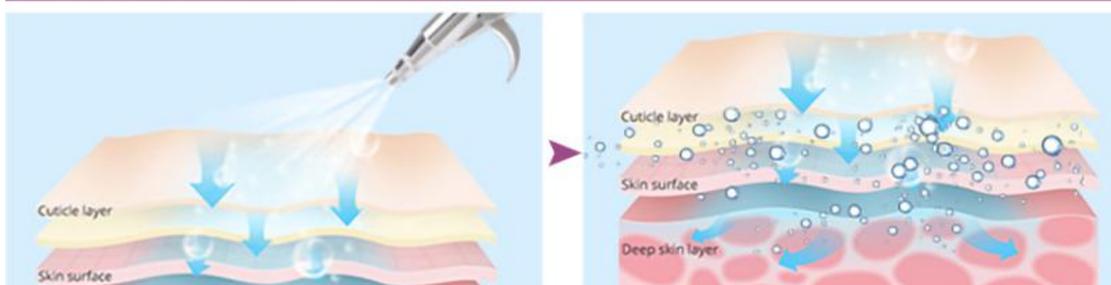

Effects

1. Fast calming, strong moisturizing.

2. Improve dull and sallow skin.
3. Improve and repair sensitive skin.
4. Improve large oil fields and shrink pores.
5. Soothe and repair skin after photoelectricity operation.
6. Relieve and repair slightly sensitive skin.

Applicable Range

1. Those with dry skin and lack of moisture.
2. Those with uneven, dull complexion.
3. Those with sensitive skin, acne-prone skin, and easily inflamed, or allergic skin.
4. Those with large, coarse pores, and vigorous oil secretion.
5. Those who have just received the photoelectric treatment.
6. Those with skin peeling and dandruff.

Inapplicable Range

1. Those who have just undergone invasive plastic surgery with unhealed wounds.
2. Those with skin diseases and infectious diseases.
3. Those who are unrealistic about the results.
4. Those who have just received injection products, such as hyaluronic acid, intradermal injection, wrinkle removal, or plastic surgery.
5. Those who are having a severe allergy (red rash).

Notes after Operation

1. Wash your face with warm water within 3 days after operation.
2. Strengthen hydration and pay attention to sun protection.
3. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
4. Apply a hydrating mask once a day for three days.
5. Avoid eating spicy or greasy food.

Cell Activation


Principles

Hydro Dermabrasion

Hydro dermabrasion utilizes water through purification into hydrogen and oxygen ionized water. It produces H₂ fine molecules in the skin surface, so that the water molecules cells can quickly penetrate into the skin, thus achieve the efficacy of hydration and brightening. Based on the traditional hydro dermabrasion, Aqua Clean

added the rotary cleaning head, which doubles the cleaning effect.

Water Vacuum To Attract Accumulation Of Sweat Pores And Skin Pores, Removal The Accumulation Of Dirt, Purifying Pores Blackheads, Tigh Pore.

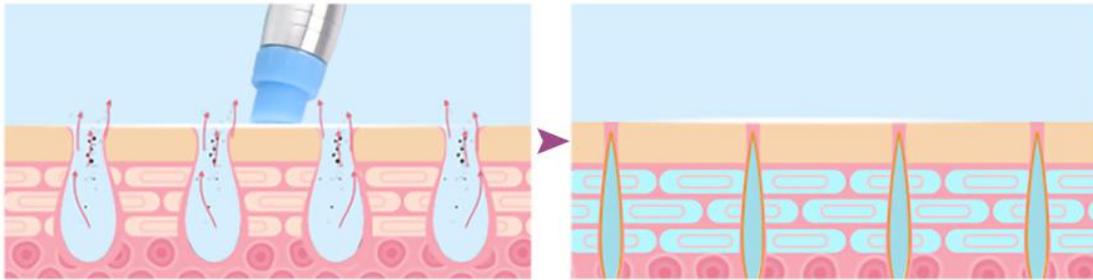

RF


RF works through the circuits formed by electrodes in pair. The optimum frequency of radio frequency biothermal effect is 3 MHz, and the maximum speed of extreme change is 8 million charged particles in tissues per second. In the rapid change of electrodes, the dermal tissue naturally resists the radio frequency current and generates thermal energy. During the process, RF promotes the metabolism and proliferation of skin collagen through high electromagnetic wave change, therefore heating up dermis layer of the skin, thus achieving effects like tightening, lifting, wrinkle removal, anti-aging and smooth skin.

Biological Effects

Radio frequency can shake millions of high frequency radio waves per second, penetrate the epidermis, directly act on the dermis, deep heat collagen cells, stimulate the growth of collagen fibers, so that it has a large number of new collagen support, making the skin more compact and elastic.

Every time energy comes into the skin, it feels a temporary sense of heat. This means that collagen has been stimulated to produce fever, resulting in tightening of the skin. While operating sliding, the radio frequency wave emitted by the therapeutic head of the super-frequency wave system, under the cold protection of the epidermis, has acted on the deep dermis through the epidermis, rapidly heating water molecules to produce safe Bio-thermal energy. When the natural friction of collagen tissue is heated up to $45^{\circ}\text{C}\sim 60^{\circ}\text{C}$, it will shrink immediately and stimulate the collagen to grow continuously. At the same time, bio-heat can effectively accelerate blood flow in adipocytes, and release free fatty acids (ffa), thereby increasing the dissolution of superficial fat, and achieve the purpose of tightening and lifting the sagging part of the body.

RF Can Deeply Heating The Collagen Cells, Stimulate The Growth Of The Collagen Fibers, Which make A Lot Of New Collagen To Support The Skin And Become More Compact And Elasticity.

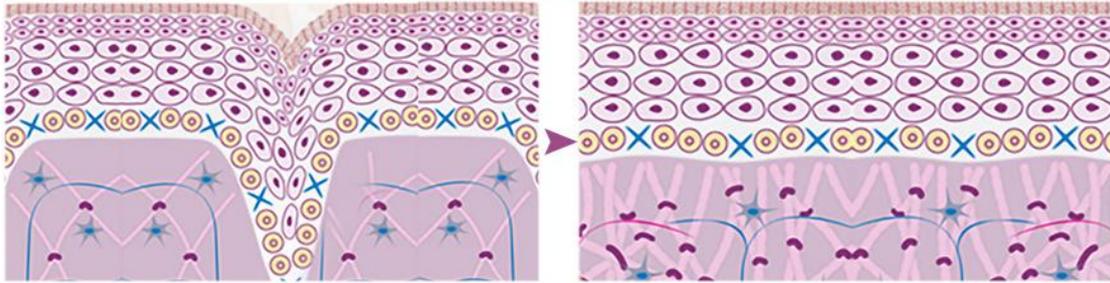

Ultrasound

Ultrasonic facial beauty instrument is based on the theory of high frequency vibration. It reaches deep into the subcutaneous tissues, 4-6 mm to be exact, with the strong vibration wave of 2.8MHz. With its high energy and permeability, it vibrates tissues in every layers of your skin. When using ultrasonic beauty instrument for massage, the warmth of ultrasonic wave can penetrate to 6MM under the skin, make the blood and lymph flow smoothly, accelerate the blood circulation of fine cells, start the biological effect of cell tissue, fade all kinds of color spots, whiten the skin and activate and regenerate the cells of deep tissue.

3MHz High Frequency Ultrasound Wave Help Nutrition Import Deeply To Dermis, To Refresh Skin Totally, It Can Do Lipolysis, Slimming Face.




Hydro Spray Jet

Using high pressure oxygen and water, it sprays water flow on the skin with molecular particles less than 80 microns. The active water molecules and pure oxygen then quickly enters into the deep layer of the skin, accelerating the skin nutrition absorption. Through high pressure force into the pores of the skin and skin crack by skin penetration into the dermis layer so as to promote the cell regeneration, it directly and quickly provides adequate nutrition for the skin, removes rubbish from deep skin layer at the same time. Hyperbaric oxygen and nutrients can stimulate the fibrous tissue of the dermis regeneration, promote metabolism, thus achieves effects like hydrated, tender and glowing skin.

Physical Effects

The hydro spray jet is equipped with a 8 kg pressure oxygen pump. It adopts the advanced principle of pressure swing adsorption (PSA, quick separation of high concentration of pure oxygen in the air, and through the pulse type push to mix 80% of pure oxygen molecules and 0.9% of the water molecules to water and transport them into cells in deeper layers of skin, and replace waste toxins wherein, quickly produce energy, promote metabolism, so as to achieve the clean, translucent and glowing skin.

The Result Of Absorb Of The Serum Or Essence Will Be More Better If the Waste And Dirt Of The Pores Have Been Removed Totally.

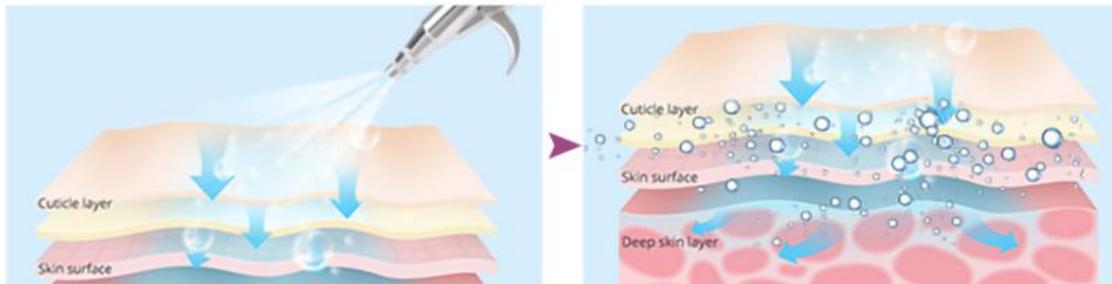

Effects

1. Deeply clean and shrink pores.
2. Dredge pores and promote absorption.
3. Deeply introduce nutrients to accelerate absorption.
4. Brightens and lifts the skin.
5. Quick hydrating, skin rejuvenation and brightening.
6. Remove dry lines, smooth fine lines and fade wrinkles.
7. Stimulate subcutaneous collagen regeneration and activate cells.
8. Resist wrinkles and prevent skin aging.

Applicable Range

1. Those with dry skin, fine lines and wrinkles.
2. Those with dark, dull skin.
3. Those with rough skin and large pores.
4. Those with excessive oil secretion and clogged pores.
5. Those with dehydrated and malnourished skin.
6. Those with poor skin absorption and lack of water in the skin base.
7. Those with skin aging caused by lack of sun protection.

Inapplicable Range

1. Those with severely sensitive skin or those in allergic phase.
2. Those with extensive facial trauma.
3. Those who are pregnant and those suffering from serious heart disease, severe

three H's(hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc.

4. Those with skin diseases and infectious diseases.
5. Those with metal implants such as stent, pacemaker, or those who are allergic to metals.
6. Those who have just received injection, such as intradermal injection, micro-needling, wrinkle removing or plastic surgeries.

Notes after Operation

1. Wash your face with warm water within three days.
2. Strengthen moisturizing and sunscreen.
3. Avoid sauna, hot spring or any strenuous exercise.
4. Do not use stimulating skin care products including scrub, exfoliating and AHAs within one week after operation.
5. Spicy, greasy food should be avoided. Staying up late, smoking and drinking are prohibited. More vegetables, fruits, and less greasy food.
6. Use facial masks at least three times a week.

Skin Anti-aging

Principles

RF

RF works through the circuits formed by electrodes in pair. The optimum frequency of radio frequency biothermal effect is 3 MHz, and the maximum speed of extreme change is 8 million charged particles in tissues per second. In the rapid change of electrodes, the dermal tissue naturally resists the radio frequency current and generates thermal energy. During the process, RF promotes the metabolism and proliferation of skin collagen through high electromagnetic wave change, therefore heating up dermis layer of the skin, thus achieving effects like tightening, lifting, wrinkle removal, anti-aging and smooth skin.


Biological Effects

Radio frequency can shake millions of high frequency radio waves per second, penetrate the epidermis, directly act on the dermis, deep heat collagen cells, stimulate the growth of collagen fibers, so that it has a large number of new collagen support, making the skin more compact and elastic.

Every time energy comes into the skin, it feels a temporary sense of heat. This means that collagen has been stimulated to produce fever, resulting in tightening of the skin. While operating sliding, the radio frequency wave emitted by the therapeutic head of the super-frequency wave system, under the cold protection of the epidermis, has acted on the deep dermis through the epidermis, rapidly heating water molecules to

produce safe Bio-thermal energy. When the natural friction of collagen tissue is heated up to $45^{\circ}\text{C}\sim 60^{\circ}\text{C}$, it will shrink immediately and stimulate the collagen to grow continuously. At the same time, bio-heat can effectively accelerate blood flow in adipocytes, and release free fatty acids (ffa), thereby increasing the dissolution of superficial fat, and achieve the purpose of tightening and lifting the sagging part of the body.

RF Can Deeply Heating The Collagen Cells, Stimulate The Growth Of The Collagen Fibers, Which make A Lot Of New Collagen To Suppot The Skin And Become More Compat And Elasticity.

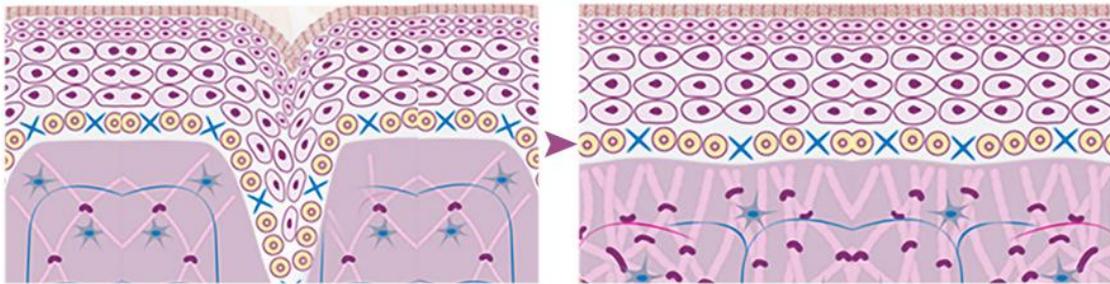


Cold Hammer

Use the principle of air conditioner. Special wafer ceramics are used to absorb energy and refrigerate at -5 degrees in an instant. The method of physical heat accumulation produced by wafer energy absorption makes one side produce heat up to 40 degrees. The fan discharges excess heat energy and achieves the effect of constant temperature and freezing alternately. It also promotes metabolism, alleviates allergic reaction, shrinks pore, calms skin, whitens and tenders skin.

Biological effects

During the cooling process, tissue temperature cools down, blood vessels and veins contract, metabolic rate reduces, cell activity inhibits, peripheral nerve sensitivity reduces, thus having a sedative and soothing effect, counteracting or reducing the symptoms of allergic reaction, such as redness, swelling, and pain, therefore achieving certain effects, such as calming the skin, reducing redness and swelling, shrinking pores and so on.

Use Cold Hammer on whole face, it can close pores and lock nutrition.

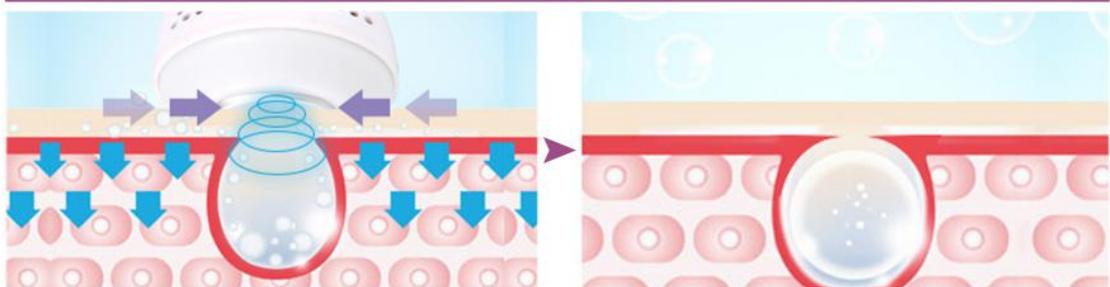

Effects

1. Improve facial relaxation and sagging.
2. Shape clear facial contour.
3. Improve facial dry lines, fine lines, wrinkles, etc.
4. Refine pores and minimize enlarged pores.
5. Improve uneven, dark and dull skin.
6. Strengthen the resistance of skin and repair fragile skin.

Applicable Range

1. Those with loose and sagging skin.
2. Those with dry skin, fine lines and wrinkles.
3. Those with dull, uneven and lusterless skin.
4. Those with coarse skin and large pores.
5. Those with skin laxity or edema after child delivery.
6. Those who work in office for long hours, facing the UV light.

Inapplicable Range

1. Those with severely sensitive skin or those in allergic phase.
2. Those with extensive facial trauma.
3. Those who are pregnant and those suffering from serious heart disease, severe three H's(hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc.
4. Those with skin diseases and infectious diseases.
5. Those with metal implants such as stent, pacemaker, or those who are allergic to metals.
6. Those who have just received injection, such as intradermal injection, wrinkle removal , or plastic surgery.

Notes after Operation

1. Wash your face with warm water within 3 days after operation.
2. Strengthen hydration and pay attention to sun protection.
3. Do not use stimulating skin care products including scrub, exfoliating and AHAs within one week after operation.
4. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
5. Apply a hydrating mask once a day for three days.
6. Avoid sauna, hot spring or any strenuous exercise.


Skin Rejuvenation

Principles

Hydra Dermabrasion

Hydra dermabrasion refers to that pure water will be purified into oxygen-hydrogen ionized water, which will produce H₂ fine molecules on the skin surface, so as to quickly permeate water molecules and cells into the skin, achieving the efficacy of being tender and translucent. On the basis of the traditional hydro facial, the innovative rotary cleaning grinding head is added, thus achieving a better skin cleaning effect.

Water Vacuum To Attract Accumulation Of Sweat Pores And Skin Pores, Removal The Accumulation Of Dirt, Purifying Pores Blackheads, Tigh Pore.

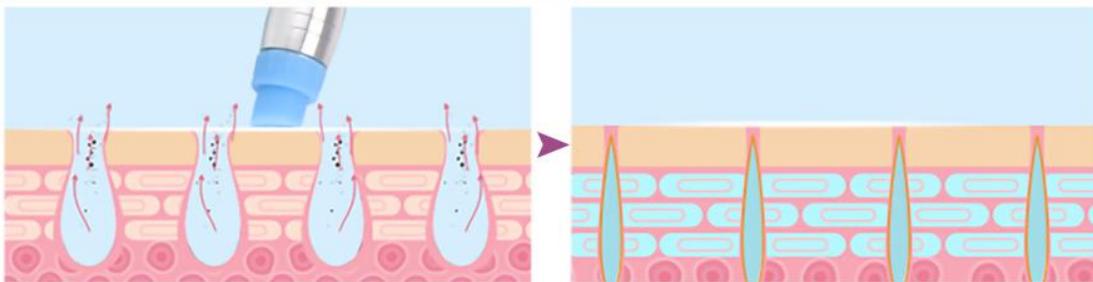


Hydro Spray Jet

Using high pressure oxygen and water, it sprays water flow on the skin with molecular particles less than 80 microns. The active water molecules and pure oxygen then quickly enters into the deep layer of the skin, accelerating the skin nutrition absorption. Through high pressure force into the pores of the skin and skin crack by skin penetration into the dermis layer so as to promote the cell regeneration, it directly and quickly provides adequate nutrition for the skin, removes rubbish from deep skin layer at the same time. Hyperbaric oxygen and nutrients can stimulate the fibrous tissue of the dermis regeneration, promote metabolism, thus achieves effects like hydrated, tender and glowing skin.

Physical Effects

The hydro spray jet is equipped with a 8 kg pressure oxygen pump. It adopts the advanced principle of pressure swing adsorption (PSA, quick separation of high concentration of pure oxygen in the air, and through the pulse type push to mix 80% of pure oxygen molecules and 0.9% of the water molecules to water and transport them into cells in deeper layers of skin, and replace waste toxins wherein, quickly produce energy, promote metabolism, so as to achieve the clean, translucent and glowing skin.

The Result Of Absorb Of The Serum Or Essence Will Be More Better If the Waste And Dirt Of The Pores Have Been Removed Totally.

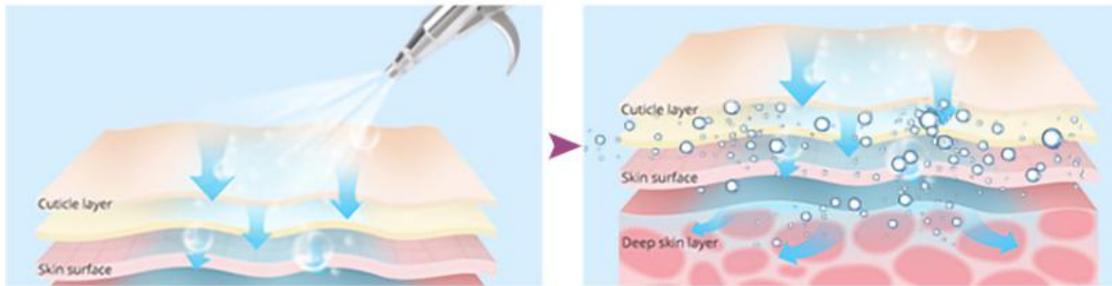

Ultrasound

Ultrasonic facial beauty instrument is based on the theory of high frequency vibration. It reaches deep into the subcutaneous tissues, 4-6 mm to be exact, with the strong vibration wave of 2.8MHz. With its high energy and permeability, it vibrates tissues in every layers of your skin. When using ultrasonic beauty instrument for massage, the warmth of ultrasonic wave can penetrate to 6MM under the skin, make the blood and lymph flow smoothly, accelerate the blood circulation of fine cells, start the biological effect of cell tissue, fade all kinds of color spots, whiten the skin and activate and regenerate the cells of deep tissue.

3MHz High Frequency Ultrasound Wave Help Nutrition Import Deeply To Dermis, To Refresh Skin Totally, It Can Do Lipolysis, Slimming Face.



Effects

1. Dredge pores and promote absorption.
2. Deeply moisturize and whiten skin.
3. Improve uneven and dull skin tone.
4. Whiten skin and increase skin gloss.
5. Deeply replenish nutrition and accelerate nutrient absorption.
6. Adjust the overall skin tone of the face to create moist, shiny skin.

Applicable Range

1. Those with dry skin and lack of nutrients.
2. Those with dull, uneven and lusterless skin.
3. Those with clogged pores and thick cuticle.

4. Those with poor skin absorption and metabolism.
5. Those with poor moisture lock-in and unbalanced skin water and oil.
6. Those who are dissatisfied with their skin.

Inapplicable Range

1. Those with severely sensitive skin or those in allergic phase.
2. Those with extensive facial trauma.
3. Those who are pregnant and those suffering from serious heart disease, severe three H's (hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc.
4. Those with skin diseases and infectious diseases.
5. Those with metal implants such as stent, pacemaker, or those who are allergic to metals.
6. Those who have just received injection, such as intradermal injection, wrinkle removal, or plastic surgery.

Notes after Operation

1. Wash your face with warm water within 3 days after operation.
2. Strengthen hydration and pay attention to sun protection.
3. Do not use stimulating skin care products including scrub, exfoliating and AHAs within one week after operation.
4. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
5. Apply a hydrating mask once a day for three days.

Overall Facial Care

Effects

1. Deeply clean skin and shrink pores.
2. Adjust the imbalance of skin water and oil.
3. Dredge pores and reduce excessive oil secretion.
4. Deeply introduce nutrition to promote skin absorption.
5. Shape defined facial outline.
6. Stimulate subcutaneous collagen regeneration.
7. Improve skin relaxation, sagging and inelasticity.
8. Lift cheeks and the corners of the eyes.
9. Improve uneven, dull and lusterless skin.
10. Prevent skin from sagging and drooping, and delay aging.
11. Adjust the overall complexion of the face to create hydrated and glowing skin.

Applicable Range

1. Those with dehydrated and malnourished skin.
2. Those with coarse skin.
3. Those with clogged pores and thick cutin.
4. Those with strong oil secretion and unbalanced skin water and oil.
5. Those with inelastic skin and unclear facial outline.
6. Those with dry lines, fine lines and wrinkles.
7. Those with facial skin laxity and sagging.
8. Those with dull, uneven and lusterless skin.
9. Those with slight aging.
10. Those who work in office for long hours, facing UV light.
11. Those who are unsatisfied with their skin tone.

Inapplicable Range

1. Those with severely sensitive skin or those in allergic phase.
2. Those with extensive facial trauma.
3. Those who are pregnant and those suffering from serious heart disease, severe three H's (hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc.
4. Those with skin diseases and infectious diseases.
5. Those with metal implants such as stent, pacemaker, or those who are allergic to metals.
6. Those who have just received injection, such as intradermal injection, wrinkle removal, or plastic surgery.

Notes after Operation

1. Wash your face with warm water within 3 days after operation.
2. Strengthen hydration and pay attention to sun protection.
3. Avoid sauna, hot spring or any strenuous exercise.
4. Do not use stimulating skin care products including scrub, exfoliating and AHAs within one week after operation.
5. Apply a hydrating mask once a day for three days.
6. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
7. Avoid food that will cause '3H' and eat less greasy food.

Part II

1. Detailed Operation

The detailed installment is as follows:



Connect the correct wire with each jack.



Press the white button at the top of the bottle and at the same time pull to retract the bottles



Twist counterclockwise to open the lid and fill in/dump out water




The light turns on when the bottle is selected and in working mode



Pull where the arrow points at to open



the exhaust bottle

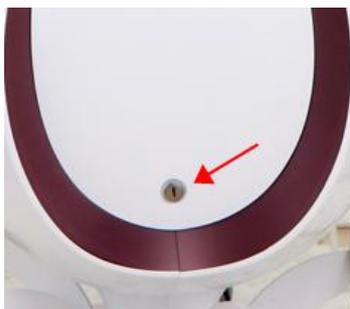

Open with a key to check the inner condition

After checking that the instrument is connected properly, the following starting interface will appear when the power supply is turned on.



1. Detailed Operation: Hydro Dermabrasion/Aqua Clean



Click  to enter the interface below



Working time, range: 0-30min (recommended to be

8-20min for each spot)



Time+/-



Suction -



Suction +



Start/Pause



Suction (Suction can be adjusted based on one's own tolerance and skin condition. For first-time users, it is recommended start from the lowest, and then slowly turns up after adaptation)



Water Volume, click to select each bottle's outflow,

maximum volume is 99



Bottle A, for purified water to be converted to hydrogen-oxygen water, suitable for all skin type to cleanse, hydrate and eliminate free radicals




Bottle B, for deep cleanse solutions, such as AHA, plant extracts, gentle, suitable for all skin types to exfoliate and deep cleanse



Bottle C, for purified water to be converted to hydrogen-oxygen water, suitable for all skin types



Bottle D, for purified water and any solutions that contains VC, suitable for all skin types to brighten skin and deep hydrate

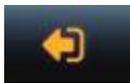

Return to home page

2. Detailed Operation: Ultrasound



Click  to go to the interface below



Working time, range: 0-30min (It is recommended to set between 8-120min for each spot)



Time+/-



Start/Pause



Energy -



Energy +



Energy (Higher energy level, strong ultrasonic vibration)




Consecutive mode, ultrasound is constantly at work after turning on the device, suitable for those with facial skin dehydration, wrinkles and coarse skin



Intermittent mode with a relatively fast interval frequency, ultrasound works intermittently after turning on the device, suitable for those skin dull and malnourished skin



Intermittent mode with a relatively slow interval frequency, ultrasound works intermittently and slowly after turning on the device, suitable for those who need to maintain current skin condition and delay aging

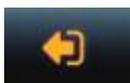

Return to home page

3. Detailed Operation: Hydro Spray Jet



Click  to go to interface below




Working time, range: 0-30min (adjust based on skin condition. For those with dry, dehydrated and sensitive skin, working time can be set relatively longer. 5-15 min for reference)



Time +/-



Start/Pause

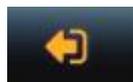


Return

4. Detailed Operation: Cool/cold Hammer



Click  to go to the interface below

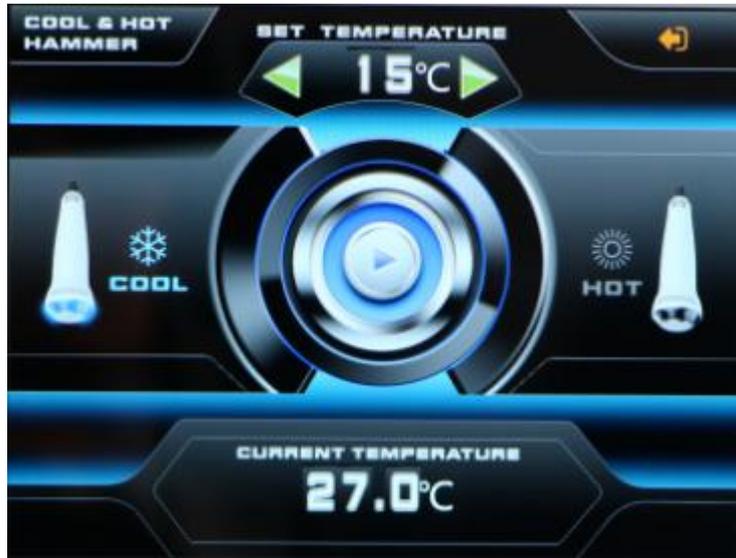

Cool Hammer Mode



Temperature, range: 15-20°C (Adjust based on each one's skin condition. 15-17°C for sensitive skin and acne skin; 17-20°C for normal skin)




Temperature +/-



Start/Pause



Current Temperature, changes after cooling hammer mode is turned on for a whole

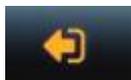


Return

6. Detailed Operation: Hot Hammer



Click [\[Image\]](#) to go to the interface below

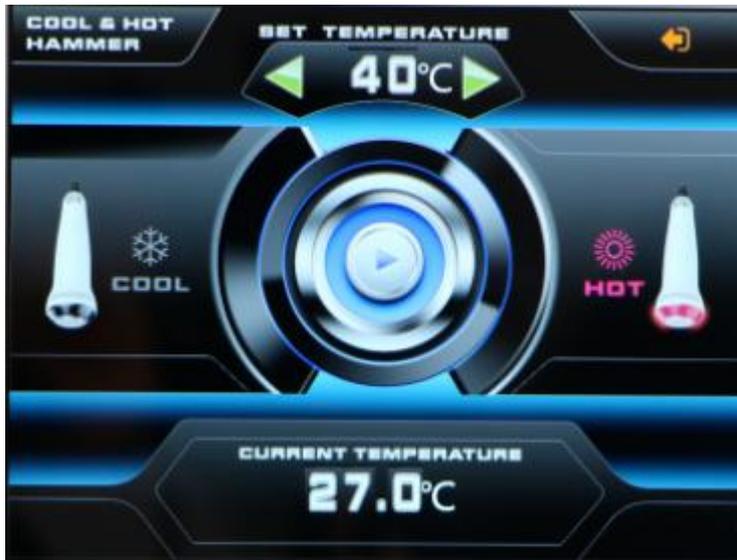

Hot Hammer Mode



Temperature, range: 40-55°C (Adjust based on each one's skin condition. 40-43°C for sensitive skin and acne skin; 44-55°C for normal skin)




Temperature +/-



Start/Pause



Current Temperature, changes after cooling hammer mode is turned on for a whole

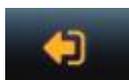

Return

7. Detailed Operation: RF



Click [RF](#) to go to the interface below



Working time, range: 0-30min (It is recommended to set between 8-15 min for each spot)



Time +/-



Start/Pause



Energy -



Energy +



Energy, adjust based on one's own comfort level. It is recommended to start from the lowest energy level and add up slowly for first-time users. Higher energy level translates to higher temperature




Consecutive mode, RF starts working after turning on the device, suitable for those who wish to tighten skin and define facial outline



Intermittent mode with a relatively fast interval rate. RF start emitting energy intermittently and quickly, suitable for those who need to fade wrinkles



Intermittent mode with a relatively slow interval rate. RF start emitting energy intermittently and slowly, suitable for first-time users and those who need to maintain the current skin condition and prevent aging

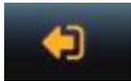

Return

2. Technical Parameters

Power supply Input: 100V-240V

Power: 190W

Ultrasonic handles for face

Frequency: 3MHz

Power: 50W

Cold therapy hammer

Temperature: 0~5 degree centigrade

Power: 10W

3. Safety Precautions

Please consult your doctor or professionals before using the device if you have the following symptoms:

1. Those who are pregnant or lactating.
2. Those with heart disease or pacemaker equipped.
3. Those whose wounds have not healed and who are recovering from the operation.
4. Those with epilepsy, severe diabetes and hyperthyroidism.
5. Those with malignant tumor, hemophilia or severe bleeding.
6. Those with skin diseases and infectious diseases.
7. Those with metal implants such as stent, pacemaker or those who are allergic to metals.

4. Dos and Dont's

1. Use a grounding pin and make sure the power socket of the device is properly connected before each operation.
2. Ensure that the voltage of the device is appropriate. If the voltage of the local power supply is unstable, we recommend that users increase to the matching power voltage.
3. In order to ensure the therapeutic effect and normal service life expectancy of the device, please use the specified accessories provided or recommended by the original manufacturer.
4. The device should not be placed in wet environment or near water, nor should they be directly exposed to sunlight.
5. Do not place the device near a strong heat source, because this may affect the life and normal use of the device.
6. Please restrain from using the device on eyes, thyroid, parathyroid, testicles, abdomen of pregnant women, and pacemakers.
7. Patients who are suffering from illness should consult with a doctor and gain permission from the doctor before using.
8. Please turn off the power switch of the device, and ensure that the total power supply of the device is turned off before the staff leaves after daily use, so as to ensure the safety of the electrical products.
9. Use device or train device operators in strict accordance with instructions in the user manual.
10. Start from the lowest energy level and slowly add up.
11. When using this device, the operating parts must be kept moist and dry skin should be avoided.
12. Please refrain from operating around eyes, ears, skin lesion or parts that had undergone plastic surgery.
13. Adjust suction subjecting to the customer's comfort level when using hydra dermabrasion handle.

14. Do not stay at one place for over 5 seconds to avoid scald or frostbite.
15. Remind customer to adjust breathing when operating on philtrum with polymer atomizing pen.
16. All vacuum tips of hydra dermabrasion can be cleaned with water and soaked in 75% alcohol.
17. Please clean the waste bottle immediately after the hydra dermabrasion treatment.
18. Hydra dermabrasion, ultrasound, and RF handles can be used alone or combined together for better beauty results.
19. Clean the instrument with normal saline after operation to ensure its cleanliness and hygiene to prolong its service life.
20. Rotary head cleaning: basic cleaning must be done to maintain the prevention of infection. After using the screw head, disposable toothbrush can be used to brush out the dirty stuff in the groove of the screw head, rinse it off with clean water, then soak with alcohol for 3 minutes, and finally rinse the alcohol ingredients.
21. Internal cleaning (after each bottle of solution is used):
Replace the probe with a cleaning head, fill the solution bottle with half a bottle of purified water, turn on the device, plug the probe by hand, adjust the water flow to the maximum, and let the water flow back to clean the residual solution in the pipe. Around a minute.
22. All parts of the instrument must be tightened, otherwise internal leakage will reduce air pressure, resulting in no suction or too little water flow.
23. Cover and pull all white dots to remove the bottles.

5. Troubleshooting & Solutions

1. The instrument can't start and the key lamp on the back of the instrument doesn't work?

- A. Ensure that the power cord is connected to a working power outlet.
- B. Ensure the fuse tube on the back of the instrument is not loose or being burnt.

2. No RF output of the instrument?

- A. Please check whether the plug connecting the instrument handle and the body is tightly connected.
- B. Please check whether the treatment area has been cleaned. Grease or oil essential oil products may cause poor contact between the treatment handle of the instrument and human body, resulting in no RF output.

3. No suction coming from the hydra dermabrasion and there's no nutrients outflow?

- A. Please check whether the waste water recovery bottle cap is tight or not, and whether the nutrient solution bottle is tight or not.
- B. Please check whether the button for adjusting suction and nutrient solution stays on 0.

C. Please check whether the output of nutrient solution is excessive.

4. No spray coming from the polymer atomizing pen?

A. Please check whether the nozzle is blocked.

B. Please check whether the regulating part of the nozzle tail is tightened.

5.The instrument can be activated, but the wrong information appears on the screen?

A. Please unplug the power plug behind the instrument and wait about 1 minute before restarting the instrument.

B. If the above method did not work, please contact the instrument distributor for assistance.

6.FAQs

1. Q: How long can I start seeing results of RF treatment?

A: Under normal circumstances within that day or a week. Skin collagen tissue is heated and produces contractions, and you can obviously feel skin tightening. Because RF works by stimulating subcutaneous tissue and bring about sustainable collagen regeneration, so the more you do it, the more visible effects you shall receive.

2. Q: Is RF harmful to skin?

A: RF treatment falls under the category of non-surgery operation. It stimulates collagen regeneration at the skin bottom and boosts metabolism, therefore has no damage to the skin. Partial redness and fever is the normal phenomenon of accelerated blood circulation, and will be resolved by oneself after a moment, so there's no need to worry.

3. Q: Will hydra dermabrasion treatment enlarge pores?

A: No. Hydro facial deep cleansing treatment follows the sequence of three steps. The first step is to use the export liquid to soften skin cutin layer; the second step is coupled with different solutions (AS1 AHA, AS2 BHA, AS3 nutrient solution; choose different solution according to different type of skin); the third step is to hydrate the skin. Hydra facial is suitable for all skin types and will not enlarge pores but rather refine skin.

4. Q: Can ultrasonic import help remove acne marks?

A: Ultrasonic import channels whitening nutrients to the skin by ultrasonic high frequency vibration, accelerate the absorption, inhibit melanin formation and fade spots, effectively brighten dull skin, and works wonder especially on dark spots caused by acne. It is green, ecological, simple, safe, reliable and has no side effect. General outdoor activities can be conducted after treatment.

5.Q: Can I wear makeup after using this instrument?

A: Absolutely, the skin after treatment is moist and plump, makeup will actually last longer. This is a safe, healthy, quick and effective way of nursing without puncturing skin or leaving scars.

6.Q: What is collagen?

A: Collagen is a kind of biological macromolecule substance, a kind of white, opaque and non-branched fibrous protein. It can supplement the nutrition needed by all layers of the skin, enhance the collagen activity in the skin, lock moisture, nourish the skin, delay aging, beautify, relieve facial relaxation, provide nutrients for the hair and so on. Collagen is a nutrient that human body must replenish to delay aging. With the growth of age, collagen will gradually be lost. At the age of 20, women have begun to age, collagen gradually decrease. At the age of 25, collagen loss reaches its peak. At the age of 40, the content is less than half of which one has at age 18. It is the loss of collagen and water that causes the wrinkles on the face of the elderly. The breakage of collagen fibers and elastic mesh will cause skin tissue to be oxidized, atrophied and collapsed. The skin will be dry, wrinkled, flabby and inelastic etc.. Therefore, in order to delay aging, collagen must be supplemented.

7. Q: What are free radicals?

A: Free radicals are a kind of harmful substances with strong oxidizability produced by mitochondria in the process of making use of oxygen. Also known chemically as a "free group," it contains a group of atoms that are not paired with electrons. It's trying to take an electron away from something else and make itself into a stable substance.

8. Q: What are antioxidants?

A: In the living system, the damage of reactive oxygen species to human body is actually an oxidation process, so to reduce the damage of free radicals, it is necessary to start from the antioxidant. Free radicals do not only exist in the body, they also come from the body. Then, there are two ways to reduce the damage of free radicals: one is, the use of endogenous free radical scavenging system, the elimination of excess free radicals; Second, exogenous antioxidants - free radical scavengers, block the invasion of free radicals on the body.

9. Q: The harm of free radical to skin?

A: Human aging process is like oxidation. What let our body cells rusty are free radicals. They first attack the skin cell membranes, destroy membranes' elasticity and suppleness, eventually its function. The balance of skin cells are damaged and skin cell immunity ability will drop rapidly, resulting in skin aging, wrinkles, loss of elasticity, luster, or even pathological changes.

7. Packing List

- 1 x Power line(We offer 100-240 V, AU/EU/UK/AU plug)
- 1 x Cold&hot hammer
- 1 x Hydro handle
- 1 x 3MHZ ultrasonic
- 1 x Face RF handle
- 1 x Spray jet handle
- 4 x Nutrient bottle
- 1 x Main machine

8.Operational Diagram

Parameter Adjustment	Product	Technique	Diagram
Skin Replenishment: 50 min, once/week			
<p>Hydro dermabrasion energy level: 20%-80%</p> <p>Operate according to different skin types of customers (try the set energy on lower arm first then on face, meanwhile check with customers constantly.)</p>	<p>Makeup remover+ Face wash+ Blackhead export liquid+ Base mask+ Cold and hot steam+ Serum+ Facial mask+ SR-DF901</p>	<ol style="list-style-type: none"> 1. Remove makeup and clean face, 5min. 2. Apply toner, 1 min. 3. Apply blackhead export liquid+hot steam (on large pores or clogged area, and toner on cheeks; for sensitive skin it is not recommended to do so but apply toner+cold steam) 5-8 min. 4. Hydro dermabrasion operation: Press gently on forehead with the crystal tip for 3-5 times. 5. Then brace the forehead tightly and stay for a second, and then gently twist the probe with 	<p>Technique5</p>  <hr/> <p>Technique10</p>  <hr/> <p>Technique12</p>

Hydro Spray Jet
energy level:
Liquid export
volume is
recommended
to set as small

Hot&Cold
Hammer
Energy:
40°C-45°C

Mode: HOT

your finger and pick it up;
each treatment area
should be right next to
another (forehead → nose →
chin → cheeks) you can
double the operation on T
zone where pores are
severely clogged, 10-15
min.

6. Clean the treatment area, 2 min.
7. Apply cleansing base mask+hot steam, 10 min.
8. Clean the treatment area, 2 min.
9. Pour (hydrating) essence into the polymer atomizing bottle (with the proportion of serum:distilled water being 2:1), 2 min.
10. Spray Jet operation: Start from chin to earlobe, then from mouth corner to ear gate. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.
11. Do the same on the other side.
12. Brace the skin vertically towards hairline on forehead to keep the treatment surface tight, 2 times.
13. Apply (hydrating) serum evenly on the face, 1 min.
14. Cold&hot hammer (hot hammer) operation: move the probe in circles from jaw to earlobe, mouth corner to ear gate, nose



Technique14



Technique16



		<p>wing to the temple, 3 times.</p> <p>15. Do the same on the other side.</p> <p>16. Lift towards hairline on forehead, 3 times.</p> <p>17. Apply facial mask, 15 min.</p> <p>18. Wash face clean, 2 min.</p> <p>19. Apply toner, essence, cream, and sunscreen.</p>	
--	--	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

Recommended treatment:

10 treatments account for one full course. After one treatment, skin is thoroughly cleaned and fully hydrated. After one full course of treatment, skin will become evenly brightened with minimized pores. After three courses, skin appears to be plump, rosy, elastic, bright and can lock in moisture for 24 hours.

Soothing&Repairing: 50 min, once/week

<p>Hot&Cold Hammer</p> <p>Energy: 15°C</p> <p>Mode: COOL</p> <p>Hydro Spray</p> <p>Jet energy level:</p> <p>Liquid export volume is recommended to set as small</p>	<p>Makeup remover+</p> <p>Face wash+</p> <p>Hydrating base mask+</p> <p>Cold and hot steam+</p> <p>Serum+</p> <p>Repairing facial mask+</p> <p>SR-DF901</p>	<ol style="list-style-type: none"> 1. Remove makeup and clean face, 5min. 2. Apply toner, 1 min. 3. Apply hydrating base mask+cold steam, 15 min. 4. Wash face clean, 2 min. 5. Pour water into the spray jet bottle, 1 min. 6. Hydro spray jet operation: Start from chin to earlobe, then from mouth corner to ear gate. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times. 7. Move vertically towards hairline on forehead, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times. 8. Apply (soothing&repairing) 	<p>Technique6</p> 
			<p>Technique7</p> 
			<p>Technique9</p> 
			<p>Technique11</p>

	<p>serum on face evenly, 1 min.</p> <p>9. Spray jet operation: Coupled with hand, move the probe from chin to earlobe, mouth corner to ear gate, nose wing to the temple, 2 times.</p> <p>10. Do the same on the other side.</p> <p>11. Coupled with hand, lift towards hairline on forehead, 2 times.</p> <p>12. Apply (soothing) essence on the face, 1 min.</p> <p>13. Cold hammer operation: Move the probe upwards in circles, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple, 3 times.</p> <p>14. Do the same on the other side.</p> <p>15. Lift towards hairline on forehead, 3 times.</p> <p>16. Apply repairing facial mask, 15 min.</p> <p>17. Remove the mask and clean, 2 min.</p> <p>18. Apply toner, serum, cream, and sunscreen.</p>	 <p>Technique13</p>  <p>Technique15</p> 
--	------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

Recommended treatment:

10 treatments account for one full course. After one treatment skin is hydrated and smooth. After a full course, skin regains its balance and will be less prone to allergy caused by change of seasons, contact with cosmetics or skin dehydration. After two courses, skin is smooth, glossy with a well functioning blood circulation and of velvety touch. Three courses help enhance the effect and prevent skin allergy.

Cells Activation: 70 min, once/week

<p>Aqua Clean energy level: 20%-80%</p> <p>Operate according to different skin types of customers (try the set energy on lower arm first then on face, meanwhile check with customers constantly.)</p> <p>RF Energy: 20%-80%</p> <p>Mode: Consecutive</p> <p>Hydro Spray Jet energy level: Liquid export volume is recommended to set as small</p>	<p>Makeup remover+ Face wash+ Blackhead Export Liquid+ Hydrating Mask+ Cold and hot steam+ Essence+ Facial mask+ SR-DF901</p>	<ol style="list-style-type: none"> 1. Remove makeup clean face, 5 min. 2. Apply toner, 1 min. 3. Apply blackhead export liquid+hot steam(on large pores or severely clogged area; apply toner on cheeks; it is recommended to apply toner+cold steam for sensitive skin rather than export liquid+hot steam), 5-8 min. 4. Aqua Clean operation: stay on forehead and press for 3-5 times. 5. Stay on forehead, then gently twist and leave it up in the air(forehead→nose→chin). Double the techniques on T zone where pores are severely clogged, 10-15min. 6. Clean the treated area, 2 min. 7. Apply hydrating base mask+hot steam, 15 min. 8. Remove the mask and clean, 2 min. 9. Massage face and apply massage cream on face evenly and caress the treatment area, 3 times. 10. Press on the following acupoints: <i>Saliva Container(Ren-24), Earth Granary(ST-4), Great Welcome(ST-5), Jaw Bone(ST-6), Below the Joint(ST-7), Welcome Fragrance(LI-20), Cheek Bone Crevice(SI-18), Temples(EM5),</i> 	<p>Technique5</p>  <hr/> <p>Technique10</p>  <hr/> <p>Technique16</p>  <hr/> <p>Technique17</p>  <hr/> <p>Technique19</p>  <hr/> <p>Technique20、 21</p>
----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

Bright Eyes(BI-1), Gathered Bamboo(BI-2), Yuyao(EM3), Silken Bamboo Hollow(TE-23), Pupil Crevice(GB-1), Container of Tears(ST-1), Four Whites(ST-2), 3 times.

11. Caress the whole face, 3 times.

12. Lift the treatment area with both hands as if plucking the string, 1 min.

13. Alternate hands in lifting the treatment area on one side as if plucking the string, 30 sec.

14. Do the same on the other side.

15. Caress the whole face, 3 times.

16. Alternate hands in lifting in three lines, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple on one side of the face, 3 times.

17. Lift towards hairline on forehead, 3 times.

18. Do the same on the other side.

19. Brace skin with middle and ring fingers on forehead as if writing 'Z', slide to the ear back and force and finish behind ear, 3 times.

20. RF operation: Start from chin, move the probe in circles, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple, 3 times.

21. Repeat the last technique, combing with hand, 3 times.

22. Do the same on the other



Technique23、 24



Technique27



Technique29



Technique31



Technique33



		<p>side.</p> <p>23. Lift towards hairline on forehead, 3 time.</p> <p>24. Coupled with hand, lift towards hairline on forehead, 3 times.</p> <p>25. Clean the treatment area, 2 min.</p> <p>26. Pour (hydrating) essence into the polymer atomizing bottle(with the proportion of serum:distilled water being 2:1), 2 min.</p> <p>27. Spray Jet operation: Start from chin to earlobe, then from mouth corner to ear gate. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>28. Do the same on the other side.</p> <p>29. Brace the skin vertically towards hairline on forehead to keep the treatment surface tight, 2 times.</p> <p>30. Apply (hydrating) serum evenly on the face, 1 min.</p> <p>31. Ultrasonic operation: Start from chin to earlobe, then from mouth corner to ear gate, nose wing to temple. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 3 times.</p> <p>32. Do the same on the other side.</p> <p>33. Move the probe horizontally in circles on forehead, 3-5 times.</p> <p>34. Apply facial mask, 15 min.</p>	
--	--	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

		<p>35. Remove the mask and wash face clean, 2 min.</p> <p>36. Apply toner, essence, cream and sunscreen.</p>	
--	--	--------------------------------------------------------------------------------------------------------------	--

Recommended Treatment:

10 treatments account for one full course. After one treatment, skin is clean and translucent. Fine lines are faded. After a full course, fine line and wrinkles area faded. After two courses, skin is hydrated, brightened and elastic. Three courses help main skin luster. RF+ultrasound can stimulate collagen regeneration, activate cells, making skin full of elasticity and collagen.

Skin Anti-aging: 60 min, once/week

<p>RF Energy: 20%-80%</p> <p>Mode: Consecutive</p> <p>Hot&Cold Hammer</p> <p>Energy: 15°C</p> <p>Mode: COOL</p>	<p>Makeup remover+ Base mask+ Cold and Cold&hot steam+ Massage cream+ Serum+ Facial mask+ SR-DF901</p>	<ol style="list-style-type: none"> 1. Remove makeup and clean face, 5 min. 2. Apply toner, 1 min. 3. Apply hydrating base mask+hot steam, 10 min. 4. Clean the treatment area, 2 min. 5. Massage face and apply massage cream on face evenly and caress the treatment area, 3 times. 6. Press on the following acupoints: <i>Saliva Container(Ren-24), Earth Granary(ST-4), Great Welcome(ST-5), Jaw Bone(ST-6), Below the Joint(ST-7), Welcome Fragrance(LI-20), Cheek Bone Crevice(SI-18), Temples(EM5), Bright Eyes(BI-1), Gathered Bamboo(BI-2), Yuyao(EM3), Silken Bamboo Hollow(TE-23), Pupil Crevice(GB-1), Container of Tears(ST-1), Four</i> 	<p>Technique6</p>  <hr/> <p>Technique12</p>  <hr/> <p>Technique13</p>  <hr/> <p>Technique15</p>
-------------------------------------------------------------------------------------------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

	<p><i>Whites(ST-2)</i>, 3 times.</p> <p>7. Caress the whole face, 3 times.</p> <p>8. Lift the treatment area with both hands as if plucking the string, 2 min.</p> <p>9. Alternate hands in lifting the treatment area on one side as if plucking the string, 1 min.</p> <p>10. Do the same on the other side.</p> <p>11. Caress the whole face, 3 times.</p> <p>12. Alternate hands in lifting in three lines, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple on one side of the face, 3 times.</p> <p>13. Lift towards hairline on forehead, 3 times.</p> <p>14. Do the same on the other side.</p> <p>15. Brace skin with middle and ring fingers on forehead as if writing 'Z', slide to the ear back and force and finish behind ear, 3 times.</p> <p>16. Clean the treated area, 2 min.</p> <p>17. Apply (anti-aging/anti-wrinkle) serum evenly all over the face, 1 min.</p> <p>18. RF operation: Start from chin, move the probe in circles, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple, 3 times.</p> <p>19. Move the probe from chin to earlobe, mouth corner to ear gate, nose wing to the</p>	 <p>Technique18</p>  <p>Technique19、 20</p>  <p>Technique22</p>  <p>Technique23、 24</p>  <p>Technique26</p>  <p>Technique28</p>
--	------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

		<p>temple to lift the treatment area, 1 time.</p> <p>20. Repeat the last technique, combing with hand, 2 times.</p> <p>21. Do the same on the other side.</p> <p>22. Move the probe towards hairline in circles, 1 time.</p> <p>23. Lift towards hairline on forehead, 1 time.</p> <p>24. Coupled with hand, lift towards hairline on forehead, 3 times.</p> <p>25. Apply serum evenly all over the face, 1 min.</p> <p>26. Cold hammer operation: Move the probe horizontally in circles from chin to earlobe, mouth corner to ear gate, nose wing to temple, 3 times.</p> <p>27. Do the same on the other side.</p> <p>28. Move the probe horizontally in circles on forehead, 3 times.</p> <p>29. Apply facial mask, 15 min.</p> <p>30. Wash face clean, 2 min.</p> <p>31. Apply toner, essence, cream and sunscreen.</p>	
--	--	------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------

Recommended Treatment:
 10 treatments account for one full course. Once done, skin feels tightened. After a full course, skin is plump and rosy. Two courses help increase skin elasticity, boost skin metabolism, fade pigmented spots, brighten dull skin, and lower the chance of skin sensitivity. Three courses help stabilize skin condition, prevent skin sagging, aging and toning facial outline.

Skin Rejuvenation: 50 min, once/week

Aqua Clean energy level: 20%-80%	Makeup remover+ Face wash+ Blackhead	<ol style="list-style-type: none"> 1. Remove makeup and clean face, 5min. 2. Apply toner, 1 min. 3. Apply blackhead export 	Technique5
----------------------------------	--------------------------------------	-------------------------------------------------------------------------------------------------------------------------------------------------------------	------------

<p>Operate according to different skin types of customers (try the set energy on lower arm first then on face, meanwhile check with customers constantly.)</p> <p>Hydro Spray Jet energy level: Liquid export volume is recommended to set as small</p> <p>Ultrasonic Energy: 20%-80%</p> <p>Mode: Consecutive</p>	<p>export liquid+ Massage cream+ Base mask+ Serum+ Facial mask+ SR-DF901</p>	<p>liquid+hot steam (on large pores or clogged area, and toner on cheeks; for sensitive skin it is not recommended to do so but apply toner+cold steam) 5-8 min.</p> <p>4. Hydro dermabrasion operation: Press gently on forehead with the crystal tip for 3-5 times.</p> <p>5. Then brace the forehead tightly and stay for a second, and then gently twist the probe with your finger and pick it up; each treatment area should be right next to another(forehead→nose→chin→cheeks) you can double the operation on T zone where pores are severely clogged, 10-15 min.</p> <p>6. Clean face, 2 min.</p> <p>7. Apply whitening base mask+hot steam on face, 15 min.</p> <p>8. Remove the mask and clean, 2 min.</p> <p>9. Pour (hydrating/whitening) essence into the spray bottle(with the proportion of serum:distilled water being 2:1), 2 min.</p> <p>10. Spray Jet operation: Start from chin to earlobe, then from mouth corner to ear gate. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>11. Do the same on the other side.</p> <p>12. Brace the skin vertically</p>	<div style="text-align: center;">  </div> <hr/> <p style="text-align: center;">Technique10</p> <div style="text-align: center;">  </div> <hr/> <p style="text-align: center;">Technique12</p> <div style="text-align: center;">  </div> <hr/> <p style="text-align: center;">Technique14</p> <div style="text-align: center;">  </div> <hr/> <p style="text-align: center;">Technique16</p> <div style="text-align: center;">  </div> <hr/>
--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

		<p>towards hairline on forehead to keep the treatment surface tight, 2 times.</p> <p>13. Apply (whitening) serum evenly on the face, 1 min.</p> <p>14. Ultrasonic operation: Start from chin to earlobe, then from mouth corner to ear gate, nose wing to temple. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 3 times.</p> <p>15. Do the same on the other side.</p> <p>16. Move the probe horizontally in circles on forehead, 3-5 times.</p> <p>17. Apply facial mask, 15 min.</p> <p>18. Remove the mask and wash face clean, 2 min.</p> <p>19. Apply toner, essence, cream and sunscreen.</p>	
--	--	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

Recommended Treatment:

10 treatments account for one full course. After one treatment, skin is clean, translucent and hydrated. After a full course, pores are minimized and skin appears to be bright. After two courses, skin is fine, glossy, and hydrated. Three courses help conserve a natural clean reservoir, making skin glossy and elastic.

Overall Facial Care: 100 min, once/week

<p>Hydro dermabrasion energy level: 20%-80%</p> <p>Operate according to different skin types of customers</p>	<p>Makeup remover+</p> <p>Face wash+</p> <p>Blackhead export liquid+</p> <p>Base mask+</p> <p>Massage cream+</p> <p>Serum+</p>	<p>1. Remove makeup and clean face, 5min.</p> <p>2. Apply toner, 1 min.</p> <p>3. Apply blackhead export liquid+hot steam (on large pores or clogged area, and toner on cheeks; for sensitive skin it is not recommended to do so but apply toner+cold steam) 5-8 min.</p>	<p>Technique5</p>  <p>Technique10</p>
---------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------

<p>(try the set energy on lower arm first then on face, meanwhile check with customers constantly.)</p> <p>Hydro Spray Jet: Liquid export volume is recommended to set as small</p> <p>Cold&hot hammer energy: 15°C</p> <p>RF energy level: 20%-80%</p> <p>Mode: Consecutive</p> <p>Ultrasound energy level: 20%-80%</p> <p>Mode: Consecutive</p>	<p>Facial mask+ Cold and hot steam+ SR-DF901</p>	<p>4. Hydro dermabrasion operation: Press gently on forehead with the crystal tip for 3-5 times.</p> <p>5. Then brace the forehead tightly and stay for a second, and then gently twist the probe with your finger and pick it up; each treatment area should be right next to another(forehead→nose→chin→cheeks) you can double the operation on T zone where pores are severely clogged, 10-15 min.</p> <p>6. Clean the treatment area, 2 min.</p> <p>7. Apply hydrating base mask+hot steam, 10 min.</p> <p>8. Clean the treatment area, 2 min.</p> <p>9. Massage face and apply massage cream on face evenly and caress the treatment area, 3 times.</p> <p>10. Press on the following acupoints: <i>Saliva Container(Ren-24), Earth Granary(ST-4), Great Welcome(ST-5), Jaw Bone(ST-6), Below the Joint(ST-7), Welcome Fragrance(LI-20), Cheek Bone Crevice(SI-18), Temples(EM5), Bright Eyes(BI-1), Gathered Bamboo(BI-2), Yuyao(EM3), Silken Bamboo Hollow(TE-23), Pupil Crevice(GB-1), Container of Tears(ST-1), Four Whites(ST-2)</i>, 3 times.</p> <p>11. Caress the whole face, 3 times.</p> <p>12. Lift the treatment area</p>	<div style="text-align: center;">  <p>Technique16</p> </div> <div style="text-align: center;">  <p>Technique17</p> </div> <div style="text-align: center;">  <p>Technique19</p> </div> <div style="text-align: center;">  <p>Technique20</p> </div> <div style="text-align: center;">  <p>Technique22</p> </div> <div style="text-align: center;">  <p>Technique23</p> </div>
-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

		<p>with both hands as if plucking the string, 2 min.</p> <p>13. Alternate hands in lifting the treatment area on one side as if plucking the string, 1 min.</p> <p>14. Do the same on the other side.</p> <p>15. Caress the whole face, 3 times.</p> <p>16. Alternate hands in lifting in three lines, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple on one side of the face, 3 times.</p> <p>17. Lift towards hairline on forehead, 3 times.</p> <p>18. Do the same on the other side.</p> <p>19. Alternate middle and ring fingers in lifting from inner eye corner, tracing lower eye lid to the temple, 3 times on one side.</p> <p>20. Use scissor hand gesture to lift the eye corner on one side, 3 times.</p> <p>21. Do the same on the other side.</p> <p>22. Circles around eyes to caress the treatment area with both hands, 3 times.</p> <p>23. Brace skin with middle and ring fingers on forehead as if writing 'Z', slide to the ear back and force and finish behind ear, 3 times.</p> <p>24. RF operation: Start from chin, move the probe in circles, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple, 3 times.</p>	 <p>Technique 24、 25</p>  <p>Technique 27、 28</p>  <p>Technique 31</p>  <p>Technique 33</p>  <p>Technique 35</p>  <p>Technique 37</p>
--	--	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

	<p>25. Repeat the last technique, combing with hand, 2 times.</p> <p>26. Do the same on the other side.</p> <p>27. Lift towards hairline on forehead, 3 time.</p> <p>28. Coupled with hand, lift towards hairline on forehead, 3 times.</p> <p>29. Clean the treatment area, 2 min.</p> <p>30. Pour (hydrating) essence into the spray bottle(with the proportion of serum:distilled water being 2:1), 2 min.</p> <p>31. Spray Jet operation: Start from chin to earlobe, then from mouth corner to ear gate. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>32. Do the same on the other side.</p> <p>33. Brace the skin vertically towards hairline on forehead to keep the treatment surface tight, 2 times.</p> <p>34. Apply (anti-aging/anti-wrinkle) serum evenly on the face, 1 min.</p> <p>35. Ultrasonic operation: Start from chin, move the probe in circles, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple, 3 times.</p> <p>36. Do the same on the other side.</p> <p>37. Move the probe towards hairline in circles, 3-5 time.</p>	 <hr/> <p>Technique39</p> 
--	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

		<p>38. Apply facial mask, 15 min.</p> <p>39. Remove the mask and clean, 2 min.</p> <p>40. Apply toner, essence, the treat with hot(cold) hammer, covering the entire face, 2-3 times.</p> <p>41. Apply cream and sunscreen.</p>	
--	--	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--

Recommended treatment:

10 treatments account for one full course. After one treatment, skin is hydrated, brightened and a lifted canthus. After a full course, skin is plump, moisturized and even-toned. After two courses, skin becomes translucent and smooth. Three courses help accentuate skin gloss. RF stimulates collagen regeneration, delay aging and restore skin to its youthful state.