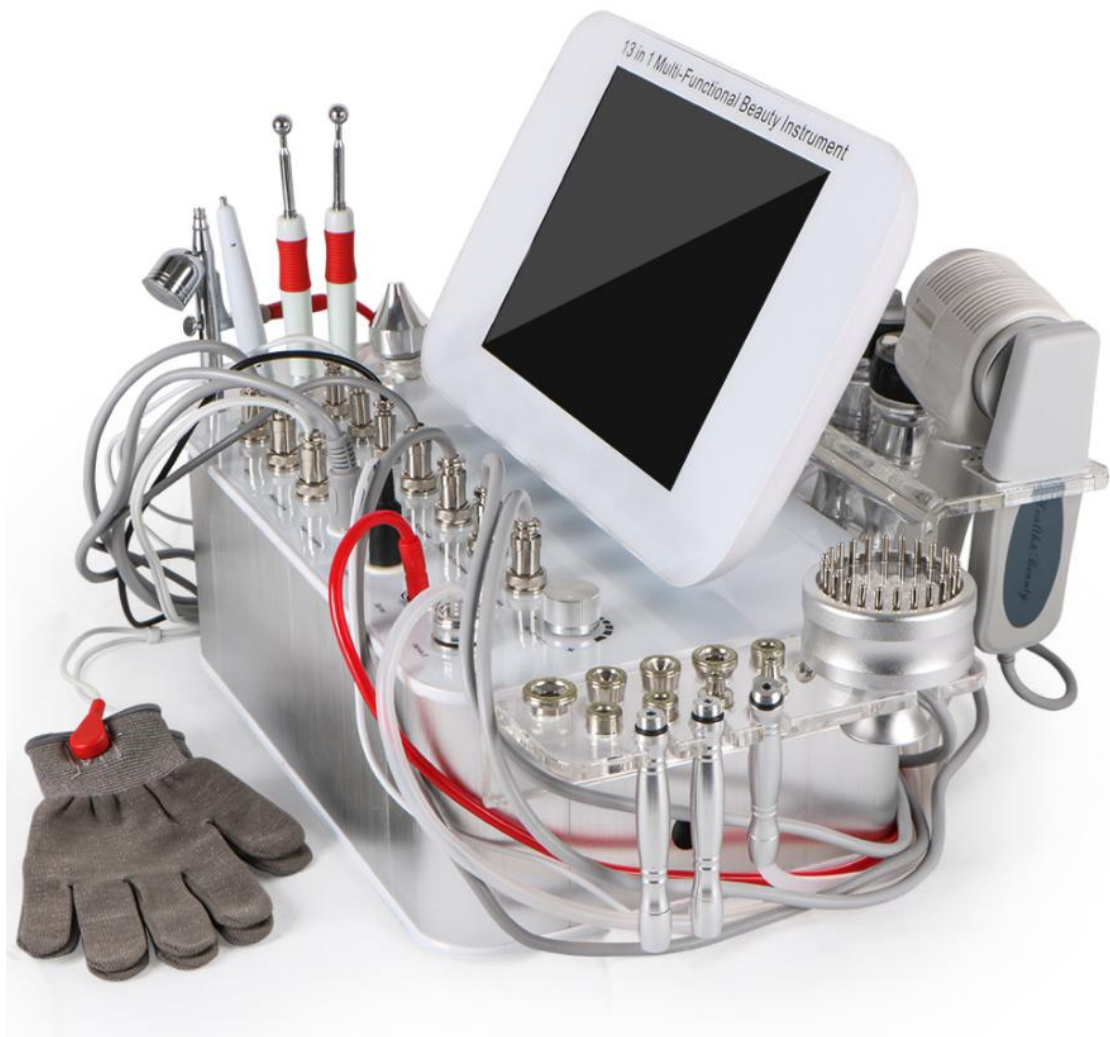


User Manual

SR-M51713



Preface

Dear users:

We're pleased to present to you our 14-in-1 beauty machine featured with tightening, lifting, acne and spots removal, lead in, cleaning and hydration. It is a multifunctional beauty machine, targeted at both face and eye area, mainly focusing on loose facial skin, lack of nutrition, dry skin, acne and sensitive skin and so on. RF, ultrasound, EMS and high frequency beauty machines are for professional use, which needs to be handled by professionals with proper training. Any improper use will result in adverse consequences. Therefore we advise any personnel read this guide thoroughly and follow the instructions strictly before operation.

We believe our quality product will bring in good returns on your end. Also, you can count on our worry-free after-sales service.

Thank you!

Table of Contents

Preface	
Part I	
Introduction	
Advantages	
Working Principles & Treatments	
Soothing & Repairing	
Anti-inflammation & Acne Treatment	
Anti-aging & Tightening	
Cleaning & Hydration	
Regain Skin Balance	
Eye Anti-aging	
Neck Anti-aging	
Spots Removal	
Part II	
1.Detailed Operations	
2.Technical Parameters	
3. Safety Precautions	
4. Dos and Don'ts	
5. Troubleshooting & Solutions	
6.FAQs	
7.Packing List	
8.Operational Diagrams	

Part I

Introduction

RF and ultrasonic and skin scrubber beauty machines are currently the most popular beauty machines that work on whitening, anti-aging, lead-in, and cleaning. It's convenient and easy to operate. It solves skin management issues for beauty-lovers. This equipment requires no injection, no medication, no operation, thus having no side-effects. It is used externally during the whole process and has an immediate effect. EMS uses electric current to stimulate muscle and lymph, leading to contraction of muscle, and thus achieving the role of edema removal and tension release. RF machines heats deeper skin layers, stimulates collagen cells to reform in dermis, increases blood circulation in hypodermis, thus achieving skin tightening and anti-aging. Coupled with cold hammer, which soothes skin and shrinks pores, this equipment takes care of your skin in all aspects.

Advantages

1. 14-in-1 multifunctional beauty machine that works on both of your facial care.
2. Various work heads can be replaced in different parts, catering to different skin types and requirements.
3. One probe is equipped with multiple functions, therefore comes with higher energy level and more even distribution of energy.
4. Painless, non-invasive during the process. Skipping any recovery periods without affecting any work and life plans.
5. No consumption, low cost and quick returns.
6. Wider treatment range and faster and more visible effects.
7. Unevenness, bleed, swelling and stasis will not appear after treatment.
8. Special probe of microdermabrasion can effectively improve wide array of skin problems and help revive clean skin.
9. Bio&Photon treatment can increase cell function, cell vitality and skin rejuvenation.
10. High frequency acne treatment deals with your troublesome acne.
11. The 3MHZ high-frequency ultrasonic wave of cleaning scrubber can effectively enhance the absorption of the skin, help the nutrition penetrate into the dermis, and comprehensively renew the skin. The high-frequency vibration refines the nutrients into small molecular particles and enhances the absorption.
12. Different essence can be selected for different skin needs. Easy and quick operation.
13. Red LED energy released by RF probe can boost blood circulation, skin cells regeneration, and acne removal.

Soothing & Repairing

Principles

Hydro Spray Jet

The product and oxygen are simultaneously injected into the air pressure equipment with the help of strong pressurized oxygen, and the granular water droplets are transformed into extremely small molecular weight to act on the skin. The tiny water droplets will be sprayed out with a specially designed nozzle at a speed of 200M/sec, and the water and oxygen will be injected into the bottom layer of human skin at an extremely high speed.

Biological Effects

Vacuum loop is formed through vacuum negative pressure, high concentration reactive oxygen species and essence are fully combined, and the skin is directly affected by high-pressure release, so as to atomize nutrients molecules, promote the removal of pore waste, not only deeply cleanse the face, but also provide lasting essence nutrition for the skin, making the skin healthy, moist, delicate and shiny.

Cold Hammer

Use the principle of air conditioner. Special wafer ceramics are used to absorb energy and refrigerate at - 5 degrees in an instant. The method of physical heat accumulation produced by wafer energy absorption makes one side produce heat up to 40 degrees. The fan discharges excess heat energy and achieves the effect of constant temperature and freezing alternately. It also promotes metabolism, alleviates allergic reaction, shrinks pore, calms skin, whitens and tenders skin.

Biological Effects

During the cooling process, tissue temperature cools down, blood vessels and veins contract, metabolic rate reduces, cell activity inhibits, peripheral nerve sensitivity reduces, thus having a sedative and soothing effect, counteracting or reducing the symptoms of allergic reaction, such as redness, swelling, and pain, therefore achieving certain effects, such as calming the skin, reducing redness and swelling, shrinking pores and so on.

BIO&Photon Treatment

LED photon therapy uses the principle of photodynamics to activate deep cells and boosts skin metabolism. After light is absorbed by the skin, the light energy is transformed into intracellular energy, which expands and strengthens microvessels and produces photochemical reaction-enzymatic reaction with the skin, increasing the activity of Catalase and superoxide dismutase (SOD), which serve as the "guard" and "cleaning worker" of cells. The breakdown of adenosine triphosphate (ATP), a source of cellular energy, increases. Increase glycogen and protein content, thereby

promoting cell metabolism and synthesis. To stimulate the body alkaline synthetic fiber growth factor (BFGF) the secretion of epidermal growth factor (EGF), making collagen fibers and increase elasticity, rearranging fibers, blocking and eliminating the formation of melanin, accelerating cell growth and blood circulation, stimulating fiber cells to produce collagen, increasing skin elasticity, repairing the aging skin, acne skin, fade spots, firm and alleviate the sun burns. It plays an important role in skin care such as killing bacteria to provide a suitable environment for new cells.

Biological Effect

Using photodynamic principle, it activates deep cells and boosts skin metabolism. After the light is absorbed by the skin, the light energy is converted into cellular energy, which can relax and strengthen the micro vessels and produce photochemical reactions with the skin -- enzymatic reactions, increasing the content of glycogen and protein, thus promoting the metabolism and synthesis of cells.

Effects

1. Fast calming, strong moisturizing.
2. Improve dull skin.
3. Improve and repair sensitive skin.
4. Improve large oil fields and shrink pores.
5. Repair skin elasticity and relieve skin after photoelectric operation.

Applicable Range

1. Those with dry, dehydrated skin.
2. Those with uneven, dull skin.
3. Those with sensitive skin and skin that are prone to redness and swelling or allergies.
4. Those with large pores, rough skin and excessive oil secretion.
5. Those who have just undergone photoelectric operation.
6. Those with facial skin peeling.

Inapplicable Range

1. Those whose wound has not healed after the invasive plastic surgery.
2. Those with skin diseases and infectious diseases.
3. Those with unrealistic expectations of results.
4. Those who have just received injections, such as hyaluronic acid, intradermal injection, wrinkle removal, or plastic surgery.
5. Those with severe allergy (rash).

Notes after Operation

1. Wash your face with warm water within 3 days after operation.

2. Strengthen hydration and pay attention to sun protection.
3. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
4. Apply a hydrating mask once a day for three days.
5. Avoid spicy or greasy food.

Anti-inflammation & Acne Treatment

Principles

Skin Scrubber

Facial cleaning scrubber is equipped with the unique technology of transforming. 2.8MHz mechanical vibration energy emitted has obvious osmosis effect on human skin. Through the medium of atomization, strong impact can instantly remove deep dirt from the pores, painlessly remove acne and pimples so that skin receive a thorough clean treatment. Low-frequency energy can emulsify sebum and secretions that are blocking pores, which can also sterilize. Mechanical vibration can not only atomizes water molecules into pores, but also can remove excess thick cutin, purify pores, decompose oil, leaving them unobstructed and tighten pores, clear and smooth skin.

Physical Effects

High-frequency vibration sound wave is combined with softening product or exfoliating cream to soften the damaged rough cuticle layer, and open pores with sound wave, and remove the garbage and waste deep in pores by high-frequency vibration. On the back of the instrument high frequency vibration combined with essence product can induce the product into the skin, increasing skin elasticity and smoothing wrinkles.

High Frequency Acne Treatment

High frequency electrotherapy instrument is an instrument that uses the vibration of high frequency electric current to generate thermal effect, ultraviolet radiation and electric current to treat skin. It has the function of promoting absorption, sterilization, anti-inflammation and accelerating wound healing. It is suitable for external treatment of acne skin and wound skin. It's application in beauty industry has long been aware of.

It is mainly composed of high frequency oscillation circuit, capacitive resistance and semiconductor devices. In addition, it is equipped with an insulated handle and a glass electric rod that can be inserted into the electrode. There is a booster transformer in the handle, and on one side there's wire for output circuit. When connected and switched on, under the action of high frequency oscillation circuit boards, it produces the continuous high voltage high frequency current, making the glass electrode

discharge ultraviolet light. Skin under the current stimulus, partial peripheral blood vessels will appear alternately contracting and expanding reaction, electric spark induces ionization of oxygen in the air and produces ozone, thus achieving effects like improving blood circulation and bactericidal anti-inflammatory.

Biological Effects

The applied current generated by the high frequency electrotherapy instrument can also change the cell membrane function of human cells, its ion channel, the receptor status on the membrane, and make the inflammatory medium unable to bind to the receptor, and make the capillary terminal shrink, thus inhibiting the occurrence and development of inflammation.

BIO&Photon Treatment

LED photon therapy uses the principle of photodynamics to activate deep cells and boosts skin metabolism. After light is absorbed by the skin, the light energy is transformed into intracellular energy, which expands and strengthens microvessels and produces photochemical reaction-enzymatic reaction with the skin, increasing the activity of Catalase and superoxide dismutase (SOD), which serve as the "guard" and "cleaning worker" of cells. The breakdown of adenosine triphosphate (ATP), a source of cellular energy, increases. Increase glycogen and protein content, thereby promoting cell metabolism and synthesis. To stimulate the body alkaline synthetic fiber growth factor (BFGF) the secretion of epidermal growth factor (EGF), making collagen fibers and increase elasticity, rearranging fibers, blocking and eliminating the formation of melanin, accelerating cell growth and blood circulation, stimulating fiber cells to produce collagen, increasing skin elasticity, repairing the aging skin, acne skin, fade spots, firm and alleviate the sun burns. It plays an important role in skin care such as killing bacteria to provide a suitable environment for new cells.

Biological Effect

Using photodynamic principle, it activates deep cells and boosts skin metabolism. After the light is absorbed by the skin, the light energy is converted into cellular energy, which can relax and strengthen the micro vessels and produce photochemical reactions with the skin -- enzymatic reactions, increasing the content of glycogen and protein, thus promoting the metabolism and synthesis of cells.

Effects

1. Improve acne, pimple, comedo, etc.
2. Shrink pores and calm skin.
3. Promote skin metabolism.
4. Increase skin elasticity.
5. Deep cleaning to help absorption.
6. Anti-inflammatory and sterilization, calm skin.

Applicable Range

1. Those with large pores and coarse skin.
2. Those with acne on the face.
3. Those with poor skin absorption and metabolism.
4. Those who run outside for a long time.
5. Those with high facial oil secretion.
6. Those with recurring facial pimples.

Inapplicable Range

1. Those who have just undergone plastic surgery and have unhealed wounds.
2. Those with skin diseases and infectious diseases.
3. Use with caution for those who are allergic to electric currents.
4. Those who have recently received certain injections, such as hyaluronic acid, intradermal injection, wrinkle removal, or plastic surgery.
5. Those who are a serious allergic reaction (rash).

Notes after Operation

1. Do not wash your face with overheated water within three days (lukewarm or cold water is recommended).
2. Enhance moisturizing and sunscreen.
3. Spicy, greasy food should be avoided. Staying up late, smoking and drinking are prohibited. Eat more vegetables, fruits, and less greasy food.
4. Use facial masks at least three times a week.
5. Less spicy, greasy food is advised.

Anti-aging&Tightening

Principles

Facial Cleansing Brush

The high frequency vibration of ultrasound brings about the vibration of the fine bristles on the brush, therefore achieving efficient cleaning and massaging without stimulation. It also helps form even, dense foam so that it cleans more thoroughly, in a way reaching the goal of beauty and wellness nourishing.

Face RF

Radio frequency can alternates electrical polarities of biological tissues on the treated area millions of times per second and change directions in the same frequency concurrently. In the rapid change of electrodes, the dermal tissue naturally resists the radio frequency current and generates thermal energy, which stimulates the dermis to

produce more collagen and fill up the gap of the lost collagens, therefore plumping up the skin and revive elasticity.

During multipolar RF treatment, polarities alternate consecutively, which produces a higher intensity energy release, heats up the bottom layers of skin. Through fast and consecutive heating of tissues, collagen regeneration is boosted, thus having a more visible result in a shorter period of time. It also comes with a wider and more intensifies treatment scope.

Biological Effect

Radio frequency can shake millions of high frequency radio waves per second, penetrate the epidermis, directly act on the dermis, deep heat collagen cells, stimulate the growth of collagen fibers, so that it has a large number of new collagen support, making the skin more compact and elastic.

Every time energy comes into the skin, it feels a temporary sense of heat. This means that collagen has been stimulated to produce fever, resulting in tightening of the skin. While operating sliding, the radio frequency wave emitted by the therapeutic head of the super-frequency wave system, under the cold protection of the epidermis, has acted on the deep dermis through the epidermis, rapidly heating water molecules to produce safe Bio-thermal energy. When the natural friction of collagen tissue is heated up to 45°C~60°C, it will shrink immediately and stimulate the collagen to grow continuously. At the same time, bio-heat can effectively accelerate blood flow in adipocytes, and release free fatty acids (ffa), thereby increasing the dissolution of superficial fat, and achieve the purpose of tightening and lifting the sagging part of the body.

Magic Gloves (EMS Massage Gloves)

The stimulating current energy sent out is effectively transmitted to the human body, and the important acupoint stimulation of the human body achieves the purpose of massage, with the magic effect of beauty skin shaping. Activate the cell, restore the skin elasticity, thus achieving the solution of some skin problems and delaying the skin aging effect of the beauty equipment.

EMS

EMS mainly uses the micro electric current to stimulate our facial acupuncture point, activate hypodermis cells, improve cell activity, accelerate facial metabolism and promote collagen formation, thus achieving effects like, skin brightening, spots fading, skin rejuvenation, wrinkle removal, skin elasticity increasing, damaged skin repairing, pore shrinking, tightening, anti-aging and antioxidant. EMS mode, also known as microelectrical stimulation, relaxes muscle spasms, increases movement, and sharpens facial lines, thereby tightening and lifting the face. It also reduces and prevents muscle atrophy, increases local blood circulation and muscle reconditioning.

Biological Effect: The use of electric current to stimulate muscle and lymph, contraction of muscle, to achieve the role of edema and tension.

Effects

1. Help to absorb nutrients and nourish dry skin.
2. Increase skin elasticity and refine skin.
3. Deeply clean and moisturize skin.
4. Reduce fine lines, dry lines, nasolabial folds and dynamic wrinkles.
5. Stimulate collagen proliferation and delay aging.
6. Improve uneven, dark and dull skin tone.
7. Tighten skin, prevent skin aging.

Applicable Range

1. Those with dull and uneven skin tone.
2. Those with skin relaxation and sagging.
3. Those with fine lines, nasolabial folds, and dynamic wrinkles.
4. Those with coarse skin and large pores.
5. Those who are under long-term exposure of UV radiation at workplace.
6. Those with lax skin, edema or obesity after birth.

Inapplicable Range

1. Those who have recently received certain injections, such as hyaluronic acid, intradermal injection, wrinkle removal, or plastic surgery.
2. Those with hypertension, heart disease, diabetes, severe thyroid and malignant tumors.
3. Those who are having an allergic reaction, or people with severe sensitive skin or allergic to metals.
4. Those with skin trauma or wound.
5. Those in menstruation, pregnancy, lactation, surgical recovery.
6. Those with skin diseases and infectious diseases.
7. Use with caution for those who are allergic to electric currents.

Notes after Operation

1. Do not wash your face with overheated water within three days (lukewarm or cold water is recommended).
2. Enhance moisturizing and sunscreen.
3. Do not soak in hot springs, steam saunas or exercise strenuously, etc. within three days.
4. Use facial masks at least three times a week.
5. Spicy, greasy food should be avoided. Staying up late, smoking and drinking are prohibited. Eat more vegetables, fruits, and less greasy food.

6. Avoid food that will cause '3 Hs' (high blood pressure, high cholesterol and high blood sugar), less greasy food is advised.

Cleaning & Hydration

Principles

Facial Cleansing Brush

The high frequency vibration of ultrasound brings about the vibration of the fine bristles on the brush, therefore achieving efficient cleaning and massaging without stimulation. It also helps form even, dense foam so that it cleans more thoroughly, in a way reaching the goal of beauty and wellness nourishing.

Skin Scrubber

Facial cleaning scrubber is equipped with the unique technology of transforming. 2.8MHz mechanical vibration energy emitted has obvious osmosis effect on human skin. Through the medium of atomization, strong impact can instantly remove deep dirt from the pores, painlessly remove acne and pimples so that skin receive a thorough clean treatment. Low-frequency energy can emulsify sebum and secretions that are blocking pores, which can also sterilize. Mechanical vibration can not only atomizes water molecules into pores, but also can remove excess thick cutin, purify pores, decompose oil, leaving them unobstructed and tighten pores, clear and smooth skin.

Physical Effects

High-frequency vibration sound wave is combined with softening product or exfoliating cream to soften the damaged rough cuticle layer, and open pores with sound wave, and remove the garbage and waste deep in pores by high-frequency vibration. On the back of the instrument high frequency vibration combined with essence product can induce the product into the skin, increasing skin elasticity and smoothing wrinkles.

Hydro Spray Jet

The product and oxygen are simultaneously injected into the air pressure equipment with the help of strong pressurized oxygen, and the granular water droplets are transformed into extremely small molecular weight to act on the skin. The tiny water droplets will be sprayed out with a specially designed nozzle at a speed of 200M/sec, and the water and oxygen will be injected into the bottom layer of human skin at an extremely high speed.

Biological Effects

Vacuum loop is formed through vacuum negative pressure, high concentration reactive oxygen species and essence are fully combined, and the skin is directly affected by high-pressure release, so as to atomize nutrients molecules, promote the removal of pore waste, not only deeply cleanse the face, but also provide lasting essence nutrition for the skin, making the skin healthy, moist, delicate and shiny.

Face Ultrasound

Ultrasonic facial beauty instrument is based on the theory of high frequency vibration. It reaches deep into the subcutaneous tissues, 4-6 mm to be exact, with the strong vibration wave of 2.8MHz. With its high energy and permeability, it vibrates tissues in every layers of your skin. When using ultrasonic beauty instrument for massage, the warmth of ultrasonic wave can penetrate to 6MM under the skin, make the blood and lymph flow smoothly, accelerate the blood circulation of fine cells, start the biological effect of cell tissue, fade all kinds of color spots, whiten the skin and activate and regenerate the cells of deep tissue.

Effects

1. Deep introduction of nutrients to accelerate absorption.
2. Adjust skin tone to make skin more transparent.
3. Deeply clean skin and shrink pores.
4. Deeply moisturize, tender and brighten skin.
5. Dredge pores and promote absorption.
6. Improve oil secretion and skin quality.

Applicable Range

1. Those with dry skin, lack of water and nutrition.
2. Those with large pores and rough skin.
3. Those with poor skin absorption, skin base water shortage.
4. Those with dull and uneven skin tone
5. Those with clogged pores and strong oil secretion
6. Those with sallow skin, no gloss.

Inapplicable Range

1. Those with severely sensitive skin or those in allergic phase.
2. Those with extensive facial trauma.
3. Those who are pregnant and those suffering from serious heart disease, severe three H's(hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc
4. Those with skin diseases and infectious diseases.
5. Those who have just received injection, such as hyaluronic acid, intradermal injection, wrinkle removal, or plastic surgery.

Notes after Operation

1. Wash your face with warm water within 3 days after operation.
2. Strengthen hydration and pay attention to sun protection.
3. Do not use stimulating skin care products including scrub, exfoliating and AHAs within one week after operation.
4. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
5. Apply a hydrating mask once a day for three days.

Regain Skin Balance

Principles

Microdermabrasion

The use of special micro-carved tube, inlaid with different thickness of fine steel particles, by the way of back and forth friction, and with the strength of vacuum suction control force, the skin will be shed a shallow layer of dead skin cells. It is non-invasive and physical. By removing dead skin cells from the surface of the skin, the regeneration ability and skin quality can be improved. After treatment, the skin will be slightly reduced, and the skin will become more smooth and tender. At the same time, combined with manual massage techniques also play an improving role in skin lifting.

Physical Effects

Vacuum negative pressure suction technology is adopted to reduce internal air pressure, which can effectively remove waste cuticle, draw out blackheads and excessive oil and other pore garbage. Also, it helps achieve free skin breathing circulation, which can dredge pores, improve large pore sizing and brighten skin tone.

Skin Scrubber

Facial cleaning scrubber is equipped with the unique technology of transforming. 2.8MHz mechanical vibration energy emitted has obvious osmosis effect on human skin. Through the medium of atomization, strong impact can instantly remove deep dirt from the pores, painlessly remove acne and pimples so that skin receive a thorough clean treatment. Low-frequency energy can emulsify sebum and secretions that are blocking pores, which can also sterilize. Mechanical vibration can not only atomizes water molecules into pores, but also can remove excess thick cutin, purify pores, decompose oil, leaving them unobstructed and tighten pores, clear and smooth skin.

Physical Effects

High-frequency vibration sound wave is combined with softening product or

exfoliating cream to soften the damaged rough cuticle layer, and open pores with sound wave, and remove the garbage and waste deep in pores by high-frequency vibration. On the back of the instrument high frequency vibration combined with essence product can induce the product into the skin, increasing skin elasticity and smoothing wrinkles.

Hydro Spray Jet

The product and oxygen are simultaneously injected into the air pressure equipment with the help of strong pressurized oxygen, and the granular water droplets are transformed into extremely small molecular weight to act on the skin. The tiny water droplets will be sprayed out with a specially designed nozzle at a speed of 200M/sec, and the water and oxygen will be injected into the bottom layer of human skin at an extremely high speed.

Biological Effects

Vacuum loop is formed through vacuum negative pressure, high concentration reactive oxygen species and essence are fully combined, and the skin is directly affected by high-pressure release, so as to atomize nutrients molecules, promote the removal of pore waste, not only deeply cleanse the face, but also provide lasting essence nutrition for the skin, making the skin healthy, moist, delicate and shiny.

Hot Hammer

Special wafer ceramics are used to absorb energy. The method of physical heat accumulation produced by wafer energy absorption makes one side produce heat up to 40 degrees. The heat energy help imports nutrients into the skin for better absorption. It also boosts blood circulation and skin metabolism.

Biological Effects

Hot hammer heating can make you feel warm, comfortable, relax muscles, balance the autonomic nerve, make pore spread on the skin, blood vessels, nutrient is absorbed quickly. It is the most gentle and comfortable import method. Alternating use of beauty has the effect of addition and multiplication, which can naturally thoroughly activate cells, improve skin quality, stimulate blood circulation acceleration, and increase the skin's metabolism.

Effects

1. Deeply clean and improve clogged and enlarged pores.
2. Remove excess corneum and unclog pores.
3. Deeply introduce nutrition and accelerate skin absorption.
4. Accelerate blood circulation and skin metabolism.
5. Regain the balance between water and oil.
6. Deeply hydrate and improve skin texture.

Applicable Range

1. Those with dry skin and lack of nutrients.
2. Those with dull, uneven and lusterless skin.
3. Those with clogged pores and thick cuticle.
4. Those with strong oil secretion and unbalanced skin water and oil.
5. Those with large pores, rough and coarse skin.
6. Those with poor skin absorption and poor metabolism.

Inapplicable Range

1. Those with severely sensitive skin or those in allergic phase.
2. Those whose nose has prosthesis.
3. Those with extensive facial trauma.
4. Those who are pregnant and those suffering from serious heart disease, severe three H's (hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc.
5. Those with skin diseases and infectious diseases.
6. Those who have just received injection, such as hyaluronic acid, intradermal injection, wrinkle removal, or plastic surgery.
7. Those with metal implants in the body, such as stents and pacemakers, or are allergic to metal.

Notes after Operation

1. Wash your face with warm water within 3 days after operation.
2. Strengthen hydration and pay attention to sun protection.
3. Do not use stimulating skin care products including scrub, exfoliating and AHAs within one week after operation.
4. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
5. Apply a hydrating mask once a day for three days.

Eye Anti-aging

Principles

Eye RF

RF works through the circuits formed by electrodes in pair. RF waves penetrate skin directly and produce thermal energy created by strong vibration from tissue resistance (1 MHz). During the treatment, RF device heats up the skin around eye area and promote collagen regeneration, which helps to tighten skin around eyes, puffiness, improve dark circles, couperose skin and fine lines.

Biological Effect

Radio frequency can shake millions of high frequency radio waves per second, penetrate the epidermis, directly act on the dermis, deep heat collagen cells, stimulate the growth of collagen fibers, so that it has a large number of new collagen support, making the skin more compact and elastic.

Every time energy comes into the skin, it feels a temporary sense of heat. This means that collagen has been stimulated to produce fever, resulting in tightening of the skin. While operating sliding, the radio frequency wave emitted by the therapeutic head of the super-frequency wave system, under the cold protection of the epidermis, has acted on the deep dermis through the epidermis, rapidly heating water molecules to produce safe Bio-thermal energy. When the natural friction of collagen tissue is heated up to 45°C~60°C, it will shrink immediately and stimulate the collagen to grow continuously. At the same time, bio-heat can effectively accelerate blood flow in adipocytes, and release free fatty acids (ffa), thereby increasing the dissolution of superficial fat, and achieve the purpose of tightening and lifting the sagging part of the body.

Eye Ultrasound

Eye ultrasound is based on the principle of high-frequency vibration that initiates a warm effect. It enters deep into the subcutaneous tissue with 20,000 HZ/ sec strong high-frequency vibration wave. Ultrasound, with its high frequency, short wavelength, and strong penetration, it can transmit like light as a straight line to the determined direction. Ultrasonic wave is a longitudinal wave that can spread smoothly in the human tissue. The ultrasonic wave that acts on human body, can produce fine frictional action on histocytes, bringing out cytoplasm activities, improving cell enzymatic activity, promoting blood circulation and metabolism, thus leaving the skin full of plumpness and elasticity and removing dark circles and bags under the eyes.

Effects

1. Relieve eye fatigue, dark circles, eye bags and edema.
2. Reduce lines around eyes and crow's feet.
3. Reduce dark spots and accelerate blood circulation.
4. Accelerate metabolism and prevent hyperpigmentation.
5. Supplement nutrition to the eyes and moisturize skin.
6. Tighten and soften skin, lift the corners of the eyes.

Applicable Range

1. Those with wrinkles, fine lines, bags and dark circles around the eyes.
2. Those with dry skin, dry lines and dynamic wrinkles.
3. Those with eyes prone to fatigue and dryness.
4. Those frequently face computer screens and mobile phones.

5. Those who often stay up late and have dark circles.
6. Those who often stay in a dry or hot environment.

Inapplicable Range

1. Those who had just undergone plastic surgery.
2. Those with hypertension, heart disease, diabetes, severe thyroid and malignant tumors.
3. Those who are having an allergic reaction, or people with severe sensitive skin or allergic to metals.
4. Those with skin trauma or wound.
5. Those who are overaging.
6. Those who are pregnant or convalescents.
7. Those with skin disease and contagious diseases.

Notices after Operation

1. Avoid excessive exposure to the sun. Sun protection is encouraged.
2. After operation, the treated area is relatively dry, moisturizing and sun protection is encouraged.
3. Within 1-3 days after operation, it is better not to use alcohol, AHA or other exfoliating products.
4. Avoid facial washing with overheated water, sauna steaming, hot springs or other strenuous exercise within 7 days after operation.
5. Moisturizers and eye masks are encouraged to use, preferably 3 times or more on eye masks a week.

Neck Anti-aging

Principles

RF

RF instrument can achieve the effect of tightening skin, lifting skin and shaping face. The optimum frequency of radio frequency biothermal effect is 3 MHz, and the maximum speed of extreme change is 8 million charged particles in tissues per second. In the rapid change of electrodes, the dermal tissue naturally resists the radio frequency current and generates thermal energy. RF wave directly penetrates the skin and generates energy by the resistance action of the skin, which makes the temperature of the skin bottom rise, rapidly and continuously heats the tissue and promotes the growth of collagen in the skin. The principle is tightening collagen and stimulating the proliferation of collagen in the dermis and then is utilized to enhance the tightness and durability in time. After 2-6

months, collagen will gradually proliferate and recombine, making sagging or relaxed skin tight and elevated.

Biological Effect

Radio frequency can shake millions of high frequency radio waves per second, penetrate the epidermis, directly act on the dermis, deep heat collagen cells, stimulate the growth of collagen fibers, so that it has a large number of new collagen support, making the skin more compact and elastic.

Every time energy comes into the skin, it feels a temporary sense of heat. This means that collagen has been stimulated to produce fever, resulting in tightening of the skin. While operating sliding, the radio frequency wave emitted by the therapeutic head of the super-frequency wave system, under the cold protection of the epidermis, has acted on the deep dermis through the epidermis, rapidly heating water molecules to produce safe Bio-thermal energy. When the natural friction of collagen tissue is heated up to 45°C~60°C, it will shrink immediately and stimulate the collagen to grow continuously. At the same time, bio-heat can effectively accelerate blood flow in adipocytes, and release free fatty acids (ffa), thereby increasing the dissolution of superficial fat, and achieve the purpose of tightening and lifting the sagging part of the body.

Face Ultrasound

Ultrasonic facial beauty instrument is based on the theory of high frequency vibration. It reaches deep into the subcutaneous tissues, 4-6 mm to be exact, with the strong vibration wave of 2.8MHz. With its high energy and permeability, it vibrates tissues in every layers of your skin. When using ultrasonic beauty instrument for massage, the warmth of ultrasonic wave can penetrate to 6MM under the skin, make the blood and lymph flow smoothly, accelerate the blood circulation of fine cells, start the biological effect of cell tissue, fade all kinds of color spots, whiten the skin and activate and regenerate the cells of deep tissue.

Magic Gloves (EMS Massage Gloves)

The stimulating current energy sent out is effectively transmitted to the human body, and the important acupoint stimulation of the human body achieves the purpose of massage, with the magic effect of beauty skin shaping. Activate the cell, restore the skin elasticity, thus achieving the solution of some skin problems and delaying the skin aging effect of the beauty equipment.

Effects

1. Improve the flabbiness, roughness and dullness of the neck skin.
2. Tighten skin and increase skin elasticity.
3. Remove double chin.
4. Accelerate lymphatic detoxification and improve facial skin.

5. Prevent cervical and lymphatic diseases.

Applicable Range

1. Those with fine lines and wrinkles on the neck.
2. Those with loose and inelastic neck skin.
3. Those with dark skin.
4. Those who often bow their heads.
5. Those who are unsatisfied with their neck skin color.

Inapplicable Range

1. Those who have just undergone plastic surgery.
2. Those with hypertension, heart disease, diabetes, severe thyroid and malignant tumors.
3. Those who are having an allergic reaction, or people with severe sensitive skin.
4. Those with skin trauma or wound.
5. Those who are overaging.
6. Those who are pregnant or convalescents.
7. Those with skin disease and other contagious diseases.

Notes after Operation

1. Sunscreen is encouraged. Keep the neck warm.
2. After operation, the treated area is relatively dry, moisturizing and sun protection is encouraged.
3. Within 1-3 days after operation, it is better not to use alcohol, AHA or other exfoliating products.
4. Avoid facial washing with overheated water, sauna steaming, hot springs or strenuous exercise within 7 days after operation.
5. Moisturizing and mask for the neck is encouraged, at least 3 times masks a week. Essence or neck cream is encouraged.
6. Avoid bowing head for a long time.

Spots Removal

Principles

Spot Removal Pen

It utilizes high frequency electric current and low voltage stabilizing transformer to transform electric current to laser and when act on skin, it can instantaneously instantaneously remove age spots, freckles, flat moles, tattoo of various sizes by

accurate targeting, drying, scabbing and peeling off. The entire process belongs to physical therapy and has a high safe level.

Effects

1. Remove age spots, freckles, café-au-lait spots, etc
2. Remove skin neoplasm, fat granules, warts, etc
3. Remove flat moles, meat moles, black moles, etc
4. Remove tattoos and tattoos of different sizes.
5. Improve corns, external hemorrhoids and rosacea on nose.

Inapplicable Range

1. Those who are prone to leave scars.
2. Those with serious heart disease, severe "3 H" (hypertension, hyperglycemia, hyperlipidemia), or other serious illness.
3. Those with dense and a large patch of spots.
4. Those with single large spots, dermal spots, patchy spots, etc
5. Use with caution for people who are allergic with electric current.

Notes after Operation

1. The scab will peel off about 4-10 days after the operation, and temporary skin erythema or pigmentation will appear in the treatment area. Pigmentation will gradually subside after one month, and will returned to normal after three months. Avoiding bright light within three months after treatment is the key to prevent the aggravation of pigment.
2. Don't eat spicy food or drink alcohol after treatment.
3. Do not take aspirin to avoid exacerbating post-treatment reactions.
4. Do not touch water or infect the treatment site after treatment, and keep it clean.

Method of Use

Mole Removal: When removing moles, pay special attention to tell the mole position. Some mole grows on the skin surface layer, whole root growth can reach deep into subcutaneous tissue. So when using the end of the needle, do not puncture the surface of moles at a time. It's best to burn in the form of small squares. After treating relatively deep positioned moles, apply healing nourishing cream that contains living cells 2-3 days prior to healing to avoid inflammation, which can prevent skin concave hole. Do not apply the nourishing cream if inflammation occurs.

Spot Removal: brace the spot position with electric needle back and forth slightly, and you can see very clearly that melanin turns charred and peels off after sweeping of

the electric. For small and surface positioned spots, one treatment will do the job. For bigger and deeper positioned pigments, it's better to divide to 2 or 3 treatments.

Part II

1. Detailed Operation

2. Technical Parameters

3. Safety Precautions

Patients with the following symptoms should be cautious about using this instrument. Please consult a doctor or a professional before using the instrument. The details are as follows:

1. Those who are pregnant or lactating.
2. Those with heart disease or who are equipped with pacemaker.
3. Those whose wounds have not healed and who are recovering from the operation.
4. Those with epilepsy, severe diabetes and hyperthyroidism.
5. Those with malignant tumor, hemophilia or severe bleeding.
6. Those with skin diseases and infectious diseases.
7. Use with caution for those who are allergic to electric currents.

4. Dos and Dont's

1. Use a grounding pin and make sure the power socket of the device is properly connected before each operation.
2. Ensure that the voltage of the device is appropriate. If the voltage of the local power supply is unstable, we recommend that users increase to the matching power voltage.
3. Use a grounding pin and make sure the power socket of the device is properly connected before each operation.
4. The device should not be placed in wet environment or near water, nor should they be directly exposed to sunlight.
5. Do not place the device near a strong heat source, because this may affect the life and normal use of the device.

6. Before treatment, please remove all metal objects from the body first, so as to avoid unexpected conditions and influence the curative effect.
7. Please restrain from using the device on eyes, thyroid, parathyroid, testicles, abdomen of pregnant women, and pacemakers.
8. Patients who are suffering from illness should consult with a doctor and gain permission from the doctor before using.
9. Please turn off the power switch of the device, and ensure that the total power supply of the device is turned off before the staff leaves after daily use, so as to ensure the safety of the electrical products.
10. Use device or train device operators in strict accordance with instructions in the user manual.
11. When operating, the instrument should fully touch the skin to avoid uneven heat.
12. Start from the lowest energy level and slowly add up.
13. When using this device, the operating parts must be kept moist and dry skin should be avoided.
14. Clean the instrument with normal saline after operation to ensure its cleanliness and hygiene and prolong its service life.
15. Do not hold the vent at the handle when operating the cold hammer.
16. Adjust the vacuum suction pressure subjecting to the customers' comfort level accordingly during microdermabrasion treatment.
17. Make sure to distinguish inhale hole from exhale hole. DO NOT insert the hydro spray jet into the inhale hole to avoid any possible damage to the machine.
18. Make sure to install rubber ring when operating microdermabrasion setting to avoid no suction or weak suction.
19. Make sure to install filter cotton when operating microdermabrasion setting to avoid filth being inhaled to the tube and damage the machine.
20. When operating the hydro spray jet on philtrum, please remind the customer to adjust his breathing.
21. When operating EMS and high frequency, please inform customers in advance to avoid frightening them.
22. Do not stay at one place for over 5 seconds to avoid scald or frostbite.
23. EMS, high frequency, RF and other probes can be used alone. The combination of several probes come with a better beauty results.
24. When changing the electrotherapy tube during high frequency treatment, please first turn off the instrument and then change the electrotherapy tube.
25. Before the use of high frequency, electric therapy tube must be installed on the handle. After the use of the instrument , remember to turn off the instrument.
26. Before sensitive skin care, please cover the skin with gauze first to avoid excessive skin irritation during high frequency treatment.
27. Please take good care of the high-frequency electric tube to avoid damage.
28. When using the cleaning brush, do not overexert yourself. Just brace the skin gently.

29. When operating the EMS massage gloves, both hands must be at work the same time, and ensure even contact with the skin. The energy should be adjusted from the minimum first, and then gradually increase after adaptation.
30. It is better not to wash the treatment area too much within 30 days after spot removal treatment (simple nursing care is enough), and not to scrub it forcibly.
31. Do not use hands to scratch the scabbed area when it gets itchy after spot removal treatment; wait for 20~30 days or so, it will fall off by itself.
32. Do not stay in one place for a long time during spot removal operation; sweep gently so as to not burn the dermis.
33. For those with dense, large patch dark spots, single large spots, dermal spots, patchy spots, it is advised not to use this instrument.

5. Troubleshooting & Solutions

1. The instrument cannot be started, but the key light on the back of the instrument does not work?

- A. Make sure the power cord is connected to a valid power socket.
- B. Whether the fuse tube on the back of the instrument is loose or burnt out.

2. No RF output of the instrument?

- A. Please check whether the plug connecting the instrument handle and the body is tightly connected.
- B. Please check whether the treatment area has been cleaned. Grease or oil essential oil products may cause poor contact between the treatment handle of the instrument and human body, resulting in no RF output.

3.No EMS Energy Output?

- A. Please check if the handle and the machine is tightly connected.
- B. Please check if the treatment area is clean. Grease or products like essential oil might interfere with the RF output.

4.No spray coming from the hydro spray jet?

- A. Please check whether the nozzle is blocked.
- B. Please check whether the regulating part of the nozzle tail is tightened.
- C. Please check whether the wrench of the spray gun is loose.

5. No response to the button of skin scrubber?

- A. Turn off the instrument and restart it.
- B. Remove the plug of the probe and install it again.

6. The microdermabrasion has no suction?

- A. Please check whether the rubber ring of the operating head is screwed too tight, resulting in air leakage.

- B. Please check whether there is any rubber ring installed.
- C. Please check whether the rubber hose is properly connected to the main engine and whether there is any damage to the rubber hose.

7. I can start the instrument, but the screen show's error message?

- A. Please unplug the power plug behind the instrument and wait for about 1 minute before restarting the instrument.
- B. If the above methods cannot be handled, please contact the instrument dealer for assistance.

6.FAQs

1. Q: How long can I start seeing results of RF treatment?

A: Under normal circumstances within that day or a week. Skin collagen tissue is heated and produces contractions, and you can obviously feel skin tightening. Because RF works by stimulating subcutaneous tissue and bring about sustainable collagen regeneration, so the more you do it, the more visible effects you shall receive.

2. Q: Is RF harmful to skin?

A: RF treatment falls under the category of non-surgery operation. It stimulates collagen regeneration at the skin bottom and boosts metabolism, therefore has no damage to the skin. Partial redness and fever is the normal phenomenon of accelerated blood circulation, and will be resolved by oneself after a moment, so there's no need to worry.

3.Q: How long do I have to wait to put on makeups after spots removal treatment?

A: Treated area will be more sensitive and fragile after spot removal treatment, so you can not put on makeup immediately, and in the recovery period, do not use a variety of chemical exfoliating cosmetics. After about 6-10 days you can start your basic makeup routine. The specific time is to be determined according to the individual skin quality and recovery. You must consult the professional personnel for selection of cosmetics. Do not use cosmetics that contain chemical composition to avoid affecting the results.

4.Q: Will the skin become thinner and more sensitive after microdermabrasion treatment?

A: No. During the process, microdermabrasion operation is just to remove the skin aging cuticle. Our skin regenerates every 28 days, so it will only make the skin become more smooth, fine and glowing. Sensitive symptoms will not appear.

6.Q: Why exfoliate?

A: The cuticle is the layer above the epidermis where the cells are completely dead. Depending on your body, the cuticle consists of 15 to 20 layers of dead cells. Human skin has a metabolic cycle, which is normally 28 days. It is the metabolic cycle of people in puberty. For adults, the metabolic cycle should be the actual age +10. But due to various reasons, dead corneous cells may not fall off in time and bring about thick aging cutins. On the other hand, it also affects the color of skin, and the skin metabolism, so other method needs to be utilized when normal metabolic process won't do the job by itself.

6.Q: Can I have high frequency treatment without dealing with my acne first?

A: It is doable, nevertheless the results differ. First of all for the immature pimple or those just started growing, it's better not to irritate them. But for mature acne, it's better to clean, then diminish inflammation, sterilize and accelerate the wound healing process.

7. Q: Do I need to apply anything before high frequency treatment?

A: It's up you. During the operation, we use a curved tube when nothing was applied, and in the case of applying something, we use a tongue or mushroom tube. On the first scenario, we're targeting at smaller and less serious acne skin. For larger or wider area of acne skin, we can use the tongue shaped tube and acne essence to achieve a more obvious treatment effect. This instrument uses the principle of high frequency current + ozone, it helps boost absorption and metabolism, so it can be used with other products for even better beauty results.

8. Q: Can I wear makeup after using this instrument?

A: Yes, except for spot removal treatment. After the other treatments, skin actually appears to be hydrated and plump, so makeup stays longer. It is a non-invasive beauty treatment, which is safe, healthy, fast and effective.

9.Q: Is it dependent?

A: No, this is for skin problems to improve and deeply clean the skin, which belongs to the maintenance and improving skin of malnutrition caused by other factors. It's used to help skin absorb nutrition products, solve other issues such as dark, yellowish skin, water shortage, rough skin, wrinkled skin. Without the treatment you skin will only go back to secretion of its own and aging, therefore not dependence factor is involved.

10.Q: What is collagen?





A: Collagen is a kind of biological macromolecule substance, a kind of white, opaque and non-branched fibrous protein. It can supplement the nutrition needed by all layers of the skin, enhance the collagen activity in the skin, lock moisture, nourish the skin, delay aging, beautify, relieve facial relaxation, provide nutrients for the hair and so on. Collagen is a nutrient that human body must replenish to delay aging. With the


growth of age, collagen will gradually be lost. At the age of 20 , women have begun to age, collagen gradually decrease. At the age of 25, collagen loss reaches its peak. At the age of 40 , the content is less than half of which one has at age 18. It is the loss of collagen and water that causes the wrinkles on the face of the elderly. The breakage of collagen fibers and elastic mesh will cause skin tissue to be oxidized, atrophied and collapsed. The skin will be dry, wrinkled, flabby and inelastic etc,. Therefore, in order to delay aging, collagen must be supplemented.

7. Packing List

- 1 x Host
- 1 x O2 sprayer
- 9 x Diamond tips
- 3 x Stainless steel wands
- 2 x Ultrasonic (flat head and point head)
- 1 x 7 Color photon LED
- 1 x Skin scrubber
- 1 x PVC vacuum hose
- 3 x Glass tubes for high frequency
- 3 x Holder for handles
- 1 x Hot Cold hammer
- 1 x Spots removal pen + 10 x needles
- 1 x Bipolar RF handle
- 1 x Quadruple RF handle
- 1 x High frequency
- 1 x Magic gloves
- 1 x Skin scrubber
- 1 x Brushes
- 1 x Galvanic

8.Operational Diagrams


Parameter Adjustment	Product	Techniques	Diagrams
Soothing&Repairing: 60 min, once/week			
<p>Hydro energy level: Liquid export volume is recommended to set as small</p> <p>Cold&Hot Hammer Energy: 20%-80%</p> <p>Mode: COLD</p> <p>BIO&Photon Energy: 20%-80%</p> <p>Mode: Green</p>	<p>Makeup remover+ Cleanser+ Hydrating base mask+ Cold&hot hammer+ Essence+ Repairing mask+ Device</p>	<ol style="list-style-type: none"> 1. Remove makeup and clean face, 5 min. 2. Apply toner, 1 min. 3. Apply hydrating facial mask+cold steam, 10 min. 4. Clean face, 2 min. 5. Pour (hydrating)essence into the hydro bottle(with the proportion of essence:distilled water being 2:1), 2 min. 6. Hydro spray jet operation: start from chin to earlobe, then from mouth corner to ear gate. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times. 7. Move vertically towards hairline on forehead, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times. 8. Apply essence evenly on the face, 1 min. 9. Cold hammer operation: Move horizontally in circles from chin to earlobe, mouth 	<p>Technique6</p>  <hr/> <p>Technique7</p>  <hr/> <p>Technique9</p>  <hr/> <p>Technique10</p>  <hr/> <p>Technique12</p>

		<p>corner to ear fate, nose wings to temple to lift the treatment area, 3 times.</p> <p>10. Lift from chin to earlobe, mouth corner to earlobe, nose wing to temple to lift the treatment area, 3 times.</p> <p>11. Do the same on the other side.</p> <p>12. Move vertically towards hairline on forehead, 3-5 times.</p> <p>13. Apply repairing facial mask + photon treatment(green LED), 15 min.</p> <p>14. Take off the mask and clean the treatment area, 2 min.</p> <p>15. Apply toner, essence, cream, and sunscreen.</p>	
--	--	---	---





Suggested Treatment:





10 treatments account for one full course. After one treatment you will feel skin is hydrated and smooth. After one course, the skin will restore a normal state, and less prone to allergy due to dehydration, cosmetics or change of season. After 2 courses, skin surface is smooth, ruddy with a boosted blood circulation and nice touch. Three course help consolidate the effect and prevent skin allergy.

Anti-inflammation&Acne Treatment: 70 min, once/week

<p>Facial Cleaning Scrubber</p> <p>Energy Level: Modal: 1 (press once); 2(press twice); 3(press thrice)</p> <p>High Frequency</p> <p>Energy: 20%-80%</p> <p>BIO&Photon</p>	<p>Makeup remover+</p> <p>Face wash+</p> <p>Cold and hot steam+</p> <p>Cleaning base mask+</p> <p>Acne essence+</p> <p>Facial mask+</p> <p>Pimple pin+ SR-M51713</p>	<ol style="list-style-type: none"> 1. Remove makeup and clean face, 5 min. 2. Apply toner, 1 min. 3. Apply cleaning base mask+hot steam, 10 min. 4. Clean face, 2 min. 5. Facial cleaning scrubber operation: drench cotton pad in water and pat on face, then tilt the scrubber at a 45°angle and slowly move around the face; start with nose and double the treatment if 	<p>Technique6</p>  <p>Technique8</p>
--	--	--	---

<p>Energy: 20%-80%</p> <p>Mode: Blue LED</p>	<p>needed on nose, 3-6 times.</p> <p>6. Start from chin, brace the skin in lines to keep the treatment surface tight, 3-6 times.</p> <p>7. Do the same on the other side.</p> <p>8. Brace the skin vertically towards hairline on forehead, 3-6 times.</p> <p>9. Wash face clean, 2 min.</p> <p>10. Use the pimple pin to clean the pimple that is ready to be popped.</p> <p>11. Install the high frequency tube of your choice.</p> <p>12. High frequency operation: Press 1-2 seconds on the treatment area with the tube until its thoroughly drained.</p> <p>13. Wash face clean, 2 min.</p> <p>14. Apply (acne treating) essence evenly all over the face, 1 min.</p> <p>15. Skin scrubber operation: Use the back side of the scrubber to lead in nutrition, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple to lift the treatment area, 3 times.</p> <p>16. Brace the skin towards hairline on the forehead to lift the treatment area, 3 times.</p> <p>17. Do the same on the other side.</p> <p>18. BIO&Photon operation: Move the probe in circular motion around the whole face, 3 times.</p> <p>19. Apply facial mask, 15 min.</p> <p>20. Wash face clean, 2 min</p> <p>21. Apply toner, essence,</p>	<div data-bbox="1161 219 1286 412" data-label="Image"> </div> <hr/> <p data-bbox="1082 443 1254 479">Technique15</p> <div data-bbox="1161 510 1286 703" data-label="Image"> </div> <hr/> <p data-bbox="1082 734 1254 770">Technique16</p> <div data-bbox="1161 801 1286 994" data-label="Image"> </div> <hr/> <p data-bbox="1082 1025 1254 1061">Technique18</p> <div data-bbox="1161 1093 1286 1285" data-label="Image"> </div> <hr/>
--	--	--

		spot treatment cream.	
<p>Recommended treatment: 10 treatments account for one full course. After one session, treatment area appears to be slightly red and hydrated. After a full course, inflammation is inhibited and acne appears to be less. After two courses, pimples are visibly lessened and inflammation is gradually disappearing. After three courses, skin is smooth and fine. At later stages, touch-up is needed to remove acne scar.</p>			
<p>Anti-aging&Tightening: 60 min, once/week</p>			
<p>Facial Cleansing Brush Energy: 20%-80%</p> <p>Face RF Energy: 20%-80%</p> <p>EMS Energy: 20%-80%</p> <p>Mode: Consecutive</p> <p>Magic Gloves Energy: 20%-80%</p> <p>Mode: Consecutive</p>	<p>Makeup remover+ Cleanser+ Massage cream+ Essence+ Facial mask+ SR-M51713</p>	<ol style="list-style-type: none"> 1. Remove makeup, 2 min. 2. Dab foaming cleanser on five spots of the face, 1 min. 3. Facial cleansing brush operation: Start from chin, moving the probe in circular motion, covering the whole face, 2 times. 4. Clean face, 2 min. 5. Apply massage cream evenly on face and caress face, 3 times. 6. Push the following acupoints (chengjiang point, dicang point, renzhong point, yingxiang point, jiache point, temple) , 3 times. 7. Caress the whole face, 3 times. 8. Use both hands to lift the treatment area as if plucking the strings, 2 min. 9. Alternate hands in lifting the treatment area as if plucking the strings, 1 min. 10. Do the same on the other side. 11. Caress the whole face, 3 times. 12. Alternate hands in lifting the treatment area, tracing from chin to earlobe, mouth corner to ear gate, nose wing 	<p>Technique3,5</p>  <hr/> <p>Technique6</p>  <hr/> <p>Technique12</p>  <hr/> <p>Technique14</p>  <hr/> <p>Technique15</p>






	<p>to temple, 3 times.</p> <p>13. Do the same on the other side.</p> <p>14. Brace skin with middle and ring fingers on forehead as if writing 'Z', slide to the ear back and force and finish behind ear, 3 times.</p> <p>15. RF operation: lift from chin to earlobe, mouth corner to ear gate, nose wing to temple; make sure one line is next to another, 3 times.</p> <p>16. Do the last technique, combined with hand, 3 times.</p> <p>17. Lift towards hairline on forehead, 3-5 times.</p> <p>18. Do the same on the other side.</p> <p>19. Wash face clean, 2 min.</p> <p>20. Apply (anti-aging/anti-wrinkle) essence evenly on face, 1 min.</p> <p>21. Install the EMS probe of your choice and have the customer hold the vacuum handle.</p> <p>22. EMS operation: Start from chin, moving in circular lines to lift the treatment area with both hands, 3-5 times.</p> <p>23. Magic gloves operation: Start from neck, lift the treatment area in lines; Move towards the hairline on forehead, 3 times.</p> <p>24. Apply facial mask, 15 min.</p> <p>25. Wash face clean, 2 min</p> <p>26. Apply toner, essence, cream and sunscreen.</p>	 <p>Technique16</p>  <p>Technique17</p>  <p>Technique22</p> 
--	---	---



Suggested Treatment:

10 treatments account for one full course. After one time, the skin will be firm and tender, providing the skin with needed nutrition. After a course of treatment, the skin is lifted and shiny. After 2 courses, the skin is smooth and plump, and the outline

gradually becomes clear. After 3 courses, it can achieve effects such as prevent and delay skin aging, improve sagging and loose skin, and increase collagen in the bottom of skin, making skin smooth, firm and glowing.

Cleaning&Hydration: 60 min, once/week

<p>Facial Cleansing Brush Energy: 20%-80%</p> <p>Facial Cleaning Scrubber Energy Level: Modal: 1 (press once); 2 (press twice); 3 (press thrice)</p> <p>Hydro energy level: Liquid export volume is recommended to set as small</p> <p>Face Ultrasound Energy: 20%-80%</p>	<p>Makeup remover+ Cleanser+ Blackhead export liquid+ Cold&hot steam+ Base mask+ Essence+ Facial mask+ SR-M51713</p>	<ol style="list-style-type: none"> 1. Remove makeup, 2 min. 2. Dab foaming cleanser on five spots of the face, 1 min. 3. Facial cleansing brush operation: Start from chin, moving the probe in circular motion, covering the whole face, 2 times. 4. Clean face, 2 min. 5. Apply blackhead export liquid+hot steam (on large pores or clogged area, and toner on cheeks; for sensitive skin it is not recommended to do so but apply toner+cold steam) 5-8 min. 6. Skin scrubber operation: Use the back side of the scrubber to lead in nutrition, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple to lift the treatment area, 3 times. 7. Brace the skin towards hairline on the forehead to lift the treatment area, 3 times. 8. Clean face, 2 min. 9. Apply cleaning base mask+hot steam, 10 min. 10. Remove the mask and clean, 2 min. 11. Pour (hydrating)essence into the hydro bottle(with the proportion of essence:distilled water being 2:1), 2 min. 12. Hydro spray jet 	<p>Technique3</p>  <hr/> <p>Technique6</p>  <hr/> <p>Technique7</p>  <hr/> <p>Technique12</p>  <hr/> <p>Technique13</p>  <hr/> <p>Technique15</p>
--	--	--	---







		<p>operation: start from chin to earlobe, then from mouth corner to ear gate. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>13. Move vertically towards hairline on forehead, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>14. Apply (hydrating) essence evenly all over the face, 2 min.</p> <p>15. Ultrasound operation: Move the probe in circles horizontally to lift the treatment area, tracing from chin to earlobe, mouth corner to ear gate, nose wing to temple, 3 times.</p> <p>16. Do the same on the other side.</p> <p>17. Move vertically towards hairline on forehead in circles, 3 times.</p> <p>18. Apply facial mask, 15 min.</p> <p>19. Wash face clean, 2 min.</p> <p>20. Apply toner, essence, cream, and sunscreen.</p>	 <hr/> <p>Technique17</p> 
--	--	---	--



Recommended treatment:

10 treatments account for one full course. After one treatment you will feel a clean, smooth and less course skin. After one full course, clogged pores, course skin and strawberry nose are improved. Three courses help your skin remain healthy, smooth and also help with better absorption of skin care products.



Regain Skin Balance: 60 min, once/week

Microdermabrasion energy level:	Makeup remover+ Face wash+	1. Remove makeup and clean face, 5 min. 2. Apply blackhead export	Technique3
---------------------------------	-------------------------------	--	------------

<p>20%-80%</p> <p>Operate according to different skin types of customers (try the set energy on lower arm first then on face, meanwhile check with customers constantly.)</p> <p>Facial Cleaning Scrubber Energy Level: Modal: 1 (press once); 2 (press twice); 3 (press thrice)</p> <p>Hydro energy level: Liquid export volume is recommended to set as small</p> <p>Hot Hammer Energy: 20%-80%</p> <p>Mode: Hot</p>	<p>Blackhead export liquid+ Cold and hot steam+ Cleaning base mask+ Acne essence+ Facial mask+ SR-M51713</p>	<p>liquid+hot steam (on large pores or clogged area, and toner on cheeks; for sensitive skin it is not recommended to do so but apply toner+cold steam) 5-8 min.</p> <p>3. Microdermabrasion operation: stay on forehead for one sec, then gently twist and leave it up in the air (forehead → nose → chin), 5 min.</p> <p>4. Operation on cheek: start from chin, and operate in dots then connects 3 dots in a line. Move in parallel lines. End with temple, 3 min.</p> <p>5. Do the same on another side, 3 min.</p> <p>6. Wash face clean, 2 min.</p> <p>7. Facial cleaning scrubber operation: drench cotton pad in water and pat on face, then tilt the scrubber at a 45° angle and slowly move around the face; start with nose and double the treatment if needed on nose, 3-6 times.</p> <p>8. Start from chin, brace the skin in lines to keep the treatment surface tight, 3-6 times.</p> <p>9. Do the same on the other side.</p> <p>10. Brace the skin vertically towards hairline on forehead, 3-6 times.</p> <p>11. Apply hydrating base mask+hot steam, 10 min.</p> <p>12. Pour (hydrating) essence into the hydro bottle (with the proportion of essence:distilled water being 2:1), 2 min.</p>	<div style="text-align: center;">  <p>Technique 4</p> </div> <div style="text-align: center;">  <p>Technique 8</p> </div> <div style="text-align: center;">  <p>Technique 10</p> </div> <div style="text-align: center;">  <p>Technique 13</p> </div> <div style="text-align: center;">  <p>Technique 14</p> </div> <div style="text-align: center;">  <p>Technique 16</p> </div>
--	--	---	---

		<p>13. Hydro spray jet operation: start from chin to earlobe, then from mouth corner to ear gate. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>16. Move vertically towards hairline on forehead, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>17. Apply essence evenly on the face, 1 min.</p> <p>18. Hot hammer operation: Move horizontally in circles from chin to earlobe, mouth corner to ear fate, nose wings to temple to lift the treatment area, 3 times.</p> <p>19. Move vertically towards hairline on forehead, 3 times.</p> <p>20. Do the same on the other side.</p> <p>22. Apply facial mask, 15 min.</p> <p>23. Wash face clean, 2 min</p> <p>24. Apply toner, essence, cream and sunscreen.</p>	 <hr/> <p>Technique17</p>  <hr/>
<p>Recommended treatment: 10 treatments account for one full course. After one session, skin is clean and translucent. After a whole session, skin is hydrated with less oil secretion. The water and oil ratio is slowly regaining the balance. After two courses, skin is fine, glossy and hydrated. Three courses help enhance the effect and prevent skin aging.</p>			
<p>Eye Anti-aging: 60 min, once/week</p>			
<p>Eye RF Energy: 20%-80%</p> <p>Eye Ultrasound Energy:</p>	<p>Makeup remover+ Cleanser+ Toner+ Essence+</p>	<p>1. Remove makeup and clean face, 5 min.</p> <p>2. Apply toner, 1 min.</p> <p>3. Apply essence evenly around eyes and gently</p>	<p>Technique3,5,10</p>


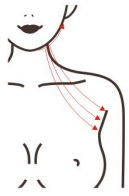
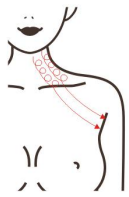
20%-80%	Eye mask+ SR-M51713	<p>massage to rub it in, 1 min.</p> <p>4. Push the following acupoints(jingming point, cuanzhu point, yuyao point, sizhukong point, temple, tongziliao point, chengqi point, sibai point), 3 times.</p> <p>5. Move hands in circular motion on eyes and push temples, 3 times.</p> <p>6. Move middle and ring fingers horizontally below lower eyelids and slide to back of the ear, 3 times.</p> <p>7. Lift middle and ring fingers from inner eye corner to upper eyelids and slide to back of the ear, 3 times.</p> <p>8. Lift eye corners with “scissor” hand gesture, 3-5 times.</p> <p>9. Repeat the previous operation on the other side, 3 times.</p> <p>10. Move hands in circular motion on eyes and push temples, 3-5 times.</p> <p>11. RF operation: move the probe beneath lower eyelid in small circles and move all the way till the temple, 3-5 times.</p> <p>12. Combined with hand, move the probe from lower eyelid to temple to lift the treatment area, 3-5 times.</p> <p>13. Lift from the beginning of the eyebrows to the end, 3-5 times.</p> <p>14. Combined with hand, move the probe to lift the corner of the eyes towards the hairline, 3-5 times.</p> <p>15. Repeat the same technique with ultrasound</p>	<div data-bbox="1161 219 1286 416"> </div> <hr/> <p data-bbox="1082 450 1238 483">Technique4</p> <div data-bbox="1161 510 1286 707"> </div> <hr/> <p data-bbox="1082 741 1238 775">Technique6</p> <div data-bbox="1161 801 1286 999"> </div> <hr/> <p data-bbox="1082 1032 1238 1066">Technique7</p> <div data-bbox="1161 1093 1286 1290"> </div> <hr/> <p data-bbox="1082 1323 1238 1357">Technique8</p> <div data-bbox="1161 1384 1286 1581"> </div> <hr/> <p data-bbox="1082 1615 1254 1648">Technique11</p> <div data-bbox="1161 1675 1286 1872"> </div> <hr/> <p data-bbox="1082 1906 1299 1939">Technique12,14</p>
---------	------------------------	--	--



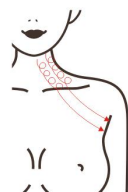
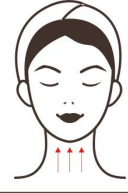


		<p>operation.</p> <p>16. Apply eye mask, 15 min.</p> <p>17. Wash eyes and face clean, 2 min.</p> <p>18. Apply toner, facial essence, eye essence, cream and sunscreen.</p>	 <p>Technique13</p> 
--	--	--	--

Suggested Treatment:

10 treatments account for one full course. Once done, the eyes are lifted and tightened, increasing blood circulation to the eyes. After one course, it lightens fine lines, dark circles under the eyes and brightens the skin. After 3 courses, the skin around the eyes is firm, tender and shiny, improving and preventing the aging of the eyes.

Neck Anti-aging: 60 min, once/week

<p>Face RF Energy: 20%-80%</p> <p>Face Ultrasound Energy: 20%-80%</p> <p>Magic Gloves Energy: 20%-80%</p> <p>Mode: Consecutive</p>	<p>Makeup remover+ Cleanser+ Toner+ Massage cream+ Essence+ Neck mask+ MS-M51713</p>	<ol style="list-style-type: none"> 1. Remove makeup and cleanse face, 5 min. 2. Apply toner, 1 min. 3. Neck massage: apply and rub oil on the chest with both hands to the back of the neck and point to feng chi and feng fu acupoint, 3 times. 4. Alternately lift the lower jaw and double chin to pass behind the ear and under the armpit, 3 times. 5. Push downwards with hand spread and hukou pointing down, tracing 3 meridian channels from the neck side to the armpit, 3 times. 6. Push downwards with 4 fingers, tracing 3 meridian channels from neck side to 	<p>Technique3</p>  <p>Technique4</p>  <p>Technique5,6,7</p>  <p>Technique8</p>
--	--	---	---

	<p>the armpit, 3 times.</p> <p>7. Push downwards with kneeling fingers till the neck turns hot and slide to the armpit, 3 times.</p> <p>8. Push downwards with hand spread and hukou pointing down from external collarbone to armpit, 3-5 times.</p> <p>9. Do the same on the other side.</p> <p>10. Neck RF operation: coupled with hand massage, lift double chin to pass behind the ear and under the armpit, 3 times.</p> <p>11. Move the device in circles on one side of neck to pass under armpit, 3 times.</p> <p>12. Slide the device from internal and external clavicle to the armpit, 3-5 times.</p> <p>13. Move the probe in circles on the whole neck area, 3-5 times.</p> <p>14. Clean neck, 2 min.</p> <p>15. Apply (anti-aging) essence evenly on the neck, 1 min.</p> <p>16. Neck ultrasound operation: Move the probe in circles on the whole neck area, 3 times.</p> <p>17. Repeat the last technique, combined with hand, 3-5 times.</p> <p>18. Magic gloves operation: Use both hands to lift the whole neck from collarbone, 3-5 times.</p> <p>19. Apply neck mask, 15 min.</p> <p>20. Clean the neck, 2 min</p> <p>21. Apply toner, neck</p>	 <p>Technique10</p>  <p>Technique11</p>  <p>Technique13</p>  <p>Technique16</p>  <p>Technique17</p> 
--	---	---

		essence, neck cream and sunscreen.	
<p>Suggested Treatments: 10 treatments account for one full course. After one treatment, skin becomes tender, smooth. After a full course, neck skin becomes smooth, speed up the neck lymphatic circulation. After 2 courses it tightens skin, fade neck lines, improve the double chin. After 3 course, skin is fine, firm and glowing, which also promote lymph drainage, improve facial dullness, some problems such as acne, and delay the aging of the skin, make skin restore young state.</p>			
<p>Spot Removal: 20 min</p>			
Spot Removal Pen Energy: Adjust the energy level subjecting to your own comfort level and size and depth of the spot/mole	Cleanser+ Makeup remover oil+ Anaesthetic + Spot removal pen+ Scar removal cream (or other scar removal products)	<ol style="list-style-type: none"> 1. Remove makeup and clean the treatment area, 5 min. 2. Apply a layer of anaesthetic on the treatment area and wait for 20 min or so. 3. Wipe it clean with tissue. 4. Adjust the energy level of the spot removal pen and start operation. 5. Spot treat on the area (1-3 times for light colored, small treatment area; 3-8 times for darker colored, bigger treatment area) 6. Apply scar removal cream or aloe gel after the treatment. 	No Diagram
<p>Normally, light spot/mole can be removed after one session. The final results differs on an individual basis. The second treatment, if needed, should be 30-45 days after the fist one.</p>			
<p>Individual differences include: One’s metabolism, postoperative care, whether you are prone to leave a scar, whether you are allergic to drugs, whether you have active moles (which can grow back after elimination), the color of the spot/mole, the type of spot/mole, skin melanin activity, etc.</p>			