

User Manual

SR-MJ1805P



Preface

To our valued users,

Thanks for choosing our 5-in-1 beauty equipment for deep cleansing, easing, moisturizing, and skin rejuvenating and brightening. This is a multifunctional equipment, which works effectively on facial skin dryness and hydropenia, clogged pores, inelastic skin, uneven skin color and etc. It applies high-tech and achieves the effect of beauty and skin care safely and effectively. SR-MJ1805P is a professional beauty equipment, which needs to be operated by people with professional training. Any improper use of this equipment may bring adverse consequences to human body, thus we advise all the people to read this manual thoroughly and follow the instructions specified hereby strictly before operation.

We believe our quality product will give you a good return, and our perfect after-sales service will make you worry-free.

Thank you.

Table of Contents

Preface
Part I
Brief Introduction
Advantages
Skin Cleansing with Active Oxygen
Skin Peeling with Diamond Dermabrasion
Easing & Repairing Management
Facial Rejuvenation & Balance Management
Ceramic Skin Restoring Management
Part II
Detailed Operations
Technical Parameters
Contraindications
Precautions for Use
Troubleshooting & Solutions
FAQs
Packing List
Operational Diagrams

Part I

Brief Introduction

Diamond dermabrasion and skin scrubber are currently the most popular equipments for deep skin cleansing. It can effectively replace the traditional facial cleansing and care programs in beauty salon, and can be operated easily, conveniently and swiftly, and solves skin problems for people pursuing beauty. Moreover, this equipment works safely and effectively, and requires no injection, no medication and no operation. It has no side effects, and is operated externally throughout, and has an instant effect. Diamond dermabrasion adopts vacuum suction technology. By decreasing internal air pressure, it effectively removes old and useless stratum corneum and sucks out wastes, like blackheads and redundant grease, from pores, thus achieving the effect of free breath and circulation for skin, dredging pores, improving large pores, and brightening skin color. Meanwhile, with the matched use of Bio & Photon phototherapy brush, which applies the principle of photodynamics, it activates cells, accelerates metabolism, rejuvenates skin and removes free radical.

Advantages

1. 5-in-1 multifunctional beauty equipment that fixes facial care problems.
2. It has multiple operating handles, which can be changed according to body part to be treated and different demands.
3. The special-designed diamond head can effectively improve facial skin problems, and deeply cleanse and purify skin.
4. It suits all kinds of skin, and can improve skin quality and flatten fine lines.
5. It's comfortable, painless, noninvasive and injury-free during treatment, and there is no convalescence, people can leave immediately, and without affecting people's normal work and life after treatment.
6. No consumption, low cost, but with quick return.
7. It has a wider treatment range, and has an instant and evident effect.
8. Unevenness, bleeding and swelling will not occur.
9. Bio & Photon phototherapy brush can activate cells, accelerate metabolism, rejuvenate skin and remove free radical.
10. Skin scrubber breaks nutrient into molecular particles with high frequency ultrasonic vibration of 3MHz, which can effectively enhance skin absorption, infuse nutrition deeply into dermis, and make skin shiny and elastic.

Skin Cleansing with Active Oxygen

Principle

Skin Scrubber


It applies the principle of ultrasound with high frequency vibration. Energy produced by mechanical vibration of 28,000 times per second has an obvious permeation effect on skin. When ultrasonic cavitation effect encounters water, it atomizes water into small bubbles. The small bubbles, through constant vibration, go deep into skin's pores breaking acne, blackheads and dark acne into granulum which will be vibrated onto skin's surface through high frequency and ultrasonic vibration. Low frequency emulsifies sebum and excreta that clogged pores, and meanwhile it can also do sterilization, atomizes water molecules, removes redundant thick cutin, purifies pores and decomposes grease through mechanical vibration after entering into pores, so as to make the pores unobstructed, contract the pores, and make the skin bright, clear, soft and smooth.

Physical Effect: Sound wave of high frequency vibration, with the matched use of softened products or exfoliating cream, can soften the impaired and rough stratum corneum, open pores, and deeply remove the wastes from pores.

Skin Exfoliating by 3MHz Ultrasound Scrubber



Utilize 3MHz high frequency ultrasound wave with exfoliating product such as AHA or exfoliating cream to soften the damaged rough epidermal stratum corneum , then open the pores, make the waste out of the pores by high frequency vibration caused by ultrasound.

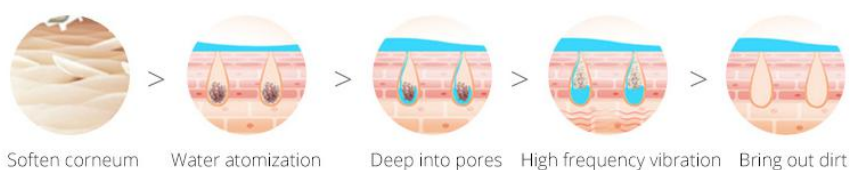

Active Oxygen Spray Bottle

By utilizing oxygen, water and strong pressure, it acts on skin with molecular particle that smaller than 80 micrometer and spray, which makes active water molecule and pure oxygen enter into deep skin rapidly, and accelerates the absorption of skin nutrition. By penetrating the pores and skin crack with strong pressure, it infiltrates from epidermis to dermis so as to stimulate cell regeneration, replenish adequate nutrition to skin directly and rapidly, and meanwhile remove wastes from deeper epidermis. High pressure oxygen and nutrient solution can stimulate the regeneration of dermis's fibrous, promote metabolism, and achieve the effect of moisturizing, skin

rejuvenating and yellowish complexion eliminating.

Physical Effect: Active oxygen spray bottle nanometerizes and atomizes oxygen molecule. Through pulsed-propulsion, it injects oxygen molecule into skin's each layer, replenish the needed moisture and nutrient to cells, thus recovering cell's vitality and achieving skin's plumpness, moisturizing and elasticity.

Double Skin Moisturizing & Nourishing-Sprayer



Bio & Photon Phototherapy Brush

It adopts the principle of high-tech photorejuvenation with photodynamics to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, light energy was transformed into intracellular energy which expands and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin. This increases activity of catalase and superoxide dismutase (SOD), which serve as the "guard" and "janitor" for cells, and increases breakdown of adenosine triphosphate (ATP), a source of cellular energy, and increases glycogen and protein content so as to accelerate cells metabolism and regeneration. It stimulates the secretion of body alkaline synthetic fiber growth factor (BFGF) and epidermal growth factor (EGF), increases collagenous fiber's elasticity, rearranges fibers, blocks and eliminates the formation of melanin, accelerates the growth of cells and blood circulation, stimulates fibrocyte tissue producing collagen, and increases the elasticity of skin. Moreover, it plays an important role in repairing aging and acne skin, and in fading spots, and in firming and lifting, as well as in alleviating sunburns.

Biological Effect: It adopts photodynamic principle to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, light energy was transformed into intracellular energy which expands and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin, and increases glycogen and protein content so as to accelerate cells metabolism

and regeneration.

Red Light: with a wavelength of 635NM. It warms up deep skin, activates collagen's activity, speeds up skin microcirculation, sculpts skin elasticity, makes skin ruddy, tender, rejuvenated and smooth, reduces wrinkles, stimulates the regeneration of skin cells and activates active ingredient through specific red light waveband.

Blue Light: with a wavelength of 462NM. It contracts pores, gets rid of oily skin, calms skin, removes acne, rapidly locks moisture and essence, makes skin more elastic, alleviates skin's inflammation, repairs dark acne, do sterilization and removes acne through specific blue light waveband.

Green Light: with a wavelength of 527NM. It soothes and repairs skin, eases pain, promotes blood circulation through specific green light waveband. It suits skin with thin stratum corneum and are prone to allergy and redness.

Purple Light: with a wavelength of 600NM. It's a dual band light that consists of red and blue light. It combines the effect of two phototherapy, especially it has very good effect on and repairing effect in curing acne and removing acne marks.

Yellow Light: with a wavelength of 590NM. It replenishes energy for skin cell, promotes gland function, helps with digestion, cures skin diseases, boosts immunity, accelerates blood circulation, and effectively repairs red marks and red spots of skin, and relives swelling, pigment. It suits for acne removing and skin brightening.

White Light: with a wavelength of 670NM. It penetrates deep skin, accelerates metabolism of active tissue, breakdowns color spots, improves fine lines and saggy skin, relaxes skin, balances secretion, and controls grease.

Turquoise Light: with a wavelength of 610NM. It can enhance cell energy in a gradual way and has a good curative effect on metabolism through specific turquoise light waveband.

Microcurrent&LED Light Therapy-Fine Lines Reduction&Skin Firming



Microcurrent-safe and painless facial helps erase fine lines and wrinkles, while firming your skin and defining your features. It improves muscle tone, reduces puffiness, increases cellular activity, and tightens pores.that is why the result will be double after LED light therapy for acne, wrinkle, the overall result is a healthier and younger looking skin, no matter your age.

Effects

1. Cleanse deep skin and improve pores.
2. Remove the excessive stratum corneum.
3. Replenish moisture and nutrition.
4. Improve skin quality and increase skin's elasticity.
5. Improve uneven and unsmooth skin.

Indications

1. People with dry or hydropenic skin.
2. People with large pores, coarse or lackluster skin.
3. People with poor skin absorption, or hydropenia of underlying skin.
4. People with dark skin, or uneven skin color.
5. People with oversecretion of grease, or oily or dry skin.
6. People who don't clean the skin properly in daily life, or with unsmooth skin.

Contraindications

1. People with severely sensitive skin, or in allergic period.
2. People with extensive trauma on face.
3. Women in pregnancy, or people with severe heart disease, serious Three Highs(hypertension, hyperglycemia, hyperlipidemia), other serious diseases, etc.
4. People with skin diseases or infectious diseases.
5. People who just had mesotherapy, microneedle, or with wound on skin.
6. People who are sensitive to electric current should use with caution.

Notes(after treatment)

1. Wash the face with warm water within three days.
2. Replenish more water, and protect yourself from the sun.
3. Don't use irritant skin care products that contain scrub, exfoliant, AHA(Alpha Hydroxyl Acid), etc.
4. Avoid staying up late, smoking or drinking alcohol. Instead, eat more vegetables, fruits and light food.
5. Apply facial mask at least three times in a week.

Skin Peeling with Diamond Dermabrasion

Principle

Diamond Dermabrasion

Embedded with tiny diamond particles of different sizes, the specialized diamond toned wand, matched with vacuum suction, grinds off a shallow layer of epidermis by rubbing back and forth. It's a noninvasive and physical dermabrasion. It removes dead skin cells of epidermis to regenerate them and to improve skin quality. After treatment, skin's fine lines lessen, and skin becomes smooth, flat, and more tender. Meanwhile, matched with hands techniques, it can achieve the effect of lifting.

Physical Effect: Diamond dermabrasion adopts vacuum suction technology. By decreasing internal air pressure, it effectively removes old and useless stratum corneum and sucks out wastes, like blackheads and redundant grease, from pores, thus achieving the effect of free breath and circulation for skin, dredging pores, improving large pores, and brightening skin color.

Double Deep Cleanse-Dermabrasion



Dermabrasion adopts diamond crystals that have been processed specially to ill skin, and use vacuum suction to remove necrotic cells from skin as well as remove the deep waste and dirty of pore, help skin double cleansing and promote growth of new skin.

Active Oxygen Spray Bottle

By utilizing oxygen, water and strong pressure, it acts on skin with molecular particle that smaller than 80 micrometer and spray, which makes active water molecule and pure oxygen enter into deep skin rapidly, and accelerates the absorption of skin nutrition. By penetrating the pores and skin crack with strong pressure, it infiltrates from epidermis to dermis so as to stimulate cell regeneration, replenish adequate nutrition to skin directly and rapidly, and meanwhile remove wastes from deeper epidermis. High pressure oxygen and nutrient solution can stimulate the regeneration of dermis's fibrous, promote metabolism, and achieve the effect of moisturizing, skin rejuvenating and yellowish complexion eliminating.

Physical Effect: Active oxygen spray bottle nanometerizes and atomizes oxygen molecule. Through pulsed-propulsion, it injects oxygen molecule into skin's each layer, replenish the needed moisture and nutrient to cells, thus recovering cell's vitality and achieving skin's plumpness, moisturizing and elasticity.

Double Skin Moisturizing & Nourishing-Sprayer




Cold Hammer

It adopts the principle of AC. It absorbs energy by using special ceramic chip, which can reach minus 5 degrees instantly. Heat energy produced by chip absorbing energy causes one side generating heat that reaches up to 40 degrees by utilizing physical heat focusing. Fan discharges the excessive heat energy so as to achieve the effect of generating constant temperature and freezing alternately. It stimulates metabolism, alleviates allergy, contracts pores, calms skin and whiten and rejuvenate skin.

Biological Effect: When in use, cold hammer reduces tissue's temperature, contracts blood vessels and veins, decreases metabolic rate, inhibits and slows down cell activity, lowers the sensitivity of nerve endings with a sedative and soothing effect, eases and

relieves pain, swelling and ache for skin, especially for allergic skin, so as to achieve the effect of skin calming, swelling and redness diminishing, and pores shrinking.

Skin Tightening-Cold Therapy

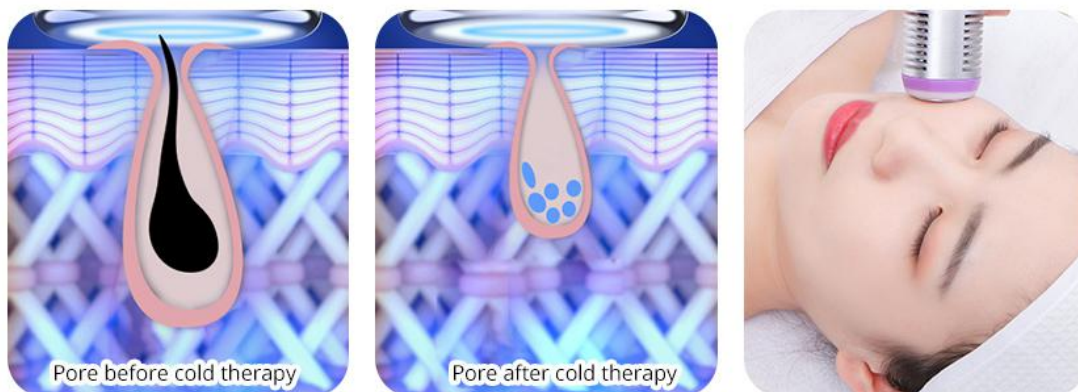

Use the cold therapy hammer to shrink pore after skin moisturizing and nutrition importation, this cooling therapy can promote skin tightening and enhance acne relieving and smoothing .

Effects

1. Remove old and useless stratum corneum.
2. Cleanse deep skin.
3. Dredge pores.
4. Brighten skin color and lift skin.
5. Improve large pores and fade fine lines.
6. Strongly remove strawberry-like nose, acne and blackheads.

Indications

1. People with slack facial skin.
2. People with clogged pores, or relatively thick stratum corneum.
3. People with dull or lackluster skin.
4. People with coarse skin or large pores.
5. People with strong grease secretion.

Contraindications

1. People with severely sensitive skin, or in allergic period.
2. People with prostheses inside nose.
3. People with extensive trauma on face.
4. Woman in pregnancy, or people with severe heart disease, serious Three Highs(hypertension, hyperglycemia, hyperlipidemia), other serious diseases, etc.
5. People with skin diseases or infectious diseases.
6. People who just had injections, such as hyaluronic acid, mesotherapy,

wrinkles removal, or plastic surgery.

Notes(after treatment)

1. Wash the face with warm water within three days.
2. Replenish more water, and protect yourself from the sun.
3. Don't use irritant skin care products that contain scrub, exfoliant, AHA(Alpha Hydroxyl Acid), etc.
4. Avoid staying up late, smoking or drinking alcohol. Instead, eat more vegetables, fruits and light food.
5. Apply hydrating mask one time each day within three days.

Easing & Repairing Management


Principle

Cold Hammer


It adopts the principle of AC. It absorbs energy by using special ceramic chip, which can reach minus 5 degrees instantly. Heat energy produced by chip absorbing energy causes one side generating heat that reaches up to 40 degrees by utilizing physical heat focusing. Fan discharges the excessive heat energy so as to achieve the effect of generating constant temperature and freezing alternately. It stimulates metabolism, alleviates allergy, contracts pores, calms skin and whiten and rejuvenate skin.

Biological Effect: When in use, cold hammer reduces tissue's temperature, contracts blood vessels and veins, decreases metabolic rate, inhibits and slows down cell activity, lowers the sensitivity of nerve endings with a sedative and soothing effect, eases and relieves pain, swelling and ache for skin, especially for allergic skin, so as to achieve the effect of skin calming, swelling and redness diminishing, and pores shrinking.

Skin Tightening-Cold Therapy

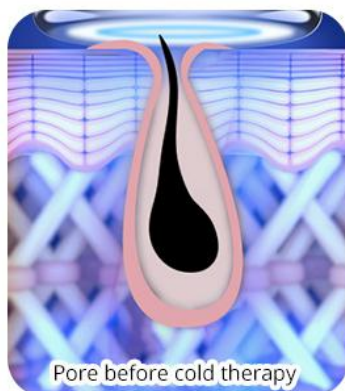

Pore before cold therapy

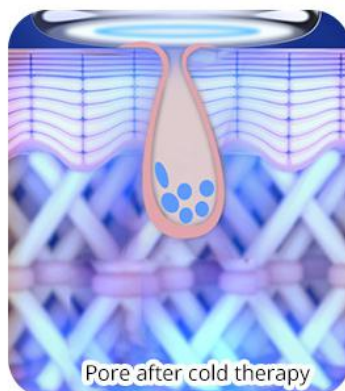

Pore after cold therapy



Use the cold therapy hammer to shrink pore after skin moisturizing and nutrition importation, this cooling therapy can promote skin tightening and enhance acne relieving and smoothing .

Active Oxygen Spray Bottle

By utilizing oxygen, water and strong pressure, it acts on skin with molecular particle that smaller than 80 micrometer and spray, which makes active water molecule and pure oxygen enter into deep skin rapidly, and accelerates the absorption of skin nutrition. By penetrating the pores and skin crack with strong pressure, it infiltrates from epidermis to dermis so as to stimulate cell regeneration, replenish adequate nutrition to skin directly and rapidly, and meanwhile remove wastes from deeper epidermis. High pressure oxygen and nutrient solution can stimulate the regeneration of dermis's fibrous, promote metabolism, and achieve the effect of moisturizing, skin rejuvenating and yellowish complexion eliminating.

Physical Effect: Active oxygen spray bottle nanometerizes and atomizes oxygen molecule. Through pulsed-propulsion, it injects oxygen molecule into skin's each layer, replenish the needed moisture and nutrient to cells, thus recovering cell's vitality and achieving skin's plumpness, moisturizing and elasticity.

Double Skin Moisturizing & Nourishing-Sprayer



Bio & Photon Phototherapy Brush

It adopts the principle of high-tech photorejuvenation with photodynamics to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, light energy was transformed into intracellular energy which expands and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin. This increases activity of catalase and superoxide dismutase (SOD), which serve as the "guard" and "janitor" for cells, and increases breakdown of adenosine triphosphate (ATP), a source of cellular energy, and increases glycogen and protein content so as to accelerate cells metabolism and regeneration. It stimulates the secretion of body alkaline synthetic fiber growth factor (BFGF) and epidermal growth factor (EGF), increases collagenous fiber's elasticity,

rearranges fibers, blocks and eliminates the formation of melanin, accelerates the growth of cells and blood circulation, stimulates fibrocyte tissue producing collagen, and increases the elasticity of skin. Moreover, it plays an important role in repairing aging and acne skin, and in fading spots, and in firming and lifting, as well as in alleviating sunburns.

Biological Effect: It adopts photodynamic principle to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, light energy was transformed into intracellular energy which expands and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin, and increases glycogen and protein content so as to accelerate cells metabolism and regeneration.

Red Light: with a wavelength of 635NM. It warms up deep skin, activates collagen's activity, speeds up skin microcirculation, sculpts skin elasticity, makes skin ruddy, tender, rejuvenated and smooth, reduces wrinkles, stimulates the regeneration of skin cells and activates active ingredient through specific red light waveband.

Blue Light: with a wavelength of 462NM. It contracts pores, gets rid of oily skin, calms skin, removes acne, rapidly locks moisture and essence, makes skin more elastic, alleviates skin's inflammation, repairs dark acne, do sterilization and removes acne through specific blue light waveband.

Green Light: with a wavelength of 527NM. It soothes and repairs skin, eases pain, promotes blood circulation through specific green light waveband. It suits skin with thin stratum corneum and are prone to allergy and redness.


Purple Light: with a wavelength of 600NM. It's a dual band light that consists of red and blue light. It combines the effect of two phototherapy, especially it has very good effect on and repairing effect in curing acne and removing acne marks.

Yellow Light: with a wavelength of 590NM. It replenishes energy for skin cell, promotes gland function, helps with digestion, cures skin diseases, boosts immunity, accelerates blood circulation, and effectively repairs red marks and red spots of skin, and relives swelling, pigment. It suits for acne removing and skin brightening.

White Light: with a wavelength of 670NM. It penetrates deep skin, accelerates metabolism of active tissue, breakdowns color spots, improves fine lines and saggy skin, relaxes skin, balances secretion, and controls grease.

Turquoise Light: with a wavelength of 610NM. It can enhance cell energy in a gradual way and has a good curative effect on metabolism through specific turquoise light waveband.

Microcurrent&LED Light Therapy-Fine Lines Reduction&Skin Firming

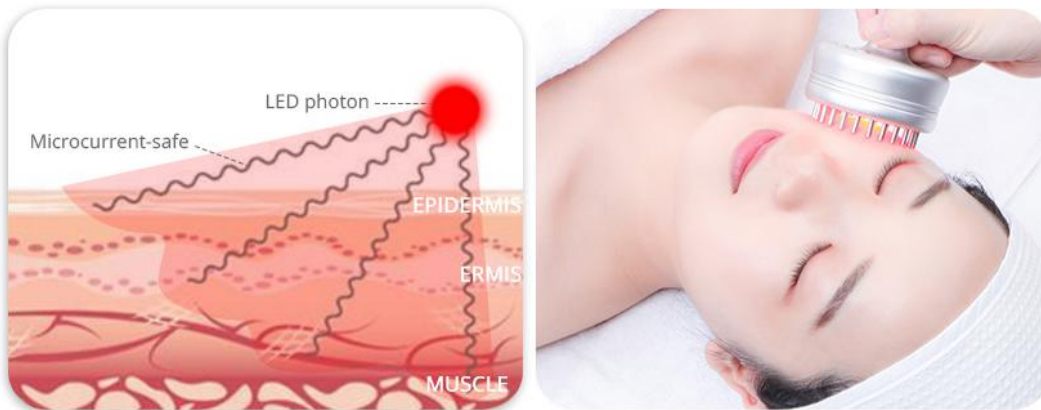

Microcurrent-safe and painless facial helps erase fine lines and wrinkles, while firming your skin and defining your features. It improves muscle tone, reduces puffiness, increases cellular activity, and tightens pores.that is why the result will be double after LED light therapy for acne, wrinkle, the overall result is a healthier and younger looking skin, no matter your age.

Effects

1. Improve symptoms like facial redness, acne and allergy.
2. Calm and ease skin.
3. Contract pores, and improve large pores.
4. Increase skin's elasticity.
5. Enhance skin's immunity, and repair sensitive skin.

Indications

1. People with sensitive or acne skin, or whose skin are prone to swelling or allergy.
2. People with slack, saggy or inelastic skin.
3. People with dry skin, fine lines, dark or uneven skin color, or lackluster skin.
4. People with large pores, coarse skin, or strong grease secretion.

Contraindications

1. People with severely sensitive skin, or in allergic period.
2. People with prostheses inside nose.
3. People with extensive trauma on face.
4. Woman in pregnancy, or people with severe heart disease, serious Three Highs(hypertension, hyperglycemia, hyperlipidemia), other serious diseases, etc.
5. People with skin diseases or infectious diseases.
6. People who just had mesotherapy, microneedle, or with wound on skin.
7. People with metal implants inside body, such as stent, pacemaker, etc., or who are allergic to metals.

Notes(after treatment)

1. Wash the face with warm water within three days.
2. Replenish more water, and protect yourself from the sun.
3. Don't use irritant skin care products that contain scrub, exfoliant, AHA(Alpha Hydroxyl Acid), etc.
4. Avoid staying up late, smoking or drinking alcohol. Instead, eat more vegetables, fruits and light food.
5. Apply hydrating mask one time each day within three days.

Facial Rejuvenation & Balance Management

Principle

Diamond Dermabrasion

Embedded with tiny diamond particles of different sizes, the specialized diamond toned wand, matched with vacuum suction, grinds off a shallow layer of epidermis by rubbing back and forth. It's a noninvasive and physical dermabrasion. It removes dead skin cells of epidermis to regenerate them and to improve skin quality. After treatment, skin's fine lines lessen, and skin becomes smooth, flat, and more tender. Meanwhile, matched with hands techniques, it can achieve the effect of lifting.

Physical Effect: Diamond dermabrasion adopts vacuum suction technology. By decreasing internal air pressure, it effectively removes old and useless stratum corneum and sucks out wastes, like blackheads and redundant grease, from pores, thus achieving the effect of free breath and circulation for skin, dredging pores, improving large pores, and brightening skin color.

Double Deep Cleanse-Dermabrasion



Dermabrasion adopts diamond crystals that have been processed specially to ill skin, and use vacuum suction to remove necrotic cells from skin as well as remove the deep waste and dirty of pore, help skin double cleansing and promote growth of new skin.

Skin Scrubber


It applies the principle of ultrasound with high frequency vibration. Energy produced by mechanical vibration of 28,000 times per second has an obvious permeation effect on human skin. When ultrasonic cavitation effect encounters water, it atomizes water into small bubbles. The small bubbles, through constant vibration, go deep into skin's pores breaking acne, blackheads and dark acne into granulum which will be vibrated onto skin's surface through high frequency and ultrasonic vibration. Low frequency emulsifies sebum and excreta that clogged pores, and meanwhile it can also do sterilization, atomizes water molecules, removes excess thick cutin, purifies pores and decomposes grease through mechanical vibration after entering into pores, so as to make the pores unobstructed, contract the pores, and make the skin bright, clear, soft and smooth.

Physical Effect: Sound wave of high frequency vibration, with the matched use of softened products or exfoliating cream, can soften the impaired and rough stratum corneum, open pores, and deeply remove the wastes from pores.

Skin Exfoliating by 3MHz Ultrasound Scrubber



Utilize 3MHz high frequency ultrasound wave with exfoliating product such as AHA or exfoliating cream to soften the damaged rough epidermal stratum corneum, then open the pores, make the waste out of the pores by high frequency vibration caused by ultrasound.

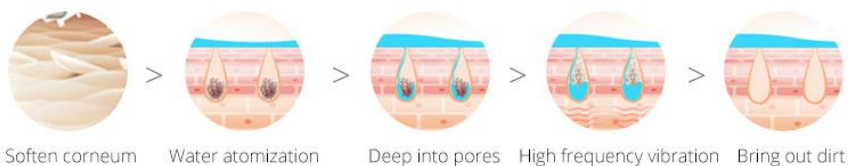

Bio & Photon Phototherapy Brush

It adopts the principle of high-tech photorejuvenation with photodynamics to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, light energy was transformed into intracellular energy which expands and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin. This increases activity of catalase and superoxide dismutase (SOD), which serve as the "guard" and "janitor" for cells, and

increases breakdown of adenosine triphosphate (ATP), a source of cellular energy, and increases glycogen and protein content so as to accelerate cells metabolism and regeneration. It stimulates the secretion of body alkaline synthetic fiber growth factor (BFGF) and epidermal growth factor (EGF), increases collagenous fiber's elasticity, rearranges fibers, blocks and eliminates the formation of melanin, accelerates the growth of cells and blood circulation, stimulates fibrocyte tissue producing collagen, and increases the elasticity of skin. Moreover, it plays an important role in repairing aging and acne skin, and in fading spots, and in firming and lifting, as well as in alleviating sunburns.

Biological Effect: It adopts photodynamic principle to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, light energy was transformed into intracellular energy which expands and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin, and increases glycogen and protein content so as to accelerate cells metabolism and regeneration.

Red Light: with a wavelength of 635NM. It warms up deep skin, activates collagen's activity, speeds up skin microcirculation, sculpts skin elasticity, makes skin ruddy, tender, rejuvenated and smooth, reduces wrinkles, stimulates the regeneration of skin cells and activates active ingredient through specific red light waveband.

Blue Light: with a wavelength of 462NM. It contracts pores, gets rid of oily skin, calms skin, removes acne, rapidly locks moisture and essence, makes skin more elastic, alleviates skin's inflammation, repairs dark acne, do sterilization and removes acne through specific blue light waveband.

Green Light: with a wavelength of 527NM. It soothes and repairs skin, eases pain, promotes blood circulation through specific green light waveband. It suits skin with thin stratum corneum and are prone to allergy and redness.


Purple Light: with a wavelength of 600NM. It's a dual band light that consists of red and blue light. It combines the effect of two phototherapy, especially it has very good effect on and repairing effect in curing acne and removing acne marks.

Yellow Light: with a wavelength of 590NM. It replenishes energy for skin cell, promotes gland function, helps with digestion, cures skin diseases, boosts immunity, accelerates blood circulation, and effectively repairs red marks and red spots of skin, and relives swelling, pigment. It suits for acne removing and skin brightening.

White Light: with a wavelength of 670NM. It penetrates deep skin, accelerates metabolism of active tissue, breakdowns color spots, improves fine lines and saggy skin, relaxes skin, balances secretion, and controls grease.

Turquoise Light: with a wavelength of 610NM. It can enhance cell energy in a gradual way and has a good curative effect on metabolism through specific turquoise light waveband.

Microcurrent&LED Light Therapy-Fine Lines Reduction&Skin Firming

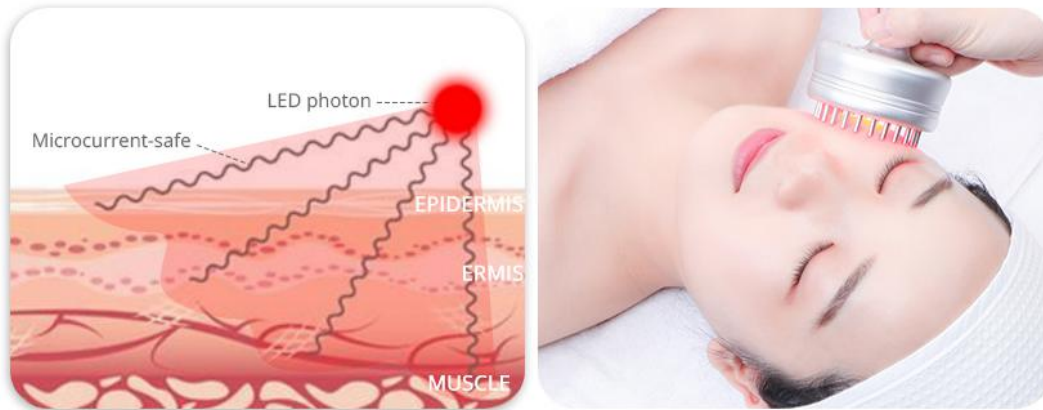

Microcurrent-safe and painless facial helps erase fine lines and wrinkles, while firming your skin and defining your features. It improves muscle tone, reduces puffiness, increases cellular activity, and tightens pores.that is why the result will be double after LED light therapy for acne, wrinkle, the overall result is a healthier and younger looking skin, no matter your age.

Active Oxygen Spray Bottle

By utilizing oxygen, water and strong pressure, it acts on skin with molecular particle that smaller than 80 micrometer and spray, which makes active water molecule and pure oxygen enter into deep skin rapidly, and accelerates the absorption of skin nutrition. By penetrating the pores and skin crack with strong pressure, it infiltrates from epidermis to dermis so as to stimulate cell regeneration, replenish adequate nutrition to skin directly and rapidly, and meanwhile remove wastes from deeper epidermis. High pressure oxygen and nutrient solution can stimulate the regeneration of dermis's fibrous, promote metabolism, and achieve the effect of moisturizing, skin rejuvenating and yellowish complexion eliminating.

Physical Effect: Active oxygen spray bottle nanometerizes and atomizes oxygen molecule. Through pulsed-propulsion, it injects oxygen molecule into skin's each layer, replenish the needed moisture and nutrient to cells, thus recovering cell's vitality and achieving skin's plumpness, moisturizing and elasticity.

Double Skin Moisturizing &Nourishing-Sprayer



Effects

1. Replenish water to deep skin and replenish nutrition.
2. Remove skin's excessive cutin, regulate water and oil balance.
3. Remove blackheads and improve strawberry-like nose.
4. Contract pores, whiten and rejuvenate skin.
5. Accelerate blood circulation and increase skin's elasticity.
6. Improve red blood streak.

Indications

1. People with unbalanced water and oil.
2. People with dry or hydropenic skin, or desquamation.
3. People with large or clogged pores.
4. People with dark skin color, or lackluster skin.
5. People with poor metabolism or absorption.
6. People with thick stratum corneum, or strong grease secretion.

Contraindications

1. People with severely sensitive skin, or in allergic period.
2. People with prostheses inside nose.
3. People with extensive trauma on face.
4. Women in pregnancy, or people with severe heart disease, serious Three Highs(hypertension, hyperglycemia, hyperlipidemia), other serious diseases, etc.
5. People with skin diseases or infectious diseases.
6. People who just had mesotherapy, microneedle, or with wound on skin.
7. People with metal implants inside body, such as stent, pacemaker, etc., or

who are allergic to metals.

Notes(after treatment)

1. Wash the face with warm water within three days.
2. Replenish more water, and protect yourself from the sun.
3. Don't use irritant skin care products that contain scrub, exfoliant, AHA(Alpha Hydroxyl Acid), etc.
4. Avoid staying up late, smoking or drinking alcohol. Instead, eat more vegetables, fruits and light food.
5. Apply hydrating mask one time each day within three days.

Ceramic Skin Restoring Management

Principle

Diamond Dermabrasion

Embedded with tiny diamond particles of different sizes, the specialized diamond toned wand, matched with vacuum suction, grinds off a shallow layer of epidermis by rubbing back and forth. It's a noninvasive and physical dermabrasion. It removes dead skin cells of epidermis to regenerate them and to improve skin quality. After treatment, skin's fine lines lessen, and skin becomes smooth, flat, and more tender. Meanwhile, matched with hands techniques, it can achieve the effect of lifting.

Physical Effect: Diamond dermabrasion adopts vacuum suction technology. By decreasing internal air pressure, it effectively removes old and useless stratum corneum and sucks out wastes, like blackheads and redundant grease, from pores, thus achieving the effect of free breath and circulation for skin, dredging pores, improving large pores, and brightening skin color.

Double Deep Cleanse-Dermabrasion




Dermabrasion adopts diamond crystals that have been processed specially to ill skin, and use vacuum suction to remove necrotic cells from skin as well as remove the deep waste and dirty of pore, help skin double cleansing and promote growth of new skin.

Skin Scrubber

It applies the principle of ultrasound with high frequency vibration. Energy produced by mechanical vibration of 28,000 times per second has an obvious permeation effect on human skin. When ultrasonic cavitation effect encounters water, it atomizes water into small bubbles. The small bubbles, through constant vibration, go deep into skin's pores breaking acne, blackheads and dark acne into granulum which will be vibrated onto skin's surface through high frequency and ultrasonic vibration. Low frequency emulsifies sebum and excreta that clogged pores, and meanwhile it can also do sterilization, atomizes water molecules, removes redundant thick cutin, purifies pores and decomposes grease through mechanical vibration after entering into pores, so as to make the pores unobstructed, contract the pores, and make the skin bright, clear, soft and smooth.

Physical Effect: Sound wave of high frequency vibration, with the matched use of softened products or exfoliating cream, can soften the impaired and rough stratum corneum, open pores, and deeply remove the wastes from pores.

Skin Exfoliating by 3MHz Ultrasound Scrubber

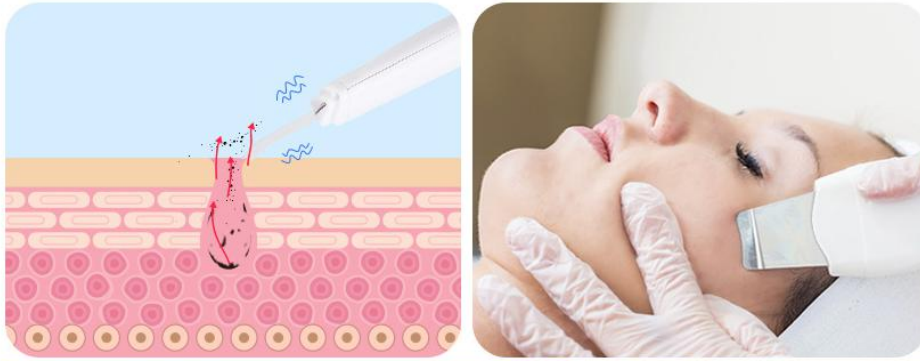

Utilize 3MHz high frequency ultrasound wave with exfoliating product such as AHA or exfoliating cream to soften the damaged rough epidermal stratum corneum , then open the pores, make the waste out of the pores by high frequency vibration caused by ultrasound.



Active Oxygen Spray Bottle

By utilizing oxygen, water and strong pressure, it acts on skin with molecular particle that smaller than 80 micrometer and spray, which makes active water molecule and pure oxygen enter into deep skin rapidly, and accelerates the absorption of skin nutrition. By penetrating the pores and skin crack with strong pressure, it infiltrates from epidermis to dermis so as to stimulate cell regeneration, replenish adequate nutrition to skin directly and rapidly, and meanwhile remove wastes from deeper epidermis. High pressure oxygen and nutrient solution can stimulate the regeneration of dermis's fibrous, promote metabolism, and achieve the effect of moisturizing, skin rejuvenating and yellowish complexion eliminating.

Physical Effect: Active Oxygen spray bottle nanometerizes and atomizes oxygen molecule. Through pulsed-propulsion, it injects oxygen molecule into skin's each layer, replenish the needed moisture and nutrient to cells, thus recovering cell's vitality and achieving skin's plumpness, moisturizing and elasticity.

Double Skin Moisturizing & Nourishing-Sprayer




Water or nutrition will be double absorbed caused by spray refining into molecules.

Cold Hammer

It adopts the principle of AC. It absorbs energy by using special ceramic chip, which can reach minus 5 degrees instantly. Heat energy produced by chip absorbing energy causes one side generating heat that reaches up to 40 degrees by utilizing physical heat focusing. Fan discharges the excessive heat energy so as to achieve the effect of generating constant temperature and freezing alternately. It stimulates metabolism, alleviates allergy, contracts pores, calms skin and whiten and rejuvenate skin.

Biological Effect: When in use, cold hammer reduces tissue's temperature, contracts blood vessels and veins, decreases metabolic rate, inhibits and slows down cell activity, lowers the sensitivity of nerve endings with a sedative and soothing effect, eases and relieves pain, swelling and ache for skin, especially for allergic skin, so as to achieve the effect of skin calming, swelling and redness diminishing, and pores shrinking.

Skin Tightening-Cold Therapy

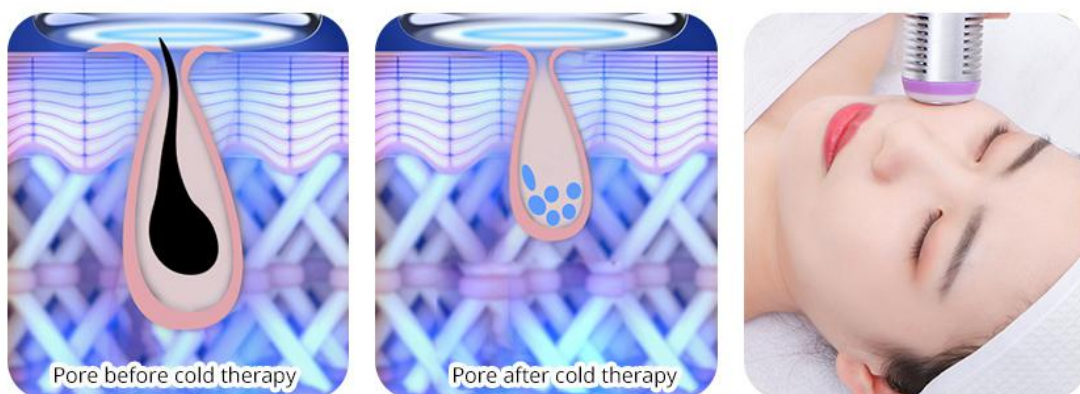

Use the cold therapy hammer to shrink pore after skin moisturizing and nutrition importation, this cooling therapy can promote skin tightening and enhance acne relieving and smoothing .

Bio & Photon Phototherapy Brush

It adopts the principle of high-tech photorejuvenation with photodynamics to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, light energy was transformed into intracellular energy which expands and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin. This increases activity of catalase and superoxide dismutase (SOD), which serve as the "guard" and "janitor" for cells, and increases breakdown of adenosine triphosphate (ATP), a source of cellular energy, and increases glycogen and protein content so as to accelerate cells metabolism and regeneration. It stimulates the secretion of body alkaline synthetic fiber growth factor (BFGF) and epidermal growth factor (EGF), increases collagenous fiber's elasticity, rearranges fibers, blocks and eliminates the formation of melanin, accelerates the growth of cells and blood circulation, stimulates fibrocyte tissue producing collagen, and increases the elasticity of skin. Moreover, it plays an important role in repairing aging and acne skin, and in fading spots, and in firming and lifting, as well as in alleviating sunburns.

Biological Effect: It adopts photodynamic principle to activate deep cells and make skin have a better metabolism. After the ray of light was absorbed by skin, light energy was transformed into intracellular energy which expands and strengthens micro-vessels and causes it to have photochemical reaction-enzymatic reaction with skin, and increases glycogen and protein content so as to accelerate cells metabolism and regeneration.

Red Light: with a wavelength of 635NM. It warms up deep skin, activates collagen's activity, speeds up skin microcirculation, sculpts skin elasticity, makes skin ruddy, tender, rejuvenated and smooth, reduces wrinkles, stimulates the regeneration of skin cells and activates active ingredient through specific red light waveband.

Blue Light: with a wavelength of 462NM. It contracts pores, gets rid of oily skin, calms skin, removes acne, rapidly locks moisture and essence, makes skin more elastic, alleviates skin's inflammation, repairs dark acne, do sterilization and removes acne through specific blue light waveband.

Green Light: with a wavelength of 527NM. It soothes and repairs skin, eases pain, promotes blood circulation through specific green light waveband. It suits skin with thin stratum corneum and are prone to allergy and redness.

Purple Light: with a wavelength of 600NM. It's a dual band light that consists of red and blue light. It combines the effect of two phototherapy, especially it has very good effect on and repairing effect in curing acne and removing acne marks.

Yellow Light: with a wavelength of 590NM. It replenishes energy for skin cell,

promotes gland function, helps with digestion, cures skin diseases, boosts immunity, accelerates blood circulation, and effectively repairs red marks and red spots of skin, and relieves swelling, pigment. It suits for acne removing and skin brightening.

White Light: with a wavelength of 670NM. It penetrates deep skin, accelerates metabolism of active tissue, breakdowns color spots, improves fine lines and saggy skin, relaxes skin, balances secretion, and controls grease.

Turquoise Light: with a wavelength of 610NM. It can enhance cell energy in a gradual way and has a good curative effect on metabolism through specific turquoise light waveband.

Microcurrent&LED Light Therapy-Fine Lines Reduction&Skin Firming



Microcurrent-safe and painless facial helps erase fine lines and wrinkles, while firming your skin and defining your features. It improves muscle tone, reduces puffiness, increases cellular activity, and tightens pores.that is why the result will be double after LED light therapy for acne, wrinkle, the overall result is a healthier and younger looking skin, no matter your age.

Effects

1. Replenish moisture and nutrition.
2. Improve skin quality and increase skin's elasticity.
3. Improve uneven and unsmooth skin.
4. Remove old and useless stratum corneum.
5. Cleanse deep skin.
6. Dredge pores and regulate grease secretion.
7. Brighten skin color and lift skin.
8. Improve large pores and fade fine lines.
9. Strongly remove strawberry-like nose, acne and blackheads.
10. Accelerate blood circulation and skin's metabolism.

Indications

1. People with large pores, coarse or lackluster skin.

2. People with dry or hydropenic skin.
3. People with poor skin absorption, or hydropenia of underlying skin.
4. People with dark skin, or uneven skin color.
5. People with oversecretion of grease, or oily or dry skin.
6. People who don't clean the skin properly in daily life, or with unsmooth skin.
7. People with slack facial skin.
8. People with clogged pores, or relatively thick stratum corneum.
9. People with dull or lackluster skin.
10. People with coarse skin or large pores.
11. People with strong grease secretion.

Contraindications

1. People with severely sensitive skin, or in allergic period.
2. People with prostheses inside nose.
3. People with extensive trauma on face.
4. Woman in pregnancy, or people with severe heart disease, serious Three Highs(hypertension, hyperglycemia, hyperlipidemia), other serious diseases, etc.
5. People with skin diseases or infectious diseases.
6. People who just had mesotherapy, microneedle, or with wound on skin.
7. People with metal implants inside body, such as stent, pacemaker, etc., or who are allergic to metals.

Notes(after treatment)

1. Wash the face with warm water within three days.
2. Replenish more water, and protect yourself from the sun.
3. Don't use irritant skin care products that contain scrub, exfoliant, AHA(Alpha Hydroxyl Acid), etc.
4. Avoid staying up late, smoking or drinking alcohol. Instead, eat more vegetables, fruits and light food.
5. Apply hydrating mask one time each day within three days.

Part II

1. Detailed Operations

Check the equipment and make sure it's properly connected. The following initial page will appear after having the power switch on.



1.1 Function Selection



Time Setting & Display



Energy Level Setting & Display



Energy Level Setting(decrease)



Energy Level Setting(increase)



Mode Selection:



Start/ Pause



Rotating Knob for Suction Setting(rotate to the right to increase suction and to the left to decrease)

1.2 Detailed Operations for Cold & Hot Hammer



Select and enter the following page.



Time Setting & Display



Energy Level Setting & Display



Energy Level Setting(decrease)



Energy Level Setting(increase)



Mode for Cold & Hot Hammer

Mode A is cooling, used for calming, repairing and contracting pores.
Mode B is heating, used for opening pores and infusion.



Start/ Pause

1.3 Detailed Operations for Bio & Photon



Select and enter the following page.



Time Setting & Display



Energy Level Setting & Display



Energy Level Setting(decrease)



Energy Level Setting(increase)



Mode for Bio & Photon

Mode A: discontinuous energy releasing, suitable for people who just started or first-time user. Mode B: discontinuous energy releasing with a relatively faster frequency, suitable for people with slight yellowish skin.

Mode C: continuous energy releasing, suitable for people with acne, coarse or dark skin.



Start/ Pause

Detailed Installations for Bio & Photon Phototherapy Brush



The operating head is as shown in the photo.



On/ Off



Mode Selection(can choose different colors)



Button for single color flashing, namely only one color of light flashes(press MODE to switch color)



Button for color switching circularly(multiple colors flash circularly)

1.4 Detailed Operations for Oxygen Spray Bottle



Select and enter the following page.



Time Setting & Display



Start/ Pause



Air Outlet(It's on the left side of rotating knob for suction setting. Insert rubber hose into this hole when operating oxygen injector.)




Open the black lid before operation, then refill water into the white bottle(can be refilled with pure water, toner, normal saline, watery essence, etc. Choose in line with skin conditions.) . It's advised to refill half of the bottle when using it every time.




Insert one end of the rubber hose into air outlet, and the other end into the place shown by the arrow. (It's prohibited to insert it into air inlet so as not to damage the equipment.)



Click , then press on the place shown by the arrow with thumb finger to spray water mist. Release the thumb finger to stop spray.

1.5 Detailed Operations for Diamond Dermabrasion



Select  and enter the following page.





Time Setting & Display



Start/ Pause



Air Inlet(It's on the right side of rotating knob for suction setting. Insert rubber hose into this hole when operating diamond dermabrasion.)



Rotating Knob for Suction Setting(rotate to the right to increase suction and to the left to decrease)

Detailed Installations for Diamond Dermabrasion Wand

Insert one end of the rubber hose into air inlet first(It's prohibited to insert it into air outlet, otherwise there is no suction.)



Install rubber ring(Rubber ring must be installed before operation, otherwise there is air leakage or without suction.)



Install filter cotton(It must be installed when operating, otherwise dirt may enter into the equipment causing damages to it. Meanwhile, the cleaning effect can be seen.)



Install the diamond head onto the wand.



Insert the other end of rubber hose into the bottom of the wand.



Turn on the equipment, set the suction level to the highest, and check whether there is suction.

Caution:

- (1) Start with the lowest energy level when just started or use it for the first time, and increase the level gradually after client got used to it.
- (2) The advised operating time for single body part is about 10 minutes.

There are 9 diamond heads in total, which can be chosen according to individual skin conditions.



This is the biggest, namely No.1 diamond head(suitable for big area or body treatment)



This is No.2 diamond head(suitable for big area or body parts treatment)



This is No.3 diamond head(suitable for big area treatment)



This is No.4 diamond head(suitable for big area treatment, face lifting, and cutin removing)



This is No.5 diamond head(suitable for cheek exfoliating, cleansing, and pore dirt sucking)



This is No.6 diamond head(suitable for sucking blackheads for face and nose, and clearing out pore obstruction)



This is No.7 diamond head(suitable for face and nose cleansing, and cutin

removing)



This is No.8 diamond head(suitable for sucking blackheads and cleansing dirt for nasion)



This is No.9 diamond head(suitable for removing stubborn blackheads on nasion or small area)

1.6 Detailed Operations for Skin Scrubber

The handle for skin scrubber is as shown in the photo.



On/ Off Press it for 3 seconds to turn on.

It's Mode 1 after turning it on, then it vibrates all the time(suitable for people with large pores, severely clogged pores and coarse skin).

One press to switch to Mode 2, which vibrates discontinuously(suitable for people with clogged pores and strong grease secretion).

Two press to switch to Mode 3, which vibrates discontinuously(suitable for people with coarse skin and who want to remove cutin).

Three press to pause.

2. Technical Parameters

Vacuum Therapy Parameters

Power: 60W
Pressure: >250kPa
Air flow: >10L/minute
Noise level: <70dB (30cm away)

Vacuum Therapy Parameters

Power: 60W
Pressure: >250kPa
Air flow: >10L/minute
Noise level: <70dB (30cm away)

Cold Hot Hammer

Cold: 0-5 degree
Hot: 45 degree

Microcurrent & Photon

7 colors : Red, Blue, Green, Purple, Orange, Turquoise, White

3. Contraindications

People with the following situations should use the equipment with caution. Please consult a doctor or professional before using this equipment. The details are as follow:

- (1) Women in pregnancy, or lactation.
- (2) People with heart disease, or heart pacemaker.
- (3) People with unhealed surgical wound, or in surgical recovery.
- (4) People with epilepsy, severe diabetes or hyperthyroidism.
- (5) People with malignant tumors, hemophiliac or severe bleeding.
- (6) People with skin diseases or infectious diseases.
- (7) People with metal implants inside body, such as stent, pacemaker, etc., or who are allergic to metals.
- (8) People who are sensitive to electric current use.

4. Precautions for Use

- (1) Plug with ground pin must be used, and power socket that already factually grounded must be ensured before using the equipment.
- (2) To make sure equipment's voltage is adaptive. If voltage of local power supply is unstable, we suggest users add voltage regulator with matching power between the mains supply and the equipment.
- (3) To guarantee curative effect and normal service life of the equipment, please uniformly use specified parts provided or suggested by original manufacturer.
- (4) The equipment can't be placed in damp places or near water, and also can't be exposed to direct sunlight.
- (5) Don't place the equipment near a strong heat source since this may affect its

service life and normal use.

- (6) Please remove all the metal objects from body before treatment so as to avoid unexpected situations which may affect curative effect.
- (7) Please restrain from using the equipment aiming at eyes, thyroid, parathyroid, testicles, pregnant woman's abdomen, pacemaker, etc.
- (8) People who are suffering from illness should use with caution unless he gets permission from a doctor.
- (9) Please turn off power switch of the equipment if no one uses it, and ensure main power is off after someone used it and before he left, thus safety of the equipment can be guaranteed.
- (10) Start with the lowest energy level when just started, and increase the level gradually after client got used to it.
- (11) Clean the equipment with normal saline after operation so as to ensure its cleanliness and hygiene, as well as extending its service life..
- (12) The operating heads can be used separately, but a better effect can be achieved with the combined use of them.
- (13) The equipment should contact skin fully so as to avoid uneven heating when in use.
- (14) When using this equipment, parts to be treated must be kept moist, and dry skin treatment should be avoided.
- (15) Keep it in mind that the air outlet and air inlet must not be mixed. Moreover, don't insert oxygen injector into air inlet, otherwise it may damage the equipment.
- (16) Rubber ring must be mounted when operating diamond dermabrasion, otherwise there is no suction or suction will be weakened.
- (17) Filter cotton must be installed when operating diamond dermabrasion, otherwise dirt may enter into air duct causing damages to the main machine.
- (18) To avoid discomfort caused by overheating, please don't touch air outlet when operating cold & hot hammer.
- (19) All the products used by oxygen injector must be watery ones.
- (20) Please remind the client to regulate breathe when operating active oxygen spray bottle.
- (21) Using the equipment and training the operators in strictly accordance with the instructions specified in the manual.

5. Troubleshooting & Solutions

- (1) The equipment can't be started, and the lights on its back are not working?
 - A. Make sure the power cord is plugged into a good power socket.
 - B. Check whether the fuse tube in the back is loose or burnt out.

- (2) The equipment can be started, but there is error message on the monitor?
 - A. Take out the plug in the back, and wait for about 1 minute and re-plug and restart it.

B. If the above-mentioned solutions are not working, please contact the equipment dealer for assistance.

(3) Active oxygen spray bottle doesn't have spray?

A. Please check whether the injector head is blocked.

B. Please check whether the connected part of injector tail is tightly twisted.

C. Please check whether the injector lever is loose.

(4) There is no response from skin scrubber's button?

A. Turn off the equipment and restart.

B. Take out the plug and re-plug.

(5) There is no suction when operating diamond dermabrasion?

A. Please check whether the rubber hose connecting to the main machine is properly connected.

B. Please check whether the rubber hose connecting to diamond dermabrasion is properly connected.

C. Please check whether the rotating knob for suction setting is set to zero.

6. FAQs

(1) Q: Does skin will become thin and sensitive after diamond dermabrasion treatment?

A: No. Diamond dermabrasion only gets rid of aging stratum corneum during treatment. Our skin refreshes every 28 days, thus diamond dermabrasion can only make skin become more and more smooth, tender, transparent and causes no sensitive symptoms.

(2) Q: Why the cutin must be removed?

A: Stratum corneum is the top layer of epidermis, on which cells are completely dead. Depending on body parts, the stratum corneum is consists of 15 to 20 layers of dead cells. Human's skin has a metabolic cycle of 28 days, which is exactly the cycle for people in adolescence. For adults, the cycle is the actual age plus 10. However, due to various reasons, the aged and dead cutin cells may not shed timely, which causes acanthokeratoderma and no transparency to skin, affects skin color and skin's metabolism and renewal. Therefore, if someone can't carry out the metabolism himself under normal conditions, then an artificial method must be used to achieve the purpose.

(3) Q: Can I wear makeup after using this equipment?

A: Absolutely. After the treatment, the skin turns moisturized and plump. It's more appropriate to wear makeup at this time. This treatment requires no skin breaking and is injury-free, safe, healthy, rapid and effective.

(4) Q: How long does it take to see the effect?

A: The skin becomes clean and bright obviously after one treatment, which will become more obvious if you stick to it. This treatment can fix problems like hydropenia of deep skin, dark skin, fine lines and aging.

(5) Q: Is there any dependency for this treatment?





A: There is no dependency. It's target at skin problems, help it improve, and cleanse deep skin, and maintains and prevents skin from having problems caused by other factors, and helps skin absorb nutrition, solve yellowish and hydrogenic skin, as well as acne skin. If quit in later stage, the skin will only back to previous conditions of secretion and senility, and it will not cause dependency.

7. Packing List

Main Machine x1
 Parts Rack x1
 Power Cord x1
 Cold & Hot Hammer x1
 Skin Scrubber x1
 Bio & Photon Phototherapy Brush x1
 Active Oxygen Spray Bottle x1
 Transparent Glass Tube x3
 Diamond Head x9
 Rubber Hose x2
 Filter Cotton x1 pack
 Black Rubber Ring x1 pack
 Fuse x2

8. Operational Diagrams


Parameter Setting	Product	Technique	Diagram
Cleansing Skin with Active Oxygen: 60 Minutes/ Once a Week			
Skin Scrubber: Mode: 1/ 2/ 3	Makeup Remover + Facial	1. Remove makeup and clean the face, 5 minutes.	Technique 4
Active Oxygen Spray Bottle: It can start to	Cleanser + Export Liquid + Base Mask	2. Apply export liquid and use hot steam(apply the liquid to parts with large pores or severely clogged	




<p>operate when it sprays water mist.</p> <p>Bio & Photon Phototherapy Brush: Advised Energy Level: 1 to 4</p> <p>Mode: A/B/C</p> <p>Color can be chosen according to user's specific demands.</p>	<p>+ Cold & Hot Steam + Essence + Facial Mask + Equipment</p>	<p>pores. Cheek can be applied to toner, and face with sensitive skin can be applied to toner and used with cold steam, but it's not advised to use hot steam on and apply export liquid to it.), 5 to 8 minutes.</p> <p>3. Skin Scrubber Technique: Wet the face with cotton pad, then move slowly on the whole face with skin scrubber tilting 45 degrees. Treat the nose first, which can be strengthened. Repeat 3 to 6 times.</p> <p>4. Lift one line after another from chin to the part below ear, from the corner of mouth to the middle of ear, from nose wing to Temple. Repeat 3 to 6 times.</p> <p>5. Do the same on the other side.</p> <p>6. Treat from forehead to hairline. Repeat 3 to 6 times.</p> <p>7. Clean the face, 2 minutes.</p> <p>8. Technique for Active Oxygen Spray Bottle: Pour essence(for whitening, moisturizing, easing, anti-aging, etc.) into oxygen injecting bottle(dilution ratio for essence and distilled water is 2 to 1), 2 minutes.</p> <p>9. Treat one line after another on the whole face, from chin to the part below ear, from the corner of mouth to the middle of ear.</p>	<div data-bbox="1066 190 1359 427" style="text-align: center;">  </div> <p data-bbox="1066 488 1230 521">Technique 6</p> <div data-bbox="1066 539 1359 757" style="text-align: center;">  </div> <p data-bbox="1066 819 1230 853">Technique 9</p> <div data-bbox="1066 871 1359 1088" style="text-align: center;">  </div> <p data-bbox="1066 1151 1246 1184">Technique 11</p> <div data-bbox="1066 1202 1359 1420" style="text-align: center;">  </div>
---	---	--	---

		<p>Rub the treated parts gently with other hand. Repeat 3 times.</p> <p>10. Do the same on the other side.</p> <p>11. Treat from forehead to hairline, and rub the treated parts gently with the other hand. Repeat 3 times.</p> <p>12. Apply facial mask, and wait for 15 minutes.</p> <p>13. Technique for Bio & Photon Phototherapy Brush: Treat the whole face in an orderly way with fixed-point stamp method, from chin to the part below ear, from the corner of mouth to the middle of ear, from nose wing to Temple, from forehead to hairline, 5 to 8 minutes.</p> <p>14. Clean the face, 2 minutes.</p> <p>15. Apply toner, essence, facial cream and sunscreen.</p> <p>16. Treatment done.</p>	
--	--	---	--

Course of Treatment Recommended:
One course consists of ten treatments. After one treatment, skin turns clear and transparent, and pores become refreshing and unobstructed. After one course, clogged pores improve, coarse skin and strawberry-like nose alleviate. After three courses, the effect stabilizes, which contributes to the better absorption of follow-up products, and make skin tender and smooth.

**Skin Peeling with Diamond Dermabrasion: 60 Minutes/
Once a Week**


<p>Suction Level for Diamond Dermabrasion: Operation and operating time can be adjusted in</p>	<p>Makeup Remover + Facial Cleanser + Export Liquid + Base Mask + Cold & Hot</p>	<p>1. Remove makeup and clean the face, 5 minutes.</p> <p>2. Apply export liquid and use hot steam (apply the liquid to parts with large pores or severely clogged pores. Cheek can be applied</p>	<p>Technique 3</p> 
---	--	--	--

<p>line with clients' skin types(Set the suction, then have a trial on lower arm first before operating on face, and ask the client whether he feels comfortable.)</p> <p>Active Oxygen Spray Bottle: It can start to operate when it sprays water mist.</p> <p>Cold Hammer: Advised Energy Level: between 2 and 4</p> <p>Mode: A</p>	<p>Steam + Essence + Facial Mask + Equipment</p>	<p>to toner, and face with sensitive skin can be applied to toner and used with cold steam, but it's not advised to use hot steam on and apply export liquid to it.), 5 to 8 minutes.</p> <p>3. Technique for Diamond Dermabrasion: Appress the device onto forehead and leave it there for 1 second, then gently revolve the device with fingers and bring it up(forehead→ nose→ chin), 5 minutes.</p> <p>4. Start with chin, aligning three points and treating it again after having every two to three points be treated, and move one line after another till Temple, 3 minutes.</p> <p>5. Do the same on the other side.</p> <p>6. Clean the face, 2 minutes.</p> <p>7. Technique for Ozone Spray Bottle: pour essence(for whitening, moisturizing, easing, anti-aging, etc.) into oxygen injecting bottle(dilution rate for essence and distilled water is 2 to 1), 2 minutes.</p> <p>8. Treat one line after another on the whole face, from chin to the part below ear, from the corner of mouth to the middle of ear. Rub the treated parts gently with the other hand. Repeat 2 times.</p> <p>9. Treat from forehead to hairline, and rub the treated</p>	<p>Technique 4</p>  <p>Technique 8, 11</p>  <p>Technique 9, 13</p> 
---	--	--	---

		<p>parts gently with the other hand. Repeat 2 times.</p> <p>10. Apply easing and repairing serum to the whole face, 1 minute.</p> <p>11. Cold Hammer Technique: Move upwards and circlewise from chin to the part below ear, from the corner of mouth to the middle of ear, and from nose wing to Temple. Repeat 5 times.</p> <p>12. Do the same on the other side.</p> <p>13. Treat the forehead with equipment moving transversely and circlewise. Repeat 3 times.</p> <p>14. Apply facial mask, and wait for 15 minutes.</p> <p>15. Clean the face, 2 minutes.</p> <p>16. Apply toner, essence, facial cream and sunscreen.</p> <p>17. Treatment done.</p>	
--	--	--	--

Course of Treatment Recommended:
 One course consists of ten treatments. After one treatment, skin becomes clean, flat and unrough. After a course, skin gets smooth, delicate and glossy. After two courses, skin turns tightened, bright and elastic. After three courses, an elastic, shiny and ruddy skin can be maintained

Easing & Repairing Management: 60 Minutes/ Once a Week





<p>Cold Hammer: Advised Energy Level: between 2 and 4 Mode: A Active Oxygen</p>	<p>Cleansing Oil + Facial Cleanser + Base Mask + Essence + Cold Steam + Cold Hammer + Repairing Facial Mask</p>	<p>1. Remove makeup and clean the face, 5 minutes.</p> <p>2. Apply hydrating mask and use cold steam, 10 minutes.</p> <p>3. Technique for Active Oxygen Spray Bottle: Pour essence(for moisturizing and easing) into oxygen</p>	<p>Technique 4</p> 
		<p>Technique 5</p>	


<p>Spray Bottle: It can start to operate when it sprays water mist.</p> <p>Bio & Photon Phototherapy Brush: Advised Energy Level: 1 to 4</p> <p>Mode: A/B/C</p> <p>Color can be chosen according to user's specific demands.</p>		<p>injecting bottle(dilution ratio for essence and distilled water is 2 to 1), 2 minutes.</p> <p>4. Treat one line after another on the whole face, from chin to the part below ear, from the corner of mouth to the middle of ear. Rub the treated parts gently with the other hand. Repeat 2 times.</p> <p>5. Treat from forehead to hairline, and rub the treated parts gently with the other hand. Repeat 2 times.</p> <p>6. Cold Hammer Technique: Lift upwards and circlewise from chin. Repeat 3 to 5 times.</p> <p>7. Do the same on the other side.</p> <p>8. Treat the forehead with equipment moving transversely and circlewise. Repeat 3 to 5 times.</p> <p>9. Apply facial mask, and wait for 15 minutes.</p> <p>10.Technique for Bio Photon Phototherapy Brush: Treat the whole face in an orderly way with fixed-point stamp method, from chin to the part below ear, from the corner of mouth to the middle of ear, from nose wing to Temple, from forehead to hairline, 5 to 8 minutes.</p> <p>11.Clean the face, 2 minutes.</p> <p>12.Apply toner, essence, facial cream and sunscreen.</p> <p>13.Treatment done.</p>	<div data-bbox="1155 215 1283 416" data-label="Image"> </div> <p data-bbox="1070 488 1230 521">Technique 6</p> <div data-bbox="1155 551 1283 752" data-label="Image"> </div> <p data-bbox="1070 819 1230 853">Technique 8</p> <div data-bbox="1155 882 1283 1084" data-label="Image"> </div>
--	--	--	--

Course of Treatment Recommended:

One course consists of ten treatments. After one treatment, skin becomes moisturized, plump and smooth. After a course, skin back to normal, which will not change due to season change, makeup contacting and allergy caused by skin hypopenia. After two courses, skin turns smooth, flexible and white touched with red, and its blood circulation improves, and it feels like velvet.

**Facial Rejuvenation & Balance Management: 60 Minutes/
Once a Week**

<p>Suction Level for Diamond Dermabrasion: Operation and operating time can be adjusted in line with clients' skin types(Set the suction, then have a trial on lower arm first before operating on face, and ask the client whether he feels comfortable.)</p> <p>Skin Scrubber: Mode: 1/2/3</p> <p>Active Oxygen Spray Bottle: It can start to operate when it sprays water mist.</p> <p>Bio & Photon Phototherapy Brush:</p>	<p>Makeup Remover + Facial Cleanser + Export Liquid + Base Mask + Cold & Hot Steam + Essence + Facial Mask + Equipment</p>	<ol style="list-style-type: none"> 1. Remove makeup and clean the face, 5 minutes. 2. Apply export liquid and use hot steam(apply the liquid to parts with large pores or severely clogged pores. Cheek can be applied to toner, and face with sensitive skin can be applied to toner and used with cold steam, but it's not advised to use hot steam on and apply export liquid to it.), 5 to 8 minutes. 3. Technique for Diamond Dermabrasion: Appress the device onto equipment leave it there for 1 second, then gently revolve the device with fingers and bring it up(forehead→ nose→ chin), 5 minutes. 4. Start with chin, aligning three points and treating it again after having every two to three points be treated, and move one line after another till Temple, 3 minutes. 5. Do the operating same on the other side. 6. Clean the face, 2 minutes. 7. Skin Scrubber 	<p>Technique 3</p>  <hr/> <p>Technique 4</p>  <hr/> <p>Technique 7</p>  <hr/> <p>Technique 8, 13</p>  <hr/> <p>Technique 10, 14</p>
--	--	--	---



<p>Advised Energy Level: 1 to 4</p> <p>Mode: A/B/C</p> <p>Color can be chosen according to user's specific demands.</p>		<p>Technique: Wet the face with cotton pad, then move slowly on the whole face with skin scrubber tilting 45 degrees. Treat the nose first, which can be strengthened. Repeat 3 to 6 times.</p> <p>8. Lift one line after another from chin to the part below ear, from the corner of mouth to the middle of ear, from nose wing to Temple. Repeat 3 to 6 times.</p> <p>9. Do the same on the other side.</p> <p>10. Treat from forehead to hairline. Repeat 3 to 6 times.</p> <p>11. Clean the face, 2 minutes.</p> <p>12. Technique for Active Oxygen Spray Bottle: Pour essence(for whitening, moisturizing, easing, anti-aging, etc.) into oxygen injecting bottle(dilution ratio for essence and distilled water is 2 to 1), 2 minutes.</p> <p>13. Treat one line after another on the whole face, from chin to the part below ear, from the corner of mouth to the middle of ear. Rub the treated parts gently with the other hand. Repeat 2 times.</p> <p>14. Treat from forehead to hairline, and rub the treated parts gently with the other hand. Repeat 2 times.</p> <p>15. Clean the face, 2 minutes.</p>	
---	--	---	---

		<p>16. Apply facial mask, and wait for 15 minutes.</p> <p>17. Technique for Bio & Photon Phototherapy Brush: Treat the whole face in an orderly way with fixed-point stamp method, from chin to the part below ear, from the corner of mouth to the middle of ear, from nose wing to Temple, from forehead to hairline, 5 to 8 minutes.</p> <p>18. Remove the mask and clean the face, 2 minutes.</p> <p>19. Apply toner, essence, facial cream and sunscreen.</p> <p>20. Treatment done.</p>	
--	--	--	--

Course of Treatment Recommended:

One course consists of ten treatments. After one treatment, skin turns clean and bright. After one course, skin becomes moisturized and plump, grease secreta decreases, water and oil back to normal gradually. After two courses, skin gets delicate, elastic and shiny.

Ceramic Skin Restoring: 90 Minutes/ Once a Week

<p>Suction Level for Diamond Dermabrasion: Operation and operating time can be adjusted in line with clients' skin types(Set the suction, then have a trial on lower arm first before operating on face, and ask the client whether he feels</p>	<p>Makeup Remover + Facial Cleanser + Export Liquid + Base Mask + Cold & Hot Steam + Essence + Facial Mask + Instrument</p>	<p>1. Remove makeup and clean the face, 5 minutes.</p> <p>2. Apply export liquid and use hot steam(apply the liquid to parts with large pores or severely clogged pores. Cheek can be applied to toner, and face with sensitive skin can be applied to toner and used with cold steam, but it's not advised to use hot steam on and apply export liquid to it.), 5 to 8 minutes.</p> <p>3. Technique for Diamond Dermabrasion: Appress the device onto equipment leave it there for 1 second, then gently revolve the</p>	<p>Technique 3</p> 
		<p>Technique 4</p> 	
		<p>Technique 7</p>	

comfortable.)

device with fingers and bring it up(forehead→ nose→ chin), 5 minutes.

4. Treat the cheek. Start with chin, aligning three points and treating it again after having every two to three points be treated, and move one line after another till Temple, 3 minutes.

5. Do the same on the other side.

6. Clean the face, 2 minutes.

7. Skin Scrubber

Technique: Wet the face with cotton pad, then move slowly on the whole face with skin scrubber tilting 45 degrees. Treat the nose first, which can be strengthened. Repeat 3 to 6 times.

8. Lift one line after another from chin to the part below ear, from the corner of mouth to the middle of ear, from nose wing to Temple. Repeat 3 to 6 times.

9. Do the same on the other side.

10. Treat from forehead to hairline. Repeat 3 to 6 times.

11. Technique for Active Oxygen Spray Bottle: Pour essence(for whitening, moisturizing, easing, anti-aging, etc.) into oxygen injecting bottle(dilution ratio for essence and distilled water is 2 to 1), 2 minutes.

12. Treat one line after



Technique 8



Technique 10



Technique 15



Technique 17



		<p>another on the whole face, from chin to the part below ear, from the corner of mouth to the middle of ear. Rub the treated parts gently with the other hand. Repeat 2 times.</p> <p>13. Treat from forehead to hairline, and rub the treated parts gently with the other hand. Repeat 2 times.</p> <p>14. Apply essence(for whitening, moisturizing, anti-aging and easing) evenly to the face, 1 minute.</p> <p>15. Cold Hammer Technique: Move circlewise and upwards from chin to to the part below ear, from the corner of mouth to the middle of ear, and from nose wing to Temple. Repeat 3 times.</p> <p>16. Do the same on the other side.</p> <p>17. Treat the forehead with equipment moving transversely and circlewise. Repeat , 3 to 5 times.</p> <p>18. Clean the face, 2 minutes.</p> <p>19. Apply facial mask, and wait for 15 minutes.</p> <p>20. Technique for Bio & Photon Phototherapy Brush: Treat the whole face in an orderly way with fixed-point stamp method, from chin to the part below ear, from the corner of mouth to the middle of ear, from nose wing to Temple, from forehead to hairline, 5 to 8 minutes.</p>	
--	--	--	--

		21. Remove the mask and clean the face, 2 minutes. 22. Apply toner, essence, facial cream and sunscreen. 23. Treatment done.	
--	--	--	--

Course of Treatment Recommended:

One course consists of ten treatments. After one treatment, skin turns refreshing obviously and pores improve. After one course, skin becomes moisturized and plump, and skin color gets even and bright. After two courses, skin becomes clean and refreshing, skin color turns transparent, ruddy and smooth. After three courses, skin's clearness, rejuvenation and smoothness can be maintained if you stick to this treatment.