

User Manual

SR-MJ1806



Preface

Dear users:

We're pleased to present to you our 6-in-1 beauty machine that mainly work on cleansing, soothing, hydrating, skin brightening and so on. It is a multifunctional beauty machine, targeted at overall facial skin care issues such as dry skin, clogged pores, acne skin and uneven skin tone. We aim at a safe and, at the same time, effective beauty result by using high-end technologies. SR-MJ1806 beauty machine is facial skin care management machine for professional use, which needs to be handled by professionals with proper training. Any improper use will result in adverse consequences. Therefore we advise any personnel read this guide thoroughly and follow the instructions strictly before operation.

We believe our quality product will bring in good returns on your end. Also, you can count on our worry-free after-sales service.

Thank you!

Table of Contents

Preface
Part I
Brief Introduction
Advantages
Working Principles & Treatments
Deep Cleaning
Soothing&Calming
Acne Treatment
Part II
1. Detailed Operations
2. Technical Parameters
3. Safety Precautions
4. Dos and Don'ts
5. Troubleshooting & Solutions
6.FAQs
7.Packing List
8.Operational Diagrams

Part I

Brief Introduction

Microdermabrasion, facial skin scrubber and high frequency acne treatments are currently the most popular beauty machines that work on deep cleaning. It's an effective replacement of all the traditional facial cleaning and facial care programs in beauty salons. Moreover, they are convenient and easy to operate. It solves various skin issues for beauty-lovers. This equipment requires no injection, no medication, no operation, thus having no side-effects. It is used externally during the whole process and has an immediate effect. Microermabrasion adopts vacuum negative pressure suction technology to reduce internal air pressure, thus effectively removing wasted cuticle layer, sucking out blackheads and excess oil and other pore garbage, in a way achieving free breathing circulation of skin, which can dredge pores, improve pores and brighten skin tone. At the same time, the combination of bio&photon treatment can also increase cell functions and vitality, remove free radicals, making the skin look youthful.

Advantages

1. 6-in-1 beauty machine that works on your overall facial care.
2. Multiple working probes can be replaced in different parts according to different requirements.
3. Special probes of microdermabrasion can improve varies of skin problems and revive a clean face.
4. Suit all skin type; help improve skin quality and fine lines.
5. Painless, non-invasive during the process. Skipping any recovery periods without affecting any work and life plans.
6. No consumption, low cost and quick returns.
7. Wider treatment area and faster and more visible results.
8. Unevenness, bleed, swelling and stasis will not appear after treatment.
9. Bio&Photon treatment can increase cell function, cell vitality and skin rejuvenation.
10. The 3MHZ high-frequency ultrasonic wave of cleaning scrubber can effectively enhance the absorption of the skin, help the nutrition penetrate into the dermis, and comprehensively renew the skin. The high-frequency vibration refines the nutrients into small molecular particles and enhances the absorption.
11. High frequency acne treatment deals with your troublesome acne.

Deep Cleaning

Principles

Facial Cleaning Scrubber

Facial cleaning scrubber is equipped with the unique technology of transforming. 2.8MHz mechanical vibration energy emitted has obvious osmosis effect on human skin. Through the medium of atomization, strong impact can instantly remove deep dirt from the pores, painlessly remove acne and pimples so that skin receive a thorough clean treatment. Low-frequency energy can emulsify sebum and secretions that are blocking pores, which can also sterilize. Mechanical vibration can not only atomizes water molecules into pores, but also can remove excess thick cutin, purify pores, decompose oil, leaving them unobstructed and tighten pores, clear and smooth skin.

Physical Effects

High-frequency vibration sound wave is combined with softening product or exfoliating cream to soften the damaged rough cuticle layer, and open pores with sound wave, and remove the garbage and waste deep in pores by high-frequency vibration.

On the back of the instrument high frequency vibration combined with essence product can induce the product into the skin, increasing skin elasticity and smoothing wrinkles.

Microdermabrasion

The skin is cleaned and tightened twice through microdermabrasion treatment that is equipped with crystal tips and wands that area specially processed for your skin to vacuum out dead skin cells, remove dirt and waste in deep pores, and make the skin doubly clean and regenerate new skin cells. Use negative pressure function to remove demodex, acarid, blackhead, whitehead, and meanwhile dredge the blocked pores.

Biological Effect

Suction generated through negative pressure, combined with the scrub head can dredge out the metabolites at the surface and blackheads, clogged gunks at root effectively. It is in a way the vacuum cleaner of skin that aims for a thorough cleansing.

Hydro Spray Jet

The product and oxygen are simultaneously injected into the air pressure equipment with the help of strong pressurized oxygen, and the granular water droplets are transformed into extremely small molecular weight to act on the skin. The tiny water droplets will be sprayed out with a specially designed nozzle at a speed of 200M/sec,

and the water and oxygen will be injected into the bottom layer of human skin at an extremely high speed.

Biological Effects

Vacuum loop is formed through vacuum negative pressure, high concentration reactive oxygen species and essence are fully combined, and the skin is directly affected by high-pressure release, so as to atomize nutrients molecules, promote the removal of pore waste, not only deeply cleanse the face, but also provide lasting essence nutrition for the skin, making the skin healthy, moist, delicate and shiny.

BIO&Photon Treatment

It uses the photodynamic principle, activates the deep cell, lets the skin carry on the metabolism better. LED colour light can penetrate the skin layer, dilate blood vessels, stimulate blood circulation, accelerate lymph circulation, enhance the role of local tissue blood oxygen saturation. LED light coupled with BIO can increase cell function, cell vitality, remove free radicals, promotes the fiber cell and endothelial cell proliferation to achieve the effect of repair skin and tender skin and eventually younger skin.

Biological Effect

Using photodynamic principle, it activates deep cells and boosts skin metabolism. After the light is absorbed by the skin, the light energy is converted into cellular energy, which can relax and strengthen the micro vessels and produce photochemical reactions with the skin -- enzymatic reactions, increasing the content of glycogen and protein, thus promoting the metabolism and synthesis of cells.

Effects

1. Atomize water into pores and bring out dirt through high-frequency vibration.
2. Refine nutrients into small molecular particles by high-frequency vibration, enhance skin absorption, and direct nutrients to the dermis.
3. Improve skin quality, coarse pores, wrinkles, thick cutin, purify skin, remove extra thick cutin, purify pores, decompose oil, tighten pores, clear, smooth and brighten skin, etc.
4. Effectively excrete the metabolites at the skin surface and the blackheads, garbage and clogs in the depths of pores, which is equivalent to the vacuum cleaner of the skin and gives the skin a thorough cleaning.

Applicable Range

1. Those with dry skin.
2. Those with large pores, rough and dull skin.

3. Those with blackheads, whiteheads, clogged pores, and various forms of acnes.
4. Those with poor skin absorption and lack of water in the skin base.
5. Those with dark and uneven skin tone.
6. Those with excessive oil secretion, oily and dry skin.
7. Those whose skin is not smooth due to improper cleaning.

Inapplicable Range

1. Those with cardiovascular and cerebrovascular diseases, and severe three H's(hypertension, hyperlipidemia, hyperglycemia).
2. Those who are suffering from serious allergy.
3. Those with skin diseases and infectious diseases.
4. Those with metal implants inside body (such as pacemaker, steel, and metal teeth, etc).
5. Those with skin damage, inflammation or lesion on parts of the body.
6. Those whose skin are having a allergic reaction.
7. Those who are in pregnancy and menstruation.
8. Those with skin wounds or cuts.

Notes after Operation

1. Use sunscreen and avoid sun exposure right after the treatment.
2. Avoid touching, scratching, or other hand contact with the treated area.
3. Apply face mask to hydrate skin, accelerate metabolism and prevent dry skin.
4. Don't change skin care products too often.
5. Make sure to remove makeups before bed.
6. Avoid high temperature.

Soothing&Calming

Principles

Hydro Spray Jet

The product and oxygen are simultaneously injected into the air pressure equipment with the help of strong pressurized oxygen, and the granular water droplets are transformed into extremely small molecular weight to act on the skin. The tiny water droplets will be sprayed out with a specially designed nozzle at a speed of 200M/sec, and the water and oxygen will be injected into the bottom layer of human skin at an extremely high speed.

Biological Effects

Vacuum loop is formed through vacuum negative pressure, high concentration reactive oxygen species and essence are fully combined, and the skin is directly affected by high-pressure release, so as to atomize nutrients molecules, promote the removal of pore waste, not only deeply cleanse the face, but also provide lasting essence nutrition for the skin, making the skin healthy, moist, delicate and shiny.

Cold Hammer

Use the principle of air conditioner. Special wafer ceramics are used to absorb energy and refrigerate at - 5 degrees in an instant. The method of physical heat accumulation produced by wafer energy absorption makes one side produce heat up to 40 degrees. The fan discharges excess heat energy and achieves the effect of constant temperature and freezing alternately. It also promotes metabolism, alleviates allergic reaction, shrinks pore, calms skin, whitens and tenders skin.

Biological effects

Cold and hot hammer has respectively cooling and heating effects on skin. During the cooling process, tissue temperature cools down, blood vessels and veins contract, metabolic rate reduces, cell activity inhibits, peripheral nerve sensitivity reduces, thus having a sedative and soothing effect, counteracting or reducing the symptoms of allergic reaction, such as redness, swelling, and pain, therefore achieving certain effects, such as calming the skin, reducing redness and swelling, shrinking pores and so on. During the heating process, it heats up tissues by raising the temperature, thus adjusting blood circulation, boosting metabolism and improve the absorption of skin care products.

BIO&Photon Treatment

LED photon therapy uses the principle of photodynamics to activate deep cells and boosts skin metabolism. After light is absorbed by the skin, the light energy is transformed into intracellular energy, which expands and strengthens microvessels and produces photochemical reaction-enzymatic reaction with the skin, increasing the activity of Catalase and superoxide dismutase (SOD), which serve as the "guard" and "cleaning worker" of cells. The breakdown of adenosine triphosphate (ATP), a source of cellular energy, increases. Increase glycogen and protein content, thereby promoting cell metabolism and synthesis. To stimulate the body alkaline synthetic fiber growth factor (BFGF) the secretion of epidermal growth factor (EGF), making collagen fibers and increase elasticity, rearranging fibers, blocking and eliminating the formation of melanin, accelerating cell growth and blood circulation, stimulating fiber cells to produce collagen, increasing skin elasticity, repairing the aging skin, acne skin, fade spots, firm and alleviate the sun burns. It plays an important role in skin care such

as killing bacteria to provide a suitable environment for new cells.

Biological Effect

Using photodynamic principle, it activates deep cells and boosts skin metabolism. After the light is absorbed by the skin, the light energy is converted into cellular energy, which can relax and strengthen the micro vessels and produce photochemical reactions with the skin -- enzymatic reactions, increasing the content of glycogen and protein, thus promoting the metabolism and synthesis of cells.

Effects

1. Improve redness, swelling, acne and allergies, etc.
2. Calm and sooth skin, and help prevent breakout.
3. Refine pores, and shrink enlarged pores.
4. Increase skin elasticity.
5. Increase skin resistance, and repair fragile skin.

Applicable Range

1. Those with sensitive skin, acne-prone skin, and easily inflamed, or allergic skin.
2. Those whose skin is flabby, saggy and inelastic.
3. Those with dry skin with fine lines, lack of moisture and uneven, dull complexion.
4. Those with large, coarse pores, and vigorous oil secretion.

Inapplicable Range

1. Those with severely sensitive skin or those in allergic phase.
2. Those whose nose has prosthesis.
3. Those with extensive facial trauma.
4. Those who are pregnant and those suffering from serious heart disease, severe three H's(hypertension, hyperlipidemia, hyperglycemia), serious diseases, etc.
5. Those with skin diseases and infectious diseases.
6. Those who have just received injection, such as hyaluronic acid, intradermal injection, wrinkle removal, or plastic surgery.
7. Those with metal implants in the body, such as stents and pacemakers, or are allergic to metal.

Notes after Operation

1. Wash your face with warm water within 3 days after operation.

2. Strengthen hydration and pay attention to sun protection.
3. Do not use stimulating skin care products including scrub, exfoliating and AHAs within one week after operation.
4. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.
5. Apply a hydrating mask once a day for three days.

Acne Treatment

Principles

Cleaning Scrubber

Facial cleaning scrubber is equipped with the unique technology of transforming. 2.8MHz mechanical vibration energy emitted has obvious osmosis effect on human skin. Through the medium of atomization, strong impact can instantly remove deep dirt from the pores, painlessly remove acne and pimples so that skin receive a thorough clean treatment. Low-frequency energy can emulsify sebum and secretions that are blocking pores, which can also sterilize. Mechanical vibration can not only atomizes water molecules into pores, but also can remove excess thick cutin, purify pores, decompose oil, leaving them unobstructed and tighten pores, clear and smooth skin.

Physical Effects

High-frequency vibration sound wave is combined with softening product or exfoliating cream to soften the damaged rough cuticle layer, and open pores with sound wave, and remove the garbage and waste deep in pores by high-frequency vibration.

On the back of the instrument high frequency vibration combined with essence product can induce the product into the skin, increasing skin elasticity and smoothing wrinkles.

Hydro Spray Jet

The product and oxygen are simultaneously injected into the air pressure equipment with the help of strong pressurized oxygen, and the granular water droplets are transformed into extremely small molecular weight to act on the skin. The tiny water droplets will be sprayed out with a specially designed nozzle at a speed of 200M/sec, and the water and oxygen will be injected into the bottom layer of human skin at an extremely high speed.

Biological Effects

Vacuum loop is formed through vacuum negative pressure, high concentration reactive oxygen species and essence are fully combined, and the skin is directly affected by high-pressure release, so as to atomize nutrients molecules, promote the removal of pore waste, not only deeply cleanse the face, but also provide lasting essence nutrition for the skin, making the skin healthy, moist, delicate and shiny.

High Frequency Acne Treatment

High frequency electrotherapy instrument is an instrument that uses the vibration of high frequency electric current to generate thermal effect, ultraviolet radiation and electric current to treat skin. It has the function of promoting absorption, sterilization, anti-inflammation and accelerating wound healing. It is suitable for external treatment of acne skin and wound skin. It's application in beauty industry has long been aware of.

It is mainly composed of high frequency oscillation circuit, capacitive resistance and semiconductor devices. In addition, it is equipped with an insulated handle and a glass electric rod that can be inserted into the electrode. There is a booster transformer in the handle, and on one side there's wire for output circuit. When connected and switched on, under the action of high frequency oscillation circuit boards, it produces the continuous high voltage high frequency current, making the glass electrode discharge ultraviolet light. Skin under the current stimulus, partial peripheral blood vessels will appear alternately contracting and expanding reaction, electric spark induces ionization of oxygen in the air and produces ozone, thus achieving effects like improving blood circulation and bactericidal anti-inflammatory.

Biological Effects

The applied current generated by the high frequency electrotherapy instrument can also change the cell membrane function of human cells, its ion channel, the receptor status on the membrane, and make the inflammatory medium unable to bind to the receptor, and make the capillary terminal shrink, thus inhibiting the occurrence and development of inflammation.

BIO&Photon Treatment

LED photon therapy uses the principle of photodynamics to activate deep cells and boosts skin metabolism. After light is absorbed by the skin, the light energy is transformed into intracellular energy, which expands and strengthens microvessels and produces photochemical reaction-enzymatic reaction with the skin, increasing the activity of Catalase and superoxide dismutase (SOD), which serve as the "guard" and "cleaning worker" of cells. The breakdown of adenosine triphosphate (ATP), a source of cellular energy, increases. Increase glycogen and protein content, thereby

promoting cell metabolism and synthesis. To stimulate the body alkaline synthetic fiber growth factor (BFGF) the secretion of epidermal growth factor (EGF), making collagen fibers and increase elasticity, rearranging fibers, blocking and eliminating the formation of melanin, accelerating cell growth and blood circulation, stimulating fiber cells to produce collagen, increasing skin elasticity, repairing the aging skin, acne skin, fade spots, firm and alleviate the sun burns. It plays an important role in skin care such as killing bacteria to provide a suitable environment for new cells.

Biological Effect

Using photodynamic principle, it activates deep cells and boosts skin metabolism. After the light is absorbed by the skin, the light energy is converted into cellular energy, which can relax and strengthen the micro vessels and produce photochemical reactions with the skin -- enzymatic reactions, increasing the content of glycogen and protein, thus promoting the metabolism and synthesis of cells.

Cold Hammer

Use the principle of air conditioner. Special wafer ceramics are used to absorb energy and refrigerate at - 5 degrees in an instant. The method of physical heat accumulation produced by wafer energy absorption makes one side produce heat up to 40 degrees. The fan discharges excess heat energy and achieves the effect of constant temperature and freezing alternately. It also promotes metabolism, alleviates allergic reaction, shrinks pore, calms skin, whitens and tenders skin.

Biological effects

Cold and hot hammer has respectively cooling and heating effects on skin. During the cooling process, tissue temperature cools down, blood vessels and veins contract, metabolic rate reduces, cell activity inhibits, peripheral nerve sensitivity reduces, thus having a sedative and soothing effect, counteracting or reducing the symptoms of allergic reaction, such as redness, swelling, and pain, therefore achieving certain effects, such as calming the skin, reducing redness and swelling, shrinking pores and so on. During the heating process, it heats up tissues by raising the temperature, thus adjusting blood circulation, boosting metabolism and improve the absorption of skin care products.

Effects

1. Clean skin and remove excessive cutin and blocked pores.
2. Calm skin, relieve skin redness, swelling, heat, pain, itching and other symptoms.
3. Tighten pores and lock in moisture.
4. Promote circulation and metabolism, and accelerate product absorption.

5. Quickly heal wounds and promote acne repair.
6. Strong hydration to prevent water loss.
7. Increase elasticity and smooth fine lines.
8. Reduce acne outbreak rate and inhibit acne growth.
9. Anti-inflammatory sterilization, reduce the chance of acne breeding.
10. Thoroughly clean the garbage of skin to prevent blockage and acne growth.
11. Relieve acne marks and pigmentation.

Applicable Range

1. Those with oily skin, acne-prone skin.
2. Those with large, coarse pores, and uneven skin.
3. Those with hyperpigmentation and uneven skin tone.
4. Those with blackheads, thick cutin and clogged pores.
5. Those with fine lines, and dull complexion.
6. Those recurrent acne condition.
7. Those with sensitive skin.

Inapplicable Range

1. Those with allergic facial condition or cuts.
2. Those with severely sensitive skin.
3. Those who are in pregnancy and surgery recovery.
4. Those with pustule or severe acne conditions.
5. Those who have just undergone plastic surgery.
6. Those with cardiovascular and cerebrovascular diseases, and severe three H's(hypertension, hyperlipidemia, hyperglycemia).
7. Those with skin diseases and infectious diseases.
8. Those with metal implants inside body (such as pacemaker, steel, and metal teeth, etc).
9. Those with skin damage, inflammation or lesion on parts of the body.
10. Those whose skin is having an allergic reaction.
11. Those who are in pregnancy and menstruation.

Notes after Operation

1. Use sunscreen and avoid sun exposure right after the treatment.
2. Avoid touching, scratching, or other hand contact with the treated area.
3. Apply face mask to hydrate skin, accelerate metabolism and prevent dry skin.
4. Don't change skin care products too often.
5. Make sure to remove makeups before bed.
6. Avoid high temperature.

7. Avoid using products containing alcohol and hormone.
8. Avoid staying up late, smoking and drinking, and eat more fruits and vegetables and less greasy food.

Part II

1. Detailed Operation

After checking that the instrument is connected properly, the following starting interface will appear when the power supply is turned on.



1.1 Function Selection



Start/pause



Time Adjustment



Time Display



Decrease of Time



Increase of Time



Model Selection



Energy Adjustment (Recommended setting for beginner is 1-2, and add up slowly afterwards.)



Decrease of Energy



Increase of Energy



Exhale hole (Insert tube to the exhale hole when operating hydro spray jet.)



Inhale hole (Insert tube to the inhale hole when operating microdermabrasion.)



Suction adjustment knob (Turn rightward to turn up the suction pressure, and turn leftward to turn down the suction pressure.)

1.2 Detailed Operation: Hot&Cold Hammer



Click  to go to the interface below



Hot&Cold Hammer Model Adjustment (Model A is cold hammer, mainly for clam, repair and minimize pores; Model B is hot hammer, mainly for opening pores and lead in nutrition.)

1.3 Detailed Operation: Bio&Photon



Click to go to the interface below



Bio Model Adjustment (Model A: intermittent energy release, suitable for beginners; model B: accelerated intermittent energy release, suitable for those who have slightly darkened skin; model C: continuous energy release, suitable for acne, rough skin.)

Photon Model Adjustment:



Start/Off



Auto alteration of color (cycle through multiple twinkling colors)



Mode selection (different colors can be chosen to stay on)



Wave: mono twinkling color (click on MODE to change to different color when needed.)

1.4 Detailed Operation: Dermabrasion



Click to go to the interface below



1.5 Detailed Operation: Cleaning Scrubber



Click the round button in the middle to turn on the device.

Click once: continuous vibration (for those with large, clogged pores and rough skin)

Click twice: accelerated intermittent vibration (for those with clogged pores and excessive oil secretion)

Click thrice: intermittent vibration (for those with rough skin and need exfoliation)

2. Technical Parameters

Water Spray Jet

Cup capacity: 7CC

Nozzle diameter: 0.3mm

Cold Hammer Handlepiece

Treatment Area: Unlimited

Measure of Treatment Area: 6cm²

Treatment Area: Air cooled

Maximum temperature: 43°C

lowest temperature: -5°C

Microcurrent+Photon Handlepiece

Treatment Area: Unlimited

Measure of Treatment Area: 27cm²

Depth of Pressure: 17 RGB tri-color LED lights

Microcurrent frequency: 62.5KHz

Max. Output Power: 15W

3. Safety Precautions

1. Those who are pregnant or lactating.
2. Those with heart disease or pacemaker equipped.
3. Those whose wounds have not healed and who are recovering from the operation.
4. Those with epilepsy, severe diabetes and hyperthyroidism.
5. Those with malignant tumor, hemophilia or severe bleeding.
6. Those with skin diseases and infectious diseases.

4. Dos and Dont's

1. For sensitive skin and tender skin, please cover the skin surface with gauze before care, so that the current can penetrate into the skin and avoid the stimulation of the current to the skin.
2. Do not leave the glass electrode tube in the same place for more than 3 seconds.
3. This device should not be used by pregnant women or patients with implanted pacemakers.
4. Be sure to attach the glass electrode tube to the handle before starting the instrument. Turn off the power after using the instrument.
5. During high frequency operation, the energy level of current should be gradually adapted from low to high. The operating head should not leave the customer's skin at all times and move gradually while inserting right amount of strength. It is advised not to stay at one place for too long to avoid burning.
6. When installing or replacing the glass electrode tube, turn off the energy adjusting knob or power supply. All kinds of glass tubes should be kept clean and dry.
7. When nursing, please take off any metal jewelry. Those with metal implants are not advised to receive the treatment.
8. Check whether the socket and plug are properly inserted and whether the operation head is tightened before operation.
9. Special odor may be emitted when using, which is a normal phenomenon.
10. For severe acne such as cyst or nodular, apply liquid iodine to the affected area after needle treatment and then carry out the operation.
11. Do not use alcohol-based skin care products as alcohol is highly flammable.
12. Freckle and dark spot skin cannot use this product, otherwise it will aggravate the freckle.
13. The lower part of the handle can be rotated to adjust the current.

14. Use sunscreen and do not touch the contact area or acne with your hands.
15. Do not use for those who are recovering from surgery and recovering from wounds on the body
16. Do not use for those with electronic implants (e.g. pacemaker metal implants).
17. Do not use for those with facial allergy and severe allergy.
18. Do not use for those with facial fillers or have undergone plastic surgery.
19. Spray bottle mainly uses water agent, essence, original liquid, normal saline, toner, etc. If it is viscous liquid, you need to dilute before spraying, otherwise the spray gun will block. Use distilled water or purified water, or mineral water diluted into water spray.
20. After use, inject clean water and spray for one round to clean the internal pipe. Empty the water and then air dry the inner pipe to avoid bacteria.
21. Care should be taken to avoid the spray inlet and nostril during bottle care.
22. Do not insert the catheter into the blackhead suction hole when using the spray bottle, in case that the solution pours into the air pump and damages the main machine.
23. The right jack of the instrument is the vent hole and the left jack is the suction hole.
24. Use sunscreen after treatment.
25. Spasm may occur if energy level is too high. Cease treatment immediately if it happens.
26. Avoid touching the air outlet when using the anti-freezing head to avoid being scalded.
27. Check whether the shim is installed when using the microdermabrasion head, otherwise it will leak air or suction is not enough.
28. The face must be wet when using the cleaning scrubber. Avoid dry skin operation.
29. When suctioning blackheads, it is better to put proper amount of cotton paper in the glass tube to prevent dirt inhalation into the air pipe and damage the host. Don't spend too much time on your face to avoid bruising.
30. If there is no suction when starting up, it is a normal phenomenon, and it can be restored to normal after a short while. The metal knob is for adjusting suction. Twist clockwise to turn up.
31. When operating the facial cleaning scrubber, the groove is outwards and the water droplets will be seen splashing around in the form of water mist. Hold the cleaning scrubber upside down and apply the flat surface to the skin for nutrients absorption and skin lifting.

5. Troubleshooting & Solutions

1. **The instrument can't start and the key lamp on the back of the instrument doesn't work?**
 - A. Ensure that the power cord is connected to a working power outlet.

B. Ensure the fuse tube on the back of the instrument is not loose or being burnt.

2. No pressure or weakened pressure?

A. Please turn off the instrument and check whether the rubber hose of the instrument is inserted tightly.

B. Please check whether the handle cover is capped tight and whether the product is watery.

C. Please check whether the rubber hose is damaged or leakage leads to poor air pressure.

D. If the above methods cannot address the issue, please contact the dealer of the instrument for assistance.

3. I can start the device, but the screen shows error message?

A. Please unplug the power plug behind the instrument and wait for about 1 minute before restarting the instrument.

B. If the above methods cannot resolve the issue, please contact the instrument dealer for assistance.

6. FAQs

1.Q: Will the skin become thinner and more sensitive after microdermabrasion treatment?

A: No. During the process, microdermabrasion operation is just to remove the skin aging cuticle. Our skin regenerates every 28 days, so it will only make the skin become more smooth, fine and glowing. Sensitive symptoms will not appear.

2.Q: Why exfoliate?

A: The cuticle is the layer above the epidermis where the cells are completely dead. Depending on your body, the cuticle consists of 15 to 20 layers of dead cells. Human skin has a metabolic cycle, which is normally 28 days. It is the metabolic cycle of people in puberty. For adults, the metabolic cycle should be the actual age +10. But due to various reasons, dead corneous cells may not fall off in time and bring about thick aging cutins. On the other hand, it also affects the color of skin, and the skin metabolism, so other method needs to be utilized when normal metabolic process won't do the job by itself.

3.Q: Can I wear makeup after using this instrument?

A: Absolutely, the skin after treatment is moist and plump, makeup will actually last longer. This is a safe, healthy, quick and effective way of nursing without puncturing

skin or leaving scars.

4.Q: How long will it take to see results?

A: After one treatment obvious results are shown, such as clean and translucent skin. It works even better on a continuous basis, improving skin problems such as dehydrating, darkening, fine lines and aging process in the deeper layers of your skin.

5.Q: Is it dependent?

A: No, this is for skin problems to improve and deeply clean the skin, which belongs to the maintenance and improving skin of malnutrition caused by other factors. It's used to help skin absorb nutrition products, solve other issues such as dark, yellowish skin, water shortage, rough skin, wrinkled skin. Without the treatment you skin will only go back to secretion of its own and aging, therefore not dependence factor is involved.

7. Packing List

- 1 x Main Machine
- 1 x Photon Micro current Handle
- 1 x Sprayer Gun
- 1 x Cold Handle
- 1 x Ultrasonic Skin Scrubber
- 1 x High Frequency Handle
- 4 x Glass Tube
- 1 x Diamond Wands
- 9 x Diamond Tips
- 1 x Power Supply Cord

8.Operational Diagram

Parameter adjustment	Product	Technique	Diagram
Deep Cleaning: 60 min, once/week			
<p>Microdermabrasion energy level: 20%-80%</p> <p>Operate according to different skin types of customers (try the set energy on lower arm first then on face, meanwhile check with customers constantly.)</p> <p>Facial Cleaning Scrubber Energy Level: Modal: 1 (press once); 2 (press twice); 3 (press thrice)</p> <p>Hydro spray jet energy level: Liquid export volume is recommended to set as small</p>	<p>Makeup remover+ Face wash+ Blackhead Export Liquid+ Hydrating Mask+ Cold and hot steam+ Essence+ Facial mask+ Device</p>	<ol style="list-style-type: none"> 1. Remove makeup and clean face, 5 min. 2. Apply blackhead export liquid+hot steam (on large pores or clogged area, and toner on cheeks; for sensitive skin it is not recommended to do so but apply toner+cold steam) 5-8 min. 3. Microdermabrasion operation: stay on forehead for one sec, then gently twist and leave it up in the air (forehead → nose → chin) 5min. 4. Operation on cheek: start from chin, and operate in dots then connects 3 dots in a line. Move in parallel lines. End with temple, 3 min. 5. Do the same on another side, 3 min. 6. Wash face clean, 2 min. 7. Apply hydrating (whitening) facial 	<p>Technique 3</p>  <hr/> <p>Technique 4</p>  <hr/> <p>Technique 12, 15</p>  <hr/> <p>Technique 14</p> 

<p>Bio&photon energy level: 2-8</p> <p>Modal: B</p> <p>Photon Light: Red</p>		<p>mask+hot steam, 10 min.</p> <p>8. Wash face clean, 2 min.</p> <p>9. Facial cleaning scrubber operation: drench cotton pad in water and pat on face, then tilt the scrubber at a 45°angle and slowly move around the face; start with nose and double the treatment if needed on nose, 3-6 times.</p> <p>10. Start from chin, brace the skin in lines to keep the treatment surface tight, 3-6 times.</p> <p>11. Do the same on the other side.</p> <p>12. Brace the skin vertically towards hairline on forehead, 3-6 times.</p> <p>13. Pour (hydrating/soothing)es sence into the hydro bottle(with the proportion of essence:distilled water being 2:1), 2 min.</p> <p>14. Hydro spray jet operation: start from chin to earlobe, then from mouth corner to ear gate. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>15. Move vertically</p>	
--	--	---	--

		<p>towards hairline on forehead, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>16. BIO&Photon treatment: start from chin, operate as if stamping on face accordingly, 5 min.</p> <p>17. Apply facial mask, 15 min.</p> <p>18. Wash face clean, 2 min.</p> <p>19. Apply toner, essence, cream, and sunscreen.</p>	
--	--	---	--

Recommended treatment:

10 treatments account for one full course. After one treatment you will see a clean and translucent skin. After a full course, skin is hydrated with less oil secretion and skin slowly regain its normal state. After 2 courses, skin is fine, glossy, hydrated and elastic.

Soothing&Calming: 60 min, once/week

<p>Cold hammer energy level: 30%-80%</p> <p>Hydro Spray Jet: Liquid export volume is recommended to set as small</p> <p>Bio&photon energy level: 2-8</p> <p>Modal: A</p> <p>Photon Light:</p>	<p>Makeup remover+ Face wash+ Hydrating Mask+ Cold steam+ Cold hammer+ Essence+ Facial mask (Tip: avoid using allergenic ingredients such as preservatives and fragrances)</p>	<ol style="list-style-type: none"> 1. Remove makeup and clean face, 5 min. 2. Apply hydrating facial mask+cold steam, 10 min 3. Pour (hydrating/soothing)essence into the hydro bottle(with the proportion of essence:distilled water being 2:1), 2 min. 4. Hydro spray jet operation: start from chin to earlobe, then from mouth corner to ear gate. Move in 	<p>Technique 4</p>  <hr/> <p>Technique 5</p>  <hr/> <p>Technique 6</p>
---	--	--	--

<p>Red,blue/green</p>	<p>parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>5. Move vertically towards hairline on forehead, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>6. Cold hammer operation: start from chin, brace the skin in circles to keep the treated surface tight, 3-5 time.</p> <p>7. Do the same on the other side.</p> <p>8. Move vertically towards hairline on forehead, 3-5 times.</p> <p>9. BIO&Photon treatment: start from chin, operate as if stamping on face accordingly, 5 min.</p> <p>10. Apply facial mask, 15 min.</p> <p>11. Wash face clean, 2 min.</p> <p>12. Apply toner, essence, cream, and sunscreen.</p>	 <hr/> <p>Technique 8</p> 
-----------------------	---	--

Suggested Treatment:
 10 treatments account for one full course. After one treatment you will feel skin is hydrated and smooth. After one course, the skin will restore a normal state, and less prone to allergy due to dehydration, cosmetics or change of season. After 2 courses, skin surface is smooth, ruddy with a boosted blood circulation and nice touch.

Acne Treatment: 60 min, once/week

<p>Facial Cleaning Scrubber: Continuous working mode</p> <p>High Frequency Head: Curved or mushroom-like</p> <p>High Frequency Energy: Twist the end of the handle to adjust energy level</p> <p>Bio&photon energy level: 2-5</p> <p>LED Photon: Red, blue</p> <p>Cold hammer energy level: 30%-80%</p>	<p>Makeup remover+ Face wash+ Blackhead Export Liquid+ Hydrating Mask+ Essence+ Cleaning Scrubber+ Hydro Spray Jet+ High Frequency Curved Head+ BIO&Photon Handle+Cold Hammer Facial mask (Tip: avoid using allergenic ingredients such as preservatives and fragrances)</p>	<ol style="list-style-type: none"> 1. Remove makeup and clean face, 5 min. 2. Apply blackhead export liquid+hot steam (on large pores or clogged area, and toner on cheeks; for sensitive skin it is not recommended to do so but apply toner+cold steam) 5-8 min. 3. Facial cleaning scrubber operation: drench cotton pad in water and pat on face, then tilt the scrubber at a 45°angle and slowly move around the face; start with nose and double the treatment if needed on nose, 3-6 times. 4. Start from chin, brace the skin in lines to keep the treatment surface tight, 3-6 times. 5. Do the same on the other side. 6. Brace the skin vertically towards hairline on forehead, 3-6 times. 7. Pour (whitening/hydrating/soothing/anti-aging)essence into the hydro bottle(with the proportion of essence:distilled water 	<p>Technique 6</p>  <hr/> <p>Technique 8, 9, 12, 13</p> 
---	--	---	--

		<p>being 2:1), 2 min.</p> <p>8. Hydro spray jet operation: start from chin to earlobe, then from mouth corner to ear gate. Move in parallel lines, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>9. Move vertically towards hairline on forehead, meanwhile brace the skin with the free hand to keep the treatment surface tight, 2 times.</p> <p>10. High frequency acne treatment: If a pimple is mature and has white tip, you need to squeeze out the pimple before treat it with high frequency curved head.</p> <p>11. It is advised to use the mushroom-like head if there are a lot of pimples on treatment area.</p> <p>12. Cold hammer operation: start from chin, brace the skin in circles to keep the treated surface tight, 3-5 time.</p> <p>13. Do the same on the other side.</p> <p>14. Move vertically towards hairline on forehead, 3-5 times.</p> <p>15. BIO&Photon treatment: start from</p>	
--	--	---	--

		chin, operate as if stamping on face accordingly, 5 min. 16. Apply facial mask, 15 min. 17. Wash face clean, 2 min. Apply toner, essence, cream, and sunscreen.	
--	--	--	--

Recommended treatment:

10 treatments account for one full course. After one treatment, skin is clean, smooth and translucent. Acne outbreaks are reduced rapidly and acne is minimized. After a full course, skin absorption and metabolism are accelerated. It also helps with acne breakouts and pore clogging. 3 courses help skin maintain a healthy and stable state, leaving it bright, tight, supple, and elastic.