
USER MANUAL

SR-MJ1808



Preface

Skin comprehensive management instrument is an indispensable skin cosmetology instrument, which can meet all the basic needs of female friends to deeply clean the skin, whiten and replenish water, and burn fat to shape one step at a time. Skin comprehensive management instrument can remove sebum and secretions, and can cooperate with blackhead extract to clean the face, while the whole cleaning process can also play a role of sterilization, make the skin healthy and beautiful.

Catalogue

Preface

Part I

| |
|---------------------|
| Operating principle |
|---------------------|

| |
|---|
| Applicable crowd and effect description |
|---|

Part II

| |
|--------------------|
| Detailed operation |
|--------------------|

| |
|---------------------------------|
| Instrument technical parameters |
|---------------------------------|

| |
|-------------|
| User Taboos |
|-------------|

| |
|-----------------------|
| Notes and Maintenance |
|-----------------------|

| |
|----------------------------|
| Operational considerations |
|----------------------------|

| |
|-------------------------------------|
| Common Faults and Treatment Methods |
|-------------------------------------|

| |
|------------------------------|
| Common Questions and Answers |
|------------------------------|

| |
|--------------|
| Packing list |
|--------------|

| |
|-------------------|
| Operation diagram |
|-------------------|

Part I

I. Dual Cleaning Principle

I. Skin Scrubber

The special energy exchange technology of spatula Scrubber has obvious penetration effect on human skin by mechanical vibration of 28000 times per second. Through medium atomization transmission, strong impact force can instantly remove deep dirt in pore, painlessly discharge acne, acne blocking pore, and make skin thoroughly clean.

Low-frequency emulsifying pore blocked sebum and secretion inducement can also sterilize, spray mist into the pore, and remove sebum and secretion function, remove dead skin.

Physical effects: High-frequency vibration sound wave combined with softening products or exfoliating ointment softens the damaged rough cuticle, and opens the pore with sound wave. High-frequency vibration removes the garbage and waste in the pore deeply.

Through the high frequency vibration and the essence products, the products can be molecularized into the skin, increasing elasticity and smoothing wrinkles.

II. Microdermabrasion

Secondary cleaning and tightening, using diamond crystals specially processed for patients'skin, using vacuum to absorb skin necrotic cells, remove deep pore dirt and waste, make the skin more clean and promote the growth of new skin. Use the negative pressure function to remove the black head and white head of the deep layer of the skin and dredge the blocked pore at the same time.

Biological effect: through negative pressure suction, combined with grinding head operation, can effectively remove metabolites on the skin surface and blackheads, garbage and blockages deep in the pore, equivalent to the skin vacuum cleaner, to thoroughly clean the skin.

III. Ultrasonic Cavitation

The warm and heat effect of ultrasound accelerates the blood circulation of tissue cells, activates the biological effect of tissue cells, renews and regenerates cells,

removes wrinkles, and restores elasticity and luster of skin. Ultrasonic waves can be used to produce foam, and remove sediment from the pores and skin tissues under the joint action. It can also promote the metabolism of the body and discharge the sediment.

Biological effects: powerful, energy-intensive, 1 million to 3 million massages per second can make skin cells vibrate, produce fine massage effect, change cell volume, thereby improving local blood and lymph circulation, enhancing cell permeability, improving tissue metabolism and regeneration, softening tissue, stimulating nervous system and cell work. It can make the skin glossy and elastic. Ultrasound can make the diffusion speed faster through vibration energy. It can penetrate nutritious products into the skin through high frequency vibration and heat energy. Ultrasound's warm and heat energy can penetrate into the skin at six centimeters below the skin. It can make the blood and lymph flow smoothly, accelerate the blood circulation of cells, activate the biological effects of cell tissues, weaken all kinds of color spots and make the skin white. Cells in deep tissues are activated and regenerated.

IV. Function

1. Spray water into the pore and bring out dirt through high frequency vibration.
2. Nutrients are refined into small molecular particles by high frequency vibration, which can enhance skin absorption and nutrients reach the dermis directly.
3. Improve skin quality, rough pore, wrinkles, thick cutin, purify skin, remove excess thick cutin, purify pore, decompose oil and smooth, tighten pore, clear and smooth skin, etc.
4. remove the dark circles with eye care essence, repair and care, eliminate dark circles, bags under the eyes and small dry lines.
5. Increase the elasticity of the skin to prevent muscle relaxation, rough skin, wrinkles and skin flaring.
6. Effectively excrete metabolites on the skin surface and blackheads, garbage and blockages deep in the pore, which are equivalent to the skin vacuum cleaner, and thoroughly clean the skin.
7. It is difficult to eliminate harmful substances in general cosmetic care to reduce the deposition of metal ions in the skin.
8. Prevention of pigmentation and whitening effect.
9. Instantly remove the dirt in the capillaries, and painlessly discharge the acne, whelk and facial black spots that obstruct the pores, so that the skin can be cleaned.

V. Applicable People

1. Dry skin and lack of water.
2. Coarse pore and dull skin.

-
3. Black-headed and white-headed pore clogging closed acne and acne crowd.
 4. Poor absorption of skin and water-deficient people at the bottom of skin.
 5. Dark skin and uneven complexion.
 6. Excessive oil secretion, oily and dry skin.
 7. Usually, the skin is not smooth because of improper cleaning.

VI. Taboo People

1. Cardiovascular and cerebrovascular diseases, three high population.
2. Serious skin allergy.
3. Patients with dermatosis and infectious diseases.
4. Metal patients (pacemaker, steel plate, metal teeth, etc.).
5. The skin has local symptoms such as injury and inflammation.
6. Skin allergic period.
7. Pregnancy and menstruation.
8. Skin wounds or wounds.

VII. Treatment Advantage

Microdermabrasion + skin scrubber + ultrasonic cavitation is a combination of three effects. Ultrasound uses high speed and small vibration to produce small bubbles, so that water molecules atomize, and under the combined action, it can discharge sediment from pores and skin tissues, and also can promote skin metabolism. Skin abrasion can deeply clean the blackheads and blockages in the pore, discharging the skin waste, deeply clear the dirt in the pore, and make the skin clean and clear. Give your skin a thorough cleaning.

VIII. Notes After Operation

1. Pay attention to sunscreen and avoid sunshine just after operation.
2. Avoid touching, grasping, buttoning, etc. with your hands.
3. pay attention to the application of multi mask to replenish moisture, prevent dry and accelerate metabolism.
4. Don't change skin care products frequently.
5. Be sure to remove your makeup before going to bed.
6. Avoid contact with high temperature.

II. Deep water replenishment and moisturization

I. Skin Scrubber

Special energy exchange technology, the energy generated by 28000 mechanical vibration per second has obvious penetration effect on human skin. Through the medium atomization transmission, strong impact force can instantly remove the deep dirt in the pore, painless discharge of acne and acne blocking the pore, so that the skin can be thoroughly cleaned.

Low-frequency emulsifying pore blocked sebum and secretion inducement can also sterilize, spray mist into the pore, and remove sebum and secretion function, remove dead skin.

Physical effects: High-frequency vibration sound wave combined with softening products or exfoliating ointment softens the damaged rough cuticle, and opens the pore with sound wave. High-frequency vibration removes the garbage and waste in the pore deeply.

Through the high frequency vibration and the essence products, the products can be molecularized into the skin, increasing elasticity and smoothing wrinkles.

II. Sprayer

Using pressurized oxygen, the product and oxygen are connected to the air pressure equipment at the same time, and the pellet water droplets are transformed into tiny molecular weight acting on the skin. Tiny droplets will be sprayed out by special designed nozzles in the form of spray at the speed of 200M seconds, and the water and oxygen will be injected into the bottom of human skin at great speed.

Biological effects: through vacuum negative pressure to form a vacuum circuit, combine the high concentration of reactive oxygen and essence, directly through the release of high pressure to the skin, so that nutrients molecules can promote the separation of pore waste, which can deeply clean the face, and can provide the skin with a lasting essence nutrition, and make the skin healthy and moist. Grease and luster.

III. Ultrasonic Cavitation

Warming effect: Through the warming effect of ultrasound, the skin surface temperature can be increased, blood circulation can be accelerated, nutrients of skin

cells can be increased, nerve excitability can be reduced, analgesic effect can be played, muscle fibers of spasm can be relaxed and spasmodic contraction can be played. Ultrasound heat is endogenous heat. 79% - 82% of the heat is transported by blood from the action zone, and 18% - 21% is dispersed from heat conduction to adjacent tissues. Therefore, customers have no obvious thermal sensation.

Chemistry: Ultrasound can strengthen the catalytic ability, accelerate the metabolism of skin cells, change the PH value of tissues to alkaline direction, alleviate the acidosis and pain accompanied by skin inflammation. Ultrasound can improve the permeability of cell membranes, make nutrients and drugs depolymerize, help skin absorb nutrients, and facilitate drug penetration into bacteria. Improve bactericidal ability.

Biological effect: Strong power, high energy, 1 million to 3 million times per second of fine massage, can make skin cells vibrate accordingly, produce micro-massage effect, make skin glossy and elastic, ultrasound through vibration energy makes the diffusion speed faster, nutritional products through high-frequency vibration and thermal energy fine progress. Into the skin, the temperature and heat of ultrasound can penetrate into the subcutaneous area of six centimeters, making the blood and lymph flow smoothly, accelerating the blood circulation of cells, activating the biological effects of cell tissues, deeply replenishing water, enhancing the absorption and penetration effect.

IV. LED Photon

LED high-tech optical skin rejuvenation cosmetology instrument uses photodynamic principle to activate deep cells, so that skin can better metabolize. After light is absorbed by skin, light energy is converted into intracellular energy, which relaxes and strengthens microvessels and produces photochemistry-enzymatic reaction with skin, and increases the activities of catalase and superoxide dismutase (SOD) as cell guards and cleaners. Increased decomposition of adenosine triphosphate (ATP) as a source of cellular energy. Increase glycogen and protein content, thereby promoting cell metabolism and synthesis. Stimulate the secretion of basic synthetic fibroblast growth factor (BFGF) epidermal growth factor (EGF) in vivo, make collagen fibers and elasticity, rearrange fibers, block and eliminate melanin formation, accelerate cell growth, accelerate blood circulation, stimulate fibrocyte tissue to produce collagen, increase skin elasticity, repair aging skin, treat sore skin, desalinate spots, enhance firmness. Nursing such as relieving sunburn plays an important role.

V. Function

1. Water atomization, deep pore, high frequency vibration, bring out dirt.
2. High frequency vibration refines nutrients into small molecular particles,

-
- effectively enhances skin absorption, and nutrients reach the dermis directly.
3. Import products to improve skin quality, pore size, wrinkles, thick cutin, purify skin, remove excess thick cutin, pore purify, decompose oil and smooth, tighten pore, clear and smooth skin, etc.
 4. Ultrasound can promote the normalization of the activity of fibrous tissue in the true slave tissue and increase the elasticity of slave skin from slave skin.
 5. Accelerating cell growth and blood circulation.
 6. Producing collagen, increasing skin elasticity, repairing aging skin, sore skin, desalinating spots, enhancing firmness, alleviating sunburn, etc.
 7. Nutritional products can enter skin more subtly through high frequency vibration and heat energy.
 8. Deep water supply and water lock to prevent water loss.

VI. Applicable People

1. Dry skin and lack of water.
2. Black-headed and white-headed pore clogging closed acne and acne crowd.
3. Sensitive skin.
4. Skin aging, wrinkles and skin.
5. Skin color is uneven and pigmented.
6. Need skin tightening to promote shiny people.
7. Poor absorption of skin and water-deficient people at the bottom of skin.
8. Skin is easily peeled, makeup removed and not tender.
9. Dry lines and fine lines appear on the skin.

VII. Taboo People

1. Cardiovascular and cerebrovascular diseases, three high population.
2. Serious skin allergy.
3. Patients with dermatosis and infectious diseases.
4. Metal patients (pacemaker, steel plate, metal teeth, etc.).
5. The skin has local symptoms such as injury and inflammation.
6. Skin allergic period.
7. Pregnancy and menstruation.
8. Skin wounds or wounds.

VIII. Treatment Advantage

Skin scrubber + Spray Gun + ultrasonic +led photon color light four effect combination, clean the skin deep into the essence, through the ultrasonic frequency along the direction of the pores into the dermal dermis, to achieve long-term effective replenishment and moisturizing, combined with the role of color light to

accelerate the blood circulation of the skin, accelerate the absorption, help the skin calm and repair, to achieve deep replenishment and water lock, so that your skin looks smoother. It is tender and shiny.

IX. Notes After Operation

1. Pay attention to sunscreen and avoid sunshine just after operation.
2. Avoid touching, grasping, buttoning, etc. with your hands.
3. pay attention to the application of multi mask to replenish moisture, prevent dry and accelerate metabolism.
4. Don't change skin care products frequently.
5. Be sure to remove your makeup before going to bed.
6. Avoid contact with high temperature.
7. Avoid products containing alcohol and hormones.

III. Microsculpture Skin Tightening

I. Microdermabrasion

Deep cleanliness and compactness, using diamond crystals specially processed for patients'skin, vacuum to absorb skin necrotic cells, remove deep pore dirt and waste, make the skin more clean and promote the growth of new skin. Use the negative pressure function to remove the deep layer of the skin of follicular mites, blackheads and whiteheads, while dredging the blocked pore, use the negative pressure to absorb the skin, to achieve skin tightening effect.

Biological effect: through negative pressure suction, combined with grinding head operation, can effectively remove metabolites on the skin surface and blackheads, garbage and blockages deep in the pore, equivalent to the skin vacuum cleaner, to thoroughly clean the skin. At the same time, the tightening and lifting effect can be achieved with the rotating lifting manipulation.

II. Ultrasonic Cavitation

Mechanical action: ultrasound has strong power and great ability. Acting on the face can make skin cells vibrate accordingly, produce fine massage effect, change cell volume, improve local blood and lymph circulation, enhance cell permeability, improve tissue metabolism, and regeneration ability, soften tissue, stimulate nervous system and cell function, make skin compact and full of elasticity. Sex,

Warming effect: Through the warming effect of ultrasound, the skin surface temperature can be increased, blood circulation can be accelerated, and nutrients of

skin cells can be increased. The heat of ultrasound is endogenous heat. 79% - 82% of the heat is transported by the blood self-acting area, and 18% - 21% is dispersed into adjacent tissues by heat conduction. Therefore, customers have no obvious thermal sensation, and the heat of skin bottom can effectively accelerate collagen protein. Regeneration makes skin full and elastic.

Biological effects: powerful, energy-intensive, 1 million to 3 million massages per second can make skin cells vibrate, produce fine massage effect, change cell volume, thereby improving local blood and lymph circulation, enhancing cell permeability, improving tissue metabolism and regeneration, softening tissue, stimulating nervous system and cell work. It can make the skin glossy and elastic. The temperature and heat energy of ultrasound can penetrate into the subcutaneous area of six centimeters, make the blood and lymph flow smoothly, accelerate the blood circulation of cells, start the biological effect of cells, stimulate the regeneration of cells, and activate and regenerate the cells of deep tissues. Achieve compact effect.

III. EMS

It mainly stimulates the acupoints of our face by micro-electric current, which can activate the cells in the bottom of our face, improve the activity of cells, accelerate the metabolism of the face and promote the formation of collagen in the dermis. It has the functions of whitening and lightening spots, rejuvenating skin, increasing skin elasticity, improving skin condition, repairing damaged skin, shrinking pore, tightening skin, anti-aging and anti-oxidation. Effectiveness. EMS mode, also known as micro-electric stimulation, can relax muscle spasm, increase movement, make facial lines clearer, so as to achieve the effect of tightening and lifting thin face.

Reduce and prevent muscle atrophy, increase local blood circulation and muscle re-exercise.

Biological effect: Use weak current to stimulate muscle and lymph, contracting muscle, to remove edema and tighten.

IV. Magic Gloves

The energy mode can be effectively transmitted to the human body, and the stimulation of the important acupoints of the human body can achieve the purpose of massage, which has the magical effect of skin beauty and shaping. Activate cells, restore skin elasticity, so as to solve some skin problems, delaying skin aging effect of beauty equipment.

V. Function

1. Enhance cell activity, promote muscle movement and restore skin elasticity.
2. Accelerate the blood circulation of microvessels, enhance cell permeability, and effectively supply nutrients to muscle tissue and skin.
3. Electrons generated by micro-current can penetrate deeply into the skin, help to repair elastic fibers and collagen layer tissue of the skin, thereby stretching and desalinating fine lines.
4. Tighten and elevate facial contour to create beautiful lines.
5. Antioxidant, whitening and lightening spots, tightening skin.
6. Accelerate skin metabolism and eliminate toxins.
7. Provide the skin with necessary nutrition, make the skin full of vitality and delay aging.

VI. Applicable People

1. People who lack luster, rough skin and blackheads.
2. People who often drink alcohol and often suffer from insomnia and poor sleep.
3. Population prone to facial varicella.
4. Population with blurred facial outline and blocked meridians.
5. Skin appears loose, sagging, not compact and full crowd.
6. Dark yellow and glossy skin.
7. Poor metabolism of skin, rough and unsmooth people.
8. Dry skin, lack of water and wrinkles.

VII. Taboo People

1. The face is allergic or has a wound.
2. Seriously sensitive muscles.
3. Pregnancy and recovery of operation.
4. Facial pus.
5. A person who has just undergone complete plastic surgery.
6. Cardiovascular and Cerebrovascular Diseases, Three High Populations.
7. Patients with dermatosis and infectious diseases.
8. Metal patients (pacemaker, steel plate, metal teeth, etc.).
9. The skin has local symptoms such as injury and inflammation.
10. Skin allergy.
11. Pregnancy and menstruation.

VIII. Treatment Advantages

Microdermabrasion + ultrasonic +EMS operation + magic gloves four effects, deep cleaning the skin, ultrasonic deep nutrition into the skin, promote dermal tissue activities, thereby increasing the elasticity of the skin, through the EMS micro current operation to stimulate the facial acupoint and accelerate the activity of cells, promote collagen regeneration, and the perfect combination of magic gloves, make skin compact. Lift, increase elasticity.

IX. Notes After Operation

1. Pay attention to sunscreen and avoid sunshine just after operation.
2. Avoid touching, grasping, buttoning, etc. with your hands.
3. pay attention to the application of multi mask to replenish moisture, prevent dry and accelerate metabolism.
4. Don't change skin care products frequently
5. Be sure to remove your makeup before going to bed
6. Avoid contact with high temperature
7. Avoid products containing alcohol and hormones

IV. Cell Regeneration

I. Skin Scrubber

Special energy exchange technology, the energy generated by 28000 mechanical vibration per second has obvious penetration effect on human skin. Through the medium atomization transmission, strong impact force can instantly remove the deep dirt in the pore, painless discharge of acne and acne blocking the pore, so that the skin can be thoroughly cleaned.

Low-frequency emulsifying pore blocked sebum and secretion inducement can also sterilize, spray mist into the pore, and remove sebum and secretion function, remove dead skin.

Physical effects: High-frequency vibration sound wave combined with softening products or exfoliating ointment softens the damaged rough cuticle, and opens the pore with sound wave. High-frequency vibration removes the garbage and waste in the pore deeply.

Through the high frequency vibration and the essence products, the products can be molecularized into the skin, increasing elasticity and smoothing wrinkles.

II. Ultrasonic Cavitation

Mechanical action: The power and power of ultrasound is strong, and the action on the face can make the skin cells vibrate accordingly, produce fine massage effect, change the cell volume, thereby improving the circulation of local blood and lymph, enhancing the permeability of cells, improving the metabolism and regeneration ability of tissue, softening tissue, stimulating nervous system and cell function, and making the skin elastic.

Biological effects: powerful, energy-intensive, 1 million to 3 million massages per second can make skin cells vibrate, produce fine massage effect, change cell volume, thereby improving local blood and lymph circulation, enhancing cell permeability, improving tissue metabolism and regeneration, softening tissue, stimulating nervous system and cell work. It can make the skin glossy and elastic. Ultrasound can make the diffusion speed faster through vibration energy. It can penetrate nutritious products into the skin through high frequency vibration and heat energy. Ultrasound's warm and heat energy can penetrate into the skin at six centimeters below the skin. It can make the blood and lymph flow smoothly, accelerate the blood circulation of cells, activate the biological effects of cell tissues, weaken all kinds of color spots and make the skin white. Cells in deep tissues are activated and regenerated.

III. EMS

Mainly through micro-electric stimulation of acupoints on our face, let the cells at the bottom of our face active, can improve cell activity, accelerate facial metabolism and promote the formation of collagen in the dermis, with whitening and light spots, rejuvenating skin, increasing skin elasticity, improving skin condition, repairing damaged skin, shrinking pore, tightening skin, anti-aging and antioxidant effect. EMS mode, also known as micro-electric stimulation, can relax muscle spasm, increase movement, make facial lines clearer, so as to achieve the effect of tightening and lifting thin face.

Reduce and prevent muscle atrophy, increase local blood circulation and muscle re-exercise.

Biological effect: Use weak current to stimulate muscle and lymph, contraction of muscle, to remove edema and tighten up the effect.

IV. LED Photon

LED high-tech optical skin rejuvenation cosmetology instrument uses photodynamic

principle to activate deep cells, so that skin can better metabolize. After light is absorbed by skin, light energy is converted into intracellular energy, which relaxes and strengthens microvessels and produces photochemistry-enzymatic reaction with skin, and increases the activities of catalase and superoxide dismutase (SOD) as cell guards and cleaners. Increased decomposition of adenosine triphosphate (ATP) as a source of cellular energy. Increase glycogen and protein content, thereby promoting cell metabolism and synthesis. Stimulate the secretion of basic synthetic fibroblast growth factor (BFGF) epidermal growth factor (EGF) in vivo, make collagen fibers and elasticity, rearrange fibers, block and eliminate melanin formation, accelerate cell growth, accelerate blood circulation, stimulate fibrocyte tissue to produce collagen, increase skin elasticity, repair aging skin, seat sore skin, desalinate spots, enhance firmness. Nursing such as relieving sunburn plays an important role.

Biological effect: Use photodynamic principle to activate deep cells, so that skin can better metabolize. When light is absorbed by the skin, it can be converted into intracellular energy, which relaxes and strengthens microvessels and produces photochemistry-enzymatic reaction with the skin, increasing the content of glycogen and protein, thus promoting cell metabolism and synthesis.

V. Function

1. Increasing the absorptivity of scleroderma.
2. Highly concentrated active agents penetrate the skin.
3. Promoting blood and lymphatic circulation.
4. Promote skin repair.
5. Promote the removal of aging substances from skin and human body.
6. Prevent pigmentation and whitening effect.
7. Quickly replenish oxygen and restore vitality to skin.
8. Anti-aging and Anti-oxidation.
9. Accelerate cell growth, promote collagen regeneration and accelerate blood circulation.
10. Deep cleaning of skin dirt and sterilization.
11. Increase skin luster and elasticity.
12. Tighten up and eliminate edema.

VI. Applicable People

1. Dry, dehydrated skin.
2. Oily, Mixed Skin, Extremely Dry and Oily Skin.
3. Coarse pore and dull skin.
4. Thin cuticle requires skin regeneration and repair.
5. Need skin tightening to promote shiny people.
6. Skin aging, wrinkles and skin.

7. Collagen loss, loosening, not full.

VII. Taboo People

1. The face is allergic or has a wound.
2. Seriously sensitive muscles.
3. Pregnancy and recovery of operation.
4. Facial pus.
5. A person who has just undergone complete plastic surgery.
6. Cardiovascular and Cerebrovascular Diseases, Three High Populations.
7. Patients with dermatosis and infectious diseases.
8. Metal patients (pacemaker, steel plate, metal teeth, etc.).
9. The skin has local symptoms such as injury and inflammation.
10. Skin allergy.
11. Pregnancy and menstruation.

VIII. Treatment Advantage

Skin scrubber + ultrasound + EMS + LED photon, four-effect combination, can promote blood circulation, promote the supply of oxygen and nutrition to the skin, through electric stimulation can increase cell activity, accelerate facial metabolism and blood circulation, promote collagen regeneration, can achieve faster cell regeneration and nutrition supplement effect. Make skin healthy, compact and full.

IX. Notes After Operation

1. Pay attention to sunscreen and avoid sunshine just after operation.
2. Avoid touching, grasping, buttoning, etc. with your hands.
3. pay attention to the application of multi mask to replenish moisture, prevent dry and accelerate metabolism.
4. Don't change skin care products frequently
5. Be sure to remove your makeup before going to bed
6. Avoid contact with high temperature
7. Avoid products containing alcohol and hormones

V. Calm and soothing

I. Skin Scrubber

Special energy exchange technology, the energy generated by 28000 mechanical vibration per second has obvious penetration effect on human skin. Through the

medium atomization transmission, strong impact force can instantly remove the deep dirt in the pore, painless discharge of acne and acne blocking the pore, so that the skin can be thoroughly cleaned.

Low-frequency emulsifying pore blocked sebum and secretion inducement can also sterilize, spray mist into the pore, and remove sebum and secretion function, remove dead skin.

Physical effects: High-frequency vibration sound wave combined with softening products or exfoliating ointment softens the damaged rough cuticle, and opens the pore with sound wave. High-frequency vibration removes the garbage and waste in the pore deeply.

Through the high frequency vibration and the essence products, the products can be molecularized into the skin, increasing elasticity and smoothing wrinkles.

II. Sprayer

Using pressurized oxygen, the product and oxygen are connected to the air pressure equipment at the same time, and the pellet water droplets are transformed into tiny molecular weight acting on the skin. Tiny droplets will be sprayed out by special designed nozzles in the form of spray at the speed of 200M seconds, and the water and oxygen will be injected into the bottom of human skin at great speed.

Biological effects: through vacuum negative pressure to form a vacuum circuit, combine the high concentration of reactive oxygen and essence, directly through the release of high pressure to the skin, so that nutrients molecules can promote the separation of pore waste, which can deeply clean the face, and can provide the skin with a lasting essence nutrition, so that the skin is healthy, moist, delicate and shiny.

III. LED Photon

LED high-tech optical skin rejuvenation cosmetology instrument uses photodynamic principle to activate deep cells, so that skin can better metabolize. After light is absorbed by skin, light energy is converted into intracellular energy, which relaxes and strengthens microvessels and produces photochemistry-enzymatic reaction with skin, and increases the activities of catalase and superoxide dismutase (SOD) as cell guards and cleaners. Increased decomposition of adenosine triphosphate (ATP) as a source of cellular energy. Increase glycogen and protein content, thereby promoting cell metabolism and synthesis. Stimulate the secretion of basic synthetic fibroblast growth factor (BFGF) epidermal growth factor (EGF) in vivo, make collagen fibers and elasticity, rearrange fibers, block and eliminate melanin formation, accelerate cell growth, accelerate blood circulation, stimulate fibrocyte tissue to produce collagen,

increase skin elasticity, repair aging skin, treat sore skin, desalinate spots, enhance firmness. Nursing such as relieving sunburn plays an important role.

Biological effect: Use photodynamic principle to activate deep cells, so that skin can better metabolize. When light is absorbed by the skin, it can be converted into intracellular energy, which relaxes and strengthens microvessels and produces photochemistry-enzymatic reaction with the skin, increasing the content of glycogen and protein, thus promoting cell metabolism and synthesis.

IV. Cold And Hot Hammer

Use the principle of air conditioner. Special wafer ceramics are used to absorb energy and refrigerate at - 5 degrees in an instant. The method of physical heat accumulation produced by wafer energy absorption makes one side produce heat up to 40 degrees. The fan discharges excess heat energy and achieves the effect of constant temperature and freezing alternately. Promote metabolism, alleviate allergic reaction, shrink pore, calm skin, whiten and tender skin.

Biological effects: The use of refrigeration and heating on the skin for the corresponding efficacy, cooling tissue temperature, blood vessels and veins contraction, reduce metabolic rate, inhibit slow cell activity, reduce peripheral nerve sensitivity, sedative and soothing effect, counteract allergic skin redness, swelling, pain, especially allergic skin. To calm the skin, reduce redness and swelling, shrink pore and other effects. When heating, it can heat tissue, raise temperature, adjust blood circulation, accelerate metabolism and promote product absorption.

V. Function

1. Promoting product absorption and penetration.
2. Relax and calm skin.
3. Eliminate redness, swelling, fever, pain, itching and discomfort.
4. Effective water replenishment to prevent water loss and lock up water.
5. Relieve acne muscles and fragile skin.
6. Shrinking pore, delicate skin, smoothing fine lines.
7. Repair skin barrier, enhance skin resistance and repair after sun exposure.
8. Clean up excess dirt and cutin.

VI. Applicable people

1. Dry skin and lack of water.
2. The pore is thick, blocked and the skin is rough and dull.
3. Sensitive skin.

-
4. Thin cuticle, red blood filament.
 5. Acne muscles, sensitive muscles.
 6. The pore is thick and rough.
 7. Dark and glossy complexion.
 8. Oily, dry and mixed skin.
 9. People with skin redness and swelling after exposure to sunlight.

VII. Taboo people

1. The face is allergic or has a wound.
2. Seriously sensitive muscles.
3. Pregnancy and recovery of operation.
4. Those with pus.
5. A person who has just undergone complete plastic surgery.
6. Cardiovascular and Cerebrovascular Diseases, Three High Populations.
7. Patients with dermatosis and infectious diseases.
8. Metal patients (pacemaker, steel plate, metal teeth, etc.).
9. The skin has local symptoms such as injury and inflammation.
10. Skin allergy.
11. Pregnancy and menstruation.

VIII. Treatment Advantage

Skin scrubber + Spray Gun + cold hammer + LED, four-in-one, calm the skin, alleviate the discomfort of sensitive skin, such as redness and swelling, shrink pore, LED light can eliminate inflammation and sterilization, promote the absorption of soothing effect of skin. It can use moisture essence, aloe vera essence, etc. to quickly restore the skin moisture wall, and enhance the stabilization effect.

IX. Notes After Operation

1. Pay attention to sunscreen and avoid sunshine just after operation.
2. Avoid touching, grasping, buttoning, etc. with your hands.
3. pay attention to the application of multi mask to replenish moisture, prevent dry. and accelerate metabolism.
4. Don't change skin care products frequently.
5. Be sure to remove your makeup before going to bed.
6. Avoid contact with high temperature.
7. Avoid products containing alcohol and hormones.

VI. Nursing Care Of Acne Elimination

I. Skin Scrubber

Special energy exchange technology, the energy generated by 28000 mechanical vibration per second has obvious penetration effect on human skin. Through the medium atomization transmission, strong impact force can instantly remove the deep dirt in the pore, painless discharge of acne and acne blocking the pore, so that the skin can be thoroughly cleaned.

Low-frequency emulsifying pore blocked sebum and secretion inducement can also sterilize, spray mist into the pore, and remove sebum and secretion function, remove dead skin.

Physical effects: High-frequency vibration sound wave combined with softening products or exfoliating ointment softens the damaged rough cuticle, and opens the pore with sound wave. High-frequency vibration removes the garbage and waste in the pore deeply.

Through the high frequency vibration and the essence products, the products can be molecularized into the skin, increasing elasticity and smoothing wrinkles.

II. Cold And Hot Hammer

Use the principle of air conditioner. Special wafer ceramics are used to absorb energy and refrigerate at - 5 degrees in an instant. The method of physical heat accumulation produced by wafer energy absorption makes one side produce heat up to 40 degrees. The fan discharges excess heat energy and achieves the effect of constant temperature and freezing alternately. Promote metabolism, alleviate allergic reaction, shrink pore, calm skin, whiten and tender skin.

Biological effects: The use of refrigeration and heating on the skin for the corresponding efficacy, cooling tissue temperature, blood vessels and veins contraction, reduce metabolic rate, inhibit slow cell activity, reduce peripheral nerve sensitivity, sedative and soothing effect, counteract allergic skin redness, swelling, pain, especially allergic skin. To calm the skin, reduce redness and swelling, shrink pore and other effects. When heating, it can heat tissue, raise temperature, adjust blood circulation, accelerate metabolism and promote product absorption.

III. Ultrasonic Cavitation

Chemistry: Ultrasound can enhance the catalytic ability, accelerate the metabolism of

skin cells, change the PH value of tissue to alkaline direction, alleviate the acidosis and pain accompanied by skin inflammation. Ultrasound can improve the permeability of cell membrane, depolymerize nutrients and drugs, help skin absorb nutrients, facilitate drug penetration into bacteria, and improve bactericidal ability.

Biological effects: powerful, energy-intensive, 1 million to 3 million massages per second can make skin cells vibrate, produce fine massage effect, change cell volume, thereby improving local blood and lymph circulation, enhancing cell permeability, improving tissue metabolism and regeneration, softening tissue, stimulating nervous system and cell work. It can make the skin glossy and elastic. Ultrasound can make the diffusion speed faster through vibration energy. It can penetrate nutritious products into the skin through high frequency vibration and heat energy. Ultrasound's warm and heat energy can penetrate into the skin at six centimeters below the skin. It can make the blood and lymph flow smoothly, accelerate the blood circulation of cells, activate the biological effects of cell tissues, weaken all kinds of color spots and make the skin white. Cells in deep tissues are activated and regenerated.

IV. Sprayer

Using pressurized oxygen, the product and oxygen are connected to the air pressure equipment at the same time, and the pellet water droplets are transformed into tiny molecular weight acting on the skin. Tiny droplets will be sprayed out by special designed nozzles in the form of spray at the speed of 200M seconds, and the water and oxygen will be injected into the bottom of human skin at great speed.

Biological effects: through vacuum negative pressure to form a vacuum circuit, combine the high concentration of reactive oxygen and essence, directly through the release of high pressure to the skin, so that nutrients molecules can promote the separation of pore waste, which can deeply clean the face, and can provide the skin with a lasting essence nutrition, so that the skin is healthy, moist, fine and glossy.

V. LED Photon

LED high-tech optical skin rejuvenation cosmetology instrument uses photodynamic principle to activate deep cells, so that skin can better metabolize. After light is absorbed by skin, light energy is converted into intracellular energy, which relaxes and strengthens microvessels and produces photochemistry-enzymatic reaction with skin, and increases the activities of catalase and superoxide dismutase (SOD) as cell guards and cleaners. Increased decomposition of adenosine triphosphate (ATP) as a source of cellular energy. Increase glycogen and protein content, thereby promoting cell metabolism and synthesis. Stimulate the secretion of basic synthetic fibroblast growth factor (BFGF) epidermal growth factor (EGF) in vivo, make collagen fibers and

elasticity, rearrange fibers, block and eliminate melanin formation, accelerate cell growth, accelerate blood circulation, stimulate fibrocyte tissue to produce collagen, increase skin elasticity, repair aging skin, treat sore skin, desalinate spots, enhance firmness. Nursing such as relieving sunburn plays an important role.

Biological effect: Use photodynamic principle to activate deep cells, so that skin can better metabolize. When light is absorbed by the skin, it can be converted into intracellular energy, which relaxes and strengthens microvessels and produces photochemistry-enzymatic reaction with the skin, increasing the content of glycogen and protein, thus promoting cell metabolism and synthesis.

VI. Function

1. Clean skin, remove cutin and pore blockage.
2. Calm the skin and relieve the symptoms of red, swollen, hot, painful and itchy skin.
3. Shrinking pore and locking water.
4. Promoting circulation, metabolism and accelerating product absorption.
5. Dilute pigments and tighten skin.
6. Effective water supplement to prevent water loss.
7. Increase elasticity and smooth fine lines.
8. Reduce the outbreak rate of acne and inhibit the growth of acne.
9. Anti-inflammation and sterilization, reduce the risk of acne breeding.
10. Clean skin garbage thoroughly to prevent blockage leading to acne growth.
11. Relieving acne marks and pigmentation.

VII. Applicable people

1. Acne muscles, oily skin.
2. The pore is thick, rough and not smooth.
3. Pigmentation and uneven complexion.
4. Blackhead, thick cuticle pore clogging crowd.
5. People with fine lines and dull skin.
6. Pox repetitive population.
7. Sensitive skin.

VIII. Taboo people

1. The face is allergic or has a wound.
2. Seriously sensitive muscles.
3. Pregnancy and recovery of operation.
4. Facial pus.
5. A person who has just undergone complete plastic surgery.

-
6. Cardiovascular and Cerebrovascular Diseases, Three High Populations.
 7. Patients with dermatosis and infectious diseases.
 8. Metal patients (pacemaker, steel plate, metal teeth, etc.).
 9. The skin has local symptoms such as injury and inflammation.
 10. Skin allergy.
 11. Pregnancy and menstruation.

IX. Treatment Advantage

Skin scrubber+ cold and hot hammer + ultrasonic + Spray Gun + LED color + high-frequency, six-effect integration, biological acne removal, calming skin, deep solution to acne problem, combined with color light and anti-inflammatory sterilization to skin, promote the elimination and repair of acne skin.

X. Notes After Operation

1. Pay attention to sunscreen and avoid sunshine just after operation.
2. Avoid touching, grasping, buttoning, etc. with your hands.
3. pay attention to the application of multi mask to replenish moisture, prevent dry and accelerate metabolism.
4. Don't change skin care products frequently.
5. Be sure to remove your makeup before going to bed.
6. Avoid contact with high temperature.
7. Avoid products containing alcohol and hormones.

Part II

I. Detailed Operation

II. Instrument Technical Parameters

III. User Taboos

1. Pregnant or lactating women.
2. Chronic heart disease or equipped with cardiac pacemaker.
3. The wound has not healed yet.
4. Epilepsy, severe diabetes and hyperthyroidism.
5. Patients with malignant tumors, hemophilia or severe bleeding.
6. Use cautiously in patients with dermatosis and infectious diseases.

IV. Notes And Maintain

1. After each use of the instrument, please remove the head of the course and clean it with clean water and keep it properly.
2. Before using the instrument, the plug with grounding pin must be used and the power socket of the instrument must be grounded.
3. Ensure that the voltage of the instrument is appropriate. If the voltage of the local power supply is unstable, we recommend that users increase the power matching regulated power supply between the municipal power supply and the instrument.
4. In order to ensure the therapeutic effect and normal service life of the instrument, please use the specified accessories provided or recommended by the original manufacturer.
5. Instruments should not be placed in wet places or near water sources, nor should they be directly exposed to sunlight.
6. Do not place the instrument near a strong heat source, because this may affect the life and normal use of the instrument.
7. Before treatment, please remove all metal objects from the body first, so as to avoid unexpected conditions and affect the curative effect.
8. Do not use in eyes, thyroid gland, parathyroid gland, testis, abdomen of pregnant women and atrial pulse generator.
9. Patients who are suffering from illness should also use this instrument carefully unless they have the permission of a doctor.
10. When no guest uses the instrument, please turn off the power switch of the instrument, and ensure that the total power supply of the instrument is turned off before the staff leaves after daily use, so as to ensure the safety of the electrical

products.

11. Use instruments or train instrument operators in strict accordance with instructions in the use manual.

V. The Use Of Precautions

1. If you feel unwell or find abnormal product, please turn off the power and stop.
2. If the intensity of the gills is too high, there may be a phenomenon of gills cramping. If the intensity of the gills is above, please stop immediately.
3. Please don't treat young people.
4. Place it on the ground where it is not easy for infants to touch.
5. If the product is abnormal, please stop and make sure that the power plug is unplugged.
6. When starting, please don't adjust the intensity too much. Start with low intensity.
7. If the skin is red and swollen and itchy, please reduce the output intensity and choose the appropriate strength for the skin.
8. Avoid touching the air vent when using the freezer, so as not to be scalded.
9. Check whether gaskets have been installed when using microcrystalline grinding, otherwise air leakage or suction will be insufficient.
10. When using a shovel, the face must be wet to avoid dry skin operation.
11. Avoid wearing metal articles when using this product. Invite customers to remove metal ornaments, which are not suitable for those with metal substances in their bodies.
12. Oxygen lance products must be water-based products.
13. Care should be taken to avoid spraying into the mouth and nostrils during oxygen spraying.
14. Clean the cups and bores before and after each use of the lance. Avoid blockage of pipe holes caused by condensation of products. Cleaning method: pour pure water into a small cup, then press the control button down and pull back, quickly eject the water from the cup.
15. When sucking blackheads, it is better to put appropriate amount of cotton paper in the glass tube to prevent dirt from sucking into the air duct and damaging the main engine. Don't stay on your face for too long to avoid bruising.
16. If there is no suction at the start of the machine, it is a normal phenomenon and can be restored to normal in a moment. Suction size can be adjusted by metal knob, clockwise for large.
17. Do not insert the catheter into the black hole when using the spray bottle, in order to prevent the cosmetic water from inverting the inhalation pump and damaging the main engine.
18. It is a normal phenomenon that there is a tingling sensation of current passing through the operation of electrons.
19. The cleaning shovel is grooved outward when it is cleaned. When it is operated, the water droplets are sprayed in mist shape, and there are no water droplets on the skin pushed by the instrument. Holding the cleaning shovel upside down and placing

the flat surface on the skin can be imported and upgraded.

VI. Common Faults And Treatment Methods

1. Does the instrument operate senselessly?
 - A. Check whether the instrument socket is plugged in properly
 - B. Check whether the handle link is inserted in place
 - C. Check whether the instrument is in contact with the product

2. The instrument can't start, and the key lamp on the back of the instrument is not on?
 - A. Ensure that the power cord is connected to an effective power outlet.
 - B. Is the fuse tube on the back of the instrument loose or burnt?

3. No current output?
 - A. Please check whether the plug connecting the instrument handle and the fuselage is tightly connected.
 - B. Please check whether the treatment area is clean. Oil or oily essential oil products may cause poor contact between the handle of the instrument and the human body, resulting in no current output.

4. Decreased current output?
 - A. Please check whether the plug connecting the instrument handle and the fuselage is tightly connected.
 - B. Please check whether the handle of the course of treatment is contaminated with non-conductive grease and other substances, which will cause poor contact and reduce the output.
 - C. Please check whether the product used is an adapter specified by the instrument.

5. Does the instrument have no or very low pressure?
 - A. Please close the instrument and check whether the hose of the instrument is tightened.
 - B. Please check whether the handle cover is tightened and whether the product is watery.
 - C. Please check whether the rubber hose is damaged or air leakage causes poor air pressure.
 - D. If the above method can not be handled, please contact the instrument distributor for assistance.

6. The instrument can be activated, but the wrong information appears on the screen?
 - A. Please unplug the power plug behind the instrument and wait about 1 minute to plug it in before restarting the instrument.
 - B. If the above method is not available, please contact the instrument distributor for

assistance.

VII. Common Questions And Solutions

1. Can I make up after the operation of this instrument?

Answer: Make-up can be completely done, after completing the skin is moist and full, at this time make-up more makeup. This is a skin-free and wound-free care, safe, healthy, fast and effective.

2. How long can I see the effect?

Answer: Once done, you can see that the skin is clear and transparent. The effect of continuous treatment is more obvious. It can improve the condition of deep water deficiency, dullness, fine lines and aging of skin.

3. Is there any patience in doing this?


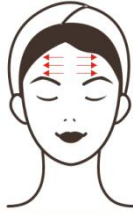



Answer: No patience, this is to improve and deeply clean the skin for skin problems, belongs to the maintenance and resistance of the skin caused by other factors, help the skin absorb nutritional products, solve skin dull yellow, water shortage, rough, wrinkled skin, if not done later, just return to the skin's own secretion and aging, will not cause patience.

VIII. Packing List

Ultrasonic handles for face x1
Cold hammer x1
Ultrasonic shape head x1
Magic Gloves x1
Diamond Dermabrasion tip x9
Diamond Dermabrasion plastic wand x3
Demabrasion machine x1
Microcurrent & Photon x 1
Galvanic winds x 2
Main machine x1
Plastic holder x1
Power supply x1

IX. Operation Diagram

| Parameter Adjustment | Product | Technique | Diagram |
|--|---------|-----------|---------|
| Double cleaning: once a week for 70 minutes | | | |

| | | | |
|--|--|--|--|
| <p>Energy regulation: 1-5 Patterns: A, B</p> | <p>Cleansing Oil +Cleansing Milk +black head liquid +microderma brasion grinding +cleaning scrubber +ultrasonic +essence +mask</p> | <ol style="list-style-type: none"> 1. Clean makeup for 5 minutes. 2. Apply blackhead extract + hot spray for 5-10 minutes (sensitive muscle can be about 2 minutes, blackhead serious can be up to 10 minutes). 3. After wetting the face with wet cotton sheets, clean the face with a spatula for 5-10 minutes (the nose with blackheads can be focused on cleaning). 4. Clean your face. 5. Apply exfoliating ointment or cleanser or skin lotion on the face and export it for 5-10 minutes with electric ion. 6. ultrasonic wave into the facial essence for 5-10 minutes. 7. ultrasonic wave into eye essence for 3-5 minutes. 8. apply cleansing and replenishing mask for 15 minutes. 9. End. |      |
|--|--|--|--|


Suggestions on the course of treatment

Ten times for a course of treatment, once done after the skin is clean and transparent, fresh and smooth pore. After one course of treatment, the problem of skin pore blockage has been improved, rough and strawberry nose has been alleviated, and skin stabilization effect has been achieved after three courses of treatment, which is conducive to better absorption of follow-up products, and skin is delicate and smooth.

Suggested price: 698/time 4980/10 times

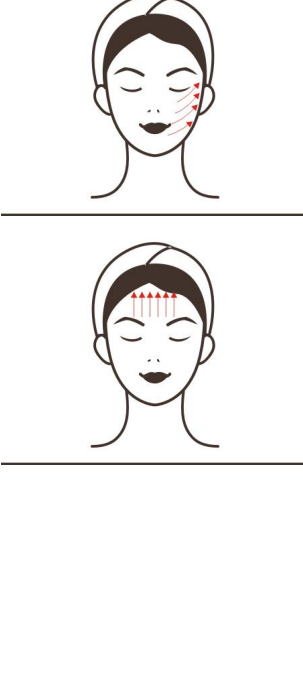
Deep moisturizing: 60 minutes once a week








| | | | |
|-----------------------------------|---|---|--|
| <p>Energy regulation: 1-5</p> | <p>Cleansing Oil + Cleansing Milk</p> | <ol style="list-style-type: none"> 1. Clean makeup for 5 minutes. 2. After wetting the face with wet cotton sheets, clean the face with a | |
|-----------------------------------|---|---|--|

| | | | |
|-----------------------|--|--|---|
| <p>Patterns: A, B</p> | <p>+clean scrubber +Spray Gun +ultrasonic +LED Photon +essence +mask</p> | <p>spatula for 5-10 minutes (the nose with blackheads can be cleaned mainly). 3. Clean your face. 4. Spray gun-mounted toner for 2 times cleaning and water replenishment for 3-5 minutes (line-up circular spraying). 5. ultrasonic wave into the facial essence for 5-10 minutes. 6. ultrasonic wave into eye essence, 3-5 minutes. 7. apply moisturizing mask for 15 minutes + micro electro color light seal operation face (choose red light). 8. Clean your face. 9. rub toner, cream, cream and sunscreen. 10. End.</p> |  |
|-----------------------|--|--|---|

Suggestions on the course of treatment
 Ten times is a course of treatment. After one time, the skin is tender and delicate. One course relieves dry skin, water shortage and peeling. Three courses stabilize the skin condition, keep the skin moisturized for a long time and prevent water loss.
 Suggested price: 698/time 4980/10 times

Microsculpture Skin Tightening: 90 minutes once a week

| | | | |
|--|--|--|---|
| <p>Energy regulation: 1-5 Patterns: A, B</p> | <p>Cleansing oil +Cleansing milk +microderma brasion grinding skin + ultrasonic + EMS +magic gloves + essence + mask</p> | <p>1. Clean makeup for 5 minutes. 2. Microcrystalline abrasive operation for 5-10 minutes (select the appropriate operating head and suction, face and forehead are pulled up by a line, nose can be pulled up in a circle). 3. Clean your face. 4. ultrasonic wave into the facial essence for 5-10 minutes (starting from the chin, rising from the front line to the front line), and introducing the compact Anti Wrinkle Essence. 5. ultrasonic wave into eye essence for 3-5 minutes. 6. EMS operation (two hands operate simultaneously from chin to chin, circle</p> |  |
|--|--|--|---|

| | | | |
|--|--|---|--|
| | | <p>upward next to line, and choose appropriate operation head) for 5-10 minutes.</p> <p>7. Put on the magic gloves after plugging in. Select the energy (adjust slowly from small to large, according to the customer's affordability) for 5-10 minutes (hands from neck to neck, rising next to line, forehead to hairline).</p> <p>8. apply mask for 15 minutes.</p> <p>9. Clean your face.</p> <p>10. rub toner, cream, cream and sunscreen.</p> <p>11. End.</p> |  <hr/>  <hr/>  <hr/>  <hr/>  <hr/>  <hr/>  |
|--|--|---|--|


Suggestions on the course of treatment

Ten times for a course of treatment, once done after the skin is clean and transparent, fresh and smooth

pore. It has compact effect. One course of treatment can relieve rough skin and improve skin contour. Three courses can stabilize skin condition, keep skin moisturized for a long time and prevent moisture loss. Skin tightening and upgrading, to achieve anti-aging effect.

Suggested price: 880/time 6980/10 times

Cell regeneration: once a week for 70 minutes

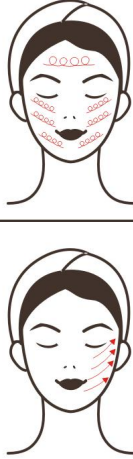
| | | | |
|--|--|---|--|
| <p>Energy regulation: 1-5 Patterns: A, B</p> | <p>Cleansing Oil + Cleansing Milk + clean Scrubber S + EMS + LED Photon + essence + mask</p> | <ol style="list-style-type: none"> 1. Clean makeup for 5 minutes. 2. Apply blackhead extract + hot spray for 5-10 minutes (sensitive muscle can be about 2 minutes, blackhead serious can be up to 10 minutes). 3. Moisten the face with cotton slices and then shovel for 5-10 minutes (the nose with blackhead can be the focus) 4. Clean your face. 5. sound wave into the facial essence for 5-10 minutes (starting from the chin, rising from the front line to the front line). 6. ultrasonic wave into eye essence for 3-5 minutes. 7. EMS operation (two hands operate at the same time from the chin line up next to the line to circle up to choose the appropriate operation head operation) 5-10 minutes. 8. apply mask 15 minutes +led color light seal operation (LED light can choose red light). 9. Clean your face. 10. rub toner, cream, cream, sunscreen. 11. End.. |  |
|--|--|---|--|

Suggestions on the course of treatment

Ten times for a course of treatment, one time after completing the skin clean and transparent, supplement the required nutrition and moisture, a course of treatment to stimulate the regeneration of collagen at the bottom of the skin, make the skin better absorb products and compact and elastic, three courses of treatment under the skin contour is clear, compact and full.

Suggested price: 698/time 4980/10 times

Calm and soothing: 60 minutes once a week

| | | | |
|--|---|---|---|
| <p>Energy regulation: 1-5 Patterns: A, B</p> | <p>Cleansing Oil + Cleansing Milk + cleaning scrubber + Spray Gun + cold head + LED Photon + essence + mask</p> | <ol style="list-style-type: none"> 1. Clean makeup for 5 minutes. 2. Moisten the face with cotton slices and then shovel for 5-10 minutes (the nose with blackhead can be the focus). 3. Clean your face. 4. Use spray gun to replenish water for 5-8 minutes. 5. evenly apply the essence to the face, then use the Cryo head operation (starting from the chin and rising from the chin to the front) for 5-10 minutes. 6. apply mask 15 minutes +led light seal operation (LED light can choose blue light, yellow light, color light). 7. Clean your face. 8. rub toner and essence. 9. End. |  |
|--|---|---|---|


Suggestions on the course of treatment

Ten times is a course of treatment. After one time, the skin is cool and refreshing, and the reddish and swollen skin is faded. One course improves the fragility of sensitive skin and reduces the sensitivity rate.

After three courses, the skin is stabilized. It is to keep the skin clean and tender in a healthy state.

Suggested price: 698/time 4980/10 times

Anti-acne nursing: 60 minutes once a week

| | | | |
|--|--|--|---|
| <p>Energy regulation: 1-5 Patterns: A, B</p> | <p>Cleansing Oil + Cleansing Milk + cleaning scrubber + cold and hot hammer + ultrasonic +LED Photon + high-frequency + essence + mask</p> | <ol style="list-style-type: none"> 1. Clean makeup for 5 minutes. 2. Moisten the face with cotton slices and then shovel for 5-10 minutes (the nose with blackhead can be the focus). 3. Secondary cleaning and replenishment of pure water in spray gun for 3-5 minutes (circular spraying from line to line upward). 4. Open the skin pore with a hot hammer for 3-5 minutes (circle the line up next to the line). 5. the ultrasonic wave is used to import the acne essence for 5-10 minutes. 6. High cycle wave acne eliminator operation for 5-10 minutes (select the appropriate operation head can be operated in full face circle, acne site strengthened). |  |
|--|--|--|---|

| | | | |
|--|--|--|--|
| | | <p>7. apply mask for 15 minutes combined with LED light seal operation (choose blue light, purple light).</p> <p>8. Clean your face.</p> <p>9. Refrigerator operation 5-10 minutes (line by line up circle).</p> <p>10. rub toner, acne essence.</p> <p>11. End.</p> | |
|--|--|--|--|

Suggestions on the course of treatment

Ten times for a course of treatment, one time after the completion of acne muscle anti-inflammatory sterilization, stability of the activity of acne, a course of treatment down to smooth acne muscle, eliminate redness and swelling, radically cure acne problems, three courses down to improve the skin acne, reduce the incidence of acne, skin back to normal state.

Suggested price: 698/time 4980/10 times