

User Manual

SR-VX8



Preface

Dear Users:

We're pleased to present to you our *8 Radar Line Carving Face Lifting Wrinkle Remove Skin Tight Rejuvenation Treatment Machine*, the latest technology that allows the heat energy to be transferred to the SMAS layer of the skin with precision and precision, which effectively stimulates the rapid growth of fibroblasts. It mainly target at skin problems like loose, sagging, aging skin and indistinct facial outline. *8 Radar Line Carving Face Lifting Wrinkle Remove Skin Tight Rejuvenation Treatment Machine* is professional beauty machine, which needs to be handled by professionals with proper training. Any improper use will result in adverse consequences. Therefore we advise any personnel read this manual thoroughly and apply with the instructions strictly before operation.

We believe our quality product will bring in good returns on your end. Also, you can count on our worry-free after-sales service.

Thank you!

Table of Contents

Preface
Part I
Brief Introduction
Advantages
Working Principles
Main Effects
Applicable Range
Inapplicable Range
Notes after Operation
Part II
1.Detailed Operations
2.Technical Parameters
3. Safety Precautions
4. Dos and Don'ts
5. Troubleshooting & Solutions
6.FAQs
7.Packing List
8.Operational Diagrams

Part I

Brief Introduction

8 Radar Line Carving Face Lifting Wrinkle Remove Skin Tight Rejuvenation Treatment Machine is currently the most popular beauty machine that work on skin anti-aging, tightening and sculpting. It addresses multiple skin problems, including cell aging, inactivity, decreased cell functions, skin dullness, wrinkles and so on. Moreover, it is convenient and easy to operate. It solves skin management issues for beauty-lovers. This equipment is safe, effective and has no side-effects. The radar line carver can effectively stimulate fibroblasts to divide new cells at a high speed through mechanical radar waves, dissolve facial adipose cells, tighten skin contour, repair damaged fiber tissue, and promote the growth of collagen cells, so as to quickly achieve the effect of slim facial outline and skin tightening.

Advantages

1. Simple, convenient and fast operation.
2. During the nursing process, it is comfortable and painless, non-invasive and injury-free, and no repair period is required.
3. No consumption, low cost and fast return.
4. It will not cause uneven appearance, bleeding and swelling and stasis.
5. Using the latest technology, the energy point is more accurate, safe and effective.
6. The unique way of probe friction beauty therapy can directly reach the depth of the target, so that the patients will not feel pain.
7. Non-invasive, proprietary radar technology, similar to skin tightening.

Principles

With mechanical radar wave good organization penetrating radar line carve, accurate would carry 65 - degree heat energy to the skin of SMAS layer (about 3.5 mm - 4.5 mm), effectively stimulate fibroblast 40 times per second speed new cell division, dissolve the facial contour, restore damaged fat cells, tighten skin, fibrous tissue, stimulate collagen cells quickly to achieve the effect of thin face, compact Q play. Radar line carving helps to improve wrinkles, skin elasticity and shape, and the formation of collagen.

The radar line carver can vibrate up to 6 to 12 million times per second. With good tissue penetration of mechanical radar wave, thermal energy of 65°C~75°C was accurately transferred to each layer of skin, 1.5 epidermal layer, 3.0 dermis layer, 4.5 fascial layer, and 4.5 collagen layer. Energy transferred heat energy from the depth of the epidermis to the fascial layer. Then ultrasonic radar line carve laser beam after

corneous layer, the transparent layer, granular layer and fine foam, nerve fibers, capillaries, base layer, the sebaceous glands, sweat glands and hair follicles, favorable nerve endings, fascia and periosteum layer after reflection to a preset level organization, make the skin temperature ascend to 65 ~ 75 °C or so, to determine the heat energy in the skin tissue fiber layer burns and damaged skin structure, the damaged structure start a growth in the supportive tissue of the skin, so, reach the role of skin to update again.

Main Effects

1. Improve sagging and loose skin.
2. Improve skin oil secretion and shrink pores.
3. Improve facial contour by lifting cheeks and canthus.
4. Improve crow's feet, lip lines, forehead lines, eye lines and other wrinkles.
5. Stimulate collagen regeneration and increase skin elasticity.
6. Promote metabolism and reverse aging.
7. Firming skin, improving double chin and clear contour.
8. Prevent cell aging and promote cell regeneration.
9. Repair damaged cells and relieve allergic skin.

Applicable Range

1. Those with sagging, dull skin.
2. Those with indistinct facial contour.
3. those with drooping mouth corners and cheeks.
4. Those with drooping canthus and obvious crow's feet.
5. Those with fast facial flabby aging.
6. Those who feel that their cheeks are too heavy and saggy.
7. Those who want to lift the cheeks, pull up the corners of the mouth and create a three-dimensional jaw line.

Inapplicable Range

1. Those who are pregnant women, menstrual period and lactation period.
2. Those with neurological diseases, myasthenia gravis, etc.
3. Those with heart attack, implantation of pacemaker or similar device.
4. Those with facial paralysis, diabetes, hyperthyroidism, advanced tumors, immune system diseases and skin diseases, such as lupus erythematosus, epilepsy, and skin cancer.
5. Those who had been injected with hyaluronic acid within 3 months.
6. Those who have had prosthesis and wire embedding in half a year.

7. Those with severe allergies, such as a rash.
8. Those with facial wounds and severe acne.

Notes after Operation

1. Do not wash your face with overheated water within three days (warm water is recommended, not exceeding your body temperature)
2. Pay more attention to hydration and sunscreen.
3. It is recommended to apply the mask 3 to 5 times in one week after operation.
4. Do not eat anything with heavy pigment such as soy sauce within a week after operation.
5. Do not use stimulating skin care products including scrubs, exfoliators and fruit acids within one week after operation.
6. Do not drink alcohol within a week after operation, eat spicy and stimulating food, eat more fruits and vegetables and light food.
7. Do not go to high-temperature sauna, yoga or other high-temperature environment within one week after operation, and do not expose to the sun.

Part II


1. Detailed Operation

Detailed installation is as follow:





After checking that the instrument is connected properly, the following starting interface will appear when the power supply is turned on.

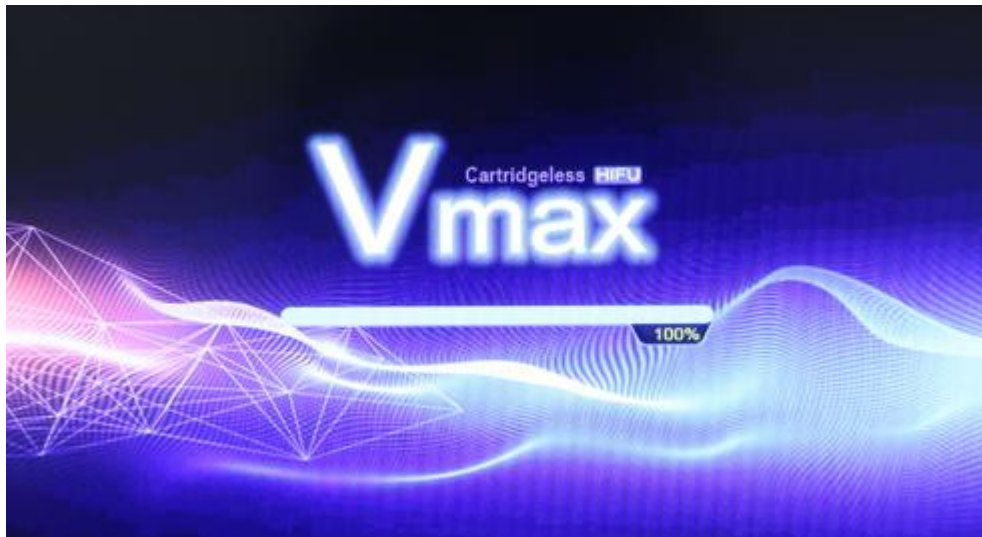

Click on the screen to go to the interface below



Probes



Lengths



Probe A (reaches into skin 3.0MM, the dermis layer)



Probe B (reaches into skin 4.5MM, the fascia layer)



Probe Selection




Working Time (It is advised to treat 10-20 min

each time on each part)



Time +/-

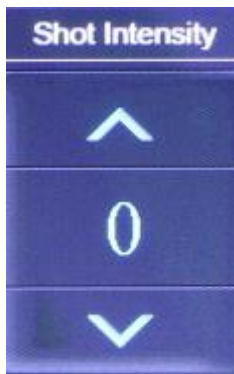

Shot Intensity/Energy +/- (It is advised to set between 5-10; low

intensity is to be set when treating on thin skin)



Energy +/-



Shot Intensity Value: Adjust based on one's own comfort level; it is advised to start with low intensity and add up slowly after adaptation; higher intensity, stronger sensation



Shot Time (It is advised to set between 2.0-3.0)



Shot Time +/-



Shot Time Value



Shot Interval (It is advised to set between 1.0-1.5)




Shot Interval +/-




Shot Interval Value



Modes

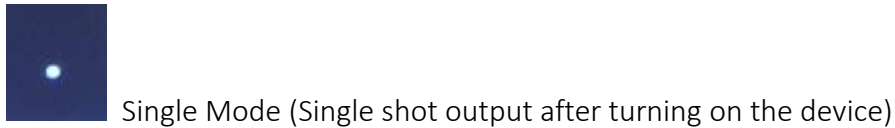

Single Mode (Single shot output after turning on the device)

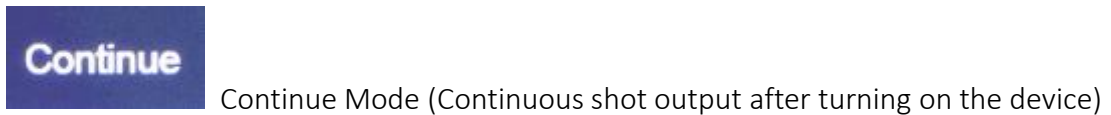


Continue Mode (Continuous shot output after turning on the device)



Mode Switch



Total Shot Time

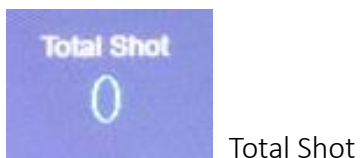

Total Shot



Start/Standby

2. Technical Parameters

Energy Setting: 0 - 100 Level

Frequency: 4MHz

Working Tim: 30 min

Interval: 0.1s - 1.5s

Duration: 0.1s - 1.5s

Treatment Depth of Probes: 3.0mm and 4.5mm

Power: 100w

Voltage: 100v—220v 50-60Hz

3. Safety Precautions

Patients with the following symptoms should be cautious about using this instrument. Please consult a doctor or a professional before using the instrument. The details are as follows:

1. Pregnant women, menstrual period and lactation period.
2. Patients with neurological diseases, myasthenia gravis, etc.
3. Heart attack, implantation of pacemaker or similar device.
4. Patients with diabetes, hyperthyroidism and advanced tumor.
5. Patients with facial paralysis, immune system diseases and skin diseases, such as lupus erythematosus, epilepsy, and skin cancer.

4. Dos and Dont's

1. Use a grounding pin and make sure the power socket of the device is properly connected before each operation.
2. Ensure that the voltage of the device is appropriate. If the voltage of the local power supply is unstable, we recommend that users increase to the matching power voltage.
3. In order to ensure the therapeutic effect and normal service life expectancy of the device, please use the specified accessories provided or recommended by the original manufacturer.
4. The device should not be placed in wet environment or near water, nor should they be directly exposed to sunlight.
5. Do not place the device near a strong heat source, because this may affect the life

and normal use of the device.

6. Before treatment, please remove all metal objects from the body first, so as to avoid unexpected conditions and influence the curative effect.

7. Please restrain from using the device on eyes, thyroid, parathyroid, testicles, abdomen of pregnant women, and pacemakers.

8. Patients who are suffering from illness should consult with a doctor and gain permission from the doctor before using.

9. Please turn off the power switch of the device, and ensure that the total power supply of the device is turned off before the staff leaves after daily use, so as to ensure the safety of the electrical products.

10. Use device or train device operators in strict accordance with instructions in the user manual.

11. Do not stay in the same position for more than 5 seconds during the operation of the instrument.

12. Please do not prohibit the operation in the area where the nerves pass from the thyroid, under the ear to the lower jaw.

13. Do not use on objects other than skin.

14. It is normal that teeth may be cold and painful if there are implants on the teeth or around the teeth during operation. At this time, you can inhale to make your mouth bulge or wrap the teeth with gauze before operation.

15. Do not operate between the eyebrows, bridge of the nose, chin and temples.

16. The forehead should be one finger apart, and the parietal region should be two fingers apart.

17. Before operation, remember to ask the customer whether it is in the three stages: pregnancy, menstruation and lactation.

18. Customers who have been injected with hyaluronic acid are advised to operate the radar line carving after 3 months.

19. If the customer has done embedding, it is suggested to operate the radar line carving after half a year.

20. If there is silicone or autologous fat filling in the operation site, avoid the operation.

21. During the operation, ask customers about their feelings and adjust the energy according to their feelings.

22. The skin on the face and forehead is different during operation, and the energy on the forehead is lower than that on the face.

5.Troubleshooting & Solutions

1. I can't turn on the device?

A. Please check the power supply of the device.

B. If the above methods cannot address the problem, please contact the instrument dealer for assistance.

2. The instrument does not work after being turned on?

- A. Please check if the working handle is properly connected with the main machine.
- B. If the above methods cannot address the issue, please contact the instrument dealer for assistance.

3. I can start the instrument, but the screen show's error message?

- A. Please unplug the power plug behind the instrument and wait for about 1 minute before restarting the instrument.
- B. If the above methods cannot address the issue, please contact the instrument dealer for assistance.

6. FAQs

1. Q: What is radar line carving?

A: Radar line carving is a kind of ultrasonic wave with good penetrating ability. The radar line carving for aesthetic purposes focuses on the skin 1.6-4.5mm (fascia layer) with high intensity positioning. By simultaneously firing 600-1200 ultrasonic waves, high-frequency friction will be generated between each other during the focusing, and thousands of heat condensation points will be aroused. The thermal stimulation generated by radar line carving can cause three stages of fascial layer cells: 1. 2. Gather energy to repair; 3. Make new cells.

2. Q: Visible result after one treatment?

A: The effect of skin repair and regeneration is a gradual process, not a sudden effect. Because it takes a certain amount of time for the muscles to build new collagen, the effect of radar line carving is gradual, not visible in a day, nor permanent.

3. Q: How long does it take to make radar line carving?

A: The operation of the radar line carver takes about 60 minutes. Slight reddening edema is normal and usually alleviates within a few hours after operation.

4. Q: Do radar lines have side effects?

A: Radar line is to use the good penetration of mechanical radar wave to effectively stimulate fibroblasts to divide new cells at a rate of 40 times per second, dissolve facial fat cells, tighten skin contour, restore damaged fiber tissue, stimulate collagen cells to regenerate, and quickly achieve the effect of thinning the face and tightening Q bullet. Radar line carving is non - invasive, non - invasive, no side effects, easy to use.

5. Q: Can I wear make up after radar carving?

A: After the radar line carving operation will appear slightly red or red swelling phenomenon, in a few hours will be alleviated, just after the operation of radar line carving day is not recommended to make up, can be in operation after 8~12 hours to

make up.

6.Q: What is collagen?

A: collagen protein is a kind of biology sex macromolecule material, it is a kind of white opaque the fiber sex protein that does not have branch chain, it can supplement the nutrition that skin each layer needs, make the collagen activity in the skin is enhanced, have lock up moisture, moist skin, defer senile, hairdressing, improve facial relaxation, raise the effect such as hair. Collagen is a nutrient that must be replenished to delay aging. As the growth of the age, the collagen will gradually erosion, women have begun to aging, after 20 years old, content is gradually decline, entering a loss peak, 25, 40, content is less than half of the age of 18, when old people face GouGou KanKan fold, collagen and moisture loss is to support the skin collagen fiber and elastic network fault, cause skin tissue by oxidation, atrophy, collapse, skin can appear dry, fold, flabby aging phenomenon such as inelastic, so, want to have to replenish collagen anti-aging.

7. Packing List

- 1 x Main Machine
- 1 x 3.0MM Handle
- 1 x 4.5MM Handle
- 2 x Accessories Frame
- 4 x Accessories Stand
- 1 x Foot Pedal
- 1 x Power Cord

8. Operational Diagrams

Parameter Adjustment	Product	Techniques	Diagrams
Skin Tightening&Lifting: 60 min, 1-2 times/week			
Shot Intensity: 5-10 Shot Time: 2.5-3.0 Shot Interval: 1.0-1.5 Mode: Continue	Makeup removal cream + cleanser + protein box (lysozyme + protein line + hyaluronic acid solution + freeze-dried powder + gel) + repair mask + SR-VX8	<ol style="list-style-type: none"> 1. Remove makeup and clean your face, 5 min. 2. Apply lotion, 1 min. 3. Take out the lysozyme and evenly apply half of the face, 2 to 3 times. 4. Lay a line of egg white from the bottom of the mouth to the temple, from the mouth to the temple, and from the nose to the temple (three lines are combined into a fan). 5. Select the appropriate operating head (4.5mm), adjust the parameters, and click start. 6. Add lysozyme to the protein strands, adding an average of 3 to 4 drops per strand. 7. Device operation: the probe is operated under the temple (to avoid the temple) by point-pressing along the position of the protein line, once or twice. 8. From the corner of the mouth down to the ear, the corner of the mouth to the ear, the nose to the temple down slowly move upward pull, 3-5 times. 	No Diagrams

		<p>9. Evenly apply the gel to the cheek area, 1 min.</p> <p>10. Operate the probe from the corner of the mouth down to the ear, the corner of the mouth to the ear, the nose to the temple down and up, 3-5 times.</p> <p>11. Apply gel evenly to forehead, 1 minute</p> <p>12. Operate the device on the forehead and pull it upward toward the hairline (avoiding the middle of the forehead and the temple), 3 to 5 times.</p> <p>13. Mix hyaluronic acid solution with freeze-dried powder and mix well. Apply to half of your face.</p> <p>14. Select the appropriate operating head (3.0mm), adjust the parameters, and click start.</p> <p>15. Device operation: from the corner of the mouth down to the ear, from the corner of the mouth to the ear, and from the nose to the temple, pull upward, 3 times.</p> <p>16. Start from the mandibular line and pull 3 times towards the cheek.</p> <p>17. (15 and 16) repeat the operation for 7-10 min.</p> <p>18. Operate the device on the forehead and pull it from the brow to the hairline, 3 times.</p> <p>19. Slowly move from the eyebrow to the end of the eyebrow, 3 times.</p> <p>20. (18 and 19) repeat the</p>	
--	--	---	--

		<p>two techniques for 3-5 min.</p> <p>21. Do the same on the other side.</p> <p>22. Apply the repair mask for 15-20 minutes.</p> <p>23. Remove membrane and clean face, 2 min.</p> <p>24. Apply repair solution or repair factor.</p>	
--	--	---	--

Suggested Treatment:
 6 treatments account for a full course. After one time, skin is lifted and appear to be slightly red. After one course, loose skin is improved and wrinkles are faded. After two courses, cheeks are lifted, skin is tightened and appear to be rosy and ruddy. Three courses help enhance the effect, reverse skin aging due to the loss of collagen cells, leaving skin young.