

# WL-929C

## 5 In 1 Slimming Machine

### Manual

#### Functions Include:

- 1- Multi-polar RF (Tripolar, Six Polar)
- 2-Vacuum
- 3- 40K Cavitation
- 4- BIO
- 5-COLD&Hot Treatment





## Warning

Thanks for using our company's products, in order to fully make use of the products, we sincerely suggests follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

## Working principal

### wave fat system

With collective strong sound wave head, strong sound wave of 40KHZ may be emitted to vibrate fat cells at top speed and produce numerous vacuum air pockets inside and outside the fat cells, robustly impact fat cells to generate introverted blast and disintegrate triglyceride into glycerol and free fatty acids. Then RF waves at frequency of 1M HZ is used for exhausting the integrated glycerol and free fatty acids through hepatoenteral circulation. Finally, vacuum RF and energy electrode are used for positioning and tightening fat. In physics, it is known as "cavitation". Micropore introverted blast inside and outside cell may lead to enhanced molecular motion and a higher energy level and this will finally cause fat cell rupture and thereby achieve the effects of body building and losing weight.



## SKET

### CH of the CONTROL PANEL

Click on the picture of the corresponding the handle enter the operation interface

## Instrument Function and Treatment Principles

### 1. Vacuum Bipolar RF handle

**Dissolve fat, lymphatic drainage, firm skin and enhance skin elasticity**



Body-shaping Equipment adopts the most advanced high-speed turbine negative pressure technology, and innovative three-dimensional conversion technology, combining with microcomputer chip controlled patent roller axis, and has an anti-geocentric attract effect on the skin.

Attracting and outspreading the fibrillar connective tissue of different cortexes, through the rhythmic massage and effectively break down subcutaneous fat and reduce cellular tissue accumulation of fat, use of the patents to massage deeply roller different parts of the body, bring up fat layer in dermis, curium and subcutaneous, and release

micro-vascular lymphatic vessels at the same time, promoting metabolism and to make fat cells in a passive movement to fat acids, even within the cells of toxins, also to be eliminated from lymphatic system in vitro, the detoxification enhance the fuction of as many as four times. Can be done manually or any other instruments can not achieve the perfect results. At the same time, skin is more elastic and luster.It is the "healthy body sculpture management" expert



## 2. Six Polar RF handle



Being integrated with the most advanced RF technology and radio frequency, the instrument may directly reach the deep-seated fat body and have the excellence of targeted positioning RF. In the fast-active state, fat cell tissue may generate friction heat, increase local temperature and remove excess fat and toxin from the body through sweat gland, enterohepatic circulation and lymph and ultimately achieve the effect of dissolving fat. With controlled depth technology, inlaid diamond particles of different sizes may rub skin along its pattern, adjust suction strength by means of vacuum draw and rub with negative pressure strongforce to directly explode thick fat. Thus, the effects are very obvious: Enhance blood circulation, accelerate the decomposition of fat, take away wastes, enhance the effect of removing cellulite, effectively improve the state of areolar tissue, remove stubborn fat and improve the sufficient absorption condition of diet products.



## 3. 40Khz Ultrasonic handle

**Strong sound wave explosion fat head**

With collective strong sound wave head, strong sound wave of 40KHZ may be emitted to human body for impacting fat cells fiercely and causing friction motion between fat cells. This may cause effective consumption of calories and moisture in fat cells and reduce the size of fat cells. What's more, sound wave vibration Vmay cause fierce impact of fat cells to make them be exploded instantaneously, reduce the amount of fat cells and thereby achieve the effects of removing fat.



**Recommended working time settings:**

Arm	Waist	Abdomen	Buttock	Thigh	Leg
4-8Min	4-8Min	5-10Min	10-15Min	10-15Min	4-8Min

**4. Tripolar facial handle**



Stick to enhance rejuvenation: use of positive and negative micro-current activation energy release skin cells, so that rapid synthesis of fibroblasts Collagen, elastic skin revert to the original compact, the skin causing t the skin to stimulate the muscle tension is increased, thereby gradually eliminate wrinkles, prevent skin aging, restoring skin elasticity, facial improvement.



**Caution for using RF**

**1.Beautician should wear insulated gloves during operation.**

2. Do not wear any aglet, both for the beautician and the users.
3. The electrode probe must be fully in contact with skin.
4. Do not make the RF electrode stay on any part without moving. Keep it moving on skin slowly.
5. Skin around eyes and forehead is very thin, the RF output intensity should be relatively weakened.
6. Do not contact skin with unipolar when using of bipolar, tripolar, hexpolar RF work head otherwise it will produce pulse discharge, gel paste etc. Products like gel maybe evaporated into a mist.

## 5. BIO:



Skin tightening after cellulite and fat breaking down;  
Lifting up, for example, faces, breasts, hips and etc.

### **BIO Operating Instruction:**

#### **Operating Instruction**

1. Connect one pair handle to BIO outlet.

**There are 3 pairs different handles available:**

**Mushroom handles:** For large area such as the cheeks, forehead, neck, etc.

**Spot handles:** For small area, such as nasal fossa, and so on.

**Handles without Cap:** Please insert the Cotton stick (which you can find them in the manual book plastic bag), then dampen the cotton with anti-wrinkle essence or other kind of essences. Its operating method is same as the other 2 kinds of handles.

**Note:** Handles with damp cotton stick conducts electricity stronger than the other 2 kinds of handles.

3. Select the time as you like about 15 minutes or 30 minutes.
4. Select the strong or weak intensity.
5. Select the mode just you like such as hip, bust, neck or face.

#### **Can be used on:**

- Micro wave Anti-wrinkle massage suitable for sagging, ageing, excrescence skin and eye bag muscle.
- Skin Maintain the pores and vessels. Clean up pores and vessels.

## 6. Hot & cold treatment instruction




The machine also has the hot & cold treatment functions. The cold treatment function is specially designed for claming down the skin after peeling. The skin is sensitive after peeling treatment, so the cold treatment procures is necessary. The hot treatment function is designed for accelerating the absorption of nutrition.

#### Hot & cold treatment instruction

1. Connect the hot & cold plug to the machine; please note the socket is in the "Hot & Cold" Area.
2. Start the machine: To turn on the machine, press the "on" button. And then start the hot & cold treatment function.
3. Please wait 3 minutes, and then you can apply the treatment. For the machine need time to warming up or cool down the treatment probe.
4. Normally one treatment program is about 5 ~10 minutes. Please note that do not stay in one treatment place over time when applying the treatment to avoid any hurt caused by cool head.
5. After treatment, please turn off the machine and then pull out the plug, make clean the probe.

### Using Guidance

#### 1. Operation schematic drawing of 40K cavitation explode work head

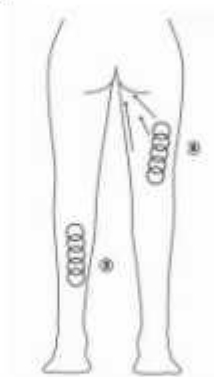

**Massage the fat of thigh in circles**

#### 2. Operation schematic drawing of sextupole RF laser work head for body

**Waist and abdomen care (25minutes, 2-3 days/time)**

- ① Massage to navel as center and gradually expanding the range



- ② Massage chest nest, both sides of former abdomen and below the




navel(into diamond) from inside to outside in circles.



- ③ Concentrated the fat from 4 directions to navel as center.

- ④ Massage from end of pelvis to groin in circles.

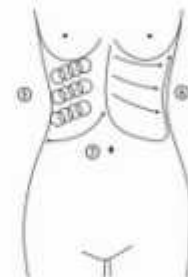

- ⑤ Massage from inner rib to outer edge in circles.

- ⑥ Massage back and forth from inner rib to outer edge in straight line

- ⑦ LiftiHip and he lower part of the body care (30minutes, 2-3days/time) used the ΦD50mm probe

Concentrated the fat to groin in circles or straight line.ng from inner rib to outer edge after massage.





### 3. Operation schematic drawing of Tripolar RF work head for face

① Massage the mandible center in circles to produce deep heat.



② Massage lower jaw in lines.

③ Massage the triangle zone of both sides of the face in circles.

③ Massage from jaw to angulus oris to ear in lines.



④ Massage from jaw to angulus oris to ear in lines.

## Bio Operation Skills



1-2

Place the device horizontally over the clavicle. Keep the balance. Move slowly down the neck to the lower jaw. Repeat several times before moving behind the ear. (pic.1-2)

Place the device on the clavicle, and slowly move them to the back of the neck. Repeat several times until the entire neck is covered. (pic.1-2)



3

Place the device on the clavicle for 1-3 minutes to open up the lymph nodes. (pic.3)



4

Pull the device slowly to the back of the neck and repeat several times to reduce neck lines. (pic.4)



5

Place the device under the jaw, and slowly move away to the back of the ears. Repeat several times, covering every inch of the skin. (pic.5)



6

Place the device under the jaw, and slowly move away to the ears, covering the entire jawline. Repeat several times if needed. (pic.6)




7

Place the device under the lower lip and slowly move them to the front of the ears. (pic.7)



8

Place the device horizontally on the corner of the lips and slowly move them to the front of the ears. This will help shape the ideal jawline. Stay at certain spots for 1-2 minutes. Repeat several times if needed. (pic.8)

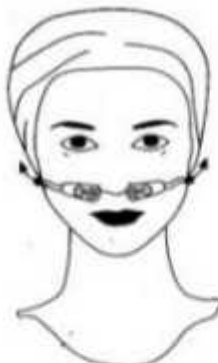

9

Place the device horizontally on both sides of the nose and slowly move them to the front of the ears. This will help shape the ideal jawline. Stay at certain spots for 1-2 minutes. Repeat several times if needed. (pic.9)

50%




10

Place the device vertically on both sides of the nose and slowly move them to the front of the ears. This will help shape the ideal jawline. Stay at certain spots for 1-2 minutes. Repeat several times if needed. (pic.10)



11

Place the device horizontally under the lower eye lids. Stay for 1-2 minutes, then slowly move to the temples, and stay for another 1-2 minutes to fade crow's feet. Repeat several times if needed. (pic.11)

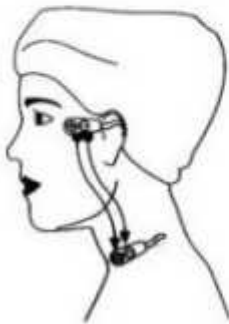

12

Place the device horizontally on the temples for 20-30 seconds, then slowly move to the clavicle. Repeat for several times and finally stay on the clavicle for 2-3 minutes to complete the treatment. (pic.12)

#### **ADVERTION**

2. Pregnant women or women during in menses.
  3. Epileptic
  4. Patients with malignancy.
  5. Patient whose wound after operation has not healed up.
  6. Acute inflammation or epidemical patients.
  7. Whom with heart diseases or with heart pacemaker..
  8. Whom with kidney (gall-stone ) disease.
  9. Who was embedded metal object or silica gel.
  10. Who in menses, birth control period, emiction incontinence period, or accepting the belly operation.
  11. Whose body always take much inner hot.
  12. Who has the genetic hypersensitivity.
-

## NOTICE

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
7. If the machine will not use for a long time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.