


User Manual

WL-UU1601

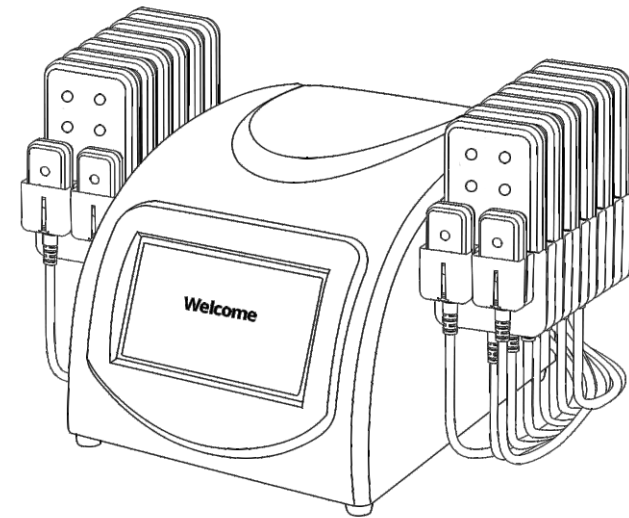

Thank you for purchasing our product.

- Please read this User's Manual carefully for information on how to use the device properly and safely.
- After reading the contents hereof, be sure to keep this User's Manual in an accessible location.
- Make sure to include this User's Manual if you are transferring ownership of the device to another person.

Contents

Part I

Safety precautions	1
Contraindications	2
Overview	2
Interface Introduction	3
Technical specification	3
Precautions	4
Maintenance & Cleaning	4
FAQ	5
Ask & Answer	6
Operation guide	7
Warranty and support	8

Safety precautions

1. Please read this manual carefully before use and familiarize yourself with the operation of this instrument and related technology.
2. This instrument is a high-precision electronic device that requires proper storage. Please avoid storing it in humid conditions, direct sunlight, or near inflammables and explosives to prevent damage.
3. Always use the designated accessory provided or recommended by the manufacturer to ensure the performance throughout its service life.
4. Do not place the instrument in high-temperature conditions, as this may negatively impact its service life and performance.
5. Do not apply the instrument on eyes, thyroid, parathyroid, testicles, pregnant women's abdomen, pacemakers, etc.
6. Patients should refrain from using this instrument unless they have obtained permission from their doctor.
7. Power off the instrument when not in use to ensure its safety.
8. If you are currently taking weight-loss drugs, please discontinue their use 1 - 2 months before using this instrument. Failure to do so may result in an extended course of treatment.
9. To maintain hygiene and extend the instrument's service life, sterilize it with saline.

Contraindications


Before using this instrument, it is essential to consult your doctor or seek advice from professionals. Details as below:

1. Pregnant or lactating woman.
2. Individuals with hypertension, heart disease or a pacemaker.
3. Individuals with unhealed wounds or those in the recovery period from surgery.
4. Individuals suffering from epilepsy, severe diabetes and hyperthyroidism.
5. Individuals suffering from malignant tumors, hemophilia and hemorrhage.
6. Individuals with skin disease or infectious disease.
7. Individuals have had a cesarean section within six months, a miscarriage within three months, or a natural birth within two months.
8. For your safety, refrain from using if you've had plastic surgery or have implants/metal objects near the treatment area within 3 months.

Overview

With the most popular and safest technologies, this LED beauty device provides you with a metamorphosis of your body. It provides the perfect solution for people seeking improvements in skin appearance and body figure without injections, drugs, or side effects. For external use with immediate effects.

Interface Introduction

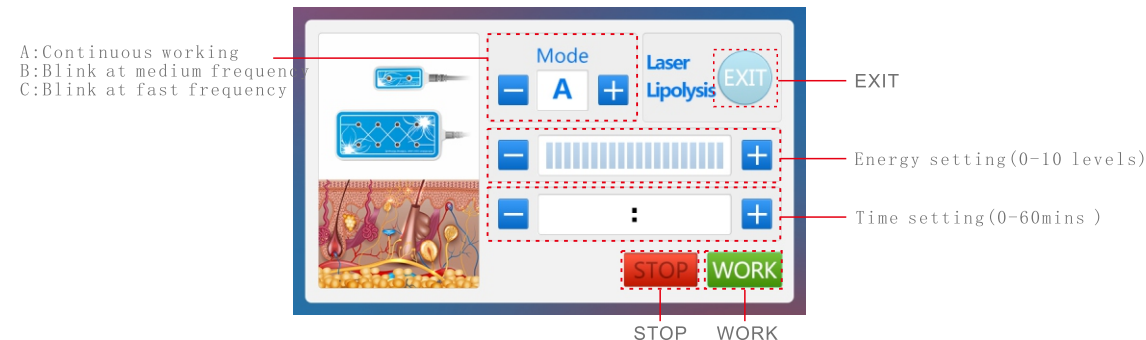

Technical Specifications

	Power supply Input: 240V/5A/50Hz 110V/5A/60Hz	Net Weight: 6.0kg
	Power: 35W	Gross Weight: 8.65kg
	Dimensions: 12.6 × 9.8 × 8.3 inch	
	Input Power: 35W	
	Wavelength: 635~650nm	
	Power: 160mw	

Precautions

1. Strictly follow the instructions in the manual to use the instrument and train operator.
2. Before using the instrument, you must use a plug with a grounding pin and ensure that the power socket is grounded.
3. Ensure that the voltage of the instrument is adapted, if the local power supply voltage is unstable, we recommend that the user add a power-matched voltage stabilized power supply between the mains and the instrument.
4. Remove all the metal objects inside the body to avoid causing accident or affecting treatment effect.
5. Stop the intake of weight drugs 1 - 2 months before the treatment, or extend the treatment course if you want to take the treatment immediately.
6. Do not use the instrument on empty abdomen, and take the treatment 1 hour later after the meal.

Maintenance & Cleaning

1. Disconnect the power cord when not in use and store the instrument in a dry, cool, and well-ventilated place.
2. Clean and sanitize the handle's surface with a damp cloth or saline, and then dry it thoroughly with a dry towel to ensure it remains dry.
3. After cleaning the instrument, cover it with a dustcloth or plastic wrap. Before use, ensure to sanitize the handle again for hygienic reasons.
4. Disassembling the instrument and attempting to clean the inside is strictly prohibited. This task should only be performed by professionals.

FAQ

1. LED fat removal treatment or liposuction surgery?

A: Compared to traditional liposuction, laser lipolysis treatment scans the fat area one by one, without causing unevenness in the skin, and conversely, leaving the skin smooth and tight without sagging. In addition, the LED energy used in LED fat removal only works on the set fat layer, not hurting the surface skin while accelerating the self-repair of skin elastic fibers, leaving the skin of the treatment area tight and smooth.

2. Will I regain weight after LED fat removal treatment?

A: The number of adipose cells in an adult remains constant, and weight gain is typically due to an increase in the size of these cells. The LED treatment employs a specific laser to stimulate adipose cells, leading them to release stored fatty acids and triglycerides, thereby reducing cell volume. Consequently, the occurrence of fat returning is highly unlikely. Usually, it will not rebound if you keep a healthy diet without overeating all day long.

3. Do I need to keep diet and take exercise during laser LED fat removal treatment?

A: The effect of laser lipolysis is remarkable, but we cannot only rely on it, if you don't keep a healthy diet after achieving idea weight after laser lipolysis, you are likely to regain the lost weight. It is important to know that there is no weight loss product in the world guarantees permanent effect. So, laser lipolysis is an auxiliary weight loss method. When the weight reaches within the ideal range, you need to control your diet and exercise reasonably after the treatment.

Ask & Answer


Ask	Cause	Answer
Instrument fails to turn on?	The power cord is disconnected to the wall socket.	Check if the power cord is well connected to the wall socket and ensure that the socket is functional.
	The fuse is loose or burn?	Replace the fuse on the back of the instrument and restart it.
The instrument has stopped functioning and cannot be restarted?	The fuse may have burned out due to unstable voltage or prolonged operation.	Please replace the fuse and restart the instrument 5 minutes later.
The screen shows an error message.	Data error?	<ol style="list-style-type: none">1. Turn off the instrument and unplug the power cord, and restart the instrument 1 minutes later.2. If the above method doesn't work, please contact the dealer for a replacement.

Operational guidance


Face (Step1: )
20mins




Belly (Step1: )
20mins

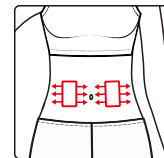

Arm (Step1: )
20mins

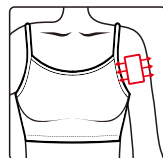

Leg (Step1: )
20mins

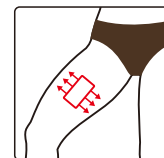

Warranty and support

We offer a one-year warranty and a three-month period for free returns and replacements from the date of purchase.

Product Name: LED Body Shaping Device
Product Model: WL-UU1601
Production Date: _____
Quality Inspection: _____

