



YH-5601

Instructions

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

◆◆Please read the instruction in detail and keep it well for further reading and consulting.

◆◆Please follow the instruction to fit and operate the machine correctly.

◆◆Please do not remove and change any accessories of the machine.

◆◆It forbids others to open and remove the mainframe except the authorized people in the company.

◆◆It forbids others to change and maintain the machine except the professional people.

◆◆Please contact us if there is any trouble during operation.

After Sale service

◆We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆How to contact us?

Phone:(+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!

Chapter 1: Safety Warnings

I. Electricity & Machinery Safety

1. The machine is either 110V And 220V. So whatever the voltage u can use before starting the machine.
2. Please keep the voltage at stable condition.

II. Contraindication Safety

1. People with heart disease, high blood pressure, or configured cardiac Pacemaker.
2. People with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
3. People with hemorrhagic diseases, trauma or who is bleeding.
4. Pregnant women
5. Medical plastic artificial or metal parts inside the body such as metal teeth or silicone breast implants.
6. People with abnormal immune system.
7. People with numb or insensitive to heat.
8. If you are feeling unwell, Prohibited in the ears, nose, eye throat and do not use this device in advance and tell your doctor.
9. Children under 12 years old are not allowed.

III. Packing Including:

- 1 x Main Machine
- 1 x Quadrople 3D Smart RF Handle
- 1 x Bio Handle
- 1 x Bipolar 3D RF with Vacuum Handle
- 1 x Vacuum Spray Handle
- 9 x Dermabrasion Tips
- 3 x Dermabrasion Wands
- 1 x Plastic Holder
- 1x Power Plug
- 2x PVC Hose
- 2x Filter
- 1x conton filter (1 bag)

Chapter 2: Before using

- I.** Before use, clean face with lotion and finish massage face to accelerate blood circulation.

II. How to install accessories.


1. Install filter and rubber band to microdermabrasion wands.



2. install red hose to machine hole (suction hole) and connect hose with glass tube or microdermabrasion wands.



3. Connect spray gum with machine hole (output hole) and black hose.

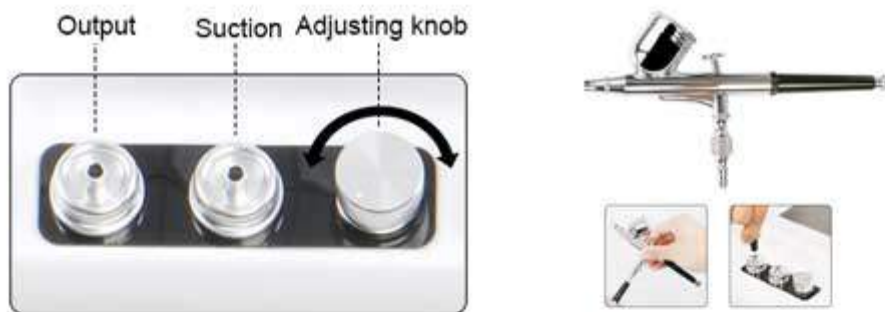

Notes: turn out the knobs of output and suction, to put in the rubber band before use.
And confirmed filter with rubber band before install to machine.



4. Connect handle BIO, face vacuum roller, quadrupole rf with machine, install in the two filter, put all handle to the holder. Plug in the power plug, turn on the red button on the back on the machine.



Chapter 3: program setting


I. Function Slection


1. When turn on the machine, you will see the welcome menu like this and then login in main menu:




2. Press “ SPRAY” enter into the function menu



(1) Click  to choose working a proper treatment time, press

 to start or stop the function.

(2)  can press to back to home page.


(3)  reset your choose.

2. Press Face Vacuum & RF into the function



(1) Click  to set the working time, setted refers to treatment time


as needed.


(2) Click  this to add or minus the energy of RF as need!


(3) 

Nor: means normal mode for this RF working model, temperature will reach the one setted at once.

Pro: professional mode, temperature will reach to the one u setted slowly.


(4)  set the vacuum suction and release by this button.

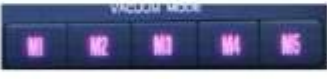
(5)  there are 5 mode of vacuum function, user could try one by one, and choose the one u like.


(6)  begin or finish your treatment by this button.


3. Choose Bio Function As Below:




(1) Click  to set the working time, setted refers to treatment time as needed.

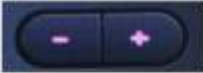(2) Click  this to add or minus the energy of BIO as need!


(3)  can turn on/off the led light of bio handle by this button.


(4)  Choose the Bio by this mode, user can try all the mode, to choose the one feel best, there are 5 mode, the mode will become stronger one by one.

(5)  begin or finish your treatment by this button.

4. Choose Face RF function as below:




(1) Click  to set the working time, setted refers to treatment time as needed.


(2) Click  this to add or minus the energy of RF as need!

(3) 

Nor: means normal mode for this RF working model, temperature will reach the one setted at once.


Pro: professional mode, temperature will reach to the one u setted slowly.


(4)  can turn on/off the led light of RF handle by this button.


(5)  begin or finish your treatment by this button.

5. Choose Dermabrasion function as below:



(1) Click  to choose working a proper treatment time, press

 to start or stop the function.

(2)  can press to back to home page.

(3)  reset your choose.

Chapter 4: User Guide

I. Steps for using the machine

Face RF→Bio→Face RF & Vacuum→Dermabrasion→Spray

1. Firstly, clean user face with clear water.
2. Use the Face RF to open skin pores, and skin lifting also. Time about 10 minutes.
3. Choose Bio to this area: sagging, ageing, excrescence skin and eye bag muscle, about 15 minutes.
4. Face Vacuum Roller for face shaping and tightening. Witin 10~15 minutes.

5. Use the dermabrasion to removal dead skin, then use the glasses tube to removal black heads, time is about 10~15 minutes.

Choose the suitable microdermabrasion tips according to different traetment area and start the treatment, massage along the skin texture.

There are 9 different microdermabrasion tips for you choose:

*Tips D240: Eyes and both sides of noise, for wrinkles, all blackhead and acne.

*Tips D200: For Scars, fine lines and imperfections.

*Tips D200: For Scars, fine lines and imperfections.


*Tps 120: body, especially for acne, knee, elbow area and striae of pregnancy.

6. Spray for skin moisturizing. Time about 15 minutes.

Notes: Next treatment should be after half of month.

II. Operation gesture for using the machine

1. **Bio/face vauum/RF** operation gesture can reference the below operation:

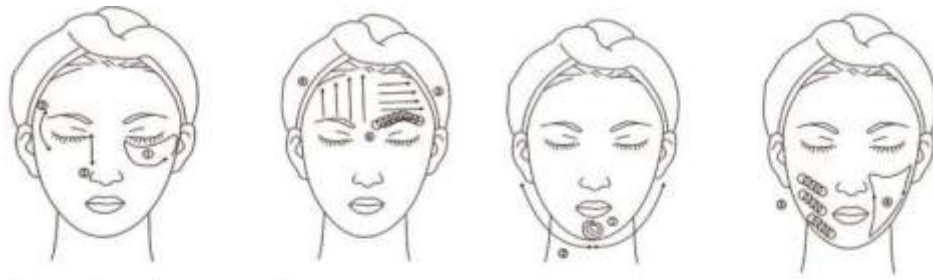

2. Dermabrasion operation gesture:

Operating Guide



STEP 1

Cleaning the facial skin and drying it.



STEP 2

Starting from the lower jaw, moving the rejuvenation from bottom to top.



STEP 3

Placing the rejuvenation instrument at the cheeks and moving from the inside to the outside.



STEP 4

Placing the rejuvenation instrument in the forehead area, moving from the middle to both sides.



STEP 5

Placing the rejuvenation instrument in the T zone, moving from top to bottom.



STEP 6

Using Toner to moisturize the skin, skin care products absorbing better.

3. Spray operation gesture:



Important Notes:

1. Clean skin with clear water.
2. Never stop the bio handle on the skin in one spot.
3. Always keeps the bio moving across the skin.
4. Never point the bio towards the chest area.
5. Dont not immerse the handle or the machine into water. Doing so could cause serious injury or death.
6. Store in a dry cool place.

Dont use on this area:

1. Sensitive skin.
2. Skin rash.
3. Damaged, inflamed, or infected skin.
4. Surgical scar less than 12months old.
5. Skin swollen.
6. The probes can not be point to eye ball.
7. Genials.

Safety Notes:

1. Never operate this equipment on a person has or suffers from the following medical conditions: Heart diseases, Sensitive Skin, Damaged, Inflamed, or Infected skin.
2. When proceed deep cleaning or Galvanic, accompanies with suitable emulsion or gel.
3. Never stop the handle's blade on the skin in one spot, always keep the blade moving on the skin.
4. Never point the blade towards the eyes or throat.
5. Do not modify the equipment.
6. Do not immerse the handle or the machine into water.
7. After usage, clean the Handle and store in a cool and dry place. The machine should be grounding.