



YH-7602SB

Instructions

Warning

Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

◆ How to contact us?

Phone:(+86) 189 -487-55929

Tel: 0755-86368500

Email: sales@mychway.com

This product does not belong to medical, please don't use for disease treatment!


Table of Contents:

| | | |
|------|--|--|
| 1. | Introduction to Manual | |
| 1.1 | Purpose | |
| 1.2 | Conventions | |
| 2. | Medical Safety..... | |
| 2.1. | Indications for Use..... | |
| 2.2. | Contraindications | |
| 2.3. | Precautions | |
| 2.4. | Patient Safety..... | |
| 3. | Features&Function..... | |
| 3.1. | Features..... | |
| 3.2. | Function..... | |
| 4. | Control Unit & Handpiece..... | |
| 5. | Operation Procedures..... | |
| 5.1. | Cavitation 40k probe..... | |
| 5.2. | Vacuum Body RF Handle | |
| 5.3. | Multipolar RF Head for Body..... | |
| 5.4. | Quadropole RF Head for Face and Body..... | |
| 5.5. | Bipolar RF Head for Face and Eyes' Around..... | |
| 5.6. | Cold probe..... | |
| 5.7. | Photon&Micro Current for Skin care | |
| 5.8. | Lipo laser..... | |
| 6. | Package Including..... | |
| 7. | Parameters..... | |
| 8. | Notice..... | |
| 9. | Illustration of Operation..... | |

1. Introduction to Manual


1.1 Purpose


This instructions for use manual provides a description of the system components, its controls and displays, instructions for its operation, and other equipment information important to the user


 **Warning:** Do Not operate this machine before reading this manual thoroughly. In addition to this manual,additional clinical training may be available by the company or your local distributor.

For more information on training available please contact your local representative.

1.2 Conventions

 **Note:** Notes designate information of special interest.

 **Caution:** Cautions alert the user to precautionary steps necessary to properly operate the system.Failure to observe these cautions may void the warranty

 **Warning:** Warnings alert the user to information that is of the highest importance and vital to the safety of the patient and user

All procedures are broken down by numbered steps.Steps must be completed in the sequence they are presented

Bulleted lists indicate general information about a particular function or procedure.They do not imply a sequential procedure.

Control names are spelled as they are on the system,and they appear in **Bold text**

2. Medical Safety

2.1. Indications for Use

The cavitation vacuum RF micro current photon machine is indicated for use as a non-invasive dermatological aesthetic treatment to:

- Body slimming,body sculpture

- Face care, lifting, whitening
- Neck skin lift

2.2. Contraindications

The cavitation vacuum RF micro current photon machine is contraindicated for use in patients with:

- Open wounds or lesions on the face and/or neck
- Severe or cystic acne on the face and/or neck

2.3. Precautions

The cavitation vacuum RF micro current photon machine has not been evaluated for use over various materials. Therefore, treatment is not recommended directly over those areas with any of the following:

- Mechanical implants
- Dermal fillers
- Implanted electrical devices in the face and/or neck
- Metal stents in the face and/or neck area

Treatment energy is not recommended for use directly on an existing keloid.

The cavitation vacuum RF micro current photon machine has not been evaluated for use in patients on an anticoagulant treatment plan


It is recommended that the following areas should be avoided during treatment:


- Thyroid gland, thyroid cartilage and trachea
- Major vessels


The cavitation vacuum RF micro current photon machine has not been evaluated for use in the following patient populations:

- Pregnant or breast feeding women
- Children
- Those with the following disease states
 - A hemorrhagic disorder or hemostatic dysfunction
 - An active systemic or local skin disease that may alter wound healing
 - Herpes simplex
 - Autoimmune disease
 - Diabetes
 - Bell's palsy

2.4. Patient Safety

 Warning: cavitation vacuum RF micro current photon machine should not be used on a patient's eyes or in a location or technique where ultrasound energy can reach the eye.

 Warning: Use this system only if you are trained and qualified to do so.

 Warning: If any problems occur during system operation, ask immediate action(s): lift the transducer off the patient's skin, press the STOP button to completely halt system operation.

3. Features&Function

3.1 Features

Painless treatment focus RF energy at the correct position, compared to other radio frequency technology. It uses low energy and high frequency, safe and effective.

Aim at the skin surface and deep location, using a complex method to control the different currents and energy, direct access to the different skin layer. Does not cause the phenomenon of uneven skin.

Selectively targeting fatty tissue, to avoid heating other fat, to achieve the fastest treatment effect.

The entire process is completed without the need of surgery and anesthesia.

With the most useful 40KHZ ultrasonic cavitation system.

Without side effects and the risk of gaining the weight back, the results are rather obvious. Does not affect the normal work and life.

3.2 Functions



Firming the arms, legs, thighs, buttocks, waist, back, abdominal muscles and reshape the body figure Improve and treat the orange peel like fatty tissues of different serious levels With the effects of soothing, decompression and pain relief (such as joint pain, sciatica), promote the regular mechanical aerobic exercise of skin tissue, rapidly relieve stress and remove fatigue. Firming, lifting and compact the skin and mould the face shape circulate and activate cells, promote metabolism and so that 100% more collagen and elastic fibers may be gained for improving the facial and body skin.

4. Control Unit & Handpiece





5. OPERATION PROCEDURES:

Press the on/off button, then it comes to the welcome interface first, then to the main function menu.

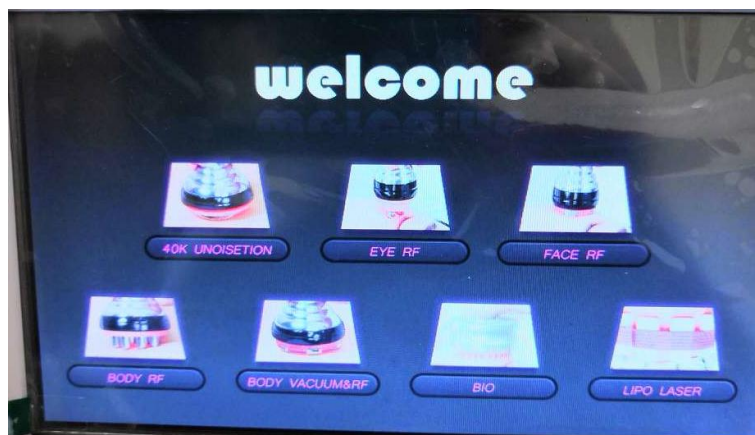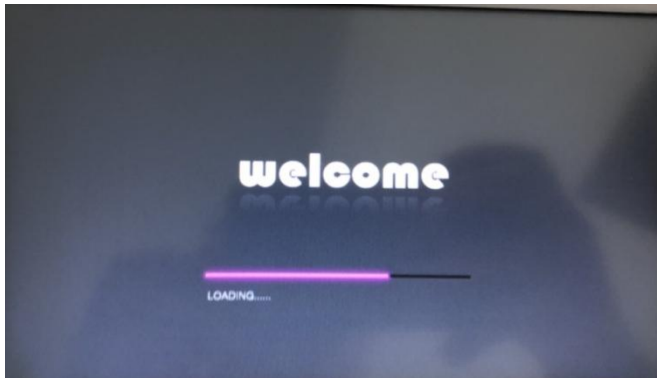



5.1 Cavitation 40k Probe

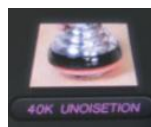

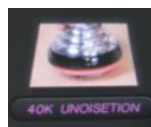
Choose the function , then it comes to the following interface:



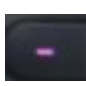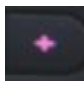

Press  to adjust the working time.


Press  to decrease the energy, and press  to increase the energy. Level 0%~100% can be adjusted.


Press  to select the model.

Model A  :continuous wave, for the area with thin fat;

Model B  :dis-continuous wave, for the area with thick fat.

Press  to start or pause the operation.

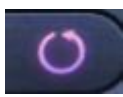Press  to clear all the settings.

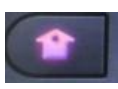Press  to back to the main function menu.

Operate with the 40KHz cavitation head for 15-20 minutes.




Warning:

Cavitation 40k Probe is forbidden to treat on back and face.


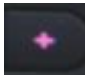
5.2 Body vacuum RF Handle




Choose the function  , then it comes to the following interface:




Press  to adjust the working time.

Press  to decrease the RF energy, and press  to increase the RF energy.

Level 0%~100% can be adjusted.

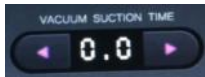
Press  to adjust the suction time.


Press  to adjust the release time.


Please note: the suction time needs to be a little longer than the release time.

You can also press  to choose the vacuum model directly.


Then you don't need to adjust the suction and release time.

Press  , the RF temperature will be up the one you set right now.

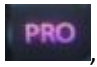Press  , the RF temperature will be up the one you set gradually.

Press  to start or pause the operation.

Press  to clear all the settings.

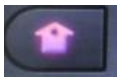Press  to back to the main function menu.

Warm tips:



Apply enough slimming cream on the treatment area.

Keep in full contact with the skin during treatment, move the probe on the surface of the cared area, in an even and thorough way


Adjust the intensity from low level to high level.

Please close eyes during led photon therapy, direct aim to eyes is forbidden

15-20 minutes on each treatment area recommended


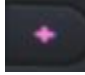
5.3 Multipolar RF Head for Body




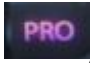
Choose the function , then it will come to the following interface:





Press  to adjust the working time.


Press  to decrease the energy, and press  to increase the energy. Level 0%~100% can be adjusted.


Press , the RF temperature will be up the one you set right now.

Press , the RF temperature will be up the one you set gradually.

Press , the led on the probe will keep on.

Press  to start or pause the operation.

Press  to clear all the settings.

Press  to back to the main function menu.


Warm tips:




Keep in full contact with the skin during treatment, move the probe on the surface of the cared area, in an even and thorough way
Adjust the intensity from low level to high level.
Please close eyes during led photon therapy, direct aim to eyes is forbidden
15-20 minutes on each treatment area recommended

5.4 Quadrupole RF Head for Face and Body


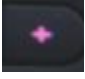


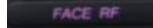
Choose the function , it will come to the following interface:

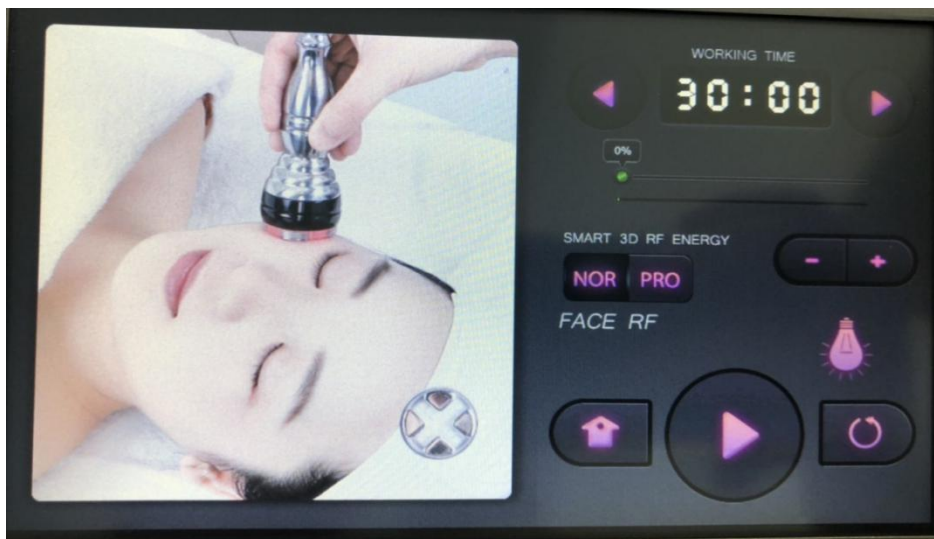






Press   to adjust the working time.


Press  to decrease the energy, and press  to increase the energy. Level 0%~100% can be adjusted.


Press , the RF temperature will be up the one you set right now.

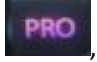Press , the RF temperature will be up the one you set gradually.

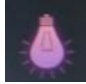Press , the led on the probe will keep on.

Press  to start or pause the operation.

Press  to clear all the settings.

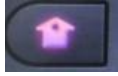Press  to back to the main function menu.

Warm tips:



Apply enough slimming cream/gel on the treatment area.

Keep in full contact with the skin during treatment, move the probe on the surface of the cared area, in an even and thorough way


Adjust the intensity from low level to high level.


Please close eyes during led photon therapy, direct aim to eyes is forbidden

15-20 minutes on each treatment area recommended



5.5 Bipolar RF Head for Face and Eyes' Around





Choose the function , then it will come to the following interface:

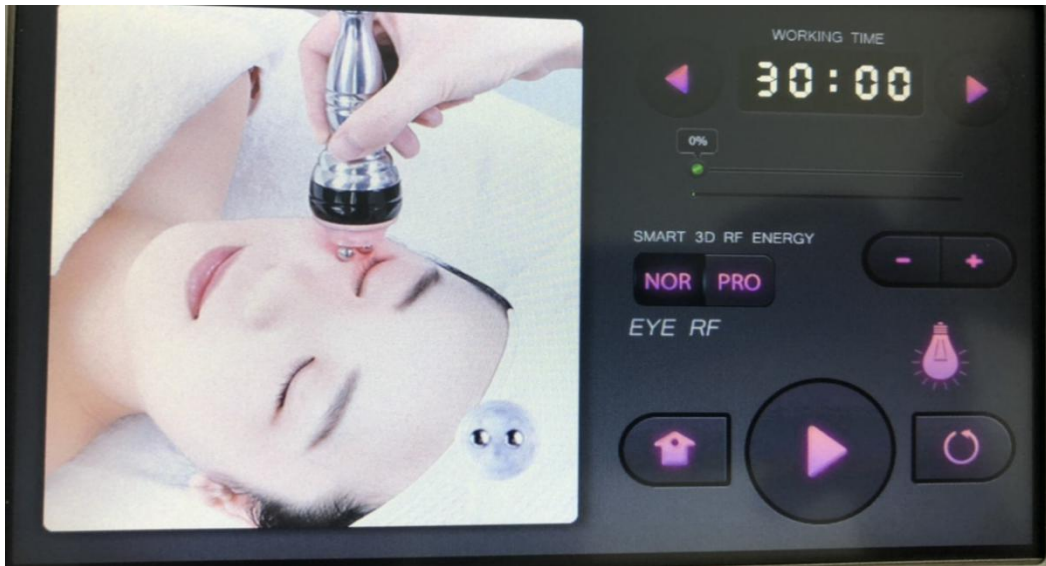

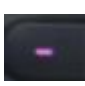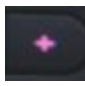

Press  to adjust the working time.

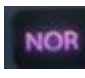
Press  to decrease the energy, and press  to increase the energy. Level 0%~100% can be adjusted.

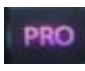
Press , the RF temperature will be up the one you set right now.

Press , the RF temperature will be up the one you set gradually.

Press , the led on the probe will keep on.

Press  to start or pause the operation.

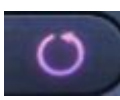Press  to clear all the settings.

Press  to back to the main function menu.

5.6 Cold probe (calming down the skin and shrinking pores)


The temperature of skin will up after the treatment of the cavitation probe, vacuum probe and RF probes. Then use the the cold probe, it helps to calm down the skin, and shrink pores.



Press the red on/off button of the probe to start/pause the probe,working time 5-10 minutes.



5.7 Photon&Micro Current for Skin care



Choose the function  , then it will come to the following interface:



Press  to adjust the working time.

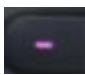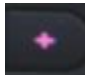Press  to decrease the energy, and press  to increase the energy. Level

0%~100% can be adjusted.




Press   , to choose the bio model.




Press  to start or pause the operation.



Press  to clear all the settings.

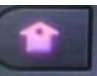

Press  to back to the main function menu.



| |
|---|
| <ul style="list-style-type: none"> ● WAVE <p>There are 2 kinds of waves to choose, Continuous Wave & Intermittent Wave</p> |
| <ul style="list-style-type: none"> ● MODE/ON <p>MODE: Choose different color (among 7 colors) ON: Start the photon function</p> |
| <ul style="list-style-type: none"> ● AUTO <p>Those 7 colors will operated automatically .</p> |
| <ul style="list-style-type: none"> ● OFF <p>Stop the photon function</p> |

Photon Curative Effect:

- Red: Vitalize be suitable for any skin, promote blood circulation, and regenerate collagen protein in order to smooth skin and heal xeroderma, exuviation, degenerate losing hair and folliculosis.
- Blue: Calm be suitable for sensitive, oil, and comedo skin-diminish and eliminate acne bacillus to remove acne, whelk, promote to compound protein with ossein,activate skin, tighten slack skin and gravity wrinkle.
- Green: Balance Normalize sebaceous glands ,Reduce black heads and sores, balance moisture.
- Purple: Drain Renew skin, reduce wrinkles
- Orange: Brighten whiten decompose pigment, and smooth scares
- Turquoise: Defense strengthen the ability to protect skin, calm down, release stress,
- White: Guide guide skin to absorb nutrients and anti-aging



5.8 Lipo Laser






Choose the function **LIPO LASER**, then it will come to the following interface:





Press  to adjust the working time.

Press  to add or reduce the energy for all the laser pads; press any of the energy bar you want  to add or reduce the energy.

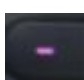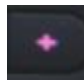
Press  to decrease the energy, and press  to increase the energy. Level 0%~100% can be adjusted.

Press  to choose the model.

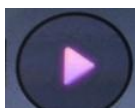Press  to start or pause the operation.

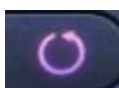Press  to clear all the settings.



Press  to back to the main function menu.



Warm tips:

Choose the proper lipo laser pad for the treatment area, use bandage to fix to the body, suggest treatment time 20~25minutes.

Lipo laser 5mw can enhance the result for slimming.

6. Package Including

- 1 x Host
- 1 x Unoisetion Cavitation 2.0 Probe
- 1 x Bipolar 3D SMART RF probe with photon for eye area
- 1x Sextupolar Vacuum 3D SMART RF for body
- 1 x Quadrupole 3D SMART RF probe with photon for face
- 1 x 8 Pole 3D SMART RF probe with photon for body
- 1 x microcurrent photon probe
- 1 x Cold probe
- 1 x Power Supply Cord(Both 110v and 220v voltage (frequency: 50/60Hz) version are available. Also we stock the USA, UK, AU, EU plug to suit for the country all over the world.)
- Big LED Laser Paddles x 6
- Small LED Laser Paddles x 2

7. parameters:

Power supply Input:100V-240V

Power :190W

40K Unoisetion Cavitation 2.0 Handle

Frequency:40KHz

Power:20W

Sextupole RF Handle

Frequeuncy:3MHz

Power:50W

Vacuum& Biploar RF Head

Frequeuncy:3MHz

Power:50W

Vacuum: <-80kPa

Pressure: >250kPa

Air flow: >10L/minute

Noise level: <70dB (30cm away)

Quadrupole RF Handle

Frequency:1MHz

Power:30W

3-Pole RF Handle

Frequency:1MHz

Power:10W

Cold hammer:

Temperature: lowest 0 degrees

Microcurrent & Photon

7 colors : Red, Blue, Green, Purple, Orange, Turquoise, White

Led laser wavelength: 635nm ~650nm


Power of each light: 5mw

Energy output :64 x 5mW =320mw (6 big pads *10 light/each+2 small pads * 2 light/each)

Mains power output: 100VA

8. Notice

1. Please insert the vacuum filter on the rear of the machine before switch machine.
Make sure the vacuum filter in the following condition

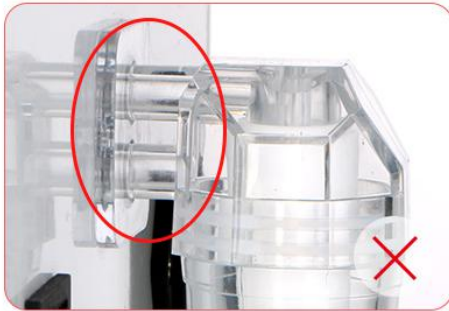

Note: Please insert this filter as deep as you can till it cant be inserted.
If the filter was not inserted completely, the vacuum probe will not have the suction.

2. If the vacuum suction is weak, please adjust the adjustment knob:
Clockwise rotation: Min to Max.



9.Illustration of Operation




1.Neck

Move in small circular motions from the inside of the neck to the outside, then lift from the neck upward to the cheek, neck treatment area can be divided into three lines(up,medium,down areas) to operate
Attention: strength should be top heavier and down lighter, operation electrodes should tight skin, repeat 8-10 times




2.Chin

Make operation head move in small circular motions by pressing tightly against the double chin and then pull back to the back of the ear, repeat 5-8 times

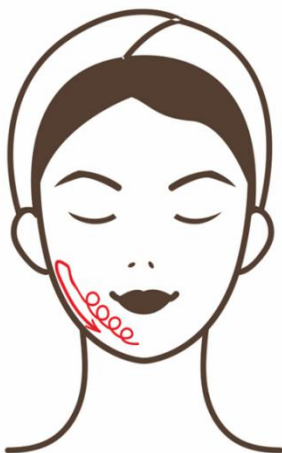


Make operation head move in small circular motions by clinging to the chin and circle out to the ear beads. Then pulled down to the chin with light force,repeat 5-8 times

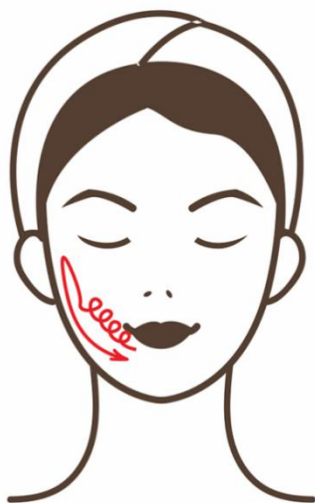

3.Mouth

Make operation head move in small circular motion by clinging to the corner of mouth, and circle out to the ear beads. Then pulled down to the chin with light force,repeat 5-8 times

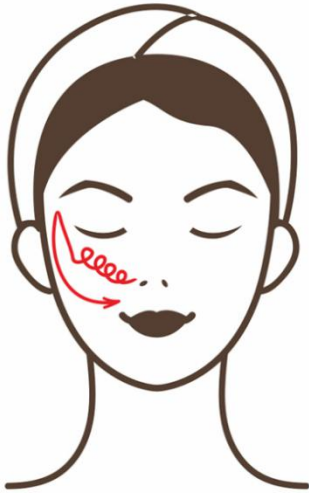


4.Cheek

Make operation head move in small circular motion by closing to the light bone of nose, circle out to the ear beads. Then pulled down to the chin with light force, repeat 5-8 times



5.Forehead

Make operation head move in small circular motion by closing to the light bone of nose, circle out to the ear beads. Then pulled down to the chin with light force, repeat 5-8 times

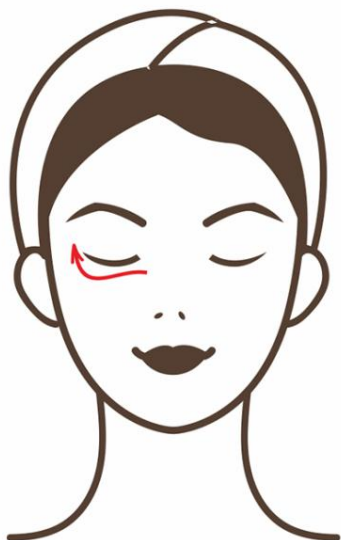

6.Eyes' around

Dredge the meridian and collaterals of eyes,
repeat 5-8 times




Move in small circular motion in the lower eye then
lift to the temple, repeat 5-8 times



Move in small circular motion from the corner of the
eye to the Jingming points to the eyebrow,
repeat 5-8 times

Belly

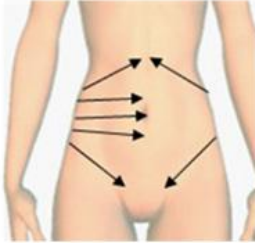

1. Move slowly from lower waist to the end of groin, repeat 3 times
2. Move slowly from lower waist to pot belly, along the gap of ribs, repeat 3 times
3. Move slowly from lower waist to belly button line, repeat 3 times




4. Separately move in small circular motions in thickness fat of body, each one in 10 circles
5. Improve anti-cellulite and the growth of collagen, tighten skin



6. From small to large by clockwise direction, circling around the navel, promote intestinal peristalsis and improve constipation

Waist

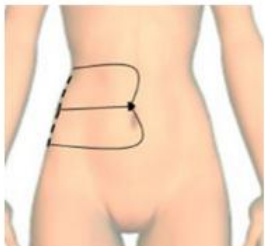

1. Knead fat back and forth in each side waist (decompose the cellulite effectively and contour waist) repeat 10 times



2. Move 10 circles in renal region
3. Separately move in small circular motions in thickness fat of buttock, each one in 10 circles
4. Move and write "8" in two sides of waist (repeat 10 times)




Lie on your side:

5. Move from the thickness fat of buttock to groin, repeat 8-10 times
6. Move slowly from the thickness fat of buttock near to axillary fossa, repeat 8-10 times

Foreleg



1. Move one time slowly from knee to groin, line by line

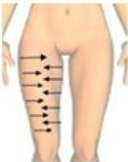

2. Move in horizontal from inside of leg to middle, down to up, slower the moving, repeat 2 times
3. Move in horizontal from outside of leg to middle, down to up, slower the moving, repeat 2 times

Caution: Lower the energy 1-3 Level when treating inside of leg



4. Separately move in small circular motions in thickness fat of body, each one in 10 circles (Improve anti-cellulite and the growth of collagen, tight skin)



5. In inside of leg, move along the muscle texture from knee to crotch, repeat back and forth movement in 4 times
6. In outside of leg, move along the muscle texture from knee to crotch, repeat back and forth movement in 4 times



7. Move from knee to groin, repeat 8 times

Backleg



1. Move slowly from the bottom of crus to popliteal fossa, repeat 3 times



2. Separately move in small circular motions in thickness fat of body, each one in 10 circles




3. Push to the up of popliteal fossa from the root of the calf, repeat 5 times

Backleg




1. Push to the bottom of buttocks from up of popliteal fossa slowly line by line, repeat once time

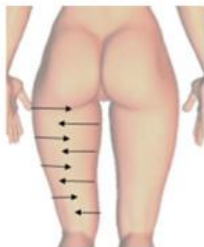

2. Move in horizontal from inside of thigh to middle, down to up, slower the moving, repeat 2 times
 3. Move in horizontal from outside of thigh to middle, down to up, slower the moving, repeat 2 times
- Caution: Lower the energy 1-3 Level when treating inside of thigh



4. Separately move in small circular motions in thickness fat of body, each one in 10 circles (Improve anti-cellulite and the growth of collagen, tight skin)

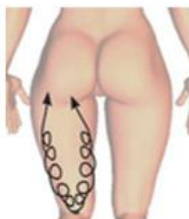

5. In inside of leg, move along the muscle texture from knee to bottom of buttocks, repeat back and forth movement in 4 times
 6. In outside of leg, move along the muscle texture from knee to bottom of buttocks, repeat back and forth movement in 4 times
- Push to the above of eight-liao point from up of knee fast line by line, repeat 8 times


Back




1. From the Da Zhui point down to governor vessel, eight-liao poin, urinary bladde and up to Da Zhui point then slide to the Da Zhui point from the governor vessel .
Caution: Repeat 3 times movement on urinary bladde of each side after governor vessel



2. From the Da Zhui point down to governor vessel, eight-liao poin, urinary bladde and up to Da Zhui point then slide to the Da Zhui point from the governor vessel .
Caution: Repeat 3 times movement on urinary bladde of each side after governor vessel

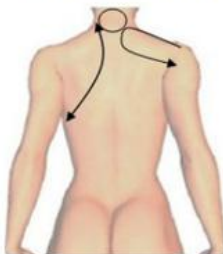

3. Move in small circle motion on dazhui point then go to daban point massage on circle motion, repeat 2 times from left to right
Caution: Move 5 times on seams of the scapula)

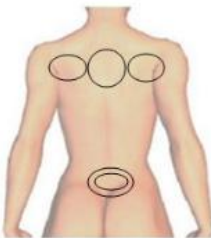

4. After playing five circle in the cardiopulmonary area, five circles should be played on the cardiopulmonary area

Back



5. Move in small circle motion from down to up, up to down, repeat 6 times, one side finished, the other side start




6. Move and write "8" in two sides of waist,back, repeat 6 times



7. Move from above of buttock to underarm until two sides finished, repeat 3 times


Outside Arm

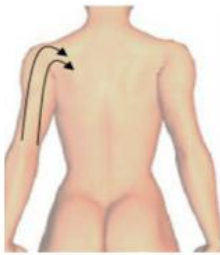

1. Move from outside of arm to underarm, down to up, slower the moving, repeat 2 times



2. Separately move in small circular motions in thickness fat of body, each one in 10 circles (Improve anti-cellulite and the growth of collagen, tight skin)

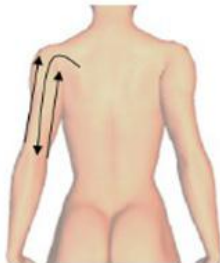


3. Move from top of outside of arm to elbow then up to underarm from two sides then down,repeat 6 times

Outside Arm




4. From up of elbow to underarm which near to back

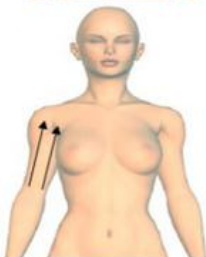

5. Move from inside of elbow to underarm, down to up, fast the moving, repeat 2 times



6. Separately move in small circular motions in thickness fat of body, each one in 10 circles (Improve anti-cellulite and the growth of collagen, tight skin)

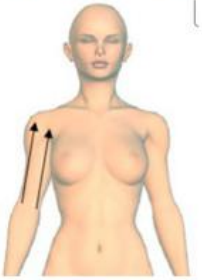

7. Move from inside of elbow to underarm, down to up, fast the moving, repeat 2 times